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
HOSPAL CARTRIDGE

Product Innovation with a Patient-Sensitive Design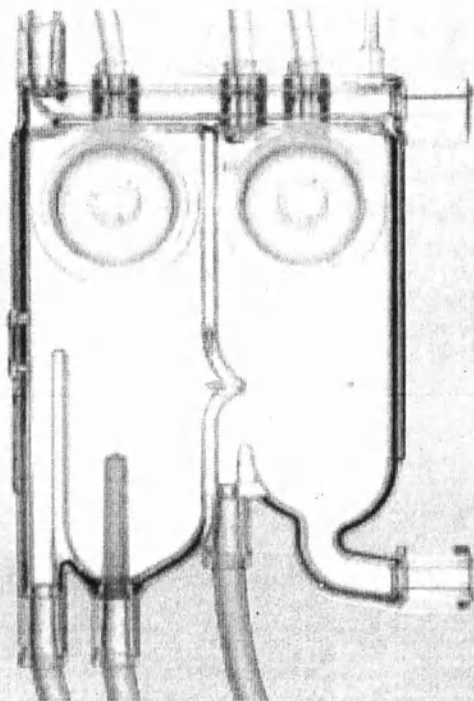

SAFETY

The patented¹ non-invasive pressure monitoring system eliminates the handling of transducer protectors. Two impermeable diaphragms allow for accurate pressure monitoring of the extra-corporeal circuit without direct contact of blood and equipment.

Also, the integrated arterial and venous chambers are pre-connected to the blood lines, reducing the number of connections compared to conventional blood tubing.

Combined, these features provide protection against cross-contamination. The risk of exposure to blood for the patient care staff is also considerably reduced.

HEMOCOMPATIBILITY

The unique low-volume design reduces the surface in contact with blood; the blood-air interface is minimized as well.

The bottom-entry chamber design provides an optimal blood flow geometry, reducing splashing and foaming generation.

The combined use of specially selected Medical Grade materials, such as a PVC formulation incorporating a plasticizer free from phthalates (DEHP), and a gentle sterilization method² (Beta: exposure to a beam of accelerated electrons) improves the patient tolerance.

AUTOMATION

The automation of the priming and rinse-back procedures of the blood circuit minimizes the time required for machine / operator interface.

ERGONOMY

The quick and easy to use design has never been equalled.

1. Product is covered by multiple U.S. and foreign patents.
2. Investigations of different clinical groups show that up to 40% of the dialysis patients are sensitised by residual ethylene oxide (EtO) from sterilization. Source: Barth, H.P. et al. Anaphylactoid Reactions In Dialysis Patients. Artificial Organs, 1986.

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

CE 0086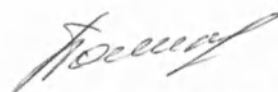

HOSPAL CARTRIDGE

Product Innovation with a Patient-Sensitive Design

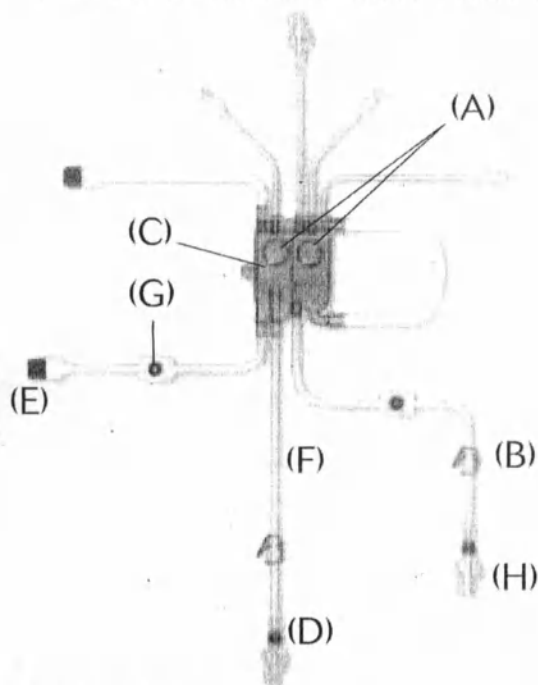- A / Pressure monitoring system
- B / Colour-coded pinch clamp in key locations
- C / Integrated chambers with bottom-entry design
- D / Colour-coded locking patient luers
- E / Colour-coded dialyzer connector caps
(for use on dialysate ports after dialysis)
- F / Kink-resistant tubing
- G / Latex-free access site, with large wings
to prevent NSI (Needle Stick Injuries)
- H / Who connector (integrated container
connector) for added convenience

SPECIFICATIONS

Materials

Tubing	Polyvinyl Chloride	PVC
Cartridge	Medical Grade, DEHP free	
Injection site	Polyterephthalate Ethylen Glycol	PETG
	Latex-free rubber	

Sterilization

Beta	Electron-beam radiation
------	-------------------------

Tubing size / volume (nominal)

Blood pathway	Ø IN = 4.17 mm / Ø OUT = 6.5 mm
Pediatric blood pathway	Ø IN = 3.18 mm / Ø OUT = 6.43 mm
Pediatric, Low-volume blood pathway	Ø IN = 2.54 mm / Ø OUT = 6.43 mm
Pump segment	Ø IN = 6.35 mm / Ø OUT = 9.43 mm
Pediatric, Low-volume pump segment	Ø IN = 3.18 mm / Ø OUT = 6.43 mm

A WIDE RANGE TO SUIT A VARIETY OF THERAPEUTIC NEEDS

Code	Application	Volume (ml)	Patient Line (cm)	Units per box
6184071	Double Needle	103	183	20
6186886	Double Needle / longer patient line	120	246	20
6186803	Double Needle / with prime line	103	183	20
6188320	Double Needle / with infusion line	120	246	20
6187694	Double Needle / pediatric	75	183	20
6187660	Double Needle / pediatric low-volume	40	145	20
6187439	Single Needle / standard	145	198	16
6187686	Single Needle / with prime line	145	198	16
6032536	Single Needle conversion kit	45	n.a.	16

HOSPAL

INTERNATIONAL MARKETING MANAGEMENT
61, avenue Tony Garnier
69007 Lyon France
Tel: 33 (0)4 37 28 11 00
Fax: 33 (0)4 37 28 11 01
www.hospal.com

Pravda