证明书
CERTIFICATE

中国国际贸易促进委员会暨中国国际商会
China Council for the Promotion of International Trade is China Chamber of International Commerce

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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THIS IS TO CERTIFY THAT:the seal of DALIAN
GOODWOOD MEDICAL CARE LTD. on the annexed DOCUMENT
is genuine.

China Council for the Promotion
of International Trade

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Authorized
Signature : Cui Ping

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(Date: JUN. 20, 2014)

Approved By
Dalian Goodwood Medical Care Ltd.

Date: May 28, 2014

(Signature)

Name of Responsible Person

Post

Stamp

Instruction for use:

«Drive medical tools: single use wooden
sterile spatulas»

Manufactured by:

“Dalian Goodwood Medical Care, Ltd.»,
China

INSTRUCTION FOR USE

Drive medical tools: single use wooden sterile spatulas

Medical device designation

Drive medical tools: Single use wooden sterile spatulas - are used in out-patient and in-patient departments in therapeutical practice for oral cavity examination. Spatulas can also be used for mixing and application of different balsams and salves. It shall be used as patient-specific.

Medical device description

Single use wooden sterile spatula is produced of environmentally-safe hard wood material. Grinding finish of the instrument and rounded edges provide comfortable operation, preventing possibility of damaging of oral cavity mucous membrane and tongue.

Basic application

Oral cavity examination when holding medical examination in therapeutical practice.

Technical characteristics

length - 150 mm $\pm$ 2.0 mm;

width - 17 mm $\pm$ 1.5 mm;

thickness - not more than 1.6 mm;

weight - not more than 2.5 g.

Instruction for use

1. Have a patient seat on a chair opposite to the source of light.
2. Open a spatula package.
3. Hold an oral cavity examination after tongue fixation with a wooden spatula.
4. Throw the spatula away to container for medical waste.

Sterilization

Devices shall be sterilized with ethylene oxide. Sterility of devices shall be guaranteed in case of unopened package only.

Usage precautions

Device remains sterile in case of undamaged packing!

For single use only. Do not use when holding medical examination in case the device is beyond its expiration date or package is damaged.

Transportation

The device shall be transported by closed transport vehicles with humidity not more than 80%, far from corrosion agents.

Storage

Single use wooden sterile spatulas shall be stored at room temperature, in well ventilated location, with humidity not more than 80%, far from corrosion agents. Keep out of the reach of children. Do not use after expiration date.

Expiration date

Expiration date of the devices is 5 years.

Disposal

The devices shall be disposed of in accordance with protocols of health care institution or local statutory legislative standards. The devices do not need special disposal.

Requirements to protection of the environment

Devices do not adversely affect human being and the environment during use, transportation and storage.

Complaints

If any quality complaints please apply to:

Limited Liability Company MK REAL KER (MK REAL KER LLC)

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THIS IS TO CERTIFY THAT: the seal of DALIAN
GOODWOOD MEDICAL CARE LTD. on the annexed DOCUMENT
is genuine.

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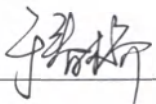
«Single use wooden sterile spatula»

Manufactured by:

“Dalian Goodwood Medical Care, Ltd.», China

Product description

Single use wooden sterile spatula.

Purpose

Single use wooden sterile spatula are used in the therapeutic practice within both out-patient and in-patient clinics for oral cavity examination. Single use wooden sterile spatula may also be used for mixing and application of various balsams and salves.

Single use wooden sterile spatula are to be used individually.

Transportation

The product has to be transported inside the covered vehicles under conditions of humidity of no more than 80% and temperature range from -50°C to + 50°C away from corrosion agents.

Storage

Single use wooden sterile spatula have to be storage in good vented compartments under conditions of humidity of no more than 80% and temperature range from -50°C to + 50°C away from corrosion agents. Keep out of reach of children. Do not use after expiration of the after termination of serviceable life period.

Labeling

The package is labeled with the data as follows:

- Name;
- Manufacturer, plant address;
- For single use only;
- Do not use in case of package integrity damage;
- Refer to application instructions;
- Ethylene oxide sterilized;
- Serviceable life period;
- Installment No.

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