Dear Patient,

Please read the following instructions carefully, because they contain important information about the correct use of this medical device. If you have any questions, please consult your doctor or pharmacist.

Instructions for use

GeloSitin® Nasal Spray

Ingredients:

GeloSitin® Nasal Spray contains: refined sesame oil (60 %), dicaprylyl carbonate, orange oil, lemon oil and an antioxidant mixture (natural extracts with high tocopherol content, palmitoyl ascorbic acid, ascorbic acid, citric acid, ethanol, triglycerides).

GeloSitin® Nasal Spray is free of preservatives.

Content:

15 ml solution

Indications:

GeloSitin® Nasal Spray is used for physical treatment and moisturisation of dry nasal mucosa, e.g. when exposed to dry air (air conditioning, heating), in cases of excessive use of decongestants, or in cases of dry rhinitis with or without crusts and scabs in the nasal vestibule or on the inner surface of the nasal wings.

Contraindications, interactions and side effects:

GeloSitin® Nasal Spray must not be used in patients with known hypersensitivity to the ingredients.

Do not use simultaneously with other local therapeutic agents.

If further medical devices or other therapeutic appliances, like nasal prongs for oxygen supply, are applied within the nose simultaneously beside GeloSitin® Nasal Spray, their compatibility has to be checked. On very rare occasions a scratchy or smarting feeling could occur in the ear-nose-region.

What must be taken into consideration when treating children?

Do not use for children under the age of three.

What must be observed during pregnancy and lactation?

GeloSitin® Nasal Spray can be used without any risk during pregnancy and lactation

Dosage instructions, method and duration of application:

The following instructions apply unless your doctor advises you otherwise.

How should you apply GeloSitin® Nasal Spray?

Please screw the enclosed dosing pump onto the glass vial prior to initial use. To do this, remove the screw cap of the vial, insert the thin tube of the dosing pump into the vial and screw the dosing pump on tight. Remove the green protective cap and always hold the spray vial upright since the pump cannot function properly otherwise. Important: prime the spray mechanism several times prior to initial use and after

longer periods of disuse, avoiding the eyes, until a consistent spray mist is produced by rapid, vigorous activation of the pump.

How much GeloSitin® Nasal Spray should you use and how often should you use it?
Administer 1-2 pump sprays into each nostril several times a day as required.

For how long can you use GeloSitin® Nasal Spray?
GeloSitin® Nasal Spray is classified as safe for use over longer periods or for interval therapy.

For hygienic reasons, we recommend use of GeloSitin® Nasal Spray on an individual basis.

Instructions and information on shelf life:

Do not use GeloSitin® Nasal Spray after the expiry date has passed.
It is recommended to discard GeloSitin® Nasal Spray after 6 to 7 weeks from opening the bottle.

Storage and Transportation:

Store GeloSitin® Nasal Spray below 25°C. Do not store GeloSitin® Nasal Spray in a refrigerator.

Store medical devices carefully! Keep out of reach of children!

Manufacturer:

G. Pohl-Boskamp GmbH & Co. KG, Kieler Strasse 11, 25551 Hohenlockstedt, Germany

Address for presentation of claims:

JSC Krasnogorskleksredstva
Russia, 143444, Moscow Region,
Krasnogorsk, distr. Opalikha,
Mira St., 25
Ph.: (495) 705-93-80
Fax: (495) 705-93-82
E-mail: krls@krls.ru
<http://www.krls.ru>

Information last updated:
November 2015

Hohenlockstedt, 2015-11-05

POHL BOSKAMP

I. Rowedder
Ines Rowedder
Advisor Medical Devices

S. Nagel
Sandra Nagel
Advisor Medical Devices

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