Siam Sempermed Corp. Ltd.
110 Kanjanavanit Road, Pahtong, Hatyai, Songkhla 90230, Thailand
Tel: (66) 074 344 663 Fax: (66) 074 344 677

Hataichanok **APPROVED**
Senior Asst. Product Manager
Ms. Hataichanok

Vanlinee Production Manager
Vanlinee
Laohachaiyakul

March 10 2016

Disposable medical examination non-sterile nitrile powder-free gloves
«Clean + Safe»

Instruction Manual

CONTENT

1	INTENDED USE	3
2	INDICATIONS FOR USE	3
3	CONTRA-INDICATIONS FOR USE	3
4	SIDE EFFECTS	3
5	DIRECTIONS FOR USE	3
6	SPECIFICATIONS	5
7	DISPOSAL AND ENVIRONMENTAL PROTECTION	6
8	TRANSPORTATION AND STORAGE	6
9	PACKAGE CONTENTS	6
10	MARKING	7
11	MANUFACTURER WARRANTY	7
12	MANUFACTURER INFORMATION	7
13	RESPONSIBLE SUPPLIER	7

1. INTENDED USE

The current Instruction Manual applies to Disposable medical examination non-sterile nitrile powder-free gloves "Clean + Safe" (hereinafter – gloves), which intended for single use in medical research, diagnostic and therapeutic procedures in order to protect the patient and investigator from cross-contamination, and also for use with contaminated medical materials.

Gloves are intended for temporary use and for use during non-invasive procedures.

Gloves have not an anatomical shape.

Conditions of use: medical and health care institutions.

Disposable medical examination non-sterile nitrile powder-free gloves "Clean + Safe" are made to be the following models: (see Annex 1)

Gloves sizes are enclosed in Annex 2.

2. INDICATIONS FOR USE:

- contact with skin or mucous membrane of the patient;
- contact with contaminated medical materials;
- contact with blood;
- contact with biological fluids;
- when there is any violation of the skin integrity of the nurse or the patient;
- carrying out medical research, diagnostic and therapeutic procedures.

3. CONTRA-INDICATIONS FOR USE

Dermatitis, the possibility of allergic reaction to device material or its components in a doctor; loss of device integrity (punctures, cuts).

4. SIDE EFFECTS

No

5. DIRECTIONS FOR USE:

5.1 General provisions

The product is intended as a hand protective device to protect healthcare personnel hands from contamination and to decrease transmission of potentially pathogenic microorganisms.

The proper technique for removing and donning of the gloves is necessary, it prevents contamination of medical professionals hands.

If the integrity of the glove is broken (for example, if it is punctured), it should be replaced as soon as possible and hands must be processed with antiseptic agent.


WARNING!

- Follow gloves donning instructions to avoid any contamination.
- Protect gloves and packaging from mechanical impact.
- Gloves are not allowed to use if the package is damaged.
- Gloves are not allowed to use after the expiration date.
- It is prohibited to touch the face with contaminated gloves.
- It is prohibited to touch surrounding surfaces: door handles, keyboard, computer mouse.
- Never wash and never use again disposable gloves.

If the indication for hand hygiene prior contact, which requires using of the gloves, it is necessary to perform hand hygiene (antiseptic treatment or washing with soap) before donning of the gloves.

5.2 Donning and removing techniques:

5.2.1 How to put on the gloves:

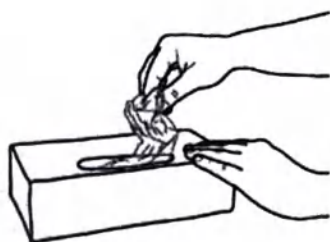
1. Remove the glove from the package (box).


2. Touch only the part of the glove, which will be located on the wrist (top edge of the cuff).

3. Put on the first glove.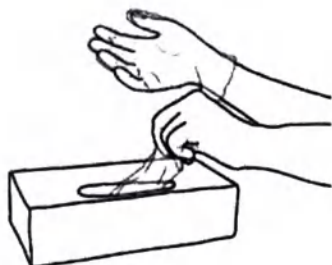

4. Take the second glove with the hand without a glove, touching the part of the glove, which will be located on the wrist.

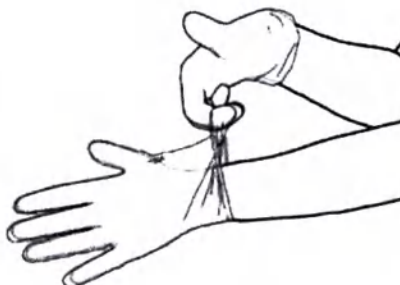5. To avoid the hand with the glove touching the skin of the forearm, remove the outer surface of the glove to put it on the bent fingers of the hand in the glove, this will allow you to put on the second glove.


6. After the gloves are put on, you cannot touch any objects of the external environment, contact with which is an indication to change the gloves.

5.2.2 Use the gloves according to the intended purpose


5.2.3 How to remove the gloves:

1. Grab one glove with the fingers at the wrist to remove it without touching the skin of the forearm, and pull it off the hand so that the glove is turned inside out.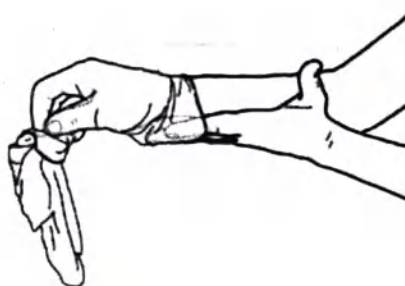

2. Hold the removed glove in the hand wearing the glove, put the fingers of the free hand between the second glove and the wrist. Remove the second glove, rolling it off the hand, and put it into the first glove.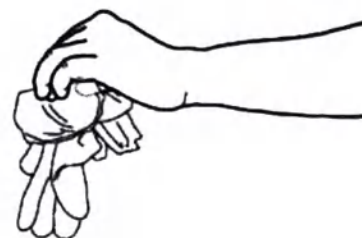

3. Dispose of the removed gloves.


Then perform the hand hygiene through sanitary antisepsis or hand washing with soap.

6. SPECIFICATIONS


6.1 Gloves comply with the palm width and the palm length standards as well as wall thickness, see Table 1.

Table 1 - Sizes of gloves and tolerance

Description	Size				
	XS	S	M	L	XL
Article	EN7XS	EN7S	EN7M	EN7L	EN7XL
	EN8XS	EN8S	EN8M	EN8L	EN8XL
Width W by size (pic.1), mm	70±10	80±10	95±10	110±10	120±10
Length L, not less (pic. 1), mm	220	220	230	230	230
Thickness, not less (pic.2), mm					
- of finger	0,05				
- of palm	0,05				

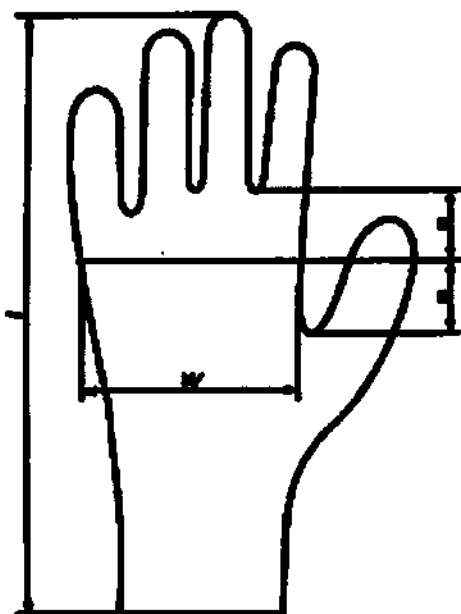Picture 1 - Measurement points of the glove width and length

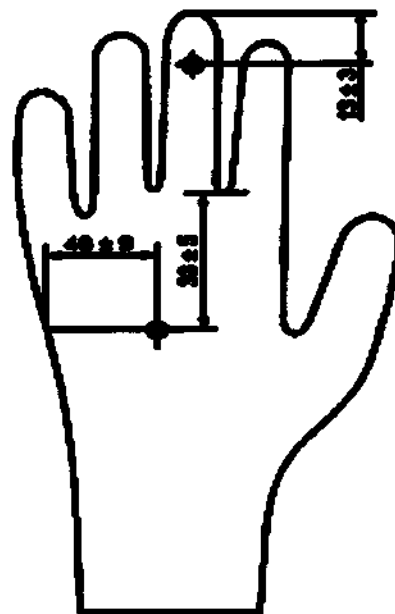Picture 2 - Measurement points of the glove wall thickness

6.2 Strength characteristics of gloves comply with Table 2

Table 2- Tensile characteristics of gloves

Characteristic	Value
1 Nominal strength before accelerated aging, MPa, not less	14
2 Nominal strength after accelerated aging, MPa, not less	14
3 percentage elongation before accelerated aging, %, not less	500
4 percentage elongation after accelerated aging, %, not less	400

6.3 Gloves are manufactured from materials shown at Table 3.

Table 3 Materials (Raw materials), which have a contact with healthcare personnel and (or) patient

No	Name	Used material		Manufacturer
1	Disposable medical examination non-sterile nitrile powder-free on-line chlorination finger textured gloves with thickening on the palm: EN7XS; EN7S; EN7M; EN7L; EN7XL	glove	NBR 265 brand synthetic latex - nitrile	Siam Sempermed Corp. Ltd., Thailand
		Blue colorant	SSI brand indulin	Siam Sempermed Corp. Ltd., Thailand
2	Disposable medical examination non-sterile nitrile powder-free on-line chlorination finger textured gloves: EN8XS; EN8S; EN8M; EN8L; EN8XL	glove	NBR 265 brand synthetic latex - nitrile	Siam Sempermed Corp. Ltd., Thailand
		Violet colorant	SSI brand indulin	Siam Sempermed Corp. Ltd., Thailand

6.4 Features

This product has the following features:

- a) non-sterile;
- b) textured on the fingers;
- c) powder-free: on-line chlorinated;
- d) Single use.

The texture of the outer surface can improve the capture of the medical instrument.

In order to avoid sticking of the gloves, we conduct modification of the surface using chlorine. Used colorants are non-toxic.

7. DISPOSAL AND ENVIRONMENTAL PROTECTION

Gloves must be assembled and transferred to a special licensed organizations engaged in the disposal of medical waste. Disposal should be carry out in accordance with local legislation current standards and requirements.

8. TRANSPORTATION AND STORAGE.

8.1 The transportation of the gloves should be carry out by any means of transport in covered vehicles in accordance with rules of cargo transportation valid for this transport.

8.2. If transported at temperature below zero, the gloves should be stored at storage conditions within 1 day before unpacking.

8.3 Gloves should be stored packed in rooms at temperature from +5 °C up to +37 °C and relative humidity not above 80%.

During storage gloves should be protected from direct sunlight, should be placed at the distance 1 m from heating appliances, should not be impacted by oil, gasoline, kerosene (and other organic solvents), as well as other corrosive substances.

9 PACKAGE CONTENTS

#	Description	Quantity, pc.
1	Disposable medical examination non-sterile nitrile powder-free gloves "Clean + Safe", *	100 pairs
2	Instruction manual	1
3	Warranty	1

*- *- the customer can choose models, the list see in Annex 1

10. MARKING

10.1 The symbols used in marking.

- Lot No:

(number)

- " SINGLE USE ONLY "

- " Manufacturing date "

(year/month)

- " Best by ... "

(year/month)

- Temperature range

- Do not expose to sunlight

- Protect from moisture

11. MANUFACTURER WARRANTY

The manufacturer guarantees compliance of the gloves to manual requirements if the operating, transportation and storage conditions are satisfied by a customer.
Guaranteed shelf life for the gloves is 3 years from manufacturing date.

12. MANUFACTURER INFORMATION:

Siam Sempermed Corp. Ltd., 110 Kanjanavanit Road, Pahtong, Hatyai, Songkhla 90230, Thailand. Manufacturing factories are pointed out in Annex 3.

13. RESPONSIBLE SUPPLIER:

Limited Liability Company "GLOMAKO" (GLOMAKO LLC), 196128, Russia, Saint-Petersburg, Varshavskaya street, house 5, building 2A, office 604-A Telephone (812)6358898

Annex 1

I. Disposable medical examination non-sterile nitrile powder-free gloves "Clean + Safe", possible models:

1. Disposable medical examination non-sterile nitrile non-powdered on-line chlorination finger textured with thickening of the palm: EN7XS; EN7S; EN7M; EN7L; EN7XL.

2. Disposable medical examination non-sterile nitrile non-powdered on-line chlorination finger textured: EN8XS; EN8S; EN8M; EN8L; EN8XL.

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