Nr. 11442

**CERTIFICATION OF COPY**

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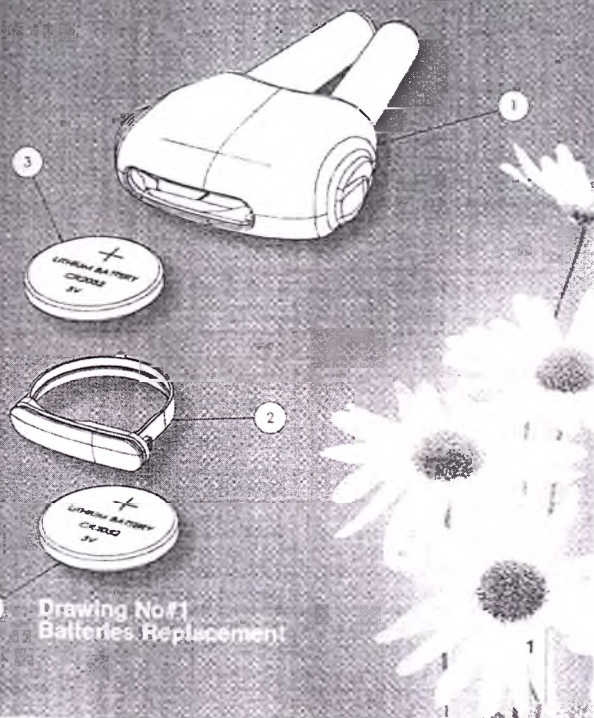
# bionette

## BioNette™

A revolutionary new device intended for the treatment of allergic rhinitis (hay fever). BioNette™ provides relief and effectively prevents symptoms associated with allergic rhinitis such as: itchy pharynx, itchy eyes, itchy ears, itchy nose, runny nose, tears, sneezing, headaches and watery eyes. The treatment is neither chemical nor pharmacological and has no known side effects. The treatment is based on the photo therapy principle (treatment with light energy). The BioNette™ device utilizes low level narrow band light technology.

## METHOD OF USE

Each treatment session lasts for (approx.) 4.5 minutes. Recommended frequency of use is two to three times a day. In order to attain maximal effectiveness and optimal relief it is recommended that use of BioNette™ begin immediately upon appearance of the first symptoms of rhinitis. Once improvement occurs, the frequency of use can be gradually reduced as required, until it ends completely. BioNette™ is a tiny device that fit into any pocket even the smallest one.

## BEFORE FIRST USE

Please pull out the battery's thin transparent seal strip.

## OPERATING INSTRUCTIONS

### How to use:

- Depress and hold both rubber buttons on the sides of the device. The two clear prongs will emit a red light.
- Ensure the red light is continuous and not flashing.
- Insert the prongs into the nose whilst pressing both rubber buttons.
- Release the two rubber buttons.
- The treatment is complete when the red light stops automatically, after approximately 4 minutes. (Once activated, it cannot be turned off part way through the treatment cycle).
- Follow the cleaning instructions detailed in the enclosed leaflet.
- Recommended frequency of use is 2-3 times a day. Effectiveness may increase with repeated use.

### Caution:

- Caution: • Parental supervision required for children. • Do not look directly at the light source. • Keep out of the sight and reach of children.

## BATTERIES REPLACEMENT

The BioNette includes 2 high quality lithium CR2032 (3V) batteries. Remove battery cap and load with (2) batteries in correct position as per drawing No#1.

## TECHNICAL DATA

|                        |                                                                         |
|------------------------|-------------------------------------------------------------------------|
| Light Treatment        | Narrow band 630 nm light                                                |
| Illumination Power     | Minimum 4.0 mW for each nostril                                         |
| Energy dose            | 2 joule                                                                 |
| Timer                  | Push-button Operated automatically turns off after (approx) 4.5 minutes |
| Timer Accuracy         | deviation: 15% +/-                                                      |
| Power Source           | 2X CR2032 (3v)                                                          |
| Current (Max)          | 0.3A                                                                    |
| Battery Lifetime       | Up to 80 treatments                                                     |
| Low Battery Indication | Active LED flashes                                                      |
| Unit Weight            | 18 gram with battery                                                    |
| Unit Size              | 52x38x15 mm                                                             |

## INTERACTIVE EFFECTS

There are no known interactive effects when the device is used in conjunction with pharmacological medications. BioNette™ can be used in conjunction with medications such as antihistamines, which are taken orally, corticosteroids, or medications taken to prevent congestion.

## SIDE EFFECTS

There are no known side effects when using the device. People suffering with severe cases of allergic rhinitis can use BioNette™ more than three times a day without any fear of adverse side effects.

## BACKGROUND ON THE BIOLIGHT SERIES OF PRODUCTS

The BIOLIGHT product line - of which BioNette™ is just one product - is a series of medical devices that utilize low level, narrow band light technology for non - invasive, non - pharmacological treatments.

These products have been developed by a team of scientists and doctors specializing in photo biology and photo-therapy. The products are supported by clinical and scientific data received from a large number of hospitals and clinics, some of which has been published in prestigious medical journals worldwide. These products constitute a significant breakthrough in non-pharmacological treatments. Their effect is impressive and they have no known side effects. Introduction of the **BioNette™** device offers a revolutionary new non-pharmacological alternative for the treatment of allergic rhinitis. Additional BIOLIGHT products include: **BIOSTICK™**, **BIOBEAM™ 660**, **BIOBEAM™ 940**, and **BIOBEAM™ ACNE**. BioNette™ is a registered trademark.

#### CLEANING

- Cleaning: The light source should be cleaned with a mild cleaning solution applied with a soft damp cloth. The **BioNette™** should never be immersed in water.
- Sterilization: Cleaning of the light sources with a cloth soaked in 70% alcohol is recommended.
- Hygienic protection: The most hygienic conditions possible should be maintained.

#### RESOLVING PROBLEMS

| Problem                     | Possible reason                        | Solution/Action                                                                         |
|-----------------------------|----------------------------------------|-----------------------------------------------------------------------------------------|
| Red Light does not light up | Batteries are old or empty             | Replace batteries                                                                       |
|                             | Batteries have been incorrectly placed | Replace batteries                                                                       |
|                             | BioNette is faulty                     | If the above solution does not resolve your problem, please contact your local supplier |
| Red Light flashes           | Batteries are weak                     | Replace batteries                                                                       |

#### MANUFACTURER'S LIMITED WARRANTY

The warranty is issued to the consumer who purchases the device and ensures that the device is free of defects, both in regard to material and its proper operating condition. The warranty is valid for two years from the date of purchase. The warranty is for repair or free of charge replacement of spare parts which at the discretion of the manufacturer were damaged during normal

operating conditions and when the device was used in accordance with the instructions supplied by the manufacturer.

#### THE WARRANTY WILL NOT APPLY IN THE FOLLOWING INSTANCES:

- Improper use.
- Repair by someone other than the manufacturer or an officially appointed representative of the manufacturer.
- Accidental damage or breakage.
- Damage caused by use of unsuitable batteries.

#### Symbols on Package

|  |                                            |  |                                                     |
|--|--------------------------------------------|--|-----------------------------------------------------|
|  | Warning must be observed strictly.         |  | Authorized representative in the European community |
|  | Manufacturer                               |  | Consult instructions for use                        |
|  | Type "BF" IEC 60601-1                      |  | Date of manufacturing                               |
|  | WEEE - Electrical and electronic equipment |  |                                                     |

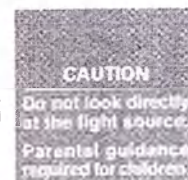
SYRO LIGHT Ltd  
15 Giron St. Yehud, 56217, IL

MEDES LIMITED, 5 Beaumont Gate, Shenley Hill, Radlett, Herts. WD7 7AR, England. Tel / Fax: + 44 19 23859810

Production is certified for: ISO 9001 (2008), MDD 2007/47 EEC (2007), ISO 13485 (2003) Cert. No. 499/499CE

The phototherapy technology & indication are patented. The BioNette device is worldwide patent pending.

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