Instruction for use – Patient Leaflet
Medical Device for patient self-administration

Product Composition

Vicks AquaForte is a vaporizing nasal spray of sea water hypertonic solution.

Filtered sea water 80% (w/w)

Purified water 20% (w/w)

- Aromatics free
- Preservative free

Chloride content: 23.20 – 27.20 g/l

Indications

- Humidification of the nasal mucosa
- Blocked nose relief
- Easing thick mucus removal

Contraindications

No contraindications are known for the intended use of the medical device hypertonic Vicks AquaForte.

Side effects

No side effects are known.

Precautions

In case concomitant use of nasal medicinal products is necessary, please contact your doctor or pharmacist.

Instructions for use

- Briefly press the nozzle before first use to prime the applicator
- Tilt your head on the side and carefully place the end of the nozzle in front of the nose
- Exert a short pressure on the nozzle (1-2 seconds)
- Let the excess solution run out and then blow your nose
- Repeat process in the other nostril

TEVA Quality Operations Germany
Merckle GmbH
Graf-Arco-Str. 3, 89079 Ulm, Germany

Instruction for use – Patient Leaflet
Medical Device for patient self-administration

- After each use, clean the nozzle with warm water

Use several times a day in each nostril.

Should only be used by one person to avoid transmission of infections.

Adults and children aged 6 years and over.

Package Information

100ml of solution in an aluminum can. Vicks AquaForte is supplied in a box containing one bottle and one leaflet.

Storage Conditions

There are no specific storage temperatures.

The recommended storage conditions are: 15°C - 25°C

Pressurized container: protect from sunlight and do not expose to temperatures exceeding 50°C.

Do not pierce or burn, even after use.

Keep out of reach of children.

Use only as intended

Transport Conditions

There are no specific transport temperatures.

The recommended transport conditions are: 15°C - 25°C

Product shelf life

36 months

Non-sterile

Non-prescription.

Interaction with other medical products

No interaction with other medical products and medical devices known

Recommendations for disposal

The medical device can be disposed together with domestic waste.

 **Quality Operations Germany**
Merckia GmbH
Graf-Arco-Str. 3, 89079 Ulm, Germany

Instruction for use – Patient Leaflet
Medical Device for patient self-administration

Manufacturer

Merckle GmbH
Graf-Arco-Str. 3
89079 Ulm
Germany

Local representative
Teva LLC
10 Shabolovka
Bld.1
Moscow 119049
Russia

LOT

- Batch code

- Use by date

- Read usage instructions

- Caution, consult documents

Version no.: 007

TEVA Quality Operations Germany
Merckle GmbH
Graf-Arco-Str. 3, 89079 Ulm, Germany

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