CE 0120

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Manufactured In

AUTHENTICALLY AND VALID

SIGD BY *Vivian Tsao*NAME *Vivian Tsao*POSITION *Business Manager*COMPANY *Global-OK Vision, Inc.*DATE *23/02/2015*

Optimum Extra™ (Roflufocon D);

Optimum Extreme™ (Roflufocon E);

Paragon HDS® 100™ (Paflufocon D)

RIGID GAS PERMEABLE CONTACT LENSES FOR
CORNEAL RESHAPING (Orthokeratology)ACTING UNDER POWER OF ATTORNEY *Overnight Wear GOV002 of May 2014, 2015*
IMPORTANT

Please read carefully and keep this information for future use.

This package insert is intended for the eye care practitioner, but should be made available to patients upon request. The eye care practitioner should provide the patient with the patient instructions that pertain to the patient's prescribed lens.

CAUTION: MDD/93/42/EEC restricts this device for sale by, or on the order of a licensed practitioner. Contact lenses for corneal reshaping should be fitted only by a trained and certified contact lens fitter. Nonsterile. Clean and condition lenses prior to use.**WARNING:** The practitioner should provide this warning to the patient.

PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN SERIOUS INJURY TO THE EYE. IT IS ESSENTIAL THAT YOU FOLLOW YOUR EYE CARE PRACTITIONER'S DIRECTIONS AND ALL LABELING INSTRUCTIONS FOR PROPER USE OF YOUR CONTACT LENSES AND LENS CARE PRODUCTS. EYE PROBLEMS INCLUDING CORNEAL ULCERS, CAN DEVELOP RAPIDLY AND LEAD TO LOSS OF VISION; THEREFORE, IF YOU EXPERIENCE EYE DISCOMFORT EXCESSIVE TEARING, VISION CHANGES, OR REDNESS OF THE EYE, OR OTHER PROBLEMS WITH YOUR EYES, IMMEDIATELY REMOVE YOUR LENSES, AND PROMPTLY CONTACT YOUR EYE CARE PRACTITIONER.

GOV LENSES FOR
CORNEAL RESHAPING OVERNIGHT WEAR

DESCRIPTION

GOV Lenses could be manufactured from the following materials: Optimum Extra™ (Roflufocon D), Optimum Extreme™ (Roflufocon E) of Contamac Ltd., or Paragon HDS 100™ (Paflufocon D) of Paragon Vision Sciences Inc. The designs have posterior surfaces consisting of four to six zones.

1. The central spherical or aspheric zone.
2. An annular "Plateau Zone(s)" surrounding the central zone with a curvature flatter (shorter radius) than the central zone.
3. An annular "Reverse Zone(s)" surrounding the central zone with a curvature steeper (shorter radius) than the central zone.
4. A spherical or aspheric "Alignment Zone(s)" generally paralleling the underlying corneal surface.
5. Peripheral curve(s) with a radius selected to create "edge lift" to promote tear flow under the lens and avoid impingement of the peripheral curve on the cornea.

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BLE CONTACT LENSES FOR LAPING (Orthokeratology)

ernight Wear *20/10/2015 14 May 2016 2015*
IMPORTANT

Keep this information for future use.
care practitioner, but should be made available to
owner should provide the patient with the patient
scribed lens.

his device for sale by, or on the order of
corneal reshaping should be fitted only by a trained
Clean and condition lenses prior to use.

de this warning to the patient.

USES AND LENS CARE PRODUCTS COULD
THE EYE. IT IS ESSENTIAL THAT YOU
TIONER'S DIRECTIONS AND ALL LABELING
E OF YOUR CONTACT LENSES AND LENS
MS INCLUDING CORNEAL ULCERS, CAN
O LOSS OF VISION; THEREFORE, IF YOU
XCESSIVE TEARING, VISION CHANGES, OR
OTHER PROBLEMS WITH YOUR EYES,
NSES, AND PROMPTLY
ITIONER.

LENSES FOR LAPING OVERNIGHT WEAR

from the following materials: Optimum Extra™
toflufocon E) of Contamac Ltd., or Paragon HDS
Sciences Inc. The designs have posterior surfaces

c zone.
surrounding the central zone with a curvature flatter
zone.
surrounding the central zone with a curvature steeper
zone.
ment Zone(s)" generally paralleling the underlying
us selected to create "edge lift" to promote tear flow
gement of the peripheral curve on the cornea.

- The lens design also includes a "rounded" edge terminus extending from the anterior to the posterior surfaces to promote comfort.
- GOV Lenses for Corneal Reshaping are to be worn overnight with removal during all or part of each following day. The contact lenses for Corneal Reshaping are available as lathe cut firm contact lenses, and are shipped dry.

ATTRIBUTES OF THE GOV LENSES

Physical Property	Optimum Extra™ (Roflufocon D)	Optimum Extreme™ (Roflufocon E)	Paragon HDS® 100™ (Paflufocon D)
Refractive Index	1.4333	1.4332	1.442
Luminous Transmittance (Blue)	90%	90%	93%
Luminous Transmittance (Green, Gray)	90%	90%	95%
Wetting Angel (Receding)	3°	6°	42°
Specific Gravity	1.166	1.155	1.10
Hardness (Shore D)	75	77	79
Water Content	< 1%	< 1%	< 1%
Oxygen Permeability	100Dk* x10 ⁻¹¹ at 35°C	130Dk* x10 ⁻¹¹ at 35°C	100Dk* x10 ⁻¹¹ at 35°C
Available colors	green, blue, gray	green, blue	green, blue

* (cm²/sec) (mL O₂/mL x mm Hg) ISO/ANSI Method, ISO 9913-1

LENS PARAMETERS AVAILABLE

Chord Diameter	7.0 to 12.5 mm
Optical Zone Diameter	3.0 to 7.0 mm
Base Curve Radius	5.0 to 13.50 mm
Plateau zone width	0.5 to 1.5mm (only in H & HP GOV Lenses)
Plateau Curve(s) Radius	up to 5mm flatter than Base Curve
Reverse Zone Width	0.5 to 2 mm
Reverse Curve(s) Radius	1.0mm flatter to 8.0mm steeper than Base Curve
Alignment Curve Width	0.5 to 2.5 mm
Alignment Curve(s) Radius	2.0 mm Flatter to 2.0 mm steeper than Base Curve
Peripheral Curve Width	0.5 to 1.5 mm
Peripheral Curve(s) Radius	2.0 mm to 10.0 mm flatter than Base Curve
Powers	-5.00 to +4.00 Diopters
Aspheric Lens Eccentricity	-1.5 to 1.5 (Oblate, Prolate or Tangent Conic)
Effective target power in Corneal Reshaping	-15.0 to +10.0 Diopters

ACTIONS

GOV Lenses for Corneal Reshaping produce a temporary reduction of refractive errors by changing the shape (flattening or steepening) of the cornea, which is elastic in nature. Slightly reducing the curvature of the cornea reduces the excessive focusing power of the myopic eye, and if the amount of corneal flattening is properly controlled, it is possible to bring the eye into correct focus and completely compensate for myopia.

Slightly increasing the curvature of the cornea increases the insufficient focusing power of the hyperopic eye, and if the amount of corneal steepening is properly controlled, it is possible to bring the eye into correct focus and completely compensate for hyperopia.

Contact lenses rest directly on the corneal tear layer and can gently influence the corneal shape. Regular contact lenses are designed to cause little or no effect, but GOV Lenses for Corneal Reshaping are designed to purposely flatten or steepen the shape of the cornea by applying gentle pressure to the center of the cornea during sleep. After the contact lens is removed, the cornea retains its altered shape for all or most of one's waking hours. The lenses are designed to be worn overnight with removal during the following day. The GOV Lenses must be worn at night on a regular schedule to maintain the corneal reshaping, or the refractive errors will revert to the pretreatment level.

INDICATIONS (USES)

GOV Lenses (Optimum Extra™ (Roflufocon D), Optimum Extreme™ (Roflufocon E), or Paragon HDS® 100™ (Paflufocon D) Rigid Gas Permeable Contact Lenses for Corneal Reshaping are indicated for use in the reduction of refractive errors in nondiseased eyes. The lenses are indicated for overnight wear in a Corneal Reshaping fitting program for the temporary reduction of up to -15.00 diopters of myopia and up to +10.0 diopters of hyperopia in eyes with astigmatism up to 3.5 diopters. The lenses may be disinfected using any a chemical cleaning and disinfection system approved in your Country.

Note: To maintain the Corneal Reshaping effect overnight lens wear must be continued on a prescribed schedule. Failure to do so can affect daily activities (e.g., night driving), visual fluctuations and changes in intended correction.

CONTRAINDICATIONS (REASONS NOT TO USE)

DO NOT USE GOV Lenses for Corneal Reshaping when any of the following conditions exist:

- Acute and subacute inflammations or infection of the anterior segment of the eye.
- Any eye disease, injury, or abnormality that affects the cornea, conjunctiva or eyelids.
- Severe insufficiency of tears (dry eyes)
- Corneal hypoesthesia (reduced corneal sensitivity).
- Any systemic disease which may affect the eye or be exacerbated by wearing contact lenses.
- Allergic reactions of ocular surfaces or adnexa which may be induced or exaggerated by wearing contact lenses or use of contact lens solutions.
- Allergy to any ingredient in a solution which is to be used to care for your contact lenses.
- Any active corneal infection (bacterial, fungal or viral).
- If eyes become red or irritated.

WARNINGS

GOV Lenses for Corneal Reshaping are shipped to the practitioner nonsterile, dry, and packed in an individual plastic case. Clean and condition lenses prior to use by using cleaning solutions approved in your Country.

Incorrect use of contact lenses and lens care products can result in serious injury to the eye. It is essential for the patient to follow the eye care practitioner's directions and all labeling instructions for proper use of contact lenses and lens care products. Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision. If the patient experiences eye discomfort, excessive tearing, vision changes, or redness of the eye, instruct the patient to immediately remove the lenses and do not wear them until instructed to do so by the eye care practitioner. All contact lens wearers must see their eye care practitioner according to the schedule given to them.

GOV Lenses for orthokeratology are to be worn overnight with removal during all or part of each following day. Wearing the lenses continuously (extended wear) presents increased risk, which increases with the number of consecutive days that the lenses are worn between removals. Although overnight Contact Lens Corneal Reshaping prescribes only overnight wear with removal during the waking hours, and although the safety risks of intermittent overnight wear may not be as great as with sustained overnight wear; there is still increased risk beginning with the first overnight period.

WARNING

The risk of ulcerative keratitis has been shown to be greater among wearers of extended wear lenses than among wearers of daily wear lenses. The risk among extended wear lens wearers increases with the number of consecutive days that lenses are worn between removals, beginning with the first overnight use. This risk can be reduced by carefully following directions for routine lens care, including cleaning of the lens storage case. Additionally, smoking increases the risk of ulcerative keratitis for contact lens wearers. It is recommended that contact lens wearers see their eye care practitioners twice each year or, if directed, more frequently.

Studies have shown that contact lens wearers who are smokers have a higher incidence of adverse reactions than nonsmokers.

ADVERSE EFFECTS (PROBLEMS AND WHAT TO DO)

Patients should be informed that the following problems may occur.

- Eyes stinging, burning, itching (irritation), or other eye pain
- Comfort is less than when lens was first placed on eye
- Feeling of something in the eye such as a foreign body or scratched area
- Excessive watering (tearing) of the eyes
- Unusual eye secretions
- Redness of the eyes
- Reduced sharpness of vision (poor visual acuity)
- Blurred vision, rainbows, or halos around objects
- Sensitivity to light (photophobia)
- Dry eyes

If the patient notices any of these conditions, the patient should **IMMEDIATELY REMOVE YOUR LENSES**. The patient should follow these instructions.

- If the discomfort or problem stops, then look closely at the lens.
- If the lens is in any way damaged, **DO NOT** put the lens back on your eye. Place the lens in the storage case and contact your eye care practitioner.
- If the lens has dirt, an eyelash, or other foreign objects on it, or the problem stops and the lens appears undamaged, you should thoroughly clean, rinse and disinfect the lens; then reinsert it.
- If the problem continues, you should **IMMEDIATELY** remove the contact lenses and consult your eye care practitioner.

When any of the above problems occurs, a serious condition such as infection, corneal ulcer, neovascularization, iritis, persistent stromal edema or GPC (giant papillary conjunctivitis) may be present. Instruct the patient to keep the lens off the eye and seek immediate professional identification of the problem and prompt treatment to avoid serious eye damage, including corneal scarring, opacification, blindness or loss of eye.

PRECAUTIONS

Eye Care Practitioner

Clinical studies have demonstrated that GOV Lenses manufactured from Optimum Extra™ (Roflucon D), Optimum Extreme™ (Roflucon E), or Paragon HDS® 100™ (Pafucon D) are safe and effective for their intended use. However, due to the small number of patients enrolled in the clinical investigation of lenses, all refractive powers, design configurations, and lens parameters available in the lens materials were not evaluated in significant numbers. This is especially true for adolescent subjects in this investigation. Consequently, when selecting an appropriate lens design and parameters, the eye care practitioner should consider all characteristics of the lens that can affect lens performance and the patient's ocular health, including, oxygen permeability, wettability, central and peripheral thickness, and optic zone diameter.

The potential impact of these factors on the patient's ocular health should be carefully weighed against the patient's need for refractive reduction; therefore, the continuing ocular health of the

patient and lens performance on the eye should be carefully monitored by the prescribing eye care practitioner. Corneal edema is more prevalent when the lens is used in high altitudes.

Each GOV Lenses is supplied nonsterile in an individual plastic case. The lens is shipped dry; The case, packing slip or invoice is marked with the keratometry (K-code), patient power (Pat.pwr.), base curve radius (B.C.), dioptric power (Pwr.), material and color of the lens, serial number, lot number, and expire date.

Patient

Patients should be informed of the following precautions.

Solution Precautions

- Different solutions cannot always be used together, and not all solutions are safe for use with all lenses. Use only recommended solutions with the contact lenses.
- Do not heat the wetting/soaking solution and lenses.
- Always use fresh unexpired lens care solutions.
- Always follow directions in the package inserts of the contact lens solutions used.
- Use only a chemical lens care system. Use of a heat (thermal) lens care system can cause damage by warping your contact lenses.
- Sterile unpreserved solutions, when used, should be discarded after the time specified in the labeling directions.
- Do not use saliva or anything other than the recommended solutions for lubricating or wetting lenses.
- Always keep the lenses completely immersed in the recommended storage solution when the lenses are not being worn (stored).

Each GOV Lenses may be disinfected using any a chemical cleaning and disinfection system approved in your Country.

Guarantee certificates

GOV Lenses keeps all indicators of quality during all expiration date, on condition of performance of storage conditions.

Expiration date

GOV Lenses expiration date are 60 months from the date of production. **DO NOT** Use after expiry date. The term of operation of lenses the patient makes 12 months then replacement of lenses is necessary.

Environment protection

GOV Lenses when using, transporting and storage don't make negative impact on the person and environment.

GOV Lenses are utilized as simple medical waste. Utilized lenses are recommended to be turned in a napkin before isolation in the garbage container.

Claims

In case of claims on quality of a medical product, the claim to send to the address:

115191, Russia, Moscow, ul. 2-ya Roshchinskaya, Bld 4, office 503

OrtoVision LLC

info@ortovision.ru

Handling Precautions

- Always wash and rinse hands before handling lenses. Do not get cosmetics, lotions, soaps, creams, deodorants, or sprays in the eyes or on the lenses. It is best to put on lenses before putting on makeup. Water-base cosmetics are less likely to damage lenses than oil-base products. Be certain that your fingers or hands are free of foreign material before touching your contact lenses, as microscopic scratches of the lenses may occur, causing distorted vision and/or injury to the eye.

- Carefully follow the handling, insertion, removal, and wearing instructions in this booklet and those prescribed by your eye care practitioner.
- Always handle your lenses carefully and avoid dropping them.
- Never use tweezers or other tools to remove your lenses unless specifically indicated for that use. Pour your lens into a container of solution.
- Do not touch the lens with your fingernails.
- To minimize lens warpage during cleaning, the lenses should be held by the hand rather than between the thumb and fingers.

Lens Wearing Precautions

- **CAUTION:** Nonsterile. Clean and condition lenses properly.
- If the lens sticks (stops moving) on the eye, follow the "What to Do for a Sticking Lens" section. The lens should move freely on the health of the eye. If nonmovement of the lens continues, consult your eye care practitioner.
- Never wear your contact lenses beyond the period specified by your practitioner.
- Avoid, if possible, all harmful or irritating vapors and fumes.
- If aerosol products such as sprays are used while wearing lenses, keep eyes closed until the spray has settled.

Lens Case Precautions

- Contact lens cases can be a source of bacterial growth. To help avoid serious eye injury, always empty and rinse the case with a rinsing solution and allow to air dry.
- Lens cases should be replaced at regular intervals as recommended by the manufacturer or eye care practitioner.

Care for a Sticking Lens

- If the lens sticks (cannot be removed), the patient should not attempt to remove it. If the lens does not move after the recommended lubricating or rewetting solution is used, lens begins to move freely on the eye before removal. If removal continues after 15 minutes, the patient should consult their eye care practitioner.

Manufacturer

No.7 Contact Lens Laboratory Ltd

Units 2 and 3 Highfields Business Park, Sidney Little
Hastings, East Sussex, TN3 89UB, UK

USA Patents: #6,361,169, #6,543,897, #6,652,095, #7,071,111
Canada Patents: #2,492,132, #2,574,742, Australia Patents: #200980120493.X
China Patents: #ZL 01818951.2, #ZL 03818340.4, #ZL 200980120493.X

Japan Patents: #4618596, #4842266, Europe Patent: #15200980120493.X
Patent: # 10-119942, Taiwan Patent: # I 383190, others pending

shown to be greater among wearers of extended wear lenses. The risk among extended wear lens wearers is higher on days that lenses are worn between removals. This risk can be reduced by carefully following the instructions for cleaning of the lens storage case. Additionally, contact lens wearers should be examined by their eye care practitioners twice each year or, if directed, more often.

Wearers who are smokers have a higher incidence of contact lens-related complications.

WHAT TO DO

Following problems may occur:
(1) Irritation, or other eye pain
(2) Lens not placed on eye
(3) Lens feels like a foreign body or scratched area
(4) Blurred vision

(5) Decreased visual acuity
(6) Inability to see distant objects

If any of these problems occur, the patient should IMMEDIATELY REMOVE the lens and follow these instructions. Do not look closely at the lens. Do NOT put the lens back on your eye. Place the lens in the lens storage case and consult your eye care practitioner.

Do not touch the lens with your fingers. Do not rub the lens with your fingers. Do not touch the lens with anything other than the recommended solutions for lubricating or wetting lenses.

Do not use saliva or anything other than the recommended solutions for lubricating or wetting lenses.

If, after 15 minutes, the problem persists, the patient should IMMEDIATELY remove the contact lenses and consult their eye care practitioner. If, after 15 minutes, the problem persists, the patient should IMMEDIATELY remove the contact lenses and consult their eye care practitioner. If, after 15 minutes, the problem persists, the patient should IMMEDIATELY remove the contact lenses and consult their eye care practitioner.

GOV Lenses manufactured from Optimum Extra™ (polyurethane), or Paragon HDS® 100™ (Paflofocon D) are available. However, due to the small number of patients, all refractive powers, design configurations, and materials were not evaluated in significant numbers. This was done in this investigation. Consequently, when selecting lenses, the eye care practitioner should consider all factors that may affect lens performance and the patient's ocular health, including corneal curvature, central and peripheral thickness, and optic zone

The patient's ocular health should be carefully weighed when selecting lenses; therefore, the continuing ocular health of the

patient and lens performance on the eye should be carefully monitored by the prescribing eye care practitioner. Corneal edema is more prevalent when the lens is used in high altitudes.

Each GOV Lenses is supplied nonsterile in an individual plastic case. The lens is shipped dry; The case, packing slip or invoice is marked with the keratometry (K-code), patient power (Pat.pwr.), base curve radius (B.C.), dioptric power (Pwr.), material and color of the lens, serial number, lot number, and expire date.

Patient

Patients should be informed of the following precautions.

Solution Precautions

- Different solutions cannot always be used together, and not all solutions are safe for use with all lenses. Use only recommended solutions with the contact lenses.
- Do not heat the wetting/soaking solution and lenses.
- Always use fresh unexpired lens care solutions.
- Always follow directions in the package inserts of the contact lens solutions used.
- Use only a chemical lens care system. Use of a heat (thermal) lens care system can cause damage by warping your contact lenses.
- Sterile unpreserved solutions, when used, should be discarded after the time specified in the labeling directions.
- Do not use saliva or anything other than the recommended solutions for lubricating or wetting lenses.
- Always keep the lenses completely immersed in the recommended storage solution when the lenses are not being worn (stored).

Each GOV Lenses may be disinfected using any a chemical cleaning and disinfection system approved in your Country.

Guarantee certificates

GOV Lenses keeps all indicators of quality during all expiration date, on condition of performance of storage conditions.

Expiration date

GOV Lenses expiration date are 60 months from the date of production. DO NOT Use after expiry date. The term of operation of lenses the patient makes 12 months then replacement of lenses is necessary.

Environment protection

GOV Lenses when using, transporting and storage don't make negative impact on the person and environment.

GOV Lenses are utilized as simple medical waste. Utilized lenses are recommended to be put in a napkin before isolation in the garbage container.

Claims

In case of claims on quality of a medical product, the claim to send to the address: 115191, Russia, Moscow, ul. 2-ya Roshchinskaya, Bld 4, office 503
OrtoVision LLC
info@ortovision.ru

Handling Precautions

- Always wash and rinse hands before handling lenses. Do not get cosmetics, lotions, soaps, creams, deodorants, or sprays in the eyes or on the lenses. It is best to put on lenses before putting on makeup. Water-base cosmetics are less likely to damage lenses than oil-base products. Be certain that your fingers or hands are free of foreign material before touching your contact lenses, as microscopic scratches of the lenses may occur, causing distorted vision and/or injury to the eye.

- Carefully follow the handling, insertion, removal, cleaning, disinfecting, storing and wearing instructions in this booklet and those prescribed by your eye care practitioner.
- Always handle your lenses carefully and avoid dropping them.
- Never use tweezers or other tools to remove your lenses from the lens container unless specifically indicated for that use. Pour your lens into your hand.
- Do not touch the lens with your fingernails.
- To minimize lens warpage during cleaning, the lenses should be cleaned in the palm of the hand rather than between the thumb and fingers.

Lens Wearing Precautions

- CAUTION: Nonsterile. Clean and condition lenses prior to use.
- If the lens sticks (stops moving) on the eye, follow the recommended directions on "Care for a Sticking Lens" section. The lens should move freely on the eye for the continued health of the eye. If nonmovement of the lens continues, you should immediately consult your eye care practitioner.
- Never wear your contact lenses beyond the period recommended by your eye care practitioner.
- Avoid, if possible, all harmful or irritating vapors and fumes when wearing lenses.
- If aerosol products such as sprays are used while wearing lenses, exercise caution and keep eyes closed until the spray has settled.

Lens Case Precautions

- Contact lens cases can be a source of bacterial growth. To prevent contamination and to help avoid serious eye injury, always empty and rinse the lens case with fresh, sterile rinsing solution and allow to air dry.
- Lens cases should be replaced at regular intervals as recommended by the lens case manufacturer or eye care practitioner.

Care for a Sticking Lens

- If the lens sticks (cannot be removed), the patient should be instructed to apply 3 to 4 drops of the recommended lubricating or rewetting solution directly to the eye and wait until the lens begins to move freely on the eye before removing it. If nonmovement of the lens continues after 15 minutes, the patient should IMMEDIATELY consult the eye care practitioner.

Manufacturer

No.7 Contact Lens Laboratory Ltd
Units 2 and 3 Highfields Business Park, Sidney Little Road, Churchfields Industrial Estate, Hastings, East Sussex, TN3 89UB, UK

USA Patents: #6,361,169, #6,543,897, #6,652,095, #7,070,275, #6,997,553, #7,360,892
Canada Patents: #2,492,132, #2,574, 742, Australia Patents: #2003258115, #2005269761
China Patents: #ZL 01818951.2, #ZL 03818340.4, #ZL 200580024766.2, #ZL 200980120493.X
Japan Patents: #4618596, #4842266, Europe Patent: #1527369, India Patent: #236549 Korea Patent: # 10-119942, Taiwan Patent: # I 383190, others pending

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