
Signed by,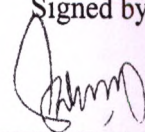

Sally Chong
International Sales & Marketing Manager

Date: 16/12/2013

Surgical sterile latex powder free gloves «Clean+Safe»

Instruction Manual

CONTENT

1	INTENDED USE	3
2	INDICATIONS FOR USE	3
3	CONTRA-INDICATIONS FOR USE	3
4	SIDE EFFECTS	3
5	DIRECTIONS FOR USE	3
6	SPECIFICATIONS	6
7	DISPOSAL AND ENVIRONMENTAL PROTECTION	7
8	TRANSPORTATION AND STORAGE	8
9	PACKAGE CONTENTS	8
10	MARKING	8
11	MANUFACTURER WARRANTY	8
12	MANUFACTURER INFORMATION	9
13	RESPONSIBLE SUPPLIER	9

1. INTENDED USE

The current Instruction Manual applies to Surgical sterile latex powder free gloves «Clean + Safe» (hereinafter – gloves), are designed for medical technician and patient protection against cross contamination during surgical operations.

Conditions of use: medical and health care institutions.

The current Instruction Manual applies to Surgical sterile latex powder free gloves «Clean + Safe» are made to be the following models: (see Annex 1)

2. INDICATIONS FOR USE:

Carrying out of the different procedures during maintaining surgical operations in the universal medical treatment and preventive care institutions.

3. CONTRA-INDICATIONS FOR USE

Dermatitis, allergic reactions to latex.

4. SIDE EFFECTS

No

5. DIRECTIONS FOR USE:

5.1 General provisions

- Do wear the correct size glove. A poorly fitting glove can limit your ability to perform the task and maybe damaged (torn or cut) more easily.
- Do change surgical gloves periodically during long cases as the protective effect of latex rubber gloves decreases with time and in apparent tear may occur
- Do keep fingernails trimmed moderately short (less than 3mm or 1/8 inch beyond the finger tip) to reduce the risk of tears
- Do pull gloves up over cuff of gown (if worn) to protect the wrists
- Do use water soluble (non fat-containing) hand lotions and moisturisers often to prevent hands from drying, cracking and chapping due to frequent hand washing and gloving
- Don't use oil-based hand lotion or creams, because they will damage latex rubber surgical and examination gloves
- Don't use store gloves in areas where there are extremes in temperature (e.g., in the sun, or near a heater, air conditioner, ultraviolet light, fluorescent light or X-ray machine). These conditions may damage the gloves (cause breakdown of the material they made of), thus reducing their effectiveness as a barrier


5.2 Pre-starting procedures:

5.2.1 Before gloves using make sure of package integrity. Then open the outer pack. The inner pack of the gloves is sterile. It is necessary to perform hand hygiene before the inner pack opening, picking up and donning of the gloves.

5.3 Procedure of work with device:

Perform hand hygiene before an "aseptic procedure by handrubbing or hand washing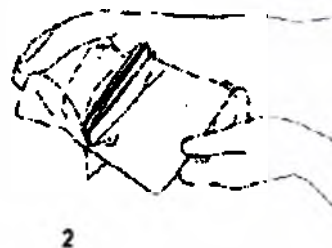


Check the package for integrity. Peel the pouch completely off the heat seal to expose the wrapper without touching it

Place the wrappers on clean, dry surface without touching the surface. Open the package and fold it towards the bottom as to unfold the wrapper and keep it open



Using the thumb and index finger of one hand, carefully grasp the folded cuff edge of the gloves

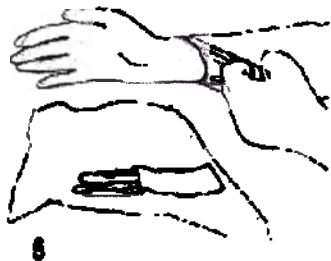
Slip the other hand into the glove in a single movement, keeping the folded cuff at the wrist level


Pick up the second glove by sliding the fingers of the gloved hand underneath the cuff of the glove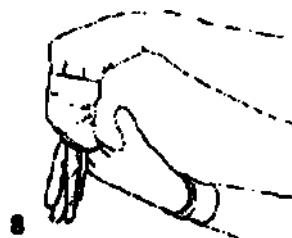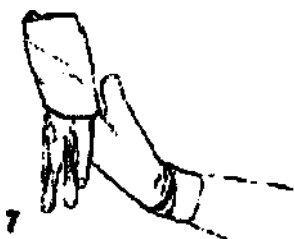

In a single movement, slip the second glove on to the ungloved hand while avoiding any contact of the gloved hand on the surfaces other than the glove to be donned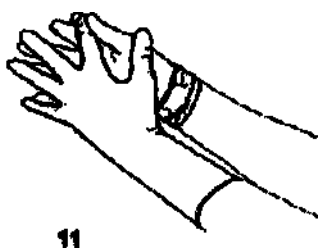

If necessary, after donning both gloves, adjust the fingers and interdigital spaces until the gloves fit comfortably

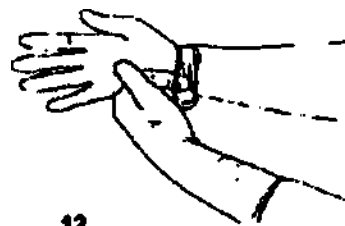Unfold the cuff of the first gloved hand by gently slipping the fingers of the other hand inside the fold, making sure to avoid any contact with a surface other than outer surface of the glove


13


14

The hands are gloved and must touch exclusively sterile devices or the previously disinfected patient's body area

Indication for removing the sterile gloves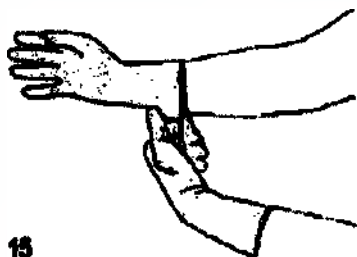

15

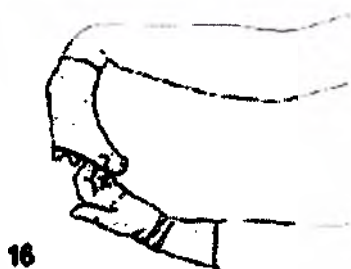
16


17

Remove the first glove by peeling it back with the fingers of the opposite hand. Remove the glove by rolling it inside out to the second finger joints (do not remove completely)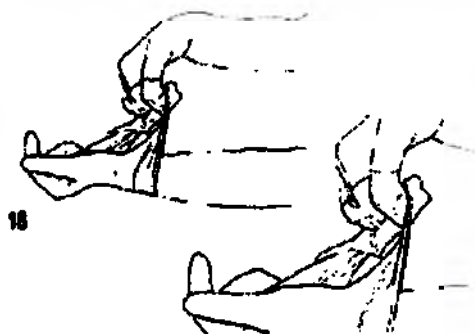


18

Remove the other glove by turning its outer edge on the fingers of the partially ungloved hand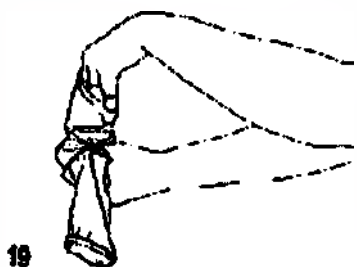

19

Remove the glove by turning it inside out entirely to ensure that the skin of the health-care worker is always and exclusively in contact with the inner surface of the glove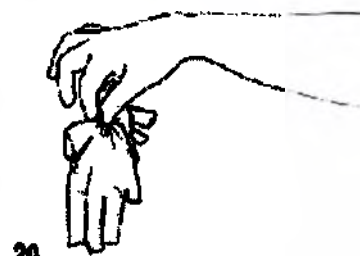

20

Discard gloves


6. SPECIFICATIONS

6.1 Sizes of the gloves describe in the Annex 2.


6.1.1 Gloves comply with the palm width and the palm length standards as well as wall thickness, (please see table 1).

Table 1 - Sizes of the surgical gloves

Description	Size							
	5,5	6	6,5	7	7,5	8	8,5	9
Width W by size (pic.1), mm	72±4	70±10	80±10	85±10	95±10	110±10	120±10	130±10
Length L, not less (pic. 1), mm	250	260	260	270	270	270	280	280
Thickness, not less (pic.2), mm								
- of finger	0,08							
- of palm	0,08							
- of cuff	0,08							

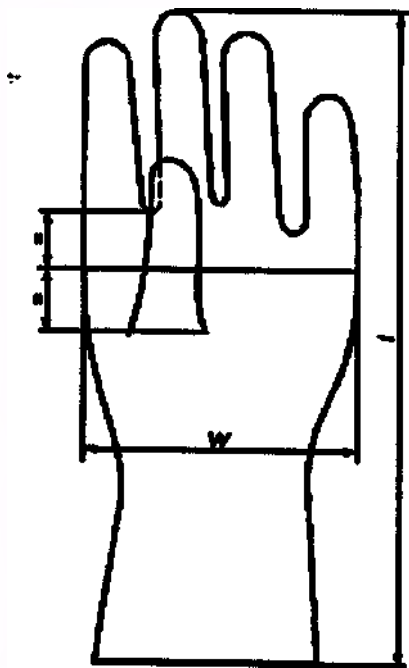Picture 1 - Measurement points of the glove width and length

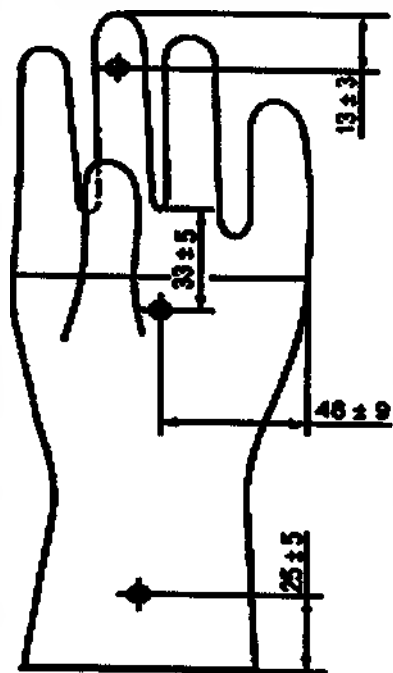Picture 2 - Measurement points of the glove wall thickness

6.2 Strength characteristics of the surgical gloves.

6.2.1 Tensile characteristics of the surgical gloves should comply with Table 2.

Table 2- Tensile characteristics of the surgical gloves.

Characteristic	Value
1 Nominal strength before accelerated aging, MPa, not less	18
2 Nominal strength after accelerated aging, MPa, not less	14
3 percentage elongation before accelerated aging, %, not less	650
4 percentage elongation after accelerated aging, %, not less	500

6.3 Gloves are manufactured from materials shown at Table 3.

Table 3 Materials (Raw materials), which have a contact with healthcare personnel and (or) patient

№	Name	Used material		Manufacturer
1	Surgical latex sterile powder free anatomically shaped polymer coated gloves with micro-textured surface: SSL1-55; SSL1-60; SSL1-65; SSL1-70; SSL1-75; SSL1-80; SSL1-85; SSL1-90;	glove	STD Revertex brand Natural rubber latex	TERANG NUSA SDN. BHD.
		Polymeric coated	2003d brand PLA (poylactide)	TERANG NUSA SDN. BHD.
2	Surgical latex double chlorinated sterile powder free hypoallergenic anatomically shaped polymer coated gloves with micro-textured surface: SSL2-55; SSL2-60; SSL2-65; SSL2-70; SSL2-75; SSL2-80; SSL2-85; SSL2-90	glove	STD Revertex brand Natural rubber latex	TERANG NUSA SDN. BHD.
		Polymeric coated	2003d brand PLA (poylactide)	TERANG NUSA SDN. BHD.

6.4 Features

This product has the following features:

- sterile;
- micro-textured surface;
- powder free:
 - polymeric coated – PLA (poylactide): SSL1-55; SSL1-60; SSL1-65; SSL1-70; SSL1-75; SSL1-80; SSL1-85; SSL1-90;
 - double chlorinated: SSL2-55; SSL2-60; SSL2-65; SSL2-70; SSL2-75; SSL2-80; SSL2-85; SSL2-90;
- Single use;
- hypoallergenic: SSL2-55; SSL2-60; SSL2-65; SSL2-70; SSL2-75; SSL2-80; SSL2-85; SSL2-90.

The texture of the outer surface can improve the capture of the medical instrument.

In order to avoid sticking of the gloves, we conduct modification of the surface using chlorine. Useful dyes, non-toxic coatings. Agents PLA (poylactide) used for the surface treatment biodegradable.

6.5 Chemicals permeability

Code Letter	Chemicals	Typical Breakthrough time
A	Methanol (CAS № 7664-93-9)	Before 15 minutes
B	Acetone (CAS № 67-64-1)	After 5 minutes
C	Acetonitrile (CAS № 75-05-08)	Before 15 minutes
D	Dichloromethane (CAS № 75-09-2)	Before 15 minutes
E	Carbon disulfide (CAS № 75-15-0)	After 240 minutes
F	Toluene (CAS № 108-88-3)	Before 5 minutes
G	Diethylamine (CAS № 109-89-7)	Before 30 minutes
H	Tetrahydrofuran (CAS № 109-99-9)	Before 15 minutes
I	Ethyl acetate (CAS № 141-78-6)	Before 45 minutes
J	n-Heptane (CAS № 142-85-5)	Before 15 minutes
K	Sodium Hydroxide 40% (CAS № 1310-73-2)	After 5 minutes
L	Sulfuric 96% (CAS № 7664-93-9)	Before 5 minutes
-	Formaldehyde 15% (CAS № 50-00-0)	Before 15 minutes

7. DISPOSAL AND ENVIRONMENTAL PROTECTION

Gloves must be assembled and transferred to a special licensed organizations engaged in the disposal of medical waste. Disposal should be carry out in accordance with local legislation current standards and requirements.

8. TRANSPORTATION AND STORAGE.

8.1 The transportation of the gloves should be carry out by any means of transport in covered vehicles in accordance with rules of cargo transportation valid for this transport.

8.2. If transported at temperature below zero, the gloves should be stored at storage conditions within 1 day before unpacking.

8.3 Gloves should be stored packed in rooms at temperature from +5 °C up to +37 °C and relative humidity not above 80%.

During storage gloves should be protected from direct sunlight, should be placed at the distance 1 m from heating appliances, should not be impacted by oil, gasoline, kerosene (and other organic solvents), as well as other corrosive substances.

9 PACKAGE CONTENTS

#	Description	Quantity, pc.
1	Surgical sterile latex powder free gloves «Clean + Safe», *	1 pair
2	Instruction manual	1
3	Warranty	1

*- the customer can choose models, the list see in Annex 1

10. MARKING

10.1 The symbols used in marking.

- Serial number

- Lot No:


(number)

- " SINGLE USE ONLY "

- " Manufacturing date "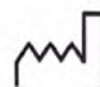

(year/month)

- " Best by ... "

(year/month)

- "STERILE UNLESS PACKAGE IS OPENED OR DAMAGED "

- "The product is made of natural latex rubber, it may cause allergy ".

- "Irradiation sterilization "

- " Powder free"

11. MANUFACTURER WARRANTY

The manufacturer guarantees compliance of the gloves to manual requirements if the operating, transportation and storage conditions are satisfied by a customer.

Guaranteed shelf life for the gloves is 5 years from the day of manufacturing.

12. MANUFACTURER INFORMATION:

TERANG NUSA SDN. BHD., 1, Jalan 8, Pengkalan Chepa 2 Industrial Zone, 16100 Kota Bharu, Kelantan, Malaysia. Tel./Fax: +609 7747171/+609 7747757, E-MAIL sales@terangnusa.com.

13. RESPONSIBLE SUPPLIER:

Limited Liability Company "GLOMAKO" (GLOMAKO LLC), 196128, Russia, Saint-Petersburg, Varshavskaya street, house 5, building 2A, office 604-A Telephone +7 (812)6358898

Annex 1

I. Surgical sterile latex powder free gloves «Clean + Safe», possible types:

1. Surgical latex sterile powder free anatomically shaped polymer coated gloves with micro-textured surface: SSL1-55; SSL1-60; SSL1-65; SSL1-70; SSL1-75; SSL1-80; SSL1-85; SSL1-90;

2. Surgical latex double chlorinated sterile powder free hypoallergenic anatomically shaped polymer coated gloves with micro-textured surface: SSL2-55; SSL2-60; SSL2-65; SSL2-70; SSL2-75; SSL2-80; SSL2-85; SSL2-90.

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