APEXMED

Surgical Gloves APEXMED Latex, Sterile, Powder Free

Operating Manual

Apexmed International bv
Keizersgracht 62-64
1015 CS Amsterdam
The Netherlands

t +31 (0)20 520 7560
f +31 (0)20 520 7510
e info@apexmed.nl
w www.apexmed.eu

Bank ING
IBAN € NL96INGB0674235010
IBAN \$ NL34INGB0020286279
BIC INGBNL2A

KvK 34159173 Amsterdam
BTW 810618114-B01

1. Intended Use.

Surgical gloves APEXMED latex, sterile, powder free are the medical devices intended for personnel and patient safety. The gloves are intended for medical personnel and patient protection from cross contamination during various surgical operations (and equal procedures), notably for hands biological protection from contamination. The gloves are intended for contact with patients and prevention of cross contamination between doctor and patient. The gloves feature corresponding sensitivity and comfort characteristics.

The gloves are resistant to blood impact, semen or cervical secretions, other biological liquids and chemical agents.

2. Indications and contraindications to the use of gloves

Indication:

- possible contact with blood;
- contact with semen or cervical secretions;
- contact with the mucous membranes of the nose, mouth;
- contact with a biological fluid, regardless of the blood presence;
- any violation of the integrity of the skin of the patient or medical personnel.

Surgical Gloves APEXMED latex, sterile, powder free made with application of technology MPXX do not cause allergic reactions, are recommended for use in people with sensitive skin and allergies to latex proteins (Allergy 1st type) and negative reactions to the powder

Contraindications and side effects:

No.

3. Safety Measures.

- All manipulations connected with risk of hands contamination with blood or serum or chemical agents, must be performed with gloves;
- All injures on hands must be covered with plaster, watertight dressing;
- In case of skin contamination with blood it must be cleaned as soon as possible considering patient's safety:
- take off the gloves;

- treat the skin with pad, wet with 70% alcohol;
- wash the skin with soap and dry with swab;
- repeat treatment with 70% alcohol.

4. Working Principle.

Surgical gloves APEXMED latex, sterile, powder free are intended for all kinds of surgical operations. Due to excellent combination of gloves thickness and strength, the gloves are recommended to use in microsurgery, ophthalmology, neurosurgery, cardiology, otolaryngology, urology and other surgical fields, requiring powder absence and high sensitivity during operation.

The device serves as protection barrier, impervious for biological agents, wearing on personnel hands during surgical operations (and equal manipulations).

5. Main Characteristics.

Surgical gloves APEXMED, latex, powder free, available types:


1. Surgical gloves APEXMED Dian, latex, sterile, powder free, textured, anatomic, non-allergic, MPXX technology, color: natural, sizes: 5,0; 5,5; 6,0; 6,5; 7,0; 7,5; 8,0; 8,5; 9,0; 9,5.


2. Surgical gloves APEXMED Aver, latex, sterile, powder free, textured, anatomic, non-allergic, MPXX technology, extra comfort, color: natural, sizes: 5,0; 5,5; 6,0; 6,5; 7,0; 7,5; 8,0; 8,5; 9,0; 9,5.

Gloves have a micro-texture that ensures a secure grip of tools.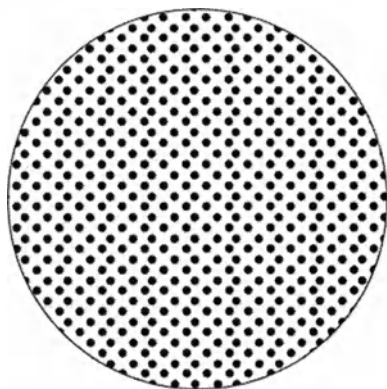

Dian

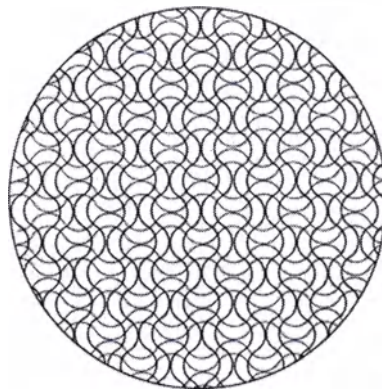Aver

Textured pattern gloves Aver contributes to a more tight retaining of the instrument in the surgeon's hand, providing extra comfort when using

Materials applied for production of surgical gloves APEXMED latex, sterile, powder free:

1. MPXX-technology cleaned latex.

MPXX technology (Maximum Protection) – is an advanced technology process of cleaning from proteins, including clinically significant (causing latex allergy of I type). Besides application of MPXX technology allows to clean the gloves from chemical additives, catalysts and coloring agents causing allergic contact dermatitis (IV type). The technology allows obtaining of material without extra covering, containing of natural latex without proteins and chemical agents. Products made by MPXX technology cause no allergy, are recommended for use among people with sensitive skin and allergy.

Compounds: natural latex, ZDBC, ZDEC, potassium hydroxide, zinc oxide, titanium dioxide, sulphur, surfactant, calcium carbonate, bentonitic clay, ionol, potassium laurate, dispersing additive.

Sterilization

Medical gloves sterilization is performed by Gamma-radiation method. This method allows sterilization of products in sealed packages and prevents repeated infection.

Strength and airtightness requirements

No.	Medical device name	Strength testing results	Airtightness testing results
1	Surgical gloves APEXMED Dian, latex, sterile, powder free, textured, anatomic, non-allergic, MPXX technology, color: natural, sizes: 5,0; 5,5; 6,0; 6,5; 7,0; 7,5; 8,0; 8,5; 9,0; 9,5	shelf life beginning: force at break: min 12,5 H, elongation at break: min 750%, shelf life end: force at break: min 9,5 H, elongation at break: min 560%	when gloves are filled with 1000 ml water there is no leakage.
2	Surgical gloves APEXMED Aver, latex, sterile, powder free, textured, anatomic, non-allergic, MPXX technology, extra comfort, color: natural, sizes: 5,0; 5,5; 6,0; 6,5; 7,0; 7,5; 8,0; 8,5; 9,0; 9,5	shelf life beginning: force at break: min 12,5 H, elongation at break: min 750%, shelf life end: force at break: min 9,5 H, elongation at break: min 560%	when gloves are filled with 1000 ml water there is no leakage.

Size table

No.	Medical device name	Size	Width, mm	Length, min (mm)	Thickness, min (mm)
1	Surgical gloves APEXMED Dian, latex, sterile, powder free, textured, anatomic, non-allergic, MPXX technology, color: natural, sizes: 5,0; 5,5; 6,0; 6,5;	5.0	67±4	250	0,13 ±0,03
		5.5	72±4	250	
		6.0	77±5	260	
			83±5	260	

No.	Medical device name	Size	Width, mm	Length, mm (mm)	Thickness, mm (mm)
	7,0; 7,5; 8,0; 8,5; 9,0; 9,5	6.5			
		7.0	89±5	270	
2	Surgical gloves APEXMED Aver, latex, sterile, powder free, textured, anatomic, non-allergic, MPXX technology, extra comfort, color: natural, sizes: 5,0; 5,5; 6,0; 6,5; 7,0; 7,5; 8,0; 8,5; 9,0; 9,5	7.5	95±5	270	
		8.0	102±6	270	
		8.5	108±6	280	
		9.0	114±6	280	
		9.5	121±6	280	

6. Work order with medical surgical gloves.

1. Treat hands.
2. Open the box, take a pair of gloves and unfold the pack.
3. Take glove for right hand by flap preventing touching of inside glove surface.
4. Close fingers of the right hand and insert them into the glove.
5. Put on the gloves not disturbing its flap opening fingers of the right hand.
6. Take the left glove with 2nd, 3rd and 4th fingers of the right hand (in glove) by the flap preventing touching of inside glove surface.
7. Close fingers of the left hand and insert them into the glove.
8. Smooth flaps on the left and then on the right hand pulling them on the sleeves.

Steps of taking off the gloves

- 1 To make a cuff on the left glove by the fingers of the right hand, touching only the outside (preventing contamination of the forearm skin or sleeves of the gown if gloves cover the lower part of the sleeves).
- 2 Take off the glove from the left hand, inverting it inside out and holding the cuff. Keep it in the right hand (preventing contamination of the environment).
- 3 To take the left glove by the left hand by the cuff on the inside (preventing contamination of the left hand).
- 4 Take off the glove from the right hand, inverting it inside out so the left glove appeared in the middle of the right (preventing contamination of

- the environment, providing infectious safety)
- 5 To put the gloves in the disinfectant solution (preventing distribution of microorganisms in the environment).
- 6 Utilize.

7. Disinfection and Disposal.

According to the current regulations, disposable gloves are subject to disinfection and disposal after use.

Immediately after use medical gloves are taken off and immersed in disinfectant solution directly in the place of use. For this purpose the following solutions are used: 6% hydrogen peroxide, 3% solution of bleach, 1.5% sodium hypochlorite, calcium 2% solution lizoformita 3000. Exposure to these disinfectants is 60 minutes.

After disinfection disposable gloves must be disposed.

Surgical gloves latex, sterile, powder free are biohazard waste. Treatment and utilization according to the set rules must be performed.

Gloves utilization is performed by means of packs, bags and containers similar to utilization of dressing materials, after preliminary disinfection. It is necessary to consider type of material for disposal.

It is not recommended to burn natural latex gloves, since toxic agents (sulfur dioxide) may form if incorrect conditions are created during combustion.

8. Operation environment Surgical Gloves APEXMED Latex, Sterile, Powder Free

- for single use;
- shelf-life – 5 years;
- protect from moisture and direct sunlight;
- storage temperatures +5 to +30 °C;
- do not use after validity term is expired.

Manufacturer assures all gloves' properties safety within 5 years from manufacture date if package is not damaged and storage conditions are kept.

9. Handling.

Transportation of the packed gloves should be carried out by all kinds of transport in covered vehicles in accordance with the rules of transportation of goods, applicable to each type of transport. Transportation by rail is carried out by the container shipments.

10. Manufacturer:

«Apexmed International B.V.»

Keizersgracht, 62-64, 1015 CS Amsterdam, the Netherlands

www.apexmed.nl, www.sensitec.nl

Production Sites:

Apexmed International B. V., Keizersgracht, 62-64, 1015 CS Amsterdam, the Netherlands.

TG Medical Sdn. Bhd., Lot 5091 Jalan Teratai, Batu 5, off Jalan Meru, 41050 Klang, Selangor D.E., Malaysia

11. Organization, authorized by the manufacturer on the acceptance and satisfaction of the requirements from consumers regarding products of inadequate quality:

"Medicom Co., Ltd.", Russian Federation, 143570 Moscow region, Novopetrovskoe settlement, Pervomayskaya str., 59, Tel./Fax: +7 (495) 651-91-16, email: medicom@inbox.ru

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