



COVIDIEN

**Общество с ограниченной ответственностью
«Ковидиен Евразия»**

105120, Россия, г.Москва, переулок Сыромятнинский 2-й, дом 1 Тел.: (495) 9336469
Факс: (495) 9336468


ИНН 7710385950, КПП 772501001, ОГРН 1027739579284, ОКНО 17163858

Фотографические изображения медицинского изделия:


Генератор электролигирующий Valleylab LS10 серии LS с одним
инструментальным выходом с принадлежностями




Генератор Valleylab серии LS

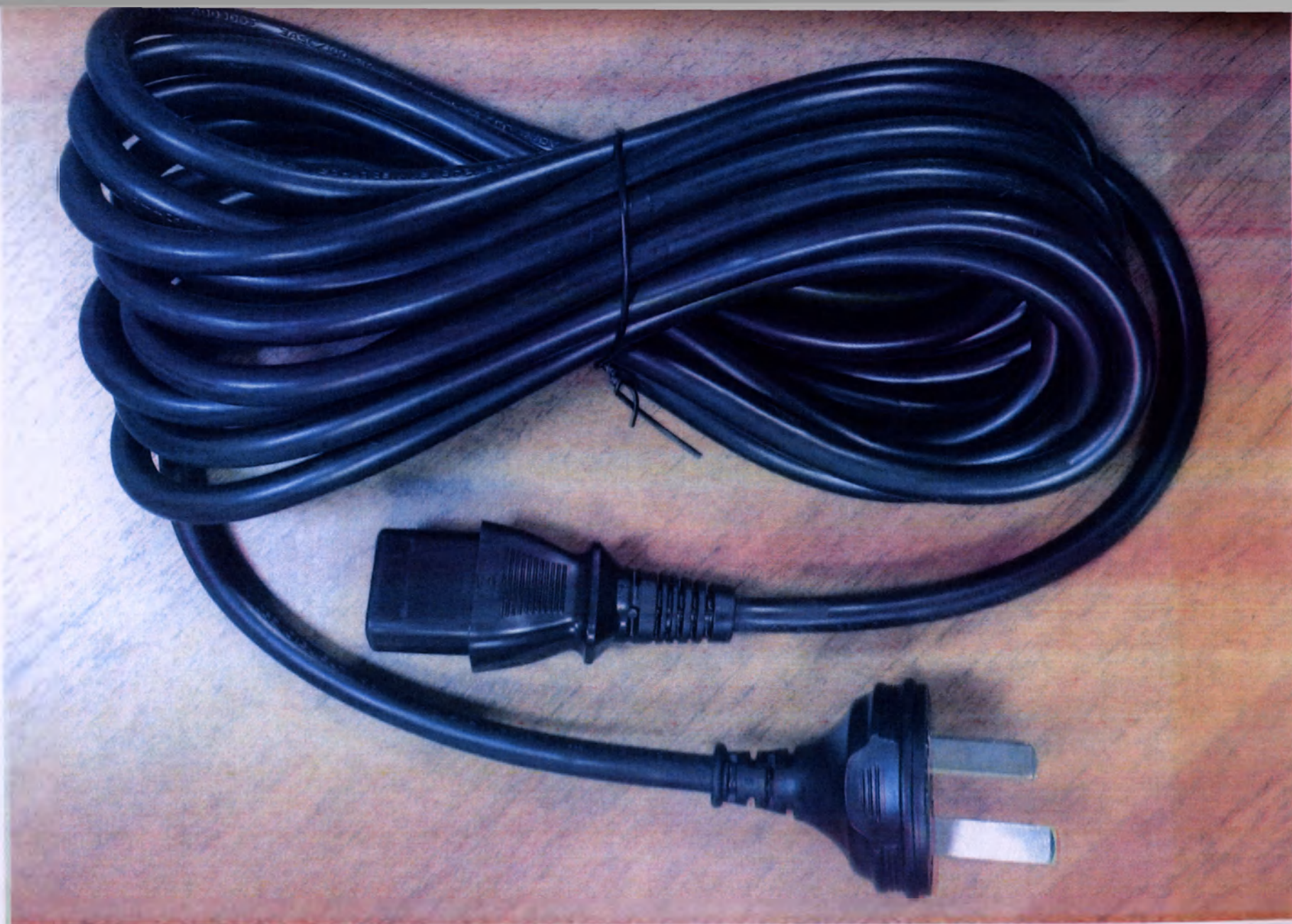

Кабель сетевой

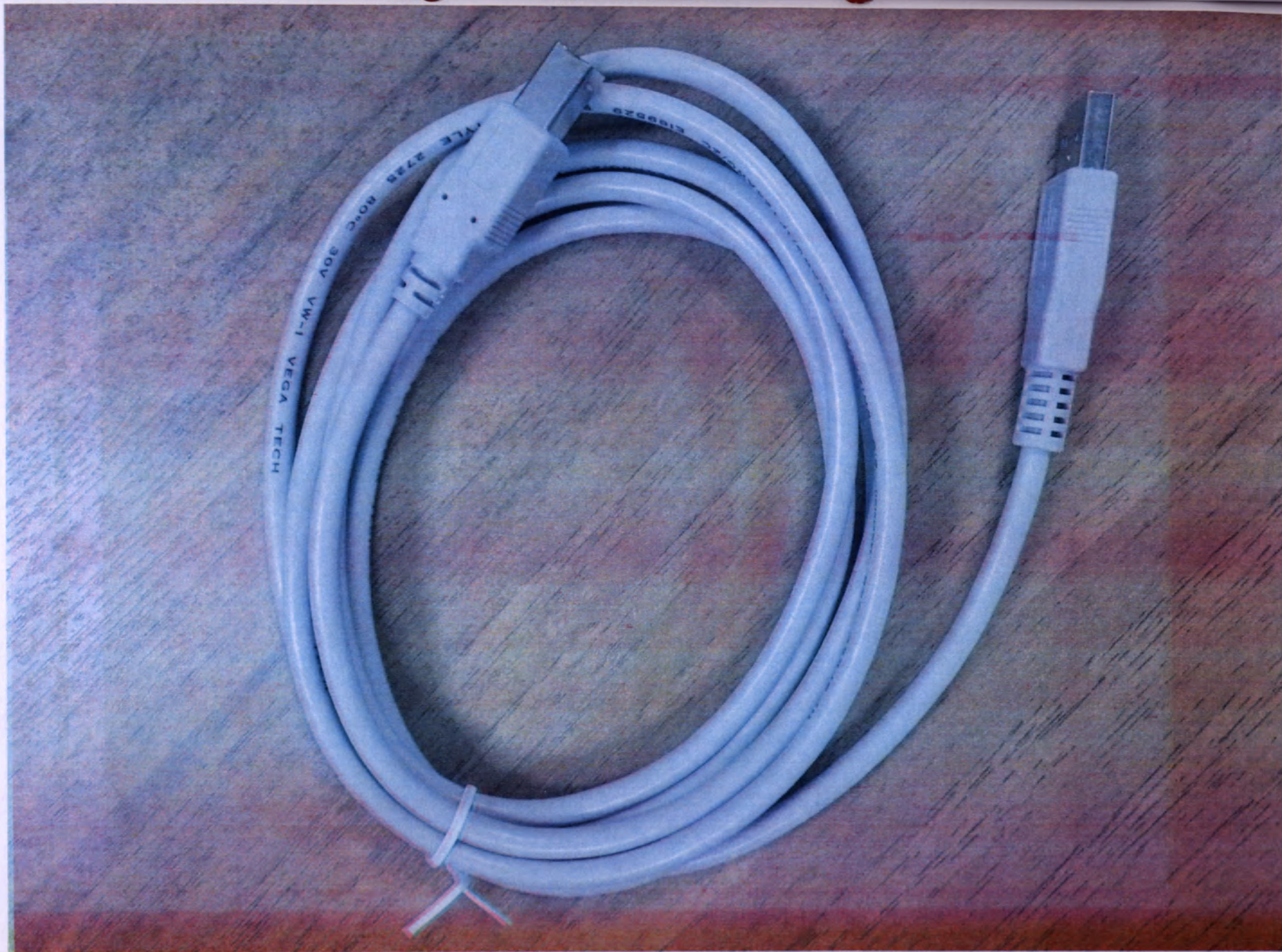

USB кабель

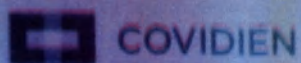

User's Guide

Valleylab™ LS10

LS Series Single Channel Vessel Sealing Generator

Valleylab LS10 Generator User's Guide
English
1070614

Руководство пользователя на бумажном носителе

Valleylab™ LS10, LS Series Single Channel Vessel Sealing Generator

Quick Reference Card

FRONT PANEL



Power button



Power on: Push and release. **Power off:** Push and hold for 3 seconds.

Instrument port



Connect LigaSure™ instrument when system status indicator is white.

Instrument status or switch stuck indicator



Red: Instrument error or hand/foot switch stuck. Instrument may be invalid, damaged, or incompatible. Check that the compatible LigaSure instrument is in proper working condition and no activation switches are pressed.

Instrument compatibility - the following LigaSure instruments are NOT compatible with this generator

LS1200	LS1500/1520	LS2071	LS2111	LS3112	LS3092	LS3544

System status indicator



White: In motion - System is performing a self test.
Constant - Ready for use, insert LigaSure instrument.
Flashing - System in service mode, power cycle system for clinical use.



Purple: Ready for sealing / sealing complete.



Purple in motion: Sealing in process.



Amber: Seal cycle incomplete alert. Inspect, regrasp, and reactivate to complete the seal cycle.

Usage limit indicator



When illuminated, indicates the inserted LigaSure instrument has already been used. It has not been recertified by the original manufacturer.

System error indicator



System error in the generator. Restart the generator. If error occurs again, contact a local Covidien customer service or global technical support - email: valleylab.technicalservice@coviden.com, US/Canada: 1-800-255-8522, International: 1-303-476-6997.

REAR PANEL



Volume knob

Adjust volume of indicator tones. Volume of error and alert tones cannot be adjusted.

ECG blanking connector

(If needed) Open cover and connect ECG monitoring system. RapidVac™ Smoke Evacuator also compatible.

Footswitch port

Only compatible with the purple LigaSure LS6300 footswitch.


For other questions, please refer to the user's guide or Valleylab™ LS10 website: www.coviden.com/valleylabLS10

Covidien, Covidien with Star and Covidien logo are trademarks of Covidien AG. Other brands and trademarks are trademarks of their respective owners.

COVIDIEN

Part Number: PT50001316

Краткая справочная карта на бумажном носителе

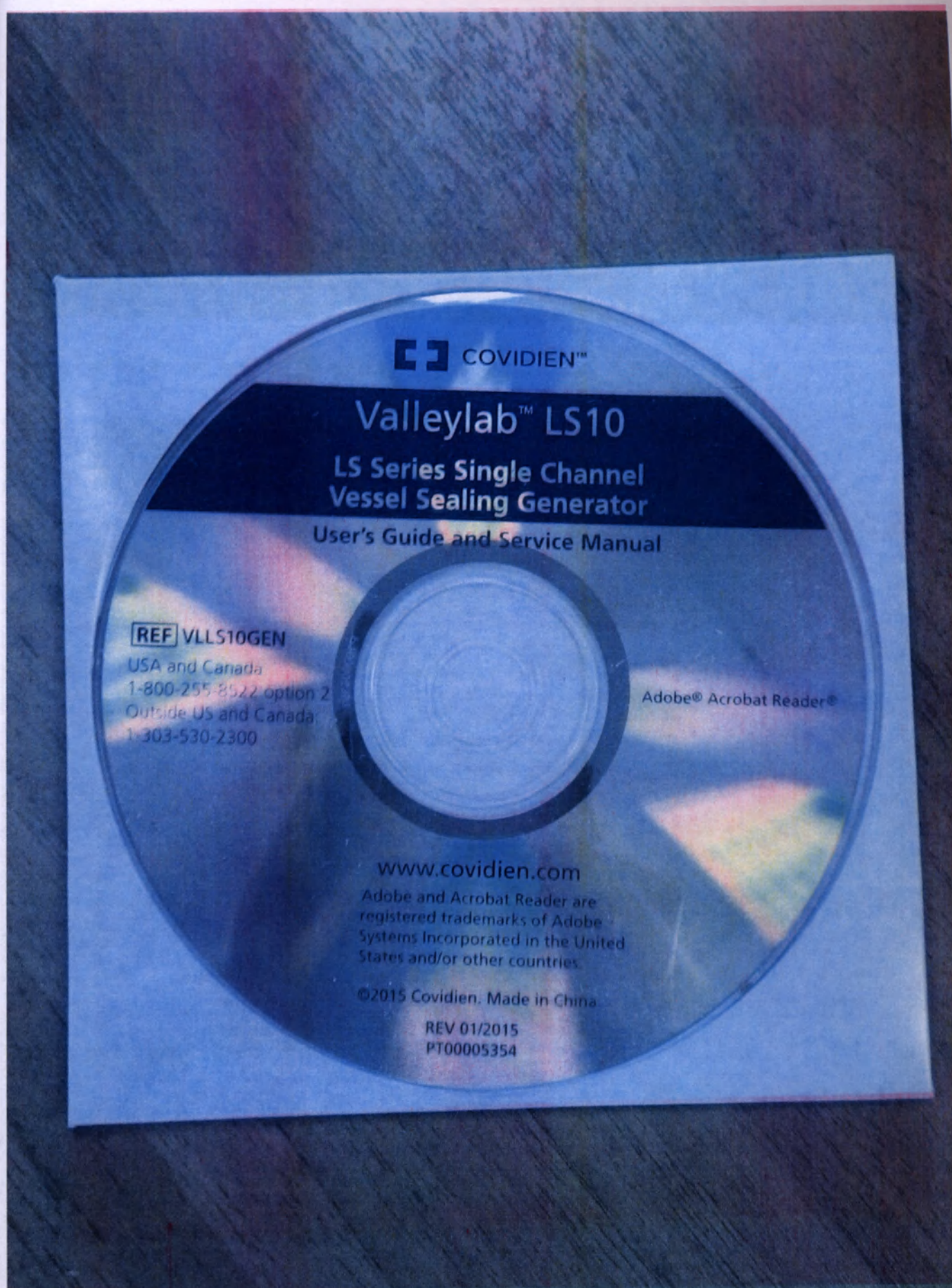

Руководство по техническому обслуживанию на CD диске

COVIDIEN ENERGY.
Performance Amplified.

Valleylab™ LS10 Generator

First-Time Setup Guide

IMPORTANT: The Valleylab LS10 generator must be set up before use. A warning symbol appears on the generator until it has been set up.



Follow the steps below to set up your generator. If you need additional help, go to www.covidien.com/valleylabexchange, select the Valleylab LS10 generator, and download the First-Time Setup Tutorial.

Downloading and Installing the Valleylab Exchange Client:



Valleylab
Exchange

1. Go to www.covidien.com/valleylabexchange. Select the Valleylab LS10 generator on the website. Download the Valleylab Exchange Installer to your desktop.
2. Locate the ValleylabExchange.exe Installer icon on your desktop. Double-click it, and follow the prompts to install the software.

NOTE: The Valleylab Exchange application works with the Windows platform. For information on which operating systems are currently supported, please refer to the Valleylab Exchange User's Guide, which can be downloaded at www.covidien.com/valleylabexchange.




VLEXClient

Set Up the Generator Location:

1. Gain access to the USB port through the back cover plate of the generator. Plug the USB A to B cable into your computer and port at the back of the generator.
2. Turn on the Valleylab LS10 generator.
3. Locate the VLEXClient icon on your desktop and double-click it to start the software.

NOTE: Do NOT login. A login and password are not required for first-time setup.

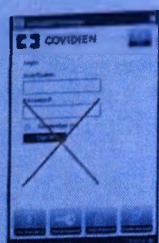

4. Click on the First Time Set-Up icon, located on the lower left corner of the Valleylab Exchange Login page.



COVIDIEN

Please see back.

Руководство по установке ПО на CD диске

Всего пронумеровано,
прошнуровано и
скреплено печатью
8 (восемь) листа (ов)

