Instructions for use

Only for dental use

Sterile Endodontic Instruments WAVEONE GOLD of Different Sizes

Manufactured by:

Maillefer Instruments Holding Sàrl, Chemin du Verger, 3, CH-1338, Ballaigues,
Switzerland. Tel.: +41(0)21 843-97 66, Fax: +41(0) 21 843-9293

1. Description
2. Photos
3. Indications
4. Contraindications
5. Precautions
6. Adverse reactions
7. Step by step instructions
8. Cleaning and sterilization
9. Conditions and term of storage, shelf- life
10. Transportation conditions.
11. Environmental safety requirements
12. Operation conditions
13. Disposal
14. Manufacturer's guarantee
15. Authorized Representative.

MAILLEFER INSTRUMENTS HOLDING Sàrl

Pierre GRAF
Chief Financial Officer

2015

1. Description

Name

Sterile Endodontic Instruments WAVEONE GOLD of Different Sizes.

Information on medical device manufacturer

Maillefer Instruments Holding Sàrl, Chemin du Verger, 3, CH-1338, Ballaigues, Switzerland.
Tel.: +41(0)21 843-97 66, Fax: +41(0) 21 843-9293.

Addresses of medical device manufacture

Maillefer Instruments Holding Sarl, Chemin du Verger, 3, CH-1338, Ballaigues, Switzerland;

Description

Sterile endodontic instruments WAVEONE GOLD of different sizes are instruments (files) for endodontic treatment, specifically for preparation and shaping of the root canal. Cutting parts of the instruments are made of heat treated nickel-titanium. Instruments are designed for single use and are supplied sterile (sterilization method - gamma radiation). These instruments are designed for use with endomotors (eg X-Smart Plus, Maillefer) and have a shank to lock in the contra-angle handles. The working part of the instruments carries a silicone stop for limiting the depth of root canal penetration.

Forms of delivery

1. Endodontic instruments WAVEONE GOLD in blisters per 4 pcs., assortment, sizes: Small, Primary, Medium, Large; Length: 21 mm.
2. Endodontic instruments WAVEONE GOLD in blisters per 4 pcs., assortment, sizes: Small, Primary, Medium, Large; Length 25 mm.
3. Endodontic instruments WAVEONE GOLD in blisters per 4 pcs., assortment, sizes: Small, Primary, Medium, Large; Length 31 mm.
4. Endodontic instruments WAVEONE GOLD in blisters per 3 pcs., sizes: Small, Primary, Medium, Large; Length 21, 25, 31 mm.
5. Endodontic instruments WAVEONE GOLD in blisters per 6 pcs., sizes: Small, Primary, Medium, Large; Length 21, 25, 31 mm.
6. Endodontic instruments WAVEONE GOLD in blisters per 1 pcs., Demo pack, size: Primary; Length 25 mm.

Materials

Component	Material
Working part	Nickel-titanium SE (Super elastic) heat treated
Handle	Brass DIN 2.0401 with Nickel/Gold plating
Label	ABS POLYLAC 737
Silicone stop	Silicone RAUMEDIC-SIK 6722

Mechanical properties

Nominal size	Maximum Bending resistance (mNm)	Minimum Torque (mNm)	Average Angle of Torsion (°)
Small	9.03	4.90	415
Primary	11.52	6.37	455
Medium	17.27	9.85	612
Large	20.98	15.00	627

Shank Roughness: Ra 1.6

Shank Hardness > 250HV5

2. Photos

Endodontic sterile instruments WAVEONE GOLD of different sizes	
Small size	
Primary size	
Medium size	
Large size	

3. Indications

WAVEONE GOLD instruments are to be used only in a clinical or hospital environment, by qualified users.

WAVEONE GOLD instruments are for use in endodontic treatment, by shaping & cleaning the root canal system.

4. Contraindications

In common with all mechanically driven root canal instruments, WAVEONE GOLD instruments should not be used in cases of severe and sudden apical curvatures.

5. Precautions

- The WAVEONE GOLD files are single-use instruments.
- Clean flutes frequently and check for signs of distortion or wear.
- Irrigate abundantly and frequently after removing any given WAVEONE GOLD file from a canal.
- Always initiate the shaping procedure with the PRIMARY WAVEONE GOLD file in the

presence of sodium hypochlorite.

- Use a gentle inward pecking motion, with short 2–3 mm amplitude strokes, to passively.

advance the WAVEONE GOLD file along a smooth, reproducible glide path.

- Remove a WAVEONE GOLD file when it does not easily progress. Clean and inspect the cutting flutes, then irrigate, recapitulate with a size 010 file and re-irrigate.
- These files are driven in precise forward/reverse reciprocating angles and must be used in reciprocating motion with dedicated Dentsply Maillefer motors.
- WAVEONE GOLD files are sold sterile and are non-autoclavable.
- WAVEONE GOLD uses a unique approach in wire processing that increases the resistance to cyclic fatigue compared to standard NiTi. Due to this processing, WAVEONE GOLD files may appear slightly curved. This is not a manufacturing defect. It is not necessary to straighten the file prior to use. Once inside the canal, the WAVEONE® GOLD file will follow the glide path, conforming to natural curvatures.

6. Adverse reactions

In common with all mechanically-driven root canal instruments, WAVEONE® GOLD files should not be used in cases of severe and sudden apical curvature due to high risk of breakage.

7. Step by step instructions

Radiographic Evaluation

Review different horizontally angulated radiographs to diagnostically determine the width, length, and curvature of any given root and canal.

Access Preparation

Create straight-line access to the canal orifice(s) with emphasis on flaring, flattening, and finishing the internal axial walls.

Precautions

- Irrigate abundantly and frequently after removing any given WAVEONE® GOLD file from a canal.
- Use a gentle inward pecking motion, with short 2–3 mm amplitude strokes, to passively advance the WAVEONE GOLD file along a smooth, reproducible glide path.
- Remove a WAVEONE GOLD file when it does not easily progress. Clean and inspect the cutting flutes, then irrigate, recapitulate with a size 010 file and re-irrigate.

WAVEONE GOLD SHAPING TECHNIQUE

- 1) Establish straight-line coronal and radicular access.
- 2) In the presence of a viscous chelator, use a size 010 hand file to verify a glide path to length. In more restrictive canals, use a size 010 hand file in any region of a canal to create a glide path.
- 3) Expand this glide path to at least 0.15 mm using either a manual or dedicated mechanical file, such as PATHFILE or PROGLIDER.
- 4) ALWAYS initiate the shaping procedure with the PRIMARY file (025/07 red) in the presence of sodium hypochlorite.
- 5) Use gentle inward pressure and let the PRIMARY file passively progress through any region of the canal that has a confirmed glide path. After shaping 2-3 mm of any given canal, remove

- 
- and clean the PRIMARY file, then irrigate, recapitulate with a size 010 hand file and re-irrigate.
- 6) Continue with the PRIMARY file, in 2-3 passes, to pre-enlarge the coronal two thirds of the canal.
 - 7) Utilize a brushing motion on the outstroke to eliminate coronal interferences or to enhance shaping results in canals that exhibit irregular cross-sections.
 - 8) In more restrictive canals, use a size 010 hand file, in the presence of viscous chelator, negotiate to the terminus of the canal. Gently work this file until it is completely loose at length.
 - 9) Establish working length, confirm patency and verify the glide path.
 - 10) Expand this glide path to at least 0.15 mm using a manual or mechanical glide path file.
 - 11) Carry the PRIMARY file to the full working length in one or more passes. Upon reaching length, remove the file to avoid over-enlarging the foramen. Inspect the apical flutes; if they are loaded with dentinal debris, then the shape is finished *.
 - 12) If the PRIMARY doesn't progress then use the SMALL file (020/07 yellow) in one or more passes to working length and then use the PRIMARY file to working length to optimize the shape
 - 13) When the shape is confirmed, proceed with 3-D disinfection protocols.
- * If the PRIMARY file is loose at length with no dentinal debris in the apical flutes, continue shaping with MEDIUM file (035/06 green) and/or LARGE file (045/05 white) until the apical flutes are loaded.

8. Cleaning and sterilization

WAVEONE GOLD is intended for single-use only.

Disinfection, cleaning and sterilization procedures are not recommended.

Re-use of WAVEONE GOLD can increase the risk of cross contamination and breakage.

9. Conditions and term of storage, shelf- life

The shelf life is 5 years. Expiry date is indicated on the packaging. Sterility is preserved for the whole shelf life if the packaging seal is not broken.

No special storage conditions are required.

10. Transportation conditions.

Special transportation conditions are not applicable. The devices must be transported in accordance with the rules applicable to each mode of transport.

11. Environmental safety requirements

The product has no negative impact on the environment during its life cycle.

12. Operation conditions

Operation: +10 - +45. These instruments can not be repaired. Specific maintenance is not required.

13. Disposal

Dispose according to the local laws or regulations of a specific medical facility.

14. Manufacturer's guarantee

The manufacturer assumes no responsibility and shall not pay compensation for any damages or accidents caused by:

- Use of the instruments that are not related to this system and may impede normal operation.
- Failure to comply with instructions for use.

The user is responsible for testing material for its suitability and use for any purpose not explicitly stated in the Instructions!

If you have any questions regarding the instrument quality, please, contact the manufacturer's authorized representative.

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