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NOTARIAL CERTIFICATE

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Quanzhou Tong Huai Notary Public Office, Fujian Province, P.R.C.

NAN AN YUAN DA SANITARY PRODUCTS FACTORY

Add: South Side of North Liumei Road, Meilin Office, Nan'An City, Fujian Province, China, Zip Code: 362300

Tel.: +86 595 28122538

Fax: +86 595 22257518

Website: www.chinayuanda.com

Email: hygieneproduct@163.com

Diapers for adults "PROTECT"

User's manual

Purpose of use: Diapers for adults «Protect» (hereinafter - diapers, product) in variants:

- for moderate degree of urinary incontinence of the following sizes: S, M, L, XL;

- for severe degree of urinary incontinence of the following sizes: S, M, L, XL;

Diapers are designed for urine and fecal matter sorption of the adult patient, suffered with moderate and severe degree of incontinence, and serve as prophylactic measure to avoid pressure injuries of bedridden patients.

Description: Diapers for adults «Protect» are non-sterile disposable underwear of pants form made of absorbing material with vertical bumpers, providing extra protection against of leakage; it is wearied between legs, collect urine and fecal matter of the adult patient, suffered with incontinence. Diapers are fitted with special Velcro, allowing for ease of dressing. When buying important to consider the waist and hips of the patient. Diapers have vertical bumpers provide

Sizes: S – waist volume 60-85 cm; M - waist volume 90-120 cm; L - waist volume 100-150 cm; XL- waist volume 120-160 cm.

Diaper structure is outlining shown on Fig.1

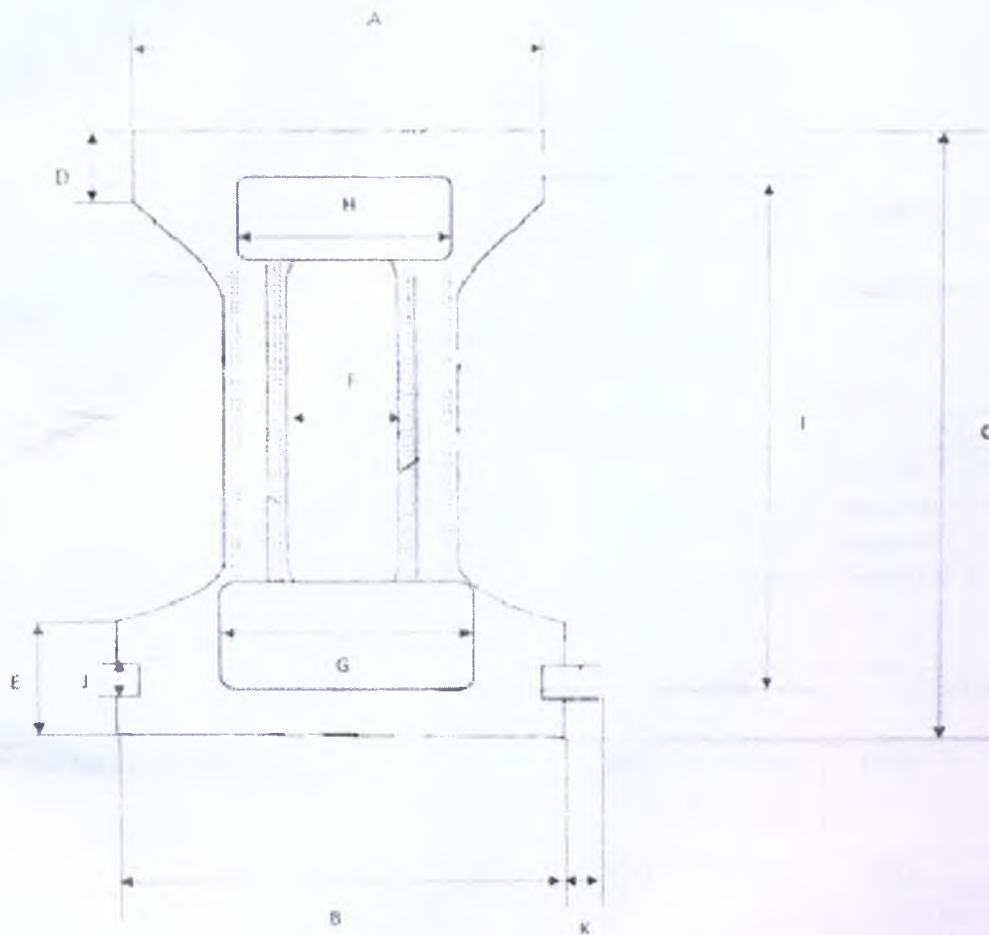
Fig.1 Diaper structure with dimensions.

A-Diaper waist frontal part's width

NAN AN YUAN DA SANITARY PRODUCTS FACTORY

Add: South Side of North Liumei Road, Meilin Office, Nan'An City, Fujian Province, China, Zip Code: 362300

Tel.: +86 595 28122538

Fax: +86 595 22257518

Website: www.chinavuanda.com

Email: hygieneproduct@163.com

- B- Diaper waist back part's width
- C- Diaper length in expanded form
- D- Diaper frontal part side's height
- E- Diaper back part side's height
- F- Absorbing layer width in the medium part
- G- Absorbing layer width in the back part
- H- Absorbing layer width in the frontal part
- I- Absorbing layer length
- J- Outpouching part of Velcro tape width
- K- Outpouching part of Velcro tape length

Tab.2 Diaper dimensions.

Parameter/ size	S	M	L	XL
A, cm within	50±0,5	64±0,5	74±0,5	83±0,5
B, cm within	50±0,5	64±0,5	74±0,5	83±0,5
C, cm within	72±0,5	80±0,5	90±0,5	99±0,5
D, cm within	18,5±0,5	19±0,5	19±0,5	20±0,5
E, cm within	22,5±0,5	23±0,5	23±0,5	25±0,5
F, cm within	17±0,3	17±0,5	17±0,5	17±0,5
G, cm within	33±0,5	36±0,5	36±0,5	36±0,5
H, cm within	28±0,5	32±0,5	32±0,5	32±0,5
I, cm within	60±0,5	68±0,5	80±0,5	80±0,5
J, cm within	2,5±0,2	2,5±0,5	2,5±0,5	2,5±0,5
K, cm within	4,5±0,2	4,5±0,5	4,5±0,5	4,5±0,5

The thickness of diaper absorbing layer for moderate degree of urinary incontinence (all sizes) is within 0,6±0,1 cm;

The thickness of diaper absorbing layer for severe degree of urinary incontinence (all sizes) is within 0,8±0,1 cm.

Tab.3 Requirements to the functional use index of adult diapers «Protect»

Size/ index	S		M		L		XL		Test method
	For the moderate degree of incontinence	For severe incontinence	For the moderate degree of incontinence	For severe incontinence	For the moderate degree of incontinence	For severe incontinence	For the moderate degree of incontinence	For severe incontinence	
Waist volume, cm	60-85		90-120		100-150		120-160		
Full liquid absorption, gr, not less	1000	1400	1300	1800	1450	2000	1450	2800	By 12.3
Absorption capacity after centrifugation, gr, not less	120	300	220	440	330	500	330	560	By 12.4
Resorption, gr,	4,4								By

NAN AN YUAN DA SANITARY PRODUCTS FACTORY

Add: South Side of North Liumei Road, Meilin Office, Nan'An City, Fujian Province, China, Zip Code: 362300

Tel.: +86 595 28122538

Fax: +86 595 22257518

Website: www.chinayuanda.com

Email: hygieneproduct@163.com

not less		12.5
Absorption time, m ³ /sec, not less	2,3	By 12.6

Requirements to the functional use index of adult diapers «Protect» are shown in Tab.3, according to size range.

Tab.4 Specification of Adult diapers «Protect»

Size/ index	S		M		L		XL	
	For the moderate degree of incontinence	For severe incontinence	For the moderate degree of incontinence	For severe incontinence	For the moderate degree of incontinence	For severe incontinence	For the moderate degree of incontinence	For severe incontinence
Diaper weight, gr within	100±10	110±10	110±10	120±10	120±10	130±10	130±10	140±10
Absorbing layer weight, gr within	56±5	58±5	57±5	60±5	62±5	65±5	67±5	70±5
SAP weight, gr within	5,5±0,5	6±0,5	6,5±0,5	7±0,5	7,5±0,5	8±0,5	8,5±0,5	9±0,5

Weight of end product 1 piece (at exact index and variant of producing) is evaluated in technical documentation and shown in Tab.4

Composition: upper covering layer (contacts the skin and temporary the mucous membrane) consists of nonwoven Hydrophilic & hydrophobic material - polypropylene PP model HPI15 (CAS № 9003-07-0), produced by ShiJiaZhuang HuanQiu Chemicals Co., Ltd, (China); distributing layer consists of tissue paper made of cellulose model WSP16 produced by Yinge, (China) and cellulose model PW416/GP CELLULOSE 4871/4872 Weyerhaeuser Company (USA); absorbing layer consists of super absorbent San-dia model IM930 (CAS №9003-04-7), produced by Sandia Corporation (Japan); protection layer consists of insulating film - polyethylene LDPE model PF22 (CAS № 9003-07-0), produced by ShiJiaZhuang HuanQiu Chemicals Co., Ltd (China); lower covering layer consists of nonwoven hydrophobic material - polypropylene PP model HPI15 (CAS № 9003-07-0), produced by ShiJiaZhuang HuanQiu Chemicals Co., Ltd (China); There are barrier elements - nonwoven Hydrophilic & hydrophobic material - polypropylene PP model HPI15 (CAS № 9003-07-0), produced by ShiJiaZhuang HuanQiu Chemicals Co., Ltd (China) and fixation elements - 3-ply tape model H6218, produced by Tekang Co. (China), elastomeric buckle - elastane model 940B, produced Hyosung (South Korea) /Asahi Kasei (Japan); layers are connected with glue - Focus Adhesive model T101/DM5305, produced by Henkel Co. Ltd (China); Composition: styrene-butadiene block copolymer (CAS №9003-55-8) – 18%; naphthenic processing oil (CAS №64742-52-5) – 24%; pentaerythritol ester of rosin (CAS № 050-26-8) – 30%; petroleum hydrocarbon resin (CAS №64742-16-1) - 27%; antioxidant -1%.

Usage: Before use, the patient should be washed away with warm water and wiped well. Put the diaper on the pelvis of the patient (if the patient is bedridden, gently put a diaper under him), flank by Velcro tape. Diaper filling time is individually - from 400 to 600 ml for 8h with a moderate degree of incontinence and more than 600 ml for 8 hours with severe incontinence.

NAN AN YUAN DA SANITARY PRODUCTS FACTORY

Add: South Side of North Liumei Road, Meilin Office, Nan'An City, Fujian Province, China, Zip Code: 362300

Tel.: +86 595 28122538

Fax: +86 595 22257518

Website: www.chinavuanda.com

Email: hygieneproduct@163.com

Contraindications: Do not use for patients with individual intolerance, the external genitalia diseases and digestive dysfunction.

Precautions: Prolonged use of the diaper can cause diaper rash. If evacuation has not happened for a few hours, the diaper should be changed to a new one.
Reusing is forbidden. Using for other purposes is forbidden.

Storage: Diapers for adult "Protect" should be stored in a clean, dry, dark place in the original package at a temperature of 2 ° C to 40 ° C.

Diapers should be stored out of the reach of children.

Do not store diapers near heaters and exposure to extreme temperatures. Do not store near objects diapers has a strong odor (household chemicals). Avoid contact with moisture on the clean diapers.

Transportation: Diapers for adult "Protect" are transported in accordance with GOST 6658 (Section 3) any kind of covered transport in accordance with the rules of carriage of goods, operating in this mode of transport. Conditions of carriage - for a group of 5 to GOST 15150 can be transported at a temperature of -40 ° C to + 40 ° C, humidity of 10-100%, atmospheric pressure 500-1060 kPa.

Manufacturer's Warranty: The manufacturer Nan An Yuanda Sanitary Products Factory, China, and it's authorized representative: Limited Liability Company "Eurasia-Trade" ("Eurasia-Trade" LLC) 660017, Krasnoyarsk region, Krasnoyarsk city, Krasnoy Army Street, 10, Building 3, room 202 www.eurasiatr.ru e-mail: info@eurasiatr.ru Tel .: +7 (391) 290-64-07 guarantees the declared diapers for adults "Protect" meet declared properties until the expiration date, subject to the rules governing the storage, use and transportation.

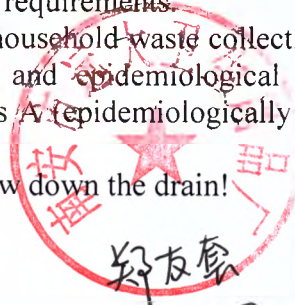
Shelf life - 3 years from manufacturing date indicated on the package. Do not use after shelf life expiration.

Diapers are one-used product and needn't service and technical support and repairing.

Utilization information: If necessary, decontamination of used diapers carried out according to Medical wastes handling requirements.

Packaging is utilized in household waste collection site. Diapers are utilized according to SanPin 2.1.7.2790-10 "Sanitary and epidemiological requirements for the management of medical waste" as wastes of class A (epidemiologically non-hazardous wastes, close to solid household wastes in composition).

Do not burn! Do not throw down the drain!

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(Translation)

(2016)MQTZ.Zi.No.416

Applicant: NAN AN YUANDA SANITARY PRODUCTS
FACTORY, of South Side of North Liumei Road, Meilin Office, Nan'An
City, with *Business License For A Sole Proprietorship Enterprise*
Registration No.: 350583100020640

Legal representative: Zheng Youtao, male

Issue under notarization: Signature and seal

This is to certify that Zheng Youtao, legal representative of NAN
AN YUANDA SANITARY PRODUCTS FACTORY, came to our notary
public office and affixed his signature and the company seal onto the
foregoing **DIAPERS FOR ADULTS "PROTECT" USER'S MANUAL**
on April 11, 2016, before me, the notary public.

Quanzhou Tonghuai Notary Public Office

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