APEXMED

Examination Gloves (Diagnostic) APEXMED Latex, Nonsterile, Powder Free

Operating Manual

Apexmed International bv
Keizersgracht 62-64
1015 CS Amsterdam
The Netherlands

t +31 (0)20 520 7560
f +31 (0)20 520 7510
e info@apexmed.nl
w www.apexmed.eu

Bank ING
IBAN € NL96INGB0674235010
IBAN \$ NL34INGB0020286279
BIC INGBNL2A

KvK 34159173 Amsterdam
BTW 810618114-B01

1. Intended Use.

Examination gloves (diagnostic) APEXMED latex, nonsterile, powder free are intended to protect patients and medical personnel from cross-contamination during various procedures. The device serves as protection barrier wearing on personnel hands during patient's examination/treatment or for other sanitary purposes. The device protects medical personnel and patients hands from contaminations. The gloves are resistant to blood impact, semen or cervical secretions, other biological liquids and chemical agents.

2. Indications for gloves applications.

- possible contact with blood;
- contact with semen or cervical secretions;
- contact with the mucous membranes of the nose, mouth;
- contact with a biological fluid, regardless of the blood presence;
- contact with objects or materials contaminated with blood or other body fluids;
- any violation of the integrity of the skin of the patient or medical personnel;
- possible contact with the patient's bedding;
- laboratory tests.

3. Safety Measures.

Usage of the gloves by persons with high sensitivity to latex is not provided. Some cases of allergic reaction have been revealed. If allergic reaction occurs, stop application of the product and appeal to doctor. The device contains natural latex and other components which may cause allergic reaction of some persons.

- All manipulations connected with risk of hands contamination with blood or serum or chemical agents, must be performed with gloves;
- All injures on hands must be covered with plaster, watertight dressing;
- In case of skin contamination with blood it must be cleaned as soon as possible considering patient's safety:
- take off the gloves;
- treat the skin with pad, wet with 70% alcohol;
- wash the skin with soap and dry with swab;

- repeat treatment with 70% alcohol.

4. Working Principle.

Examination gloves (diagnostic) APEXMED latex, nonsterile, powder free are intended to protect patients and medical personnel from cross-contamination during various procedures.

The device serves as protection barrier, impervious for biological agents, wearing on personnel hands during patient's examination/treatment or for other sanitary purposes. The device protects medical personnel and patients hands from contaminations. The gloves are resistant to blood impact, semen or cervical secretions, other biological liquids and chemical agents.

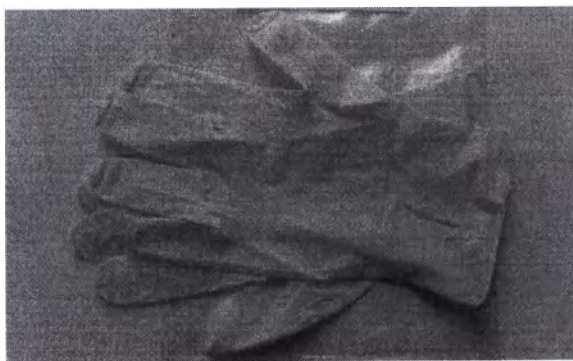
5. Main Characteristics.

Examination gloves (diagnostic) APEXMED nonsterile, powder free, are made of natural rubber latex, available textured and smooth, universal and ambidextrous, with/without polymer cover, treated or not with MPXX technology, dental, with standard cuff and elongated. Available variants are:


1. Examination (diagnostic) gloves APEXMED Campa, latex, non-sterile, powder free, smooth, ambidextrous (for both hands), non-allergic, MPXX technology, color: natural, sizes: XS, S, M, L, XL.

2. Examination (diagnostic) gloves APEXMED Sovie, latex, non-sterile, powder free, textured, ambidextrous (for both hands), non-allergic, MPXX technology, color: natural, sizes: XS, S, M, L, XL.

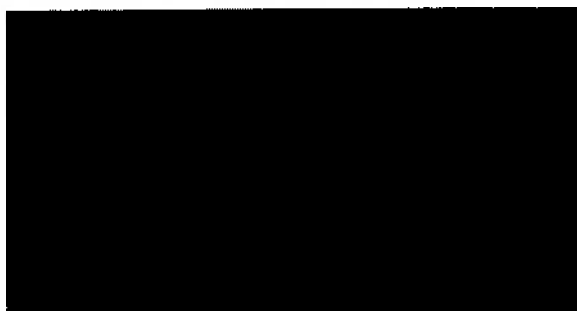
3. Examination (diagnostic) gloves APEXMED Denty dental, latex, non-sterile, powder free, textured, ambidextrous (for both hands), with polymer cover, color: green, sizes: XS, S, M, L, XL.

4. Examination (diagnostic) gloves APEXMED Mentha dental, latex, non-sterile, powder free, textured, ambidextrous (for both hands), with polymer cover, mint flavor, color: green, sizes: XS, S, M, L XL.

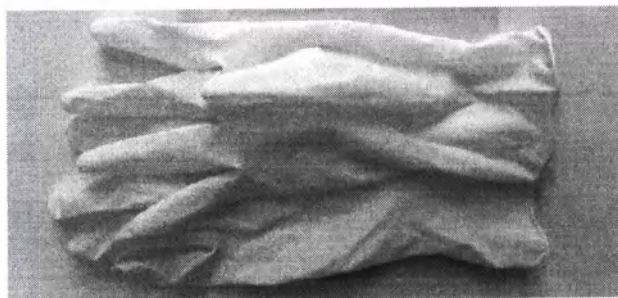
5. Examination (diagnostic) gloves APEXMED Poins latex, non-sterile, powder free, textured, ambidextrous (for both hands), with polymer cover, color: natural, sizes: XS, S, M, L, XL.

6. Examination (diagnostic) gloves APEXMED Semp, latex, non-sterile, powder free, smooth, anatomic, elongated, color: natural, sizes: XS, S, M, L, XL.

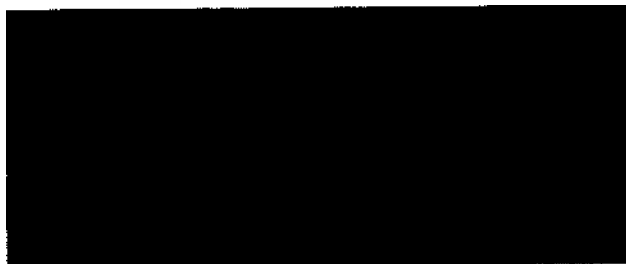7. Examination (diagnostic) gloves APEXMED Ovil, latex, non-sterile, powder free, textured, anatomic, elongated, color: natural, sizes: XS, S, M, L, XL.

8. Examination (diagnostic) gloves APEXMED Desmol, latex, non-sterile, powder free, textured, anatomic, with polymer cover, elongated, color: green, sizes: XS, S, M, L, XL.

Materials applied for production of examination (diagnostic) gloves APEXMED latex, nonsterile, powder free:

1. Natural latex – material referred to elastomeric group which is non-toxic and does not emit other harmful substances. It is high-molecular compound capable to reversible deformations (due to the fact that latex molecule has twisted spiral shape).
2. MPXX-technology cleaned latex.

MPXX-technology (Maximum Protection) – cleaning technology which allows to get material without extra covers, containing only cleaned natural latex without proteins and chemical additives. Products made by MPXX-technology are recommended for use among persons with sensitive skin and inclined to allergy.

Compounds: natural latex, ZDBC, ZDEC, polymer phenol, zinc oxide, sulphur, EPL, titanium dioxide (PW6), coloring agent (PG7).

Strength and airtightness requirements

No.	Medical device name	Strength testing results	Airtightness testing results
1	Examination (diagnostic) gloves APEXMED Campa, latex, non-sterile, powder free, smooth, ambidextrous, anallergic, MPXX-technology, color: natural, sizes: XS, S, M, L, XL	shelf life beginning: force at break: min 9 H, elongation at break: min 650%, shelf life end: force at break: min 7 H, elongation at break: min 500%, force required to achieve 300% elongation: min 6 H (shelf life beginning)	when gloves are filled with 1000 ml water there is no leakage.
2	Examination (diagnostic) gloves APEXMED Sovie, latex, non-sterile, powder free, textured, ambidextrous, anallergic, MPXX-technology, color: natural, sizes: XS, S, M, L, XL	shelf life beginning: force at break: min 9 H, elongation at break: min 650%, shelf life end: force at break: min 7 H, elongation at break: min 500%	when gloves are filled with 1000 ml water there is no leakage.
3	Examination (diagnostic) gloves APEXMED Denty, dental, latex, non-sterile, powder free, textured, ambidextrous, with polymer cover, color: green, sizes:	shelf life beginning: force at break: min 9 H, elongation at break: min 650%, shelf life end: force at break: min 7 H, elonga-	when gloves are filled with 1000 ml water there is no leakage.

No.	Medical device name	Strength testing results	Airtightness testing results
	XS, S, M, L, XL	tion at break: min 500%	
4	Examination (diagnostic) gloves APEXMED Denty, dental, latex, non-sterile, powder free, textured, ambidextrous, with polymer cover, color: green, sizes: XS, S, M, L, XL	shelf life beginning: force at break: min 9 H, elongation at break: min 650%, shelf life end: elongation at break: min 7 H, elongation at break: min 500%	when gloves are filled with 1000 ml water there is no leakage.
5	Examination (diagnostic) gloves APEXMED Mentha, dental, latex, non-sterile, powder free, textured, ambidextrous, with polymer cover, с ароматом мяты, color: green, sizes: XS, S, M, L XL	shelf life beginning: force at break: min 9 H, elongation at break: min 650%, shelf life end: force at break: min 7 H, elongation at break: min 500%	when gloves are filled with 1000 ml water there is no leakage.
6	Examination (diagnostic) gloves APEXMED Semp, latex, non-sterile, powder free, smooth, анатомические, удлиненные, color: natural, sizes: XS, S, M, L, XL	shelf life beginning: force at break: min 7,0 H, elongation at break: min 650%, shelf life end: force at break: min 6,0 H, elongation at break: min 500%	when gloves are filled with 1000 ml water there is no leakage.
7	Examination (diagnostic) gloves APEXMED Ovil, latex, non-sterile, powder free, textured, anatomic, elongated, color: natural, sizes: XS, S, M, L, XL	shelf life beginning: force at break: min 9,0 H, elongation at break: min 650%, shelf life end: force at break: min 7,0 H, elongation at break: min 500%	when gloves are filled with 1000 ml water there is no leakage.
8	Examination (diagnostic) gloves APEXMED Desmol, latex, non-sterile, powder free, textured, anatomic, with polymer cover, elongated, color: green, sizes: XS, S, M, L, XL	shelf life beginning: force at break: min 9,0 H, elongation at break: min 650%, shelf life end: force at break: min 7,0 H, elongation at break: min 500%	when gloves are filled with 1000 ml water there is no leakage.

Size table

No.	Medical device name	Size	Width, mm	Length, min (mm)	Thickness, min (mm)
1	Examination (diagnostic) gloves APEXMED Campa latex non-sterile, powder free, smooth, ambidextrous, anallergic, MPXX-technology, color: natural, sizes: XS, S, M, L, XL	XS	76 ± 3	240	Fingers: 0.12±0,02 Palm: 0.10±0,02 Cuff: 0.10±0,02
		S	84 ± 3		
		M	94 ± 3		
		L	105 ± 3		
		XL	113 ± 3		
2	Examination (diagnostic) gloves APEXMED Sovie latex, non-sterile, powder free, textured, ambidextrous, anallergic, MPXX-technology, color: natural, sizes: XS, S, M, L, XL	XS	76 ± 3	240	Fingers: 0.12±0,02 Palm: 0.10±0,02 Cuff: 0.10±0,02
		S	84 ± 3		
		M	94 ± 3		
		L	105 ± 3		
		XL	113 ± 3		
3	Examination (diagnostic) gloves APEXMED Denty dental, latex, non-sterile, powder free, textured, ambidextrous, with polymer cover, color: green, sizes: XS, S, M, L, XL	XS	76 ± 3	240	Fingers: 0.12±0,02 Palm: 0.10±0,02 Cuff: 0.10±0,02
		S	84 ± 3		
		M	94 ± 3		

No.	Medical device name	Size	Width, mm	Length, min (mm)	Thickness, min (mm)
		L	105 ± 3		
		XL	113 ± 3		
4	Examination (diagnostic) gloves APEXMED Denty dental, latex, non-sterile, powder free, textured, ambidextrous, with polymer cover, color: green, sizes: XS, S, M, L, XL	XS	76 ± 3	240	Fingers: 0.12±0,02 Palm: 0.10±0,02 Cuff: 0.10±0,02
		S	84 ± 3		
		M	94 ± 3		
		L	105 ± 3		
		XL	113 ± 3		
5	Examination (diagnostic) gloves APEXMED Mentha dental, latex, non-sterile, powder free, textured, ambidextrous, with polymer cover, mint flavor, color: green, sizes: XS, S, M, L XL	XS	76 ± 3	240	Fingers: 0.12±0,02 Palm: 0.10±0,02 Cuff: 0.10±0,02
		S	84 ± 3		
		M	94 ± 3		
		L	105 ± 3		
		XL	113 ± 3		
6	Examination (diagnostic) gloves APEXMED Semp latex, non-sterile, powder free, smooth, anatomic, elongated, color: natural,	XS	76 ± 3	270	Fingers 0.17±0,02 Palm
		S	84 ± 3		

No.	Medical device name	Size	Width, mm	Length, min (mm)	Thickness, min (mm)
	sizes: XS, S, M, L, XL				0.15±0,02
		M	94 ± 3		Cuff 0.13±0,02
		L	105 ± 3		
		XL	113 ± 3		
7	Examination (diagnostic) gloves APEXMED Ovil latex, non-sterile, powder free, textured, anatomic, elongated, color: natural, sizes: XS, S, M, L, XL	XS	76 ± 3	270	Fingers 0.17±0,02 Palm 0.15±0,02 Cuff 0.13±0,02
		S	84 ± 3		
		M	94 ± 3		
		L	105 ± 3		
		XL	113 ± 3		
8	Examination (diagnostic) gloves APEXMED Desmol latex, non-sterile, powder free, textured, anatomic, with polymer cover, elongated, color: green, sizes: XS, S, M, L, XL	XS	76 ± 3	260-280	Fingers: 0.12±0,02 Palm: 0.10±0,02 Cuff: 0.09±0,02
		S	84 ± 3		
		M	94 ± 3		
		L	105 ± 3		

No.	Medical device name	Size	Width, mm	Length, min (mm)	Thickness, min (mm)
		XL	113 ± 3		

6. Work order with medical examination (diagnostic) gloves

1. Treat hands.
2. Open the box and get a pair of gloves.
3. Take one of the gloves, shake to spread.
4. Then either with some help or by yourself put the right hand into the glove and pull until it fit tightly.
5. Repeat the same actions with your right hand.
6. Take off the gloves after the procedure.

Steps of taking off the gloves

- 1 To make a cuff on the left glove by the fingers of the right hand, touching only the outside (preventing contamination of the forearm skin or sleeves of the gown if gloves cover the lower part of the sleeves).
- 2 Take off the glove from the left hand, inverting it inside out and holding the cuff. Keep it in the right hand (preventing contamination of the environment).
- 3 To take the left glove by the left hand by the cuff on the inside (preventing contamination of the left hand).
- 4 Take off the glove from the right hand, inverting it inside out so the left glove appeared in the middle of the right (preventing contamination of the environment, providing infectious safety)
- 5 To put the gloves in the disinfectant solution (preventing distribution of microorganisms in the environment)
- 6 Utilize. Gloves utilization is performed by means of plastic bags, packs and containers as utilization of dressing materials after preliminary disinfection. It is necessary to consider type of material for disposal. It is not recommended to burn natural latex gloves, since wrong combustion conditions may lead to toxic staff formation (sulphur dioxide).

7. Disinfection and Disposal.

According to the current regulations, disposable gloves are subject to disinfection and disposal after use.

Immediately after use medical gloves are taken off and immersed in disinfectant solution directly in the place of use. For this purpose the following solutions are used: 6% hydrogen peroxide, 3% solution of bleach, 1.5% sodium hypochlorite, calcium 2% solution lizoformita 3000. Exposure to these disinfectants is 60 minutes.

After disinfection disposable gloves must be disposed.

Examination gloves (diagnostic) latex, nonsterile, powder free are biohazard waste. Treatment and utilization according to the set rules must be performed.

Gloves utilization is performed by means of packs, bags and containers similar to utilization of dressing materials, after preliminary disinfection.

It is not recommended to burn natural latex gloves, since toxic agents (sulfur dioxide) may form if incorrect conditions are created during combustion.

8. Operation environment.

- for single use;
- shelf-life – 5 years;
- protect from moisture and direct sunlight;
- storage temperatures +5 to +30 °C;
- do not use after validity term is expired.

Manufacturer assures all gloves' properties safety within 5 years from manufacture date if package is not damaged and storage conditions are kept.

9. Handling.

Transportation of the packed gloves should be carried out by all kinds of transport in covered vehicles in accordance with the rules of transportation of goods, applicable to each type of transport. Transportation by rail is carried out by the container shipments.

10. Manufacturer:

«Apexmed International B.V.»

Keizersgracht, 62-64, 1015 CS Amsterdam, the Netherlands

www.apexmed.nl, www.sensitec.nl

Production Sites:

Apexmed International B. V., Keizersgracht, 62-64, 1015 CS Amsterdam, the Netherlands.

TG Medical Sdn. Bhd., Lot 5091 Jalan Teratai, Batu 5, off Jalan Meru, 41050 Klang, Selangor D.E., Malaysia

11. Organization, authorized by the manufacturer on the acceptance and satisfaction of the requirements from consumers regarding products of inadequate quality:

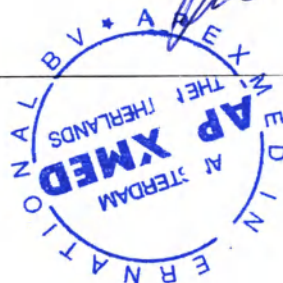
"Medicom Co., Ltd.", Russian Federation, 143570 Moscow region, Novopetrovskoe settlement, Pervomayskaya str., 59, Tel./Fax: +7 (495) 651-91-16, email: medicom@inbox.ru

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