Directions for use

For dental use only

Ceram.x one universal dental material with accessories

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1 Description

1.1 Name

Ceram.x one universal dental material with accessories.

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Ceram.x one universal dental material – the Single Translucency System, comprises seven shades of intermediate translucency comparable to conventional composites optimal for fast and easy restorations of anterior and posterior teeth.

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1.6 Delivery forms

Ceram.x one universal dental material is delivered in 3 g syringes and has different shades: a1, a2, a3, a3.5, a4, c1, c2.

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DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

1.7 Composition and materials

1.7.1 *ceram.x one universal dental material in 3 g syringes of different shades: a1, a2, a3, a3.5, a4, c1, c2.*

- Methacrylate modified polysiloxane (organically modified ceramics)
- Ethoxylated bisphenol-A-dimethacrylate
- Dodecandioldi-methacrylate (DDDMA)
- EGAMA-ATMPO adduct
- Camphor quinone
- Ethyldimethylamino-benzoate (DMAEB)
- 2,5-Dihydroxybenzene-1,4-dicarboxylic acid diethylester (Lumilux)
- 2-hydroxy-4-methoxybenzophenone
- Highly dispersed silicon dioxide двуокись кремния
- Barium-alumo-boro-silicate glass
- Titanium dioxide
- Black iron oxide
- Yellow iron oxide
- Red iron oxide.

1.7.2 *Syringe materials:*

Component	Material
Barrel, piston seal, plunger, cap	Polypropylene
Screw	Polybutylene terephthalate

1.7.3 *Dentin & Enamel Shade Guide*

Component	Material
Shade Guide Holder	Polybutylenterephthalat PBT Pocan 1505, BASF AG
Shade (fingers)	Polycoarbonat Trirex 3022 IR

1.8 Technical specifications of *ceram.x one universal*

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Withstand irradiation with ambient light, showing no pre-polymerization for at least 100 seconds (for all shades).

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2 Images

2.1 Ceram.x one universal dental material in 3 g syringes of different shades: a1, a2, a3, a3.5, a4, c1, c2 (from left to right).

2.2 Dentin & Enamel Shade Guide.

2.3 iShade label.

3 Indications

Ceram.x one universal dental material is designed for restoration of anterior and posterior teeth. Products should be used only by qualified medical personnel in clinical institutions.

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7 Step-by-step instructions

7.1 Shade selection

Ceram.x one Universal restorative offers seven shades of intermediate translucency (a1-a4, c1, c2) and is especially suitable for fast, single-shade treatment procedures.

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- The VITAPAN 2 classical shade guide together with the shade identification label (iShade label). With the enclosed iShade label you may select the suitable ceram.x one Universal shade(s) corresponding to the VITA shade range.
- The ceram.x one shade tabs which are made from original ceram. x one restorative material.

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1. Fix the delivered iShade label at the rear side of the VITAPAN classical shade guide.
2. Select the VITAPAN classical shade to be matched.
3. Select the corresponding ceram.x one shade according to the recipe on the iShade label.
4. ceram.x one universal restorative. Each VITA shade is reproduced by one of the seven ceram.x one universal shades.

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To select the tooth color you may use the delivered shade guides for ceram.x one Universal restorative.

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1. Select the ceram.x one shade most closely matching the color of the tooth. Consider the central region of the tooth.

7.1.3 Shade selection hints

- Clean the tooth surface with prophylaxis paste to remove plaque or surface stain.
- Select the shade while teeth are hydrated.

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

7.2 Cavity preparation

1. Clean the tooth surface with a fluoride free prophylaxis paste (e.g. Nupro prophylaxis paste (Dentsply Professional, USA)).
2. Prepare cavity (unless not necessary, e.g. cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure

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7.6 Applying ceram.x one universal restorative

1. Fill the whole cavity incrementally. Each layer ≤ 2 mm.
2. Cure each layer according to the table below [Light-curing].

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Damage to the device.

1. Do not clean or disinfect with aggressive agents (e.g. solutions based on chlorine, glutaraldehyde, phenolic solvents, iodophors, alcohol, orange oil, or acetone) as they may affect the color shade over time.
2. Always make sure individual tabs are completely reprocessed before placing back into shade guide holder.
3. Never autoclave the individual tabs.

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9 Storage and shelf life

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

Store the product in a ventilated place protected from light.

Store in sealed packs at temperatures between 6 °C and 28 °C

Keep out of direct sunlight and protect from moisture.

Do not freeze.

Do not use after expiration date:

Shelf life – 2 years

Humidity can adversely affect the properties of the material.

For un-sealed syringes, the following is recommended:

Store in a dry environment (< 80% relative humidity);

Use within 3 months.

10 Transportation.

The devices should be transported according to the rules applicable to each transport mode at temperatures between 6 °C and 28 °C and relative humidity < 80%.

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Directions for use

For dental use only

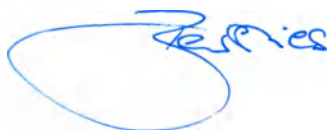
Ceram.x one dentin & enamel dental material with accessories

manufactured by

«DENTSPLY DeTrey GmbH», De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-192; +49-(0)7531-583-660-192

- 1 Description
- 2 Images
- 3 Indications
- 4 Contradictions
- 5 Warnings
- 6 Adverse events
- 7 Step-by-step instructions
- 8 Cleaning and disinfection
- 9 Storage and shelf life
- 10 Transportation.
- 11 Environmental requirements
- 12 Condition of use
- 13 Disposal
- 14 Manufacturer's warranties
- 15 Authorized Representative

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

2015

1 Description

1.1 Name

Ceram.x one dentin & enamel dental material with accessories.

1.2 Description

Ceram.x one dental material – is a light-cured, radiopaque restorative material for restoration of anterior and posterior teeth. Ceram.x one restorative combines Nanotechnology with improved organically modified Ceramic particles, resulting in a Nano-Ceramic restorative with unique features: Natural esthetics by simple procedure, extraordinary low monomer release and superior handling characteristics.

Ceram.x one restorative provides two shading systems in one product to cover different clinical and esthetic demands.

Ceram.x one dentin & enamel dental material – restorative, the Single Translucency System, comprises seven shades of intermediate translucency comparable to conventional composites optimal for fast and easy restorations of anterior and posterior teeth.

Accessories:

Dentin & Enamel Shade Guide - A shade guide for selection ceram.x one shades.

iShade label - A stick scale of colors for the transferring VITA color scale to ceram.x one color scale.

1.3 Information about medical device manufacturer/developer

«DENTSPLY DeTrey GmbH», De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-192; +49-(0)7531-583-660-192; e-mail: webmaster@dentsply.de

1.4 Addresses of medical device manufacturing sites

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1.6 Delivery forms

Ceram.x one dentin & enamel dental material are delivered in 3 g syringes and has different shades: d1, d2, d3, d4, e1, e2, e3.

The material can be delivered separately or as a component of ceram.x one dentin & enamel Starter Kit with the accessories.

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

1.7 Composition and materials

1.7.1 *ceram.x one dentin & enamel dental material*

- Methacrylate modified polysiloxane (organically modified ceramics)
- Ethoxylated bisphenol-A-dimethacrylate
- Dodecandioldi-methacrylate (DDDMA)
- EGAMA-ATMPO adduct
- Camphor quinone
- Ethyldimethylamino-benzoate (DMABE)
- 2,5-Dihydroxybenzene-1,4-dicarboxylic acid diethylester (Lumilux)
- 2-hydroxy-4-methoxybenzophenone
- Highly dispersed silicon dioxide двуокись кремния
- Barium-alumo-boro-silicate glass
- Titanium dioxide
- Black iron oxide
- Yellow iron oxide
- Red iron oxide.

1.7.2 *Syringe materials:*

Component	Material
Barrel, piston seal, plunger, cap	Polypropelene
Screw	Polybutylene terephthalate

1.7.3 *Dentin & Enamel Shade Guide*

Component	Material
Shade Guide Holder	Polybutylenterephthalat PBT Pocan 1505, BASF AG
Shade (fingers)	Polycoarbonat Trirex 3022 IR

1.8 Technical specifications of ceram.x one dentin & enamel

1.8.1 *Appearance:*

Homogeneous paste without impurities.

1.8.2 *Consistency:*

Material	Shades	Consistency
ceram.x one dentin	d1, d2, d3, d4	13 - 19 mm
ceram.x one enamel	e1, e2, e3	16 – 19 mm

1.8.3 *Curing time and Depth of cure (according ISO 4049:2009):*

Material	Shades	Curing time		Depth of cure
		Power ≥ 500 mWatt/cm ²	Power ≥ 800 mWatt/cm ²	
ceram.x one dentin	d1, d2, d3, d4	40 s	30 s	Not less than 2 mm
ceram.x one enamel	e1, e2, e3	10 s	10 s	Not less than 2 mm

1.8.4 *Sensitivity to Ambient Light (according to ISO 4049:2009):*

Withstand irradiation with ambient light, showing no pre- polymerization for at least 100 seconds (for all shades).

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

1.8.5 Opacity:

Material	Shades	Opacity, %
ceram.x one dentin	d1	62 ± 2
	d2	55 ± 2
	d3	50 ± 2
	d4	45 ± 2
ceram.x one enamel	e1	24 ± 1.5
	e2	22 ± 1.5
	e3	20 ± 1.5

1.8.6 Main technical properties of the polymerized material:

Parameter	Units	Value
Compressive strength	MPa	320
Yield strength	MPa	130
Flexural strength	MPa	110
Flexural modulus	GPa	8,5
Shrinkage (Archimedes; v)	%	2,3
Radiopacity	mm Al	2

2 Images

2.1 Ceram.x one dentin & enamel dental material in 3 g syringes of different shades: e1, e2, e3, d1, d2, d3, d4 (from left to right).

2.2 Dentin & Enamel Shade Guide.

2.3 iShade label.

Handwritten signature

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

3 Indications

Ceram.x one dentin & enamel dental material is designed for restoration of anterior and posterior teeth.

Products should be used only by qualified medical personnel in clinical institutions.

4 Contradictions

Patients who have a history of allergic reaction to acrylate or methacrylate plastics or any of the other material components.

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Warnings

The material ceram.x one contains polymerizable methacrylate monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons. The material is acid agent and can cause thermal injury.

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- Use protective measures for the dental personnel and patients such as glasses and rubber dam in accordance with recommendations of your local dental association.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. It is recommended to use adequate isolation such as rubber dam.
- To prevent a bottle tip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the devices are handled offside the dental unit with clean/disinfected gloves.
- Tightly close the syringe immediately after use.

To prevent a syringe from exposure by splashes and drops of biological fluids or material from untreated hands it is necessary to take syringe only with hands in clean/sterile gloves. In case of exposure dispose a syringe.

Interactions:

- Do not use eugenol- and hydrogen peroxide-containing materials in conjunction with this dental material since they may cause softening of polymer components and/or interfere with hardening of the product.
- If it is refrigerated, allow material to reach room temperature prior to use.

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7 Step-by-step instructions

7.1 Shade selection

ceram.x one Dentin & Enamel restorative offers four dentin shades of low translucency (d1-d4) and three enamel shades of high translucency (e1-e3). With these shades you may copy every VITA shade. ceram.x one Dentin & Enamel restorative is especially suitable for esthetically demanding situations.

Concerning shade selection procedure there are two possibilities. You can use:

- The VITAPAN 2 classical shade guide together with the shade identification label (iShade label). With the enclosed iShade label you may select the suitable ceram.x one Universal or Dentin & Enamel shade(s) corresponding to the VITA shade range.
- The ceram.x one shade tabs which are made from original ceram. x one restorative material.

7.1.1 Shade selection with VITAPAN classical shade guide and iShade label

1. Fix the delivered iShade label at the rear side of the VITAPAN® classical shade guide.
2. Select the VITAPAN classical shade to be matched.
3. Select the corresponding ceram.x one shade according to the recipe on the iShade label.
4. ceram.x one Dentin & Enamel restorative. Each VITA shade is reproduced by a combination of one dentin shade and one enamel shade.

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7.1.2 Shade selection with delivered ceram.x one shade tabs.

To select the tooth color you may use the delivered shade guides for ceram. x one Dentin & Enamel restorative.

Avoid contamination of the shade guide.

To prevent the shade guide from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. For shade selection, remove individual tabs from shade guide holder. Always make sure individual tabs are completely reprocessed before placing back into shade guide.

1. Select the dentin shade most closely matching the color of the moist dentin visible in the cavity or consider the cervical region of the tooth.
2. Select the enamel shade most closely matching the color of the tooth. Consider the incisal, occlusal or proximal region of the tooth.

7.1.3 Shade selection hints

- Clean the tooth surface with prophylaxis paste to remove plaque or surface stain.
- Select the shade while teeth are hydrated.
- When working with ceram.x one Dentin & Enamel restorative you should select a slightly darker rather than a lighter dentin shade. The succeeding enamel layer tends to lighten the optical result.

7.2 Cavity preparation

1. Clean the tooth surface with a fluoride free prophylaxis paste (e.g. Nupro prophylaxis paste (Dentsply Professional, USA)).
2. Prepare cavity (unless not necessary, e.g. cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

7.3 Matrix placement and wedging (Class II restorations)

1. Place a matrix (e.g. AutoMatrix or Palodent Plus matrix system (Dentsply Caulk, USA)) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or BiTine ring (Dentsply Caulk, USA) placement is recommended.

7.4 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application.

Refer to adhesive manufacturer's Instructions for Use for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of the material.

7.5 Application

Apply material in increments immediately after the application of the adhesive. The material is designed to be cured in increments up to a 2 mm depth/thickness.

Avoid contamination of the syringe.

To prevent the syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves.

1. Remove the cap.
2. Turn the handle of the syringe slowly in a clockwise direction and dispense the necessary amount of the material onto a mixing pad.
3. Point the front tip of the syringe upwards and turn the handle anti-clockwise to prevent oozing of the material.
4. Immediately close the syringe with the cap.
5. Protect the restorative material on the mixing pad against light.

7.6 Applying ceram.x one Dentin & Enamel restorative

Rebuild the tooth with respect to its anatomical structure. Consider that depending on the thickness of the enamel and dentin layers, the optical result may vary from the selected VITA shade.

1. Rebuild the dentin with the selected dentin shade (thickness of increment ≤ 2 mm).
2. Cure each increment of dentin according to the table below [Light-curing].
3. Rebuild the enamel layer with the selected enamel shade (thickness of increment ≤ 2 mm).
4. Cure each increment of enamel according to the table below [Light-curing].

7.7 Light-curing

Light-cure each area of the restoration surface with a suitable curing light designed to cure materials containing camphorquinone (CQ) initiator, i.e. spectral output containing 470 nm. Minimum light output must be at least 500 mW/cm², exposure time according to the shade as listed in the table below. Refer to curing light manufacturer's Instructions for Use for compatibility and curing recommendations.

Curing time for 2 mm increments:

Output	≥ 500 mW/cm ²	≥ 800 mW/cm ²
ceram.x one Dentin shades (d1-d4)	40 sec	30 sec
ceram.x one Enamel shades (e1-e3)	10 sec	10 sec

7.8 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.
2. Use Enhance finishing devices (Dentsply Caulk, USA) for additional finishing.
3. Prior to polishing, check that surface is smooth and free of defects.
4. For achieving a very high luster of the restoration, we recommend Enhance and PoGo, finishing and polishing instruments, Prisma Gloss Regular and Extrafine polishing paste (Dentsply Caulk, USA).

For contouring, finishing, and/or polishing, follow the manufacturer's Instructions for Use.

8 Cleaning and disinfection

The products are supplied non-sterile. Un-sealed syringes containing the material can be used within 3 months. Sterilisation of the material and accessories is not acceptable.

8.1 Cleaning and disinfection of Shade Guide

NOTICE

Cross contamination.

Infection.

1. To prevent the shade guide from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. For shade selection, remove individual tabs from shade guide holder.
2. Always make sure individual tabs are completely reprocessed before placing back into shade guide holder.

NOTICE

Wrong cleaning or disinfection method.

Damage to the device.

1. Do not clean or disinfect with aggressive agents (e.g. solutions based on chlorine, glutaraldehyde, phenolic solvents, iodo-phors, alcohol, orange oil, or acetone) as they may affect the color shade over time.
2. Always make sure individual tabs are completely reprocessed before placing back into shade guide holder.
3. Never autoclave the individual tabs.

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

8.1.1 Manual cleaning

1. Discard gloves according to local regulations.
2. Disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations.

Use according to disinfectant solution manufacturer's Instructions for Use.

3. Use a new clean pair of examination gloves.
4. Clean device with a soft brush, warm water, and an appropriate cleaning solution until free of visible contamination. Pay special attention to device seams and insertions.
5. Remove cleaning solution residue with a cloth soaked with water.
6. Dry device with a lint-free single-use cloth.

8.1.2 Manual disinfection

1. After cleaning [see Manual cleaning] thoroughly wipe all device surfaces with a single-use cloth in combination with a water-based, bactericidal, virucidal and fungicidal disinfection solution approved according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions.
2. Remove disinfectant solution residue with a cloth soaked with water.
3. Dry device with a lint-free single-use cloth.

9 Storage and shelf life

- Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.
- Store in sealed packs at temperatures between 6 °C and 28 °C (43 °F and 82 °F).
- Keep out of direct sunlight and protect from moisture.
- Do not freeze.
- Do not use after expiration date:

ceram.x one – 2 years

Humidity can adversely affect the properties of the material. For unsealed syringes, the following is recommended:

- Store in a dry environment (< 80% relative humidity).
- Use within 3 months.

10 Transportation.

The devices should be transported according to the rules applicable to each transport mode at temperatures between 6 °C and 28 °C and relative humidity <80%.

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12 Condition of use

Use at: +10 - +45 and relative humidity <80%. The devices are not subject to repair. No special maintenance is required.

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Directions for use

For dental use only

Prime&Bond One Etch&Rinse Dental Adhesive Material with Accessories

Manufactured by:

DENTSPLY DeTrey GmbH, De-Trey-Strasse 1, 78467 Konstanz, Germany
Tel: +49-(0)7531-583-192; +49-(0)7531-583-660-192

- 1 Description
- 2 Images
- 3 Indication for use
- 4 Contradictions
- 5 Warnings
- 6 Adverse events
- 7 Step-by-step instructions
- 8 Cleaning and Sterilization
- 9 Storage and shelf life
- 10 Transportation
- 11 Environmental requirements
- 12 Condition of use
- 13 Disposal
- 14 Manufacturer's warranties
- 15 Authorized Representative

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

2015

1 Description

1.1 Name:

Prime&Bond One Etch&Rinse Dental Adhesive Material with Accessories.

1.2 Information about medical device manufacturer/developer:

«DENTSPLY DeTrey GmbH»,

De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-192; +49-(0)7531-583-660-192; e-mail: webmaster@dentsply.de

1.3 Addresses of medical device manufacturing sites:

«DENTSPLY DeTrey GmbH», Германия

De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-192; +49-(0)7531-583-660-192; e-mail: webmaster@dentsply.de

1.4 Indication for use:

Dental self-etching adhesive material for direct and indirect restorations.

1.5 Description:

Universal self-etching adhesive in a bottle is specially designed to bond composite materials to enamel and dentin as well as to metals and ceramic.

Prime&Bond One Etch&Rinse universal self-etching adhesive offers excellent performance due to high bond strength to enamel and dentin, easy and comfortable application and a high degree of application robustness.

Prime&Bond One Etch&Rinse is singly used with light-cured restorative materials and cements. When it is mixed with “Self Cure Activator” (DeTrey, Germany, available separately), it is used with self-cure and dual-cure materials. In this case refer to instruction for using “Self Cure Activator”.

1.6 Delivery forms:

1. Adhesive Prime&Bond One Etch&Rinse in a bottle 3,5 ml with applicator tips – 50 pcs., CliXdish mixing well – 1 pcs., Instructions for Use – 1 pcs.

2. Adhesive Prime&Bond One Etch&Rinse in a bottle 2,5 ml with the Instructions for Use – 1 pcs.

Accessories:

1. Applicator tips – no more than 100 pcs.;

ADDITIONALLY, PRIME & BOND ONE ETCH & RINSE IN BOTTLE 3,5 ML WITH APPLICATOR TIPS AND CLIXDISH MIXING WELL COULD BE INCLUDED INTO CERAM.X ONE UNIVERSAL STARTER KIT AND CERAM.X ONE DENTIN & ENAMEL STARTER KIT.

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

1.7 Composition and materials

1.7.1 Prime&Bond One Etch & Rinse Adhesive

Component	Chemical name
Phosphoric acid modified acrylate resin (PENTA)	2,2,4,4,4-penta-oxymethylenacrycol-3-oxo-1-phosporic acid
Urethane dimethacrylate resin (UDMA)	2,7,7,9,15-Pentamethyl-4,13-dioxo-3,14-dioxa-5,12-diaza-hexadecan-1,16-diyl-dimethacrylat
Carboxylic acid modified dimethacrylate (TCB resin)	Bytane -1,2,3,4-tetracarboxylic acid, bis[2-hydroethyl-methacrylate] ester
Triethylenglycol-dimethacrylate (TEGDMA)	Triethylenglycoldimethylacrylate
2-hydroxyethylmeth-acrylat (HEMA)	2-propenoic acid, 2-methyl-, 2-hydroxyethyl ester
Camphor quinone (CQ)	(±)-2,3-dioxo-1,7,7-trimethyl-bicyclo-(2,2,1)-heptan
Ethyl-4-dimethylamonibenzoate (DMABE)	Ethyl-4-dimethylaminobenzoate
Tert.-butyl-hydroquinone (TBHQ)	2-tert.-butyl-1,4-benzene diol
Nanofiller	Functionalized amorphous silica
Tert.-butanol	2-methyl-propanol-2

1.7.2 Bottle for adhesive

КОМПОНЕНТ	Материал
Bottle	Low density polyethylene, high density polyethylene, EVOH, ADMER.
Cap	Polypropylene
Dropper	Polyethylene

1.7.3 Applicator tips

Applicator tips part	Composition (material)
Handle	Polypropylene homopolymer
	Stabilizers
Nylon brush	Nylon 6,6 Polyamide
	Titanium dioxide
	Absorb water
Glue	Toluene (DGEBA) Part A
	Xylain (Amidopolyamines) Part B

1.7.4 CliXdish mixing well CliXdish

Polyamide.

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

1.8 Technical specifications of prime&bond one etch & rinse adhesive

1.8.1 Shear bond strength:

Dentin moisture	Bond strength, MPa
Optimum	39
Over-dried	31
Over-wet	32

*Test method: after 1800 thermal cycles (5/55 °C), n=10; 5 mm/min (Latta MA, 2009).

1.8.2 Micro-shear fatigue resistance:

Parameter	Units	Value
Fatigue resistance*	MPa	30

*Test method: median micro-shear fatigue resistance (μ SFR) method, 10 000 cyclic actions 2Hz (35°C).

1.8.3 Cure time:


Material	Cure time	
	Output ≥ 500 mWatt/cm ²	Output ≥ 800 mWatt/cm ²
prime & bond one etch & rinse adhesive	20 s	10 s

2 Images

2.1 Prime&Bond One Etch&Rinse adhesive in bottle x 3.5 ml – 1 piece; applicator tips – 50 pieces; CliXdish mixing well – 1 piece

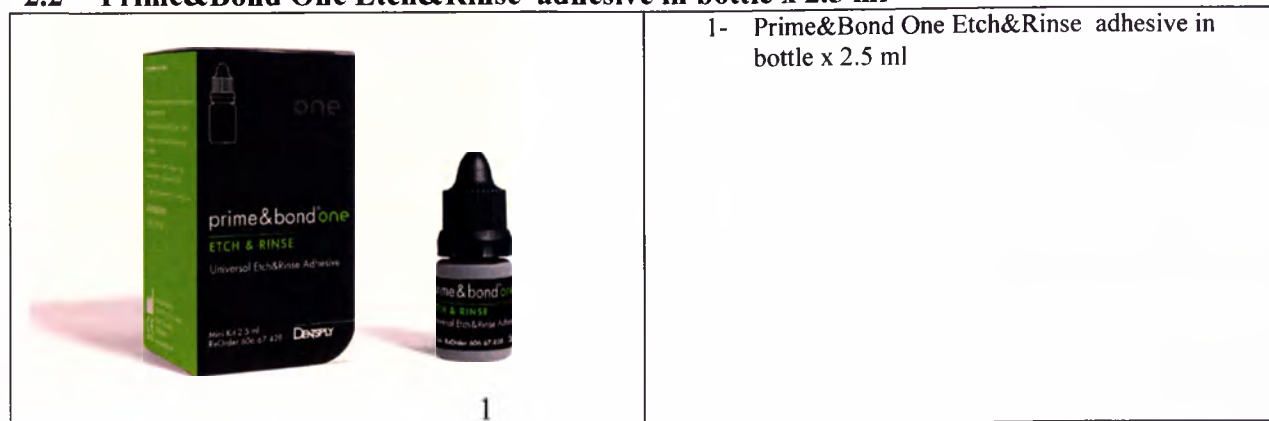2.2 Prime&Bond One Etch&Rinse adhesive in bottle x 2.5 ml

Handwritten signature in blue ink.

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

3 Indication for use

Dental self-etching adhesive material for direct and indirect restorations.

Prime&Bond One Etch&Rinse adhesive is designed for direct restorations of all cavity classes in combination with light-cured composite restorative materials.

Prime&Bond One Etch&Rinse adhesive is 5th generation adhesive for:

- Direct, light-cured composite and compomer restoratives.
- Indirect restorations; light-cured, composite cemented inlays, onlays, crowns, bridges and retainers.
- Composite/compomer, ceramic and amalgam repairs.
- Cavity varnish for use with fresh amalgam.

It is necessary to refer to instruction for using “Self Cure Activator” when using Prime&Bond One Etch&Rinse for the following indications:

- Direct, chemical and dual-cure composite restoration and restoration of residual limb.
- Indirect restorations; fixation on cement of chemical and dual-cure inlays, onlays, crowns, bridges and retainers.
- Fixation on cement of chemical or dual-cure endodontic post cementation.

When using Prime&Bond One Etch&Rinse for adhesive bonding of direct amalgam restorations please refer to the Instructions for use of “Amalgam Bonding Accessory Kit” (available not in all the countries).

4 Contradictions

- Patients who have a history of allergic reaction to acrylate or methacrylate plastics or any of the other adhesive components.
- Direct application to dental pulp (direct pulp capping).

5 Warnings

Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

Warnings

The material contains polymerizable acrylate and methacrylate monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental personnel and patients such as glasses and rubber dam in

accordance with recommendations of your local dental association.

- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. It is recommended to use adequate isolation such as rubber dam.
- To prevent a bottle tip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the devices are handled offside the dental unit with clean/disinfected gloves.
- Tightly close the bottle immediately after use.
- Use in a well-ventilated area.

Avoid inhaling vapor.

- Flammable: Keep away from sources of ignition. Take precautionary measures against static discharges.
- Avoid Prime&Bond One Etch&Rinse adhesive saturating the gingival retraction cord. If Prime&Bond One Etch&Rinse soaks into the cord, it may set hard and bond the cord to the underlying tooth surface making removal difficult.
- Applicator tips are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.

Interactions:

- Do not use eugenol- and hydrogen peroxide-containing materials in conjunction with Prime&Bond One Etch&Rinse adhesive since they may cause softening of polymer components and/or interfere with hardening of the product.
- If mineral-impregnated (e.g., ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.
- Prime&Bond One Etch&Rinse is a light-cured material. Protect from ambient light.
- If it is refrigerated, allow material to reach room temperature prior to use.

6 Adverse events

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

7 Step-by-step instructions

7.1 Direct restoration (light-cured composite resin and compomers)

7.1.1 Cleaning

1. Clean uninstrumented enamel and dentin with a rubber cup and pumice or a non-fluoride cleaning paste such as nupro prophylaxis paste (Dentsply Professional, USA).
2. Wash thoroughly with water spray and air dry.
3. Clean freshly instrumented enamel and dentin with water spray, then air dry.

7.1.2 Pulp protection

1. In deep cavities cover the dentin close to the pulp (less than 1 mm) with a hard-setting calcium hydroxide liner (Dycal calcium hydroxide liner (Dentsply Caulk, USA), see instructions for use) leaving the rest of the cavity surface free for bonding.

7.1.3 Tooth conditioning/dentin pre-treatment

When used as a bonding agent for direct restorative materials, it is recommended to follow the total etch technique described in this chapter. Prior to use, please see the instructions for use of the respective conditioner.

Application of 34%-36% phosphoric acid conditioner

1. After application of rubber dam or other suitable isolation technique, apply the respective conditioner.
2. Apply the respective conditioner to the cavity surfaces starting at the enamel margins.
3. For best results, condition enamel for at least 15 seconds and dentin for 15 seconds.

Rinsing and blot drying

1. Remove conditioner with aspirator tube and/or vigorous water spray and rinse conditioned areas thoroughly for at least 15 seconds.
2. Remove rinsing water completely by blowing gently with an air syringe or by blot drying with a cotton pellet. Do not desiccate dentin.
3. Proceed immediately to application of prime&bond one etch&rins adhesive.

Contamination.

Restoration failure.

1. Once the surfaces have been properly treated, they must be kept uncontaminated.
2. If salivary contamination occurs, thoroughly clean with vigorous water spray, dry, and repeat conditioning procedure for 5 seconds only. Rinse and dry as described above.

7.1.4 Dosage

1. To prevent the bottle from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the bottle is handled offside the dental unit with clean/disinfected gloves.
2. Dispense prime&bond one etch&rins adhesive directly onto a clean applicator tip, making sure that the bottle does not come in direct contact with the applicator tip, or place one to two drops of prime&bond one etch&rins adhesive into a clean CliXdish mixing well or standard dappen dish.
3. Replace cap promptly.

Technique Tip: Material in a closed CliXdish mixing well will remain usable for up to 15 minutes. Material dispensed to a standard mixing well must be used immediately.

7.1.5 Application and curing

1. Using the applicator tip, apply the product wetting all cavity surfaces uniformly. Avoid pooling. This surface should remain undisturbed for 20 seconds.
2. Evaporate solvent by thoroughly drying with clean, dry air from a dental syringe for at least 5 seconds. Use a vacuum aspirator to prevent spatter of the adhesive onto the mucosa. Surface should have a uniform glossy appearance. If not, repeat application and air dry.
3. Cure prime&bond one etch&rins adhesive for 10 seconds 2 using a curing light.

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check curing output.

Check curing light for minimum light output of at least 800 mW/cm². Cure for at least 20 seconds if light output is between 500 and 800 mW/cm².

4. Place restorative material over the cured prime&bond one etch&rins adhesive as per restorative material manufacturer's instructions for use.

Detrey

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

7.2 Light-cured composite, ceramic and amalgam repairs

7.2.1 Preparation

1. Roughen and clean fractured restoration surface with a fi diamond bur and create mechanical retention, if possible.

7.2.2 Tooth conditioning/dentin pre-treatment

1. Etch fractured tooth surface with the respective conditioner as outlined in 7.1.3 tooth conditioning/dentin pre- treatment.

7.2.3 Treatment of restoration

1. Etch fractured restoration surface with the respective conditioner for at least 15 seconds as outlined in 7.1.3 tooth conditioning/dentin pre-treatment.

7.2.4 Application and curing

1. Apply adhesive to the fractured restoration surfaces and light-cure 2 the adhesive [7.1.5 application and curing].

7.2.5 Completion

1. Complete repair with placement and curing of desired shade(s) of light-cured composite restorative following manufacturer's instructions for use.

7.3 Cementation of light transmitting indirect restoration with a light-cured resin cement

7.3.1 Treatment of restoration

1. Treat internal bonding surface of restoration according to manufacturer's or dental laboratory's instructions for use, i.e., etching, mechanical roughening and/or Silane Coupling agent application.
2. Apply a single coat of prime&bond one etch&rinse adhesive to the internal bonding surface of the restoration. immediately air dry for 5 seconds.
Light-curing of applied prime&bond one etch&rinse adhesive is not necessary.

7.3.2 Treatment of tooth preparation

1. Clean and condition tooth preparation, apply and light- cure the material as described in 7.1 Direct restoration (light-cured composite resin and compomers).

7.3.3 Cementation

1. Prepare and apply light-cured resin cement according to manufacturer's instructions for use.

7.4 Cavity varnish for use with fresh amalgam

When used as a cavity varnish, prime&bond® one etch&rinse adhesive is not an amalgam adhesive.

7.4.1 Preparation

1. Finish preparation by removing existing restorations and/ or caries.

7.4.2 Cleaning

Clean preparation and place a hard-setting calcium hydroxide liner (Dycal calcium hydroxide liner (Dentsply Caulk, USA), see instructions for use) if needed, according to:

1. Cleaning [7.1.1 Cleaning].
2. Pulp protection [7.1.2 Pulp protection].

7.4.3 3.4.3 Tooth conditioning/dentin pre-treatment

1. Rinse and carefully air dry cavity preparation, but do not desiccate exposed dentin.
2. Condition enamel and dentin following section 7.1.3 tooth conditioning/dentin pre-treatment.

7.4.4 Application

1. Apply and light cure prime&bond one etch&rinse adhesive as described for direct restorations, see section 7.1.3 tooth conditioning/dentin pretreatment.

7.4.5 Completion

1. Place and condense amalgam as per manufacturer's directions.

8 Cleaning and Sterilization

The products are supplied non-sterile. Applicators are for single use. Sterilization of dental material and accessories is not required. Except for Clixdish mixing well that is reusable. It can be cleaned, disinfected and sterilized.

Cross-contamination.

Infection.

1. Do not reuse single use products. Dispose in accordance with local regulations.
2. Reprocess reusable products as described below.

8.1 CliXdish mixing well is reusable. Cleaning, disinfection and sterilization procedure (if necessary):

8.1.1 Manual cleaning:

1. Discard gloves according to local regulations. Disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's Instructions for Use.
2. Use a new clean pair of examination gloves.
3. Disassemble the device.
4. Remove adhesive residue with a cloth.
5. Clean the device with a soft brush, warm water, and an appropriate cleaning solution until free of visible contamination. Pay special attention to device seams and insertions.
6. Remove cleaning solution residue with a cloth soaked with water.
7. Dry the device with a lint-free single-use cloth.

8.1.2 Manual disinfection:

1. After cleaning (8.1.1 Manual cleaning) thoroughly wipe all device surfaces with a single-use cloth in combination with an alcohol-based, bactericidal, virucidal and fungicidal disinfection solution approved according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions.
2. Remove disinfectant solution residue with a cloth soaked with water.
3. Dry the device with a lint-free single-use cloth.

The device may be manually disinfected for up to 200 times.

8.1.3 Automatic Washer-Disinfector:

As an alternative to manual cleaning and disinfection, the device can also be processed in a washer-disinfector (for example, Miele Washer-Disinfector G7835 CD using program Vario TD with neodisher MediClean (alkaline detergent) and neodisher Z (acid neutralization and cleaning detergent); both detergents from Dr. Weigert, Hamburg, Germany (not products or registered trademarks of DENTSPLY International, Inc.).

Use only properly maintained, calibrated, and approved washer-disinfector according to ISO 15883-1.

1. Disassemble the device.

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

2. Remove adhesive residue with a cloth.
3. Place the device in the washer-disinfector allowing water and detergent to enter into and drain off from device openings during washer-disinfection program and follow washer-disinfector's Instructions for Use.
4. Run washer-disinfection program with A0 value ≥ 3000 (e.g. 5 min at ≥ 90 °C) using appropriate detergents.

The device may be cleaned in the washer-disinfector for up to 500 times.

8.1.4 Sterilization (if required)

The device can also be autoclaved.

Steam autoclave the device unwrapped at 134 °C/ 2 bar for 3 minutes 30 seconds.

The device may be disinfected for up to 200 times and sterilized for up to 20 times.

9 Storage and shelf life

Expiration date – 2 years. Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store the product in a well-ventilated area and protect from sunlight.
- Store the product at temperatures between 2°C and 24 °C. Use the product at room temperature.
- Protect from moisture.
- Do not freeze.
- Do not use after expiration date.

10 Transportation

The devices should be transported according to the rules applicable to each transport mode at temperatures between 2 °C and 24 °C.

11 Environmental requirements

The product does not have an adverse impact on the environment during its life cycle.

12 Condition of use

Use at: +10 - +45. The devices are not subject to repair. No special maintenance is required.

13 Disposal

Disposal in accordance with the local regulations and the policies and procedures of your medical facility.

14 Manufacturer's warranties

The manufacturer is not responsible and shall not indemnify for any possible damage or accidents caused by:

- Use of components unrelated to the product which can affect normal operation.
- Noncompliance with following the instructions for use.

The user is responsible for testing of materials for fitness and use for any purpose not expressly specified in the instructions!

For product quality issues, please contact the Authorized Manufacturer's Representative.

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Directions for use

For dental use only

DeTrey Conditioner 36 etching gel in syringe with accessories

manufactured by

DENTSPLY DeTrey GmbH, De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-192; +49-(0)7531-583-660-192

- 1 Description
- 2 Photo
- 3 Indications for use
- 4 Contradictions
- 5 Warnings
- 6 Averse events
- 7 Step-by-step instructions
- 8 Cleaning and sterilization
- 9 Storage and shelf life
- 10 Transportation
- 11 Environmental requirements
- 12 Condition of use
- 13 Disposal
- 14 Manufacturer's warranties
- 15 Authorized Representative

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1.4 Indication for use

A conditioner gel in syringe for etching dentin/enamel.

1.5 Description

DeTrey Conditioner 36 etching gel – is a blue-tinted gel containing 36% phosphoric acid for the etching of tooth enamel and conditioning of dentine prior to bonding with resin-based materials.

Accessories:

Application needles

Application needle for applying DeTrey Conditioner 36 gel. Put on the syringe.

1.6 Delivery forms

DeTrey Conditioner 36 etching gel is available in plastic syringe 3 ml with application needles (25 pc.) as a part of sets *ceram.x* one universal Starter Kit and *ceram.x* one dentin & enamel Starter Kit.

1.7 Materials

1.7.1 DeTrey Conditioner 36 etching gel in syringe 3 ml

Composition:

- Phosphoric acid
- Highly dispersed silicon dioxide
- Detergent
- Pigment
- Water

1.7.2 Syringe materials:

Component	Material
Cylinder, Plunger	Polypropylene (Exxon Mobil PP 1013 H1)

1.7.3 Application needle

Component	Material
Tube	Stainless Steel SUS304
Hub	Polypropylene (Exxon Mobil PP 1013 H1)

1.8 Technical specifications

Not acceptable because of accessory purpose of the product.

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

2 Photo

2.1 DeTrey Conditioner 36 etching gel in syringe 3 ml

2.2 Application needles

3 Indication for use

- Enamel etch technique in connection with composite and compomer restorations
- Total etch technique in connection with composite and compomer restorations.

4 Contradictions

Application to dentine close to the pulp (less than 1 mm).

5 Warnings

1. DeTrey Conditioner 36 contains 36% phosphoric acid. Causes burns. Avoid contact with oral tissues, eyes and skin. If accidental contact occurs, flush affected area with generous amounts of water. In case of contact with the eyes, immediately rinse with plenty of water and seek medical attention.
2. DeTrey Conditioner 36 gel should extrude easily: DO NOT USE EXCESSIVE FORCE. Replace original cap of DeTrey Conditioner 36 tightly after each use to avoid evaporation. Discard needle after use, as needles may clog if gel is allowed to dry inside.
3. Make sure the needle tip is securely fastened to the syringe prior to application of DeTrey Conditioner 36.

Interactions with dental materials

Some liners and bases may be etched by phosphoric acid. In general, this does not impair their barrier function.

A handwritten signature in blue ink, possibly reading "DeTrey".

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7 Step-by-step instructions

7.1 1. Cleaning

Clean unprepared enamel with a rubber cup and pumice or a cleaning paste such as Nupro Prophylaxis Pastes. Wash thoroughly with water spray and air-dry.

Clean freshly cut enamel and dentine with water spray and then air-dry.

7.2 Acid Conditioning of Enamel and Dentine (Total Etch Technique)¹

It is recommended to follow the Total Etch Technique described below:

Remove cap from the end of the syringe. Attach disposable needle tip to end of the syringe. Turn tip clockwise to assure that it is fully seated. Gently extrude gel directly to surface/s to be treated. Needle tip may be bent for easy access. Start to apply DeTrey Conditioner 36 gel (36% phosphoric acid) to the cavity surfaces at the enamel margins. For best results, condition enamel for at least 15 seconds and dentine for 15 seconds or less.

Discard needle after use as needle may clog if gel is allowed to dry inside. Replace original cap.

CAUTION: Replace cap tightly after each use to avoid evaporation. Gel should extrude easily: **DO NOT USE EXCESSIVE FORCE.**

¹ Alternatively to the Total Etch Technique, the conventional Enamel Etch Technique can be followed. In this case, the enamel margins only are treated with DeTrey Conditioner 36 for at least 15 seconds. Then rinse and dry as described below.

7.3 Rinsing and Drying

Remove gel with aspirator tube and /or vigorous water spray and rinse conditioned areas thoroughly for at least 15 seconds.

Further treatment of the conditioned area depends on the adhesive material which is applied following the conditioning procedure. Consult respective directions for use. Once the surfaces have been properly treated, they must be kept uncontaminated. If salivary contamination occurs, thoroughly clean with vigorous water spray, dry, and repeat conditioning procedure of enamel for 5 seconds only. Rinse as described above. Proceed with adhesive procedure.

8 Cleaning and Sterilization

The products are supplied non-sterile. Sterilization of materials and accessories is not accommodate. Application needles are for single use.

9 Storage and shelf life

- Close the cap immediately after use.
- Keep out of direct sunlight and protect from moisture.
- Store in at temperature to 24 °C.

Expiration date:

- DeTrey Conditioner 36 etching gel – 5 years
- Application needles – 5 years

10 Transportation

The devices should be transported according to the rules applicable to each transport mode at a temperature 6-28 °C and relative humidity <80%.

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