Directions for use

For dental use only

Ceram.x one universal dental material

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1.1 Name

Ceram.x one universal dental material.

1.2 Information about medical device manufacturer/developer

«DENTSPLY DeTrey GmbH», De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-192; +49-(0)7531-583-660-192; e-mail: webmaster@dentsply.de

1.3 Addresses of medical device manufacturing sites

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Ceram.x one universal dental material – the Single Translucency System, comprises seven shades of intermediate translucency comparable to conventional composites optimal for fast and easy restorations of anterior and posterior teeth.

1.6 Delivery forms

Ceram.x one universal dental material is delivered in 3 g syringes and has different shades: a1, a2, a3, a3.5, a4, c1, c2.

1.7 Composition and materials

1.7.1 *ceram.x one universal dental material in 3 g syringes of different shades: a1, a2, a3, a3.5, a4, c1, c2.*

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1.8 Technical specifications of ceram.x one universal

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Withstand irradiation with ambient light, showing no pre-polymerization for at least 100 seconds (for all shades).

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2 Images

Ceram.x one universal dental material in 3 g syringes of different shades: a1, a2, a3, a3.5, a4, c1, c2 (from left to right).

3 Indications

Ceram.x one universal dental material is designed for restoration of anterior and posterior teeth. Products should be used only by qualified medical personnel in clinical institutions.

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To prevent a syringe from exposure by splashes and drops of biological fluids or material from untreated hands it is necessary to take syringe only with hands in clean/sterile gloves. In case of exposure dispose a syringe.

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7 Step-by-step instructions

7.1 Shade selection

Ceram.x one Universal restorative offers seven shades of intermediate translucency (a1-a4, c1, c2) and is especially suitable for fast, single-shade treatment procedures.

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- The VITAPAN 2 classical shade guide together with the shade identification label (iShade label). With the enclosed iShade label you may select the suitable ceram.x one Universal shade(s) corresponding to the VITA shade range.
- The ceram.x one shade tabs which are made from original ceram. x one restorative material.

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1. Fix the delivered iShade label at the rear side of the VITAPAN classical shade guide.
2. Select the VITAPAN classical shade to be matched.
3. Select the corresponding ceram.x one shade according to the recipe on the iShade label.
4. ceram.x one universal restorative. Each VITA shade is reproduced by one of the seven ceram.x one universal shades.

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To select the tooth color you may use the delivered shade guides for ceram.x one Universal restorative.

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1. Select the ceram.x one shade most closely matching the color of the tooth. Consider the central region of the tooth.

7.1.3 Shade selection hints

- Clean the tooth surface with prophylaxis paste to remove plaque or surface stain.
- Select the shade while teeth are hydrated.

7.2 Cavity preparation

1. Clean the tooth surface with a fluoride free prophylaxis paste (e.g. Nupro prophylaxis paste (Dentsply Professional, USA)).
2. Prepare cavity (unless not necessary, e.g. cervical lesion).
3. Use adequate isolation such as rubber dam.

DENTSPLY DETREY GmbH

De-Trey-Straße 1

D-78467 Konstanz

4. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure

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1. Remove the cap.
2. Turn the handle of the syringe slowly in a clockwise direction and dispense the necessary amount of the material onto a mixing pad.
3. Point the front tip of the syringe upwards and turn the handle anti-clockwise to prevent oozing of the material.
4. Immediately close the syringe with the cap.
5. Protect the restorative material on the mixing pad against light.

7.6 Applying ceram.x one universal restorative

1. Fill the whole cavity incrementally. Each layer ≤ 2 mm.
2. Cure each layer according to the table below [Light-curing].

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«DENTSPLY DeTrey GmbH», De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-192; +49-(0)7531-583-660-192

- 1 Description
- 2 Images
- 3 Indications
- 4 Contradictions
- 5 Warnings
- 6 Adverse events
- 7 Step-by-step instructions
- 8 Cleaning and disinfection
- 9 Storage and shelf life
- 10 Transportation.
- 11 Environmental requirements
- 12 Condition of use
- 13 Disposal
- 14 Manufacturer's warranties
- 15 Authorized Representative

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

1 Description

1.1 Name

Ceram.x one dentin & enamel dental material with accessories.

1.2 Description

Ceram.x one dental material – is a light-cured, radiopaque restorative material for restoration of anterior and posterior teeth. Ceram.x one restorative combines Nanotechnology with improved organically modified Ceramic particles, resulting in a Nano-Ceramic restorative with unique features: Natural esthetics by simple procedure, extraordinary low monomer release and superior handling characteristics.

Ceram.x one restorative provides two shading systems in one product to cover different clinical and esthetic demands.

Ceram.x one dentin & enamel dental material – restorative, the Single Translucency System, comprises seven shades of intermediate translucency comparable to conventional composites optimal for fast and easy restorations of anterior and posterior teeth.

1.3 Information about medical device manufacturer/developer

«DENTSPLY DeTrey GmbH», De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-192; +49-(0)7531-583-660-192; e-mail: webmaster@dentsply.de

1.4 Addresses of medical device manufacturing sites

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1.6 Delivery forms

Ceram.x one dentin & enamel dental material are delivered in 3 g syringes and has different shades: d1, d2, d3, d4, e1, e2, e3.

The material can be delivered separately or as a component of ceram.x one dentin & enamel Starter Kit with the accessories.

1.7 Composition and materials

1.7.1 *ceram.x one dentin & enamel dental material*

- Methacrylate modified polysiloxane (organically modified ceramics)
- Ethoxylated bisphenol-A-dimethacrylate
- Dodecandioldi-methacrylate (DDDMA)
- EGAMA-ATMPO adduct
- Camphor quinone
- Ethyldimethylamino-benzoate (DMABE)
- 2,5-Dihydroxybenzene-1,4-dicarboxylic acid diethylester (Lumilux)
- 2-hydroxy-4-methoxybenzophenone
- Highly dispersed silicon dioxide двуокись кремния
- Barium-alumo-boro-silicate glass
- Titanium dioxide
- Black iron oxide
- Yellow iron oxide
- Red iron oxide.

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

1.7.2 Syringe materials:

Component	Material
Barrel, piston seal, plunger, cap	Polypropylene
Screw	Polybutylene terephthalate

1.7.3 Dentin & Enamel Shade Guide

Component	Material
Shade Guide Holder	Polybutylenterephthalat PBT Pocan 1505, BASF AG
Shade (fingers)	Polycoarbonat Trirex 3022 IR

1.8 Technical specifications of ceram.x one dentin & enamel

1.8.1 Appearance:

Homogeneous paste without impurities.

1.8.2 Consistency:

Material	Shades	Consistency
ceram.x one dentin	d1, d2, d3, d4	13 - 19 mm
ceram.x one enamel	e1, e2, e3	16 - 19 mm

1.8.3 Curing time and Depth of cure (according ISO 4049:2009):

Material	Shades	Curing time		Depth of cure
		Power ≥ 500 mWatt/cm ²	Power ≥ 800 mWatt/cm ²	
ceram.x one dentin	d1, d2, d3, d4	40 s	30 s	Not less than 2 mm
ceram.x one enamel	e1, e2, e3	10 s	10 s	Not less than 2 mm

1.8.4 Sensitivity to Ambient Light (according to ISO 4049:2009):

Withstand irradiation with ambient light, showing no pre- polymerization for at least 100 seconds (for all shades).

1.8.5 Opacity:

Material	Shades	Opacity, %
ceram.x one dentin	d1	62 ± 2
	d2	55 ± 2
	d3	50 ± 2
	d4	45 ± 2
ceram.x one enamel	e1	24 ± 1.5
	e2	22 ± 1.5
	e3	20 ± 1.5

1.8.6 Main technical properties of the polymerized material:

Parameter	Units	Value
Compressive strength	MPa	320
Yield strength	MPa	130
Flexural strength	MPa	110
Flexural modulus	GPa	8,5
Shrinkage (Archimedes; v)	%	2,3
Radiopacity	mm Al	2

2 Images

Ceram.x one dentin & enamel dental material in 3 g syringes of different shades: e1, e2, e3, d1, d2, d3, d4 (from left to right).

3 Indications

Ceram.x one dentin & enamel dental material is designed for restoration of anterior and posterior teeth.

Products should be used only by qualified medical personnel in clinical institutions.

4 Contradictions

Patients who have a history of allergic reaction to acrylate or methacrylate plastics or any of the other material components.

5 Warnings

Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

Warnings

The material ceram.x one contains polymerizable methacrylate monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons. The material is acid agent and can cause thermal injury.

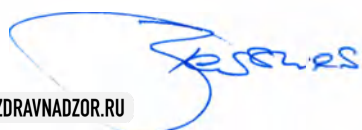
- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

Precautions:

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental personnel and patients such as glasses and rubber dam in accordance with recommendations of your local dental association.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. It is recommended to use adequate isolation such as rubber dam.
- To prevent a bottle tip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the devices are handled offside the dental unit with clean/disinfected gloves.
- Tightly close the syringe immediately after use.

To prevent a syringe from exposure by splashes and drops of biological fluids or material from

untreated hands it is necessary to take syringe only with hands in clean/sterile gloves. In case of exposure dispose a syringe.

Interactions:

- Do not use eugenol- and hydrogen peroxide-containing materials in conjunction with this dental material since they may cause softening of polymer components and/or interfere with hardening of the product.
- If it is refrigerated, allow material to reach room temperature prior to use.

6 Adverse events

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

7 Step-by-step instructions

7.1 Shade selection

ceram.x one Dentin & Enamel restorative offers four dentin shades of low translucency (d1-d4) and three enamel shades of high translucency (e1-e3). With these shades you may copy every VITA shade. ceram.x one Dentin & Enamel restorative is especially suitable for esthetically demanding situations.

Concerning shade selection procedure there are two possibilities. You can use:

- The VITAPAN 2 classical shade guide together with the shade identification label (iShade label). With the enclosed iShade label you may select the suitable ceram.x one Dentin & Enamel shade(s) corresponding to the VITA shade range.
- The ceram.x one shade tabs which are made from original ceram. x one restorative material.

7.1.1 Shade selection with VITAPAN classical shade guide and iShade label

1. Fix the delivered iShade label at the rear side of the VITAPAN classical shade guide.
2. Select the VITAPAN classical shade to be matched.
3. Select the corresponding ceram.x one shade according to the recipe on the iShade label.
4. ceram.x one Dentin & Enamel restorative. Each VITA shade is reproduced by a combination of one dentin shade and one enamel shade.

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DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

7.1.2 Shade selection with delivered ceram.x one shade tabs.

To select the tooth color you may use the delivered shade guides for ceram. x one Dentin & Enamel restorative.

Avoid contamination of the shade guide.

To prevent the shade guide from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. For shade selection, remove individual tabs from shade guide holder. Always make sure individual tabs are completely reprocessed before placing back into shade guide.

1. Select the dentin shade most closely matching the color of the moist dentin visible in the cavity or consider the cervical region of the tooth.
2. Select the enamel shade most closely matching the color of the tooth. Consider the incisal, occlusal or proximal region of the tooth.

7.1.3 Shade selection hints

- Clean the tooth surface with prophylaxis paste to remove plaque or surface stain.
- Select the shade while teeth are hydrated.
- When working with ceram.x one Dentin & Enamel restorative you should select a slightly darker rather than a lighter dentin shade. The succeeding enamel layer tends to lighten the optical result.

7.2 Cavity preparation

1. Clean the tooth surface with a fluoride free prophylaxis paste (e.g. Nupro prophylaxis paste (Dentsply Professional, USA)).
2. Prepare cavity (unless not necessary, e.g. cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

7.3 Matrix placement and wedging (Class II restorations)

1. Place a matrix (e.g. AutoMatrix or Palodent Plus matrix system (Dentsply Caulk, USA)) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or BiTine ring (Dentsply Caulk, USA) placement is recommended.

7.4 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application.

Refer to adhesive manufacturer's Instructions for Use for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of the material.

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

7.5 Application

Apply material in increments immediately after the application of the adhesive. The material is designed to be cured in increments up to a 2 mm depth/thickness.

Avoid contamination of the syringe.

To prevent the syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves.

1. Remove the cap.
2. Turn the handle of the syringe slowly in a clockwise direction and dispense the necessary amount of the material onto a mixing pad.
3. Point the front tip of the syringe upwards and turn the handle anti- clockwise to prevent oozing of the material.
4. Immediately close the syringe with the cap.
5. Protect the restorative material on the mixing pad against light.

7.6 Applying ceram.x one Dentin & Enamel restorative

Rebuild the tooth with respect to its anatomical structure. Consider that depending on the thickness of the enamel and dentin layers, the optical result may vary from the selected VITA shade.

1. Rebuild the dentin with the selected dentin shade (thickness of increment ≤ 2 mm).
2. Cure each increment of dentin according to the table below [Light-curing].
3. Rebuild the enamel layer with the selected enamel shade (thickness of increment ≤ 2 mm).
4. Cure each increment of enamel according to the table below [Light-curing].

7.7 Light-curing

Light-cure each area of the restoration surface with a suitable curing light designed to cure materials containing camphorquinone (CQ) initiator, i.e. spectral output containing 470 nm. Minimum light output must be at least 500 mW/cm², exposure time according to the shade as listed in the table below. Refer to curing light manufacturer's Instructions for Use for compatibility and curing recommendations.

Curing time for 2 mm increments:

Output	≥ 500 mW/cm ²	≥ 800 mW/cm ²
ceram.x one Dentin shades (d1-d4)	40 sec	30 sec
ceram.x one Enamel shades (e1-e3)	10 sec	10 sec

7.8 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.
2. Use Enhance finishing devices (Dentsply Caulk, USA) for additional finishing.
3. Prior to polishing, check that surface is smooth and free of defects.
4. For achieving a very high luster of the restoration, we recommend Enhance and PoGo, finishing and polishing instruments, Prisma Gloss Regular and Extrafine polishing paste (Dentsply Caulk, USA).

For contouring, finishing, and/or polishing, follow the manufacturer's Instructions for Use.

8 Cleaning and disinfection

The products are supplied non-sterile. Use un-sealed syringes within 3 months. Sterilization is not acceptable.

9 Storage and shelf life

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

Store the product in a ventilated place protected from light.

Store in sealed packs at temperatures between 6 °C and 28 °C

Keep out of direct sunlight and protect from moisture.

Do not freeze.

Do not use after expiration date:

Shelf life – 2 years

Humidity can adversely affect the properties of the material.

For un-sealed syringes, the following is recommended:

Store in a dry environment (< 80% relative humidity);

Use within 3 months.

10 Transportation.

The devices should be transported according to the rules applicable to each transport mode at temperatures between 6 °C and 28 °C and relative humidity < 80%.

11 Environmental requirements

The product does not have an adverse impact on the environment during its life cycle.

12 Condition of use

Use at: +10 - +45. The devices are not subject to repair. No special maintenance is required.

13 Disposal

Disposal in accordance with the local regulations and the policies and procedures of your medical facility.

14 Manufacturer's warranties

The manufacturer is not responsible and shall not indemnify for any possible damage or accidents caused by:

- Use of components unrelated to the product which can affect normal operation.
- Noncompliance with following the instructions for use.

The user is responsible for testing of materials for fitness and use for any purpose not expressly specified in the instructions!

For product quality issues, please contact the Authorized Manufacturer's Representative.

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DENTSPLY DeTREY GmbH