Directions for use

For dental use only

Prime&Bond One Select Dental Adhesive Material with Accessories

Manufactured by:

DENTSPLY DeTrey GmbH, De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-192; +49-(0)7531-583-660-192

- 1 Description
- 2 Photos
- 3 Indication for use
- 4 Contradictions
- 5 Warnings
- 6 Adverse events
- 7 Step-by-step instructions
- 8 Cleaning and sterilization
- 9 Storage and shelf life
- 10 Transportation
- 11 Environmental requirements
- 12 Condition of use
- 13 Disposal
- 14 Manufacturer's warranties
- 15 Authorized Representative

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

1 Description

1.1 Name

Prime&Bond One Select Dental Adhesive Material with Accessories.

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1.5 Description:

One-component self-etching adhesive is designed for selective enamel etching, cleaning of mouth cavity from smear layer and photopolymerization and it is used for bonding synthetical light cure materials with enamel and dentin while direct restorations.

1.6 Delivery form

1. Prime&Bond One Select adhesive in easy-squeeze plastic bottle x 3.5 ml with screw cap; applicator tips (5- pieces) in a plastic container, CliXdish mixing well (without primary packaging) are packed in a carton box and DFU.
2. Prime&Bond One Select adhesive in easy-squeeze plastic bottle x 2.5 ml with screw cap is packed in a carton box and DFU.

Accessories:

Applicator tips – no more than 100 pcs.;

1.7 Composition and materials

1.7.1 Prime&Bond One Select Adhesive

Component	Chemical name
BAP	1,3-Bisacrylamide-propane
BAPEN	1,3-Bisacrylamide-pentane
BAA-TCD	3,(4), 8,(9)-Bisacrylamide-methyl)- tricyclo-5.2.1.0 2,6-decane
DHPOBA	Ethyl 2-[5-dihydrophosphoryl-5,2- dioxapenthyl]acrylate
AMPS	2-Acrylamide-2-methylpropane- sulfoacids
Butylate 1,2-benzenediol	2-Tper.-butyl-1,4-benzenediol
2-camphanone-quinone	(+)-2,3-Dioxi-1,7,7-trimethyl-bicyclo-(2,2,1)-heptane
Dimethylamino-benzene carbonitrile	4-(dimethylamino) benzene carbonitrile
Tert-butanol	t-Butanol
Demineralized aqua	Aqua

1.7.2 Bottle for adhesive

Component	Material
Bottle	Low density polyethylene, high density polyethylene, EVOH, ADMER
Cap	Polypropylene
Dropper	Polyethylene

1.7.3 Applicator tips

Applicator tips part	Composition (material)
Handle	Polypropylene homopolymer Stabilizers
Nylon brush	Nylon 6,6 Polyamide Titanium dioxide Absorb water
Glue	Toluene (DGEBA) Part A Xylain (Amidopolyamines) Part B

1.7.4 CliXdish mixing well

Polyamide.

1.8 Technical specifications prime&bond one select adhesive

1.8.1 Shear bond strength

Application method	Surface	Bond strength, MPa
Self-etching	Enamel	34
	Dentin	35
Total self-etching	Enamel	35
	Dentin	45

* Test method: after 6000 thermal cycles (5/55 °C), n=10; 5 mm/min (Latta MA, 2009).

1.8.2 Cure time:

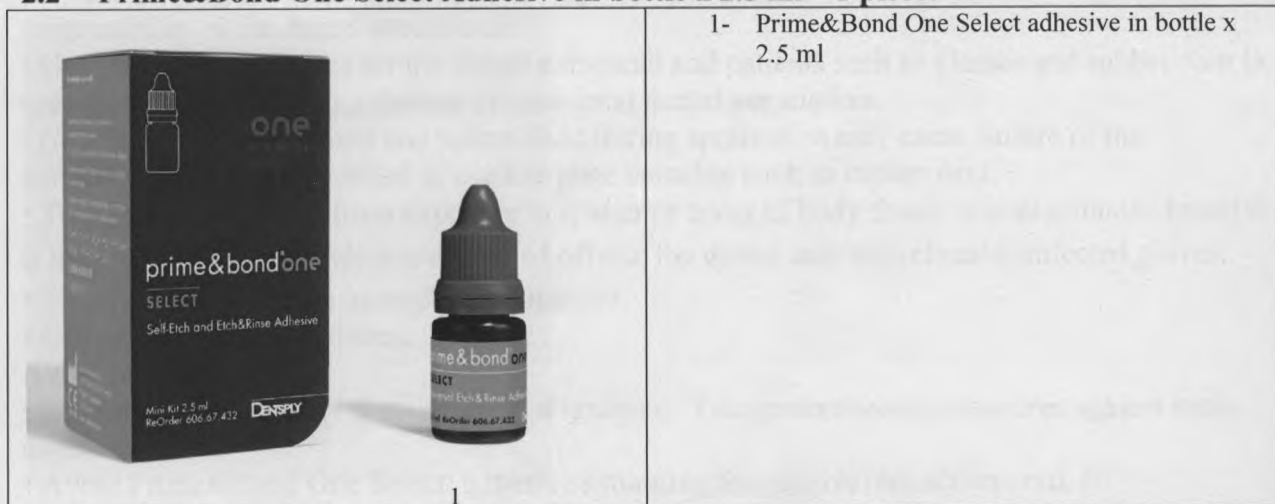
Material	Cure time	
	Output ≥ 500 mWatt/cm ²	Output ≥ 800 mWatt/cm ²
Prime & Bond One Select Adhesive	20 s	10 s

2 Photos

2.1 Prime&Bond One Select adhesive in bottle x 3.5 ml – 1 piece; applicator tips – 50 pieces; CliXdish mixing well – 1 piece

	1- Applicator tips 2- CliXdish mixing well 3- Prime&Bond One Select adhesive in bottle x 3.5 ml
---	---

2.2 Prime&Bond One Select Adhesive in bottle x 2.5 ml – 1 piece:

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4 Contradictions

- Patients who have a history of allergic reaction to acrylate or methacrylate plastics or any of the other adhesive components.
- Direct application to dental pulp (direct pulp capping).
- Use with chemical or dual-cure materials

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5.3 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

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Avoid inhaling vapor.

- Flammable: Keep away from sources of ignition. Take precautionary measures against static discharges.
- Avoid Prime&Bond One Select adhesive saturating the gingival retraction cord. If Prime&Bond One Select soaks into the cord, it may set hard and bond the cord to the underlying tooth surface making removal difficult.
- Applicator tips are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.

5.4 Interactions

- Do not use eugenol- and hydrogen peroxide-containing materials in conjunction with Prime&Bond One Select adhesive since they may cause softening of polymer components and/or interfere with hardening of the product.
- If mineral-impregnated (e.g., ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.
- Prime&Bond One Select is a light-cured material. Protect from ambient light.
- If it is refrigerated, allow material to reach room temperature prior to use.

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7 Step-by-step instructions

7.1 Cavity preparation

1. Prepare the cavity so that no residual restorative material is left.
2. In cases where no preparation or instrumentation has been carried out, roughen the surface to be bonded with a finishing bur.
3. For restoration of cervical lesions, finish enamel and dentin margins. Enamel margins should be beveled to mask the restoration/tooth interface. This may require instrumentation.
4. Rinse surface with water spray.
5. Remove rinsing water completely by blowing gently with an air syringe or by blot-drying with a (cotton) pellet. Do not desiccate the tooth substance.
6. Use a rubber dam or cotton rolls to isolate the cavity from contamination. If contamination occurs, thoroughly clean with vigorous water spray and repeat steps 4 and 5.

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

7.2 Pulp protection

In deep cavities cover the dentin close to the pulp (less than 1 mm) with a hard-setting calcium hydroxide liner (Dycal calcium hydroxide liner (Dentsply Caulk, USA), see Instructions for Use), leaving the rest of the cavity surface free for bonding.

7.3 Selective enamel-etch or etch-and-rinse tooth conditioning/dentin pre-treatment

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➡ Selective enamel-etch mode

1. After application of rubber dam or other suitable isolation technique, apply the respective conditioner.
2. Apply the respective conditioner to the enamel surfaces of the cavity.
3. For best results, condition enamel for at least 15 seconds.

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Rinsing and blot drying

1. Remove conditioner with aspirator tube and/or vigorous water spray and rinse conditioned areas thoroughly for at least 15 seconds.
2. Remove rinsing water completely by blowing gently with an air syringe or by blot drying with a cotton pellet. Do not desiccate dentin.
3. Proceed immediately to application of prime&bond one select adhesive.

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7.4 Dosage

Easy squeeze bottle

1. Dispense prime&bond one select adhesive into a CliXdish mixing well or standard dappen dish or mixing well.
2. Clean the dropper of the bottle after each application using a soft paper tissue.
3. Replace cap promptly.
4. Technique Tip: Material in a closed CliXdish mixing well will remain usable for up to 30 minutes. Material dispensed into a standard mixing well must be used immediately.

7.5 Application and curing for all modes (self-etch, selective enamel-etch, and etch-and-rinse mode)

1. Using the applicator tip, apply the product sufficiently (larger cavities may require 2 or more applications) wetting all cavity surfaces uniformly. Avoid pooling.
2. Then gently agitate the adhesive for 20 seconds.

Insufficient evaporation of solvent.

Inadequate polymerization.

1. Follow strictly the instructions of the following step.

3. Evaporate solvent thoroughly. Begin with a moderate stream of clean, dry air from a dental syringe close and strong enough to keep the adhesive layer slightly moving. Continue evaporation with more forceful air until there is no more movement of the adhesive, but for at least 5 seconds. Use a vacuum aspirator to prevent spatter of excess adhesive onto the mucosa.

4. Surface should have a uniform, glossy appearance. If not, apply prime&bond one select adhesive once more and repeat step 3. Surface should not show areas of excessive adhesive thickness or pooling.

5. Cure prime&bond one select adhesive for 10 seconds.

Ensure uniform exposure of all surfaces.

Insufficient curing.
Inadequate polymerization.
1. Check compatibility of curing light.
2. Check curing cycle.
3. Check curing output.

6. Immediately place restorative material over the cured prime&bond one select adhesive as per restorative material manufacturer's Instructions for Use.

Multiple cavities.
When restoring multiple cavities it is advisable to fully restore one cavity (adhesive and restorative) before proceeding to the next cavity.

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Cross-contamination.
Infection.
1. Do not reuse single use products. Dispose in accordance with local regulations.
2. Reprocess reusable products as described below.

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8.1.1 Manual cleaning:

1. Discard gloves according to local regulations. Disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's Instructions for Use.
2. Use a new clean pair of examination gloves.
3. Disassemble the device.
4. Remove adhesive residue with a cloth.
5. Clean the device with a soft brush, warm water, and an appropriate cleaning solution until free of visible contamination. Pay special attention to device seams and insertions.
6. Remove cleaning solution residue with a cloth soaked with water.
7. Dry the device with a lint-free single-use cloth.

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8.1.3 Automatic Washer-Disinfector:

As an alternative to manual cleaning and disinfection, the device can also be processed in a washer-disinfector (for example, Miele Washer-Disinfector G7835 CD using program Vario TD with neodisher MediClean (alkaline detergent) and neodisher Z (acid neutralization and cleaning detergent); both detergents from Dr. Weigert, Hamburg, Germany (not products or registered trademarks of DENTSPLY International, Inc.)). Use only properly maintained, calibrated, and approved washer-disinfector according to ISO 15883-1.

1. Disassemble the device.
2. Remove adhesive residue with a cloth.
3. Place the device in the washer-disinfector allowing water and detergent to enter into and drain off from device openings during washer-disinfection program and follow washer-disinfector's Instructions for Use.
4. Run washer-disinfection program with A0 value ≥ 3000 (e.g. 5 min at ≥ 90 °C) using appropriate detergents.

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8.1.4 Sterilization (if required)

The device can also be autoclaved.

Steam autoclave the device unwrapped at 134 °C/ 2 bar for 3 minutes 30 seconds.

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- Store the product in a well-ventilated area and protect from sunlight.
 - Store the product at temperatures between 2 °C and 24 °C. Use the product at room temperature.
 - Protect from moisture.
 - Do not freeze.
- Do not use after expiration date.

10 Transportation

The devices should be transported according to the rules applicable to each transport mode at the temperature 2- 24 °C.

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Directions for use

For dental use only

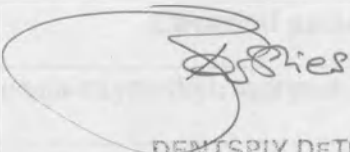
Prime&Bond One Etch&Rinse Dental Adhesive Material with Accessories

Manufactured by:

DENTSPLY DeTrey GmbH, De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-192; +49-(0)7531-583-660-192

- 1 Description
- 2 Images
- 3 Indication for use
- 4 Contradictions
- 5 Warnings
- 6 Adverse events
- 7 Step-by-step instructions
- 8 Cleaning and Sterilization
- 9 Storage and shelf life
- 10 Transportation
- 11 Environmental requirements
- 12 Condition of use
- 13 Disposal
- 14 Manufacturer's warranties
- 15 Authorized Representative

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

1 Description

1.1 Name:

Prime&Bond One Etch&Rinse Dental Adhesive Material with Accessories.

1.2 Information about medical device manufacturer/developer:

«DENTSPLY DeTrey GmbH»,

De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-192; +49-(0)7531-583-660-192; e-mail: webmaster@dentsply.de

1.3 Addresses of medical device manufacturing sites:

«DENTSPLY DeTrey GmbH», Германия

De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-192; +49-(0)7531-583-660-192; e-mail: webmaster@dentsply.de

1.4 Indication for use:

Dental self-etching adhesive material for direct and indirect restorations.

1.5 Description:

Universal self-etching adhesive in a bottle is specially designed to bond composite materials to enamel and dentin as well as to metals and ceramic.

Prime&Bond One Etch&Rinse universal self-etching adhesive offers excellent performance due to high bond strength to enamel and dentin, easy and comfortable application and a high degree of application robustness.

Prime&Bond One Etch&Rinse is singly used with light-cured restorative materials and cements. When it is mixed with "Self Cure Activator" (DeTrey, Germany, available separately), it is used with self-cure and dual-cure materials. In this case refer to instruction for using "Self Cure Activator".

1.6 Delivery forms:

1. Adhesive Prime&Bond One Etch&Rinse in a bottle 3,5 ml with applicator tips – 50 pcs., CliXdish mixing well – 1 pcs., Instructions for Use – 1 pcs.

2. Adhesive Prime&Bond One Etch&Rinse in a bottle 2,5 ml with the Instructions for Use – 1 pcs.

Accessories:

1. Applicator tips – no more than 100 pcs.;

Additionally, prime & bond one etch & rinse in bottle 3,5 ml could be included into ceram.x one universal Starter Kit and ceram.x one dentin & enamel Starter Kit.

1.7 Composition and materials

1.7.1 Prime&Bond One Etch & Rinse Adhesive

Component	Chemical name
Phosphoric acid modified acrylate resin (PENTA)	2,2,4,4,4-penta-oxymethylenacrycol-3-oxo-1-phosporic acid
Urethane dimethacrylate resin (UDMA)	2,7,7,9,15-Pentamethyl-4,13-dioxo-3,14-dioxa-5,12-diaza-hexadecan-1,16-diyl-dimethacrylat

Carboxylic acid modified dimethacrylate (TCB resin)	Bytane -1,2,3,4-tetracarboxylic acid, bis[2-hydroethyl-methacrylate] ester
Triethylenglycol-dimethacrylate (TEGDMA)	Triethylenglycoldimethylacrylate
2-hydroxyethylmeth-acrylat (HEMA)	2-propenoic acid, 2-methyl-, 2-hydroxyethyl ester
Camphor quinone (CQ)	(±)-2,3-dioxo-1,7,7-trimethyl-bicyclo-(2,2,1)-heptan
Ethyl-4-dimethylamonibenzoate (DMABE)	Ethyl-4-dimethylaminobenzoate
Tert.-butyl-hydroquinone (TBHQ)	2-tert.-butyl-1,4-benzene diol
Nanofiller	Functionalized amorphous silica
Tert.-butanol	2-methyl-propanol-2

1.7.2 Bottle for adhesive

Компонент	Материал
Bottle	Low density polyethylene, high density polyethylene, EVOH, ADMER.
Cap	Polypropylene
Dropper	Polyethylene

1.7.3 Applicator tips

Applicator tips part	Composition (material)
Handle	Polypropylene homopolymer
	Stabilizers
Nylon brush	Nylon 6,6 Polyamide
	Titanium dioxide
	Absorb water
Glue	Toluene (DGEBA) Part A
	Xylain (Amidopolyamines) Part B

1.7.4 CliXdish mixing well CliXdish

Polyamide.

1.8 Technical specifications of prime&bond one etch & rinse adhesive

1.8.1 Shear bond strength:

Dentin moisture	Bond strength, MPa
Optimum	39
Over-dried	31
Over-wet	32

*Test method: after 1800 thermal cycles (5/55 °C), n=10; 5 mm/min (Latta MA, 2009).

1.8.2 Micro-shear fatigue resistance:

Parameter	Units	Value
Fatigue resistance*	MPa	30

*Test method: median micro-shear fatigue resistance (μSFR) method, 10 000 cyclic actions 2Hz (35°C).

1.8.3 Cure time:

Material	Cure time	
	Output	Output
	≥500	≥800


	mWatt/cm ²	mWatt/cm ²
prime & bond one etch & rinse adhesive	20 s	10 s

2 Images

2.1 Prime&Bond One Etch&Rinse adhesive in bottle x 3.5 ml – 1 piece; applicator tips – 50 pieces; CliXdish mixing well – 1 piece

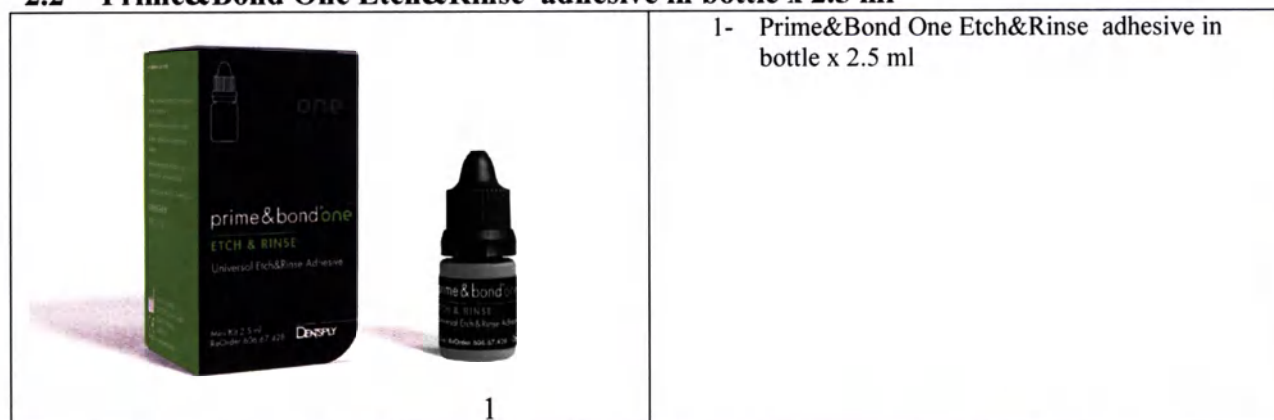2.2 Prime&Bond One Etch&Rinse adhesive in bottle x 2.5 ml

3 Indication for use

Dental self-etching adhesive material for direct and indirect restorations.

Prime&Bond One Etch&Rinse adhesive is designed for direct restorations of all cavity classes in combination with light-cured composite restorative materials.

Prime&Bond One Etch&Rinse adhesive is 5th generation adhesive for:

- Direct, light-cured composite and compomer restoratives.
- Indirect restorations; light-cured, composite cemented inlays, onlays, crowns, bridges and retainers.
- Composite/composer, ceramic and amalgam repairs.
- Cavity varnish for use with fresh amalgam.

It is necessary to refer to instruction for using “Self Cure Activator” when using Prime&Bond One Etch&Rinse for the following indications:

- Direct, chemical and dual-cure composite restoration and restoration of residual limb.
- Indirect restorations; fixation on cement of chemical and dual-cure inlays, onlays, crowns, bridges and retainers.
- Fixation on cement of chemical or dual-cure endodontic post cementation.

When using Prime&Bond One Etch&Rinse for adhesive bonding of direct amalgam restorations please refer to the Instructions for use of “Amalgam Bonding Accessory Kit” (available not in all the countries).

Handwritten signature

4 Contradictions

- Patients who have a history of allergic reaction to acrylate or methacrylate plastics or any of the other adhesive components.
- Direct application to dental pulp (direct pulp capping).

5 Warnings

Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

Warnings

The material contains polymerizable acrylate and methacrylate monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental personnel and patients such as glasses and rubber dam in accordance with recommendations of your local dental association.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. It is recommended to use adequate isolation such as rubber dam.
- To prevent a bottle tip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the devices are handled offside the dental unit with clean/disinfected gloves.
- Tightly close the bottle immediately after use.
- Use in a well-ventilated area.

Avoid inhaling vapor.

- **Flammable:** Keep away from sources of ignition. Take precautionary measures against static discharges.
- Avoid Prime&Bond One Etch&Rinse adhesive saturating the gingival retraction cord. If Prime&Bond One Etch&Rinse soaks into the cord, it may set hard and bond the cord to the underlying tooth surface making removal difficult.
- Applicator tips are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.

Interactions:

- Do not use eugenol- and hydrogen peroxide-containing materials in conjunction with Prime&Bond One Etch&Rinse adhesive since they may cause softening of polymer components and/or interfere with hardening of the product.
- If mineral-impregnated (e.g., ferric compounds) retraction cords and/or hemostatic solutions

are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.

- Prime&Bond One Etch&Rinse is a light-cured material. Protect from ambient light.
- If it is refrigerated, allow material to reach room temperature prior to use.

6 Adverse events

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

7 Step-by-step instructions

7.1 Direct restoration (light-cured composite resin and compomers)

7.1.1 Cleaning

1. Clean uninstrumented enamel and dentin with a rubber cup and pumice or a non-fluoride cleaning paste such as nupro prophylaxis paste (Dentsply Professional, USA).
2. Wash thoroughly with water spray and air dry.
3. Clean freshly instrumented enamel and dentin with water spray, then air dry.

7.1.2 Pulp protection

1. In deep cavities cover the dentin close to the pulp (less than 1 mm) with a hard-setting calcium hydroxide liner (Dycal calcium hydroxide liner (Dentsply Caulk, USA), see instructions for use) leaving the rest of the cavity surface free for bonding.

7.1.3 Tooth conditioning/dentin pre-treatment

When used as a bonding agent for direct restorative materials, it is recommended to follow the total etch technique described in this chapter. Prior to use, please see the instructions for use of the respective conditioner.

Application of 34%-36% phosphoric acid conditioner

1. After application of rubber dam or other suitable isolation technique, apply the respective conditioner.
2. Apply the respective conditioner to the cavity surfaces starting at the enamel margins.
3. For best results, condition enamel for at least 15 seconds and dentin for 15 seconds.

Rinsing and blot drying

1. Remove conditioner with aspirator tube and/or vigorous water spray and rinse conditioned areas thoroughly for at least 15 seconds.
2. Remove rinsing water completely by blowing gently with an air syringe or by blot drying with a cotton pellet. Do not desiccate dentin.
3. Proceed immediately to application of prime&bond one etch&rinse adhesive.

Contamination.

Restoration failure.

1. Once the surfaces have been properly treated, they must be kept uncontaminated.
2. If salivary contamination occurs, thoroughly clean with vigorous water spray, dry, and repeat conditioning procedure for 5 seconds only. Rinse and dry as described above.

7.1.4 Dosage

1. To prevent the bottle from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the bottle is handled offside the dental unit with clean/disinfected gloves.
2. Dispense prime&bond one etch&rinse adhesive directly onto a clean applicator tip, making sure that the bottle does not come in direct contact with the applicator tip, or place one to two

drops of prime&bond one etch&rinse adhesive into a clean CliXdish mixing well or standard dappen dish.

3. Replace cap promptly.

Technique Tip: Material in a closed CliXdish mixing well will remain usable for up to 15 minutes. Material dispensed to a standard mixing well must be used immediately.

7.1.5 Application and curing

1. Using the applicator tip, apply the product wetting all cavity surfaces uniformly. Avoid pooling. This surface should remain undisturbed for 20 seconds.

2. Evaporate solvent by thoroughly drying with clean, dry air from a dental syringe for at least 5 seconds. Use a vacuum aspirator to prevent spatter of the adhesive onto the mucosa. Surface should have a uniform glossy appearance. If not, repeat application and air dry.

3. Cure prime&bond one etch&rinse adhesive for 10 seconds 2 using a curing light.

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.

2. Check curing cycle.

3. Check curing output.

Check curing light for minimum light output of at least 800 mW/cm². Cure for at least 20 seconds if light output is between 500 and 800 mW/cm².

4. Place restorative material over the cured prime&bond one etch&rinse adhesive as per restorative material manufacturer's instructions for use.

7.2 Light-cured composite, ceramic and amalgam repairs

7.2.1 Preparation

1. Roughen and clean fractured restoration surface with a fi diamond bur and create mechanical retention, if possible.

7.2.2 Tooth conditioning/dentin pre-treatment

1. Etch fractured tooth surface with the respective conditioner as outlined in 7.1.3 tooth conditioning/dentin pre-treatment.

7.2.3 Treatment of restoration

1. Etch fractured restoration surface with the respective conditioner for at least 15 seconds as outlined in 7.1.3 tooth conditioning/dentin pre-treatment.

7.2.4 Application and curing

1. Apply adhesive to the fractured restoration surfaces and light-cure 2 the adhesive [7.1.5 application and curing].

7.2.5 Completion

1. Complete repair with placement and curing of desired shade(s) of light-cured composite restorative following manufacturer's instructions for use.

7.3 Cementation of light transmitting indirect restoration with a light-cured resin cement

7.3.1 Treatment of restoration

1. Treat internal bonding surface of restoration according to manufacturer's or dental laboratory's instructions for use, i.e., etching, mechanical roughening and/or Silane Coupling agent application.

2. Apply a single coat of prime&bond one etch&rinse adhesive to the internal bonding surface of the restoration. immediately air dry for 5 seconds.

Light-curing of applied prime&bond one etch&rinse adhesive is not necessary.

Fraser

7.3.2 Treatment of tooth preparation

1. Clean and condition tooth preparation, apply and light- cure the material as described in 7.1 Direct restoration (light-cured composite resin and compomers).

7.3.3 Cementation

1. Prepare and apply light-cured resin cement according to manufacturer's instructions for use.

7.4 Cavity varnish for use with fresh amalgam

When used as a cavity varnish, prime&bond® one etch&rinse adhesive is not an amalgam adhesive.

7.4.1 Preparation

1. Finish preparation by removing existing restorations and/ or caries.

7.4.2 Cleaning

Clean preparation and place a hard-setting calcium hydroxide liner (Dycal calcium hydroxide liner (Dentsply Caulk, USA), see instructions for use) if needed, according to:

1. Cleaning [7.1.1 Cleaning].
2. Pulp protection [7.1.2 Pulp protection].

7.4.3 3.4.3 Tooth conditioning/dentin pre-treatment

1. Rinse and carefully air dry cavity preparation, but do not desiccate exposed dentin.
2. Condition enamel and dentin following section 7.1.3 tooth conditioning/dentin pre-treatment.

7.4.4 Application

1. Apply and light cure prime&bond one etch&rinse adhesive as described for direct restorations, see section 7.1.3 tooth conditioning/dentin pretreatment.

7.4.5 Completion

1. Place and condense amalgam as per manufacturer's directions.

8 Cleaning and Sterilization

The products are supplied non-sterile. Sterilization of dental material and accessories is not required. Except for CliXdish mixing well that is reusable. It can be cleaned, disinfected and sterilized.

Cross-contamination.
Infection.

1. Do not reuse single use products. Dispose in accordance with local regulations.
2. Reprocess reusable products as described below.

8.1 CliXdish mixing well is reusable. Cleaning, disinfection and sterilization procedure (if necessary):

8.1.1 Manual cleaning:

1. Discard gloves according to local regulations. Disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's Instructions for Use.
2. Use a new clean pair of examination gloves.
3. Disassemble the device.
4. Remove adhesive residue with a cloth.
5. Clean the device with a soft brush, warm water, and an appropriate cleaning solution until free of visible contamination. Pay special attention to device seams and insertions.

DENTSPLY DETREY GMBH
De-Trey-Straße 1
D-78467 Konstanz

6. Remove cleaning solution residue with a cloth soaked with water.
7. Dry the device with a lint-free single-use cloth.

8.1.2 Manual disinfection:

1. After cleaning (8.1.1 Manual cleaning) thoroughly wipe all device surfaces with a single-use cloth in combination with an alcohol-based, bactericidal, virucidal and fungicidal disinfection solution approved according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions.
2. Remove disinfectant solution residue with a cloth soaked with water.
3. Dry the device with a lint-free single-use cloth.

The device may be manually disinfected for up to 200 times.

8.1.3 Automatic Washer-Disinfector:

As an alternative to manual cleaning and disinfection, the device can also be processed in a washer-disinfector (for example, Miele Washer-Disinfector G7835 CD using program Vario TD with neodisher MediClean (alkaline detergent) and neodisher Z (acid neutralization and cleaning detergent); both detergents from Dr. Weigert, Hamburg, Germany (not products or registered trademarks of DENTSPLY International, Inc.).

Use only properly maintained, calibrated, and approved washer-disinfector according to ISO 15883-1.

1. Disassemble the device.
2. Remove adhesive residue with a cloth.
3. Place the device in the washer-disinfector allowing water and detergent to enter into and drain off from device openings during washer-disinfection program and follow washer-disinfector's Instructions for Use.
4. Run washer-disinfection program with A0 value ≥ 3000 (e.g. 5 min at $\geq 90^\circ\text{C}$) using appropriate detergents.

The device may be cleaned in the washer-disinfector for up to 500 times.

8.1.4 Sterilization (if required)

The device can also be autoclaved.

Steam autoclave the device unwrapped at 134°C / 2 bar for 3 minutes 30 seconds.

The device may be disinfected for up to 200 times and sterilized for up to 20 times.

9 Storage and shelf life

Expiration date – 2 years. Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store the product in a well-ventilated area and protect from sunlight.
- Store the product at temperatures between 2°C and 24°C . Use the product at room temperature.
- Protect from moisture.
- Do not freeze.
- Do not use after expiration date.

10 Transportation

The devices should be transported according to the rules applicable to each transport mode at temperatures between 2°C and 24°C .

11 Environmental requirements

The product does not have an adverse impact on the environment during its life cycle.

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

12 Condition of use

Use at: +10 - +45. The devices are not subject to repair. No special maintenance is required.

13 Disposal

Disposal in accordance with the local regulations and the policies and procedures of your medical facility.

14 Manufacturer's warranties

The manufacturer is not responsible and shall not indemnify for any possible damage or accidents caused by:

- Use of components unrelated to the product which can affect normal operation.
- Noncompliance with following the instructions for use.

The user is responsible for testing of materials for fitness and use for any purpose not expressly specified in the instructions!

For product quality issues, please contact the Authorized Manufacturer's Representative.

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