LABORATOIRES GILBERT

S.A.S. au capital de 5.048.000 euros

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Mme Deverly

(seen exclusively to certify the above signature)

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The device for lavage and irrigation of nasal cavities for children

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**The device for lavage and irrigation of nasal
cavities for children MARIMER Baby, nasal
spray, versions: 50 ml, 100 ml**

Composition for 100 ml

Active substance: sterile purified seawater –
28 ml
Excipient: purified water - H₂O – up to 100 ml
pH 6,5-8,5;
NaCl 9 g/L.

50 or 100 ml of the sterile solution in a polyethylene container inside of an aluminum bottle with a polyethylene spraying nozzle and a polypropylene cap.

The bottle and the instruction for use are put into a carton.

Description of medical device

Everyday hygiene of nasal cavities provides protection the nasal mucosa against aggressive environment (cold, the presence of allergens in the air, dryness or air pollution, etc.), reduces the risk of allergic or oto-rhino-laryngological infectious diseases, it helps to maintain respiratory function of the nose. Children under 6 months exclusively breathe nose, so it's especially important to wash the nasal cavity of infants daily.

«MARIMER Baby» - is sterile seawater saline solution, where the concentration of sodium chloride amount to 9 g/L, which is equivalent to isotonic solution. Seawater of the Atlantic ocean (region of Saint-Malo, France) is used; it contains natural mineral salts and microelements. «MARIMER Baby» is recommended for daily nasal lavage and wetting of nasal mucosa of babies right from birth. Special valve provides solid protection against bacterial contamination of the solution in a balloon (the solution remains sterile).

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Microdiffusion system of provides deep and gentle irrigation of mucosa that makes microelements effectively.

Innovative spraying nozzle is specifically designed for newborns; the tip size corresponds to physiological structure of the nasal passage of a newborn baby. The spraying nozzle has a wide tip and gradually thickens to the base in order to avoid deep penetration into the nasal passage of a child, and the special stopper provides additional protection.

Also the spraying nozzle has a rounded shape to avoid the risk of injury when used. The bottle can be sprayed in any position.

Prescription

Sterile isotonic solution of purified seawater is meant for penetration, cleaning, freeing and wetting of mucosa of nasal passages and nasal cavities. The solution is self administered by pressing the spraying nozzle in the course of postoperative, preventive or symptomatic treatments of the diseases of the nose. The solution is contained in a polyethylene container inside of an aluminum bottle, equipped with a polyethylene spraying nozzle with a valve, which provides maintenance of sterility during the shelf life.

Indications for use

The device for lavage and irrigation of nasal cavities for children MARIMER Baby, nasal spray, is used for babies, including newborns, for:

- Daily hygiene of nasal cavities and to preserve the protective properties of mucosa under conditions of increased dryness or pollution of air (conditioning, central heating etc.);

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- Elimination therapy for the prevention and treatment of oto-rhino-laryngological and allergic diseases, as well as viral infections (influenza, cold).

Usage during the period of pregnancy and breast-feeding

The device can be used during the period of pregnancy and lactation.

Way of use and dosage

The device should be used under normal climatic conditions.

One spray per nostril 1-4 times a day before meal preferably.

«MARIMER Baby» can be used in children in any position: lying down, sitting or standing (it is recommended to take into account the general condition of a child and age characteristics).

- Turn the child's head to one side;
- Carefully insert the tip into the nasal passage above and press the spraying nozzle;
- Turn the child's head to another side and make spraying into another nasal passage;
- Straighten the head;
- In newborns – wipe nasal discharge, at a runny nose – use an aspirator, for example, baby nasal aspirator «Marimer»
- Older children are recommended to blow the nose gently.

After each use rinse the spraying nozzle with warm soapy water, clean with water and dry.

Contraindications

Do not use in newborn premature babies.

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Sterilization by ozonation assures the solution during the claimed shelf

Precautions for use

The content of the bottle is under pressure. Keep away from sources of heat and open flame. Do not smoke. Avoid direct sunlight and heat above 50 °C. Do not pierce or burn the bottle even after use.

Keep out of the reach of children.

Usage and storage conditions

The device for lavage and irrigation of the nasal cavities for children MARIMER Baby, nasal spray, while in operation is resistant to climatic factors for temperatures from 10 °C to 35 °C and a relative humidity of no more than 80 %.

Store at room temperature.

Transportation

Transportation can be performed by any type of enclosed transport in accordance with haulage rules applicable for this kind of transport.

Transportation must comply with standards for transportation of medical devices and is carried out in special containers approved by the company Laboratoires GILBERT, France, and resistant to mechanical impact and changes in climatic factors.

Transport conditions:

At a temperature from 0 °C to 50 °C and a relative humidity of no more than 80 %.

Procedures for disposal and destruction

In accordance with local requirements for the disposal of household waste.

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E-mail: abbott-russia@abbott.com

*The organization is authorized by the MA
holder to accept consumer claims:*

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