Directions for use

For dental use only

Spectrum Dental Light-Cured Restorative Material

Manufactured by:

DENTSPLY DeTrey GmbH, De-Trey-Strasse 1, 78467 Konstanz, Germany
Tel: +49-(0)7531-583-0; +49-(0)7531-583-104

2015

1 Description

1.1 Name:

Spectrum Dental Light-Cured Restorative Material.

1.2 Information about medical device manufacturer/developer:

«DENTSPLY DeTrey GmbH»,

De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-0; +49-(0)7531-583-104; e-mail: webmaster@dentsply.de

1.3 Addresses of medical device manufacturing sites:

«DENTSPLY DeTrey GmbH», Германия

De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-0; +49-(0)7531-583-104; e-mail: webmaster@dentsply.de

1.4 Description:

Spectrum is a light-cured, radiopaque micro-hybrid composite for anterior and lateral restorations. The composite is delivered in traditional syringes and is available in a selection of VITA shades.

1.5 Indication for use:

Dental light-cured restorative materials are used for restoration of anterior and lateral teeth.

Scope: dentistry.

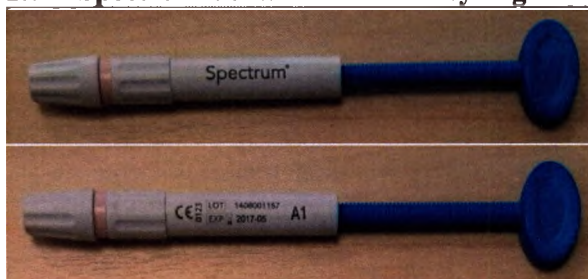
These products should be used only in clinical environments by qualified dental personnel.

1.6 Delivery forms:

Spectrum Dental Light-Cured Restorative Material in syringe 4g – 1 pc per pack. Is available in a selection of shades A1, A2, A3, A3.5, A4, B1, I-B1, B2, B3, C2, C3, O-A2, O-A3.5. The material is supplied as a component of Spectrum kits (only shades: A1, A2, A3, A3.5, B2, O-A2, O-A3.5) or can be supplied separately (all shades).

2 Images

2.1 Spectrum dental material in syringes x 4 g (shade A1 as example):

2.2 Package and labeling (shade A1 as example):

3 Composition and materials

3.1 Spectrum dental material:

- Triethyleneglycol dimethacrylate (TEGDMA)
- Dimethacrylate
- Multifunctional polymethacrylate
- Camphorquinone
- Ethyl-4(dimethylamino)benzoate
- Butylated hydroxy toluene (BHT)
- UV stabilizer
- Barium-aluminium-borosilicate glass (mean particle size < 1 μm)
- Silanated barium-boron-fluoro-alumino-silicate glass (mean particle size < 1 μm)
- Highly dispersed silicon dioxide (particle size 10-20 nm)
- Fluorescent agent
- Ferric oxide pigments
- Ferrous oxide pigments

3.2 Syringe:

- Polypropylene;
- Polybutylene terephthalate.

4 Technical specifications

4.1 Spectrum dental material in syringes x 4 g; shades: A1, A2, A3, A3.5, A4, B1, I-B1, B2, B3, C2, C3, O-A2, O-A3.5:

4.1.1 Appearance: homogeneous paste

4.1.2 Weight of the material in the syringe $4 \pm 0.01g$

4.1.3 Layer thickness: 3,2- 4,5 mm

It characterizes the deformation of the pasty material with significant load.

4.1.4 Consistency: 16 – 21 mm

At the temperature 37°C.

4.1.5 Depth of Cure: (ISO 4049:2000)

A3,5, A4, B3, C4, O-A2, O-A3,5	minimum 2,0 mm
A1, A2, A3, B1, B2, C2, C3, I-B1	minimum 2,5 mm

4.1.6 Opacity / Colour

Shades	Opacity, %
A1, A2, A3, A3,5, A4, B1, B2, B3, C2, C3, C4	39 - 43
O-A2, O-A3,5	48-52
I-B1	27-31

4.1.7 Extrudability: 50 – 120N

4.1.8 The main technical characteristics of the polymerized material:

Characteristics	Units	Value
Flexural strength	MPa	≥ 80
Water Sorption	$\mu g/mm^3$	≤ 40
Solubility	$\mu g/mm^3$	≤ 7.5
Color Stability	No change	No change
Radiopacity	mm Al	≥ 1

4.2 Syringe

Not applicable

5 Indication for use

Restorations of all type cavities in anterior and posterior teeth.

6 Contradictions

Patients who have a history of allergic reaction to acrylate or methacrylate plastics or any of the other material components.

7 Warnings

Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

Warnings

The material contains methacrylates and polymerizable monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons. The material is an acid agent and may cause burns.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental personnel and patients such as glasses and rubber dam in accordance with recommendations of your local dental association.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. It is recommended to use adequate isolation such as rubber dam.
- To prevent a bottle tip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the devices are handled offside the dental unit with clean/disinfected gloves.
- Tightly close syringes immediately after use.
- To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringes are handled with clean/disinfected gloves. Discard syringes if contaminated.
- Interactions:
 - Do not use eugenol- or hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening of the product.
 - If refrigerated, allow material to reach room temperature prior to use.

8 Adverse events

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

9 Step-by-step instructions

9.1 Shade selection

Shade selection should be made prior to the restorative procedure while the teeth are hydrated. Remove extraneous plaque or surface stain if any with prophylaxis paste. Use the Spectrum shade guide provided with samples of original Spectrum restorative material. The color coding dot on the shade guide matches the colored label on the syringe.

Alternatively, a VITAPAn classical shade guide may be used. The Spectrum shade corresponds to the central part of the respective VITA tooth.

**Avoid contamination of the shade guide.**

To prevent the shade guide from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. For shade selection, remove individual tabs from shade guide holder. Always make sure individual tabs are completely reprocessed before placing back into shade guide.

9.2 Cavity preparation

1. Clean the tooth surface with a fluoride free prophylaxis paste (e.g. nupro prophylaxis paste).
2. Prepare cavity (unless not necessary, e.g. cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

9.2.1 Matrix placement and wedging (Class II restorations)

1. Place a matrix (e.g. AutoMatrix or Palodent Plus matrix system) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or BiTine ring placement is recommended.

9.3 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application

Refer to adhesive manufacturer's Instructions for use for pulp protection, tooth conditioning and/or adhesive application. once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of the material.

9.4 Application

Apply material in increments immediately after the application of the adhesive. The material is designed to be cured in increments up to a 3 mm depth/thickness

**Avoid contamination of the syringe.**

To prevent the syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves.

9.5 Light-curing

1. Light-cure each area of the restoration surface with a suitable curing light designed to cure materials containing camphorquinone (CQ) initiator, i.e. spectral output containing 470 nm. Minimum light output must be at least 500 mW/cm². Refer to curing light manufacturer's Instructions for use for compatibility and curing recommendations.
2. Cure each layer according to the table below.

**Insufficient curing.****Inadequate polymerization.**

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check curing output.

Curing Light Output $\geq 500 \text{ mW/cm}^2$

Shades	Increment/ layer	Curing time
A1; A2; A3; A3,5; B1; Incisal-B1; B2; C2; C3; D3	3 mm	20 sec
Opaque-A2; opaque-A3,5; A4; B3; C4	2 mm	20 sec

9.6 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.

2. Use Enhance finishing devices for additional finishing.
3. Prior to polishing, check that surface is smooth and free of defects.
4. For achieving a very high luster of the restoration, we recommend Enhance and PoGo, finishing and polishing instruments, Prisma Gloss Regular and Extrafine polishing paste.

For contouring, finishing, and/or polishing, follow the manufacturer's Instructions for use.

10 Cleaning, disinfection and sterilization

The products are supplied non-sterile. Sterilization of dental material and accessories is not required.

11 Storage and shelf life

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store at temperatures between 2 °C and 28 °C.
- Keep out of direct sunlight and protect from moisture.
- Do not freeze.
- Do not use after expiration date. Shelf life of Spectrum - 3 years
- Store in a dry environment (< 80% relative humidity).

12 Transportation

The devices should be transported according to the rules applicable to each transport mode at temperatures between 2 °C and 28 °C and relative humidity <80%.

13 Environmental requirements

The product does not have an adverse impact on the environment during its life cycle.

14 Condition of use

Materials are correctly operated intraoral in presence of physiological liquids at nominal temperatures: 32 ° to 42 ° C. The products cannot be repaired. Special service is not required.

15 Disposal

Dispose in accordance with the sanitary rules and regulations of the Russian Federation 2.1.7.2790-10 SanPiN "Sanitary requirements for handling of medical waste." Hazard category of medical waste - B.

16 Symbols used in labeling:

Symbol	Explanation
	Manufacturer's name and address
	CE mark
	PCT mark
	Serial number (batch/lot number)
	Expiration date
	Caution: consult the Instructions for Use

	Temperature limits; both lower and upper temperature limits are noted next to horizontal lines
	Protect from sunlight
	See webpage for instructions for use
	The thickness of the solidification
	Required power light source
	Required solidification time

17 Manufacturer's warranties

The manufacturer is not responsible and shall not indemnify for any possible damage or accidents caused by:

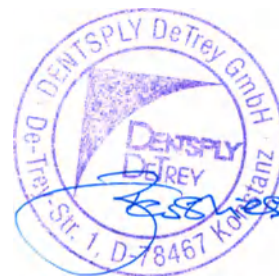
- Use of components unrelated to the product which can affect normal operation.
- Noncompliance with following the instructions for use.

The user is responsible for testing of materials for fitness and use for any purpose not expressly specified in the instructions!

For product quality issues, please contact the Authorized Manufacturer's Representative.

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