
FACE A FACE
PARIS
WOOW

"APPROVE"

ARCHITECTURES SAS, France

The President, Pascal JAULENT

Thursday 16th April, 2015

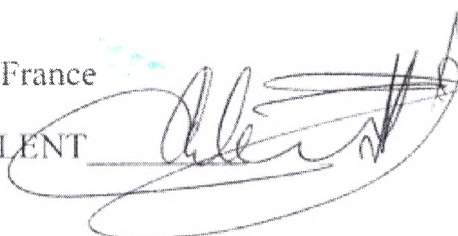**Instruction for Use
of frames for vision correction spectacles (optical frames)**

1. PURPOSE: Metal, plastic, combined, semi-rimless frames for vision correction spectacles (optical frames) with transportation demo-lens made by the company ARCHITECTURES SAS are designed for fixation of spectacle lenses and proper positioning of lenses relative to eyes.

2. BASIC TECHNICAL CHARACTERISTICS AND DESIGN

Optical frames consist of the frame, temples, nose pads and hinges. Hinges may be simple and spring.

Optical frames are characterized by three (3) major geometric parameters: width (a), width of a bridge (b), and length of temple to the curvature (L). Dimensions used for frames made by ARCHITECTURES SAS company are listed in the Table:

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FACE A FACE

PARIS

WOOW

Optical frames		Distance between vertical tangents to the facet rim gutter (a).	Minimal distance between vertical tangents to the base of the facet rims gutter (b).	Length of temple from the hinge axis to the beginning of the curvature (L), mm	Frame weight without lenses, g
Metal optical frames	Men	51±0,2 mm	17±0,2 mm	110±1,0	19.0±1,0
	Women	50±0,2 mm	16±0,2 mm	90±1,0	19.0±1,0
	Unisex	51±0,2 mm	17±0,2 mm	110±1,0	19.0±1,0
Plastic optical frames	Men	57±0,2 mm	15±0,2 mm	110±1,0	20.5±1,0
	Women	55±0,2 mm	16±0,2 mm	90±1,0	20.2±1,0
	Unisex	57±0,2 mm	18±0,2 mm	110±1,0	20.4±1,0
Combined optical frames	Men	57±0,2 mm	19±0,2 mm	110±1,0	19.9±1,0
	Men	55±0,2 mm	19±0,2 mm	110±1,0	19.9±1,0
	Women	50±0,2 mm	18±0,2 mm	90±1,0	19.4±1,0
	Unisex	52±0,2 mm	18±0,2 mm	110±1,0	19.8±1,0
Semi-rimless optical frames	Men	52±0,2 mm	17±0,2 mm	110±1,0	19.0±1,0
	Women	50±0,2 mm	18±0,2 mm	90±1,0	19.0±1,0
	Unisex	52±0,2 mm	18±0,2 mm	110±1,0	19.0±1,0

The surface of the frames shall not bear sharp edges and burrs, cavities, scratches, bubbles, cuts of solder visible by unarmed eye.

Temples should turn around the axis smoothly and without adhesion when moving.

Decoration is not used.

Decorative coatings – acrylic varnish from methylmethacrylate with butylakrilat and vinyl monomer-styrene.

The spectacle frames are subject to ISO EN 12870.

Ch

FACE A FACE

PARIS

WOOW

Optical frame are selected with the account for anthropometric data of the patient.

3. CONTRAINDICATIONS TO USE

The following may be considered to be contraindication to use:

- Individual intolerability of vision correction when wearing spectacles;
- Impossibility of vision correction when wearing spectacles.

4. MATERIALS APPLIED

The frames are made of materials which cannot cause irritation, allergic or toxic reactions in the course of their use when contacting human skin.

Plastic frames are produced from cellulose acetate (Parade 209389).

Metal frames are produced from:

copper-nickel alloys (Monel KA 500)

temple tips- from cellulose acetate (Parade 209389)

and nose pads - from silicone LSR50 shore.

Combined frames are produced from:

copper-nickel alloys (Monel KA 500) and Aluminium alloy AG4MC (5086)

temple - cellulose acetate (Parade 209389)

and nose pads - silicone LSR50 shore.

Semi-rimless frames are made of:

copper-nickel alloys (Monel KA 500)

temple tips - cellulose acetate (Parade 209389)

and nose pads – silicone LSR50 shore and fixing thread is of nylon polymer Nylon.

Transportation demo-lenses are made of acryl Polycryl Clear.

The materials comply with requirements of biological safety.

5. LABELING

Optical frames made by the company ARCHITECTURES SAS are labeled on the temples. The right side bears the name of the manufacturer. The left side bears the label stating technical parameters: horizontal width of the lens to be set (a); width of the bridge (b); full length of the temple; code or name of the model; code of the color or coating.

FACE A FACE

PARIS

WOOW

6. PRE-OPERATION: optical frame, after the insertion of the lenses, should not lose the initial shape. Rims must be symmetrical. Chips, scratches, and gaps between the spectacle lenses and rims of the frame, as well as other defects of the spectacles' assembly are intolerable.

Spectacle lenses inserted in the frame should not be displaced and fall out of the frame under usual conditions of the spectacles' usage.

7. RULES OF STORAGE AND USE:

- Significant mechanical effort should not be applied to the frame;
- Frame should be kept in a case or in other packing which can protect the frame from mechanical damage;

Full average service life of the frame is not less than three (3) years.

Storage conditions: in warmed premises, at the ambient temperature from $+5^{\circ}\text{C}$ to $+50^{\circ}\text{C}$ and relative air humidity of 80%.

During the storage, effect of aggressive substances (acids or alkali) and organic solvents or their vapors should be avoided.

8. USAGE CONDITIONS

Optical frames can be used in all climatic zones at any time of a day.

9. PRECAUTIONS FOR USE

Using the optical frames, one should be cautious when engaging in sports and doing works suggesting sharp head movements, bending, jumps, and shocks.

10. POSSIBLE SIDE EFFECTS WHEN USING

No side effects related to the use of these optical frames have been found.

11. INFORMATION ON POSSIBLE EFFECT OF USE OF THE FRAMES ON THE ABILITY TO DRIVE VEHICLES OR OPERATE MECHANISMS

For drivers having vision problems, optical frames provide the possibility of driving transportation means and operating mechanisms (when optical frame are used as part of spectacles).

FACE A FACE
— PARIS —
WOOW

12. STERILITY

Not sterile.

13. MAINTENANCE AND REPAIR

Optical frames must be repaired in special laboratories which have a license for manufacturing, maintenance and repair of frames.

14. WARRANTY

Shelf life of the frames: for plastic and combined, semi-rimless frames- 5 years; for metal frames - 10 years.

The full service life of the frames is at least three (3) years, with the number of swings of temples not more than 30 000.

15. TRANSPORTATION

Transportation of optical frames can be carried out by all types of transport in the sheltered vehicles in accordance with rules of cargo transportation effective for the given type of transport.

16. UTILIZATION

Optical frames are subject to disposal as household wastes

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