Nano-Hybrid Flowable Composite

EN

DIRECT RESTORATIVES

Instructions for use

English

Definition

Inspiro Nano-Hybrid Flowable Composite is a light cured, radiopaque, highly filled homogenous Nano-Hybrid Flowable Composite filling material for restorations that meets the highest physico-mechanical and esthetic standards.

The system consists of 2 components:

Dentin "Body" (B) is provided in 7 intensity levels (Bi1 to Bi7) and Enamel "Skin" (S) in 3 shades: Skin Bleach (SB), Skin Neutral (SN) and Skin White (SW).

Composition

Barium dental glass
Nano-particles
Bis-GMA based resin matrix
Pigments
Additives
Catalyst

Technical data

Filler ratio: 65% by weight or 38% by volume. The range of inorganic filler particle size is from 0.02 to 3µm.

Indications

- Small to medium size class V restorations (cervical caries, root erosion, wedge-shaped defects)
- Small restorations of all types in non-functional areas
- Initial layer underneath class I and II restorations
- Blocking out of undercuts in the preparation of veneers, inlays & onlays
- Repair of bis-acryl temporaries

Recommended application

1. Select the shade using the inspiro Body & Skin Shade Guide.
2. Provide appropriate isolation. It is recommended to use a dental dam.
3. Prepare the cavity according to the general requirements for adhesive techniques.
4. Clean the cavity using water spray.
5. Dry the cavity using air spray.
6. Apply a pulp protection (e.g. calcium hydroxide), if necessary; then, only cover areas close to the pulp and apply a pressure-resistant cement.
7. Apply matrix / interproximal dental wedge if needed
8. Condition the tooth and apply the selected bonding agent according to manufacturer's instructions.
9. Apply inspiro Nano-Hybrid Flowable Composite in layers of max. 2mm.
10. Polymerize each layer for 20s using a light output intensity of at least 500mW/cm². With a light output intensity of 1100mW/cm², the individual layers can be cured in 10s. When using a metal matrix (e.g. OptraMatrix), also polymerize from the buccal or lingual/palatal sides after removing the matrix.
11. Finish the restoration using suitable finishing instruments or finegrit diamonds.
12. Check the occlusion.
13. Polish using silicone polishers as well as polishing discs and strips.

Essential notes

- Light curing units should have an output of 450nm and should be checked regularly. The curing light output intensity should be at least 500mW/cm². Place the curing light as close as possible to the restorative material.
- Do not remove the oxygen-inhibition layer, since this is needed to create a sufficient bond to the next layer.
- Follow the instructions for use for all materials used together with inspiro Nano-Hybrid Flowable Composite.

Contraindications

Application of inspiro Nano-Hybrid Flowable Composite is contraindicated if a dry working area or the recommended application technique is not possible, if the pulp is exposed, or if there are any known allergies to any components in the inspiro Nano-Hybrid Flowable Composite.

Side effects

Side effects are not known to date. In individual cases, hypersensitivity to components of the material cannot be excluded. Avoid contact of the paste with the skin. If accidental contact occurs, wash immediately using soap and water. In case of contact with eyes, rinse immediately using plenty of water, and consult a physician.

Interactions

Avoid materials containing eugenol, moisture or oily air, since they can inhibit polymerization at the contact area. The use of cationic mouthwashes, plaque indicators or chlorohexidine may lead to discolorations.

Classification

Inspiro Nano-Hybrid Flowable Composite fulfills the requirements of ISO 4049:2000. The specified curing depth of 2mm (20s) for inspiro Nano-Hybrid Flowable Composite is prepared according to ISO 4049:2000. The specified layer thickness of 2mm and light curing time of 20s for each layer is recommended for all cases.

Storage, use and transportation

Products can be used, transported and stored at normal room temperature (15-25°C / 59-77°F). Please close syringe/ composite tip immediately after each use to prevent premature setting from ambient light.

Shelf life and marking

Expiration date and the lot number are marked on the syringes as well as the packaging. Do not use after the date of expiration.

Disposal considerations

After the use the device has to be disposed as a contaminated healthcare waste.

The disposal of the product does not pose any danger, however, the waste must be disposed of according to national or regional regulations.

Recycling

The garbage after processing has to be utilized as special waste according to the Directives EC 91/156/CEE on waste, 91/689/CEE about dangerous wastes, both 94/62/CE about packing and packing waste and according to the national legislation on this matter.

Important

The product is guaranteed to be free from defects at the time of shipping. All of our products are designed and manufactured to meet the highest quality standards. We cannot accept any liability for failure of products due to misuse or application which is not in accordance with the present instruction for use.

To ensure for optimal clinical results, we recommend that all inspiro components in this system are used in conjunction with each other.

Note

Only to be supplied to dentists and dental laboratories or upon their request. Keep out of the reach of children! For dental use only!

Date of issue

June 2013

Effect Shades

EN

DIRECT RESTORATIVES

Instructions for use

English

Definition

Effect Shade is a light-curing, thin-flowing composite which has been specifically developed for the characterization of composites. Due to the special pigmentation, Effect Shades allow color individualization of restorations.

Composition

Inorganic fillers in a matrix of dental resins. Filler content: 64% by weight = 38% by volume. The variation width of the inorganic filler particles is between 0.02-3µm.

Indications

Light-curing, thin-flowing composite for the characterization of composite restorations.

Recommended use

1. Select colors needed for the desired characterizations.
2. Place a small amount of the selected color on a mixing palette.
3. Apply the material using a thin brush or any other appropriate instrument in a layer thickness of ≤ 1mm.
4. Light-cure Effect Shades using commercially available light-curing units (e.g. LED polymerization device) for at least 20s.
5. As a general rule, Effect Shades should be covered with a layer of restorative composite, flowable composite or a restoration such as a veneer. If this is not done, the effect shade might be worn away when the restoration is polished or as well when submitted to brushing and function.
6. Effect Shades may only be mixed with one another (to obtain new color shades); they are not recommended to be mixed with other products.

Please note

- Light-curing units should have a light intensity of at least 500mW/cm², emit at 450nm and their output should be regularly checked. The light should be placed as close as possible to the material.
- To prevent premature polymerization due to ambient light, the syringes should be closed immediately after use and the prepared materials on the mixing palette should be covered with a black lid.
- It is important that the surface of the restoration is not contaminated before applying Effect Shades.
- Ensure that the covers of the color stains are not mixed up with one another.

Contraindications

Do not use the material in the case of a known allergy to one of the components.

Side effects

There are no known side effects to date. Isolated cases of hyper-sensitivity to components of the material cannot be excluded. Avoid skin contact with the paste. If there is accidental contact, wash with soap and water. In the event of eye contact, immediately rinse with plenty of water and seek medical attention.

Interactions

Avoid materials containing eugenol, moisture or oily air, since they can inhibit polymerization at the contact area. The use of cationic mouthwashes, plaque indicators or chlorohexidine may lead to discolorations.

Storage, use and transportation

Products can be used, transported and stored at normal room temperature (15-25°C / 59-77°F).

Please close syringe/ composite tip immediately after each use to prevent premature setting from ambient light.

Shelf life and marking

Expiration date and the lot number are marked on the syringes as well as the packaging. Do not use after the date of expiration.

Disposal considerations

After the use the device has to be disposed as a contaminated healthcare waste.

The disposal of the product does not pose any danger, however, the waste must be disposed of according to national or regional regulations.

Recycling

The garbage after processing has to be utilized as special waste according to the Directives EC 91/156/CEE on waste, 91/689/CEE about dangerous wastes, both 94/62/CE about packing and packing waste and according to the national legislation on this matter.

Important

The product is guaranteed to be free from defects at the time of shipping. All of our products are designed and manufactured to meet the highest quality standards. We cannot accept any liability for failure of products due to misuse or application which is not in accordance with the present instruction for use.

To ensure for optimal clinical results, we recommend that all inspirocomponents in this system are used in conjunction with each other.

Note

Only to be supplied to dentists and dental laboratories or upon their request. Keep out of the reach of children! For dental use only!

Date of issue

June 2013

Nano-Hybrid Restorative Composite

EN

DIRECT RESTORATIVES

Instructions for use

English

Definition

Inspiro Nano-Hybrid Composite is a light cured, radiopaque, highly filled homogenous nano-hybrid composite filling material for restorations that meets the highest physico-mechanical and esthetic standards

The system consists of 2 components:

Dentin „Body“ (B) in 6 different intensity levels (shades: Bi0 to Bi5) and Enamel „Skin“ (S) in 5 shades: Skin Bleached (SB), Skin White (SW), Skin Neutral (SN), Skin Transparent (ST), Skin Ivory (SI).

Composition

Barium dental glass
Nano-particles
Bis-GMA based resin matrix
Pigments
Additives
Catalyst

Technical data

Filler ratio: 82% by weight or 65% by volume. The range of inorganic filler particle size is from 0.02 to 3µm.

Indications

Direct applications:

Filling of cavity classes I, II, III, IV, V
All other types of direct restorations
(e.g.: form corrections, diastema closure, etc...)

Indirect applications:

Inlays
Onlays
Veneers
Cementation of edelweiss DIRECT VENEAR & OCCLUSION VD

Dentin „Body“ (B) is specially indicated for:

Reconstruction of natural dentin

Enamel „Skin“ (S) is specially indicated for:

Reconstruction of natural enamel

Recommended application

1. Select the shade using the inspiro Body & Skin Shade Guide.
2. Provide appropriate isolation. It is recommended to use a dental dam.
3. Prepare the cavity according to the general requirements for adhesive techniques.
4. Clean the cavity using water spray.
5. Dry the cavity using air spray.
6. Apply a pulp protection (e.g. calcium hydroxide), if necessary; then, only cover areas close to the pulp and apply a pressure-resistant cement.
7. Apply matrix / interproximal dental wedge if needed
8. Condition the tooth and apply the selected bonding agent according to manufacturer's instructions.

9. Apply inspiro Nano-Hybrid Restorative Composite in layers of max. 2mm.
10. Polymerize each layer for 20s using a light output intensity of at least 500mW/cm². With a light output intensity of 1100mW/cm², the individual layers can be cured in 10s. When using a metal matrix (e.g. OptraMatrix), also polymerize from the buccal or lingual/palatal sides after removing the matrix.
11. Finish the restoration using suitable finishing instruments or finegrit diamonds.
12. Check the occlusion.
13. Polish using silicone polishers as well as polishing discs and strips.

Essential notes

- Light curing units should have an output of 450nm and should be checked regularly. The curing light output intensity should be at least 500mW/cm². Place the curing light as close as possible to the restorative material.
- Do not remove the oxygen-inhibition layer, since this is needed to create a sufficient bond to the next layer.
- Follow the instructions for use for all materials used together with inspiro Nano-Hybrid Composite.

Contraindications

Application of inspiro Nano-Hybrid Composite is contraindicated if a dry working area or the recommended application technique is not possible, if the pulp is exposed, or if there are any known allergies to any components in the inspiro Nano-Hybrid Composite.

Side effects

Side effects are not known to date. In individual cases, hypersensitivity to components of the material cannot be excluded. Avoid contact of the paste with the skin. If accidental contact occurs, wash immediately using soap and water. In case of contact with eyes, rinse immediately using plenty of water, and consult a physician.

Interactions

Avoid materials containing eugenol, moisture or oily air, since they can inhibit polymerization at the contact area. The use of cationic mouthwashes, plaque indicators or chlorhexidine may lead to discolorations.

Classification

Inspiro Nano-Hybrid Composite fulfills the requirements of ISO 4049:2000. The specified curing depth of 2mm (20s) for inspiro Nano-Hybrid Composite is prepared according to ISO 4049:2000. The specified layer thickness of 2mm and light curing time of 20s for each layer is recommended for all cases.

Storage, use and transportation

Products can be used, transported and stored at normal room temperature (15-25°C / 59-77°F). Please close syringe/ composite tip immediately after each use to prevent premature setting from ambient light.

Shelf life and marking

Expiration date and the lot number are marked on the syringes as well as the packaging. Do not use after the date of expiration.

Disposal considerations

After the use the device has to be disposed as a contaminated healthcare waste. The disposal of the product does not pose any danger, however, the waste must be disposed of according to national or regional regulations.

Recycling

The garbage after processing has to be utilized as special waste according to the Directives EC 91/156/CEE on waste, 91/689/CEE about dangerous wastes, both 94/62/CE about packing and packing waste and according to the national legislation on this matter.

Important

The product is guaranteed to be free from defects at the time of shipping. All of our products are designed and manufactured to meet the highest quality standards. We cannot accept any liability for failure of products due to misuse or application which is not in accordance with the present instruction for use.

To ensure for optimal clinical results, we recommend that all inspiro components in this system are used in conjunction with each other.

Note

Only to be supplied to dentists and dental laboratories or upon their request. Keep out of the reach of children! For dental use only!

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SECTION 1: Identification of the substance/mixture and of the company/undertaking

MSDS ID: Nano hybrid composite (REF 2054)
Nano hybrid flowable composite (REF 2060)
Effect shades (REF 2096)

Company /undertaking identification

Company name: edelweiss DR AG
Street: Unter Altstadt 28, Mercandor
Place: 6300 Zug, CH
Telephone: +41 79 570 20 50
Internet: www.edelweisssdr.com e-mail: info@edelweisssdr.com

SECTION 2: Hazard identification

Classification of the substance or mixture

Indications of danger : Irritant
R-phrases:
Irritating to eyes, respiratory system and skin.
May cause sensitization by skin contact.
Harmful to aquatic organisms, may cause long-term adverse effects in the aquatic environment.

GHS classification

Hazard Statements:
May cause respiratory irritation.
Causes serious eye irritation.
Causes skin irritation.
May cause an allergic skin reaction.
Harmful to aquatic life with long lasting effects.

Label elements

Signal word: Warning
Pictograms: GHS07

Hazard statements

H335 May cause respiratory irritation.
H319 Causes serious eye irritation.
H315 Causes skin irritation.
H317 May cause an allergic skin reaction.
H412 Harmful to aquatic life with long lasting effects.

Precautionary statements

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P302+P352 IF ON SKIN: Wash with plenty of soap and water.
P280 Wear protective gloves/protective clothing/eye protection/face protection.
P273 Avoid release to the environment.

Safety Data Sheet

According to Regulation (EC) No. 1907/2006

Product Code

Nano hybrid composite (REF 2054)
Nano hybrid flowable composite (REF 2060)
Effect shades (REF 2096)

SECTION 3: Composition/information on ingredients

Mixtures

Chemical characterization

chemical characterization (preparation): Acrylate.-resin.

SECTION 4: First aid measures

Description of first aid measures

General information

Seek medical attention if problems persist. Change contaminated clothing.

After inhalation

Move victim to fresh air. Put victim at rest and keep warm.

After contact with skin

After contact with skin, wash immediately with: Water and soap.

After contact with eyes

In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

After ingestion

Immediately get medical attention. Do not induce vomiting.

SECTION 5: Firefighting measures

Extinguishing media

Suitable extinguishing media

Water fog. Extinguishing powder. Sand. Foam. Carbon dioxide (CO₂).

Extinguishing media which must not be used for safety reasons

High power water jet.

Special hazards arising from the substance or mixture

CO_x, NO_x

SECTION 6: Accidental release measure

Personal precautions, protective equipment and emergency procedures

Provide adequate ventilation.

Environmental precautions

Do not empty into drains or the aquatic environment.

Methods and material for containment and cleaning up

Collect mechanically. Absorb with liquid-binding material (e.g. sand, diatomaceous earth, acid- or universal binding agents).

SECTION 7: Handling and storage

Precautions for safe handling

Advice on safe handling

Keep container tightly closed. Wear suitable protective clothing and gloves.

Safety Data Sheet

According to Regulation (EC) No. 1907/2006

edelweiss
DIRECT RESTORATIVES

Product Code

Nano hybrid composite (REF 2054)
Nano hybrid flowable composite (REF 2060)
Effect shades (REF 2096)

Conditions for safe storage, including any incompatibilities

Requirements for storage rooms and vessels

Keep container tightly closed in a cool, well-ventilated place.
Store only in original container.

SECTION 8: Exposure controls/personal protection

Control parameters

Exposure controls

Occupational exposure controls

Ensure adequate ventilation of the storage area.

Protective and hygiene measures

When using do not eat or drink.

Respiratory protection

The following must be prevented: inhalation.

Hand protection

Tested protective gloves are to be worn: Suitable material: NBR (Nitrile rubber).

Eye protection

Tightly sealed safety glasses.

SECTION 9: Physical and chemical properties

Information on basic physical and chemical properties

Physical state: Paste
Colour: yellow
Odour: like: ester

pH-Value (at 20 °C):

Test method
not applicable

Changes in the physical state

Boiling point:	> 200 °C
point of decomposition:	not determined
Flash point:	> 100 °C
Density:	1,8 g/cm ³
Vapour density: (at 20 °C)	> 1

SECTION 10: Stability and reactivity

Conditions to avoid

Light. heat.

Incompatible materials

Keep away from strong acids, leachates, heavy metal salts and reducing materials.

Hazardous decomposition products

Can be released in case of fire: Gas / vapours, irritant. (Acrylate., pungent)

Further information

Substances sensitive to light.

Safety Data Sheet

According to Regulation (EC) No. 1907/2006

Product Code

Nano hybrid composite (REF 2054)
Nano hybrid flowable composite (REF 2060)
Effect shades (REF 2096)

SECTION 11: Toxicological information

Information on toxicological effects

Acute toxicity

LD50: Rat. 2.000 mg/kg

Specific effects in experiment on an animal

Frequently or prolonged contact with skin may cause dermal irritation.

Irritation and corrosivity

Irritant effect on the eye: Frequently or prolonged contact with skin may cause dermal irritation.

Sensitising effects

May cause sensitization by skin contact.

Additional information on tests

Contains Methacrylic esters.: May produce an allergic reaction.

SECTION 12: Ecological information

Toxicity

Acute fish toxicity LC50: OECD 203 / 96h: >10 mg/l

Persistence and degradability

Preparation not tested.

Further information

Do not empty into drains or the aquatic environment. Leakage into the environment must be prevented.

SECTION 13: Disposal considerations

Waste treatment methods

Advice on disposal

Can be burnt together with household waste in compliance with official regulations in contact with approved waste disposal companies and with authorities in charge. Paste: Carry out a burning of hazardous waste according to official regulations.

Waste disposal number of waste from residues/unused products

180106 WASTES FROM HUMAN OR ANIMAL HEALTH CARE AND/OR RELATED RESEARCH (except kitchen and restaurant wastes not arising from immediate health care); wastes from natal care, diagnosis, treatment or prevention of disease in humans; chemicals consisting of or containing dangerous substances
Classified as hazardous waste.

SECTION 14: Transport information

Other applicable information

Not a hazardous material with respect to these transportation regulations.

SECTION 15: Regulatory Information

Safety, health and environmental regulations/legislation specific for the substance or mixture

National regulatory information

Water contaminating class (D): 2 - water contaminating

Safety Data Sheet

According to Regulation (EC) No. 1907/2006

2/8

Product Code

Nano hybrid composite (REF 2054)
Nano hybrid flowable composite (REF 2060)
Effect shades (REF 2096)

SECTION 16: Other information

Full text of R-phrases referred to under sections 2 and 3

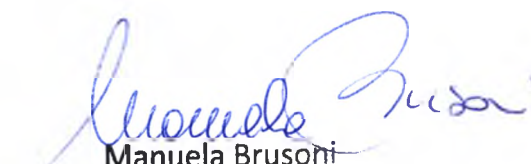
36/37/38	Irritating to eyes, respiratory system and skin.
43	May cause sensitization by skin contact.
52/53	Harmful to aquatic organisms, may cause long-term adverse effects in the aquatic environment.

Full text of H-Statements referred to under sections 2 and 3

H315	Causes skin irritation.
H317	May cause an allergic skin reaction.
H319	Causes serious eye irritation.
H335	May cause respiratory irritation.
H412	Harmful to aquatic life with long lasting effects.

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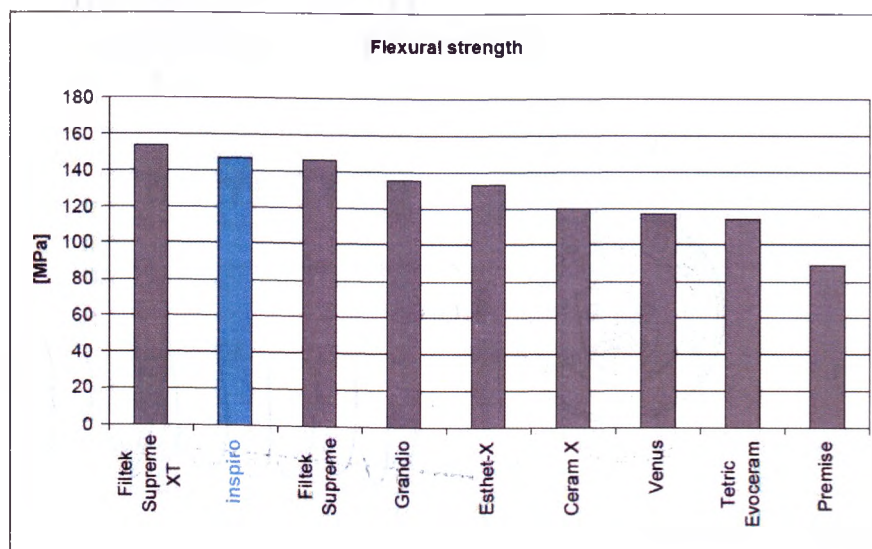
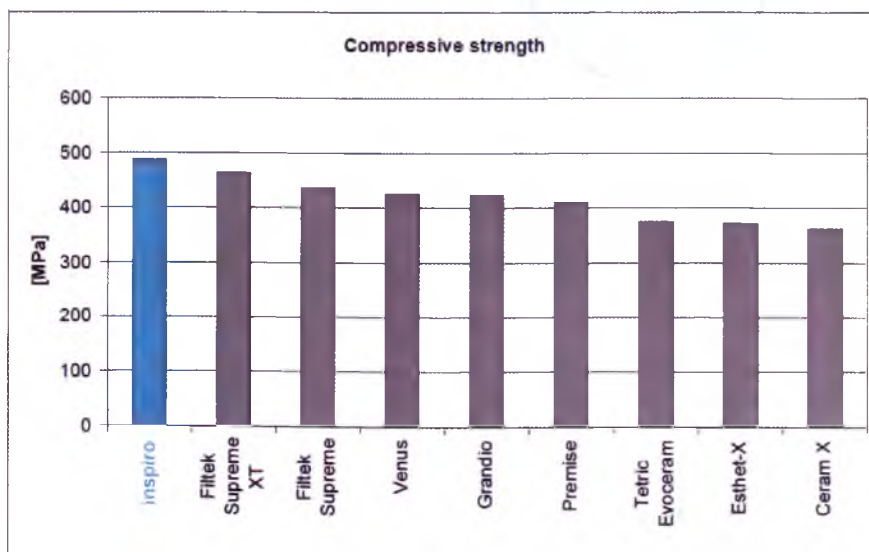
inspiro™ PHYSICAL PROPERTIES

Inspiro physical properties

There are different physical characteristics of interest which determine the material resistance to intra-oral stresses, the ability of the restoration to strengthen the remaining tooth structure, as well as the potential to transmit stresses due to material polymerization within surrounding structures.

1. Compressive and flexural strength are typical values which reflect the material's ability to resist functional stresses. Inspiro presents very high values for both properties and outperforms the majority of main competitors.

0.1

Manuela Brusoni
EDELWEISS DR AG
Unter Altstadt 28, Mercantour
6300 Zug, Switzerland

DIRECT RESTORATIVES

inspiro direct

I, Laurent BRECHBUHL, notary public in Geneva/Switzerland, hereby certify the signature affixed on the other side by Mrs Manuela Brusoni, who binds with his single signature Edelweiss DR AG, a share company in Zug/Switzerland.

Geneva, February 13, 2015

BRECHBUHL & RODRIGUEZ
NOTAIRES

38, route de Malagnou - Case postale

1211 GENEVE 17

Tél. +41 22 707 15 00 - Fax +41 22 735 00

E-mail notaires@brnotaires.ch

APOSTILLE

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1. Pays: Suisse

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M. L. Brechbuhl

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notaire.

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4

Attesté

16 FEV. 2015

6. le

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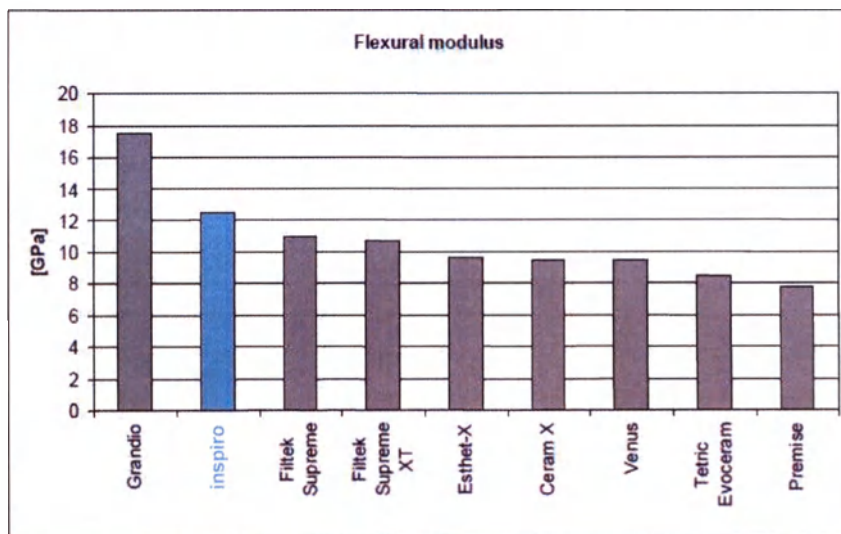
4678


10. Signature

Andrea GIL FERNANDEZ

2. Next to flexural strength, the flexural modulus is of paramount importance to provide reinforcement to the remaining tooth structure; then, a relatively high modulus is required. However, a very stiff material with particularly high modulus would unfortunately augment polymerization stresses as this later property is directly governed by linear/volumetric shrinkage and material's E-modulus. Inspiro provides here an optimal balance in-between the aforementioned properties.

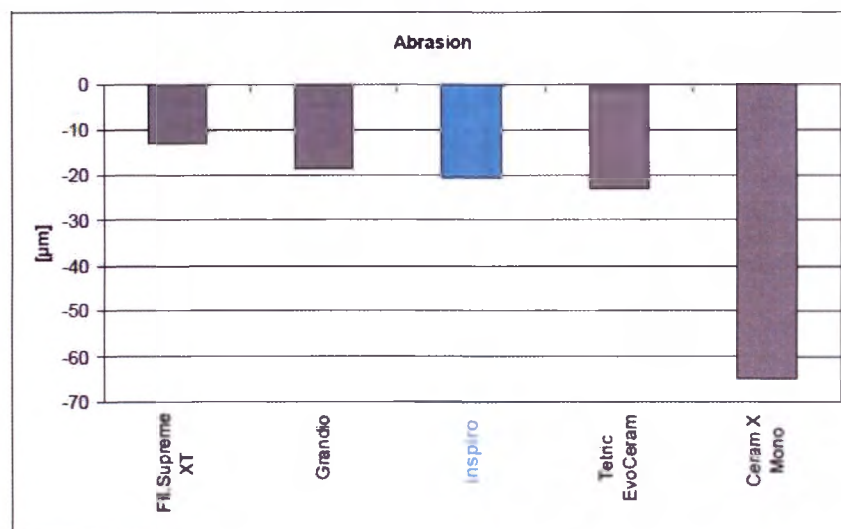
3. Abrasion is another important property for satisfactory long-term restoration performance, in posterior areas particularly. Due to the high filler level and the use of the homogeneously dispersed nano particles, inspiro has a very hard surface. Moreover, the restoration exhibits an optimal and stable, long-term gloss retention in comparison with standard hybrid or inhomogeneous nano-hybrid composites.

4. Polymerization shrinkage is a common, adverse event of all dental resin composite materials; related stresses have potential to impact the adhesive interface quality. To minimize this undesirable property, a low volumetric/linear shrinkage is needed; however, lowering polymerization stresses by reducing the E-modulus of the material is not desired as it would reduce the reinforcement and stabilization effect of the restoration on the residual tooth structure.

Inspiro exhibits a linear shrinkage similar to most leading brands in this product category (nano-hybrids) and significantly lower stresses than traditional micro-hybrids. However, Inspiro is not intended for a bulk-filling procedure and an appropriate layering/filling technique is recommended for large cavities, in particular for class I and II restorations.

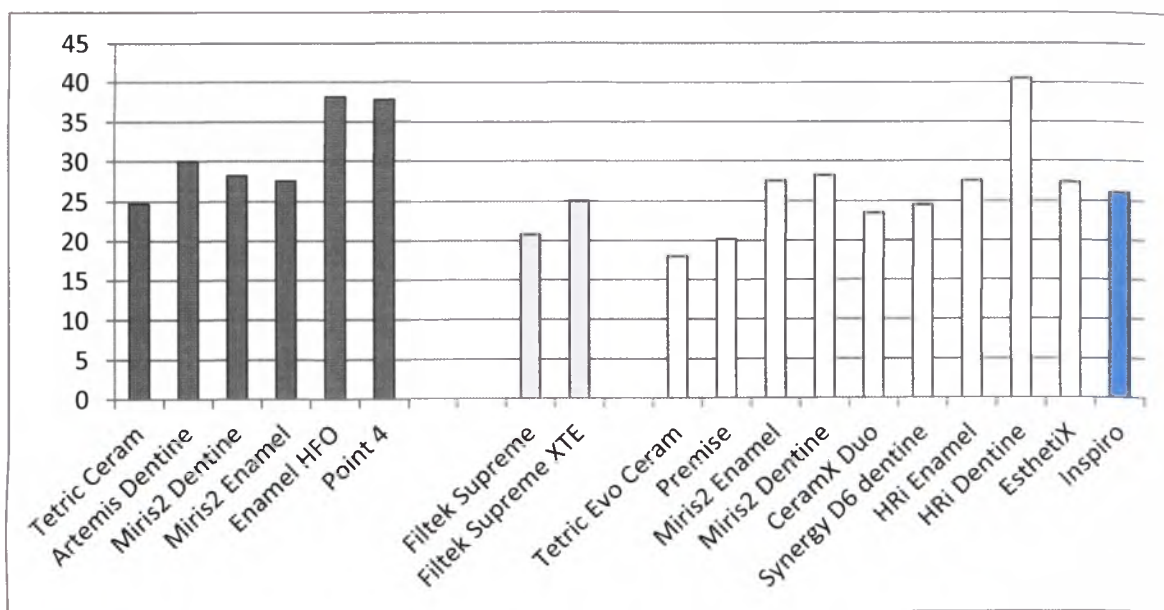LINEAR SHRINKAGE DISPLACEMENT (μm @ 3min)

Measurements (University of Geneva) according to:

1. Stavridakis MM, Lutz F, Johnston WM, Krejci I. Linear displacement and force induced by polymerization shrinkage of resin-based restorative materials. Am J Dent. 2003 Dec;16(6):431-8.
2. Stavridakis MM, Dietschi D, Krejci I. Polymerization shrinkage of flowable resin-based restorative materials. Oper Dent. 2005 Jan-Feb;30(1):118-28

Materials and Method

The materials tested were 5 resin-based hybrid composites (Tetric Ceram, Artemis, Miris, Enamel HFO, Point 4), 2 nano-filled composites (Filtek Supreme, Filtek Supreme XTE), and 8 nano-hybrid composites (Tetric Evo Ceram, Premise, Miris2, CeramX Duo, Synergy D6, Enamel HRI, Esthetix, Inspiro). Each property was measured for 180 seconds from the start of curing with the help of custom made devices in eight specimens for each material. Statistical evaluation of the data was performed with one-way ANOVA, Tukey's Studentized Range (HSD) Test ($P=0.05$) and simple linear regression.

Summary of inspiro physico-chemical characteristics

Type of Material Universal restorative material based on nano-technology.

Class of Material Light-cure composite based on dimethacrylates with fillers.

Relevant Standards Conforms to ISO 4049:2000 (Dentistry - Polymer-based filling, restorative and luting materials). Type 2, class 2, group 1.

Toxicological properties Minimal toxicological risk for completely cured material. Some patients may be sensitive to components.

Storage @ Room temperature, dry (15-25 °C / 59-77 °F).

Shelf-Life When stored properly until expiry date (see product labeling).

Typical Composition

Dental glass / silicone dioxide	82 [%]
Bis-GMA-based dental resins	17 [%]
Additives, pigments, catalysts	1 [%]
Inorganic filler particle size	0.02 - 3 [µm]
Volume of inorganic fillers	65 [%-vol.]

Working Properties

Working time (full operating light) ≥ 01:00 [minutes]

Recommended thickness of layers ≤ 2 [mm]

Recommended curing time (halogen lamp) ≥ 20 [seconds]

Depth of cure (after recommended curing time) ≥ 2 [mm]

Physical Properties

Compressive strength	480 [MPa]
Flexural strength	150 [MPa]
Radiopacity	230 [% AI]
Water sorption	25 [µg/mm³]
Water solubility	5 [µg/mm³]

LINEAR SHRINKAGE DISPLACEMENT

University of Geneva, according to the methodology of Stavridakis et al, 2003 & 2005.

The materials tested were 5 resin-based hybrid composites (Tetric Ceram, Artemis, Miris, Enamel HFO, Point 4), 2 nano-filled composites (Filtek Supreme, Filtek Supreme XTE), and 8 nano-hybrid composites (Tetric Evo Ceram, Premise, Miris2, CeramX Duo, Synergy D6, Enamel HRI, EsthetX, Inspiro). Each property was measured for 180 seconds from the start of curing with the help of custom made devices in eight specimens for each material. Statistical evaluation of the data was performed with one-way ANOVA, Tukey's Studentized Range (HSD) Test ($P=0.05$) and simple linear regression.

Color Stability of Recent Composite Resins

Olivier Duc^a, Enrico Di Bella^b, Ivo Krejci^a, Stefano Ardu^a

^aDivision of Cariology and Endodontology, Section of Dental Medicine, Faculty of Medicine, University of Geneva, Geneva, Switzerland

^bDepartment of Economics and Quantitative methods, University of Genoa, Genoa, Italy

Objectives: To evaluate the color stability of 8 recently developed resin-composites when exposed to various staining agents.

Methods: Six disc shaped specimens made out of 8 resin-composite materials (Tab. 1) were immersed in artificial saliva, coffee, coke, tea, orange juice and red wine. Initial color (T0) of the 288 specimens was assessed by a calibrated reflectance spectrophotometer (SpectroShade, MHT) (Fig. 1) over a black as well as a white background. All specimens were kept in an incubator (INP-500, Memmert GmbH) at 37 ° C for 4 weeks. Colorant solutions were changed every 7 days to avoid bacteria or yeast contamination.

After 4 weeks of storage in the colorants (T1), spectrophotometric measurements were repeated and both, ΔE and ΔE_{00} were calculated.

Table 1: list of materials evaluated

Resin composite	Manufacturer	Group N°
Estelite posterior A2	Tokuyama Dental	1
ELS A2	Saremco	2
Saremco microhybride A2	Saremco	3
Filtek Supreme XTE A2 enamel	3M Espe	4
Inspiro skin neutral	Edelweiss	5
Venus Diamond A2	Heraeus Kulzer	6
Miris 2 enamel neutral regular	Coltène-Whaledent	7
Filtek Silorane A2	3M Espe	8


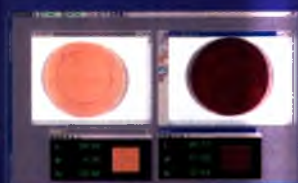
Figure 1

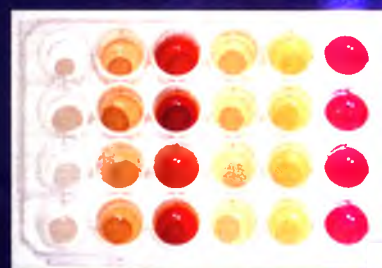Results: When analysed over a white background, median ΔE values varied from 0.0 (Filtek Silorane) to 19.1 (Filtek Supreme XTE)

When analysed over a black background median ΔE varied from 1.5 (Filtek Silorane) to 19.6 (Filtek Supreme XTE)



When analysed over a white background median ΔE_{00} varied from 7.0 (Filtek Silorane) to 21.6 (Filtek Supreme XTE)

When analysed over a black background median ΔE_{00} varied from 4.4 (Filtek Silorane) to 20.8 (Filtek Supreme XTE)

Significant differences were detected when comparing values at T0 and T1 and when comparing median ΔE and ΔE_{00} values between materials independent on the background (Mann-Whitney, Bonferroni and Fisher's LSD post hoc tests).

The correlation between ΔE and ΔE_{00} over a white background was 0.9932 while it was 0.9873 over a black background

	White B.	Black B.	White B.	White B.
	Median ΔE	Median ΔE	Median ΔE_{00}	Median ΔE_{00}
	T1-T0	T1-T0	T1-T0	T1-T0
1	12.5	7.0	12.9	9.5
2	14.3	6.1	16.5	13.3
3	11.2	6.5	14.0	9.9
4	19.1	19.6	21.6	20.8
5	7.4	5.9	20.5	13.7
6	14.9	11.2	19.6	12.2
7	14.9	6.2	15.2	10.4
8	0	1.5	7.0	4.4

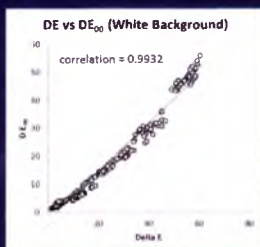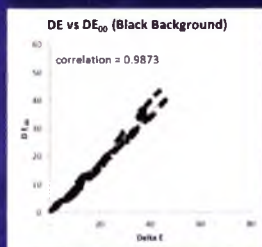Groupings according to the Fisher's LSD test applied on ΔE_{00} values (Groups with the same letter are not significantly different (p<0.05))

Group	1	2	3	4	5	6	7	8
White B.	A	B	C	D	E	F	G	H
Black B.	A	B	C	D	E	F	G	H

Groupings according to the Fisher's LSD test applied on ΔE values (Groups with the same letter are not significantly different (p<0.05))

Group	1	2	3	4	5	6	7	8
White B.	A	B	C	D	E	F	G	H
Black B.	A	B	C	D	E	F	G	H


Conclusions: Resin composite materials had a significantly ($p < 0.05$) different staining behaviour depending on their chemical composition and manufacturing technology. Under the conditions of the present study, there was no significant difference between ΔE and ΔE_{00} . The correlation coefficients between ΔE and ΔE_{00} were very high for both, black and white background.

REF#	Product description	Package content
INSPIRO TOOL BOXES		
3010	inspiro Homogeneous Nano-Hybrid Composite Master Kit Compule	90 x Dentin 0,3 g Compule / 80 x Enamel 0,3 g Compule / 7 x Dentin Flowable 1,5 g Syringe / 3 x Enamel Flowable 1,5 g Syringe / 5 x Effect Shades 1,5 g Syringe / 20 x Application Tips / Shade Guide
3020	inspiro Homogeneous Nano-Hybrid Composite Master Kit Syringe	6 x Dentin 3 g Syringe / 5 x 3 g Enamel Syringe / 7 x Dentin Flowable 1,5 g Syringe / 3 x Enamel Flowable 1,5 g Syringe / 5 x Effect Shades 1,5 g Syringe / 20 x Application Tip / Shade Guide
3030	inspiro Homogeneous Nano-Hybrid Composite Intro Kit Compule	70 x Dentin 0,3g Compule, 60 x Enamel 0,3 g Compule, 3 x Dentin Flowable 1,5 g Syringe, 2 x Enamel Flowable 1,5 g Syringe, 3 x Effect Shades 1,5 g Syringe, 20 x Application Tips, Shade Guide
3040	inspiro Homogeneous Nano-Hybrid Composite Intro Kit Syringe	4 x Dentin 3 g Syringe, 3 x 3 g Enamel Syringe, 3 x Dentin Flowable 1,5 g Syringe, 2 x Enamel Flowable 1,5 g Syringe, 3 x Effect Shades 1,5 g Syringe, 20 x Application Tip, Shade Guide
3050	inspiro Homogeneous Nano-Hybrid Composite Mini Kit Anterior Esthetics, Syringes	3 x Dentin 3 g Syringe, 2 x Enamel 3 g Syringe, 2 x Dentin Flowable 1,5 g Syringe, 1 x Enamel Flowable 1,5 g Syringe, 10 x Application Tip, Shade Guide
3055	inspiro Homogeneous Nano-Hybrid Composite General Mini Kit, Syringes	3 x Dentin 3 g Syringe, 2 x Enamel 3 g Syringe, 1 x Dentin Flowable 1,5 g Syringe, 1 x Enamel Flowable 1,5 g Syringe, 1 x Effect Shade 1,5 g Syringe, 10 x Application Tip, Shade Guide
HOMOGENEOUS NANO-HYBRID COMPOSITE DENTIN COMPULES REFILLS		
3200	inspiro Homogeneous Nano-Hybrid Composite Body i0	10 x 0,3 g Compule
3210	inspiro Homogeneous Nano-Hybrid Composite Body i1	10 x 0,3 g Compule
3215	inspiro Homogeneous Nano-Hybrid Composite Body i1	20 x 0,3 g Compule
3220	inspiro Homogeneous Nano-Hybrid Composite Body i2	10 x 0,3 g Compule
3225	inspiro Homogeneous Nano-Hybrid Composite Body i2	20 x 0,3 g Compule
3230	inspiro Homogeneous Nano-Hybrid Composite Body i3	10 x 0,3 g Compule
3235	inspiro Homogeneous Nano-Hybrid Composite Body i3	20 x 0,3 g Compule
3240	inspiro Homogeneous Nano-Hybrid Composite Body i4	10 x 0,3 g Compule
3245	inspiro Homogeneous Nano-Hybrid Composite Body i4	20 x 0,3 g Compule
3250	inspiro Homogeneous Nano-Hybrid Composite Body i5	10 x 0,3 g Compule
HOMOGENEOUS NANO-HYBRID COMPOSITE ENAMEL COMPULES REFILLS		
3300	inspiro Homogeneous Nano-Hybrid Composite Skin Bleach	10 x 0,3 g Compule
3305	inspiro Homogeneous Nano-Hybrid Composite Skin Bleach	20 x 0,3 g Compule
3310	inspiro Homogeneous Nano-Hybrid Composite Skin White	10 x 0,3 g Compule
3315	inspiro Homogeneous Nano-Hybrid Composite Skin White	20 x 0,3 g Compule
3320	inspiro Homogeneous Nano-Hybrid Composite Skin Neutral	10 x 0,3 g Compule
3325	inspiro Homogeneous Nano-Hybrid Composite Skin Neutral	20 x 0,3 g Compule
3330	inspiro Homogeneous Nano-Hybrid Composite Skin Transparent	10 x 0,3 g Compule
3340	inspiro Homogeneous Nano-Hybrid Composite Skin Ivory	10 x 0,3 g Compule
HOMOGENEOUS NANO-HYBRID COMPOSITE DENTIN SYRINGES REFILLS		
3400	inspiro Homogeneous Nano-Hybrid Composite Body i0	1 x 3 g Syringe
3410	inspiro Homogeneous Nano-Hybrid Composite Body i1	1 x 3 g Syringe
3420	inspiro Homogeneous Nano-Hybrid Composite Body i2	1 x 3 g Syringe
3430	inspiro Homogeneous Nano-Hybrid Composite Body i3	1 x 3 g Syringe
3440	inspiro Homogeneous Nano-Hybrid Composite Body i4	1 x 3 g Syringe
3450	inspiro Homogeneous Nano-Hybrid Composite Body i5	1 x 3 g Syringe
HOMOGENEOUS NANO-HYBRID COMPOSITE ENAMEL SYRINGES REFILLS		
3500	inspiro Homogeneous Nano-Hybrid Composite Skin Bleach	1 x 3 g Syringe
3510	inspiro Homogeneous Nano-Hybrid Composite Skin White	1 x 3 g Syringe
3520	inspiro Homogeneous Nano-Hybrid Composite Skin Neutral	1 x 3 g Syringe
3530	inspiro Homogeneous Nano-Hybrid Composite Skin Transparent	1 x 3 g Syringe
3540	inspiro Homogeneous Nano-Hybrid Composite Skin Ivory	1 x 3 g Syringe
HOMOGENEOUS NANO-HYBRID COMPOSITE DENTIN FLOWABLE REFILLS		
3610	inspiro Homogeneous Nano-Hybrid Flowable Body i1	1 x 1,5 g Syringe
3620	inspiro Homogeneous Nano-Hybrid Flowable Body i2	1 x 1,5 g Syringe
3630	inspiro Homogeneous Nano-Hybrid Flowable Body i3	1 x 1,5 g Syringe
3640	inspiro Homogeneous Nano-Hybrid Flowable Body i4	1 x 1,5 g Syringe
3650	inspiro Homogeneous Nano-Hybrid Flowable Body i5	1 x 1,5 g Syringe
3660	inspiro Homogeneous Nano-Hybrid Flowable Body i6	1 x 1,5 g Syringe
3670	inspiro Homogeneous Nano-Hybrid Flowable Body i7	1 x 1,5 g Syringe
HOMOGENEOUS NANO-HYBRID COMPOSITE ENAMEL FLOWABLE REFILLS		
3700	inspiro Homogeneous Nano-Hybrid Flowable Skin Bleach	1 x 1,5 g Syringe
3710	inspiro Homogeneous Nano-Hybrid Flowable Skin White	1 x 1,5 g Syringe
3720	inspiro Homogeneous Nano-Hybrid Flowable Skin Neutral	1 x 1,5 g Syringe
HOMOGENEOUS NANO-HYBRID EFFECT SHADES REFILLS		
3800	inspiro Homogeneous Nano-Hybrid Effect Shade Azur	1 x 1,5 g Syringe
3810	inspiro Homogeneous Nano-Hybrid Effect Shade Ice	1 x 1,5 g Syringe
3820	inspiro Homogeneous Nano-Hybrid Effect Shade Opaque	1 x 1,5 g Syringe
3830	inspiro Homogeneous Nano-Hybrid Effect Shade Chroma	1 x 1,5 g Syringe
3840	inspiro Homogeneous Nano-Hybrid Effect Shade Fissure	1 x 1,5 g Syringe
ACCESSORIES		
3900	inspiro Shade Guide	1 piece
3910	inspiro Application Tips	20 pieces

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inspir^o
direct

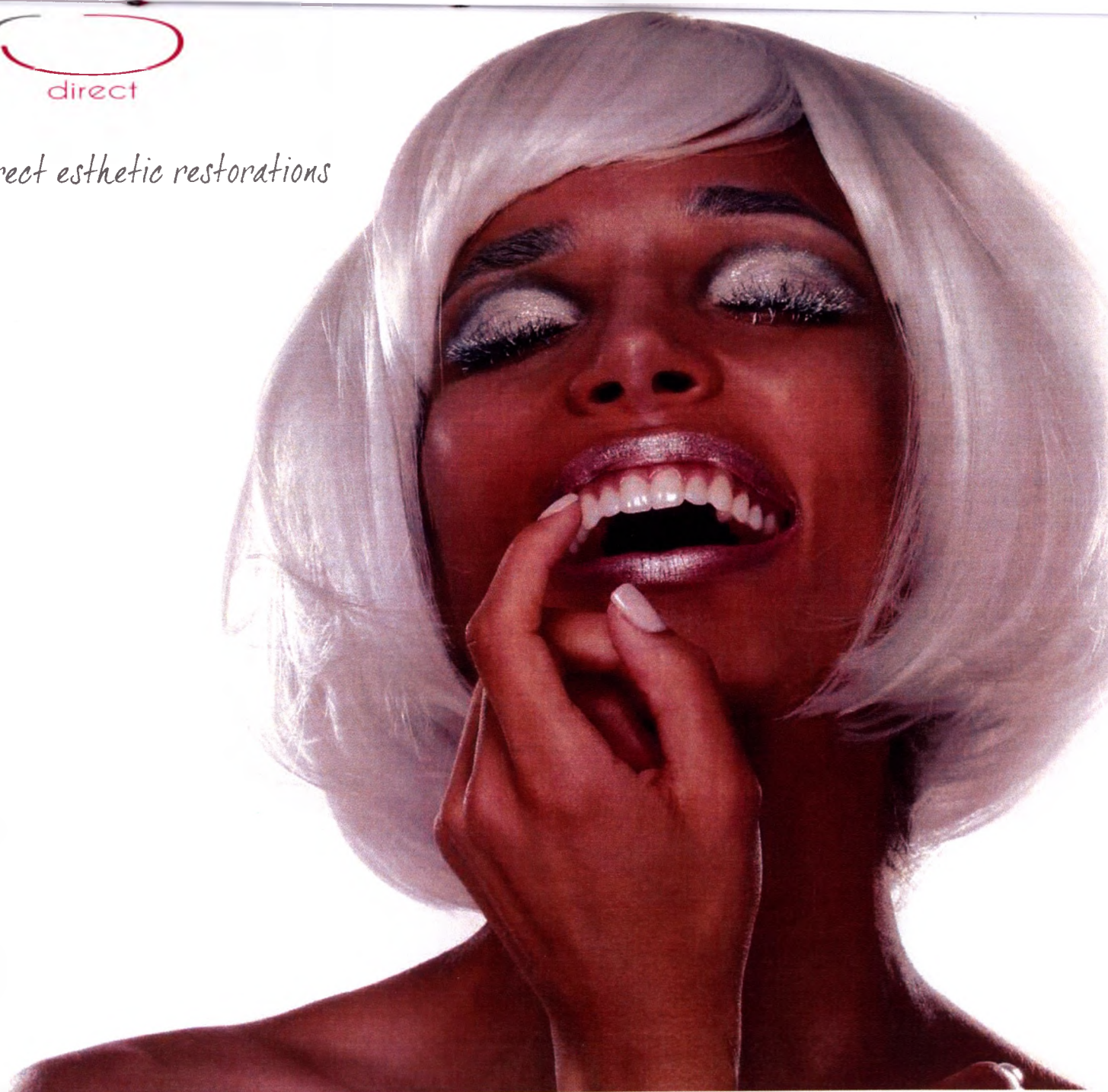... the "Swiss haute couture" in direct esthetic restorations

DIRECT RESTORATIVES

*" Perfection consists not in doing
extraordinary things, but in doing
ordinary things extraordinarily well "*

Angelique Arnauld

DIRECT RESTORATIVES

inspiro
direct

... the "Swiss haute couture" in direct esthetic restorations

EdelweissDR is introducing a new level in direct esthetic restorations with its **inspiro** direct system, developed under the guidance of Dr. Didier DIETSCHI. This material expands the possibilities in free-hand bonding techniques following the well established "NATURA" LAYERING CONCEPT leading to uncompromised esthetic and functional results thanks to the new, improved homogeneous nano-hybrid technology combining surface smoothness and mechanical strength...

Try and convince yourself!

DIRECT RESTORATIVES

Natural Layering Concept

The concept was introduced in 1995 by Prof. Dr. D. Dietsch and published for the first time in 1997. It is based on the idea of approaching nature as closely as possible, using a simplified and more reliable layering technique with two layers only (dentin & enamel) to perfectly mimic the tooth structure and appearance. **inspiro** composite system represents the latest development in the application of the Natural Layering Concept, with improved optical properties and a more comprehensive, logical choice of color and material consistencies to obtain restorations of unsurpassed esthetic integration and quality.

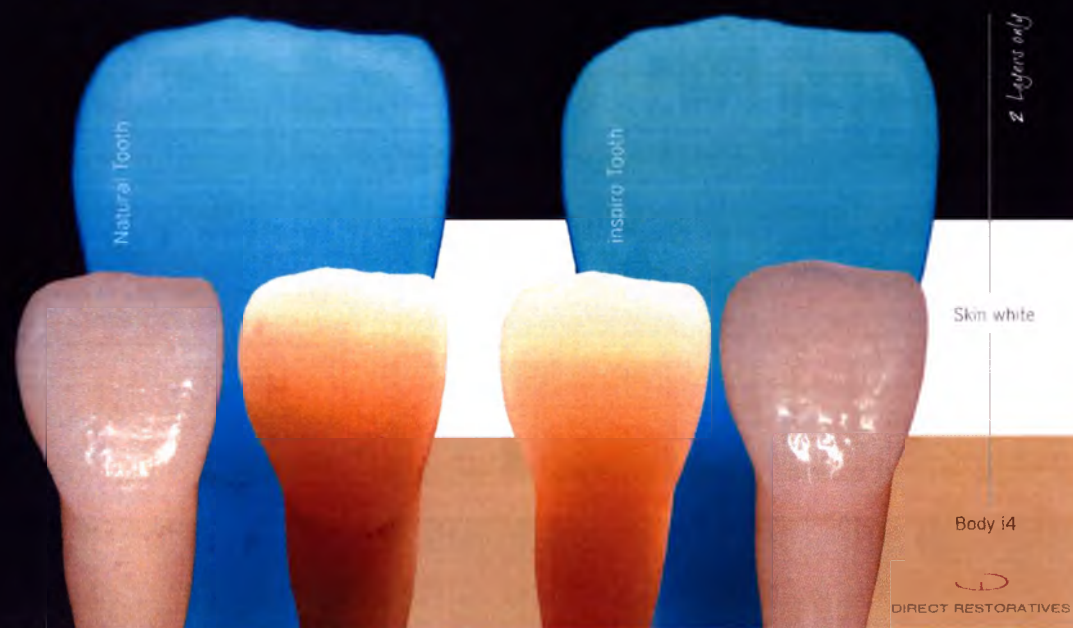
A COMPREHENSIVE ASSORTMENT OF DENTIN & ENAMEL SHADES FOR A 2-LAYER APPROACH

8 intensity levels in **BODY SHADES** for covering all variations in dentin chroma. The system offers 6 chroma levels for the restorative material and 7 chroma levels for the flowable material. Intensity 0 was developed for bleached teeth and masses with chroma intensity extending beyond the VITA color range (intensity 5 to 7) for optimal integration of class V restorations on dentin mamelons. This comprehensive choice of body shades will allow the clinician to cover all dentin chroma variations found in nature. All body shades exhibit the same color (true) and a similar opacity and fluorescence, greatly simplifying shade selection, as compared to other systems.

5 tints and transparency levels in **SKIN SHADES** for mimicking all natural enamel variations, which extend esthetic possibilities and system reliability. BLEACH and WHITE skin shades provide the brightness and lightness needed for young patients and rejuvenated smiles; skin NEUTRAL is a very common enamel tint to be used for adult patients, while skin HIGH and TRANSPARENT complete the enamel shades for mature teeth which require more depth and warm tints. 3 out of the 5 restorative skin shades are also available in a **flowable consistency**.

...layering does not need to be complicated to be highly esthetic!

DIRECT RESTORATIVES

inspiro comprehensive color range & shade guide

The **inspiro** direct shade guide allows a precise and reliable color evaluation. It has been developed as an integral tool of the "Natural Layering Concept" and includes 2 distinctive restorative layers (body & skin). It has smaller dentin/body samples, which can be inserted into enamel/skin samples, to demonstrate any 2-LAYER combination. The selection of enamel/skin is to be made visually where more enamel can be observed, namely in the incisal or proximal areas. It is recommended to use glycerin in between both composite samples.

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
Step 1: selection of dentin chroma in the cervical area

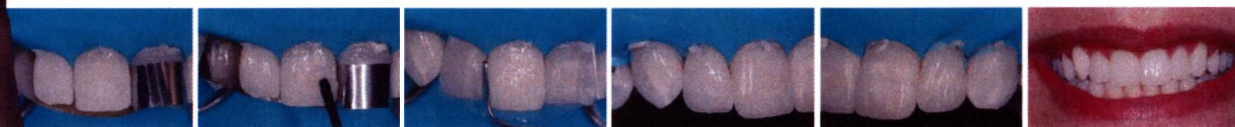Step 2: combination of dentin & enamel samples

EFFECT SHADES & POLYCHROMATIC RESTORATIONS

Teeth can present unlimited color and light variations. For the large majority of them, the basic application of the "Natural Layering Concept" will perfectly render the beauty of natural tissues. For the few cases which require additional coloring, the **inspiro** system provides all needed effect shades, which can be used for the following characterization:

AZUR EFFECT

serves to enhance blue-opalescent effects of the incisal edge in front teeth. It is not too intense so that over-characterization can be avoided.

CHROMA EFFECT

is used to increase chroma locally when required. It is applied over the normal dentin build-up to mimic all kind of "orange" intensive effects such as dentin sclerosis, counter-opalescence and some forms of pigmentation, due for instance to aging.

ICE EFFECT

helps to simulate widespread enamel opacities and serves then to create those discrete, natural "imperfections" such as light fluorosis or surface demineralization.

OPAQUE EFFECT

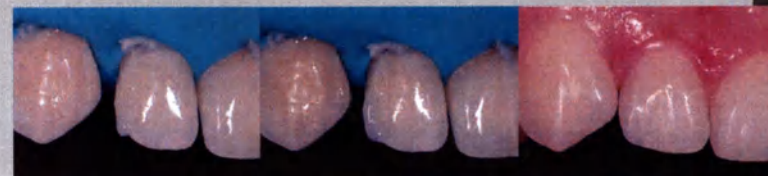
is mainly used in combination with other effect shades (i.e. ICE or CHROMA) to produce "opaquers" of desired shade and chroma to cover either severely discolored tissues or metals.

FISSURE EFFECT


is to be placed in the depth of occlusal fissures of class I and II restorations to improve their esthetic integration and stimulate the artistic talent of practitioners when doing posterior restorations.

Ice effect

Azur effect

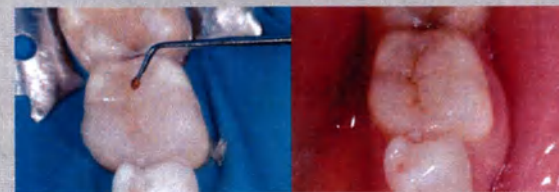Fissure effect

AZUR
CHROMA
ICE

NANO PARTICLES


A new homogenous Nano-Hybride Composite:

Fillers with particle sizes in the nanometer range ("nano particles") show a strong tendency to aggregate and agglomerate. In this "cluster" form, they do not reach their full potential for improvement of the mechanical and surface properties of the composite; in fact, such aggregated and agglomerated nano particles have been known for a long time and used in dental materials for many years (impression materials are composites for instance). **inspiro** on the contrary, uses a new technology which takes advantage from true nano-particle technology where, differently from all other so called "non-homogeneous" nano-hybrid composites, this ultrafine filler is separated and evenly distributed in the matrix. Here, the nano-particles can reach their full potential for enhancement of surface quality, flexural strength and modulus, among many other unique physicochemical properties.

LINEAR SHRINKAGE DISPLACEMENT

(University of Geneva; according to the methodology of Stavridakis et al, 2003 & 2005)

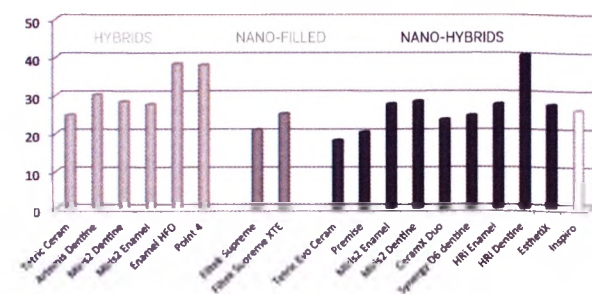Fig. 4. Linear shrinkage displacement


Mechanical properties & benefits:


- Low shrinkage due to nano-technology and high amount of filler, 83%
- Very good physical and mechanical properties
- Good abrasion resistance
- Antibacterial surface due to zinc and fluoride particles in the filler
- Excellent polishability and gloss retention
- Natural fluorescence and opalescence

(Source: internal data from EdelweissDR)

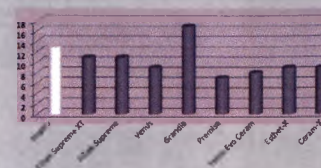
FLEXURAL MODULUS (GPa)

FLEXURAL STRENGTH (MPa)

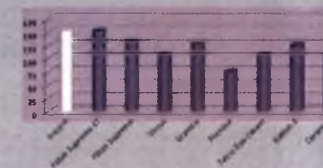COMPRESSIVE STRENGTH (MPa)

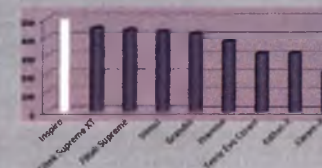Properties	Restorative shades	Flowable shades
Filler content	83% (W) / 65% (V)	85.5% (W) / 38% (V)
Particle size range	0.02-3 µm	0.02-3 µm
Glass filler type	Barium glass with Zinc-Fluoride & nanoparticles	Barium glass with Zinc-Fluoride & nanoparticles
Glass average size	0.7µm	0.7µm
Matrix system	Bis-GMA	Bis-GMA
Flexural modulus	12.5 GPa	6 GPa
Flexural strength	150 MPa	120 MPa
Compressive strength	480 MPa	350 MPa
Working time	~1 min	~1 min

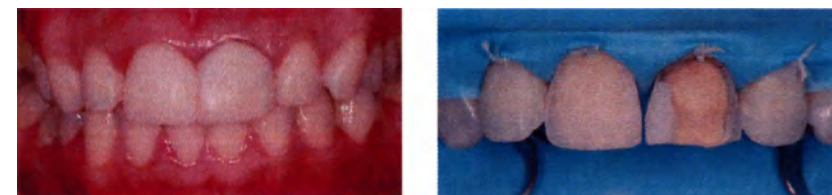
DIRECT RESTORATIVES

Form Corrections

DIRECT RESTORATIVES

Dr. Didier DIETSCHI, D.M.D., Ph.D., PD

Direct Veneers

...“simple and effective layering approach” does not mean to accept any compromise on the esthetic treatment outcome, on the contrary!

Dr. Didier Dietseki

"What do I find so special about the inspiro system?"

1. The **inspiro** system is based on a simple and rational shading concept, consisting of universal dentin shades and multi-tint, multi-translucency enamels, which I have refined over the last decade. Using this straightforward approach, I am able to achieve the right combination for mimicking any natural tooth appearance.

2. The **inspiro** effect shades give even more freedom to develop our "artistry" and achieve unsurpassed esthetic results following a simple, effective layering approach

3. I can pre-visualize and control the color integration thanks to the special bi-laminar **inspiro** shade guide; it means that when the **inspiro** shade sample matches the natural teeth, I will get it right!

4. I have an accurate selection of restorative and flowable consistencies suitable for various different clinical situations, from posterior restorations to more complex smile rehabilitations... effectively

For once, clinical excellence can be the result of simplicity and reliability!

Swiss haute couture

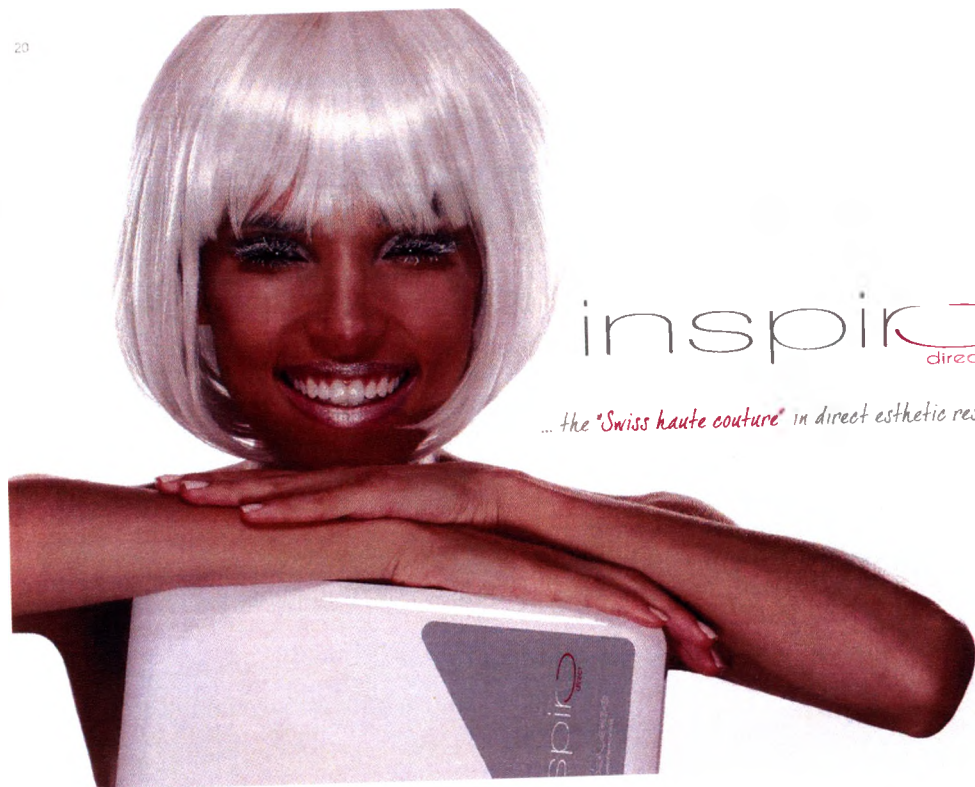
in esthetic dentistry

Esthetic Rehabilitation

Posterior Restorations



inspiro
direct

... the "Swiss haute couture" in direct esthetic restorations

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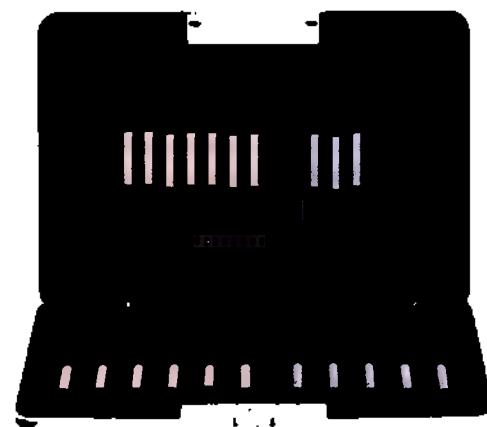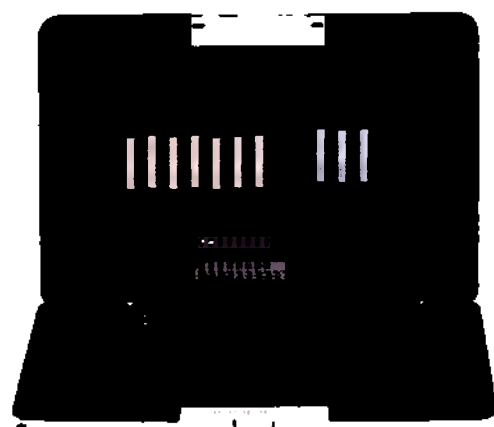Instruction of use Inspiro Shade Guide

Use: As per clinical photos shown below,

1. Select the dentin shade by placing the dentin sticks in the cervical are of the tooth to be restored.
2. Select visually an enamel on the basis of the tint and translucency of the tooth natural enamel.
3. Apply a drop of water or glycerine gel into the enamel sample, then place the dentin into the enamel sample.
4. Check the color at the incisal margin. Eventually change the enamel sample until the perfect match is found.

Characteristic: Bi-Laminar Shade Guide made with the inspiro composite system for easy shade selection and predictable color matching.

Parameters - size: 10.5 x 5 x 0.5cm,
 - material: - Tip = inspiro composite
 - Base = ABS
 - Stick = PC

Expiry date: the shade guide does not have expiry date.

Conditions storage: no particular storage conditions are required.

VITA	A1	B1	C1	A2	B2	C2	D4	A3	B3	C3	D5	A4	B4	C4	D6
tooth	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26
tooth	W/N	T	W/N	T	W/N	T	W/N	T	W/N	T	W/N	T	W/N	T	W/N

Instruction of use Inspiro application tips

Use:

1. Remove the syringe cap
2. Screw the application tip on
3. Apply a gentle pressure on the piston of the syringe to extrude the material.

Characteristic: Application tip for flowable and effect shade materials, with narrow gauge in order to precisely apply the material as well as avoid waste.

Parameters - Size: length - 33.0; bore diameter syringe - 3.0

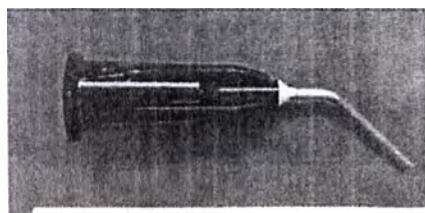
- Material: Dispensing tip, single use, black

Hub: Polypropylene (PP)

Tube: Stainless steel SUS304

Expiry date: the shade guide does not have expiry date.

Conditions storage: no particular storage conditions are required.

Manuela Brusoni
EDELWEISS DR AG
Unter Altstadt 28, Mercandor
6300 Zug, Switzerland

10. 06. 2016

June 10th, 2016

To whom it may concern

We, Edelweiss DR AG, located at Unter Altstadt 28, Mercandor, Zug, Switzerland, hereby certify that the *inspiro*TM kits, precisely item no. 3010, 3020, 3030, 3040, 3050, 3055 do not have a specific Instruction for Use because they simply contain a selection of items Restorative/Flowable/Effect Shade, also sold as refills, whose IFU is enclosed in the box.

Issued by:
Manuela Brusoni
EDELWEISS DR AG
Unter Altstadt 28, Mercandor
6300 Zug, Switzerland

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