INSTRUCTIONS FOR USE FOR MEDICAL PRODUCT

Solution for mouth and throat rinsing for children and adults
NATUMA Aquaflu throat 125 ml
manufactured by YS Lab, France

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1. Main characteristics

Description:

Solution for mouth and throat rinsing for children and adults NATUMA Aquaflu throat 125 ml is a spray in a metal container under pressure with an internal protective anti-corrosion coating and ergonomic throat nozzle. The contents of container is a clear, colorless, odorless liquid with a salty taste, a sterile sea water solution, with a hypertonic NaCl concentration 19-23 g/l. Hypertonic seawater solution provides local moisturizing effect, reduces pain while swallowing, dryness and burning sensations, reduces swelling and congestion in mucosa. Seawater solution reduces edema in mucosa. It washes off viruses and bacteria from the surface of mucosa in the mouth, throat, tonsils, reducing microbial contamination. It enhances the therapeutic efficacy of drugs applied to the mucosa.

This drug is intended for children over 1 year old and adults.

2. Technical specification

The drug's technical and functional specifications are as follows:

Parameter name	Description
Product description and design	Aerosol in a metal container under pressure. Four-layer bag with the content provided with a special one-way valve maintaining sterility of the contents during the period of use by preventing the reverse current of the contents and preventing entering the ambient air into the bag is placed inside the container.
Packaging materials	Container - aluminum (ALU 8); bag with a one-way valve: four-layer bag - polypropylene (PP 75) - polyamide (OPA 15) - aluminum foil (ALU 8) - polyester (PETP 12); the valve body - polypropylene (HJ 325MO); valve piston - polyacetal (POM Kepital F2503); spring - stainless steel (AIS 316); inner sealing - rubber (DB 219); outer sealing - butyl (U 133); valve cup - aluminum (ALU 8); nozzle - polyethylene (PE HE7541).
Product form	Throat spray
Dimensions (mm) h - height; d - diameter	Container (h - 148 ± 1 ; d - 45 ± 1) sprayer (h - 70 ± 1 ; d - 15 ± 1)
The fill volume (ml)	125
Weight (g)	Container 160 ± 5 ; spray tip $1,97 \pm 0,3$
Contents characteristics	Clear colorless and odorless liquid with a salty taste which is a sterile sea water solution with hypertensive NaCl concentration
Composition of content	natural sea water 45 %, purified water 53,84 %
The content of chemical elements in seawater	Sodium, Na (mg/l) - 10750.0 Manganese, Mn (μ g/l) - 9 Chloride, Cl (mg/l) - 19780 Brome, Br (mg/l) - 67 Magnesium, Mg (mg/l) - 1290 Calcium, Ca (mg/l) - 404 Potassium, K (mg/l) - 403

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	Iodine, I ($\mu\text{g/l}$) 60 Zinc, Zn ($\mu\text{g/l}$) 0.5 Copper, Cu ($\mu\text{g/l}$) 0.2 Selenium, Se ($\mu\text{g/l}$) 0.3
The content of chemical elements in solution of a medical device, (%)	Sodium, Na – 0,47 Manganese, Mn – 0,0000004 Chloride, Cl - 0,87 Brome, Br – 0,003 Magnesium, Mg - 0,056 Calcium, Ca – 0,018 Potassium, K – 0,018 Iodine, I - 0,000003 Zinc, Zn – 0,00000002 Copper, Cu – 0,000000009 Selenium, Se – 0,000000009
pH	6.0-8.5
NaCl content (g/l)	19-23
Storage conditions	Store in a dry place protected from direct sunlight at a temperature range of +5 ° C to +25 ° C, out of reach of children.
Sterility	Sterile. Irradiation sterilization. Maintains sterility during the whole period of use.
Magnitude of volume of a substance extracted	>120,0
Relative density at 20 ° C (density solution) ml	1.008-1.016
Source of raw materials for the aerosol substance from which the manufacturer permits the preparation of a medical product	Nitrogen, Sea water from the North of Brittany on the Pink Granite Coast (Cote de Granit Rose), in the Cotes d'Armor department, in Pleumeur-Bodou, on the island "Ile Grande" (22560), more than 22 meters deep. Purified water
Maximum pressure for unclosing of aerosol valve	should not exceed 40 N (4 kp)
Allowable steam leakage through seal of nipple	should not exceed 0,1 $\pm$ 0,05 cm ³ during 6 hours under pressure (1,20*0,05 MPa
Allowable case deformity under overpressure	Allowable deformity of valve (case) under overpressure no more 0,3 $\pm$ 0,05 mm und overpressure of vapor: (1,00 $\pm$ 0,05) MPa – for aluminum case
Stability to axial force of compression (opening/closing)	Aerosol valve should withstand axial force of compression not less 150 N (15 kp) during 500 work cycles (opening/closing)
Spray head's nozzle pressure	Spray head's nozzle should withstand pressure without falling out: Plastic nozzle - (1,00 $\pm$ 0,05) MPa
Force of tube removal off pocket shank	Not less 60 N (6 kp). Allowable gap between tube's back and pocket - no more 0,8 mm.
Volume of a single dose	100-300 μL
Internal burst pressure	1,5 MPa
Withstand the internal pressure of balloon without causing permanent deformation	1,2 MPa;

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The material of internal protective anti-corrosion coating	Internal coating: SISTIAGA® 1420-14 Aluminum Container Coating pigmented epoxy phenolic based lacquer for use in monobloc aerosol c The material has protective and anti-corrosion effect. The coating is 1 cured and solid. The value of electrical current flowing through the coating is not more 20 mA
Outer balloon coating	The outer balloon coating is chemically resistant and resistant to hot water Mark of outer balloon coating's material is a paint SWS13-POM WeiZb 50, Item-No. 91-01449, manufacturer is GRAFE Color Batch Gm Germany.

3. Intended use of the medical product

Solution for mouth and throat rinsing for children and adults NATUMA Aquaflu throat 125 ml is intended for the prevention and for complex therapy of acute and chronic inflammatory diseases of the mouth and throat (infectious, viral, bacterial, fungal, allergic, atrophous, acute and chronic pharyngitis, acute (angina) and chronic tonsillitis, epiglottitis, laryngitis, stomatitis, gingivitis, before and after surgery in dentistry and otolaryngology. It is also used for the comprehensive treatment of ARVI (acute respiratory viral infections) and influenza, influenza prevention while epidemics, preparation of mouth and throat for application of medicines.

4. Indications and contraindications for use:

Indications for use:

- It is indicated for children over 1 year old and adults.
- Prevention and comprehensive treatment of acute and chronic inflammatory diseases of the mouth and throat (infectious, viral, bacterial, fungal, allergic, atrophic):
 - acute and chronic pharyngitis
 - acute (angina) and chronic tonsillitis
 - epiglottitis
 - laryngitis
 - stomatitis
 - gingivitis
- Before and after surgical intervention in dentistry and otolaryngology;
- Comprehensive ARVI and influenza treatment, prevention during epidemics;
- Preparations of mucosa for application of medicines.

Contraindications:

Individual intolerance to the components of container contents.

Special instructions:

Avoid contact with eyes.

Pregnancy and breast-feeding is not a contraindication.

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5. Preparation and Application:

- Put the nozzle on the container;
- Direct a spray nozzle to the focus of inflammation (depending on the disease - to the rear wall of the pharynx, tonsil), carry out the washing procedure for several seconds;
- For children over one year and adults - repeat the procedure for 3-4 injections 4-6 times a day.

6. Instructions for sanitization

Treat ergonomic nozzle with boiling water or with a solution of 70 percent ethanol before use of this medical product.

7. Packaging and labeling

Metal container with ergonomic nozzle for the throat in a carton box with instructions for use.

Container bears the following information:

- name and address of manufacturer company, country;
- product name in Russian and English;
- intended use of the product;
- method of use;
- dispensing information;
- number of the registration certificate;

Package contains:

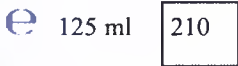
- name and address of manufacturer company, country;
- product name in Russian and English;
- intended use of the product;
- method of use;
- dispensing information;
- address for the claims submission;
- number of the registration certificate

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Marking

Cartoon box

Marking	Meaning
	Net weight 125 ml
	Refer to the instructions for use
 Fabriqu� par : Yslab 2, rue F�lix Le Dantec 29000 Quimper - France	Name and the address of manufacturer
	Code of products' certification in the European Union
	Batch code Expiry date (indicated on the box's bottom)
	Has been sterilized
	Packing from cartoon
	Self recycling (do not contaminate the environment)
	Non-toxic material
	The Unified Mark of products on the market of states - members of the Customs Union
Warning: Pressurized container: may explode when heated. Keep away from heat, sparks, open flames, hot surfaces. No smoking. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50 ° C. Keep away from children. Store at temperatures between +5 ° C to +25 ° C.	Features of use and storage

Si ge Social : YSLAB – 2 rue F lix Le Dantec – 29000 QUIMPER– FRANCE
SAS au capital de 100 000 Euros – RCS Quimper 433 203 593 – Code APE 2042Z

Metal container

Marking	Meaning
 125 ml 	Net weight 125 ml
 Fabriqué par : Yslab 2, rue Félix Le Dantec 29000 Quimper - France	Name and the address of manufacturer
	Manufacture date
 0459	Code of products' certification in the European Union
 	Batch code Expiry date (indicated on the box's bottom)
	Has been sterilized
 	Packing from aluminum (container) and plastic (nozzle and cap)
	Self recycling (do not contaminate the environment)
	Non-toxic material
	The Unified Mark of products on the market of states - members of the Customs Union
Warning: Pressurized container: may explode when heated. Keep away from heat, sparks, open flames, hot surfaces. No smoking. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50 ° C. Keep away from children. Store at temperatures between +5 ° C to +25 ° C.	Features of use and storage

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8. Terms of storage, transportation, disposal

Shelf life is 3 years. Use before the date indicated on the package.

Store in a dry place protected from direct sunlight at a temperature range of +5 ° C to +25 ° C, out of reach of children.

Solution for mouth and throat rinsing for children and adults NATUMA Aquaflu throat 125 ml is transported by all kinds of vehicles in covered vehicles in accordance with regulations applicable for the given type of transport at temperatures from +1 ° C to +40 ° C.

Solution for mouth and throat rinsing for children and adults NATUMA Aquaflu throat 125 ml at the end of life should be utilized in accordance with the applicable national regulations, together with other industrial waste.

Cure for nose cavity lavage « NATUMA Aquaflu throat 125 ml » refers to Class A of medical waste classification in accordance with SanPiN's sanitary - epidemiological rules and norms 2.1.7.2790-10.

9. The manufacturer, warranty

Solution for mouth and throat rinsing for children and adults NATUMA Aquaflu throat 125 ml is manufactured by YS Lab, Le Forum, 2 rue Felix Le Dantec, QUIMPER 29000, France.

The quality of the medical product and YS Lab, France production facilities conform all the international quality standards that is confirmed by certificate ISO 13485:2012 No.12656 Revision 5 and declaration of conformity for the product.

The manufacturer guarantees the quality of the product till the expiration date.

Complaints about the quality of this medical product on the territory of the Russian Federation should be sent to "PharmaStar, LLC", corp. 1A, 7, 2 Baumanskaya str., 104005 Moscow;
tel. 8-495-969-21-84.

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