

Keller Funnel™

U.S. and International Patents Pending



Keller Medical, Inc.

Do Not Reuse: For Single Patient Use

Ranges below are based on implant volume in cc's and are suggestions only. Optimum trim location will vary based on surface texture, shape, cohesiveness, projection, base width, and height.

Smooth Round: Trim as shown.

High Profile Smooth: Trim 0.5 cm larger.

Textured Round: Trim 0.5 to 1 cm larger.

Form Stable (Anatomical Implants):
Trim 1 to 2 cm larger.

X = Implant Volume in CC's

8.0 cm

7.5 cm

7.0 cm

6.5 cm

6.0 cm

5.5 cm

5.0 cm

4.5 cm

Keller Medical, Inc.

www.kellermedical.com
900 SA Central Expressway, Suite 111, Irvine, CA 92618-1111
949.252.2100

Keller Funnel™

most comfortable, most effective, most reliable
funnel for use with any type of implant
system. It is made of soft, flexible material
and is designed to be used with any type of
implant system.

REF

KS-005



EXP

2012-09



LOT

K11040101



QTY

2011-09



STERILE R

1

2

3

QTY

EO REP

Rx Only

CE 0086

© 2008 Keller Medical, Inc.

Model: KS-005 Rev: 1.0

U.S. and International Patent Pending LA 022-01 Rev. 1

