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D. Wall

08.12.2015

Bitmos

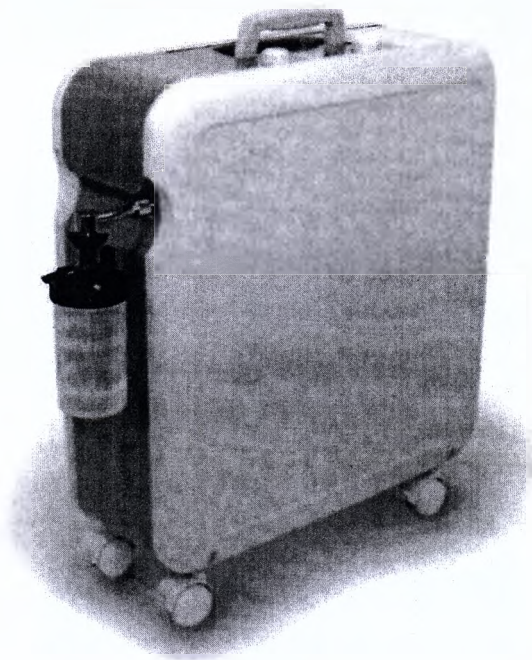
Bitmos GmbH
Himmelgeister Str. 37
D 40225 Düsseldorf
Germany

Bitmos
MEDIZINTECHNIK

Instructions for Use

Oxygen concentrator "oxy6000"

Read the instructions for use
before working with the unit.

Before
taking into operation
check system components,
control, emergency stop
and safety equipment
for condition, damage
and correct function!

© Bitmos GmbH
Himmelgeister Str. 37
D-40225 Düsseldorf

Phone: +49 (0) 211 60101030
Fax: +49 (0) 211 60101050

E-Mail: info@bitmos.de
Internet: www.bitmos.de

Dok-ID: 41-1201.EN

Version: 3 16.07.2015

1 Preliminary statement.....	6
2 General	7
2.1 Information in these instructions for use	7
2.2 Type plate	8
2.3 Liability and warranty	8
2.4 Warranty	8
2.5 Explanation of symbols	9
2.6 Copyright protection.....	10
2.7 Return and waste disposal	10
2.8 Customer service	10
3 Safety	11
3.1 General	11
3.2 Customer's responsibility	11
3.3 Intended use	12
3.4 Dangers which may arise from the unit	12
3.5 What to do if a hose caught fire	15
4 Design and function.....	16
4.1 General description.....	16
4.2 Design	16
5 Technical data	18
6 Transport, packaging and storage.....	20
6.1 Transport inspection	20
6.2 Storage	20
7 Taking into service	21
7.1 Before assembling	21
7.2 Choosing the location	22
7.3 Assembly	23
7.3.1 Without external humidification	23
7.3.2 With external humidification	26
8 Operation	29
8.1 Taking into service	29
8.2 Adjusting the oxygen volume flow	30
8.3 Status messages and alarms	31

8.3.1 Status messages	31
8.3.2 Status message	32
8.3.3 Alarm priorities	32
8.3.4 Alarm categories	33
9 Maintenance	36
9.1 Safety	36
9.2 General notes	36
9.2.1 Cleaning	36
9.2.2 Disinfecting	36
9.3 Maintenance plan	37
9.4 Maintenance work	39
10 Spare parts	41
11 Explanation of symbols on the unit	42

Preliminary statement

1 Preliminary statement

Your doctor has found that you require an additional oxygen supply. With the oxy6000 you received a German brand product for oxygen supply, which has been developed on the basis of the latest knowledge in both medical engineering and electronics. Permanent quality inspections ensure uniform quality on the highest level.

The oxy6000 is a highly reliable oxygen concentrator, intended for use in homes or at home, as well as for clinical applications.

However, should problems arise with the oxy6000, you may contact your dealer at any time.

This medical care product from Bitmos is labelled with the CE-sign according to MDD (Medical Device Directive).

Only use the oxy6000 after a medical indication and only in compliance with the doctor's orders by following these instructions for use.

If side effects or extreme health restrictions occur during the therapy, you should immediately consult your doctor.

2 General

2.1 Information in these instructions for use

These instructions for use describe the installation, operation and maintenance of the unit. Strict compliance with the stated notes on safety and instructions for use is a prerequisite for safe and proper work with the unit.

Moreover, compliance with the accident prevention instructions valid at the location of use and the general safety regulations is mandatory.

This instruction manual is part of the product and should be kept near the unit so that it is at any time available for personnel involved in installation, operation, maintenance and cleaning.

The graphic illustrations in this manual may perhaps differ slightly from the actual design of the unit.

General

2.2 Type plate

The type plate of the oxy6000 is attached to the back of the unit above the coarse dust filter.

2.3 Liability and warranty

All details and notes for the operation, maintenance and cleaning of the unit are made to the best of our knowledge taking into consideration our experience and knowledge gained up to now.

We reserve the right to make technical changes to the machine dealt with in these instructions for use within the context of ongoing development.

Translations are also carried out to the best of knowledge. We do not accept any liability for errors in translation. The German version of the instructions for use, which is also delivered with the unit, is the definitive version.

Texts and illustrations do not necessarily correspond to the scope of delivery. The drawings and graphics are not to scale 1:1.

Read instructions for use carefully before starting operation of the unit!

The manufacturer will not assume liability for damage or disruptions that occur as a result of non-compliance with the instructions for use.

Handing over these instructions for use to third parties is not permitted and results in the obligation for compensation.

2.4 Warranty

Warranty and possible after-warranty service is provided by the Buyer or the Company appointed by the Buyer.

Differing from our General Terms of Business we grant an extended warranty of 30.000 operating hours for all functional parts (e.g. compressor, control board, valve technology, etc.) used in our oxygen concentrator oxy6000. This extended warranty is valid for maximum 5 years from the date of purchase from the manufacturer.

Indispensable condition is regular check and documenting, according to the Technical Information of the device.

Warranty service will consist in replacement, free of charge, defective parts on new, at obligatory observance of terms of carrying out the service works specified in the maintenance instructions and representations to the manufacturer of the act with the indication of causes of defect, serial number of the device, business hours, serial number and operation hours of the functional unit (at its malfunction) and screen copies of the test program.

Filters and zeolites, and also damages by misuse and device use at treatment, mechanical damages are excluded from guarantees (for example during transport), and also at non-use of original spare details.

Warranty of the manufacturer is limited to free delivery of new spare blocks or details for replacement of defective parts. Defective parts have to be returned to the manufacturer for verification of the warranty claim.

Local and travel expenses connected are not covered by the manufacturer.

If the device is returned to the producer free of charge during the warranty period, any repair work on replacement of defective details and blocks will be made free of charge.

2.5 Explanation of symbols

Important safety and equipment related notes in these instructions for use are highlighted by symbols. These notes must be strictly adhered to in order to avoid accidents, personal injuries and damage to property.

WARNING!

This symbol warns of dangers that can lead to adverse effects on health, injuries, permanent physical damage or to death.

Strictly comply with all notes regarding work safety, and be particularly careful in these situations.

WARNING! Danger of electric current!

This symbol draws attention to dangerous situations involving electrical currents. There is a danger of serious injury or death if the safety notes are not complied with. The related work may only be carried out by qualified electricians.

ATTENTION!

Information highlighted with this symbol must be strictly complied with in order to avoid damage to the machine, malfunction and/or breakdown.

NOTE!

This symbol highlights hints and information to be observed for efficient and non-disrupted operation of the unit.

General

2.6 Copyright protection

These instructions for use are to be treated confidentially. They should only be used by persons who have been authorized accordingly. It may only be passed on to third parties with the written consent of the manufacturer.

All documents are protected by copyright protection law.

It is not permissible to pass on or reproduce the documents, either as a whole or excerpts thereof, to evaluate or communicate their content, unless explicitly entitled to do so. Any violations are liable to prosecution and require compensation.

We reserve the right to exercise industrial property rights.

2.7 Return and waste disposal

- If the unit has been delivered through a parcel service and not directly by a dealer you should keep the packaging material for possible service instances.
- If there is no corresponding agreement regarding the return of packing material, then the packing material remains with the customer. He is responsible for environmental waste disposal in accordance with the applicable waste disposal legislation.
- After use the unit may be returned to the dealer, who is then responsible for proper disposal of the unit.
- Non-infectious used accessories (e.g. nasal cannula) can be disposed off as domestic waste
- Infectious accessories (e.g. nasal cannula of an infected user) must be disposed of through a specially approved waste disposal company. Addresses are available from your local municipality.

2.8 Customer service

Service work should normally be carried out by your local authorized dealer.

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3 Safety

This section provides an overview over all important safety aspects for safe and trouble-free operation of the unit.

The individual chapter additionally contain actual safety notes highlighted with symbols, which will help to avoid immediate dangers.

3.1 General

The machine is built according to the currently applicable rules of technology and is safe to operate.

However, dangers may still arise from the unit if it is incorrectly operated or used for purposes it is not intended for.

Any persons using this unit must have read and understood these instructions before starting operation. This also applies if the person in question has already worked with just such a unit or similar equipment or was trained by the manufacturer.

Knowing the content of these instructions for use is a prerequisite for the avoidance of mistakes and for safe and trouble-free operation of the unit.

Neither changes nor conversions may be carried out on the equipment, which have not been explicitly authorized by the manufacturer, to avoid dangers and to ensure optimal performance.

All safety decals and operating signs on the unit must be kept well legible at all times. Damaged or illegible decals must be replaced immediately.

3.2 Customer's responsibility

These instructions for use must be kept near the unit, so that it is available for the user at any time.

Apart from the notes on safety mentioned in this manual, all generally valid safety and accident prevention instructions must also be observed and adhered to.

The machine may only be operated in a technically perfect condition and if operationally safe.

The information contained in the instruction manual is complete and must be adhered to without limitation.

Safety

3.3 Intended use

The operational safety of the unit is only assured when used for the purpose it is intended for, as specified in the instructions for use.

The oxy6000 solely intended for use within the scope of a medical therapy for the additional, non-life-sustaining supply of oxygen. The unit may thereby be used in hospitals, homes or at home.

The unit must only be used according to the medical indication and only in accordance with the doctor's prescription by following these instructions for use

The intended use also includes the compliance with the assembly instructions, as well as the notes on cleaning and maintenance of the unit.

Any other use of the unit beyond these limits is prohibited and is not considered as unintended use! Claims of any kind against the manufacturer and/or his authorized representatives resulting from damage caused by unintended use of the unit are excluded. The customer is solely liable for any damage resulting from unintended use.

3.4 Dangers which may arise from the unit

The unit was subjected to a risk analysis. The resultant construction and design of the unit corresponds to the current status of technology.

However, there is still a remaining risk!

The unit requires responsibly minded and cautious operation. Improper operation or operation by unauthorized persons can endanger persons.

WARNING! Risk of health damage!

If an absolutely safe oxygen supply is required, it is strictly necessary to have a second, independent oxygen source available as replacement (e.g. a mobile oxygen savings system with an oxygen cylinder).

If the patient or the operator notices at any time that the available amount of oxygen is not sufficient, you should immediately contact your dealer and/or doctor.

WARNING! Risk of health damage!

Particular supervision is required if the unit is to be used in the vicinity of children or bedfast persons. The unit must under no circumstances be used with children without additional supervision!

WARNING! Danger of side effects!

If side effects or extreme health restrictions occur during the therapy, you should immediately consult your doctor.

**WARNING! Fire hazard caused by oxygen!**

Oxygen is vital, but in concentrators with only a few percentage points above the normal oxygen content in the air it is a highly dangerous fire accelerant. There are only a few materials which will not burn off like an explosion under a raised concentration of oxygen.

Therefore:

- Oxygen may only be handled by trained or specially instructed persons!
- The misuse of oxygen, e.g. to cool down or improve the ambient air, to cool down and dust or blow off of persons, clothes, furniture etc. is dangerous and therefore prohibited!
- Smoking and handling ignition sources and open flames is prohibited when working with oxygen!
- Keep a minimum distance of 2 metres from all spark generating equipment and open fire!
- After having stayed in a possible oxygen saturated atmosphere you should thoroughly aerate your cloths, because oxygen adheres to the clothes very well! An ignition source, e.g. a burning cigarette, could easily cause burning of your clothes.
- Materials that do not burn in air may burn very vigorously and even spontaneously in oxygen or oxygen enriched air. This already applies for an enrichment of only a few percent!
- Oil and grease (even creams and gels) can react in an explosion like manner when coming in contact with oxygen. It is therefore mandatory to keep the unit free of oil and grease!
- Oxygen considerably increases the temperature of a flame and the speed of combustion!
- Do not fill the moisturizer with inflammable fluids!

**WARNING! Danger of electric current!**

Electric energies can cause severe injury. Damaged insulation or components cause a danger to life.

Therefore:

- Work on the equipment must only be performed by trained professionals.
- Pull the mains plug out before starting work on the unit!
- Check mains leads for damage before every use.

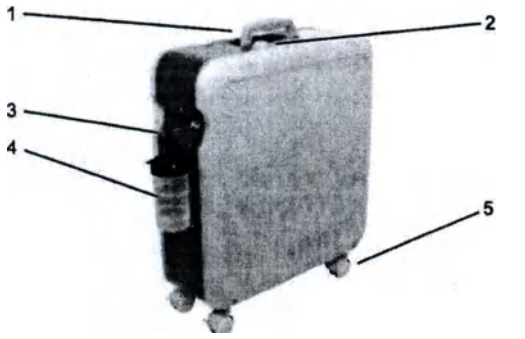
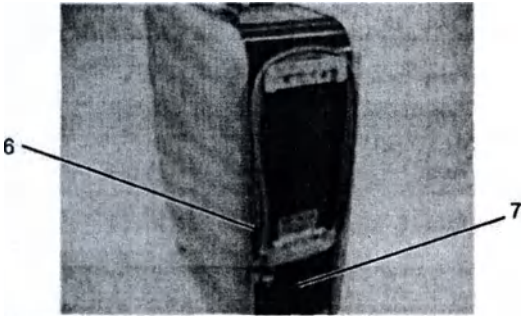
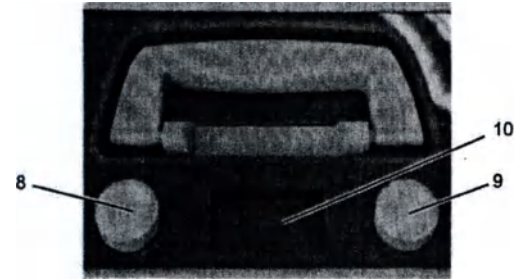
Design and function

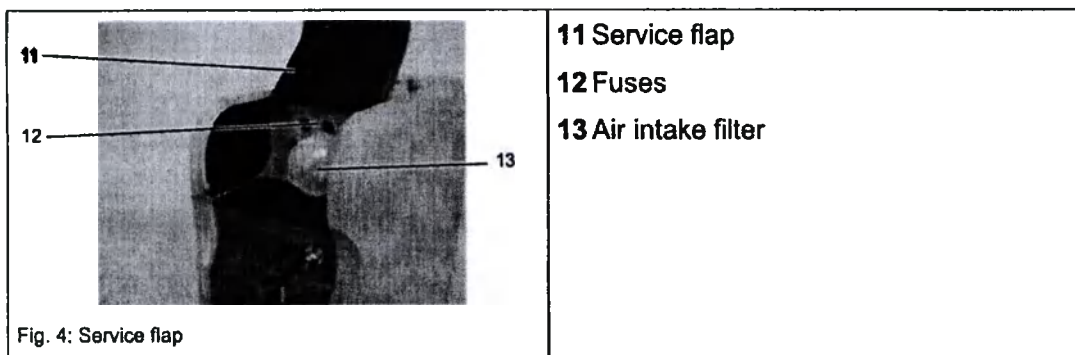
4 Design and function

4.1 General description

The oxy6000 oxygen concentrator for has been optimized for the oxygen supply at home. The electronically controlled oxygen concentrator separates the oxygen from the ambient air and supplies the user through the nasal cannula with highly concentrated oxygen.

4.2 Design

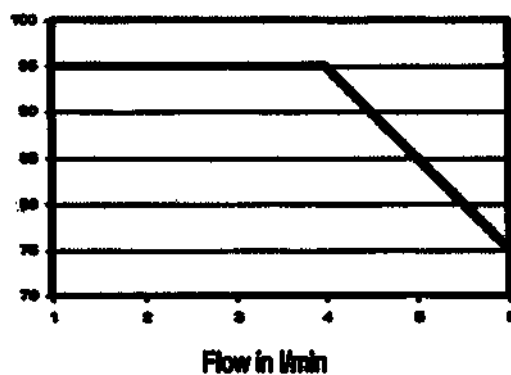
 Fig. 1: Front	Design: 1 Carrier handle 2 Control panel 3 Angle connection 4 Humidifier 5 Steerable castors
 Fig. 2: Back	6 Mains lead 7 Coarse dust filter
 Fig. 3: Control panel	8 On/Off-switch 9 Volume flow regulator 10 LC display

Technical data
5 Technical data

version	oxy6000
Classification acc. to MPG	Ila
Operating voltage	230 V, 50 Hz
Ambient operating temperature	Operation: +10 to +40 °C Storage: -20 to +70 °C
Sound pressure level	< 35 dB(A)
Power input	360 W
Coarse dust filter	in back of housing
Air intake filter	behind service flap
Bacteria filter	inside the device
Fuses	mains side: 2 x TT2,5A H 250 V internal: 1 x T1,0A L 250 V
Interface	USB
Temperature range	+10 to +40 °C
Atmospheric pressure range	700 mbar to 1060 mbar
Weight	19.8 kg
Dimensions (HxWxD)	(53.5 x 20.3 x 52) cm without castors
Warranty of manufacturer	30.000 operating hours, max. 5 years (see chapter 2.4)
O ₂ -concentration	1 to 4 l/min. 95 % - 3 % 4 to 5 l/min. 85 % +/- 3 % 5 to 6 l/min. 75 % +/- 3 %
Status display O ₂ -concentration (at operating temperature)	82% status display 60% deficiency display
max. recommended volume flow	6 l/min
Volume flow	1 - 6 l/min acc. to pre-setting
max. output pressure:	60 kPa
Overpressure relief	275 kPa (compressor)

Characteristic, at a counter pressure of 0 kPa, O₂ concentration in %

6 Transport, packaging and storage

The following should be noted when transporting the oxy6000:

- The unit should only be shipped and transported in its original packaging.
- For transport, e.g. by car, the unit may stand upright or lay on one of the two large flat sides.
- Open the transport box from the top. Do not stand the transport carton upside down or on one of its narrow sides.

6.1 Transport inspection

It is highly recommended to check the complete delivery for completeness and possible transport damage, immediately after receipt.

In case of externally detectable transport damage you should not accept the delivery, or only with reservation. Acknowledge the receipt only with reservation (e.g. on the freight document). Specify the expected damage and inform the manufacturer immediately.

Hidden damage should be claimed immediately after detection, because damage claims can only be lodged within the applicable claims periods.

The packaging material should be saved, it may be needed if the unit has to be returned.

6.2 Storage

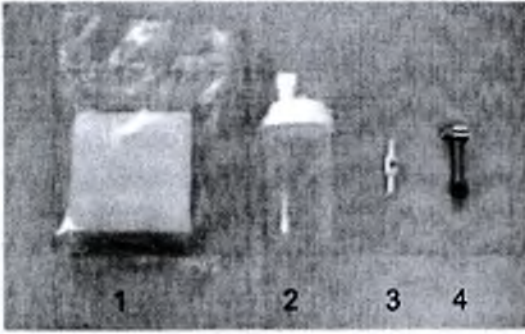
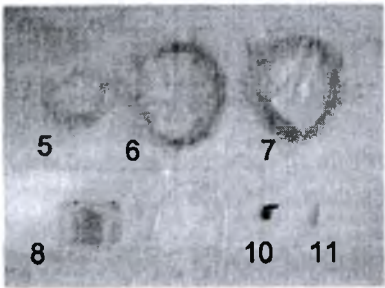
If the package is to be stored before it is taken into service, please observe the following instructions:

- Store in a dry environment. Relative humidity: max. 60 %.
- It must be assured that the package is not stored outdoors.
It must also be assured that the floor used for storage is dry over the entire storage period.
- Storage temperature -20 to +70°C.
- Store in a dust-free environment.
- Avoid mechanical shocks and damages.

7 Taking into service

7.1 Before assembling

Check before assembling whether all components needed for correct operation are available.

 Fig. 5: Scope of delivery 1	Unit: • Unit oxy6000 • User manual
 Fig. 6: Scope of delivery 2	Further components: 1 Coarse dust filter 2 Humidifier 3 Fire Stop 4 Angular connector
 Fig. 7: Scope of delivery 3	5 Nasal cannula for oxygen 2m 6 Nasal cannula for oxygen 5m 7 Oxygen safety hose for oxygen 15m 8 Wall bracket for humidifier 9 Oxygen safety hose for oxygen 0.33 cm 10 Angular connector for humidifier 11 Oxygen outlet connector

Taking into service

7.2 Choosing the location

Please consider the following when choosing the location:

- The unit should have 30cm clearance from any walls, curtains and other large objects (e.g. cupboards), to ensure unrestricted entry of air through the back of the unit.
- The unit is air cooled. It must therefore not be placed near heaters, etc. At such a location there is a risk of overheating.

ATTENTION!

The Fire stop is a thermal fuse designed to stop the flow of oxygen in the event that the downstream PVC tube is ignited. In doing so, the fire in the PV tube is inclined to extinguish because PVC will not normally burn in air. The Fire stop is fitted directly to the outlet of each oxygen delivery device, typically an oxygen flowmeter or the outlet of an oxygen concentrator!

Read through this entire manual before installing the Fire stop. As with all medical equipment, attempting to use or install this device without a thorough understanding of its operation and limitations may result in patient or user injury

1. This device is intended to be fitted to the outlet of a medical oxygen or oxygen concentrator where breathing circuits that have low static operating pressures not exceeding those stated in the device specification. It must not be used for other applications.
2. This device is a flow direction sensitive device. It must be placed with respect to the correct flow direction; otherwise it is ineffective and does not protect the patient in case of fire.
3. Do not install this device near an open flame or near a source of excessive heat that is likely to exceed 40°C.
4. Oxygen is not flammable but the presence of oxygen will drastically increase the rate and severity of combustion. Oil and/or grease in the presence of oxygen become highly combustible. Do not use oil or grease on this device. In particular, do not lubricate the seals or hose barb connector!
5. Never administer oxygen or undertake oxygen therapy while smoking or when near an open flame.

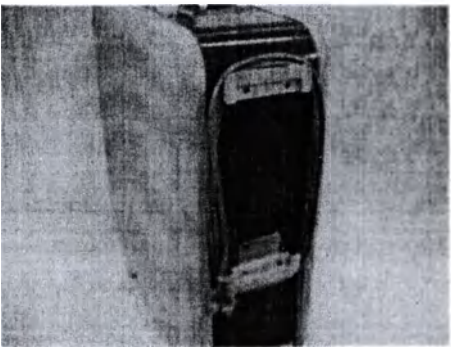
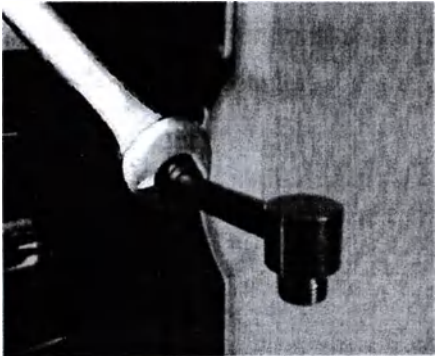
7.3 Assembly

There are generally two possible ways to use the unit.

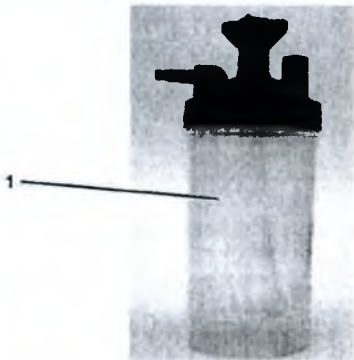
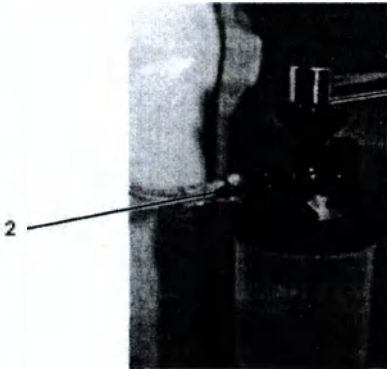
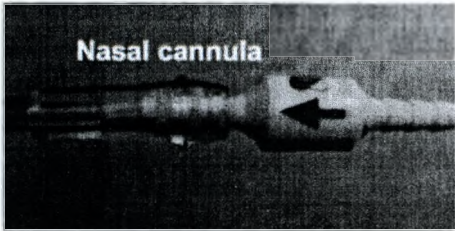
- 1 The unit is standing next to the user during use.
- 2 The user uses the "Set for external humidification". In this case the unit can stand in another room.

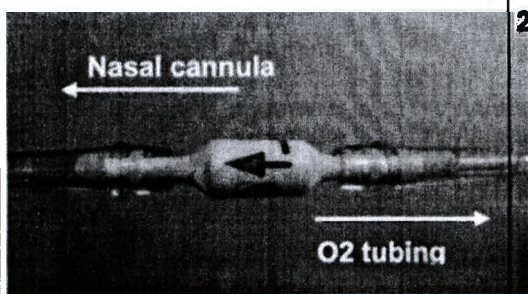
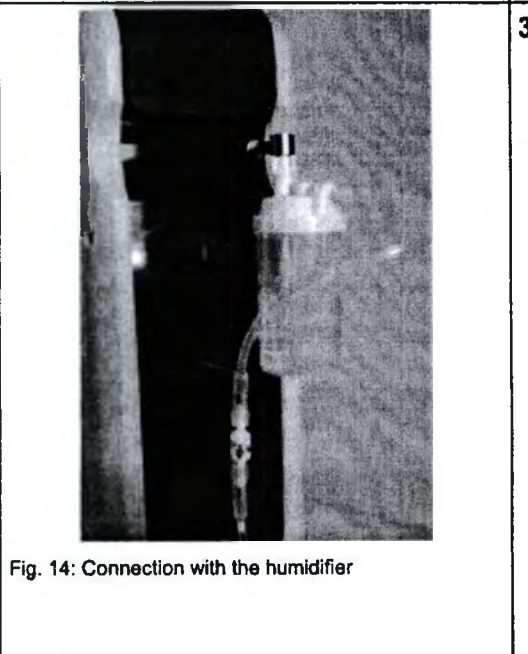
7.3.1 Without external humidification

If the unit is to stand next to the user, it is started up as follows:

 Fig. 8: Mains lead	1 Insert the mains lead into a professionally grounded socket. ! ATTENTION! The oxy6000 is only designed for operation on a 230 Volt, 50Hz mains network.
 Fig. 9: Angle connection	2 The metal oxygen outlet is installed using a wrench (metric size 17).

Taking into service

 Fig. 10: Humidifier Max.-mark	3 Fill the humidifier up to the top MAX.-mark (1).
 Fig. 11: Humidifier assembly	4 Fasten the humidifier on the angular connection (1). 5 Connect the 0.33m oxygen hose to the humidifier outlet (2).
 Fig. 12: Connection to the nasal cannula	1 Plug the Fire stop in oxygen flow direction into the nasal cannula. ! ATTENTION! Observe flow direction indicated by the arrow! ! ATTENTION! For the best protection of the patient, place the Fire stop in the oxygen tubing routing as close to the patient as possible!

	2 Connect the other terminal to the short oxygen tubing (0.33 m).
	3 Connect the oxygen tubing to the humidifier.  NOTE! <i>The humidifier can be used directly at the oxygen concentrator or with the kit for external humidification (near to the patient).</i>

WARNING! Danger of tripping over!

On the back of the oxy6000 you will find a winding device for the mains lead. This should be used when the unit is out of use, to avoid any danger of tripping over.

Taking into service

7.3.2 With external humidification

If the unit does not need to stand next to the user, the installation is carried out as follows:

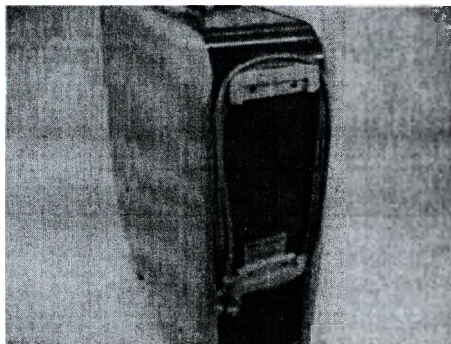
Fig. 15: Mains lead

- 1 Insert the mains lead into a professionally grounded socket.**

ATTENTION!

The oxy6000 is only designed for operation on a 230 Volt, 50Hz mains network.

Fig. 16: Adapter for ext. humidification

- 2 Screw the angle connection to the oxygen outlet of the oxy6000 with a wrench (metric size 17).**

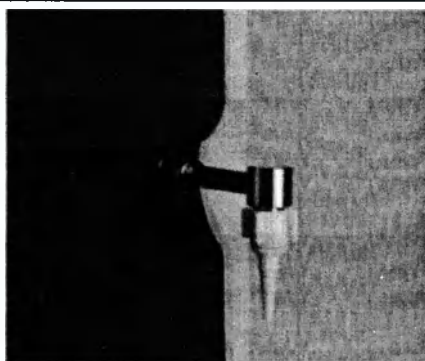
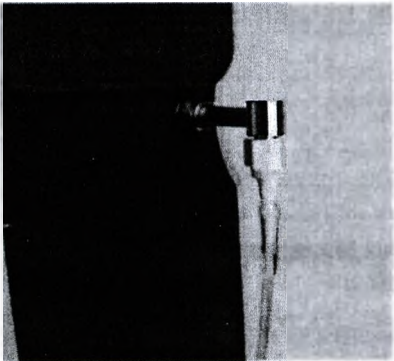
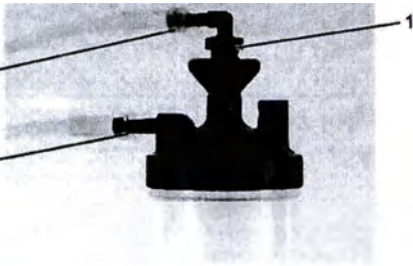
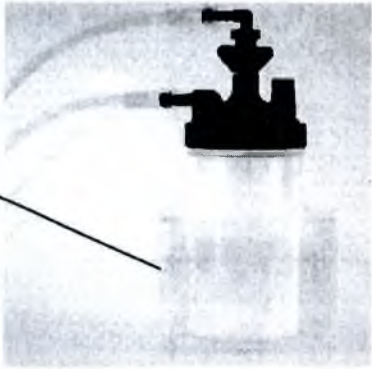
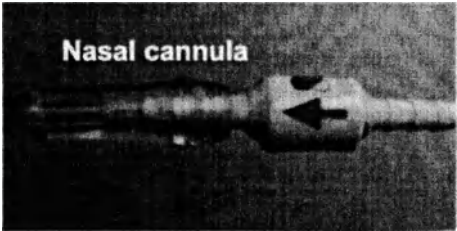
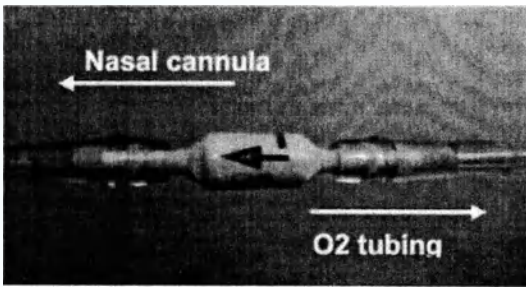
Fig. 17: Oxygen outlet connector

- 3 Connect the oxygen outlet connector..**

 Fig. 18: Oxygen outlet connector	4 Connect the 15m-oxygen safety tubing to the connector.
 Fig. 19: Humidifier Max.-mark	5 Fill the humidifier up to the top MAX.-mark (1).
 Fig. 20: Humidifier assembly	6 Screw the angle adapter (1) to the humidifier. 7 Plug the 15 m safety hose for oxygen (extension hose) onto the angle adapter (2). 8 Push the 0.33-m safety hose for oxygen onto the outlet connector of the humidifier (3).

Taking into service

 Fig. 21: Humidifier bracket	9 Stand the humidifier into the bracket (1).  NOTE! <i>The bracket has the function of preventing the humidifier from falling over. It is therefore recommended to mount the bracket to a wall, a cupboard, etc.</i>
 Fig. 22: Connection to the nasal cannula	1 Plug the Fire stop in oxygen flow direction into the nasal cannula.  ATTENTION! Observe flow direction indicated by the arrow!  ATTENTION! For the best protection of the patient, place the Fire stop in the oxygen tubing routing as close to the patient as possible!
 Fig. 23: Connection with the O2 tubing	2 Connect the other terminal to the short oxygen tubing (0.33 m).

WARNING! Danger of tripping over!

On the back of the oxy6000 you will find a winding device for the mains lead. This should be used when the unit is out of use, to avoid any danger of tripping over.

8 Operation

WARNING! Health risk!

Inappropriate use of the oxy6000 can lead to severe personal and/or material damage.

You should therefore only start up the unit in strict compliance with the instructions for use and the notes on safety.

8.1 Taking into service

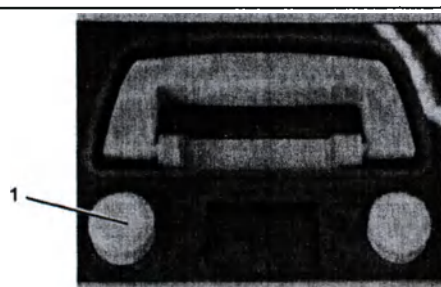
Fig. 24: Control panel

- 1** Start the unit by actuating the **mains switch (1)** on top of the unit.
 - The oxy6000 runs a self-test.
 - The oxygen production starts after the self-test.
 - The LC-display continuously shows the current oxygen volume flow and the equipment status.

NOTE!

During the start-up phase the oxygen alarm appears for 2 minutes. This alarm should disappear after two minutes. If this does not happen, the unit may be defective.

During the start-up phase the unit also shows the operating hours. This information disappears after 30 seconds.

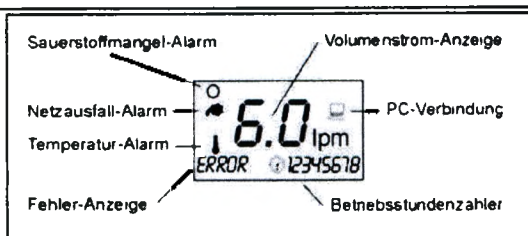
Fig. 25: LC display

- Information concerning the operating status of the unit can be found in the display.

Fig. 26: Nasal cannula

- 2 Wear your nasal cannula.**
Insert both cannula openings into your nose.
Place both feed hoses over your ears.
Pull the sling with the sliding piece tight under your chin.

WARNING! Risk of health damage!
Do not smoke while operating the oxy6000! Oxygen must **NOT** be used near open fire, sparks, glowing objects, etc.!

8.2 Adjusting the oxygen volume flow

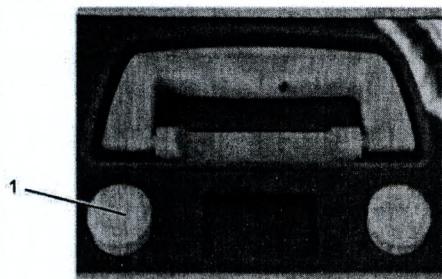
Fig. 27: Adjusting the volume flow

- 1 Press the adjustment regulator (1) for at least 2 seconds.**
 - When a signal sounds and the volume flow indication in the LC display flashes the adjustment mode has been activated.
- 2 Change the volume flow by turning the adjustment regulator.** Turning clockwise increases the volume flow; turning the regulator anti-clockwise reduces the volume flow.

NOTE!

Perform the following steps to adjust:

- 0 - 2 l/min in steps of 0.1
- 2 - 4 l/min in steps of 0.2
- 4 - 6 l/min in steps of 0.5

- 3 The settings are accepted by pressing the adjustment regulator again.** The input is once again acknowledged by an audible signal.

NOTE!

	<i>The supplier may have restricted the available volume flow range to customized settings, such as 1 -4 lpm.</i>
--	---

 NOTE!

It may take up to 2 minutes until the actual volume flow corresponds with the setting. If the output is closed, the output pressure can increase up to 0.7bar. The maximum gas output temperature is maximum 6 degree higher than the ambient temperature.

 ATTENTION!

After selecting a new volume flow, all alarms and status messages are disabled for approximately one minute. Also already activated alarms are disabled, but might reoccur after the a.m. minute.

8.3 Status messages and alarms

8.3.1 Status messages

 NOTE!

Status messages are NOT accompanied with acoustical warning signals!

Operation

8.3.2 Status message

Status message / LC display	Description
Oxygen status alarm 	The oxygen concentrator oxy6000 is equipped with an innovative multi-function sensor to monitor the oxygen concentration of the oxygen output. If this drops below 82%, the oxygen status alarm is triggered. Countermeasures – Check humidifier and hoses for leaks – Inform the customer service.  NOTE! <i>According to the specifications of the oxy6000, the oxygen status alarm is an allowed situation and not an error state for volume flow setting of greater than 5.0 lpm.</i>

8.3.3 Alarm priorities

We differentiate between two alarm priorities:

Alarm priority	Description
Medium priority:	Quick countermeasures by the user are required.
Low priority:	Utmost attention of the user is required.

 NOTE!

The alarm priorities can audibly be differentiated by different alarm sound sequences. With a higher priority the number of alarm signals per unit of time increases.

8.3.4 Alarm categories

Alarm category / LC display	Description
Temperature alarm 	Medium priority The operating temperature inside the concentrator is too high. Countermeasures: – The unit should be switched off immediately. – Check whether the air flow into the unit is restricted. Also make sure that the unit has a sufficient clearance to other objects (wall, cupboard, etc.). – It may be necessary to replace the coarse dust filter in the back wall of the unit. Further information concerning this matter can be found in the section "Maintenance".  NOTE! <i>The oxygen supply is immediately stopped to protect the patient. However, the compressor keeps on running.</i>
System alarm 	Medium priority Micro-processor fault. Countermeasures – Switch off the unit immediately and inform the customer service.  NOTE! <i>Output and generation of oxygen is stopped for safety reasons.</i>

Operation

Alarm category / LC display	Description
Mains failure alarm  ERROR	Medium priority The power supply for the unit is interrupted. This causes immediate function failure of the oxy6000 Countermeasures The following should be checked: – Is the mains lead properly plugged into the socket? – Has a fuse tripped? Check the fuse, replace if necessary.  NOTE! <i>Further information concerning checking the equipment fuse of the oxy6000 can be found in section "Maintenance".</i> Note If a function test concerning the mains failure alarm is to be performed, you should proceed as follows: – Pull the mains plug out of the socket. – Switch on the unit. The alarm works, if it is activated during the switch-on self-test.  NOTE! <i>If the alarm is triggered again after switching on, you should inform the service department.</i>
Lack of oxygen alarm  ERROR	Low priority The oxygen concentrator oxy6000 is equipped with an innovative multi-function sensor to monitor the oxygen concentration of the oxygen output. If it is detected that the actual concentration does not match the technical specifications, the alarm is activated. . Countermeasures – Check humidifier and hoses for leaks – Inform the customer service.

Alarm category / LC display	Description
Sensor alarm 	Low priority There is a malfunction of the multi-function sensor, quantity and concentration of oxygen cannot be determined with sufficient accuracy. Countermeasures – Inform the customer service. – Search for kinks of the oxygen supply tubing.
Volume flow alarm 	Low priority The actual volume flow does not match the setting. Countermeasures – Check whether the oxygen hose is buckled or squeezed. – Check whether the accessories are correctly connected. – Switch from longer tubing lengths to shorter tubing lengths.  NOTE! <i>If the volume flow output is too high, the output and generation of oxygen is temporarily interrupted for safety reasons.</i>

WARNING! Risk of health damage!

Before resuming operation of the unit make sure that fault and cause of fault have been rectified professionally.

Maintenance

9 Maintenance

9.1 Safety

WARNING! Danger of electric current!

Before starting cleaning the unit must be switched off and disconnected from the mains supply.

9.2 General notes

Cleanliness is a prerequisite for the success of an oxygen therapy at home. The specified cleaning intervals must therefore strictly adhered to!

9.2.1 Cleaning

- The unit should be cleaned with a damp (not wet) cloth, so that not fluid can enter.
- You should only use commercial cleansing agents (e.g. washing-up liquid).
- Aggressive cleansers must not be used under any condition!

9.2.2 Disinfecting

- Any commercial disinfectant can be used for disinfecting. An up-to-date list is available from the manufacturer.
- The information for use issued by the disinfectant manufacturer must be strictly complied with.

9.3 Maintenance plan

Maintenance and cleaning work must be performed at regular intervals as specified in the table below.

Interval	Cleaning work
daily	The humidifier must be cleaned and disinfected every day.
daily (in case of stationary or ambulant use)	Nasal cannula should be disposed of and replaced every day.
daily (when used in homes or for nursing care)	Clean the nasal cannula .
weekly	If the " Set for external humidification " is used, it must be cleaned and disinfected every week.
every 14 days earlier if necessary	The oxy6000 must be cleaned with a damp cloth and disinfected after.
every 4 weeks	Replace the coarse dust filter
every 4 weeks (when used in homes or for nursing care)	Replace the nasal cannula .
annually / after 5000 operating hours	Replace the air intake filter  NOTE! <i>With extremely dirty ambient air the filter must be changed earlier.</i>
with changing patients	The oxy6000 must be cleaned with a damp cloth and disinfected after.
with changing patients	Replace the humidifier .
with changing patients	Replace the coarse dust filter
with changing patients	Replace the air intake filter
with changing patients	If the " Set for external humidification " is used, the

Maintenance

Interval	Cleaning work
	bracket must be cleaned and disinfected.
with changing patients	Replace the nasal cannula.  WARNING! Health risk! In order to avoid cross infection, each user of the oxy6000 should wear his/her own nasal cannula.
after infections	After an infection a new nasal cannula should be used to prevent re-infection.
after service work	The oxy6000 must be cleaned with a damp cloth and disinfected after.
after service work	Nasal cannula should be disposed of and replaced by a new one.
after service work	Replace the coarse dust filter
after service work	Replace the air intake filter .

Interval	Inspection
annually	Safety inspection

9.4 Maintenance work

Maintenance work	Description
Cleaning the humidifier	1 Unscrew the humidifier from the angle connection 2 Unscrew the lid from the humidifier and pour out any water residues 3 Clean the humidifier with clear, warm water 4 Disinfect the humidifier as instructed 5 Fill fresh, distilled water into the humidifier 6 Turn the lid back on 7 Fasten the humidifier on the angle connection  NOTE! When changing patients the humidifier must be disposed of. Sterile water system If the oxy6000 is used with a sterile water system, the following must be observed: – Sterile water systems must not be cleaned and refilled. – The old water flask must be disposed of. – The information provided by the sterile water producer (package insert) must be observed!
Cleaning the nasal cannula	1 Disconnect the hose of the nasal cannula from the oxy6000 . 2 Clean the nasal cannula in warm soapsuds. You may alternatively use a weak acetic solution (10% vinegar, 90% water). 3 Rinse the nasal cannula with lots of clear water. 4 Let the nasal cannula dry in air. The nasal cannula may only be used again for the therapy after it has properly dried.

Maintenance

Maintenance work	Description
Replacing the coarse dust filter	1 Remove the coarse dust filter cover/bracket from the back of the oxy6000. 2 Remove the coarse dust filter and place a new filter into the bracket. 3 Reattach the coarse dust filter cover.
Replacing the air intake filter	1 Open the service flap. 2 Pull the old air intake filter with a slight twist off the filter off the filter holder. 3 Plug the new filter element on. 4 Close the service flap.
Checking the equipment fuse	WARNING! Danger of electric current! Before checking the fuse switch off the unit and pull the mains plug out of the socket. 1 Open the service flap. 2 Apply slight pressure and turn the fuse cover anti-clockwise. 3 Remove the fuse. 4 Inspect the fuse visually. 5 If necessary replace the fuse 6 Reinstall the fuse

10 Spare parts

ATTENTION!

The intended use of the equipment is only possible when using approved accessories. The use of accessories that have not been designed for use with this unit, can severely affect the performance of the unit.

The following article numbers should be used when ordering:

Article number	Designation
41-1201.EN	Instructions for use for oxy6000, english
41-7001	Angle connector for humidifier
41-7002	Set for external humidification
41-7003	Hose connection with cap nut
41-7004	Bracket for humidifier
41-7005	Humidifier, for refilling
80-1001	Nasal cannula, 2 m
80-1002	Nasal cannula, 5 m
41-7008	Coarse dust filter for oxy6000
41-7009	Air intake filter
41-7010	Bacteria filter
80-1014	Oxygen safety hose 15,0 m
80-1011	Oxygen safety hose 2,0 m
80-1015	Oxygen safety hose 0,33 m
80-1031	Fire stop

Explanation of symbols on the unit
11 Explanation of symbols on the unit

Symbol	Explanation
	Attention, observe instructions for use!
	Attention, observe instructions for use!
IP 21	Degree of protection for the ingress of liquids and small parts
	Applied part type BF
	Class II
CE 0197	Notified Body: TÜV Rheinland LGA Products GmbH
I / O	On / Off
	Do not smoke !
	No open fire !
	No oil or grease !
	Do not remove any covers!
	Do not dispose of in household waste !

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The translation of this text was performed by me, the translator Mamedov Timur Djavanshirovich. The requirements regarding the text of the translation (maximum correctness, literate statement) have been explained to me.

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I, Akimov Gleb Borisovich, Notary of the city of Moscow, certify the authenticity of signature, made by the translator Mamedov Timur Djavanshirovich in my presence. His identity is established.

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It is registered in the register under №
Fee collected: 100 rubles

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Stitched up, numbered and sealed (45) pages

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