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Hydraulic Temporary Restorative

■ RECOMMENDED INDICATION

To be used by a dental professional as a temporary filling material.

■ CONTRAINDICATION

Contraindicated for patients with known hypersensitivity to hydraulic temporary restoratives. In case of allergy, refer to physician

■ DIRECTIONS FOR USE

1. Dry the prepared cavity with a cotton pellet (do not desiccate)
2. Apply the necessary amount of MD-Temp into the cavity with the use of a suitable instrument
3. Let the patient bite down on the filling to check for interferences in case of Class I cavities. Otherwise, contour filling by adapting it to the cavity walls
4. Remove excess by wiping it off with a moist cotton pellet towards the margins of the cavity
5. As complete setting takes place after an hour, instruct the patient to avoid pressure or abrasion until such time
6. Removal of the filling may be done with the use of rotary instruments or any appropriate hand instrument

■ NOTES

- Close the container immediately after removal of the material required.
- Do not mix or contaminate with any other material

■ STORAGE

- Store at temperatures 10°C ~ 25°C (50°F ~ 77°F)

■ COMPONENTS

- 40g jar individual box
- Box size : 35 X 35 X 36 mm

■ SHELF LIFE

- 2 years from the manufactured date.

■ SYMBOLS

: Use By : Batch Code

: CE-mark and identification number of notified body,
product conforms to the essential requirements of the
medical device directive 93/42/EEC

: Authorized representative in the EU

: Attention, see instruction for use

: Manufacturer

Manufactured by META BIOMED CO., LTD.

: Head Office & Factory

#414-12, Maehwang-Dong, Cheongju City, Chungbuk 361-140, Korea

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