INSTRUCTION

For ophthalmic spectacle frames

1. **PURPOSE:** Ophthalmic spectacle frames made of metal, plastic, combined produced by "Andy Wolf Fashion GmbH", meant for holding the lenses in a correct fixed position over the eyes.

2. MAIN TECHNICAL CHARACTERISTICS AND CONSTRUCTION

Ophthalmic spectacle frames consist of a front, temples, nosepads, hinges. Hinges may be simple and spring-based.

Ophthalmic spectacle frames are characterized by 3 main geometric parameters: width (a), bridge width (b) and temple length to the curve (L). The sizes used for spectacle frames by "Andy Wolf Fashion GmbH" are given in the table:

Ophthalmic spectacle frames		Distance between the vertical tangents to the facet groove of the rim (a). (mm)	Minimum distance between the vertical tangents to the bottom of the facet groove of the rim (b). (mm)	Temple length from the hinge axis to the curve (L) (mm)	Temple mass without lenses (g)
Ophthalmic spectacle frames (metal)	men	$56 \pm 0,2$	$18 \pm 0,2$	$110 \pm 1,0$	$19,0 \pm 1,0$
	women	$54 \pm 0,2$	$16 \pm 0,2$	$90 \pm 1,0$	$19,0 \pm 1,0$
	unisex	$56 \pm 0,2$	$18 \pm 0,2$	$110 \pm 1,0$	$19,0 \pm 1,0$
	kids	$52 \pm 0,2$	$14 \pm 0,2$	$90 \pm 1,0$	$18,0 \pm 1,0$
Ophthalmic spectacle frames (plastic)	men	$56 \pm 0,2$	$18 \pm 0,2$	$110 \pm 1,0$	$20,5 \pm 1,0$
	women	$54 \pm 0,2$	$16 \pm 0,2$	$90 \pm 1,0$	$20,2 \pm 1,0$
	unisex	$56 \pm 0,2$	$18 \pm 0,2$	$110 \pm 1,0$	$20,4 \pm 1,0$
	kids	$52 \pm 0,2$	$14 \pm 0,2$	$90 \pm 1,0$	$20,0 \pm 1,0$
Ophthalmic spectacle frames (combined)	men	$56 \pm 0,2$	$18 \pm 0,2$	$110 \pm 1,0$	$19,9 \pm 1,0$
	women	$54 \pm 0,2$	$16 \pm 0,2$	$90 \pm 1,0$	$19,4 \pm 1,0$
	unisex	$56 \pm 0,2$	$18 \pm 0,2$	$110 \pm 1,0$	$19,8 \pm 1,0$
	kids	$52 \pm 0,2$	$14 \pm 0,2$	$90 \pm 1,0$	$19,0 \pm 1,0$

On the surface of the frames there may be no sharp edges or burrs, shells, scratches, bubbles, sagging solder visible to the naked eye.

Temples should rotate on the axis smoothly, without jamming.

Decorative trim (if any) is from nickel silver (N18 CUNI18ZN20), or crystals

Ophthalmic spectacle frames correspond to the standards: ISO EN 12870.

Ophthalmic spectacle frames are chosen according to the anthropometric parameters of the patient.

3. The following may be considered CONTRAINDICATIONS:

- individual intolerance to sight correction by means of spectacles;
- impossibility of sight correction by means of spectacles.

4. MATERIALS:

Frames are made of materials that do not cause irritation, allergic or toxic reactions during their exploitation upon contact with human skin.

Plastic frames are produced of cellulose acetate polymer.

Metal and combined frames are produced of nickel silver (N18 CUNI18ZN20) and cellulose acetate polymer, temple tips are produced of cellulose acetate polymer, nose pads are produced of silicone LSR50.

Transportation inserts are produced of acryl Polycryl Clear.

Materials comply with the requirements of biological safety.

5. MARKING:

Ophthalmic spectacle frames produced by Andy Wolf Fashion GmbH have marking on the temples.

On the right temple there is the name of the manufacturer, producer country.

On the left temple there are technical parameters: model code or name; color or coating code; horizontal width of the fitted lens (a); bridge width (b) and full temple length.

6. PREPARING THE ITEM FOR USE: The spectacle frame should not lose its original shape after fitting the lenses. The rims should be symmetrical. No dents, scratches or gaps between spectacle lenses and rims, nor any other assembling defects are allowed.

Spectacle lenses fixed in the frame should not move or fall out of the rims during normal spectacle use.

7. TERMS OF STORAGE AND USE:

- Do not apply significant mechanical efforts to the frame;
- Keep the frame in a case or any other package to protect it from mechanical damage;
- Full average term of use is no less than 3 years..
- Terms of storage: in heated storages under the environmental temperature from + 5°C to + 50°C, relative air moisture 80%.
- No impact of aggressive substances (acids, alkali) and organic solvents and their fumes is allowed during storage.

8. TERMS OF USE

Ophthalmic spectacle frames are used in all climate zones at any time of the day.

9. PRECAUTIONS FOR USE

While using ophthalmic spectacle frames it is necessary to exercise caution practising sports and activities involving sudden head movements, bends, jumps and bumps.

10. POSSIBLE SIDE EFFECTS

No side effects during use of medical product were detected.

11. INFORMATION ON THE POSSIBLE IMPACT OF THE USE ON ABILITY TO DRIVE VEHICLES, MACHINERY

Drivers with vision problems may drive vehicles and machinery using spectacle frames (in the manufacture of frames for corrective glasses).

12. STERILITY

Unsterile.

13. MAINTENANCE AND REPAIRS

Ophthalmic spectacle frames should be repaired in special laboratories licensed for production, maintenance and repairs.

14. WARRANTY OBLIGATIONS

Storage life: plastic and combined frames – 5 years, metal frames – 10 years.

Full storage life – not less than 3 years with temple bends not more than 30000.

15. TRANSPORTATION

Ophthalmic spectacle frames are transported by all kinds of transport in covered vehicles in accordance with the rules of transportation of goods applicable to this kind of transport.

16. DISPOSAL

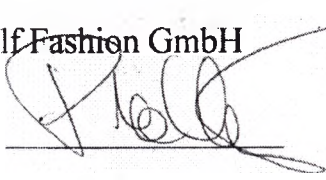
Ophthalmic spectacle frames should be disposed of as household waste.

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Certification Register No: 1769/15/bb

I herewith certify the authenticity and genuineness of the foregoing signature of Katharina Plattner, born on August 13, 1975 residing at Petschenweg 21, A-8114 Stübing, acting as managing director with individual signing power for, in the name and on behalf of **Andy Wolf Fashion GmbH** with principal place of business at Am Ökopark 21, A-8230 Hartberg. -----

After today's duly inspection of the commercial Register kept by electronic data processing I herewith certify pursuant to Section 89a Notaries Act that the individual above mentioned is acting in the name of **Andy Wolf Fashion GmbH** as above mentioned and that she as managing director is duly authorized to sign individually for, in the name and on behalf of **Andy Wolf Fashion GmbH** with its corporate seat in Hartberg, registered with the Commercial Register of the Commercial Court of Graz under Court File Number FN 282827 y. -----

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