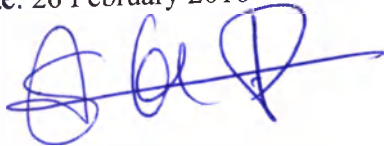
Approved by

Stanley DORVILLY
European Product registration Specialist

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M. Dorvilly

(seen exclusively to certify the above signature)

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Instructions for use

Sterimar Nasal Spray (for rinsing of nasal cavity)

SOFIBEL – Laboratoire Fumouze

110-114, rue Victor Hugo, 92686, Levallois Cedex France

2016

NAME OF THE MEDICAL DEVICE

Sterimar Nasal Spray:

- 1. Sterimar Isotonic For Adults (Sterimar Nasal hygiene), 100 ml, 50 ml.
- 2. Sterimar Isotonic for Children (Sterimar Baby Nasal hygiene), 100 ml, 50 ml.
- 3. Sterimar Hypertonic For Adults (Sterimar Blocked nose Nypertonic Cu Aduid), 100 ml, 50 ml.
- 4. Sterimar Hypertonic For Children (Sterimar Blocked nose Nypertonic baby/child), 100 ml, 50 ml.

Production sites:

1. SOFIBEL – Laboratoire Fumouze, 110-114, rue Victor Hugo, 92686, Levallois Cedex France.
2. Church & Dwight UK LTD, Wear Bay Road, Folkestone, Kent, CT 19 6PG, United Kingdom.

2. PURPOSE OF MEDICAL DEVICE:

2.1. Intended use

Sterimar Nasal Spray is a nasal decongestant. It represents isotonic or hypertonic solution of diluted natural seawater containing no preservatives, antihistamine agents, steroids and other additives. Upon spray preparation seawater undergoes the multistep purification process, including reverse osmosis filtration, UV treatment and ozonation, and is mixed with purified water to reach the intended osmotic concentration. Solution retains important minerals and microelements that are abundantly contained in seawater.

2.2. Indications

Sterimar Nasal Spray manufactured by SOFIBEL – Laboratoire Fumouze, France, is indicated for relief of nasal congestion and nasal mucosal edema in adults and children.

Otolaryngologists recommend Sterimar Nasal Spray for treatment of sinusitis, acute respiratory viral infections, allergic rhinitis, and for rinsing nasal passages to remove dust and crusts.

2.3. Contraindications

Sterimar Nasal Spray is safe for use by all family members (children in age of 3 months and elder), pregnant and for people for whom traditional medications are contraindicated. It can be used for gentle nasal humidification on everyday basis.

2.4. Adverse reactions

actions that may occur:

- Mucosal irritation (burning sensation, itching, tingling, pricking);
- Slight bleeding.

These reactions pass spontaneously without complications after stopping the treatment.

Mechanism of action

Mechanism of action: solution is contained in mini-bottle, the sprayer is fixed on the bottle. Then user pushes out the solution by pressing the nozzle. Spray feed can be easily dosed: the volume of spraying solution is always constant.

After spreading on the surface of mucosa spray causes several effects:

- ✓ Rinses nasal passages, removing dust, crusts, mucus, allergens and inflammatory mediators.
- ✓ Enhances mucociliary function and elimination of mucus.
- ✓ Has a softening effect in nasal inflammation (cold, nasopharyngitis, allergic rhinitis, sinusitis).
- ✓ Reduces bacterial growth.
- ✓ Limits the risk of superinfection.
- ✓ Optimizes the effect of basic therapy.

Description of mechanism of action

In case of catching a cold or when allergic reactions occurs the inflammations causes mucosal edema, nasal congestion and excessive mucus production. Hypertonic salty solution (seawater) is rinsing the mucosa and forms a pressure drop between solution and mucosal interstitium (osmotic effect), helping to reduce swelling of nasal passages and to ease breathing. Isotonic salty solution acts only as an agent for moistening and mechanical cleaning (rinsing) of allergens, mucus, dust etc. away from mucosa.

Mechanism of action of Sterimar Spray is also based on the following:

- Providing effective natural relief of nasal congestion when catching a cold, in rhinosinusitis and allergic rhinitis, by reducing swelling of mucosa
- Reducing pressure in nasal sinuses
- Moistening of nasal cavities

"Sterimar Nasal Cavity Spray (Sterimar)" manufactured by SOFIBEL – Laboratoire Fumouze, France, can also be used as additional medication in a combination with other treatment options. It helps to reduce the need of administration of decongestants.

2.4 Application

- Remove the protective cap.
- Ensure the tip is present.
- Hold the bottle under any angle and accurately insert the tip into one nostril.
- For moistening effect: push the tip for 1 -2 seconds.
- Blow your nose.
- For irrigation: tilt your head on one side, insert the nozzle, push and hold until the liquid begins to flow out from the nose.
- Use 2 -6 times per day (or more) as recommended by a doctor.

For hygienic purposes each bottle should be used only by one patient.

2.5 Conditions for use

Use at room temperature - 2 °C to 40 °C.

Spray can be used at home.

3. DESCRIPTION OF ACTION

3.1 Sterimar Isotonic for Adults (Sterimar Nasal hygiene), 100 ml, 50 ml.

Sterimar Isotonic Spray for Adults (Sterimar Nasal hygiene) is a fully natural isotonic spray, containing seawater. To provide a softening effect on mucosa the salt concentration is reduced to the level, which is natural for human body cells. It cleans nasal passages from allergens, dust and other air pollutants, and performs moistening effect. The spray reduces symptoms of allergic reactions, allergic rhinitis, acute viral infections, rhinitis and sinusitis. Sterimar Isotonic for Adults (Sterimar Nasal hygiene) is made of cleanest water from coast of Bretagne. It contains no preservatives and additives.

Sterimar Nasal Spray (Sterimar Nasal hygiene) is safe for use by all family members (children in age of 3 months and elder), pregnant and for people for whom traditional medications are contraindicated, on doctor's prescription. Spray has no adverse effects. It can be used for gentle nasal humidification on everyday basis.

Sterimar Isotonic for Adults (Sterimar Nasal hygiene) is recommended for treatment of sinusitis, acute respiratory viral infections, allergic rhinitis, and for rinsing nasal passages to remove dust and crusts.

Sterimar Isotonic for Children (Sterimar Baby Nasal hygiene), 100 ml, 50 ml.

Sterimar Isotonic for Children (Sterimar Baby Nasal hygiene) is a 100% natural isotonic solution that can effectively and softly clean the child's congested nose. The mechanism of action is based on cleaning and humidifying nasal passages for creation of effective protection.

Sterimar Isotonic for Children uses the microdiffusion system, which provides soft spraying of a seawater on the nasal mucosa. Tiny drops of seawater soften and dissolve nasal discharge. It helps to clean out nasal passages.

Sterimar Isotonic for Children (Sterimar Baby Nasal hygiene) is a nasal spray for kids, containing seawater. It is safe for children of 0-3 years.

Sterimar Isotonic for Children (Sterimar Baby Nasal hygiene) contains no antihistamine agents, steroids, preservatives and other additives.

3 Sterimar Hypertonic for Adults (Sterimar Blocked nose Nypertonic Cu Aduid), 100 ml, 50

Sterimar Hypertonic for Adults (Sterimar Blocked nose Nypertonic Cu Aduid)) is a fully natural solution for treatment of diseases of the nose.

Sterimar Hypertonic for Adults (Sterimar Blocked nose Nypertonic Cu Aduid) is made of cleanest water from coast of Bretagne, water purity is confirmed by testing. **Sterimar Hypertonic for Adults (Sterimar Blocked nose Nypertonic Cu Aduid)** is a hypertonic solution with a slightly elevated salty content when comparing with human cells, which provides natural and soft decongestant effect.

4 Sterimar Hypertonic for Children (Sterimar Blocked nose Nypertonic baby/child), 100 ml, 10 ml.

Sterimar Hypertonic for Children (Sterimar Blocked nose Nypertonic baby/child) is a medical device. It is recommended for use in children since 3 month:

- ✓ Has a softening effect in nasal inflammation (cold, nasopharyngitis, allergic rhinitis, sinusitis).
- ✓ Reduces bacterial growth.
- ✓ Limits the risk of superinfection.
- ✓ Optimizes the effect of basic therapy.

Sterimar Hypertonic for Children (Sterimar Blocked nose Nypertonic baby/child) was developed specially for use in infants from 3 months and elder kids: salt content in a hypertonic solution and pressure upon spraying are designed specially for children nasal mucosa.

Spray is packed into aluminum bottle with nozzle spray and lacquered coating inside. Outer package is carton box. Instructions for use are embedded in a box. Mini bottle (bottle) contains 50 ml (about 150 doses) or 100 ml of spray. The device is not sterile.

Materials for manufacturing the medical device.

Name of the device:	Material	Contact with patient's body	Sterility
1. Sterimar Isotonic for Adults (Sterimar Nasal hygiene), Sterimar Isotonic for Children (Sterimar Baby Nasal hygiene)	Diluted natural seawater, 31.82 ml, Purifies water 100 ml. Chlorides – 572, 76 mg Sodium – 350,02 mg Sulfate – 82, 732 mg Magnesium – 41, 366 mg Calcium – 12, 728 mg Potassium – 11, 137 mg Brome – 1,9092 mg Strontium – 0,25456 mg	Device with a short contact with nasal mucosa	Non-sterile
2. Sterimar Hypertonic for Adults (Sterimar Blocked nose Nypertonic Cu Aduid)	Diluted natural seawater, 75 ml, Purifies water 100 ml. Chlorides – 1350 mg Sodium – 825 mg Sulfate – 195 mg Magnesium – 97,5 mg Calcium – 30 mg Potassium – 26,25 mg Brome – 4,5 mg Strontium – 0,6 mg	Device with a short contact with nasal mucosa	Non-sterile
3. Sterimar Hypertonic for Children (Sterimar Blocked nose Nypertonic baby/child)	Diluted natural seawater, 60 ml, Purifies water 100 ml. Chlorides – 1080 mg Sodium – 660 mg Sulfate – 156 mg Magnesium – 78 mg Calcium – 24 mg Potassium – 21 mg Brome – 3,6 mg Strontium – 0,48 mg	Device with a short contact with nasal mucosa	Non-sterile
2. Bottle, 100 ml., 50 ml.	Aluminium 1050AC (purity > 99,5%) with coated inner layer – interior lacquer (epoxyphenol J3121)	Device with a short contact with intact skin on fingers. It represents the primary package of Sterimar Nasal Spray	Non-sterile
3. Valve pack	Pocket – foil PETP 12/ALU 8/OPA 15/PP 75 Spring plate – Polypropylene	Device that does not contact with patient. Included	Non-sterile

Name of the device:	Material	Contact with patient's body	Sterility
	HJ325MO Spring – stainless steel AISI 316 C: ≤0,08% Cr: 16-18% Si: ≤0,75% Mn: ≤0,2% S: ≤0,03% P: ≤0,045% Fe: 80,9-82,9% Plunger – polyoxymethylene Kepital F25-03 Coupling (mounting ring) – Aluminum External gasket – Butyl Rubber IIR Inner gasket – Buna-nitrile DB219	in the primary packaging of Sterimar nasal Spray	
4. Spray Head	Polyethylene MG9641	Device with a short contact with intact skin on fingers.	Non-sterile
5. Tip	Polypropylene PP BRASKEM Random 7051 –	Device with a short contact with intact skin on fingers.	Non-sterile

SAFETY REQUIREMENTS

Following requirements should be met when using the device:

- Avoid contact with eyes.
- Not for oral administration
- Container under pressure: keep away from sun light, do not use at temperature exceeding 50°C.
- Do not pierce even after use.
- Do not use longer than 10 or 15 days in a row, if other is not recommended by a doctor.
- After each use clean the tip with soapy water, rinse and dry.
- If your child has a nose injury or if a surgery of nasal passages was carried out shortly before the use of the medication, doctor's consultation is to be performed.

5. Step-by-step instruction

For use of Sterimar Isotonic for Adults and Sterimar Hypertonic for Adults:

- Remove the protective cap.
- Ensure the tip is present.
- Hold the bottle under any angle and accurately insert the tip into one nostril.
- For moistening effect: push the tip for 1 -2 seconds.
- Gently blow your nose.
- For irrigation: tilt your head on one side, insert the nozzle, push and hold until the liquid begins to flow out from the nose.

Use 2-6 times per day (or more) as recommended by a doctor.

For use of Sterimar Isotonic for Children and Sterimar Hypertonic for Children:

Sterimar for Children is convenient for use under any angle. Design of a tip enables its use in the smallest nostrils.

- Prepare spray by shortly pressing the tip.
- Put a child on a side of place him tilting the head to the side.
- Gently insert the nozzle into the nostril.
- Shortly press the tip - wipe off excessive liquid.
- Repeat with another nostril.
- Clean the nozzle with warm soapy water.

Use as often as necessary, 2 to 6 times per day (or more), as recommended by a doctor.

ENVIRONMENT PROTECTION REQUIREMENTS

Upon use, transportation and storage **Sterimar Nasal Spray** does not perform negative effects on human and environment.

7. DISINFECTION AND STERILITY

Device is non-sterile.

For disinfection it is recommended to clean and rinse the tip (spray nozzle) with soap and water and dry naturally after each use.

8. TRANSPORTATION AND STORAGE

Device can be transported by any kind of covered transport in accordance with

Transportation of the device should be performed using any kind of covered transport in accordance with actual rules for the current type of transport.

Device transportation conditions:

- Ambient temperature 2°C to 30 °C;
- Relative humidity 10% to 85%, no condensation;
- Atmosphere pressure not regulated.

Storage conditions:

- Ambient temperature 2 °C to 50 °C;
- Relative humidity 10% to 85%, no condensation;
- Atmosphere pressure not regulated.

Keep away from exposure to liquids, direct heat and light. Do not use after expiry date.

9. PACKAGING AND LABELLING

Sterimar Nasal Spray is intended for individual use. It is supplied packaged into cardboard box.

Symbol	Definition
	Manufacturer
	Use before
	Upper limit of temperature range
	CE mark – sign of conformity with the requirements of Directive 93/42/EEC and ID of EU accredited certification Body.

LOT

Lot number

10. REFUSE AND DISPOSAL. SHELF LIFE.

Refuse and disposal of the device and its waste products are defined in national and regional regulatory rules.

Expired or used medical device is to be disposed in accordance with protocols of a medical facility, and with requirements of local regulations.

Device's shelf life is 3 years since the date of manufacture printed on the packaging.

11. WARRANTY

Manufacturer guarantees that the device is safe if the requirements of at attached instructions on transportation, storage and application were met during the whole shelf life.

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