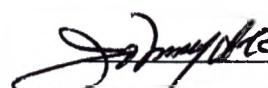
Managing Director

"Valentine International Ltd."

 Johnny Chiu /
«Apr.» 27, 2016r.

User manual

Wheelchairs Barry for Disabled with Additional Executions

. INTRODUCTION

Dear Customer,

Thank you for the confidence you have placed in us by purchasing the wheelchairs Barry for disabled.

Each wheelchairs Barry for disabled is tested by the manufacturer for electrical safety and functionality, and leaves the factory in the best possible condition.

This manual contains information about the functions necessary for everyday use of the wheelchairs Barry for disabled. You should have this manual constantly at hand, and regularly consult it as a practical handbook.

"Valentine International Ltd."

DESCRIPTION AND OPERATION

Wheelchairs Barry are designed only for transporting patients and people with disabilities of the musculoskeletal system and can be propelled by the sitter or by a companion. Only those who physically and mentally are capable to properly operate (set in motion, control, brake) the manual wheelchair Barry are allowed to use it unaided. Your wheelchair Barry should correspond to your health condition.

2.1. Purpose of the product


Wheelchairs Barry for disabled with additional executions to move adults with disabilities musculoskeletal with the possibility of installation of sanitary equipment.

2.2. Description of the product composition

1. Wheelchair

The wheelchair is delivered pre-assembled.

The wheelchair consists of various parts. In the diagram marked only its basic elements.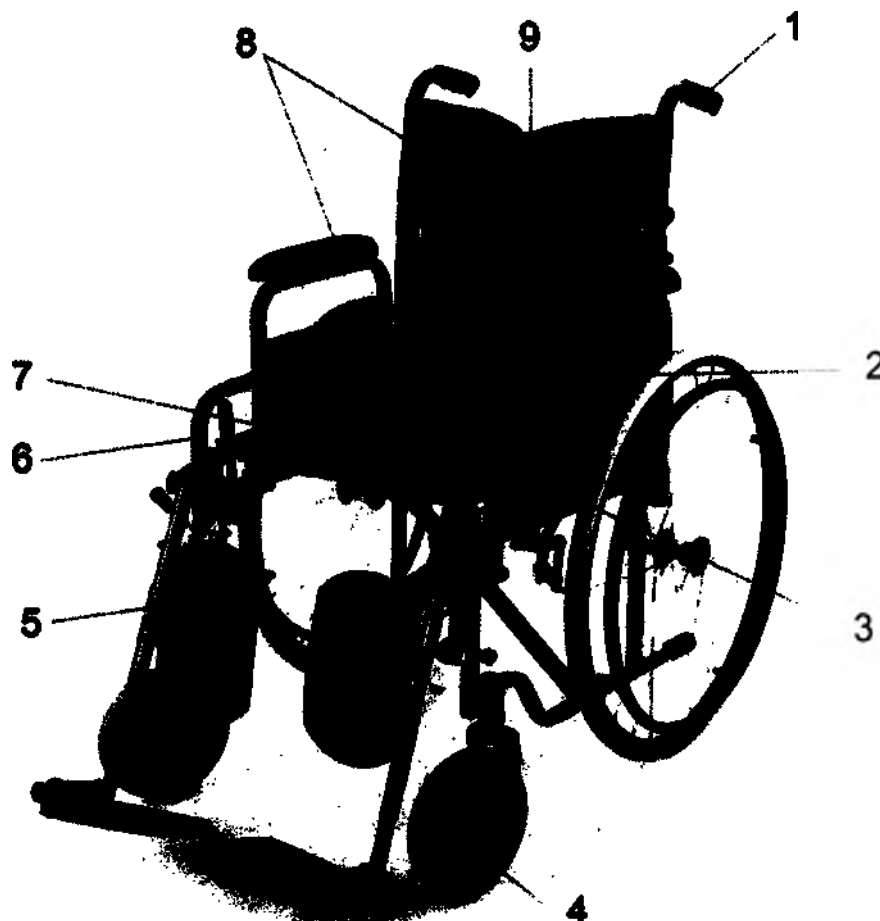

1—Push handles

2—Rear wheel (driving wheel)

3—Brake

4—Front wheel (castor)

5—Footrests

6—Frame

7—Seat

8—Armrests


9—Backrest

- **Push handles** are used to move the wheelchair with a man. Push handle anatomical feature packed with anti-slip material- polypropylene (grade: 1500J) for the convenience of an accompanying person. Push handle not height adjustable. Handles for pushing is not height adjustable.
- **Rear wheel (driving wheel)** it consists of a wheel, and the axis of the hoops. The rear wheels provide stability and allow strollers putting it into motion by means of hoops. Material: polyurethane (grade: PMC-790).
- **Brake** used to fix the standing wheelchair. The brake is applied at long stops wheelchair. The brake is equipped with anatomic attachment of anti-slip material - polypropylene (grade: 1500J).
- **Front wheel (castor)** it consists of a wheel and a fork. The front wheels provide contact with the front of the carriage floor and direct the movement of the carriage by repositioning the plug.
Material: polyurethane (grade: PMC-790).
- **Footrest** it consists of an adjustable strut and the foot panel supporting feet.
Material: polypropylene (grade: 1500J).
- **Frame** it is a load-bearing structure of the wheelchair, which consists of the side frame members, and folding system, including elements of the seat and backrest.
Frame Material: Carbon steel with corrosion-resistant coating of chromium (make: Q 235 C). In embodiments (Barry W1 (art. 5019 W2), Barry W24 (art. 5019 W24), Barry W2 (art. 5019 W2P), Barry W2 SF (art. 5019C0102SFR) frame is painted with paint «Ningbo Honesty Electric Technical» green (green pigment - chrome green lead).
- **Seat** is removable, easy to clean, thermally and chemically inert, resistant to urine, blood, ointments, oils, greases and all disinfectants. On the wheelchair seat is standard, if desired client has the ability to install different sized seats (seats with different sizes are presented in an attachment).
Material: nylon 100% (grade: Oxford 1680D, dye iDye Poly (Black)).
- **Armrest** wheelchairs fitted with a non-removable armrests type it provides the entire unit solidity, long life and stability. The armrests have a stepped form, which are characterized by high maneuverability and allows the user to easily drive up to the various interior items.
It is forbidden to lift the arms of the wheelchair!
Material: nylon 100% (grade: Oxford 1680D, dye iDye Poly (Black)).
- **Backrest** is removable, easy to clean, thermally and chemically inert, resistant to urine, blood, ointments, oils, greases and all disinfectants.
On the wheelchair set the standard back, if desired client has the ability to install a universal back (represented in the attachment) with adjustable width.
Material: nylon 100% (grade: Oxford 1680D, dye iDye Poly (Black)).

2. Operating Manual (hard copy, in Russian).

Accessories:

1. Seat width 36 cm (art. 001)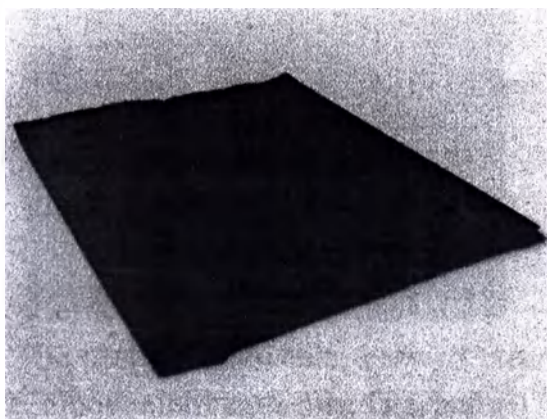


The seat is removable, easy to clean, thermally and chemically inert, and resistant to urine, blood, ointments, oils, fats, and all disinfectants.

If desired, the customer has the ability to install seat width 36 cm seat instead of the included standard wheelchairs.

Seat material: nylon 100% (grade: Oxford 1680D, dye iDye Poly (Black)).

The size of the seat width 36 cm (W×D): 36 × 40 cm; Weight: 0.47 kg.

2. Seat width 38 cm (art. 001)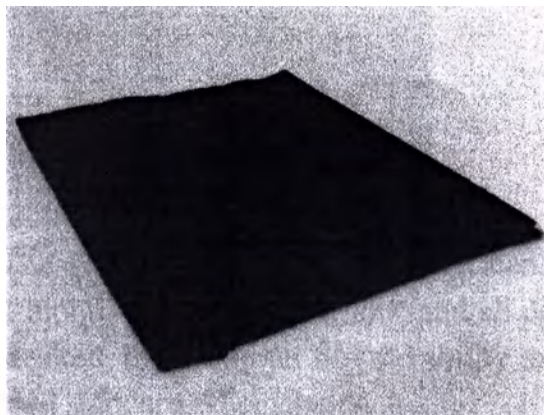


The seat is removable, easy to clean, thermally and chemically inert, and resistant to urine, blood, ointments, oils, fats, and all disinfectants.

If desired, the customer has the ability to install seat width 38 cm seat instead of the included standard wheelchairs.

Seat material: nylon 100% (grade: Oxford 1680D, dye iDye Poly (Black)).

The size of the seat width 38 cm (W×D): 38 × 40 cm; Weight: 0.47 kg.

Seat 41 cm (art. 001)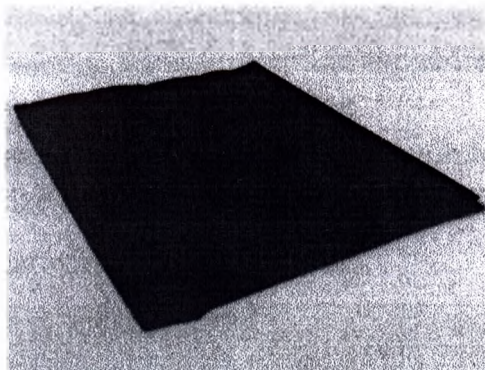


The seat is removable, easy to clean, thermally and chemically inert, and resistant to urine, blood, ointments, oils, fats, and all disinfectants.

If desired, the customer has the ability to install seat width 41 cm seat instead of the included standard wheelchairs.

Seat material: nylon 100% (grade: Oxford 1680D, dye iDye Poly (Black))

The size of the seat width 41 cm (W×D): 41 × 40 cm; Weight: 0.47 kg.

4. Seat width 43 cm (art. 001)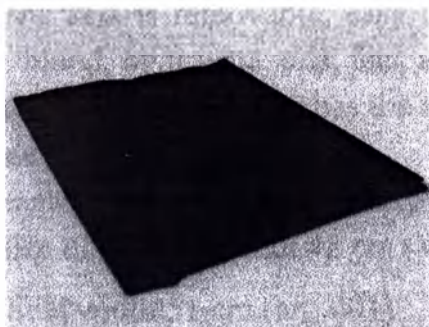


The seat is removable, easy to clean, thermally and chemically inert, and resistant to urine, blood, ointments, oils, fats, and all disinfectants.

If desired, the customer has the ability to install seat width 43 cm seat instead of the included standard wheelchairs.

Seat material: nylon 100% (grade: Oxford 1680D, dye iDye Poly (Black))

The size of the seat width 43 cm (W×D): 43 × 40 cm; Weight: 0.47 kg.

5. Seat width 46 cm (art. 001)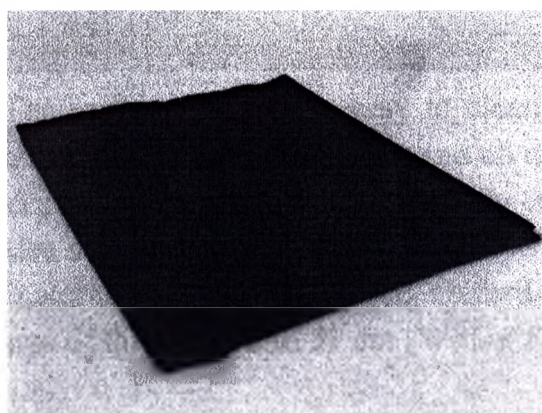


The seat width is removable, easy to clean, thermally and chemically inert, and resistant to urine, blood, ointments, oils, fats, and all disinfectants.

If desired, the customer has the ability to install seat width 46 cm seat instead of the included standard wheelchairs.

Seat material: nylon 100% (grade: Oxford 1680D, dye iDye Poly (Black))

The size of the seat width 46 cm (W×D): 46 × 40 cm; Weight: 0.47 kg.

6. Seat width 48 cm (art. 001)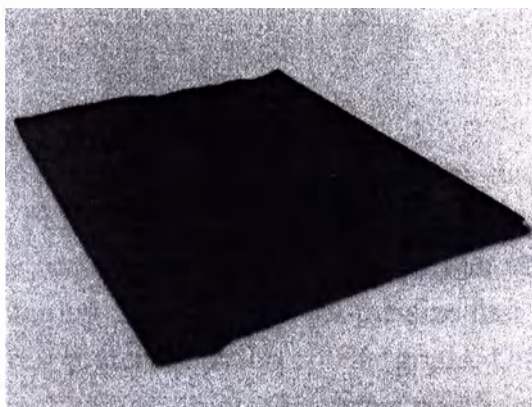


The seat is removable, easy to clean, thermally and chemically inert, and resistant to urine, blood, ointments, oils, fats, and all disinfectants.

If desired, the customer has the ability to install seat width 48 cm seat instead of the included standard wheelchairs.

Seat material: nylon 100% (grade: Oxford 1680D, dye iDye Poly (Black))

The size of the seat width 48 cm (W×D): 48 × 40 cm; Weight: 0.47 kg.

7. Seat width 51 cm (art. 001)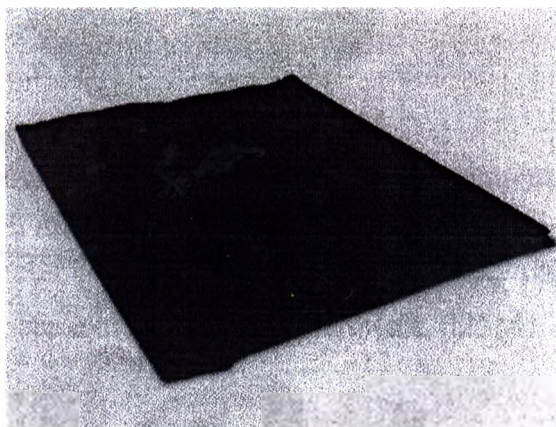


The seat is removable, easy to clean, thermally and chemically inert, and resistant to urine, blood, ointments, oils, fats, and all disinfectants.

If desired, the customer has the ability to install seat width 51 cm seat instead of the included standard wheelchairs.

Seat material: nylon 100% (grade: Oxford 1680D, dye iDye Poly (Black))

The size of the seat width 51 cm (W×D): 51×40 cm; Weight: 0.47 kg.

Backrest (art. 002)

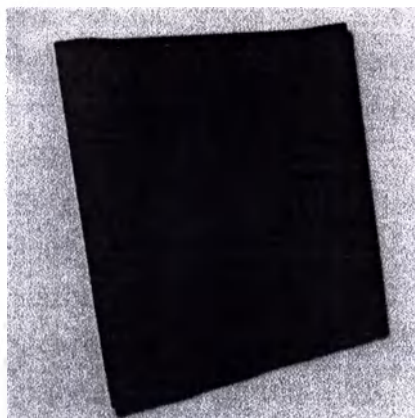The backrest is removable, easy to clean, thermally and chemically inert, and resistant to urine, blood, ointments, oils, fats, and all disinfectants.

The back is universal, because under changes the width of the seat it is possible by means of fastening velcro-adjust for the size that is needed.

Backrest material: nylon 100% (grade: Oxford 1680D, dye iDye Poly (Black)).

Backrest height: 46 cm; Backrest width (min-max): 34–70 cm; Weight: 0.51 kg.

9. Front wheel without fork (art. 016)

Front wheel without fork it is a replacement part for the wheelchair.

Wheel material: polyurethane (grade: PMC-790).

Diameter of front wheel without fork: 20 cm; Rim diameter: 10 cm; Weight: 0.8 kg.

10. Spare extra wheel magnum (art. 025M)

Spare extra wheel magnum with a hub for a quick-release axle is fitted with spokes and a hoop for manual propulsion. The wheel has rubber tires without inner tubes (cast tires).

If desired, the client can be installed to replace the wheel magnum wheels included In standard complete set of the wheelchair.

Material:

Spokes: carbon steel (grade: ML45) with corrosion resistant chrome plating.

Hub: carbon steel (grade: ML45) with corrosion resistant chrome plating.


Rim: carbon steel (grade: ML45) with corrosion resistant chrome plating.

Hoop: carbon steel (grade: ML45) with corrosion resistant chrome plating.

Tire: polyurethane (grade: PMC-790).

Spare extra wheel magnum diameter: 60 cm; Rim diameter: 54 cm; Weight: 3 kg.

11. Footrest left (art. 047L)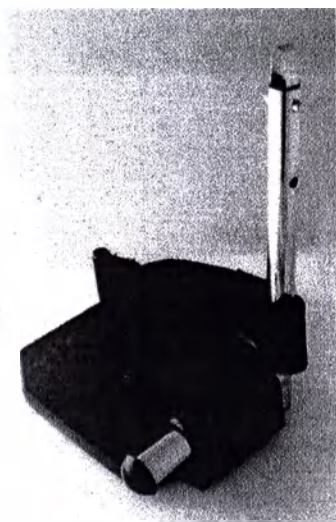


The footrest left consists of an adjustable stand and a foot panel to support the foot.

Footrest left it is a replacement part for the wheelchair.

Footrest left material: polypropylene (1500J).



Footrest left height: 17 cm; Weight: 0.6 kg.

12. Footrest right (art. 047R)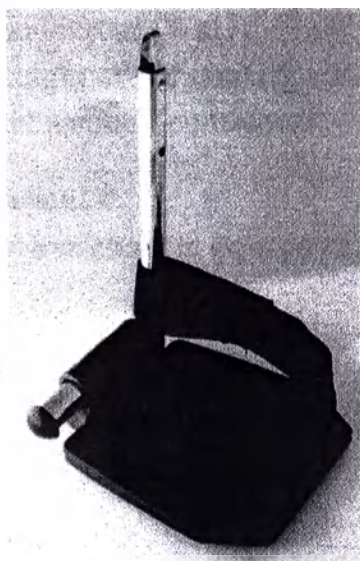

The footrest right consists of an adjustable stand and a foot panel to support the foot.
Footrest right it is a replacement part for the wheelchair.
Footrest right material: polypropylene (1500J).
Footrest right height: 17 cm; Weight: 0.6 kg.

13. Sanitary equipment Potty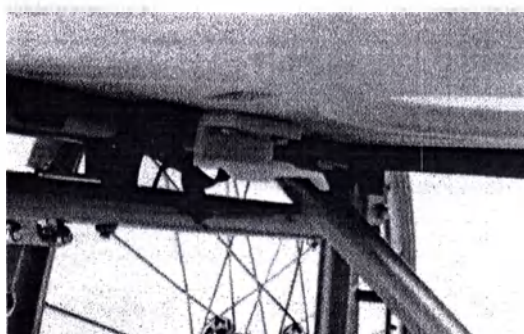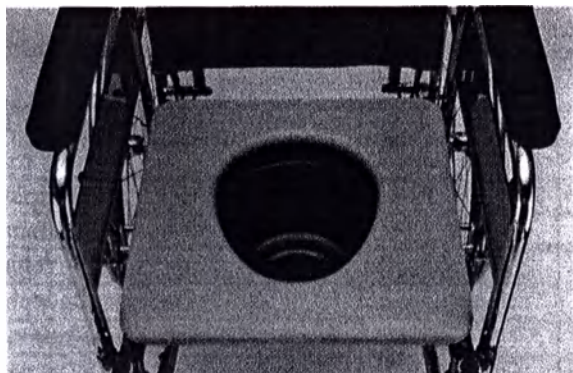

According to the patient's wishes sanitary equipment can be mounted on the wheelchair seat instead. Sanitary equipment and securely attached to the frame via slots. Sanitary equipment consists of a sanitary toilet seat and tank. To the toilet seat fixed slots for fixing the sanitary container. Sanitary container can be extracted for hygiene procedures. Sanitary container and commode wheelchairs removable made of washable material - polypropylene (1500J).

Weight sanitary equipment Potty: 2 kg

Size toilet seat (WxD): 51x40 cm

The height of the sanitary container: 17 cm.; The diameter of the sanitary container: 21 cm.;

Weight sanitary containers: 0.6 kg.

Attention!!!

The wheelchair is equipped with sanitary equipment only on request as an accessory.

Technical Description

Wheelchairs Barry for disabled with additional executions manufactured and released without tolerances, according to the specifications given in Tables 1-3.

Technical description of wheelchairs Barry for disabled

Table 1

Model/Specifications	Barry W1 (art. 5019 W2)	Barry W24 (art. 5019 W24)	Barry W2 (art. 5019W2P)	Barry W2 SF (art. 5019C0102SFR)
Weight, kg	17	17	18	17,5
Folding	yes	yes	yes	yes
Dimensions (W × L × H), cm	60 × 98 × 87	60 × 98 × 87	61 × 100 × 88	60 × 98 × 87
Seat dimensions (W × D), cm	46 × 40	46 × 40	46 × 40	46 × 40
Seat angle	Non-adjustable	Non-adjustable	Non-adjustable	Non-adjustable
Seat height adjustment	Non-adjustable	Non-adjustable	Non-adjustable	Non-adjustable
Floor-to-seat height, cm	51	51	50	50
Backrest	Folding	Folding	Folding	Folding
Backrest angle	Non-adjustable	Non-adjustable	Non-adjustable	Non-adjustable
Backrest height, cm	46	46	46	46
Footrests	Fixed, height adjustable, with shank support	Fixed, height adjustable, with shank support	Removable, folding, height adjustable, with shank support	Removable, height adjustable, with shank support
Footrest length, cm	30–40	30–40	30–40	44–54
Armrests	Fixed	Fixed	Removable	Removable
Armrest height, cm	28	28	24	24
Type of rear wheels	Solid cast, made of polyurethane, w/protector	Pneumatic, made of polyurethane, w/protector	Solid cast, made of polyurethane, w/protector	Solid cast, made of polyurethane, w/protector
Rear wheel diameter, cm	60	60	60	60
The diameter of the disc of the rear wheel, cm	54	54	54	54
Type of front wheels	Cast	Cast	Cast	Cast
Front wheel diameter, cm	20	20	20	20
The diameter of the front wheel drive, cm	10	10	10	10
Upholstery material	Nylon	Nylon	Nylon	Nylon
Maximum load, kg	100	100	100	100
Minimum turn width, cm	100	100	100	100
Minimum turn radius, cm	65	65	65	65
Folding (Unfolding) force, H	48	48	48	48
Backrest fixing force, H	48	48	48	48
Length and footrest angle adjustment force, H	35	35	35	35
Maximum clearance between surface and wheel, cm	0.2	0.2	0.2	0.2
Parking brake handle force, H	48	48	48	48
Force to the handles back, H	70	70	70	70
Force to the wheel rim, H	35	35	35	35
The force moving the wheelchair with a rated load on the seat, H	40	40	40	40
Age group	For Adults	For Adults	For Adults	For Adults

Technical description of wheelchairs Barry for disabled

Table 2

Model/Specifications	Barry W3 (art. 5019C0103SF)	Barry W4	Barry W5
Weight, kg	18	19	19
Folding	yes	yes	yes
Dimensions (W × L × H), cm	62 × 100 × 92	65 × 107 × 97	65,5 × 107 × 98
Seat dimensions (W × D), cm	46 × 40	46 × 40	46 × 40
Seat angle	Non-adjustable	Non-adjustable	Non-adjustable
Seat height adjustment	Non-adjustable	Non-adjustable	Non-adjustable
Floor-to-seat height, cm	50	50	51
Backrest	Folding	Folding	Folding
Backrest angle	Non-adjustable	Non-adjustable	Non-adjustable
Backrest height, cm	46	46	46
Footrests	Removable, folding, height adjustable, with heel supports	Removable, height adjustable, with shank support	Removable, height adjustable, with shank support
Footrest length, cm	30–40	34–49	43–50
Armrests	Removable	Removable Folding	Removable Folding
Armrest height, cm	24	26	27
Type of rear wheels	Pneumatic, made of polyurethane, w/protector	Solid cast, made of polyurethane, w/protector	Cast
Rear wheel diameter, cm	60	60	60
The diameter of the disc of the rear wheel, cm	54	54	54
Type of front wheels	Cast	Cast	Cast
Front wheel diameter, cm	20	20	20
The diameter of the front wheel drive, cm	10	10	10
Upholstery material	Nylon	Nylon	Nylon
Maximum load, kg	100	100	100
Minimum turn width, cm	100	100	100
Minimum turn radius, cm	65	65	65
Folding (Unfolding) force, H	48	48	48
Backrest fixing force, H	48	48	48
Length and footrest angle adjustment force, H	35	35	35
Maximum clearance between surface and wheel, cm	0.2	0.2	0.2
Parking brake handle force, H	48	48	48
Force to the handles back, H	70	70	70
Force to the wheel rim, H	35	35	35
The force moving the wheelchair with a rated load on the seat, H	40	40	40
Age group	For Adults	For Adults	For Adults

Technical description of wheelchairs Barry for disabled

Table 3

Model/Specifications	Barry W6	Barry W7	Barry W8
Weight, kg	18	18	16,5
Folding	yes	yes	yes
Dimensions (W × L × H), cm	65 × 105 × 89	65 × 110 × 89	60 × 98 × 85
Seat dimensions (W × D), cm	46 × 40	46 × 40	46 × 40
Seat angle	Non-adjustable	Non-adjustable	Non-adjustable
Seat height adjustment	Non-adjustable	Non-adjustable	Non-adjustable
Floor-to-seat height, cm	51	51	49
Backrest	Folding	Folding	Folding
Backrest angle	Non-adjustable	Non-adjustable	Non-adjustable
Backrest height, cm	46	46	46
Footrests	Removable, height adjustable and angle, with shank support	Removable, folding, height adjustable and angle, with heel supports	Removable, folding, height adjustable, with heel supports
Footrest length, cm	43–50	43–50	36–46
Armrests	Removable Folding	Removable Folding	Removable Folding
Armrest height, cm	25	25	23
Type of rear wheels	Cast	Solid cast, made of polyurethane, w/protector	Solid cast, made of polyurethane, w/protector
Rear wheel diameter, cm	60	60	60
The diameter of the disc of the rear wheel, cm	54	54	54
Type of front wheels	Cast	Cast	Cast
Front wheel diameter, cm	20	20	20
The diameter of the front wheel drive, cm	10	15	10
Upholstery material	Nylon	Nylon	Nylon
Maximum load, kg	120	120	120
Minimum turn width, cm	100	100	100
Minimum turn radius, cm	65	65	65
Folding (Unfolding) force, H	48	48	48
Backrest fixing force, H	48	48	48
Length and footrest angle adjustment force, H	35	35	35
Maximum clearance between surface and wheel, cm	0.2	0.2	0.2
Parking brake handle force, H	48	48	48
Force to the handles back, H	70	70	70
Force to the wheel rim, H	35	35	35
The force moving the wheelchair with a rated load on the seat, H	40	40	40
Age group	For Adults	For Adults	For Adults

INTENDED USE

Wheelchairs Barry for disabled with additional executions to move adults with disabilities musculoskeletal with the possibility of installation of sanitary equipment.

3.1. Indications

- Paraplegia;
- Meningomyelitis;
- Progressive muscular dystrophy;
- Infantile cerebral palsy;
- Multiple sclerosis;
- Hemiplegia of the central body part;
- Post-traumatic brain damage;
- Moderate limb deformities;
- Dysmelia;
- Phocomelia;
- Significant osteogenesis;
- Spasms, minor joint damage;
- Polyarthrosis;
- Polyarthritis, ankylosing spondylitis;

3.2. Contraindications

- Disturbance of mental functions with significant judgment and intelligence decline, frequent epileptiform fits, and significant disorientation; significant vision disorders.
- Drug and alcohol dependence
- As indicated by the doctor

3.3. Possible side effects

Side effects, when using wheelchairs Barry for disabled, do not exist.

3.4. Preparation for use


Wheelchairs mechanical Barry with accessories by design are complex technological devices, the correct use of which depends not only on the physical condition, but also the safety of the patient as a whole. Therefore it is necessary to read the instruction manual carefully and comply with even the most basic safety rules.

3.5. Using the Product

- Carefully unpack and remove the wheelchair Barry from the package. If necessary, the package can be reused for subsequent storage and transportation.
- After unpacking, carefully inspect to ensure the manual wheelchair Barry is in a good condition.

1. Unfolding the Wheelchair

- **With one hand, take the armrest and the seat element on the same side of the wheelchair and slightly tilt the wheelchair toward yourself (so that the wheels on the opposite side lift off the floor).**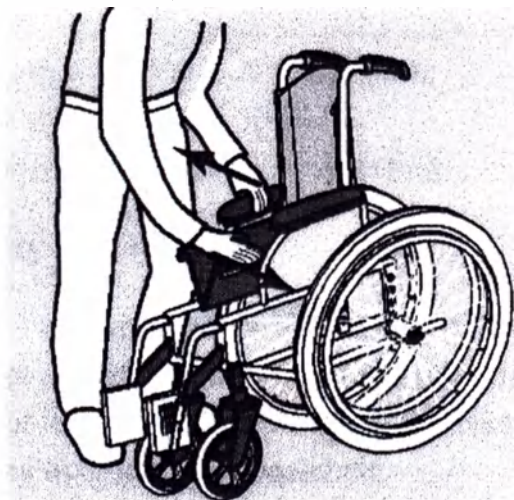

- Press on the seat cover until the element supporting the seat unfolds to its entire length. The seat should be fully extended.
- Apply both parking brakes of the wheelchair.

3.5.2. Setting the Wheelchair in Motion


The wheelchair can be set in motion by turning the hoops mounted on the wheels or when it is pushed by a companion.

The parking brake is used to secure the position of the standing wheelchair. The parking brakes are located on both sides of the device. They are used for prolonged stops of the wheelchair.

3.6. Folding the Wheelchair

Hold the two edges of the seat (front and rear) and pull them up.

To avoid the folding mechanism damaging the upholstery, fold the wheelchair by pulling the seat up.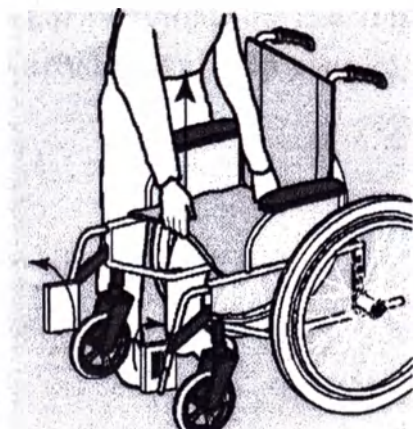

Conditions of Use

- **Wheelchairs Barry** for disabled may be used only on hard and flat surfaces.
- **Wheelchairs Barry** for disabled are suitable for multiple uses. When the wheelchair is used by another patient, it shall be wiped or sanitized and inspected.

During the operation must be maintained following parameters:

Temperature: + 10°C to + 35°C

Relative Humidity: 60 - 85%

Atmospheric pressure: 500 - 1060 hPa

3.8 Safety Precautions

- Do not attempt to reach any object, if this requires you to move forward on the seat.
- Do not attempt to pick up objects from the floor by leaning between your knees.
- Do not lean over the top edge of the backrest to reach objects located behind you, as doing so may overturn the wheelchair.
- Always apply both parking brakes.
- The parking brakes can be used only after stopping the wheelchair: never use them to reduce the speed of the wheelchair, or as a support when getting in or out of the seat.
- It is not recommended to travel over staircases and high obstacles without assistance.
- When passing over daily obstacles (steps, thresholds), position the back wheels of the wheelchair in the direction of movement.
- Companions should never try to raise the wheelchair by holding its removable parts.
- When climbing or descending on inclined surfaces, the companion should hold the wheelchair by its handles.
- Do not leave the wheelchair on an inclined surface.
- When getting in or out of the manual wheelchair Barry, apply the parking brakes to the back (driving) wheels.
- At the beginning of motion, avoid sudden movements and jolts, as they may overturn the wheelchair.
- Do not use the wheelchair without inflating the tires to a pressure of 304kPa-405kPa
- Do not overinflate the tire. It may break and cause injury.
- To avoid injury, keep your fingers away from moving parts.

DISINFECTION

To maximize the service life of manual wheelchairs Barry for disabled, please pay close attention to the following recommendations:

- Clean the upholstery and metal parts with a soft wet cloth. If necessary, the seat and backrest can be washed at 40°C with conventional or liquid detergent.
- Always wipe your wheelchair dry after it is used in the rain.
- If your wheelchair gets dirty, wipe off wet dirt immediately (when possible) and dry your wheelchair thoroughly.
- When caring for manual wheelchairs Barry for disabled, is it not recommended to use alcohol-based, abrasive cleaners, and cleaners with dye or clean your wheelchair with stiff brushes.
- Wheel axle should be cleaned with a clean cloth and a small amount of oil to lubricate.
- After each use, the sanitary equipment should be washed with mild detergent, then rinse under running water, then dry.

5. MAINTENANCE

Wheelchairs Barry with additional executions are designed and manufactured for long-term use. Their term of service in compliance with the rules of operation of 5 years, depending on the conditions and frequency of use.

For this purpose, regularly inspect your manual wheelchair Barry to ensure its constant safety and check its operability, as described below.

Perform a minimum visual inspection of condition and operability of the manual wheelchairs Barry at frequent and regular intervals, as well as each time before the next use.

Regular Checks:

- 1) Make sure that the wheelchair can be easily folded and unfolded. Make sure that the wheelchair can move in a straight line (with no resistance or sidewise deviation).
- 2) Make sure that disengaged parking brakes do not touch the tires. Make sure that the parking brakes operate properly. Make sure that connections are not worn and that the gap between the brake and the tire is properly adjusted.
- 3) Check the folding system for worn or deformed parts.
- 4) Make sure that all screws are tight.
- 5) Make sure that the backrest and seat upholstery are in good condition.
- 6) Make sure that the wheels are parallel to the frame.
- 7) Make sure that the hoops have no irregularities.
- 8) Make sure that the spokes are not deformed, loose or broken.
- 9) Make sure that all screw connections are tight.
- 10) Check the tire wear.
- 11) Make sure that all components of the entire sanitary equipment and have no cracks.

It is recommended to perform a full inspection of the wheelchair at a service center at least once a year.

6. ONGOING REPAIRS

Maintenance and repair of wheelchairs Barry for disabled, including warranty repair, is carried out within the territory of the Russian Federation by the authorized representative of the manufacturer – LTD SIMS 2 at: 6 Novoposelkovaya St., Moscow, 125363, tel: (495) 792-31-90 (multichannel), e-mail: info@sims2.ru.

7. STORAGE AND SERVICE LIFE

Packed wheelchairs Barry for disabled shall be stored in a dry, ventilated area in accordance with fire safety rules in conditions preventing contamination, mechanical damage, and exposure to sunlight.

Environmental temperature: from - 40°C to + 50°C

Relative humidity: from 60% to 80 %

Atmospheric pressure: 500 - 1060 hPa

Shelf life of manual wheelchairs Barry for disabled: 5 years.

Wheelchairs Barry for disabled are designed for 5 years of reliable service subject to regular maintenance as per the User Manual supplied with each product.

8. TRANSPORTATION

Wheelchairs Barry for disabled can be transported by any type of transportation, ensuring protection against mechanical damage and precipitation during loading and unloading operations.

Environmental temperature: from - 40°C to + 50°C

Relative humidity: from 60% to 80 %


Atmospheric pressure: 500 - 1060 hPa

9. PACKING

Wheelchairs Barry for disabled are supplied in cardboard boxes sized for the wheelchairs.

LABELING

Sample label

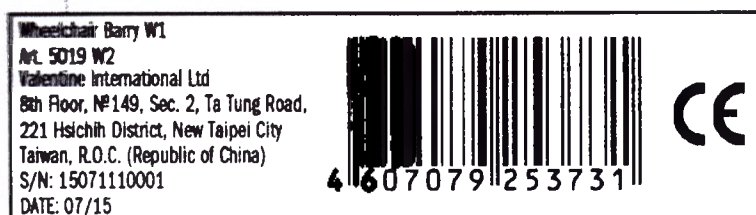Information on the individual packing:

Product name

Art.

Manufacturer's name

Manufacturer's address

Manufacturing country

Serial No.

Manufacturing date

CE marking

Marking for other versions is similar to that shown above.

11. DISPOSAL AND DESTRUCTION PROCEDURE

Upon expiry of their service life or in the event of damage to manual wheelchairs Barry for disabled beyond repair, the user shall dispose of the decommissioned equipment, after it is disinfected, as per SanPiN 2.1.7.2790-10 and Art. 23 of the Federal Law *On Circulation of Medical Products*.

Upon disposal of manual wheelchairs Barry for disabled, their woven fabric, plastic, and metal parts must be separated from each other.

12. WARRANTY

The manufacturer's warranty covers any manufacturing defects within 12 months after the sale of manual wheelchairs Barry for disabled and includes the free replacement of any defective parts. Coverage under the warranty is initiated by providing shipping documents and a written claim.

This warranty does not apply to the following conditions:

- Mechanical damage.
- Defects resulting from natural wear and tear or due to improper use.

When a manufacturing defect is found during the warranty period, the user may contact the authorized manufacturer distributor in the Russian Federation, LTD SIMS 2, at: 6 Novoposelkovaya St., Moscow 125363, tel.: +7 (495) 792-31-90 (multi line), e-mail: info@sims2.ru.

13. CLAIM

When a defective or inconsistent product is found, the claim is accepted by the authorized manufacturer distributor in the Russian Federation, LTD SIMS 2, at: 6 Novoposelkovaya St., Moscow 125363, tel.: +7 (495) 792-31-90 (multi line), e-mail: info@sims2.ru.

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