NIKKISO Europe GmbH

Desbrocksriede 1

D-30855 Langenhagen

Tel.: +49 (511) 67 9999 0

Fax: +49 (511) 67 9999 11

www.nikkiso-europe.eu

The Bank of Tokyo-Mitsubishi UFJ

BLZ: 300 107 00 Konto: 097568

BIC: BOTK DE 33

IBAN: DE 35 300 107 000 000 097 568

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EF-01D / EF-02D

Filter for dialysate purification

	Refer to the instruction before use / Vor der Verwendung Anleitungen beachten / Voir le manuel d'utilisation avant usage / Consultare le istruzioni prima dell'uso / Consulte el manual de instrucciones antes de su uso / Raadpleeg voor gebruik de handleiding / Przed użyciem należy zapoznać się z instrukcją / Hänvisa till instruktionen före användning / Gennehlæs instruktionsbogen før brug / Se instruksjon før bruk / Lue ohjeet ennen käyttöä / См. инструкции перед употреблением / Αναφερθείτε στις οδηγίες πριν την χρήση / Kullanymdan önce talimatlara başvurun / Veja o manual de instruções antes de utilizar
	Sterilized by gamma rays / Sterilisierungsmethode: Gammastrahlen / Sterilisé par rayons gamma / Sterilizzato ai raggi gamma / Esterilizado por rayos gamma / Gesteriliseerd m.b.v. gammastralen / Sterylizacja promieniami gamma / Steriliserad av gammastrålningar / Steriliseret med gammastråler / Steriliseret med gammastråler / Sterilisoitu gammasäteillä / Стерилизован гамма-лучами / Αποστείρωση με ακτίνες γάμα / Gama üyünlaryyla sterilize edilmib / Esterilização com raios γ (gama)
	Batch number / Chargennummer / Numéro de lot / Numero di lotto / Numero de lote / Partijnummer / Numer partii / Partinummer / Vapartinummer / Partinummer / Eränumero / Номер партии / Αριθμός παρτίδας / Parti Numarası / Número de lote
	Date of Manufacturing / Herstellungsdatum / Date de fabrication / Data di fabbricazione / Fecha de fabricación / Fabricagedatum / Data produkcji / Tillverkningsdatum / Produktionsdato / Produksjonsdato / Valmistuspäivä / Дата изготовления / Ημερομηνία Κατασκευής / Ymalat Tarihi / Data de fabricaço
	Expiry Date / Verfallsdatum / Date d'expiration / Data di scadenza / Fecha de caducidad / Vervaldatum / Data ważności / Utgångsdatum / Udløbsdato / Utløpsdato / Viimeinen käyttöpäivä / Срок годности / Ημερομηνία Λήξεως / Son Kullanma Tarihi / Data de expiração
	Authorized representative in the European community / Europäische Vertretung / Représentant autorisé dans l'Union Européenne / Referente autorizzato nella Comunità Europea / Representación autorizada en la Comunidad Europea / Geautoriseerde vertegenwoordiger in de Europese Gemeenschap / Autoryzowany przedstawiciel w Unii Europejskiej / Auktoriserad representant i Europeiska gemenskaperna / Autoriseret repræsentant i EU / Autorisert representant i EU / Valtutettu edustaja Euroopan yhteisössä / Уполномоченный представитель в Европейском Сообществе / Εξουσιοδοτημένος αντιπρόσωπος εντός της Ευρωπαϊκής κοινότητας / Avrupa topluluğundaki yetkili temsilci / Representante autorizado pela comunidade Europeia
	Manufacturer / Hersteller / Fabricant / Produttore / Fabricante / Fabrikant / Producent / Tillverkare / Producent / Produsent / Valmistaja / Производитель / Κατασκευαστής / Ymalatçı / Produtor

NIKKISO Europe GmbH
Desbrocksriede 1, D-30855, Langenhagen, Germany

NIKKISO CO., LTD.
20-3, Ebisu 4-Chome Shibuya-ku, Tokyo, Japan

KA-9101-004H (Revised date : 2015-08-31)

INSTRUCTION MANUAL

Filter for dialysate purification

EF-01D/EF-02D

WARNING

- Before using EF-01D/EF-02D (hereinafter, "the filter"), thoroughly read this instruction manual as well as the dialysis equipment's instruction manual.
- Follow instructions from the physician when using the filter.
- The filter is designed specifically for NIKKISO dialysis equipments, and should not be used with other equipments.

Intended Use

The filter is used to remove bacteria and endotoxin from the dialysis fluid.

Usage

Attach the filter to the dialysate filter holder on the dialysis equipment, and operate as instructed in the dialysis equipment's instruction manual.

Precautions

- Do not use the filter if the filter or its package appears to be damaged.
- Do not use the filter past the expiration date indicated on the label.
- Use the filter promptly once the sterilization bag has been opened.
- Check that there is no bend, warping, or other abnormalities with the piping connected to the filter. If an abnormality is observed, do not use until resolved.
- Ensure that the dialysis water and dialysis fluid satisfy the quality requirements described in the dialysis equipment's instruction manual.
- A large volume of air in the filter may be an indication of leakage. Replace with a new filter.
- The filter leakage test is conducted automatically before treatment. If "TFD214" is displayed on the dialysis equipment, immediately replace with a new filter.
- During treatment, check that there is no leakage from the filter. If a leak is observed, immediately replace with a new filter. A leaking filter may result in excessive removal of water from patient.
- Prevent contact of substances such as chemicals and oils from outer surface of the filter.
- Be careful when using chemicals for cleaning of a hemodialysis machine to which the filter is equipped.
- If such substances adhered to the outer surface of the filter, wipe them off with a soft water-wet cloth immediately.

Cleaning of filter

- Wipe the dirt and dust off with soft dry cloth from the outer surface of the filter. If the dirt still remains, wipe it off with a soft water-wet cloth.
 - When disinfection is required, wipe the outer surface of the filter with a soft well-wrung cloth dampened in diluted sodium hypochlorite solution. (dilute concentration : 0.05% to 0.1%) Finally, wipe it again with a soft water-wet cloth immediately.
- Do not use ethanol for disinfection. Use the sodium hypochlorite solution within the range of directed dilute concentration.

Cleaning and Disinfection

- Always clean and disinfect the filter before treatment.
- For details on the chemical used for cleaning and disinfection, refer to the dialysis equipment's instruction manual.
- After cleaning and disinfection, rinse thoroughly with water to remove any residual chemical.

Replacement

- The filter must be replaced before 750 cumulative hours of operation.
- To check the operating hours, refer to the dialysis equipment's instruction manual.

Disposal

- When disposing the filter, take appropriate measures to prevent it from polluting the surrounding environment.
- Dispose the filter in accordance with the local laws and regulations.

Storage

- Store the filter in a clean room, under a temperature of 4 to 30 °C (39 to 86 °F).
- Do not store in an environment with chemicals, oil, high temperature, high humidity, direct sunlight, UV light, dust, salt, sulfur, or any other factors that may have a harmful effect on the filter.

Sterilization Method

γ (gamma) ray sterilization

WARRANTY AND LIMITATION OF LIABILITY

- The manufacturer warrants that the device has been manufactured in accordance with its specifications and in compliance with ISO13485, applicable industry standards, and regulatory requirements.
- The device with manufacturing defects will be replaced if defect is reported with lot number.
- The manufacturer shall not be liable for any misuse, improper handling, non-compliance with instruction manual and cautionary notes and for any damage incurred subsequent to the manufacturer's delivery of the device.

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