MediSpine™

Spinal Needle 

INSTRUCTIONS FOR USE

INTENDED USE: Spinal needle is used to injection local anesthetics in order to provide regional anesthesia.

PRODUCT SPECIFICATION:

S.NO	GAUGE	NEEDLE OD and ID $\pm 0.01\text{mm}$	STYLET OD	EFFECTIVE LENGTH OF NEEDLE $\pm 0.4\text{mm}$	COLOUR CODE
1.	18 G	1.26-0.96	0.90 mm $\pm 0.05\text{mm}$	75/90/150	PINK
2.	19 G	1.06-0.75	0.70 mm $\pm 0.05\text{mm}$	75/90/150	CREAM
3.	20 G	0.90-0.60	0.55 mm $\pm 0.05\text{mm}$	75/90/150	YELLOW
4.	21G	0.81-0.51	0.45 mm $\pm 0.05\text{mm}$	75/90/150	GREEN
5.	22 G	0.70-0.40	0.35 mm $\pm 0.01\text{mm}$	75/90/150	BLACK
6.	23 G	0.66-0.40	0.35 mm $\pm 0.01\text{mm}$	75/90/150	BLUE
7.	24G	0.58-0.33	0.25 mm $\pm 0.01\text{mm}$	75/90/150	DARK PINK
8.	25 G	0.55-0.30	0.25 mm $\pm 0.01\text{mm}$	75/90/150	ORANGE
9.	26G	0.46-0.30	0.25 mm $\pm 0.01\text{mm}$	75/90/150	BROWN
10.	27G	0.41-0.25	0.20 mm $\pm 0.01\text{mm}$	75/90/150	LIGHT GREY

MATERIAL USED IN SPINAL NEEDLE:

S.No	Component	MATERIAL
1.	SS PIPE	STAINLESS STEEL
2.	STYLET	STAINLESS STEEL
3.	STYLET HUB	ACRYLONITRILE BUTADIENE STYRENE (ABS) + MASTER BATCH
4.	NEEDLE	SS PIPE
5.	NEEDLE HUB	POLYCARBONATE
6.	NEEDLE HUB INSERT	POLYCARBONATE
7.	PIPE TYPE NEEDLE COVER	POLYPROPYLENE

THE PROCEDURE FOR USING:

- Choose the corresponding size of spinal needle.
- Prepare patient for the procedure lying on one side or in sitting position.
- Select the area for introduction of needle and work up the same with antiseptic solution.
- Taking all aseptic Precautions, anaesthetize the puncture are with local anesthetic.
- Advance the spinal needle through the dura-mater into the subarachnoid space. As you pass through the dura-mater you will normally feel slight resistance followed by a "click" which indicates that you are in the subarachnoid space.

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- After the "click", slowly remove the stylet from the spinal needle and confirm the presence of CSF. If there is no CSF reflux to indicate correct positioning in the subarachnoid space, rotate the spinal needle then aspirate if necessary to check for CSF. If there is still no CSF, remove the spinal needle and carefully adjust the position of the introducer needle, or repeat the puncture procedure at a new site.
- When dural puncture is achieved (and confirmed by CSF reflux through the spinal needle): proceed to CSF collection if performing lumbar puncture; when performing spinal anaesthesia, hold the needle firmly when attaching the syringe (to avoid changing the needle position), then inject the anaesthetic.
- Withdraw the spinal needle.
- Apply sterile material on puncture area.
- Dispose the needle and syringe according to the medical waste disposal rules.

WARNINGS:

- In case of problem or resistance during insertion, it is advisable to remove the spinal and introduce needles simultaneously
- Maintain strict aseptic conditions throughout the procedure
- If there is paraesthesia in the lower limbs or pelvis the device must be removed.
- The product should be used according to the instructions for use.

GLOBAL MEDIKIT DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.

- The product should not be reprocessed.
- Visually inspect and carefully check the product and packaging before use. Improper transport and handling may cause structural and/or functional damage to device or packaging.
- The product is guaranteed non-toxic, sterile & non-pyrogenic if the package has not been opened or damaged.
- Do not clean or resterilise. For single use only.
- Discard after use in accordance with local / regional / international regulations
- Store in cool & dry place. Do not expose to extremes temperature.
- Transportation material carried by all modes of transport in compliance with the regime of temperature from +5 to + 26C, at a relative humidity of 60 to 80%.
- The product should be used immediately after opening the packaging.
- Shelf life 5 years.

CONTRAINDICATIONS:

- Lack of patient consent
- Coagulation problems or anti-coagulation therapy
- Anatomical deformities
- Localised infection at the puncture site, or general infection(septicaemia)
- Allergy to local anaesthetic
- Intracranial hypertension

COMPLICATIONS:

The known complications are those associated with spinal anaesthesia techniques: hypotension, cardiac arrest, respiratory arrest, haematoma or epidural abscess, backache, acute urine retention, headache, problems with vision and hearing, nausea, vomiting etc... These are rare. However these devices must only be used by qualified personnel, with the necessary means available to treat these complications if necessary.

CAUTION:

The use of a smaller diameter will cause less CSF leakage through human dura. It is accepted that this CSF leakage is the contributing cause of the PDPH associated with spinal anesthesia.

Global Medikit Limited

Corporate Identity Number : U33119DL2001PLC110470

3, Dr. G. C. Narang Marg, Delhi - 110007 (INDIA) • Tel : +91-11-27662182 / 88 / 89 • Email : domestic@globalmedikit.in • Website : www.globalmedikit.in

000 ЦПММ INFO@CIRMI.RU WWW.GOSZDRAVNADZOR.RU Khasra No. 323, (MI) Camp Road, Selaqui, Dehradun - 248197 (UTTRAKHAND)

IMPORTANT:

- Always introduce spinal needle with stylet.
- Don't force the needle through resistance. If patient complains of pain or paresthesia during the procedure, abandon the procedure and manage the patient for same accordingly.

HAZARDS OF MEDISPINE REUSE:

- Transmission of infection from one patient to other patient
- Blunting of the bevel causes difficulty in reintroducing the needle and causes more pain.
- Reuse causes loosening of stylet at the hub and may cause tissue coring inside needle.
- Reuse weakens the body of the needle and may cause accidental breakage of needle inside body.

WARRANTY :

GLOBAL MEDIKIT warrants its products free from defects in workmanship and material under proper handling and use. This warranty voids if failure or defect is due to improper handling or storage or factors related to the patient, his / her Diagnosis, Treatment, Surgical Procedures or any other matter beyond GLOBAL MEDIKIT control, directly affecting GLOBAL MEDIKIT products and the results obtained from their use. Liability of GLOBAL MEDIKIT limited upto replacement of the Product only. GLOBAL MEDIKIT disclaims any responsibility for possible consequences resulting from improper use. If, despite this, still be found to be defective, please contact our authorized representative.

Global Medikit Limited

Khasra No. 323 (MI) Camp Road, Selaqui,
Dehradun, Uttarakhand, India.

: Obelis s.a. Boulevard Général Wahis 53, 1030 Brussels, BELGIUM
Tel: +(32) 2 732-59-54 Fax: +(32) 2 732-60-03 E-mail: mail@obelis.net

Authorized representative:

LLC "Registration and Certification Assistance Center"

7 Vellinga Street, office 10, Podolsk, Moscow Region, 142100, Russia

Customer Care : +7 495 724-10-67

Email : centre@med-reg.ru

 Catalogue number	 Use by / Expiry Date	 Date of manufacture
 Batch code	 Do not reuse	 Manufacturer
 Sterilized using ethylene oxide	 Do not resterilize	 See instructions for use
 Do not use if package is damaged	 Authorised representative in the European Community	 0434

ANUP KUMAR MAMGAIN
(HEAD QA CUM MANAGEMENT REPRESENTATIVE)

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