APPROVE
R&S Consumer Goods GmbH
General manager

  
Axel Roth
CONSUMER GOODS
Wasserburger Land 364
81227 München - Germany
Tel: +49-40-89-67578500
Email: info@rsgermany.de
Date 05/08/2016
Stamp

INSTRUCTION GUIDE
Natural Rubber Latex Male Condom

FOR THE RUSSIAN MARKET

1. Description and action.....	page 3
1.1. Indications for the product use.....	page 3
1.2. Technical characteristics.....	page 3
1.3. Product components and package.....	page 6
1.4. Labeling.....	page 7
2. Wearing indications.....	page 9
3. Wearing contraindications.....	page 9
4. Proper use.....	page 10
4.1. Wearing limitations.....	page 10
4.2. Preparing the product for wearing.....	page 10
4.3. Product use.....	page 10
5. Product maintenance.....	page 10
6. Storage and transporting.....	page 10
7. Disposal.....	page 10
8. Manufacturer warranty.....	page 11
9. Manufacturer contact information.....	page 11
10. Technical specification.....	page 12

This Natural rubber latex male condom is used for contraception and for prophylactic purposes (help prevent pregnancy and the transmission of sexually transmitted diseases).

1) **Description and action**

1.1) **Indications for the product use**

The non-sterile male condom is a single-use disposable medical device made of natural rubber latex as the principal ingredient.

Medical Function: Contraception and/or prevention against “Sexually Transmitted Diseases (STD’s)” including AIDS; under specified conditions of use.

Mode of Operation: Condoms are worn over the erected penis before the latter enters vagina or before commencing sexual intercourse.

The device is classified as Class IIb as per Rule 14 of annex-IX of MDD 93/42/EEC and 2007/47/EG.

1.2) **Technical characteristics**

The device is made of natural rubber latex as the principal ingredient, chemical ingredients and stabilizers. The device composition includes the following:

NAME, BRAND	CHEMICAL NAME	FUNCTION	RATIO
NATURAL RUBBER LATEX HA	CIS-1,4-POLYISOPRENE	RAW MATERIAL	~ 92%
TAMOL NN 9104	TAMOL	SUSPENSION AGENT	<1%
ZDEC	ZINC DIETHYLDITHIOCARBAMATE	ACCELERATOR	<2%
WINGSTAY L	DI-TERT-BUTYL-4-METHYLPHENOL	ANTIOXIDANT	<1%
ZDBC	ZINC DIBUTYLDITHIOCARBAMATE	ACCELERATOR	<1%
ZINC OXIDE ACTIVE	ZINC OXIDE	ACTIVATOR	<2%
GROUND SULPHUR	SULPHUR	VULCANIZING AGENT (CROSS-LINKING)	<2%
POTASSIUM HYDROXIDE FLAKES	POTASSIUM HYDROXIDE	STABILIZING AGENT	<1%
EMULAN OG	FATTY ALCOHOL, ETHOXYLATED	STABILIZER	<1%
POTASSIUM OLEATE SOLUTION 20% COMPOSITION: OLEIC ACID: ~4% POTASSIUM HYDROXIDE (KOH): ~8% DEIONIZED WATER: ~78%	POTASSIUM OLEATE	STABILIZER	<2%

LUBRICANT SPECIFICATIONS

Silicone-based or water-based lubricants may be added to some types of devices.

Silicon-based lubricant

Most types of condoms are lubricated with odorless and flavorless silicone oil. It is compatible with natural rubber latex and recommended for use on condoms of all types:

Specifications:

Product name (brand): BRB Silicone oil DM 350 EP

INCI name: Dimethicone

Pure substance/mixture: Substance

Ratio and minimum quantity per condom	Mg
Silicone Lubricant	450
	0
Total	450

Formulation summary of condoms with glowing effect

The glowing effect of the “glow in the dark” condom types is caused by the ingredients UCAR POLYPHOBE 106 HE and LUMILUX EFFECT GREEN added to the device.

Formulation summary of condoms with glowing effect and silicone-based lubricant:

Brand / Name	Chemical name	Function	Ratio (%)
Compounded latex	Formulation see chapter 2.1	Primary raw material	95.84
BRB Silicone oil DM 350 EP	Dimethicone	Lubncant	2.25
UCAR POLYPHOBE 106 HE	Polyphobe	Glowing agent	0.021
LUMILUX EFFECT GREEN	Zinc sulphide	Glowing agent	1.88

Water based lubricant with cooling effect

Some variants of condones may use water based lubricant with cooling effect. The cooling effect in the skin is caused by mint oil or Mentha Piperita Oil (INCI).

Water based lubricants are widely recommended and compatible with all kinds of condoms.

Ingredients (INCI)	Function	Ratio (%)
Aqua	Diluent	10-15
Dimethicone	Emollient	15-25
Propylene Glycol	Humectant	30-40
Glycerin	Lubricant/Solvent	20-30
Isopropyl Palmitate	Emollient	3.5-6.5
Cetearyl Isononanoate	Emollient	3.5-6.5
Ceteareth-20	Emulsifier	3.5-6.5
Ceteareth-12	Emulsifier	3.5-6.5
Cetearyl Alcohol	Viscosity regulator	0.5-0.8
Glyceryl Stearate	Viscosity regulator	0.5-0.8
Cety palmitate	Emollient	3.5-6.5
Sodium Polyacrylate	Emulsifier	0.5-0.8
Mentha Piperita Oil	Fragrance/Cooling agent	0.01-0.02

Ratio and minimum quantity per condom	Mg
Silicon-based lubricant	200
Water Based Cooling Lubricant	450
Total	650

Water based lubricant with warming effect

Some variants of condones may use water based lubricant with warming effect. The warming effect in the skin is caused by a higher proportion of glycerin vs. purified water.

Water based lubricants are widely recommended and compatible with all kinds of condoms.

Ingredients of warming lubricant, function and ratio:

Ingredients (INCI)	Function	Ratio (%)
Aqua	Diluent	40.96
Glycerin	Lubricant / denaturant / solvent, humectant	55.90
Sodium Benzoate	Preservative / anti-fungal agent	0.50
Hydroxyethyl cellulose	Thickener	1.50
Lactic Acid	Buffering / PH regulation / Humectant / Skin conditioning	0.14
Sodium Lactate	Buffering / Humectant	0.60
Allantoine	Moisturizing agent / soothing	0.40

Ratio and minimum quantity per condom	Mg
Silicon-based lubricant	200
Water Based Warming Lubricant	450
Total	650

SPECIFICATIONS OF FLAVORS AND COLORS

The following flavors and colors are added to the following types:

Type	Flavor name	Color Name
VITALIS strawberry / red	Strawberry flavor oil 4439	Colony Carmine LTX 100-ID
VITALIS chocolate / black	Chocolate Flavor oil 11614	Colony Black N 131
VITALIS banana / yellow	Banana flavor oil 4431	Flexonyl Yellow DGR-LA
VITALIS Tutti Fruit / Blue	Tutti Frutti Flavor oil 9885	Colanyl Blue A2R 131
VITALIS orange / orange	Orange Flavor oil 4373	Flexonyl Orange SP-G-VP 3584
VITALIS mint / green	Spearmint Flavor oil 13138	Colanyl Green GG 131
VITALIS apple / green	Green Apple Flavor oil 2842	Colanyl Green GG 131
VITALIS coconut / transparent	Coconut Flavor oil 12023	Transparent / No color
VITALIS Vanilla / Gold	Vanilla Flavor 4135	Iriodin®302 gold Satin
VITALIS sensation	(No flavor)	Colanyl Carmine LTX 100-ID
ON) strawberry / red	Strawberry flavor oil 4439	Colanyl Carmine LTX 100-ID
ON) chocolate / black	Chocolate Flavor oil 11614	Colanyl Black N 131
ON) banana / yellow	Banana flavor oil 4431	Flexonyl Yellow DGR-LA

Nominal Length:

General measurements of lengths are the following:

- ≥ 160mm length for condoms with width ≥ 47.0mm
- ≥ 178mm length for condoms with width ≥ 50.0mm
- ≥ 193mm length for condoms with width > 57.0mm
- ≥ 223mm length for condoms with width ≥ 64.0mm

Nominal width:

Condoms with the following nominal widths are manufactured: 47mm, 49mm, 53mm, 57mm, 60mm, 64mm or 69mm. Deviation from the average measured width and claimed nominal thickness:

Each individual condom: ± 2 mm

Mean of the lot: ± 1mm

Description on packages inform about the nominal width.

The width of the small rim of the rolled condoms should be 1,0 -1,6 mm

Nominal wall thickness:

Nominal wall thickness and tolerance between average measured mid-body thickness and claimed nominal thickness:

Normal condoms: $\geq 0.065 \text{ mm} \pm 0.01$

Strong condoms: $\geq 0.080 \text{ mm} \pm 0.01$

Thin condoms: $0.050\text{-}0.055 \text{ mm} \pm 0.01$

(Very thin condoms: $< 0.050\text{mm} \pm 0,008$, not manufactured by R&S)

Weight (without lubrication):

1.5-2.30gm

Burst pressure:

Not less than 1.0kPa for all condoms less strong condoms

Not less than 2.0kPa only for strong condoms (thickness $\geq 0.080\text{mm}$)

Burst volume:

The minimum burst volume depends on the thickness is not less than the following:

16,0 dm³ for condoms with a mid-body width greater than or equal to 45,0 mm and less than 50,0 mm

18,0 dm³ for condoms with a mid-body width greater than or equal to 50,0 mm and less than 56,0 mm

22,0 dm³ for condoms with a mid-body width greater than or equal to 56,0 mm and less than 65,0 mm

28,0 dm³ for condoms with a mid-body width greater than or equal to 65,0 mm and not more than 75,0 mm

Tensile Properties for strong condoms only (thickness $\geq 0.080\text{mm}$):

Elongation at break: minimum 700%

Force at break: minimum 100N

Shape:

Variants may have cylindrical (parallel walls) shape, flare shape or anatomic shape.

Description on packages and on leaflets inform about these specifications.

Texture:

Variants may be plain, dotted, ribbed or both. Description on packages and on leaflets inform about these specifications.

Reservoir tip (deposit):

Variants may or may not have a reservoir tip for sperm. Description on packages and on leaflets mention if the specific condom has a reservoir tip or not.

The reservoir length (for variants with a reservoir tip) should be $18 \pm 2 \text{ mm}$

All options condom, see the technical specification.**1.3) Product components and package**

Every single condom is packed in an individual sealed aluminum foil package opaque to light. The foil is laminated and includes a layer of impermeable flexible aluminum foil of a minimum thickness of 8 micrometers, and layers of plastic materials suitable for the mechanical protection of the metal foil and for printing and sealing. The condoms are packed in consumer packaging which can be boxes or bags containing, 1,3, 6, 10, 12, 36, 50, 100 or 1000 condoms.

1.4) Labeling

The single unit must contain the minimum information

Required information	Used symbol, if the case
• Name of product	
• Brand	
• CE 0197	CE 0197
• LOT with symbol	LOT
• Expiry with symbol	

The consumer packaging must contain information

Required information	Used symbol, if the case
Full description of the condom, for example whether or not it has a reservoir tip, is parallel sided or shaped, is colored or natural, textured or plain, has a flavor or fragrance, and is lubricated or not.	
Number of condoms contained	
Nominal width of the condom	
The nominal length of the condom if this is shorter than 160 mm	
Whether the condom is lubricated or dry. If the condom or lubricant is fragranced or flavored	
Manufacturer symbol	
Lot number, manufacturer's identifying reference for traceability	
Lot number	
Expiry date writing form	YYYY-MM
Expiry date symbol	
Instruction for storage: Condoms must be stored in a cool dry place, away from heat and direct sunlight	
Protect from heat symbol:	
Information concerning lubricant. In case lubricant contains aromatics or perfumes this has also to be stated on the shelf box.	
Statement that the condoms are made from natural rubber latex	
CE Mark with notified body number and appropriate symbols	CE 0197
Brand name	
Manufacturers name and address	
Statement to read the leaflet	
The normative: ISO 4074	
Single use only symbol	
Read user information symbol:	
Additional information can be added according to local regulations or customer requirement	

The leaflet must contain the minimum information

Required information	Used symbol, if the case
Brand name	
Instructions how to use the condom: The condom should be placed on the erect penis before any contact occurs between the penis and the partner's body to assist in the prevention of sexually transmitted diseases and pregnancy. Open the condom packaging carefully with your fingers and not with sharp objects. Be careful not to damage the condom with the fingernails, jewelry etc. when applying. Hold the reservoir between thumb and forefinger and remove any air remaining in the tip of the condom by squeezing between thumb and forefinger. After ejaculation withdraw the penis along with the condom from the vagina before the penis softens. Remember to hold onto the condom at the base of the penis. Additional lubrications: If you need additional lubrication, use only the correct type of lubricant which is recommended for use with condoms. Avoid the use of oil based lubricants such as petroleum jelly, baby oil, body lotions, massage oils, butter, margarine etc., as these are deleterious to the integrity of the condom. Important: Dispose of the condom in a waste bin. Do not flush condoms down the toilet. Use each condom only once, as otherwise the protection is not guaranteed! Latex condoms normally do not cause side effects. If allergic reactions are experienced, consult your doctor. If you are using prescription or non-prescription medication on the genitals such as ointments or creams, ask your doctor or pharmacist about their tolerability with condoms. When properly used, condoms are highly effective. They may prevent pregnancy and help reduce the risk of spreading many sexually transmitted diseases. However, no contraceptive can guarantee 100% protection.	
Reference to the standard ISO 4074	
Manufacturer with symbol:	
CE Mark with notified body number and appropriate symbols	CE 0197
Protect from heat symbol:	
Read user manual symbol:	
Latex symbol:	

Additional information may be added according to local regulations or customer requirements. The precautions can be shown in icons in the instructions for use in order to make the comprehension easier.

2) Indications

When properly used, condoms are highly effective. They may prevent pregnancy and help reduce the risk of spreading many sexually transmitted diseases. However, no contraceptive can guarantee 100% protection.

3) Contraindications unknown

Side effects unknown

Latex condoms normally do not cause side effects. If allergic reactions are experienced, consult your doctor. If you are using prescription or non-prescription medication on the genitals such as ointments or creams, ask your doctor or pharmacist about their tolerability with condoms.

4) Precautions

4.1) Wearing limitations

- Condoms must be stored in a cool dry place, away from heat and direct sunlight;
- Single use only;
- If you need additional lubrication, use only the correct type of lubricant which is recommended for use with condoms. Avoid the use of oil based lubricants such as petroleum jelly, baby oil, bodylotions, massage oils, butter, margarine etc., as these are deleterious to the integrity of the condom.
- Dispose of the condom in a waste bin. Do not flush condoms down the toilet.

4.2) Handling

The condom should be placed on the erect penis before any contact occurs between the penis and the partner's body to assist in the prevention of sexually transmitted diseases and pregnancy. Open the condom packaging carefully with your fingers and not with sharp objects. Be careful not to damage the condom with the fingernails, jewelry etc. when applying. Hold the reservoir between thumb and forefinger and remove any air remaining in the tip of the condom by squeezing between thumb and forefinger.

4.3) Product use

After ejaculation withdraw the penis along with the condom from the vagina before the penis softens. Remember to hold onto the condom at the base of the penis.

Important: Dispose of the condom in a waste bin. Do not flush condoms down the toilet. Use each condom only once, as otherwise the protection is not guaranteed!

5) Maintenance

Condom single use only.

6) Storage and Transportation

General storage conditions for natural rubber condoms:

Store condoms in a cool dry place away from direct sunlight

Optimal storage conditions are between 15°C to 35 °C and at relative humidity of 50% to 90%.

Transportation conditions:

Primary packaging: Device should be foiled and sealed. Packed in dispenser boxes or plastic bags.

Secondary packaging: Carton

Distribution methods: Sea freight, air freight, lorry and courier services.

7) Disposal

Dispose of the condom in a waste bin. Do not flush condoms down the toilet.

Disposal of used items should be carried out according to the sanitary rules and Russian standards 2.1.7.2790-10. Used condoms relate to waste of class B (epidemiologically hazardous waste).

8) Manufacturer warranty

WE, R&S CONSUMER GOODS GMBH, hereby declare that we guarantee to deliver the Natural Rubber Latex Condoms with the brands VITALIS, ON and MYSIZE, including all their different types, with the specifications stated in the attached Technical specification. See attached the Technical specification.

Shelf life: 5 years

9) Manufacturer contact information

R&S CONSUMER GOODS GMBH
WASSERBURGER LANDSTR. 264
81827 MÜNCHEN GERMANY
PHONE: 089-87579508
FAX: 089-87579509
E-MAIL: RSMUNICH@AOL.COM

Authorized representative of the Manufacturer in the Russian Federation:

Limited Liability Company "Globalization" (LLC "Globalization")
350000, Krasnodar region, Krasnodar, ul. Krasnoflotskaya, 18, of. 4
phone: +7 (861) 245-50-22
e-mail: info@gloobal.ru

Technical specification

Brand and type / Specifications	VITALIS natural	VITALIS comfort plus	VITALIS ribbed	VITALIS sensation	VITALIS color & flavor	VITALIS strong	VITALIS x-large
Description	Normal, standard condom	Standard condom, anatomic shape	Textured condom, ribbed	Textured condom, ribbed, dotted	Colored and flavored condom	More resistant condom	Bigger condom
Additional benefit	Normal condom	Better fit to penis shape	Stimulating effect	Better fit and stimulating effect	More fun	Better protection	Better fit for bigger penis
Length (mm)	185±5	185±5	185±5	185±5	185±5	180±5	190±5
Width (mm)	53±2	53±2	52±2	53±2	53±2	53±2	57±2
Thickness (mm)	0,065 ±0.010	0,065 ±0.010	0,065 ±0.010	0,065 ±0.010	0,065 ±0.010	0,065 ±0.010	0,065 ±0.010
Color	No	No	No	Pink	Various	No	No
Flavor	No	No	No	No	Various	No	No
Type lubricant	Silicon based	Silicon based	Silicon based	Silicon based	Silicon based	Silicon based	Silicon based
Quantity lubricant (mg)	≥450	≥450	≥450	≥450	≥450	≥450	≥450
Shape	Cylindrical	Contour	Cylindrical	Anatomical	Cylindrical	Cylindrical	Cylindrical
Texture	Plain	Plain	Ribbed	Ribbed, dotted	Plain	Plain	Plain
Reservoir Tip	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Force at break (N)	N.A.	N.A.	N.A.	N.A.	N.A.	≥100	N.A.
Elongation at break (%)	N.A.	N.A.	N.A.	N.A.	N.A.	≥700	N.A.
Min. Burst Volume (dm ³)	≥18	≥18	≥18	≥18	≥18	≥18	≥22
Min. Burst pressure (kPa)	≥1,0	≥1,0	≥1,0	≥1,0	≥1,0	≥2,0	≥1,0

Brand and type / Specifications	VITALIS super thin	VITALIS stimulation & warming	VITALIS delay & cooling	VITALIS glow in the dark	VITALIS strawberry / red	VITALIS chocolate / black	VITALIS banana / yellow
Description	Condom with extra thin walls	Condom with warming effect	Condom with cooling effect	Condom glowing in the dark	Colored condom with flavor	Colored condom with flavor	Colored condom with flavor
Additional benefit	More feeling	Warming and stimulating effect	Cooling and delaying effect	More fun	More fun	More fun	More fun
Length (mm)	185±5	185±5	185±5	185±5	185±5	185±5	185±5
Width (mm)	53±2	53±2	53±2	53±2	53±2	53±2	53±2
Thickness (mm)	0,055 ±0,010	0,065 ±0,010	0,065 ±0,010	0,065 ±0,010	0,065 ±0,010	0,065 ±0,010	0,065 ±0,010
Color	No	No	No	No	Red	Black	Yellow
Flavor	No	No	No	No	Strawberry	Chocolate	Banana
Type lubricant	Silicon based	Water based with warming effect	Water based with cooling effect	Silicon based with glowing effect	Silicon based	Silicon based	Silicon based
Quantity lubricant (mg)	≥450	≥650	≥650	≥450	≥450	≥450	≥450
Shape	Cylindrical	Cylindrical	Cylindrical	Cylindrical	Cylindrical	Cylindrical	Cylindrical
Texture	Plain	Plain	Plain	Plain	Plain	Plain	Plain
Reservoir Tip	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Force at break (N)	N.A.	N.A.	N.A.	N.A.	N.A.	N.A.	N.A.
Elongation at break (%)	N.A.	N.A.	N.A.	N.A.	N.A.	N.A.	N.A.
Min. Burst Volume (dm ³)	≥18	≥18	≥18	≥18	N.A.	N.A.	N.A.
Min. Burst pressure (kPa)	≥1,0	≥1,0	≥1,0	≥1,0	≥1,0	≥1,0	≥1,0

Brand and type / Specifications	VITALIS tutti frutti / blue	VITALIS orange / orange	VITALIS mint / green	VITALIS apple / green	VITALIS coconut / transparent	VITALIS vanilla / gold
Description	Colored condom with flavor	Colored condom with flavor	Colored condom with flavor	Colored condom with flavor	Colored condom with flavor	Colored condom with flavor
Additional benefit	More fun	More fun	More fun	More fun	More fun	More fun
Length (mm)	185±5	185±5	185±5	185±5	185±5	185±5
Width (mm)	53±2	53±2	53±2	53±2	53±2	53±2
Thickness (mm)	0.065 ±0.010	0.065 ±0.010	0.065 ±0.010	0.065 ±0.010	0.065 ±0.010	0.065 ±0.010
Color	Blue	Orange	Green	Green	No color	Gold
Flavor	Tutti frutti	Orange	Mint	Apple	Coconut	Vanilla
Type lubricant	Silicon based	Silicon based	Silicon based	Silicon based	Silicon based	Silicon based
Quantity lubricant (mg)	≥450	≥450	≥450	≥450	≥450	≥450
Shape	Cylindrical	Cylindrical	Cylindrical	Cylindrical	Cylindrical	Cylindrical
Texture	Plain	Plain	Plain	Plain	Plain	Plain
Reservoir Tip	Yes	Yes	Yes	Yes	Yes	Yes
Force at break (N)	N.A.	N.A.	N.A.	N.A.	N.A.	N.A.
Elongation at break (%)	N.A.	N.A.	N.A.	N.A.	N.A.	N.A.
Min. Burst Volume (dm ³)	N.A.	N.A.	N.A.	N.A.	N.A.	N.A.
Min. Burst pressure (kPa)	≥1,0	≥1,0 *	≥1,0	≥1,0	≥1,0	≥1,0

Brand and type / Specifications	ON) strawberry / red	ON) chocolate / black	ON) banana / yellow	ON natural feeling	ON stimulation	ON fruit & color
Description	Colored condom with flavor	Colored condom with flavor	Colored condom with flavor	Normal, standard condom	Textured condom, dotted	Colored and flavored condom
Additional benefit	More fun	More fun	More fun	Normal condom	Stimulating effect	More fun
Length (mm)	185±5	185±5	185±5	185±5	185±5	185±5
Width (mm)	54±2	54±2	54±2	54±2	54±2	54±2
Thickness (mm)	0,065 ±0.010	0,065 ±0.010	0,065 ±0.010	0,065 ±0.010	0,065 ±0.010	0,065 ±0.010
Color	Red	Black	Yellow	No	No	Various
Flavor	Strawberry	Chocolate	Banana	No	No	Various
Type lubricant	Silicon based	Silicon based	Silicon based	Silicon based	Silicon based	Silicon based
Quantity lubricant (mg)	≥450	≥450	≥450	≥450	≥450	≥450
Shape	Cylindrical	Cylindrical	Cylindrical	Cylindrical	Cylindrical	Cylindrical
Texture	Plain	Plain	Plain	Plain	Dotted	Plain
Reservoir Tip	Yes	Yes	Yes	Yes	Yes	Yes
Force at break (N)	N.A	N.A	N.A	N.A	N.A	N.A
Elongation at break (%)	N.A	N.A	N.A	N.A	N.A	N.A
Min. Burst Volume (dm ³)	N.A	N.A	N.A	≥18	≥18	≥18
Min. Burst pressure (kPa)	≥1,0	≥1,0	≥1,0	≥1,0	≥1,0	≥1,0

Brand and type / Specifications	MYSIZE 47	MYSIZE 49	MYSIZE 53	MYSIZE 57	MYSIZE 60	MYSIZE 64	MYSIZE 69
Description	Condom with special size	Condom with special size	Condom with special size	Condom with special size	Condom with special size	Condom with special size	Condom with special size
Additional benefit	Better fit using the right size	Better fit using the right size	Better fit using the right size	Better fit using the right size	Better fit using the right size	Better fit using the right size	Better fit using the right size
Length (mm)	160±3	163±3	178±3	178±3	193±3	223±3	223±3
Width (mm)	47±2	49±2	53±2	57±2	60±2	64±2	69±2
Thickness (mm)	0,065 ±0.010	0,065 ±0.010	0,065 ±0.010	0,065 ±0.010	0,065 ±0.010	0,065 ±0.010	0,065 ±0.010
Color	No	No	No	No	No	No	No
Flavor	No	No	No	No	No	No	No
Type lubricant	Silicon based	Silicon based	Silicon based	Silicon based	Silicon based	Silicon based	Silicon based
Quantity lubricant (mg)	≥450	≥450	≥450	≥450	≥450	≥450	≥450
Shape	Cylindrical	Cylindrical	Cylindrical	Cylindrical	Cylindrical	Cylindrical	Cylindrical
Texture	Plain	Plain	Plain	Plain	Plain	Plain	Plain
Reservoir Tip	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Force at break (N)	N.A.	N.A.	N.A.	N.A.	N.A.	N.A.	N.A.
Elongation at break (%)	N.A.	N.A.	N.A.	N.A.	N.A.	N.A.	N.A.
Min. Burst Volume (dm³)	≥16	≥16	≥18	≥22	≥22	≥22	≥28
Min Burst pressure (kPa)	≥1,0	≥1,0	≥1,0	≥1,0	≥1,0	≥1,0	≥1,0

File No. P 920 / 2016

Hereby I, Dr. Peter Schubert, civil law notary in Munich, certify that the above is the true signature, acknowledged in my presence, of

Mr Axel Roth, born on 13 May 1966,

Business address:

R & S consumer goods GmbH

Wasserburger Landstr. 264, 81827 München,

personally known to me and recently identified by his valid German identity card.

Furthermore upon inspection of the Commercial Register of the Munich Court – HRB 117286 - from today I certify, that Mr. Axel Roth as Director (= Geschäftsführer) of the company is entitled to act solely for and on behalf of

R & S consumer goods GmbH

Wasserburger Landstr. 264, 81827 München, Germany,

a German company having its legal place of business in Munich.

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