Contiplex® – Catheterset

B | BRAUN

B. Braun Melsungen AG
Medical Division
P. O. Box 11 20
D-34209 Melsungen
Tel. +49(0)5661 71-0
Fax +49(0)5661 71 27 77

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Instructions for use

Puncture region must be properly shaved, disinfected and be covered with sterile towels. After correct placement of the arm a local anesthetic agent should be injected around the intended puncture site so that a skin-wheel will be set. When the local anesthetic agent is acting an incision should be made with a lancet.

Mount cannula on syringe filled with cold physiological sodium chloride solution. Advance in direction of the axillary artery through the skin incision with constant piston pressure technique at an angle of approx. 30 – 40°.

When reaching neurovascular fascia the tissue resistance increases steadily up to maximum. The pressure felt with the constant piston technique disappears immediately after the cannula has passed the fascia.

Advance Teflon cannula now situated in perivascular space over the steel introducer needle until green plastic connector hub has reached the skin level. Remove introducer needle.

Connect syringe with Teflon cannula and inject cold physiological sodium chloride solution after aspiration control to determine exact positioning of cannula (cause paraesthesia).

After correct positioning of the Teflon cannula, introduce catheter approx. 1-3 cm beyond the cannula tip into the neurovascular fascia.

The cone on the catheter should be plugged into the hub of the Teflon cannula thus increasing the longitudinal stability of catheter considerably (fig. 1).

Catheter markings in cm-distances from 5 to 15 cm:

- 9 cm: broad marking up to 9.5 cm
- 10 cm: 2 rings
- 15 cm: 3 rings
- 20 cm: 4 rings

Using a threading assist guide the catheter leaves the distal end of the cannula with the beginning of the broad marking according to the different cannulae length.

- 33 mm at 7.0 cm (REF 4892402)
- 45 mm at 8.0 cm (REF 4893611)
- 55 mm at 9.0 cm (REF 4892210, 4894235, 4894243, 4893638)
- 100 mm at 14.2 cm (REF 4894391)

Hold catheter in place and remove Teflon cannula.

Insert the catheter as far as possible into Perifix catheter connector and close.

The Catheter-Connector works according to the Click & Ready principle.

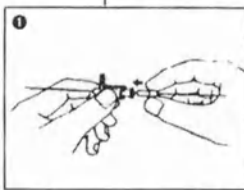
- Open the catheter connector lid as far as possible.
- Insert the catheter into the funnel shaped opening on the connector tip (fig. 2)
- Push the catheter well into the connector until it clearly reaches a stop (fig. 3)
- Close the connector lid by pressing down firmly on the tip of the lid until the lid engages with a clear click (fig. 4)
- After removal of the protective cap attach the connector to the Perifix filter (if provided) with a twist action (Luer Lock) (fig. 5)

If necessary: Re-opening the connector:

Hold the connector firmly between two fingers and press the connector tip on the upper side of the lid firmly downwards. The lid will spring open with an audible click.

CAUTION:

The catheter must be inserted into the connector as far as possible in order to avoid any leakage or disconnection. Do not use if packaging is damaged. Do not resterilise. The catheter connector and injection filter should not be wet with disinfectants containing alcohol as this may result in crack in housing.

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