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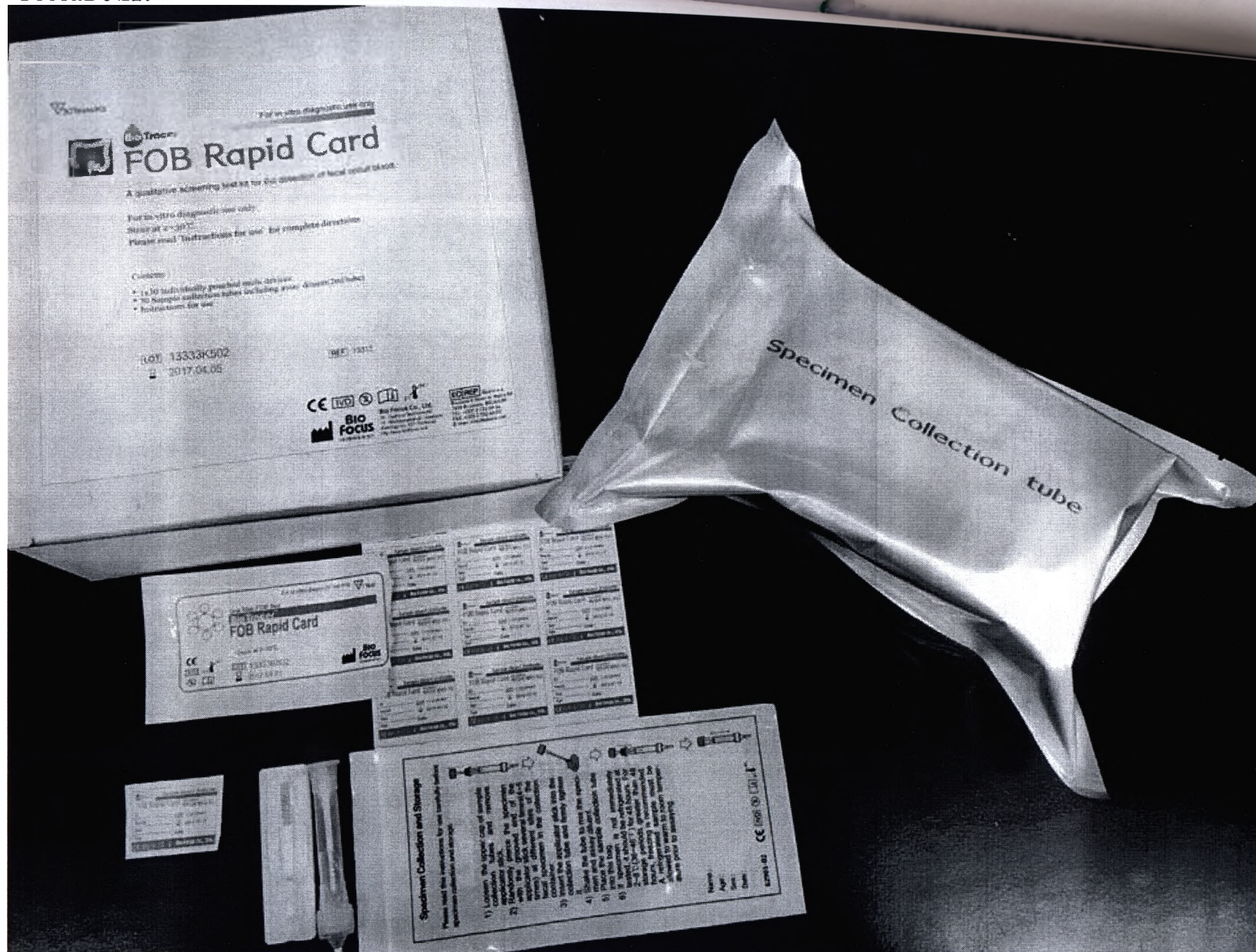
For in vitro diagnostic use only

FOB Rapid Card

A qualitative screening test kit for the detection of fecal occult blood.

For in vitro diagnostic use only

Store at 2-30°C

Please read 'Instructions for use' for complete directions


Contents :

- 1x30 individually pouched multi devices
- 30 Sample collection tubes including assay diluent(2ml/tube)
- Instructions for use

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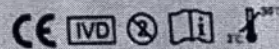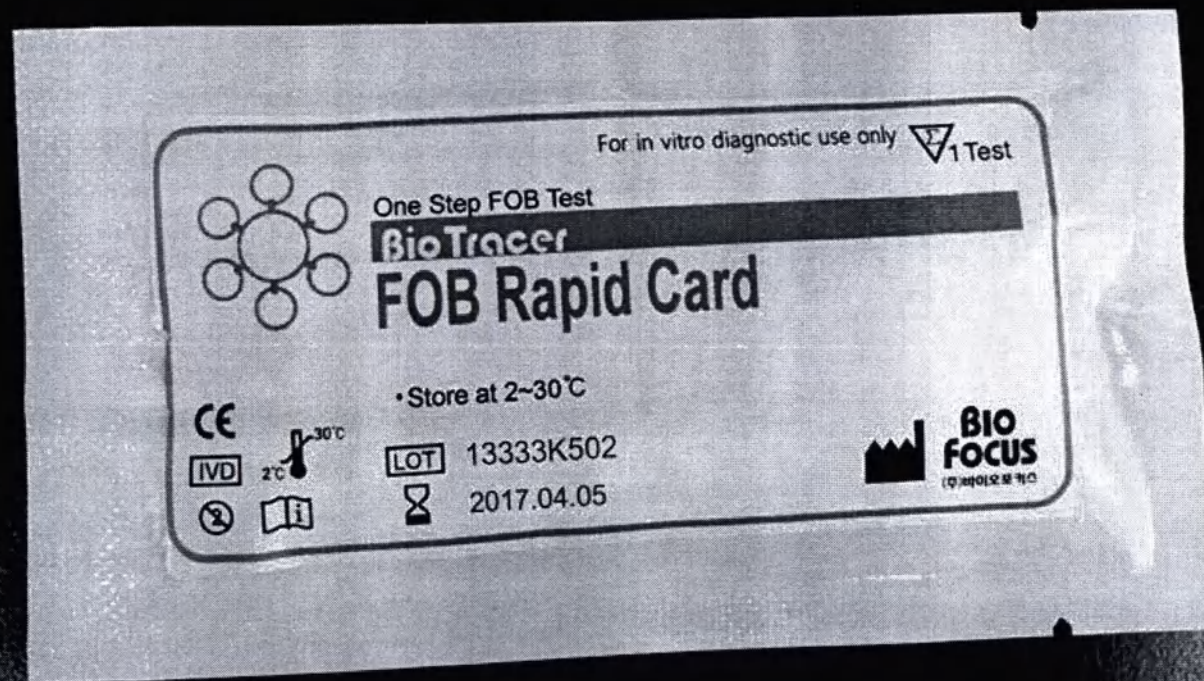
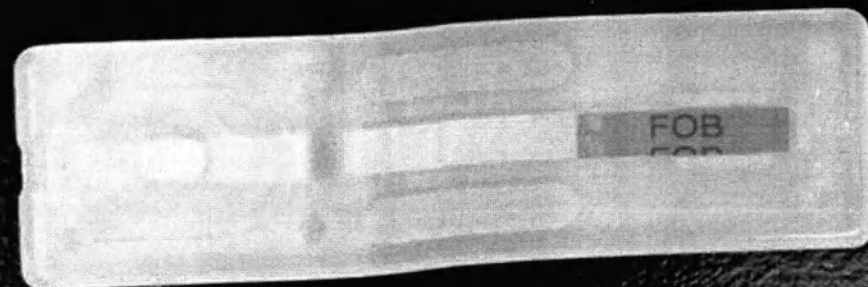
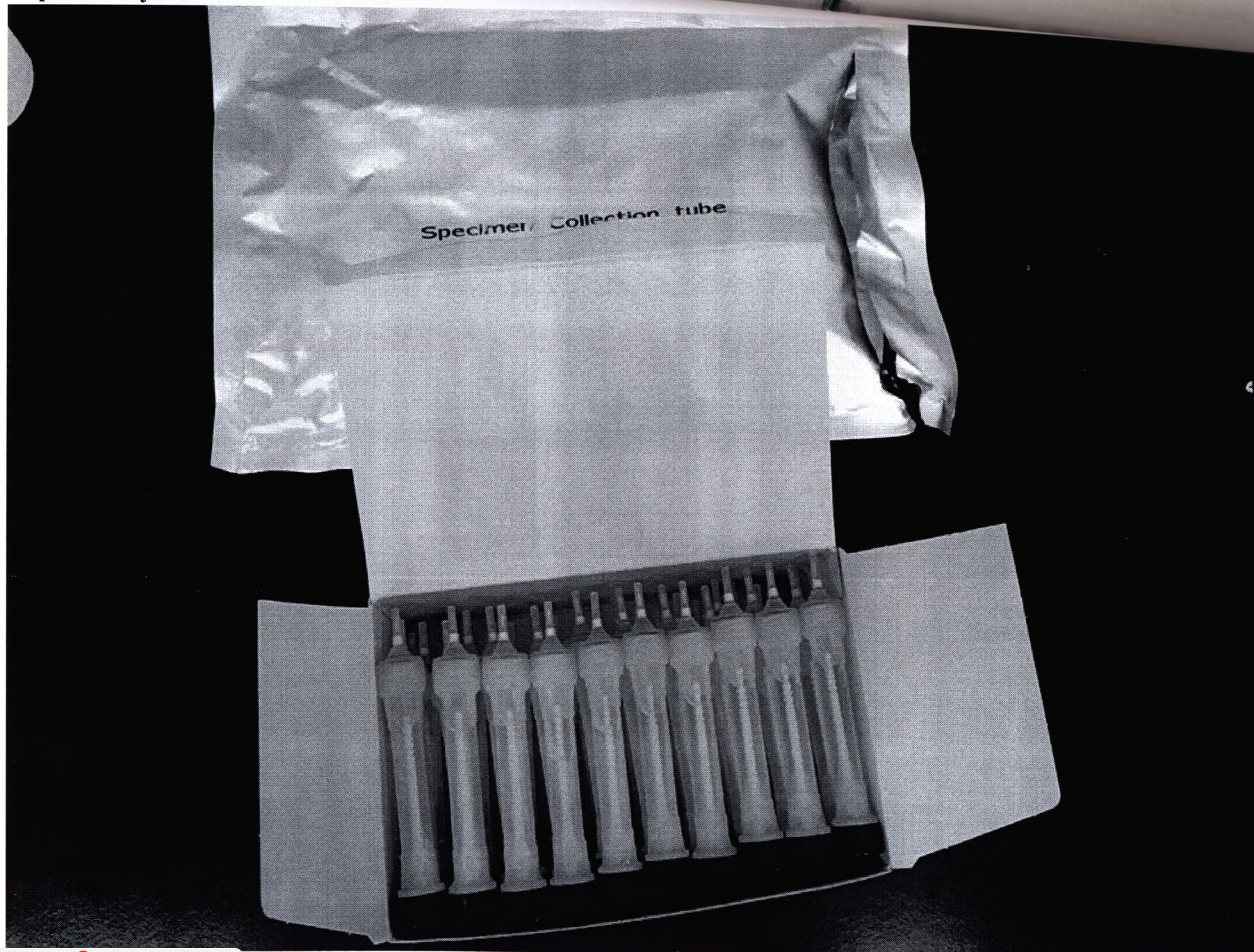
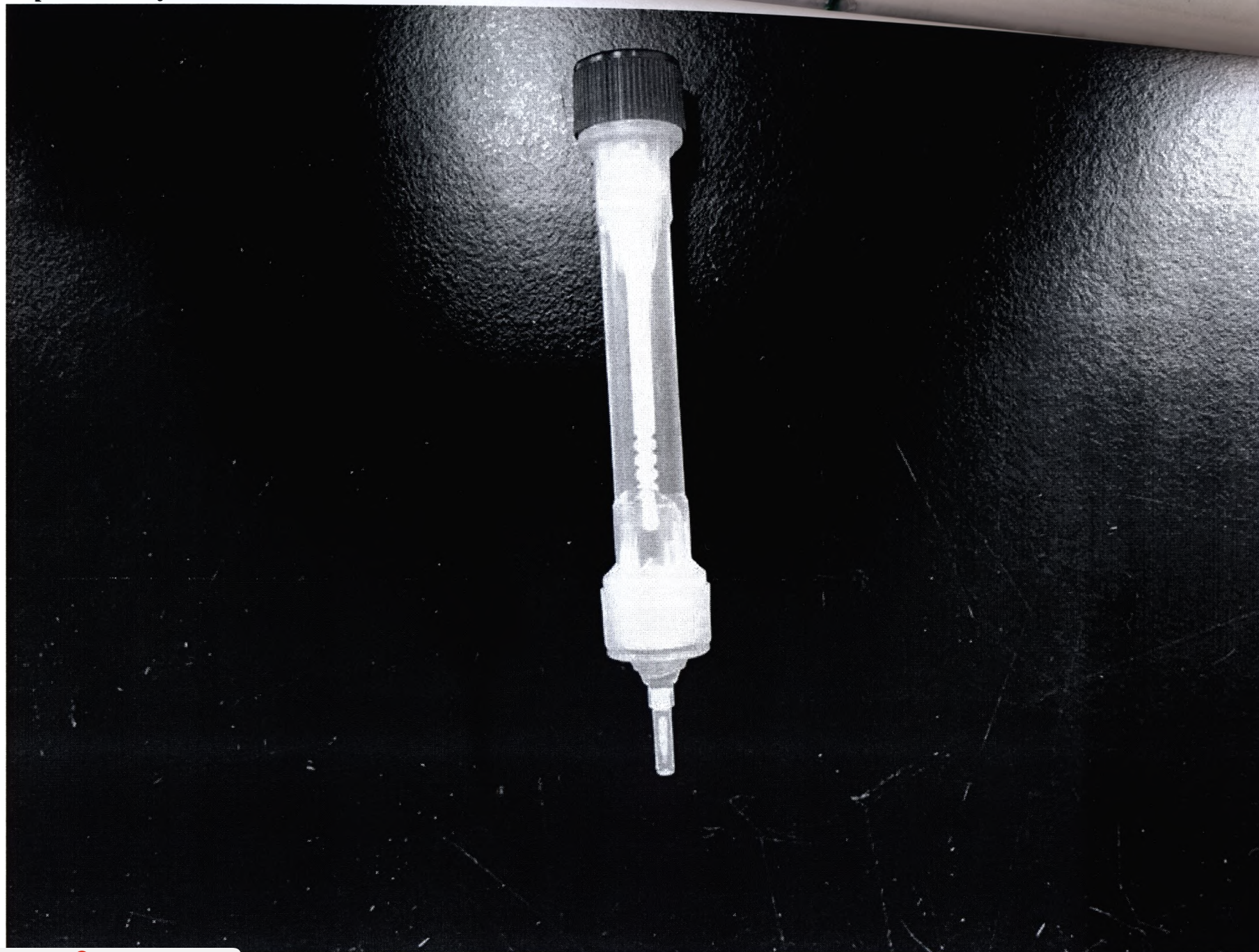
Specimen Collection and Storage

Please read this instructions for use carefully before specimen collection and storage.

- 1) Loosen the upper cap of sample collection tubes and remove applicator stick.
- 2) Randomly pierce the specimen with the grooved end of the applicator stick several times (4-5 times) at different sites of the fecal specimen in the collection container.
- 3) Insert the applicator stick into the collection tube and firmly tighten it.
- 4) Shake the tube to mix the specimen and assay diluent.
- 5) Place the sample collection tube into this bag.
- 6) If specimen is not immediately tested, it should be refrigerated at 2-8°C (36-46°F) for 48 hours. For storage periods greater than 48 hours, freezing is recommended. A refrigerated sample must be allowed to warm to room temperature prior to assaying.

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- 1) Loosen the upper cap of sample collection tubes and remove applicator stick.
- 2) Randomly pierce the specimen with the grooved end of the applicator stick several times (4-5 times) at different sites of the fecal specimen in the collection container.
- 3) Insert the applicator stick into the collection tube and firmly tighten it.
- 4) Shake the tube to mix the specimen and assay diluent.
- 5) Place the sample collection tube into this bag.
- 6) If specimen is not immediately tested, it should be refrigerated at 2-8 °C (36-46 °F) for 48 hours. For storage periods greater than 48 hours, freezing is recommended. A refrigerated sample must be allowed to warm to room temperature prior to assaying.

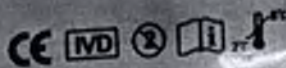
Name :

Age :

Sex :

Date :

62901-02

	Sample diluent 2ml/bottle
FOB Rapid Card	
ID :	LOT 13333K501
Name:	2017.01.05
Sex :	
Age :	Date :
CE   Bio focus co., LTD.	