

3M ESPE
Pentamix™ Lite
Automatic Mixing Unit

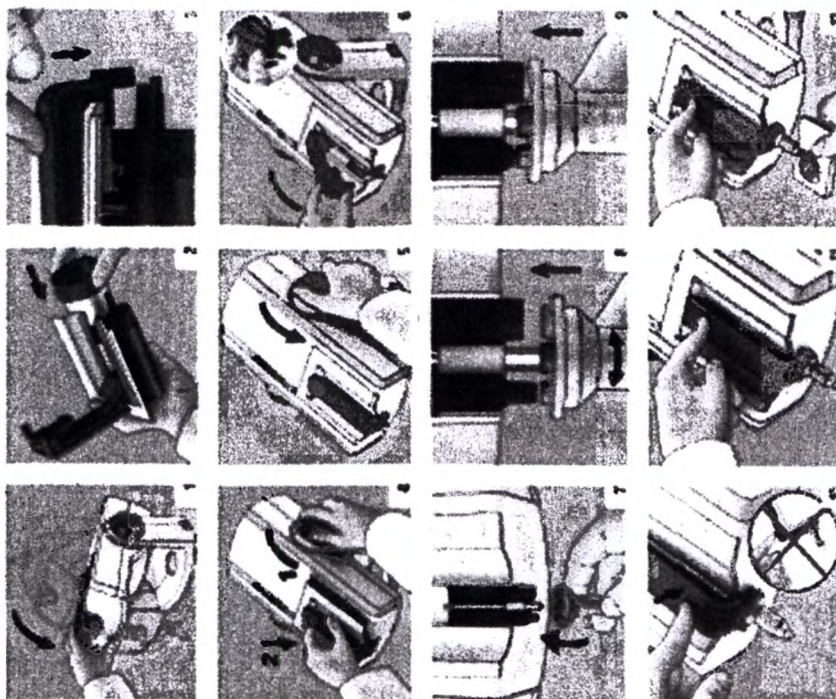3M ESPE
Pentamix™ Lite.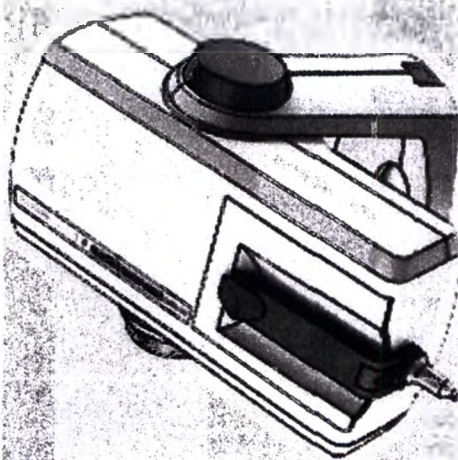

Automatic Marking Unit

ANALYST AND AUTOMATION VECTOR 31 MARCH 1981

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Automated materials

Aracely Quiroz

1. *Chlorophyll a* (Chl a) is the primary photosynthetic pigment in most plants and algae. It is a green pigment that absorbs light energy in the blue and red regions of the visible spectrum. Chl a is essential for the light-dependent reactions of photosynthesis, where it converts light energy into chemical energy in the form of ATP and NADPH.

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THE UNIVERSITY OF CHICAGO

Pentamix™ Lite

Automatic Mixing Unit
Operating Instructions

Table of Contents

	Page
1. Safety Instructions	5
2. Product Description	5
3. Technical Data	6
4. Installation	6
5. Operation	6
5.1 Selecting the Unit into Working Position	6
5.2 Loading the Cartridge	6
5.3 Inserting the Cartridge	6
5.4 Attaching the Mixing Tip	6
5.5 Sticking a New Cartridge	6
5.6 Loading the Impression Tray and Penta Electronic Syringe	6
5.7 Removing the Cartridge	7
5.8 Re-loading a Penta Empty Cartridge	7
5.9 Re-loading a Cartridge	7
5.10 Storage of Partially Used Cartridges	7
5.11 Fill Level Gauge	7
5.12 Parking the Unit	7
5.13 Transporting the Unit	7
5.14 Disposal of Consumable Materials	7
5.15 Return of Old Electric and Electronic Equipment for Disposal	7
6. Malfunctions and Error Messages	8
7. Maintenance and Care	8
7.1 Replacement of the Plunger Discs	8
7.2 Care	8
7.3 Maintenance and Repair	8
8. Customer Information	8
8.1 Warranty	8
8.2 Limitation of Liability	8

1. Safety Instructions

ATTENTION! Read these Operating Instructions carefully before plugging in and operating the unit! As with all technical units, the trouble-free function and safe operation of this unit can be assured only if standard safety procedures and the special safety instructions in these Operating Instructions are observed.

3M Deutschland GmbH guarantees trouble-free operation of the Pentamix system only if 3M ESPE system components are used exclusively.

1. The unit may be used only in strict accordance with the following instructions. We accept no liability for any damage arising from any other or improper use of this unit.
2. If the unit is moved from a cold into a warm environment, condensation can result in a hazardous situation. Never begin operating the unit until it has reached the ambient temperature.
3. The power plug must be connected to a suitably grounded socket before the unit is operated. If an extension cable is used, make sure that it also provides a groundwire.
4. Position the unit so that the power plug is accessible at all times.
5. The unit must be placed on a non-slip, level surface to insure stable placement.
6. Always turn off the unit and disconnect the plug from the power source before changing the plunger discs.
7. When changing the plunger discs, position the smooth side of the new discs towards the foil bag (the opposite side of the discs with the fan-shaped depressions must face away). Otherwise, the foil bag might be damaged.
8. To avoid electric shock, do not insert any objects into the unit.
9. Use only genuine 3M ESPE parts when replacing defective components as directed in these Operating Instructions. We assume no liability for any damage caused by the use of non-3M ESPE parts.
10. If, for any reason, it is possible that the safety of a unit has been compromised, that unit must be removed from operation and identified in such a way that it is not inadvertently operated again by a third party. Safety may have been compromised, for example, if the unit does not work as specified or is visibly damaged.
11. Keep solvents, flammable liquids, and potential heat sources away from the unit as they may damage the plastic housing.
12. Do not allow any cleaning agents to enter the unit during cleaning as they could cause an electric shock or a dangerous malfunction.
13. Only a properly trained, authorized technician may open the unit housing and repair the unit.

2. Product Description

Pentamix™ Lite is an automatic mixing unit for the preparation of Penta® impression materials. 3M ESPE Penta impression materials have been specifically designed for this unit and are supplied in special foil bags. This is the only form of packaging which can be used in the Pentamix Lite units. Every Penta® foil bag is sealed with a PentaSeal® sealing cap. The external geometry of the sealing cap insures its proper positioning on the cartridge. When the Pentamix unit is turned on, a special internal device opens the foil bag automatically once sufficient pressure has been built up by the plunger. The connection pieces located on the PentaSeal® sealing caps serve for proper attachment of the mixing tip.

The cartridges and the PentaSeal® sealing caps act together as a stable unit for the preparation of paste in the Pentamix unit and allow a change of products without intermediate cleaning of the device and with no loss of paste. Do not operate the Pentamix Lite unit using any elements other than Penta® cartridges for Pentamix Lite and Penta® Mixing Tips - Red.

- Tip): if the mixer is operated while containing hardened material, the catalyst nipple of the sealing cap can be damaged or may break away.
- Put the Perma Elastomer Syringe (after removing the plunger) with its fill opening directly onto the mixing tip and press the "Start" button (Fig. 4).
- CAUTION:** Not all Perma Impression materials can be syringed. Please refer to the instructions for use of the respective product.

5.7 Removing the Cartridge

- Do not press the "Start" button while removing the cartridge.
- Turn the plunger adjustment knob anticlockwise with your right hand until it stops and rod it in place. The plungers are now in the uppermost position.
- Using your left hand, remove the cartridge from the device (Fig. 5).

5.8 Re-starting a Partially Empty Cartridge

- After prolonged storage (especially for silicone), use an instrument to remove any plugs that may have formed from the outflow openings of the fill bags.
- It may be necessary to start the unit without a mixer to check the paste flow.
- Attach a new mixer and start the mixing procedure.

5.9 Replacing a Cartridge

A cartridge is easily empty if we measured that you have a second cartridge ready for use at hand so that you will be able to replace the exhausted cartridge quickly. This cartridge should be prepared as described under 5.5 "Starting a New Cartridge".

Do not forget to move the plungers into contact with the bags!

Scheme of the mixing tip: see 5.6

5.10 Storage of Partially Used Cartridges

- Fill bags which have been started and partially emptied must be stored only in the cartridge and with attached and filled mixer.
- Store any cartridges with partially used fill bags either horizontally or with the mixing device (Fig. 6). If stored separately, the impression material may flow backwards, which would prevent the reentering flow or cause the base and catalyst pastes to mix.
- When there is a partially used cartridge in the unit and it is in park position, the plunger does not let the fill bags in the cartridge. It is not necessary to remove the cartridge before putting the unit.

5.11 Fill Level Gauge

The fill level gauge shows how much material remains in the fill bags (Fig. 7).

When the fill bags are completely empty, the red fill gauge marker disappears, and the unit starts off automatically.

5.12 Parking the Unit

The Perma-Lite unit can easily be tilted into a compact park position when its operation is temporarily not required.

- When mixing the unit, use caution to avoid the possibility of injury from jamming or crushing your fingers.

- Before leaving the mixing apparatus of the unit containing a cartridge, always be sure that the plunger does not in the cartridge and that the mixing tip which has been used is still in place. The cartridge may otherwise fall out of the unit and be damaged.
- With one hand, steady the mixing apparatus of the Perma-Lite mixing unit backward from the working position until it audibly snaps into the park position (Fig. 8).
- Do not push the unit beyond the first position as this could lead to technical problems.

5.13 Transporting the Unit

- When in the park position, the Perma-Lite unit can be easily transported and stored.
- When transporting the unit, use caution to avoid the possibility of injury from jamming or crushing your fingers.
- Move the unit into the park position before transport: see 5.2.
- Turn off the power switch on the Perma-Lite unit and remove the power cable from the unit.
- Grasp the unit from the side with one hand (Fig. 20) and carry it using the carrying handle in the housing (Fig. 19).

5.14 Disposal of Consumable Materials

Dispose of used mixing tips and empty fill bags in suitable waste. Be sure to comply with local waste disposal regulations.

5.15 Return of Old Electric and Electronic Equipment for Disposal

- Collection**
Users of electric and electronic equipment are required to collect their old equipment separately from other waste in accordance with the regulations of the specific country. Old electric and electronic equipment must not be disposed of with unsorted household waste. This separate collection is a prerequisite for recycling and reprocessing as an important method for preserving environmental resources.

2. Return and Collection Systems

When your Perma-Lite is no longer usable, do not dispose of it in dumps with household waste. B&B Deutschland GmbH has set up special disposal facilities to handle the equipment. Discuss about the procedure for the specific country can be obtained from the pertinent B&B subsidiary.

3. Meaning of the Symbols

The EU Directive prohibits the disposal of any electric or electronic device marked with these symbols in combination with household waste.

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