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**certified copy**

Carl-Schurz-Straße 1  
41453 Neuss  
Germany

Postal Address:  
ESPE Platz  
82229 Seefeld  
Germany

☎ +49 (0)8152 700-0  
☎ +49 (0)8152 700-1366  
Internet: [www.3MESPE.com](http://www.3MESPE.com)  
E-Mail: [info3MESPE@3M.com](mailto:info3MESPE@3M.com)  
WEEE-Reg.-Nr. DE 36963167  
VAT-ID: DE 120679179

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To whom it may concern

February 09, 2016

### Registration documents for Express XT - IFU

We 3M Deutschland GmbH, Carl-Schurz-Str. 1, 41453 Neuss, herewith confirm that the following pages are an excerpt of the Information for Use.

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3M, ESPE, Express, Garant, Penta and Pentamix  
are trademarks of 3M or 3M Deutschland GmbH.  
© 2015, 3M. All rights reserved.

44000768786/02  
(44 00 07 687 86/02)  
(4400 0768883/02)

## en ENGLISH

### Product Description

These Instructions for Use apply to the subsequently listed vinyl polysiloxane impression materials, which are used for one-step and two-step impression technique.

Not all products are available in all countries.

| Product Name                   | ISO 4823 | ISO 4823 Consistency | Paste Color | Mixing Ratio (Vol.) | Mixing Device                   |
|--------------------------------|----------|----------------------|-------------|---------------------|---------------------------------|
| Express™ XT Penta™ Putty       | Type 0   | putty                | green       | 5:1                 | Pentamix 2 and younger versions |
| Express™ XT Penta™ Putty Soft  | Type 0   | putty                | ocher       | 5:1                 | Pentamix 2 and younger versions |
| Express™ XT Penta™ H           | Type 1   | heavy-bodied         | ocher       | 5:1                 | all Pentamix versions           |
| Express™ XT Penta™ H Quick     | Type 1   | heavy-bodied         | green       | 5:1                 | all Pentamix versions           |
| Express™ XT Regular Body       | Type 2   | medium-bodied        | petrol      | 1:1                 | Garant Dispenser                |
| Express™ XT Regular Body Quick | Type 2   | medium-bodied        | caramel     | 1:1                 | Garant Dispenser                |
| Express™ XT Light Body         | Type 3   | light-bodied         | magenta     | 1:1                 | Garant Dispenser                |
| Express™ XT Light Body Quick   | Type 3   | light-bodied         | pink        | 1:1                 | Garant Dispenser                |

Express XT Penta Putty and Express XT Penta Putty Soft can only be mixed in Pentamix versions from September 1999 on; earlier Pentamix units (before September 1999) extrude the material too slowly.

Express XT Penta H can be mixed in all versions of Pentamix mixing devices. All other products are statically mixed and directly applied using the Garant™ Dispenser.

- These Instructions for Use should be kept for the duration of product use. For all additionally mentioned products, please refer to the corresponding Instructions for Use.

### Indications

- All precision impressions (e.g.: crown, bridge, inlay, and onlay preparations)

### Precautionary Measures

3M MSDSs can be obtained from [www.mmm.com](http://www.mmm.com), or contact your local subsidiary.

### Preparation

- Block out undercuts or interdental spaces sufficiently to aid impression removal from the teeth after setting as otherwise impression removal from the mouth will be difficult or could lead to the extraction of natural teeth or prostheses. Blocking is particularly necessary when using Express XT Penta Putty or any other impression material with increased shore hardness.

### Impression Tray:

All impression trays generally used for precision impressions are suitable.

Rigid impression trays are recommended for two-step impression technique and when using Express XT Penta Putty and Express XT Penta Putty Soft.

- For sufficient adhesion, apply VPS Tray Adhesive on the tray and allow it to dry completely, the ideal drying time is 5-15 minutes.

#### **Penta Material:**

- ▶ Place the Penta foil bag into the designated 3M ESPE Penta cartridge.
- ▶ Insert the loaded cartridge into the Pentamix mixing unit.
- ▶ Attach a red Penta™ Mixing Tip.
- ▶ Turn the plungers of the mixing unit downward with the handwheel until resistance is noticed; do not force.
- ▶ When using new foil bags, start mixing and extrude a small amount of paste until it is uniform in color. Discard this first amount of paste.
  - Paste color: please see "Product Description".
  - Material that has set inside the Penta Mixing Tip should not be extruded by force as this may damage the Penta Cartridge and Penta Mixing Tip and result in the formation of leaks.
- ▶ Please follow the section "Impression Taking" for tray filling procedure.

#### **Garant Dispenser:**

- ▶ Insert the cartridge with the wash impression material into the Garant Dispenser.
- ▶ Prior to application, check that both Garant cartridge openings are not clogged.
- ▶ When using new Garant cartridges, extrude a small amount of paste without the mixing tip until base paste and catalyst are evenly extruded.
- ▶ Attach a yellow Garant™ Mixing Tip to the cartridge.
- ▶ For intraoral application, attach a yellow Garant Intraoral Tip onto the Garant Mixing Tip.
- ▶ If required: enlarge the exit opening for the intraoral tip by cutting it with a scalpel.
- ▶ Make sure that base paste and catalyst are mixed completely and are extruded in a uniform color.
  - Paste color: see "Product Description".
  - Material that has set inside the Garant Mixing Tip should not be extruded by force as this may damage the Garant cartridge and Garant Mixing Tip and result in the formation of leaks.

#### **Retraction**

Suitable retraction agents include aluminum hydroxide chloride, aluminum sulfate, or ferric sulfate solutions.

- ▶ Keep the areas from which the impression is to be taken dry during impression.
- ▶ The use of retraction cords may be required for subgingival preparations.
- ▶ Prior to the impression, thoroughly remove residual retraction material by flushing and drying.

#### **Retraction**

Suitable retraction agents include aluminum hydroxide chloride, aluminum sulfate, or ferric sulfate solutions.

- ▶ Keep the areas from which the impression is to be taken dry during impression.
- ▶ The use of retraction cords may be required for subgingival preparations.
- ▶ Prior to the impression, thoroughly remove residual retraction material by flushing and drying.

#### **Times**

|                             | Working Time<br>at 23°C/74°F<br>from start of<br>mixing*<br>min:sec                 | Intraoral<br>Syringe<br>Time<br>min:sec                                             | Intraoral<br>Setting<br>Time<br>min:sec                                             |
|-----------------------------|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|
| Symbols on product          |  |  |  |
| Express XT Penta Putty      | 1:30                                                                                | -                                                                                   | 3:00                                                                                |
| Express XT Penta Putty Soft | 1:30                                                                                | -                                                                                   | 3:30                                                                                |
| Express XT Penta H          | 2:00                                                                                | -                                                                                   | 3:30                                                                                |
| Express XT Penta H Quick    | 1:30                                                                                | -                                                                                   | 2:30                                                                                |

|                                                                           | Working Time<br>at 23°C/74°F<br>from start of<br>mixing*<br>min:sec | Intraoral<br>Syringe<br>Time<br>min:sec | Intraoral<br>Setting<br>Time<br>min:sec |
|---------------------------------------------------------------------------|---------------------------------------------------------------------|-----------------------------------------|-----------------------------------------|
| Express XT Regular Body                                                   | 2:00                                                                | 1:00                                    | 3:30                                    |
| Express XT Regular Body Quick                                             | 1:30                                                                | 0:40                                    | 2:30                                    |
| Express XT Light Body                                                     | 2:00                                                                | 1:00                                    | 3:30                                    |
| Express XT Light Body Quick                                               | 1:30                                                                | 0:40                                    | 2:30                                    |
| Intraoral setting time when using one-step technique in combination with: |                                                                     |                                         |                                         |
| Express XT Penta Putty                                                    |                                                                     |                                         | 3:00**                                  |
| Express XT Penta Putty Soft or<br>Express XT Penta H                      |                                                                     |                                         | 3:30                                    |
| Express XT Penta H Quick                                                  |                                                                     |                                         | 2:30<br>(using Quick wash materials)    |

\* Start of mixing = Entry of paste into the mixing tip.

\*\* Due to the higher temperature of the Penta Putty paste extruded from the Pentamix mixing unit, the intraoral setting of the syringe material is accelerated.

These are the processing times for products that have been stored at a temperature of 23°C/74°F and 20%–80% relative humidity. These times are reduced due to higher temperatures of the product or prolonged if the temperature of the product is lower.

### Impression Taking

#### One-step Impression Technique:

| Recommended product combinations                                                  |                                                                 |
|-----------------------------------------------------------------------------------|-----------------------------------------------------------------|
| Express XT Penta Putty or<br>Express XT Penta Putty Soft or<br>Express XT Penta H | Express XT Light Body or<br>Express XT Regular Body             |
| Express XT Penta H Quick                                                          | Express XT Light Body Quick or<br>Express XT Regular Body Quick |

- ▶ At the start of mixing the Penta impression material, be sure to observe the entry of the paste into the Penta Mixing Tip and ensure that base paste and catalyst paste flow evenly into the mixing tip. The mixed paste must be extruded in a uniform color free of streaks.
  - Paste color: see "Product Description".
- ▶ Load the impression tray prepared with adhesive, keeping the mixing tip permanently immersed in the paste to avoid the formation of air bubbles.
- ▶ While the tray is being loaded, apply the wash impression material around the cleaned and dried preparation from the bottom up. Keep the application nozzle permanently immersed in the paste to avoid the formation of air bubbles.
  - Depending on the number of units to be syringed, initiate application so that the tray loading and application of material around the preparation are completed simultaneously.
  - Do not exceed processing times of tray material and wash material to avoid incomplete and inaccurate impressions.
- ▶ Slowly insert the loaded tray into the mouth parallel to the long axes of the prepared teeth and hold it in place without exerting pressure.
- ▶ Remove the impression from the mouth after completed setting.

#### Two-step Impression Technique:

| Recommended product combinations                      |                                                         |
|-------------------------------------------------------|---------------------------------------------------------|
| Express XT Penta Putty or<br>Express XT Penta H Quick | Express XT Light Body or<br>Express XT Light Body Quick |

- ▶ After start of mixing the Penta impression material, be sure to observe the entry of the paste into the Penta Mixing Tip and ensure that base paste and catalyst paste flow evenly into the mixing tip. The mixed paste must be extruded in a uniform color, free of streaks.
  - Paste color: see "Product Description".
- ▶ Load the impression tray prepared with adhesive, keeping the mixing tip permanently immersed in the paste to avoid the formation of air bubbles.
- ▶ Slowly insert the loaded tray into the mouth parallel to the long axes of the prepared teeth and hold it in place without exerting pressure.
- ▶ Remove the impression from the mouth after completed setting; clean and dry.
- ▶ Excise undercuts and interdental septa, etc., and cut spiliways if required.
- ▶ Clean and dry the impression thoroughly to provide for optimal coalescence of the two materials.
- ▶ Insert the wash impression material into the preliminary impression at the appropriate sites and/or apply around the cleaned and dried preparations, while keeping the application nozzle permanently immersed in the paste to avoid the formation of bubbles.
- ▶ Replace the preliminary impression in the mouth and hold it in place.
- ▶ Remove the impression from the mouth after completed setting.

#### After Impression Taking

- ▶ Thoroughly examine and explore the sulcus of the prepared teeth and surrounding dentition. Remove any residual cured impression material from the mouth.

#### Hygiene

- ▶ Place the impression in a standard disinfectant solution. The duration complies with the instructions of the manufacturer. Do not leave the impression submerged in the disinfection solution for more than the specified amount of time as otherwise the impression may be damaged.
- ▶ After disinfection, clean the impression of any remaining disinfection solution by rinsing under running water for approx. 15 seconds.

#### Model Preparation

- ▶ If Express XT Penta Putty or Express XT Penta Putty Soft have been used to take an impression, the impression can be poured two hours after removal from the mouth. Impressions of all other product combinations can be poured 30 minutes after removal from the mouth at the earliest. There are no other time limitations for pouring.
- ▶ In order to attain a bubble-free model, rinse the impression with water shortly before pouring, remove excess water or use a silicone surfactant.
- ▶ Pour the impression with a standard dental die or model stone.  
The impressions can be silver or copper plated.

#### Cleaning

- ▶ Garant Dispenser: Remove paste that has not set using an alcohol-soaked cloth. The dispenser handle and plunger can be autoclaved up to a temperature of 135°C/275°F. Disassemble the dispenser before autoclaving. Glutaraldehyde-based solutions can be used for disinfection.
- ▶ Prior to setting, the paste can be removed from surfaces with ethanol.
- ▶ Avoid exposing the material to clothing since the material cannot be removed by chemical cleaning.

#### Notes

- Completely remove the oxygen inhibition layer of composite materials, e.g., fillings or core buildups; it may impair or retard the setting of silicone impression materials.
- Keep the filled Penta or Garant Mixing Tip on top of the cartridge to serve as a closure until the next use. Removing and replacing the spent Penta or Garant Mixing Tip may lead to carry-over of paste and ensuing formation of clogs.
- Vinyl polysiloxane impression materials must not be combined with condensation type silicones or polyether impression materials. Even small traces can affect the setting process.
- Disposable latex gloves impair the setting of silicone impression materials. Vinyl gloves are more suitable.
- Under no circumstances may impressions come into contact with solvent-containing fluids. This could result in expansion and inaccurate stone models.

**Storage and Stability**

Store the product at 15–25° C/59–77° F.

Do not use after the expiration date.

Store set impressions at temperatures below 30° C/86° F.

**Customer Information**

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

**Warranty**

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

**Limitation of Liability**

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

Information as of February 2015

RUS: Authorised representative of the manufacturer in RF:

ZAO "3M Russia", Russia 121614, Moscow, Krylatskaya str. bldg. 17/3  
T. +74957847474

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