Coloplast A/S
Holtedam 1
DK-3050 Humlebæk
Denmark

Tlf. 49 11 11 11
Tel. +45 49 11 11 11
www.coloplast.com

Coloplast

Approved
15 May 2016
by

RA Coordinator,
Lene Vibeke Jacobsen,
Coloplast A/S


Lene Vibeke Jacobsen

APPLICATION GUIDANCE

Alterna® New Generation One-Piece Ostomy Care Products, design variants:

- **Alterna® One-Piece closed bags, transparent, nontransparent, with filter, of different sizes and volumes, mini caps, with integrated flat and convex baseplates, different sizes of cut holes, for adults and children (paediatric)**
- **Alterna® Free One-Piece closed bags, transparent, nontransparent, of different sizes and volumes, with filter, with integrated flat and convex baseplates, different sizes of cut holes**

by Coloplast A/S, Denmark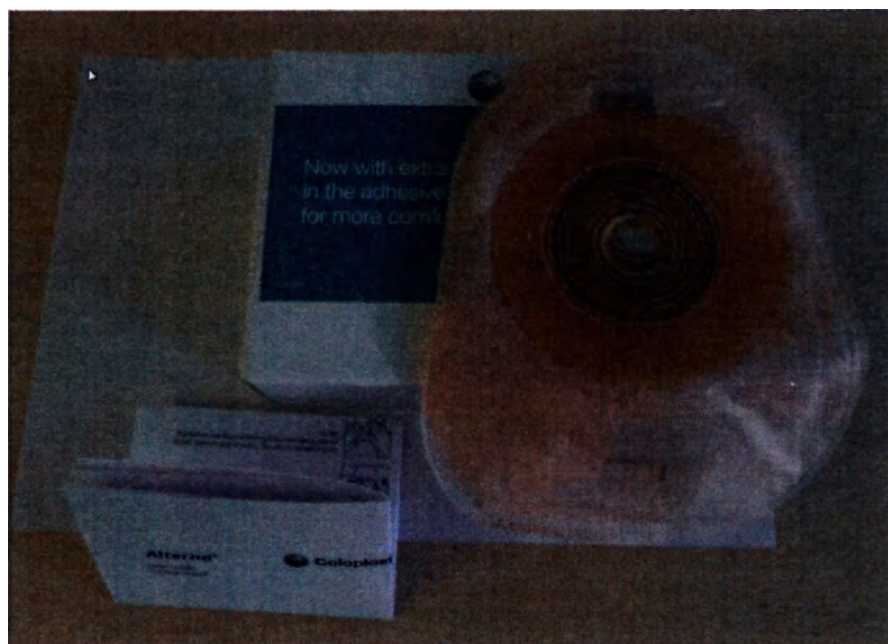

Gebruik: 392680 01 0000,0017
60200-00578/2016

16-08-2016
200,00

Coloplast A/S, Denmark, 2016

NAME OF THE MEDICAL DEVICE

Alterna® New Generation One-Piece Ostomy Care Products, design variants:

- **Alterna® One-Piece closed bags, transparent, nontransparent, with filter, of different sizes and volumes, mini caps, with integrated flat and convex baseplates, different sizes of cut holes, for adults and children (paediatric)**
- **Alterna® Free One-Piece closed bags, transparent, nontransparent, of different sizes and volumes, with filter, with integrated flat and convex baseplates, different sizes of cut holes**

Medical device production sites

1. **Coloplast A/S**, Høtvedvej 1, 3050 Humlebaek, Denmark
2. **Coloplast Hungary KFT**, Buzavirag Ut.15, 2800 Tatabánya, Hungary

COMPLETE SET OF THE MEDICAL DEVICE

30 one-piece closed ostomy bags Alterna ® or Alterna® Free with integrated flat baseplates or 10 one-piece closed ostomy bags Alterna ® or Alterna® Free with integrated convex baseplates with Instruction for Use of the Medical Device in carton box.

PURPOSE

To collect intestinal discharge from the stoma and protect the peristomal area skin in patients who have had an operation with colostomy exteriorization onto the abdominal wall.

Field of application: coloproctology.

Intended use: for patients with colostomies.

Class according to the potential risk level – I according to GOST R 51609-2000 and the order of the Ministry of Healthcare of Russia No. 4н of (regarding the approval of the nomenclative classification of medical devices)

Code according to All-Russian Classification of Products **93 9890**

Type of the medical device according to the nomenclative classification of medical devices – **152450**

COMPOSITION, DESCRIPTION, ACTION

1 piece open ostomy bag Alterna® and Alterna® Free is a closed ostomy bag with integrated flat or convex hydrocolloid adhesive plate with spiral structure.

The spiral structure of the Alterna® adhesive consisting of alternating adhesive layers ensures reliable adhesion and protection.

The light adhesive layer is absorbent retaining the normal skin condition and pH level.

The dark adhesive layer ensures reliable adhesion.

Such a combination of adhesives ensures good skin protection from intestinal discharge and painless removal of the ostomy bag baseplate.

The Alterna® adhesive flows at the skin's natural temperature and fills all the skin wrinkles and contours, preventing intestinal waste leakage under the plate but not tightening the skin or preventing body movement.

Alterna® adhesive baseplates are thick at the stoma and become thinner towards the edges.

The new-generation Alterna® baseplates are distinct in their combination of safety and ability to retain healthy skin under the baseplate.



Safety:

- the plates are safely stuck to the peristomal skin and removed without pain
- erosion-resistant, protecting the skin from irritation

Retaining healthy skin under the baseplate:


- absorbing excessive moisture from the peristomal skin surface
- maintaining the skin's natural pH

Due to a more flexible basefilm, the new-generation Alterna® baseplates are softer, more flexible and the 6 flex zones make the plate follow body's contours enabling ostomy patients to move freely and creating additional comfort during their use.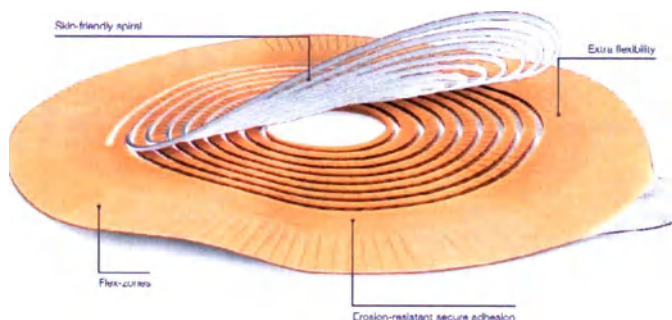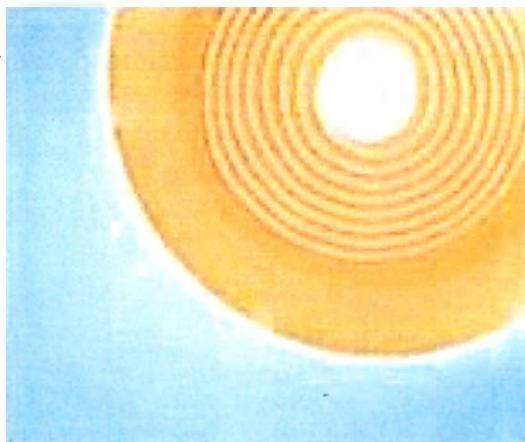

The adhesive baseplates of new-generation Alterna® one-piece closed ostomy bags have a thin and soft protective cover with a cutting guide for cutting a hole for the stoma, and a modified ear to remove the cover which prevents rolling and sticking to the adhesive.

1 piece closed ostomy bags with integrated flat and convex baseplates can be used by patients with normal retracted and flat stomas.

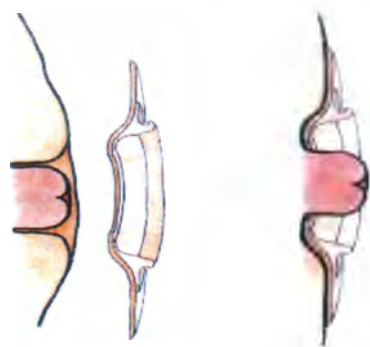The enhanced flange of convex plates of ostomy bags Alterna® и Alterna® Free provides pressure on the peristomal area allowing the stoma move outwards the skin level or skinfold.

1 piece closed ostomy bags with integrated flat and convex baseplates can be used by patients with normal retracted and flat stomas.

1 piece closed ostomy bags Alterna® have symmetric anatomic shape, integrated plates are round.

1 piece closed ostomy bags Alterna® Free have asymmetric anatomic shape, integrated plates are oval, therefore they have different maximum cut hole horizontally and vertically.

1 piece closed ostomy bags Alterna® should be changed 2-3 times a day, so the adhesive layer of the adhesive plate of 1 pc closed bag have been investigated for frequent change of the ostomy bags; the plate of the ostomy bags can be easily removed without skin damaging.

The bags of 1 piece closed ostomy bags Alterna® и Alterna® Free are made of multilayer odour-resistant transparent or non-transparent film, have the filter, beige one-sided soft non-woven layer (closed ostomy bags Alterna® and transparent closed ostomy bags Alterna® Free) or beige two-sided soft non-woven (non-transparent bags Alterna® Free), The non-woven permits the skin under the ostomy bag to breathe making the bag agreeable to wear and protecting from irritation. One-sided soft non-woven layer and two-sided soft non-woven are strong, easy to clean and fast drying.

1 piece closed ostomy bags Alterna® and Alterna® Free are also comfortable to use due to the unique built-in multi-layer filter, which neutralizes the odour and regulates the amount of the gas in the bag.

Alterna® Ostomy Bag has a filter rectangular in shape. The carbon filter of Alterna® Free Ostomy Bags are larger in size than that of Alterna® Ostomy Bags, has a banana-shape.

1 pc closed ostomy bag Alterna® and Alterna® Free can have different size and volume: for children (paediatric), maxi, midi, mini, mini-caps.

1 pc closed ostomy bag Alterna® mini-cap is a one-piece closed bag of minimum volume integrated hydrocolloid adhesive plate with spiral structure. Consisting of alternating adhesive layers and indicated for the patients with colostomies.

The adhesive baseplate have a protective cover with a cutting guide for cutting a hole for the stoma. The bag is made of multilayer odour-resistant non-transparent film, have built-in multi-layer filter and an inner absorbent pad. Cut hole is 20-55 mm.

Alterna® mini-caps are designed for patients with a colostomy (with solid stool and rare regular discharge) and can be used for bathing, outdoor activities and sports.

1 piece closed ostomy bags Alterna® are convenient, reliable and leak-free, they have an anatomical shape and are discreet under clothing.

INDICATIONS / CONTRAINDICATIONS

Indications

Colostoma with solid or demi-solid intestinal discharge.

Contraindications:

Absent

USE GUIDELINES

Information

Coloplast accepts no liability for any injury or loss that may arise if this device is used in a manner contrary to Coloplast's current recommendations

Warning


- Re-use of the single use product may create a potential harm to the user.
- Reprocessing, washing, disinfection and/or sterilization may compromise product characteristics, causing additional risk of physical harm or infection to the user.

Attention

- Consult your health care professional before you start using ostomy bag with integrated convex plates.

Preparation

- Cut a hole in the adhesive to fit the exact size and shape of the stoma using a pair of small curved scissors and the cutting guide on the adhesive. You can use the enclosed stoma measuring guide to measure the size and shape of the stoma. An exact fit between the hole and the stoma is important to reduce the risk of leakage and skin problems.

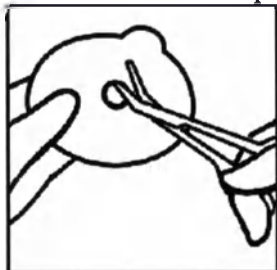
- Thoroughly clean the stoma and the surrounding skin area. The skin must be completely dry before application of the adhesive.

Application

- Peel off the protective film from the adhesive.

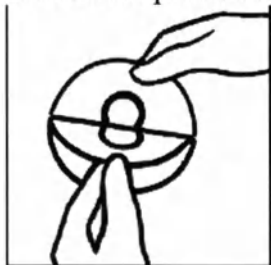- Align the baseplate around the stoma and apply pressure on the baseplate to ensure fast **adhesion and a perfect fit between the adhesive and the skin around the stoma.**

- Apply pressure on the bag to ensure fast adhesion and a perfect fit between the adhesive and the skin around the stoma.


Removal

- Gently remove the bag by pulling the removal ear on the adhesive away from the stomach and roll the adhesive slowly downwards.
Apply light pressure on your skin with your free hand to help removal.

Filter label

- If required, to prevent gas from escaping, place one of the enclosed filter labels on the filter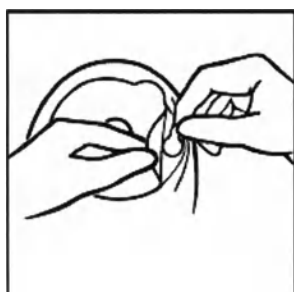

OPERATING CONDITIONS

At 20 - 30 °C, relative humidity 40 - 70 %; atmospheric pressure 720 - 780 mmHg

STORAGE CONDITIONS AND SHELF LIFE

Store in the original manufacturer's package in accordance with storage conditions 2 of GOST 15150.

Store in a horizontally in a dry place protected from light at the room temperature (from 8 to 25°C).

Do not expose to heating and freezing.

The devices are stable until the labeled expiry date, provided that the recommended storage conditions are assured. The shelf life is 3 years.

TRANSPORTATION

Transportation can be arranged by any means of transport according to the medical device transportation requirements in a horizontal position at 0 to 25°C, relative humidity 40-60%.

Avoid direct sunlight, heating and freezing.

DISPOSAL

New-Generation One-Piece Ostomy Care Products: Alterna closed ostomy bags belong to class A according to the hazard classification (according to sanitary regulations and standards SanPiN 2.1.7.2790-10).

Dispose the used materials according to the specified hazard class of medical wastes in accordance with SanPiN 2.1.7.2790-10.

Dispose of the used device with the household waste.

STERILIZATION

A non-sterile device.

LIST OF NATIONAL AND INTERNATIONAL NORMATIVE DOCUMENTS/STANDARDS, WHICH THE MEDICAL DEVICE COMPLIES TO

The document complies with the requirements of the national standards: GOST R 50444-92, GOST R ISO 10993-2011 (parts 1, 5, 9, 10, 11, 12, 18), GOST R ISO 10993-2009 (part 2), GN 2.3.3972-00, MU1.1.037-95, GOST R 52770-2007, GOST 31214-2003, Collection of guiding teaching materials for toxic-hygienic examination of polymeric materials and products on the basis of medical supplies MZ USSR, 1987

The document complies with the requirements of the international standards: EC Declaration of Conformity to Council Directive 93/42/EEC, Certificate of compliance to quality system ISO 9001:2008, ISO 13485-2003, DS/EN ISO 13485-2012

LABELING

	Manufacturing date
	Lot Number
	Expiry date
	Catalogue Number
	Manufacturer name and address
	Do not reuse
	Keep away from sunlight (printed on retail and shipper box)
	Horizontal storage
	See Instructions for Use

WARRANTY LIABILITIES AND RECLAMATIONS

Manufacturer:

Representative Office in Russia:

Coloplast A/S, Høtvedam 1, 3050
Hømlebaek, Denmark;
tel./fax: +45 49 11 11 11;
www.coloplast.com

Coloplast LLC
Bldg 2, 72 Leningradsky Highway, 125315 Moscow,
Russia
Tel.: +7,495,937 53 90 Fax: +7,495,937 53 91,
info@coloplast.ru

This is to certify that **Mrs. Lene Vibeke Jacobsen** today in my presence at the Notarial Office approved and signed the above document.

No conspicuous corrections or addenda were found in the document.

She has proved her identity by showing driving license.

The district court in Elsinore (Helsingør), Denmark, August 16, 2016.

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Re: Confirmation of Coloplast A/S, RA Coordinator, about documentation on the Medical Device **Alterna® New Generation One-Piece Ostomy Care Products, design variants:**

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To whom it may concern:

Dear Sirs,

Coloplast A/S in the person of Lene Vibeke Jacobsen, RA Coordinator, confirms the correctness of the Addendum to Application Guidance, compiled by Coloplast A/S.

This Addendum has been compiled for registration of the medical device

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in the Russian Federation.

Lene Vibeke Jacobsen
RA Coordinator

Addendum to Application Guidance for Medical Device

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by«Coloplast A/S», Denmark

Additional parameters and characteristics

I. Overall dimensions

Table 1. Overall dimensions (length, width, device thickness, weight, volume)

	Length, mm*	Width, mm*	Device thickness, mm	Weight, g**	Volume, ml**
Alterna® ostomy bag 1-piece closed Paediatric Opaque 10-35 mm	148,3	139	2,5- 4,5	11	210
Alterna® ostomy bag 1-piece closed Paediatric Transparent 10-35 mm	148,3	139	2,5- 4,5	11	210
Alterna® ostomy bag 1-piece closed Opaque 10-70 mm Maxi	223,8	148	2,5- 4,5	14	519
Alterna® ostomy bag 1-piece closed Opaque 20-55mm Maxi	223,8	148	2,5- 4,5	15	538
Alterna® ostomy bag 1-piece closed Transparent maxi 20-55 mm	223,8	148	2,5- 4,5	15	538
Alterna® ostomy bag 1-piece closed Transparent maxi 10-70 mm	223,8	148	2,5- 4,5	14	519
Alterna® ostomy bag 1-piece closed Opaque 20-55 mm Midi	183,8	148	2,5- 4,5	11	327
Alterna® ostomy bag 1-piece closed Transparent 20-55 mm Midi	183,8	148	2,5- 4,5	11	327
Alterna® ostomy bag 1-piece closed mini opaque 20-55 mm	148,3	139	2,5- 4,5	11	159
Alterna® ostomy bag 1-piece closed Opaque minicap 20-55 mm	109	109	3,1-5,1	11	26
Alterna® ostomy bag 1-piece closed Opaque convex 15-43 mm Maxi	223,8	148	8,1*	26	502
Alterna® Free ostomy bag 1-piece closed Opaque 20-75 mm Maxi	216,4	150,6	4,5-6,5	14	504
Alterna® Free ostomy bag 1-piece closed Transparent 20-75 mm Maxi	216,4	150,6	4,5-6,5	14	504

Alterna® Free ostomy bag 1-piece closed Opaque 20-65 mm Midi	183,4	137,7	4,5-6,5	13	325
Alterna® Free ostomy bag 1-piece closed Transparent 20-65 mm Midi	183,4	137,7	4,5-6,5	13	325
Alterna® Free ostomy bag 1-piece closed Opaque 20-65mm Mini	143,9	135	4,5-6,5	12	137
Alterna® Free ostomy bag 1-piece closed Transparent 20-65mm Mini	143,9	135	4,5-6,5	12	137
Alterna® Free ostomy bag 1-piece closed Transparent Maxi convex 15-43 mm	216,4	150,6	6,5*	25	500
Alterna® Free bag 1-piece closed Opaque Midi convex 15-33 mm	183,4	137,7	6,5*	23	279

* - Permissible deviation +/- 5 %

** - Permissible deviation +/- 10 %

This is to certify that **Ms. Lene Vibeke Jacobsen** today in my presence at the Notarial Office approved and signed the above document.

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Ms. Lene Vibeke Jacobsen has proved her identity by showing driving license.

The Court of Elsinore, Denmark, 17 november, 2016.

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