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CERTIFICATE

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China Council for the Promotion of International Trade is China Chamber of International Commerce

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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THIS IS TO CERTIFY THAT: the seal of Huaiyin Medical Instruments Co., Ltd. on the annexed APPLICATION SHEET is genuine.

China Council for the Promotion
of International Trade

(66)

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Authorized
Signature:

Zhao Weilu

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(Date: Jan. 23, 2019)

"Seen and approved"

Head of

Huaiyin Medical Instruments Co., Ltd.

General Manager

L.S.

Application sheet

Single-edged surgical instruments CERTUS, with accessories:

1. Sterile disposable surgical scalpel. Plastic handle. Carbon steel blade;
2. Sterile disposable surgical scalpel. Plastic handle. Stainless steel blade;
3. Sterile disposable surgical scalpel with a protective slider. Plastic handle. Carbon steel blade;
4. Sterile disposable surgical scalpel with a protective slider. Plastic handle. Stainless steel blade;
5. Sterile disposable surgical blade made of carbon steel;
6. Sterile disposable surgical blade made of stainless steel;
7. Sterile disposable surgical blade made of carbon steel for the removal of sutures.

Accessories:

Handle.

Produced by Huaiyin Medical Instruments Co., Ltd, China

Sterile disposable surgical scalpel. Plastic handle. Carbon steel blade

Sterile disposable surgical scalpel. Plastic handle. Stainless steel blade.

Purpose

- designed for dissecting soft tissues for all types of surgical interventions

Indications for use

- The dissection of soft tissues: skin, subcutaneous tissue, aponeuroses, muscles, as well as the lancing of polyps.

Contraindications

- The product should not be used for chondrotomy and bone cutting, for carrying out microsurgical incisions under an operating microscope and for visual organ surgery.

Precautionary measures

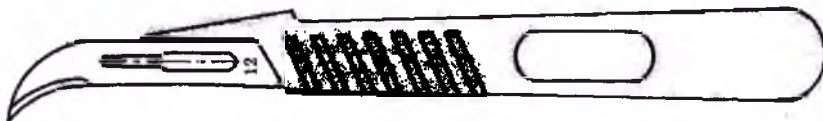
- Be careful when working with the tool, avoid contact with the cutting surface of the blade. Do not use a scalpel if the blade is pointing up, unless the incision is made along the probe. Do not try to change the shape of the blade or sharpen it. It is forbidden to put the used instrument to sterile and vice versa.

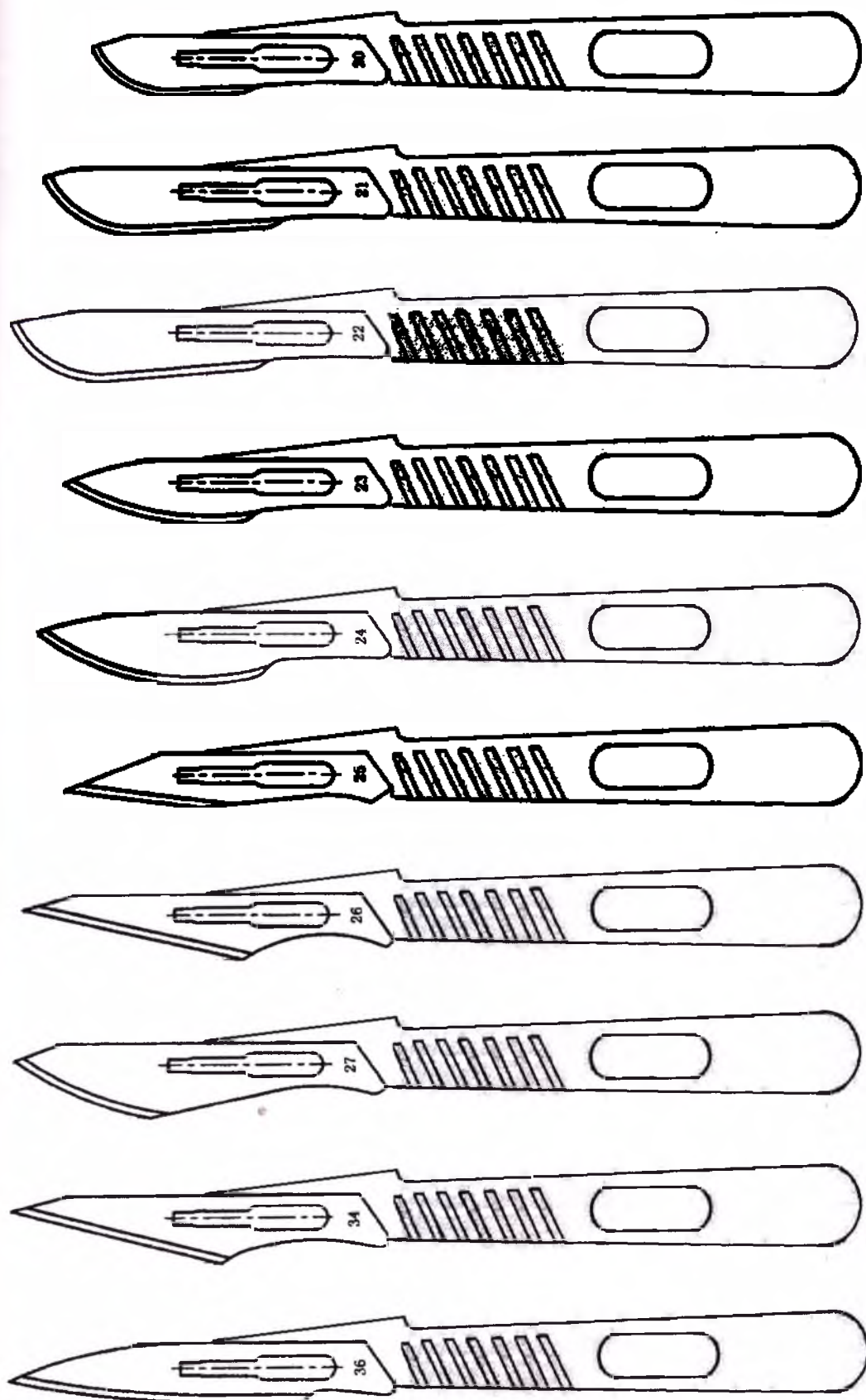
Particular complications

-When dissecting soft tissues, microscopic irregularities of the cutting-edge tip can contribute to the formation of micro-flaps and micro-tears at the edges of the wound, leading to wound abscess and the formation of rough scars.

Size range

The size of the scalpel has a numeric or alphanumeric designation and is determined by the shape of the blade (size): No. 9, 10, 11, 12, 13, 14, 15, 15C, 16, 18, 19, 20, 21, 22, 23, 24, 25, 26, 27, 34, 36

Blade size No.	9	10	11	12	13
Mass without security cap, g	4.9 ± 0.5	5.2 ± 0.5	5.0 ± 0.5	4.9 ± 0.5	5.0 ± 0.5
Mass with security cap, g	7.3 ± 0.5	7.6 ± 0.5	7.4 ± 0.5	7.3 ± 0.5	7.4 ± 0.5
Length, mm.	144.0 ± 2.0	144.5 ± 2.0	144.8 ± 2.0	140.4 ± 2.0	144.8 ± 2.0
Width, mm.	12.0 ± 0.9	10.9 ± 0.9	9.5 ± 0.75	12.4 ± 0.9	8.8 ± 0.75
Thickness, mm.	4.0 ± 0.6				

Blade size No.	14	15	15C	16	18
Mass without security cap, g	5.2 ± 0.5	5.1 ± 0.5	5.0 ± 0.5	5.0 ± 0.5	5.8 ± 0.5
Mass with security cap, g	7.6 ± 0.5	7.5 ± 0.5	7.4 ± 0.5	7.4 ± 0.5	8.2 ± 0.5
Length, mm.	150.7 ± 2.0	142.3 ± 2.0	141.4 ± 2.0	138.6 ± 2.0	154.9 ± 2.0
Width, mm.	9.5 ± 0.75	7.4 ± 0.75	7.3 ± 0.75	12.4 ± 0.9	13.8 ± 0.9
Thickness, mm.	4.0 ± 0.6				

Blade size No.	19	20	21	22	23
Mass without security cap, g	5.5 ± 0.5	5.7 ± 0.5	6.1 ± 0.5	6.2 ± 0.5	5.9 ± 0.5
Mass with security cap, g	7.9 ± 0.5	8.1 ± 0.5	8.5 ± 0.5	8.6 ± 0.5	8.3 ± 0.5
Scalpel length, mm.	154.0 ± 2.0	147.7 ± 2.0	155.6 ± 2.0	160.9 ± 2.0	152.4 ± 2.0
Width, mm.	14.0 ± 0.9	13.1 ± 0.9	14.7 ± 0.9	15.2 ± 0.9	14.0 ± 0.9
Thickness, mm.	4.0 ± 0.6				

Blade size No.	24	25	26	27	34	36
Mass without security cap, g	6.1 ± 0.5	5.8 ± 0.5	6.1 ± 0.5	6.2 ± 0.5	5.5 ± 0.5	5.9 ± 0.5
Mass with security cap, g	8.5 ± 0.5	8.2 ± 0.5	8.5 ± 0.5	8.6 ± 0.5	7.9 ± 0.5	8.3 ± 0.5
Length, mm.	157.3 ± 2.0	148.6 ± 2.0	162.9 ± 2.0	158.9 ± 2.0	158.3 ± 2.0	159.5 ± 2.0
Width, mm.	14.9 ± 0.9	12.6 ± 0.9	15.3 ± 0.9	15.3 ± 0.9	14.4 ± 0.9	13.1 ± 0.9
Thickness, mm.	4.0 ± 0.6					

Completeness

- The scalpel consists of:
 1. Plastic handle. Material: medical ABS-plastic (acrylonitrile butadiene styrene, chemical formula $(C_8H_8)_x \cdot (C_4H_6)_y \cdot (C_3H_3N)_z$ – shock-resistant technical thermoplastic resin based on acrylonitrile copolymer with butadiene and styrene). Scalpel handles are made taking into account all ergonomic requirements. In use, a scalpel is a direct continuation of the surgeon's hand and provides an accurate implementation of the planned incision. The handle grip has an ergonomic rest for the surgeon's finger and a ribbed surface. The measuring scale with a length of 6 cm and a scale interval of 1 mm is applied to the handle.
 2. Sterile disposable surgical blade made of carbon steel or stainless steel (description and characteristics in full compliance with the sections "Sterile disposable surgical blade made of carbon steel" and "Sterile disposable surgical blade made of stainless steel"). The blade is securely fixed in its mounting seat. The blades are described in the previous section.
 3. Removable plastic security cap. Purpose: protection of the scalpel work surface (blade) and protection of medical personnel from possible injuries when removing the tool. Length 66 ± 1.5 mm, height 16 ± 0.9 mm, maximum width 6.0 ± 0.6 mm, minimum width 4 ± 0.6 mm, wall thickness 1 ± 0.5 mm, mass 2.35 ± 0.50 g. The rough surface provides more comfortable removal of the cap from the tool
- Material: LDPE 52518 grade polyethylene;
- A strip of rust-preventive paper is located inside the cap, completely eliminating the occurrence of corrosive processes;

Packaging

- individual packaging made of aluminum composite material PET/AL/PE – polyester and composite foil with a thickness of 0.08 mm, completely prevents the oxygen transmission. The packaging is sealed by heat sealing;

The size of individual packaging:

Item number	Product name	The size of individual packaging, mm.
1	Sterile disposable surgical scalpel. Plastic handle. Stainless steel or carbon steel blade (size range No. 9, 10, 11, 12, 13, 14, 15, 15C, 16, 18, 19, 20, 21, 22, 23, 24, 25, 26, 27, 34, 36)	209x35 ± 2 mm

- The retail container is made of cardboard and includes 10 scalpels;
- The shipping box is made of cardboard and includes 50 retail containers, 500 scalpels.

Shelf life and storage conditions

- shelf life (sterility) 5 years;
- the products should be stored in clean, dry and ventilated premises with a humidity of less than 80%. The range of permissible temperatures is from -30° to + 50°. During transportation, avoid violations of the package integrity, direct sunlight, moisture and exposure to high temperature.

Warranty period

- The warranty period is 5 years from the production date.

Operation.

1. The range of permissible temperatures is from +10° to +35°.
2. Select the required scalpel size;
3. After checking the integrity of the retail container, remove the scalpel in an individual packaging. Ensure the integrity of the individual packaging. Make sure the shelf life (sterility) has not expired. It is FORBIDDEN to use the products if the individual packaging is damaged or the shelf life has expired.
4. Open the individual packaging and remove the scalpel, following the aseptic procedures;
5. Remove the security cap. The scalpel is ready to use.
6. After use, dispose of the scalpel in accordance with the instruction approved by the head of the health care institution. The instruction defines the responsible employees and the procedure for handling medical waste in the institution.

Sterile disposable surgical scalpel with a protective slider. Plastic handle. Stainless steel blade

Sterile disposable surgical scalpel with a protective slider. Plastic handle. Carbon steel blade

Purpose

- designed for dissecting soft tissues for all types of surgical interventions. The scalpel provides maximum safety during surgical procedures due to the presence of a protective slider, which allows you to close the scalpel blade with one thumb movement and eliminates the possibility of re-use of the product.

Indications for use

- The dissection of soft tissues: skin, subcutaneous tissue, aponeuroses, muscles, as well as the lancing of polyps.

Contraindications

- The product should not be used for chondrotomy and bone cutting, for carrying out microsurgical incisions under an operating microscope and for visual organ surgery.

Precautionary measures

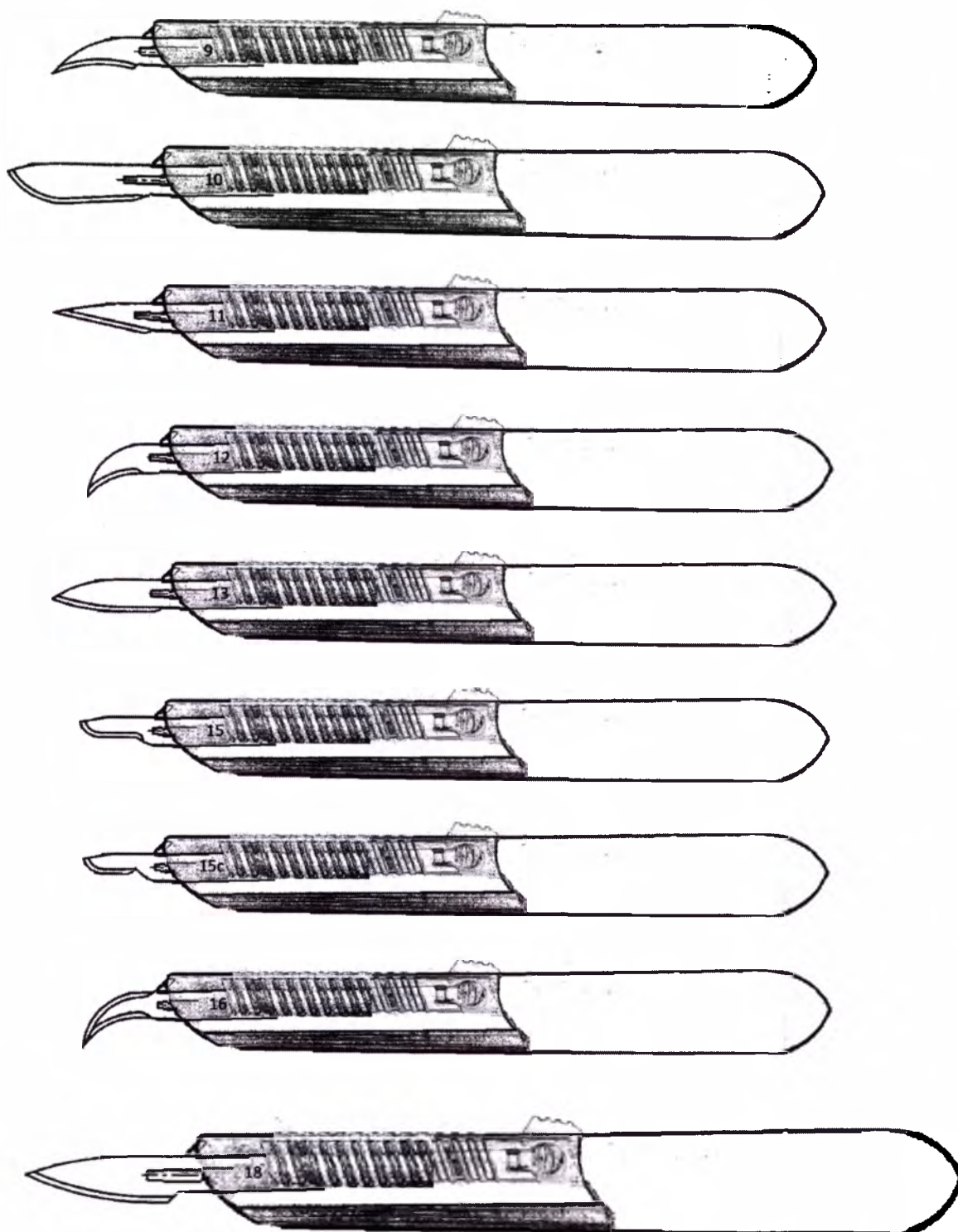
- Be careful when working with the tool, avoid contact with the cutting surface of the blade. Do not use a scalpel if the blade is pointing up, unless the incision is made along the probe. Do not try to change the shape of the blade or sharpen it. It is forbidden to put the used instrument to sterile and vice versa.

Particular complications


- When dissecting soft tissues, microscopic irregularities of the cutting-edge tip can contribute to the formation of micro-flaps and micro-tears at the edges of the wound, leading to wound abscess and the formation of rough scars.

Size range

- The size of the scalpel has a numeric or alphanumeric designation and is determined by the shape of the blade (size): 9, 10, 11, 12, 13, 15, 15C, 16, 18, 19, 20, 21, 22, 23, 24, 25, 26, 27, 34, 36

Blade shapes No. 19-36

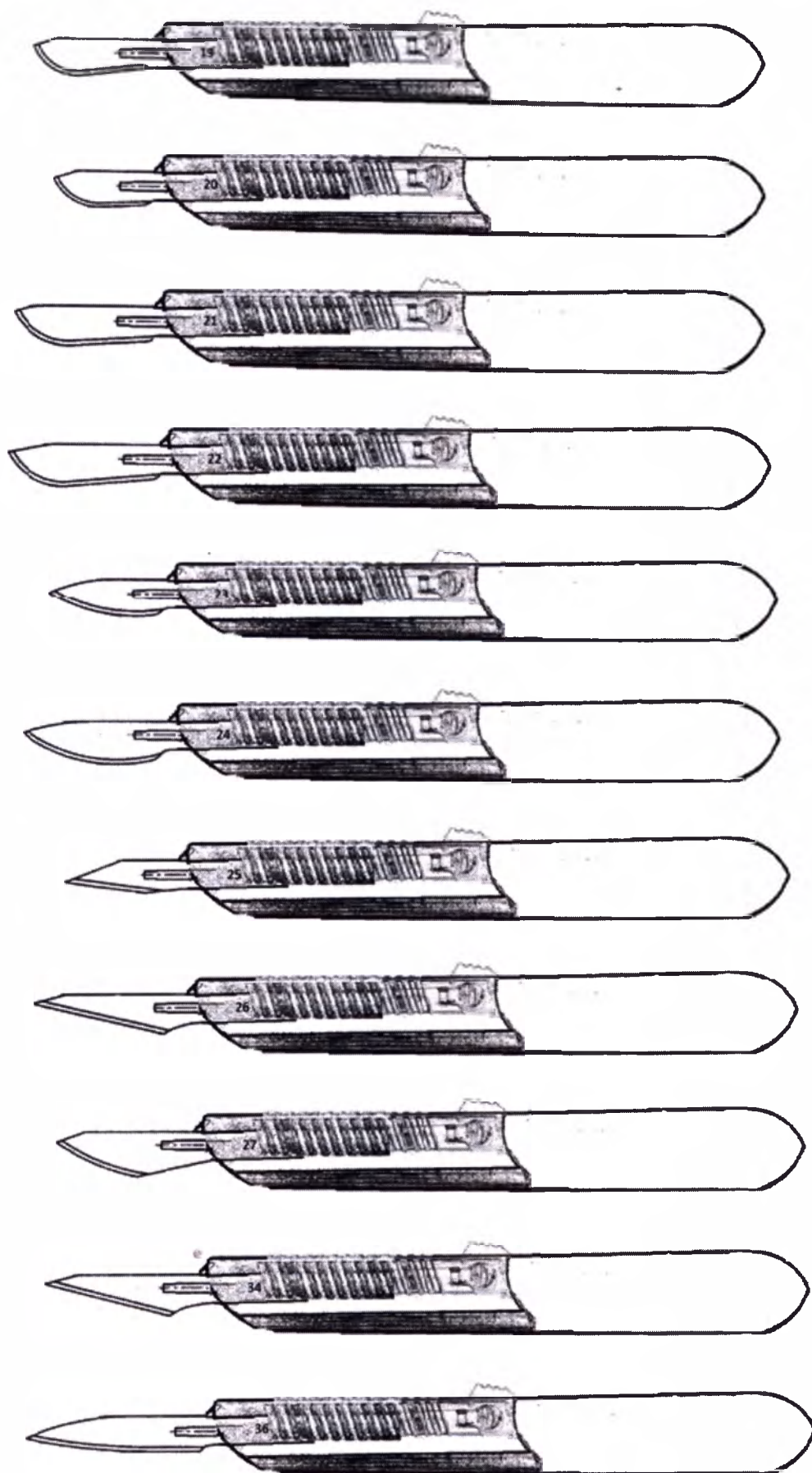Blade size No.	9	10	11	12	13
Mass, g	8.07 ± 0.5	8.01 ± 0.5	7.80 ± 0.5	7.78 ± 0.5	7.84 ± 0.5
Operational length, mm.	142.4 ± 2	147.3 ± 2	148.0 ± 2	142.7 ± 2	146.7 ± 2
"New tool" position length, mm.	151.0 ± 2				
"Locked" position length, mm.	155.0 ± 2				
Width, mm.	15.0 ± 0.9				
Thickness, mm.	10.0 ± 0.75				

Blade size No.	15	15C	16	18	19
Mass, g	8.00 ± 0.5	7.84 ± 0.5	7.84 ± 0.5	10.70 ± 0.5	10.70 ± 0.5
Operational length, mm.	144.3 ± 2	144.7 ± 2	144.0 ± 2	178.1 ± 2	175.7 ± 2
"New tool" position length, mm.	151.0 ± 2			190.0 ± 2.3	
"Locked" position length, mm.	155.0 ± 2			194.0 ± 2.3	
Width, mm.	15.0 ± 0.9			17.0 ± 0.9	
Thickness, mm.	10.0 ± 0.75				

Blade size No.	20	21	22	23	24
Mass, g	10.57 ± 0.5	10.95 ± 0.5	11.11 ± 0.5	10.70 ± 0.5	10.85 ± 0.5
Operational length, mm.	169.7 ± 2	178.2 ± 2	183.0 ± 2.3	175.0 ± 2	179.0 ± 2
"New tool" position length, mm.	190.0 ± 2.3				
"Locked" position length, mm.	194.0 ± 2.3				
Width, mm.	17.0 ± 0.9				
Thickness, mm.	10.0 ± 0.75				

Blade size No.	25	26	27	34	36
Mass, g	10.56 ± 0.5	10.99 ± 0.5	10.89 ± 0.5	10.76 ± 0.5	10.74 ± 0.5
Operational length, mm.	174.3 ± 2	184.6 ± 2.3	181.7 ± 2.3	181.7 ± 2.3	184.2 ± 2.3
"New tool" position length, mm.	190.0 ± 2.3				
"Locked" position length, mm.	194.0 ± 2.3				
Width, mm.	17.0 ± 0.9				
Thickness, mm.	10.0 ± 0.75				

Completeness

- The scalpel consists of:

1. Plastic handle. Material: medical ABS-plastic (acrylonitrile butadiene styrene, chemical formula $(C_8H_8)_x \cdot (C_4H_6)_y \cdot (C_3H_3N)_z$ – shock-resistant technical thermoplastic resin based on acrylonitrile copolymer with butadiene and styrene). Scalpel handles are made taking into account all ergonomic requirements. In use, a scalpel is a direct continuation of the surgeon's hand and provides an accurate implementation of the planned incision. The measuring scale with a length of 6 cm and a scale interval of 1 mm is applied to the handle. The scalpel handle is equipped with a slider guide path, front and rear guide grooves of the button-locking device, a through groove that allows the slider to move and the operation of the button-locking mechanism, as well as the stops securing the slider in the locked position.

- The slider guide path is designed to hold the slider on the handle of the scalpel and to ensure its straight movement along the handle. Location – the lower bound of the scalpel handle. Above the slider guide path, three arrows are applied to the surface of the handle, indicating the movement direction of the slider to bring the scalpel to the "ready for use" position

- The front guide groove. The groove has a blade-side stopper that ensures the fixation of the button stem and the position of the slider, which excludes the possibility of its further use;

- The rear guide groove provides locking of a fixing stem and ensures the "locked" position of the slider;

- The through groove allows the slider button-locking mechanism to function and allows the slider to be fixed in the "new tool" and "ready to use" positions. The through groove is located inside the front guide groove symmetric about its right and left edges.

2. Sterile disposable surgical blade made of carbon steel or stainless steel

3. The protective plastic slider is equipped with a button-locking mechanism and a finger rest. Purpose: protection of the scalpel work surface (blade) and protection of medical personnel from possible injuries when removing the tool and its further disposal. The scalpel slider has three positions relative to the scalpel handle: "new tool", "ready to use", "locked". In the "new tool" position, the button stem is at the level of the through groove flare, the slider is fixed, the scalpel blade is closed. In the "ready to use" position, the button stem is at the level of the through groove flare on the opposite side from the blade, the slider is fixed, the blade is open. In the "locked" position, the button stem and the locking stem are located behind the stops, the slider is fixed, the blade is closed, further use is impossible. The ribs are applied on three sides of the slider surface for comfortable and reliable retention of the scalpel. The finger rest is designed to provide a comfortable movement of the slider when bringing the tool into the operating and non-operating position (blade closing). The slider button-locking device is located on the front and rear surfaces and is intended for fixing the slider in one of the "blade open"/"blade closed" positions (operating and non-operating position of the tool), as well as ensuring the subsequent closing of the blade and locking the slider after using the tool. The slider button is located on the front surface. The locking stem is located on the pressure plate on the backside of the slider.

- Material: Material: medical ABS-plastic (acrylonitrile butadiene styrene, chemical formula $(C_8H_8)_x \cdot (C_4H_6)_y \cdot (C_3H_3N)_z$ – shock-resistant technical thermoplastic resin based on acrylonitrile copolymer with butadiene and styrene)

Packaging

- individual packaging can be made in two versions:

1. Made of aluminum composite material PET/AL/PE – polyester and composite foil;

The size of individual packaging:

Item number	Product name	The size of individual packaging, mm.
1	Sterile disposable surgical scalpel with a protective slider. Plastic handle. Carbon steel or stainless steel blade (size range No. 9, 10, 11, 12, 13, 14, 15, 15C, 16, 18, 19, 20, 21, 22, 23, 24, 25, 26, 27, 34, 36)	235x36 ± 2mm

2. Blister packaging, medical paper, PET plastic film.

The size of individual packaging:

Item number	Product name	The size of individual packaging, mm.
1	Sterile disposable surgical scalpel with a protective slider. Plastic handle. Carbon steel or stainless steel blade (size range No. 9, 10, 11, 12, 13, 14, 15, 15C, 16)	196x42 ± 2mm
2	Sterile disposable surgical scalpel with a protective slider. Plastic handle. Carbon steel or stainless steel blade (size range No. 18, 19, 20, 21, 22, 23, 24, 25, 26, 27, 34, 36)	233x45 ± 2mm

- The retail container is made of cardboard and includes 10 scalpels;
- The shipping box is made of cardboard and includes 50 retail containers, 500 scalpels.

Shelf life and storage conditions

- shelf life (sterility) 5 years;
- the products should be stored in clean, dry and ventilated premises with a humidity of less than 80%. The range of permissible temperatures is from -30° to + 50°. During transportation, avoid violations of the package integrity, direct sunlight, moisture and exposure to high temperature.

Warranty period

- The warranty period is 5 years from the production date.

Operation

1. The range of permissible temperatures is from +10° to +35°.
2. Select the required scalpel size;
3. After checking the integrity of the retail container, remove the scalpel in an individual packaging. Ensure the integrity of the individual packaging. Make sure the shelf life (sterility) has not expired. It is FORBIDDEN to use the products if the individual packaging is damaged or the shelf life has expired.
4. Open the individual packaging and remove the scalpel, following the aseptic procedures;
5. Press the slider button and move the slider in the direction indicated by the arrows, till it goes click. The scalpel blade is open, the slider is fixed, the scalpel is ready to use.
6. After using the tool, press the slider button and move the slider in the direction away from yourself, till it goes click. The scalpel blade is closed. Lock the slider to lock the scalpel in order to prevent its subsequent use. To do this, hold the scalpel with one hand and with the other hand move the slider in the direction away from yourself until its clicking position. Make sure the scalpel slider is locked. To do this, press the button and try to move the slider towards yourself. If the scalpel is locked, the slider should stay put and the scalpel blade closed.
7. Dispose of the scalpel in accordance with the instruction approved by the head of the health care institution. The instruction defines the responsible employees and the procedure for handling medical waste in the institution.

Sterile disposable surgical blade made of carbon steel

Sterile disposable surgical blade made of stainless steel

Sterile disposable surgical blade made of carbon steel for the removal of sutures

Purpose

- sterile disposable surgical blades are designed for dissecting soft tissues for all types of surgical interventions;

- sterile disposable surgical blades for the removal of sutures are intended for the removal of postoperative sutures;

Indications for use

- The dissection of soft tissues: skin, subcutaneous tissue, aponeuroses, muscles, lancing of polyps and removal of postoperative sutures.

Contraindications

- The product should not be used for chondrotomy and bone cutting, for carrying out microsurgical incisions under an operating microscope and for visual organ surgery.

Precautionary measures

- Use tweezers to attach and then remove the surgical blade from the handle. Be careful when working with the tool, avoid contact with the cutting surface of the blade. Do not use a scalpel if the blade is pointing up, unless the incision is made along the probe. Do not try to change the shape of the blade or sharpen it. It is forbidden to put the used instrument to sterile and vice versa.

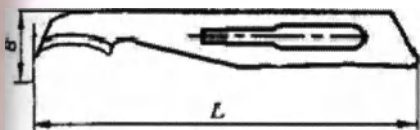
Particular complications

-When dissecting soft tissues, microscopic irregularities of the cutting-edge tip can contribute to the formation of micro-flaps and micro-tears at the edges of the wound, leading to wound abscess and the formation of rough scars.


Blade shape (size)

- has a numeric, alphanumeric or letter designation;

- the form of surgical blades for the removal of sutures depending on the size (number):

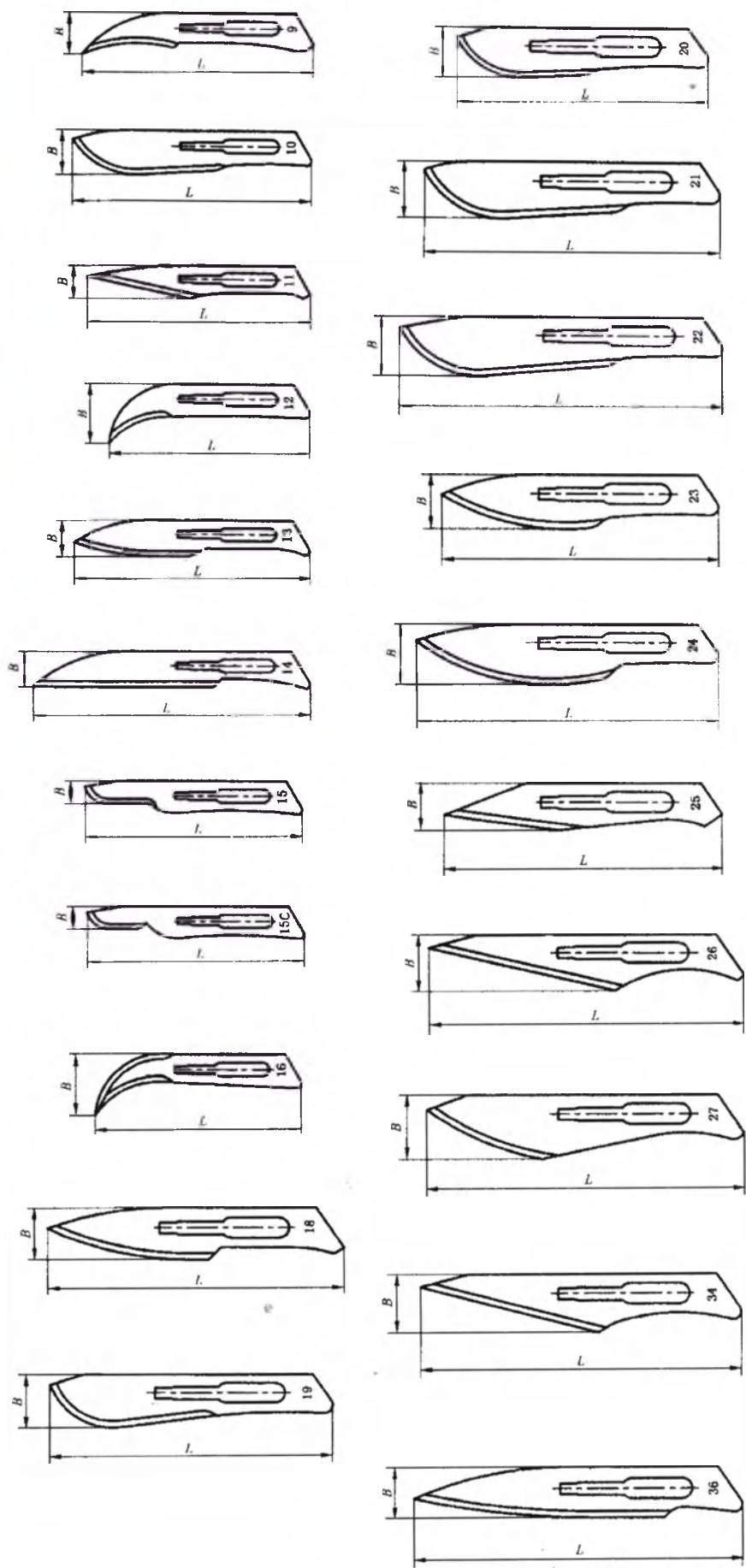- the shape of surgical blades depending on the size (number):

The size of the blade for the removal of sutures (length, mm)	45mm	65mm	110mm
Mass, g.	0.49 ± 0.10	1.15 ± 0.10	1.8 ± 0.10
Length, mm.	44.6 ± 1.25	65.0 ± 1.50	110.0 ± 1.75
Width, mm.	6.4 ± 0.75	7.0 ± 0.75	6.3 ± 0.75

Blade size No.	9	10	11	12	13
Mass, g.	0.56 ± 0.10	0.69 ± 0.10	0.51 ± 0.10	0.46 ± 0.10	0.47 ± 0.10
Length, mm.	42.00 ± 1.25	43.00 ± 1.25	42.60 ± 1.25	38.50 ± 1.25	43.20 ± 1.25
Width, mm.	8.00 ± 0.75	7.80 ± 0.75	6.50 ± 0.75	9.50 ± 0.75	6.20 ± 0.75


Blade size No.	14	15	15C	16	18
Mass, g.	0.63 ± 0.10	0.50 ± 0.10	0.48 ± 0.10	0.46 ± 0.10	0.88 ± 0.10
Length, mm.	49.5 ± 1.25	40.40 ± 1.25	40.00 ± 1.25	38.00 ± 1.25	54.50 ± 1.50
Width, mm.	6.10 ± 0.75	4.40 ± 0.60	4.40 ± 0.60	9.80 ± 0.75	9.80 ± 0.75

Blade size No.	19	20	21	22	23
Mass, g.	0.94 ± 0.10	0.79 ± 0.10	1.11 ± 0.10	1.27 ± 0.10	0.86 ± 0.10
Length, mm.	51.80 ± 1.50	46.00 ± 1.25	54.00 ± 1.50	59.10 ± 1.50	50.9 ± 1.50
Width, mm.	9.90 ± 0.75	9.00 ± 0.75	10.70 ± 0.90	11.00 ± 0.90	9.9 ± 0.75

Blade size No.	24	25	26	27	34	36
Mass, g.	1.07 ± 0.10	0.62 ± 0.10	1.06 ± 0.10	1.17 ± 0.10	1.07 ± 0.10	1.00 ± 0.10
Length, mm.	54.90 ± 1.50	47.10 ± 1.25	60.50 ± 1.50	57.40 ± 1.50	57.70 ± 1.50	58.90 ± 1.50
Width, mm.	11.00 ± 0.90	9.00 ± 0.75	11.20 ± 0.90	11.30 ± 0.90	10.50 ± 0.90	8.90 ± 0.75

Operation

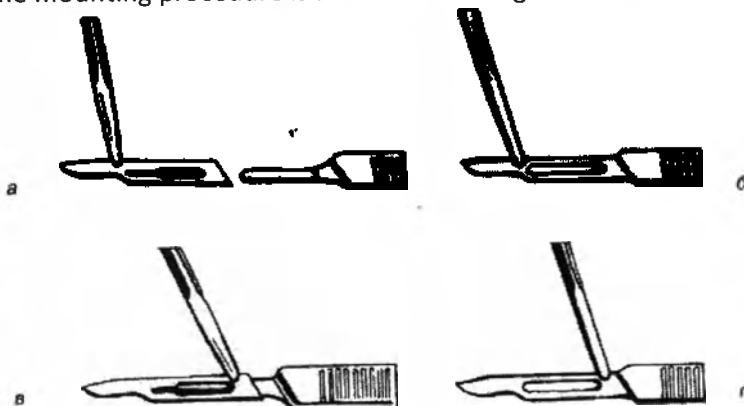1. The range of permissible temperatures is from $+10^{\circ}$ to $+35^{\circ}$.
2. Select the required blade size;
3. After checking the integrity of the retail container, remove the blade in an individual packaging. Ensure the integrity of the individual packaging. Make sure the shelf life (sterility) has not expired. It is FORBIDDEN to use the products if the individual packaging is damaged or the shelf life has expired.
4. Open the individual packaging and remove the blade, following the aseptic procedures;
5. Attach the sterile blade to the sterile scalpel handle through the special mounting hole of the blade.
The mounting procedure is shown in the diagram:

Surgical blades 65 mm and 110 mm long for the removal of sutures can be used with or without a scalpel handle.

6. After using, remove the blade from the scalpel handle in reverse order. Be careful to avoid injury.
7. After use, dispose of the blades in accordance with the instruction approved by the head of the health care institution. The instruction defines the responsible employees and the procedure for handling medical waste in the institution.

Completeness

- disposable blade made of carbon or stainless steel;

Packaging

- individual packaging made of aluminum composite material PET/AL/PE – polyester and composite foil with a thickness of 0.08 mm, completely prevents the oxygen transmission. Rust-preventive paper, wrapping the blade, completely eliminates the occurrence of corrosive processes. The packaging is sealed by heat sealing;

The size of individual packaging:

Item number	Product name	The size of individual packaging, mm.
1	Sterile disposable surgical blade made of stainless or carbon steel (size range No. 9, 10, 11, 12, 12D, 13, 14, 15, 15C, 16, 18, 19, 20, 21, 22, 23, 24, 25, 26, 27, 34, 36)	90x24 ± 2mm
2	Sterile disposable surgical blade made of carbon steel for the removal of sutures. (length 45 mm (45C))	90x24 ± 2mm
3	Sterile disposable surgical blade made of carbon steel for the removal of sutures. (length 65 mm)	90x24 ± 2mm
4	Sterile disposable surgical blade made of carbon steel for the removal of sutures (length 110 mm)	151x30 ± 2mm

- The retail container is made of cardboard and includes 100 blades;
- The shipping box is made of cardboard and includes 50 retail containers, 5000 blades.

Shelf life and storage conditions

- shelf life (sterility) 5 years;
- the products should be stored in clean, dry and ventilated premises with a humidity of less than 80%. The range of permissible temperatures is from -30° to + 50°. During transportation, avoid violations of the package integrity, direct sunlight, moisture and exposure to high temperature.

Warranty period

- The warranty period is 5 years from the production date.

General section for sterile disposable surgical blades and scalpels:

Package Marking

Individual packaging

1.1. Product name;

1.2. Product size (numerical);

1.3. Lot number (batch). LOT: _____

1.4. Date of manufacture. Month and year MFG: __ _

1.5. Expiration Date EXP: __ _

1.6. Single-use information; 

1.7. The manufacturer's name, address, telephone number, the country of goods origin;

1.8. Certificate of Validation number and date;

1.9. Marking of the opening place 

2. Retail container

2.1. Product name;

2.2. Product size (numerical);

2.3. Lot number (batch). LOT: _____

2.4. Date of manufacture. Month and year MFG: __ _

2.5. Expiration Date EXP: __ _

2.6. Single-use information ; 

2.7. The number of pieces in the package;

2.8. Information about the sterilization method;

sterile	R
---------	---

2.9. The manufacturer's name, address, telephone number, the country of goods origin;

2.10. Name, address, telephone number of the manufacturer's representative in the territory of the Russian Federation;

2.11. Certificate of Validation number and date;

2.12. Marking of the opening place 

2.13. The phrase "Sterile if the individual packaging is intact"

3. Shipping package

- 3.1. Product name;
- 3.2. Product size (numerical);
- 3.3. Quantity in the box;
- 3.4. Information about the sterilization method;

sterile	R
---------	---
- 3.5. Lot number (batch). LOT: _____
- 3.6. Date of manufacture. Month and year MFG: ____ - ____ - ____
- 3.7. Expiration Date EXP: ____ - ____ - ____
- 3.8. Single-use information ; 
- 3.9. The manufacturer's name, address, telephone number, the country of goods origin;
- 3.10. Name, address, telephone number of the manufacturer's representative in the territory of the Russian Federation;
- 3.11. Certificate of Validation number and date;
- 3.12. Package size (length x width x height);
- 3.13. Net Weight, Gross Weight;
- 3.14. Requirements for transportation and storage (symbol legend).

- keep away from heat;

- permissible storage humidity 80% maximum;

- The range of permissible storage temperatures from -30° to +50°

Disposal

Surgical blades and disposable scalpels are classified as Class B – epidemiologically hazardous waste.

1. Surgical blades and scalpels are subject to obligatory purification (disinfection)/decontamination. The choice of the disinfection/decontamination method is determined by the capabilities of the health care/pharmaceutical institution and is carried out when developing a scheme for handling medical waste. If the health care/pharmaceutical institution does not have a site for

disinfection/decontamination of class B waste or a centralized system for disposal of medical waste adopted in the administrative territory, the class B waste is decontaminated by the personnel of the organization at the places where this waste is formed using chemical/physical methods;

2. Surgical blades and scalpels are collected in a disposable solid (puncture-proof) yellow (or with a yellow marking) packaging (containers). The container should have a close-fitting lid, which excludes the possibility of spontaneous opening. Transfer of class B waste outside the unit in open containers is not allowed;

3. During the final packaging of Class B waste to remove it from the unit (organization), disposable containers with Class B waste are marked "Waste. Class B" with the name of the organization, division, date, and name of the person responsible for collecting the waste;

4. Class B medical waste in enclosed disposable packages is placed in containers and then transported from the departments to a waste management site or a room for temporary storage of medical waste until it is transported by specialized organizations to a disinfection/decontamination site. Unauthorized access to the temporary storage area of medical waste is prohibited.

Containers should be made of materials resistant to physical impact, high and low temperatures, detergents and disinfectants, closed with lids, which design should not allow their spontaneous opening;

5. It is allowed to transfer Class B non-disinfected medical waste packaged in special disposable packaging (containers) from remote structural units (health centers, doctor's offices, medical and obstetric centers) and other healthcare delivery places to the medical organization to ensure their subsequent disinfection/decontamination;

6. When collecting medical waste, it is prohibited:

- to dispose of class B waste manually;
- to transfer (reload) unpackaged Class B waste from one container into another;
- to tamp Class B waste;
- to carry out any operations with waste without gloves or the necessary personal protective equipment and clothing;
- to use soft disposable packaging to collect sharp medical instruments and other sharp objects;
- to install disposable and reusable waste collection containers at a distance of less than 1 m from heating appliances.

7. If an employee is injured while handling medical waste, which is potentially dangerous in terms of infection (prick, cut with violation of the skin and/or mucous membranes integrity), it is necessary to take emergency preventive measures.

Handle

Purpose

- Designed for fixing a surgical blade

Size range

- has a numeric size designation No. 3 and No. 4;

Table of matching handle sizes to surgical blade sizes

Handle size	Blade size
No. 3	Sterile disposable surgical blade made of stainless or carbon steel (size range No. 9, 10,

	11, 12, 12D, 13, 14, 15, 15C, 16) Sterile disposable surgical blade made of stainless steel for the removal of sutures. (length 45 mm (45C); 65 mm, 110 mm)
No. 4	Sterile disposable surgical blade made of stainless or carbon steel (size range No. 18, 19, 20, 21, 22, 23, 24, 25, 26, 27, 34, 36)

Steel grade

- Steel grade 304 according to AISI

Packaging

- individual packaging is made of a polymer film with a thickness of 50 microns;

The size of individual packaging:

Item number	Product name	The size of individual packaging, mm.
1	handle No. 3 and No. 4	200x20 ± 2 mm

- The retail container is made of cardboard and includes 10 handles;

- The shipping box is made of cardboard and includes 40 retail containers, 400 handles.

Shelf life and storage conditions

- shelf life is unlimited;

- the products should be stored in clean, dry and ventilated premises with a humidity of less than 80%. The range of permissible temperatures is from -30° to + 50°. During transportation, avoid violations of the package integrity, direct sunlight, moisture and exposure to high temperature.

Warranty period

- Warranty period – 2 years from the date of sale.

Operation

1. The range of permissible temperatures is from +10° to +35°.
2. Select the required handle size;
3. Sterilize the handle before use;
4. Open the individual packaging and remove the blade, following the aseptic procedures;
5. Attach the sterile blade to the sterile handle through the special mounting hole of the blade. The mounting procedure is shown in the diagram:

6. After using, remove the blade from the handle in reverse order. Be careful to avoid injury;
7. Clean the tool. Use streaming water and a hard brush to manually clean the instrument of residual blood and dirt. Put the tool into the hot water of the ultrasound bath (ultrasonic) and process it for 40 minutes using ultrasound;
8. Disinfect the tool by chemical means;
9. Sterilize the tool;

High-pressure steam sterilization (autoclaving)

1. Load the cleaned and disinfected handles into sterilizing drums. The side holes of the drums must be open before sterilization. Close the drum lid tightly.
2. Load the sterilizing drums into the autoclave.
3. Close the autoclave with an airtight lid and perform the necessary manipulations to begin its work in a certain mode. Control the autoclave operation with pressure gauge and thermometer indicators.

Sterilize in one of three modes:

At a temperature of 134°C, with a pressure of 1.1 atmospheres – 1 hour,

At a temperature of 134°C, with a pressure of 1.5 atmospheres – 45 minutes,

At a temperature of 134°C, with a pressure of 2 atmospheres – 30 minutes.

4. After sterilization is complete, open the autoclave door slightly and dry the sterilizing drums. When removing the sterilizing drums from the autoclave, close the holes in the walls of the drums and mark the date of sterilization.

Hot air (dry heat) sterilization

1. Place the cleaned and disinfected handles on the shelves of the hot-air sterilizer. Dry the handles for 30 minutes at 80°C with the door of the hot-air sterilizer ajar.
2. Close the door of the hot-air sterilizer and sterilize the tools for 1 hour at 180°C.
3. After the sterilization time has expired, wait until the hot-air sterilizer is cooled to 50-70°C and open the door slightly. After the final cooling of the hot-air sterilizer, unload sterile tools from the chamber.

10. Dispose of the handles in accordance with the instruction approved by the head of the health care institution. The instruction defines the responsible employees and the procedure for handling medical waste in the institution.

Package Marking

1. Retail container

1.1. Product name;

1.2. Product size (numerical);

3. Lot number (batch). LOT: _____
4. Date of manufacture. Month and year MFG: ____ - ____ - ____
5. Expiration Date EXP: ____ - ____ - ____
6. The number of pieces in the package;
7. The manufacturer's name, address, telephone number, the country of goods origin;
8. Name, address, telephone number of the manufacturer's representative in the territory of the Russian Federation;
9. Certificate of Validation number and date;
10. Marking of the opening place 
11. Markings "Sterile", "Non-pyrogenic", "Non-toxic";
2. Shipping package
- 2.1. Product name;
- 2.2. Product size (numerical);
- 2.3. Quantity in the box;
- 2.4. Lot number (batch). LOT: _____
- 2.5. Date of manufacture. Month and year MFG: ____ - ____ - ____
- 2.6. Expiration Date EXP: ____ - ____ - ____
- 2.7. The manufacturer's name, address, telephone number, the country of goods origin;
- 2.8. Name, address, telephone number of the manufacturer's representative in the territory of the Russian Federation;
- 2.9. Certificate of Validation number and date;
- 2.10. Package size (length x width x height);
- 2.11. Net Weight, Gross Weight;
- 2.12. Requirements for transportation and storage (symbol legend).

- keep away from heat;

- permissible storage humidity 80% maximum;

- The range of permissible storage temperatures from -30° to +50°

Disposal

1. Handles are subject to mandatory purification (disinfection)/decontamination. The choice of the disinfection/decontamination method is determined by the capabilities of the health care/pharmaceutical institution and is carried out when developing a scheme for handling medical waste. If the health care/pharmaceutical institution does not have a site for disinfection/decontamination of waste or a centralized system for disposal of medical waste adopted in the administrative territory, the waste is decontaminated by the personnel of the organization at the places where this waste is formed using chemical/physical methods;
2. Sterilize the tools by autoclaving;
3. After instrumental methods of disinfection using physical methods and autoclaving of handles, such tools can be accumulated, temporarily stored, transported, destroyed and disposed of together with Class A waste – epidemiologically safe waste, close in composition to solid household waste.
4. Class A waste is collected in reusable containers or disposable bags. The bags can be any color, except for yellow and red. Disposable bags are placed on special carts or inside reusable containers. Waste collection containers and carts should be labeled "Waste. Class A". Filled reusable containers or disposable bags are delivered using labor-saving tools and are reloaded into labeled containers designed to collect waste of this class, installed on a special site (room). The transportation of Class A waste is organized taking into account the sanitation chart adopted for this territory in accordance with the requirements of sanitary legislation to the maintenance of the territories of populated areas and the production and consumption waste handling.

Supplier: MBT LLC

195427, St. Petersburg, Tikhoretsky Avenue, 25, b. 2, apt. 81

Phone: +7 -911-928-53-51, mail: mb-technology@mail.ru

Manufacturer: Huaiyin Medical Instruments Co., Ltd.

No.8 West Ming Yuan Road, Huaian, Jiangsu, China.

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