APPROVED BY

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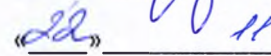
Adult Diapers

Types:

Adult Diapers Giggles, sizes: XS, S, M, L, XL

Adult Diapers Dr.Comfort, sizes: S, M, L

Adult Pants Giggles, sizes: M, L

Manufacturer:

Sevincler Saglik Urunleri San. ve Tic. A.S., Turkey

2016

Table of Contents

Table of Contents.....	2
1 Product Description.....	3
2 Details of Product Manufacturer:	3
3 Manufacturing Site.....	3
4 Intended Use of Product established by Manufacturer	3
5 Classification	3
6 Specifications	4
6.1 Material Composition of Adult Diapers	4
6.2 Material Composition of Adult Pants	5
6.3 Adult Diapers: Giggles.....	6
6.4 Adult Diapers Dr. Comfort	8
6.5 Adult Pants Giggles.....	9
7 Packaging.....	9
7.1 Consumer Package	9
7.2 Shipping Package.....	10
8 Marking and Labelling	10
8.1 Marking of Consumer Packages.....	10
9 Product Performance.	11
9.1 Microbiology	11
9.2 Physical and Chemical Properties of Products.....	11
9.3 Contraindications and possible side effects	12
9.4 Performance of Adult Diapers Giggles.....	12
9.5 Performance of Adult Diapers Dr.Comfort	12
9.6 Performance of Adult Pants Giggles.....	12
10 Safety Requirements	13
11 Environmental.....	13
12 Transportation, storage, disposal	14
12.1 Transport Rules:	14
12.2 Storage Information:	14
12.3 Disposal.....	14
13 Use.....	15
13.1 Instruction for Use of Adult Diapers:.....	15
13.2 Instructions for Use of Giggles Adult Pants	16
14 Manufacturer's Warranty	17
15 Contacts.....	17

1 Product Description

Adult Diapers, Styles:

- Adult Diapers:Giggles, sizes XS, S, M, L, XL
- Adult Diapers: Dr.Comfort, sizes: S, M, L
- Adult Pants: Giggles, sizes: M, L.

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+90(342) 3375700, 4444150

info@sevinclersaglik.com.tr, www.sevincler.com.tr

4 Intended Use of Product established by Manufacturer

Adult diapers and pants are designed to receive and retain bodily fluids of adults. They find application in nursing of patients with incontinence or bedridden patients and in some other circumstances.

These products can be used in care providing institutions and/ or at home.

5 Classification

The products come in different sizes depending on the patient waist size. The details of available product sizes are given in Table 1.

Table 1. Table 1.

Product Type	Patient Waist Size, cm	International Name of Product Type	Size Symbol
Extra small	40-60	Extra small size	XS
Small	50-85	Small size	S
Medium	85-125	Medium size	M
Large	100-150	Large size	L
Extra large	120-160	Extra large size	XL

6 Specifications

6.1 Material Composition of Adult Diapers

Adult diapers consist of a number of working layers and additional components that do not participate in the absorbing process but make the use of the products more comfortable. The material composition of an adult diaper is shown in Fig.

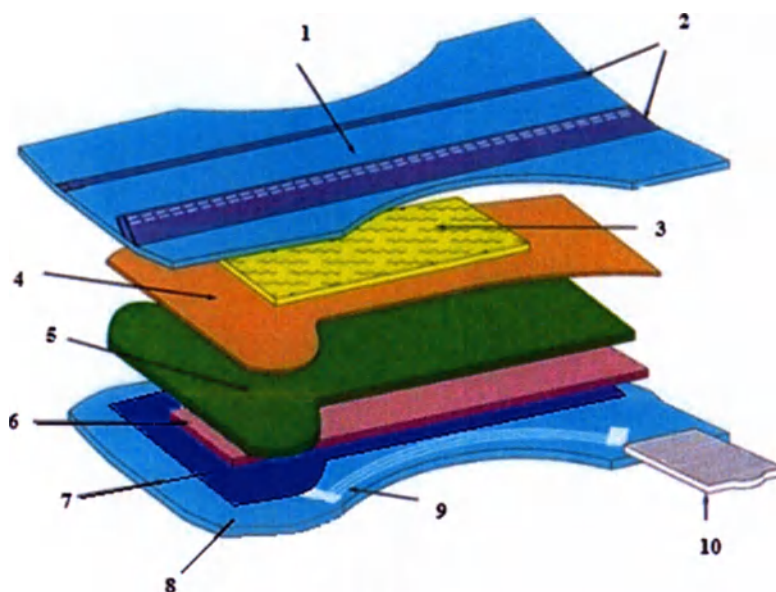
Fig. 1. Material Composition of Adult Diapers

- 1- The top layer is made of a soft nonwoven textile fabric.
- 2- The main part of the diaper consists of leak-proof protective barriers and two elasticized tapes attached thereto.
- 3- The Absorbing and distribution layer (ADL) is also made of a breathable nonwoven. ADL' function is to accelerate the fluid absorption and spread the fluid over the diaper in order to prevent its accumulation in one spot.
- 5- The absorbent core arranged under ADL. This layer comprises a mixture of a superabsorbent polymer (SAP) and fluffed pulp. The absorbent core is sandwiched between the nonwoven sheets (top barrier sheet (-4) and bottom backsheet (-6)), which keep the absorbent core in place even when the diaper is full.
- 7- The absorbent core and the backsheets are covered with a protective layer. It is a polyethylene layer with prints on it (Giggles trade mark). This layer helps lock all the fluid accumulated inside the diaper and avoid the leaks.

- 8- The polyethylene layer is covered with another nonwoven layer with a WETNESS INDICATOR zone. The latter consists of graphic symbols imprinted on the outside of the diaper, that fade when the diaper needs changing.
- 9- Protective side bands. These three elasticized bands fit snugly around the wearer's leg without putting pressure on the skin for the comfort of usage.
- 10- Velcro tapes. Two tapes per side. These tapes are made of Velcro material to make the diaper re-sealable. You can open and close the diaper as many times as necessary.

Therefore, it can be seen that the diapers made by Sevinçler Sağlık Ürünleri comprise the top protective layer, ADL, absorbent core, protective layer, bottom backsheet, barrier elements, fasteners, wetness indicator.

6.2 Material Composition of Adult Pants

The composition of the adult pants is similar to that of the adult diapers. However, there are some difference between these products. See Fig. 3.

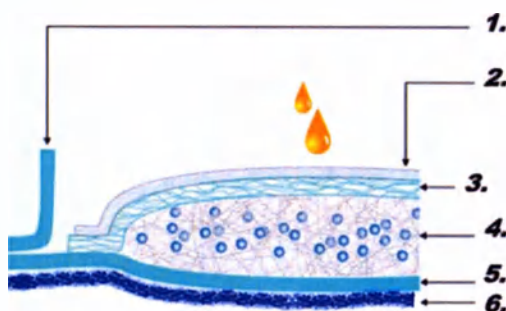
Fig. 3. Material Composition of Adult Pants

1- protective barriers, 2 – top layer, nonwoven , 3 – distribution layer, breathable nonwoven, 4 – absorbent core, SAP and fluffed pulp, 5 – protective layer, polyethylene film, 6 – bottom backsheet, nonwoven.

It can be seen that the inside structure of the adult pants differs from that of the adult diapers, specifically, it has no textile sheets on top and under the absorbent core.

The outside of the adult pants is also different from the adult diapers. See Fig. 3.

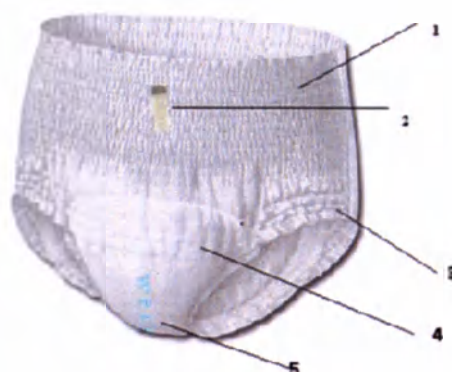
Fig 3 . Adult Pants. Schematic Diagram

1 - elasticized waistband, 2 - adhesive tapes used for folding the pants before disposal, 3- protective side bands for legs, 4- elasticized tapes for the maximum comfort of usage, 5 - as in diapers, wetness indicator imprinted on the outside of the product (date and time of manufacture, size).

6.3 Adult Diapers: Giggles

Product sizes inspected by the Quality Control: XS/Extra small, S/Small, M/Medium, L/Large and XL/Extra large.

1-Inspected size: XS/Extra Small (30 pcs. per pack)

Sl. No.	Description	Value	Tolerance
1	SAP (g)	7	(+-) 0.6
2	Fluffed Pulp (g)	35	(+-)3
3	Width (mm)	420	(+-)10
4	Length (mm)	640	(+-)10
5	Length of absorbent core (mm)	550	(+-)10
6	Side barrier (width, mm)	430	(+-)10
7	Top layer (width)	235	(+-)5
8	Side straps (length, mm)	25	(+-)3

2-Inspected size: S/Small (9/30 pcs. per pack)

Sl. No.	Description	Value	Tolerance
1	SAP (g)	8	(+-) 0.6
2	Fluffed Pulp (g)	40	(+-)3
3	Width (mm)	520	(+-)10
4	Length (mm)	700	(+-)10
5	Length of absorbent core (mm)	550	(+-)10
6	Side barrier (width, mm)	430	(+-)10
7	Top layer (width)	235	(+-)5
8	Side straps (length, mm)	25	(+-)3

3-Inspected size: M/Medium (8/30 pcs. per pack)

Sl. No.	Description	Value	Tolerance
1	SAP (g)	9	(+-) 0.6
2	Fluffed Pulp (g)	45	(+-)3
3	Width (mm)	690	(+-)10
4	Length (mm)	800	(+-)10
5	Length of absorbent core (mm)	635	(+-)10
6	Side barrier (width, mm)	580	(+-)10
7	Top layer (width)	235	(+-)5
8	Side straps (length, mm)	30	(+-)3

4-Inspected size: L/Large (7/30 pcs. per pack)

Sl. No.	Description	Value	Tolerance
1	SAP (g)	12	(+-) 0.6
2	Fluffed Pulp (g)	50	(+-)4
3	Width (mm)	790	(+-)10
4	Length (mm)	920	(+-)10
5	Length of absorbent core (mm)	740	(+-)10
6	Side barrier (width, mm)	680	(+-)10
7	Top layer (width)	235	(+-)5
8	Side straps (length, mm)	30	(+-)3

5-Inspected size: XL/Extra Large (6/30 pcs. per pack)

Sl. No.	Description	Value	Tolerance
1	SAP (g)	18	(+-) 0.6
2	Fluffed Pulp (g)	70	(+-)4
3	Width (mm)	800	(+-)10
4	Length (mm)	950	(+-)10
5	Length of absorbent core (mm)	740	(+-)10
6	Side barrier (width, mm)	680	(+-)10
7	Top layer (width)	235	(+-)5
8	Side straps (length, mm)	30	(+-)3

Summary: The quality inspection results obtained on the adult diapers, Giggles type, are in full conformity with the applicable standards, It has been found that these products contained no harmful substances that could affect the human health. The adult diapers, Giggles type are made of the following materials: hotmelt adhesive, polyethylene, polypropylene, nonwoven, pulp, polyacrylate (SAP), polyester fiber, elastic.

6.4 Adult Diapers Dr. Comfort

Product sizes inspected by the Quality Control Dr. Comfort Adult Diapers: S/Small, M/Medium and L/Large.

1-Inspected size: S/Small (9/30pcs. per pack)

Sl. No.	Description	Value	Tolerance
1	SAP (g)	6	(+-) 0.6
2	Fluffed Pulp (g)	35	(+-)3
3	Width (mm)	500	(+-)10
4	Length (mm)	700	(+-)10
5	Length of absorbent core (mm)	550	(+-)10
6	Side barrier (width, mm)	430	(+-)10
7	Top layer (width)	235	(+-)5
8	Side straps (length, mm)	23	(+-)3

2-Inspected size: M/Medium (8/30 pcs. per pack)

	Description	Value	Tolerance
1	SAP (g)	8	(+-) 0.6
2	Fluffed Pulp (g)	40	(+-)3
3	Width (mm)	690	(+-)10
4	Length (mm)	800	(+-)10
5	Length of absorbent core (mm)	635	(+-)10
6	Side barrier (width, mm)	580	(+-)10
7	Top layer (width)	235	(+-)5
8	Side straps (length, mm)	23	(+-)3

3-Inspected size: L/Large (7/30 pcs. per pack)

Sl. No.	Description	Value	Tolerance
1	SAP (g)	10	(+-) 0.6
2	Fluffed Pulp (g)	45	(+-)4
3	Width (mm)	790	(+-)10
4	Length (mm)	920	(+-)10
5	Length of absorbent core (mm)	740	(+-)10
6	Side barrier (width, mm)	680	(+-)10
7	Top layer (width)	235	(+-)5
8	Side straps (length, mm)	23	(+-)3

Summary: The quality inspection results obtained on the adult diapers, Comfort type are in full conformity with the applicable standards.

It has been found that these products contained no harmful substances that could affect the human health. Dr. Comfort adult diapers are made of the following materials: hotmelt adhesive, polyethylene, polypropylene, nonwoven, pulp, polyacrylate (SAP), polyester fiber, elastic.

6.5 Adult Pants Giggles

Product sizes inspected by the Quality Control: M/Medium (waist size 70x110 cm), L/Large (waist size 95x135 cm).

1-Inspected size: M/Medium (10 pcs. per pack)

Sl. No.	Description	Value	Tolerance
1	SAP (g)	10	(+-) 0.6
2	Fluffed Pulp (g)	30	(+-)3
3	Width (mm)	630	(+-)10
4	Length (mm)	800	(+-)10
5	Length of absorbent core (mm)	480	(+-)10

2-Inspected size: L/Large (10 pcs. per pack)

Sl. No.	Description	Value	Tolerance
1	SAP (g)	12	(+-) 0.6
2	Fluffed Pulp (g)	33	(+-)3
3	Width (mm)	760	(+-)10
4	Length (mm)	840	(+-)10
5	Length of absorbent core (mm)	500	(+-)10

Summary: The quality inspection results obtained on the adult pants, Giggles type are in conformity with the applicable standards.

It has been found that these products contain no harmful substances that could affect the human health. Adult pants, Giggles type are made of the following materials: hotmelt adhesive, hydrophilic nonwoven, fluffed pulp, polyacrylate (SAP), polyester fiber, elastic.

7 Packaging

7.1 Consumer Package

The adult diapers in the amount established by the Manufacturer

- Adult Diapers Giggles, sizes:
XS-30 pcs., S-9/30 pcs., M-8/30 pcs., L-7/30 pcs., XL-6/30 pcs.
- Adult Diapers Dr.Comfort, sizes:
S-9/30 pcs., M-8/30 pcs., L-7/30 pcs.
- Adult Pants Giggles, sizes:
M -10 pcs.,L-10 pcs.

are placed in low density polyethylene packs for protection during shipping and storage. Marking of consumer packaging is made in Russian.

7.2 Shipping Package

The adult diapers are placed in a polyethylene bag, 4 packs per bag in case of 30-pcs. packs or 12 packs per bag in case of 6, 7, 8, 9-pcs. packs. The adult pants are shipped in cartons, 4 packs per carton. The shipping package is in conformity with the applicable standards. It is designed to protect the products against damage during shipping by different means of road and sea transport, transshipment, storage and handling, provided that the proper handling practices are applied.

8 Marking and Labelling

8.1 Marking of Consumer Packages

Labeling must be accurate, verifiable and readable. Labeling applied to the packaging by a printing method, where in layering paint is not allowed. Each consumer package of the adult diapers is labelled with the following information:

- information about the manufacturer (name and address with indication of country of manufacture in English and Russian, phone, email address, website, phone service);
- the name of the country-manufacturer: the inscription "Made in Turkey";
- product description, trade mark, group and size (in accordance with waist size);
- instructions for use (pictograms and text);
- No. of products per pack;
- batch no. containing the date of manufacture (DD/MM/YY);
- expiry date;
- list of ingredients;
- distinctive characteristics of the diaper in accordance with the technical performance (diaper-pants in the form of a drawing);
- the presence of the wetness indicator in the form of drawing and text;
- symbol "Do not deposit into toilets";
- symbol "No litter";
- symbol "Do not re-use";
- details of the Manufacturer's certificates;
- bar code;
- symbol "Keep dry";
- symbol "Keep away from sunlight";
- sign "Temperature range" storage;
- identification of the authorized representative of the manufacturer in Russia (name, address, telephone number);
- number of the registration certificate of the Russian Federation.

Not allowed to put information about the special properties of the diaper without the appropriate documentation.

9 Product Performance.

9.1 Microbiology

Since manufacturers of the disposable hygiene products are obliged to care about the health and well being of the users, Sevinçler Sağlık Ürünleri is committed to the strict adherence of the following microbiological standards. Therefore, our products are in conformity with EU Directive 2001/95 / EC.

Table 2. Microbiological characteristics of diapers

Description	Amount per diaper, (g)
Mold	negative
Yeast	negative
E.coli	negative
Staphylococci	negative
Pseudomonas aeruginosa	negative
Bioburden	negative

9.2 Physical and Chemical Properties of Products

Form:	Solid
Colour:	Patterns on white background
Odour:	Characteristic
Freezing point:	N/A
Boiling point:	N/A
Flash point:	N/A
Ignition temperature:	N/A
Oxidizing properties	N/A
Self-ignition point:	N/A
Lower explosive limit:	N/A
Higher explosive limit:	N/A
Vapourization rate:	N/A
Gas pressure:	N/A
Apparent density:	N/A

Relative vapour density:	N/A
Water solubility:	N/A
pH value	5.5 - 7.5
log Pow	N/A
Viscosity (dynamic)	N/A
Viscosity (kinematic)	N/A

9.3 Contraindications and possible side effects

If you follow the instructions, contraindications and possible side effects are not observed.

9.4 Performance of Adult Diapers Giggles

Description	Value				
Description	Giggles size XS	Giggles size S	Giggles size M	Giggles size L	Giggles size XL
Unit Weight (g)	57±3	66±3	76±3	88±3	115±3
Absorbency (g), not less	1300	1550	1900	2150	3000
Sorption capacity after centrifugation (g), not less	280	320	450	550	700
Absorption rate (cm ³ /s), not less	3,0	3,0	3,0	3,0	3,0
Resorption (g), not more	3,0	3,0	3,0	3,0	3,0

9.5 Performance of Adult Diapers Dr.Comfort

Description	Value		
Description	Dr.Comfort size S	Dr.Comfort size M	Dr.Comfort size L
Unit Weight (g)	56±3	73±3	87±3
Absorbency (g), not less	1300	1500	1800
Sorption capacity after centrifugation (g), not less	250	380	460
Absorption rate (cm ³ /s), not less	2,7	2,7	2,7
Resorption (g), not more	3,5	3,5	3,5

9.6 Performance of Adult Pants Giggles

Description	Value	
Description	Giggles size M	Giggles size L
Unit Weight (g)	69±3	77±3

Absorbency (g), not less	1500	1800
Sorption capacity after centrifugation (g), not less	330	440
Absorption rate (cm ³ /s), not less	2,8	2,8
Resorption (g), not more	3,8	3,8

10 Safety Requirements

The products by Sevinçler Sağlık Ürünleri are in full conformity with the following requirements:

The products conform to all certificates and directives that are in place in the Russian federation and the Eurasian Economic Community customs union.

No external defects are allowed in the products including mechanical damage (broken edges, cuts, damaged fasteners, etc.), stains of various origin, inclusions visible to a naked eye.

The printed symbols on the diapers must be clear, without blurring or spaces. No surface lifting or scuffing is allowed.

The technical documentation for the specific type of products or age must contain the details of the composition, linear dimensions, configuration and style of the diapers.

The products have no strong chemical odour.

The products contain no toxic substances that could affect the human health.

To ensure the safe use of the products, please adhere to the following rules:

- Do not deposit the used diapers into toilets.
- Do not use if defective or the package broken.
- Keep away from open flame.
- Follow the instructions for use.
- Change when necessary.
- Do not use after the expiry date.

11 Environmental

During the manufacture, all the waste is disposed of in accordance with the national environmental law. No hazardous chemical substances form in the manufacture cycle. The only waste the company generates is the waste of raw materials or products that do not meet the specifications. Such waste is collected and sent to recycling.

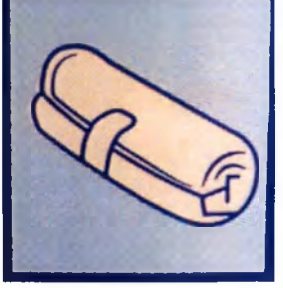
We do not use any water in the manufacture, so not water waste is generated. We do not generate any gaseous waste, therefore, there is no impact on the environment.

13 Use

13.1 Instruction for Use of Adult Diapers:

Pictogram	Description
	Place the diaper under the patient so that the side fasteners are behind them.
	Make sure that the side barriers, that hold the fluid, fold inwards.
	Secure the side fasteners on the front tape, symmetrically on both sides. You can open and close the diaper as many times as necessary. After use, discard the diaper in a specialized container. Do not deposit it into the toilet.

13.2 Instructions for Use of Giggles Adult Pants

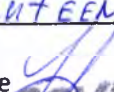
Pictogram	Description
	To start using the disposable pants on, pull them on upwards as regular underwear, keeping the adhesive tape on your back.
	Make sure that the pants sit on your waist so that the gore fits to your body snugly.
	To change the pants, break the side seams from your waist downwards. Open the pants and carefully take them off.
	After taking the disposable pants off, carefully roll them and secure with the adhesive tape before discarding.

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