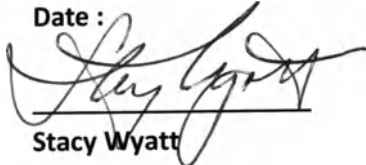
26422
Approved by
Kerr Corporation, USA

Date :

Stacy Wyatt
Senior Product Manager

Instruction for Use for Medical Device
Polymerization Dental Lamp DEMI ULTRA with Accessories

Legal Manufacturer:

Kerr Corporation, 1717 West Collins Avenue, Orange, CA 92867, USA

Contract Manufacturer:

Plexus (Xiamen) Co LTD., No 6 Xiangxing 2 Road, Modern Logistics Zone, Xiamen, 361006, China.

Metrex Research LLC, 28210 Wick Road, Romulus, Michigan 48174, USA.

Suzhou Pac-Dent Technology Co.,Ltd, Xiangcheng District Yuanhe Technology Zone, Suchou City, Jiangsu Province, Jiansu 215000, China.

Table of contents

1. Product Information	3
1.1. Description and Indications for Use	3
1.2. System components.....	3
1.3. Features	4
1.4. Controls	4
1.5. Operating Modes	5
1.6. Periodic Level Shifting (PLS).....	5
2. Product Safety Warnings and Cautions.....	5
2.1. Warnings:	5
2.2. Cautions:.....	8
3. Product Setup and First-Time Use	9
3.1. Power Supply Assembly.....	9
3.2. Using the Charging Dock.....	10
4. Product Testing	11
4.1. Test the LED Light Attachment with the Radiometer	11
4.2. Verifying Radiometer Readings with a Hardness Disk Kit	12
5. Product Use	12
5.1. Adjust the LED Light Attachment and Light Shield	12
5.2. Verify the Handpiece is Charged	Error! Bookmark not defined.
5.3. Select Curing Time.....	Error! Bookmark not defined.
5.4. Activate the Curing Light.....	Error! Bookmark not defined.
5.5. Other Product Use Notes.....	14
5.6. Recommended Curing Times	14
6. Product Care and Maintenance	15
6.1. Operating Environment	15
6.2. Transport and Storage Environment.....	15
6.3. Troubleshooting	15
6.4. Cleaning	15
7. Product Specifications	17

CALIFORNIA ALL-PURPOSE ACKNOWLEDGMENT

CIVIL CODE § 1189

A notary public or other officer completing this certificate verifies only the identity of the individual who signed the document to which this certificate is attached, and not the truthfulness, accuracy, or validity of that document.

State of California)

County of Orange)

On August 2, 2014 before me, Patricia Contreras, Notary Public
Date Here Insert Name and Title of the Officer

personally appeared Stacy Wyatt
Name(s) of Signer(s)

who proved to me on the basis of satisfactory evidence to be the person(s) whose name(s) is/are subscribed to the within instrument and acknowledged to me that he/she/they executed the same in his/her/their authorized capacity(ies), and that by his/her/their signature(s) on the instrument the person(s), or the entity upon behalf of which the person(s) acted, executed the instrument.

I certify under PENALTY OF PERJURY under the laws of the State of California that the foregoing paragraph is true and correct.

WITNESS my hand and official seal.

Signature Patricia Contreras
Signature of Notary Public

1	Light Shield
1	Power Supply with Adapter Set
1	Instructions for Use
1	Demi Ultra Disposable Barrier Bags (pack of 5)

8. Parts and Accessories.....	17
9. Disposal Instruction	17

1. Product Information

1.1. Description and Indications for Use

The Demi Ultra is a light emitting diode (LED) visible curing device used for the polymerization of light-cured materials by dental professionals.

IMPORTANT: Read all instructions carefully prior to set-up and use of the Demi Ultra. Listed below are the components, features, controls and operating modes of the system.

1.2. System components

Quantity	Description
1	Demi Ultra Handpiece
1	8 mm LED Light Attachment
1	Charging Dock with LED Radiometer
1	Light Shield
1	Power Supply with Adapter Set
1	Instructions for Use
1	Demi Ultra Disposable Barrier Bags (pack of 5)

1.3. Features


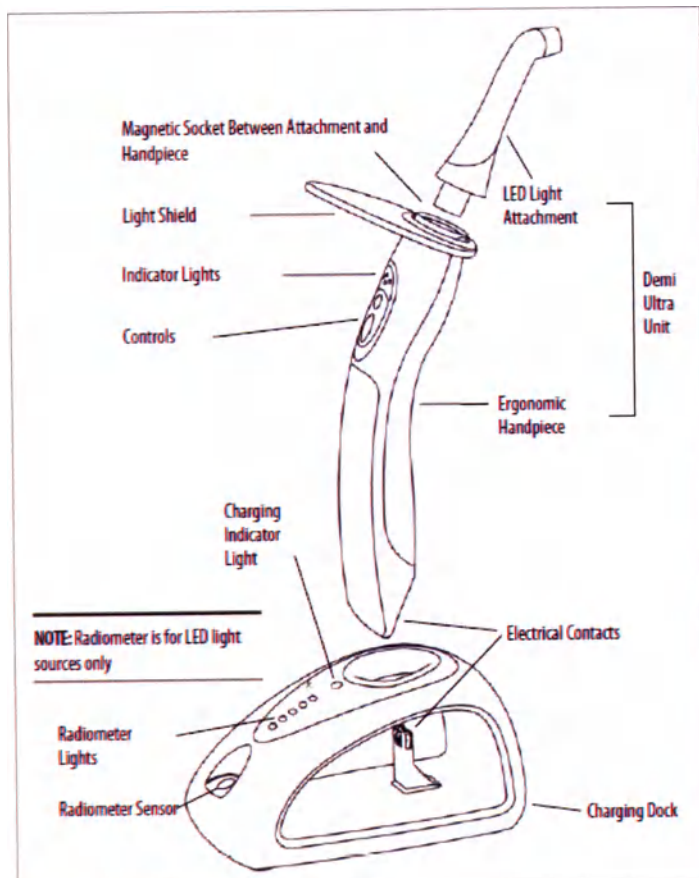
Figure 1 – Demi Ultra System

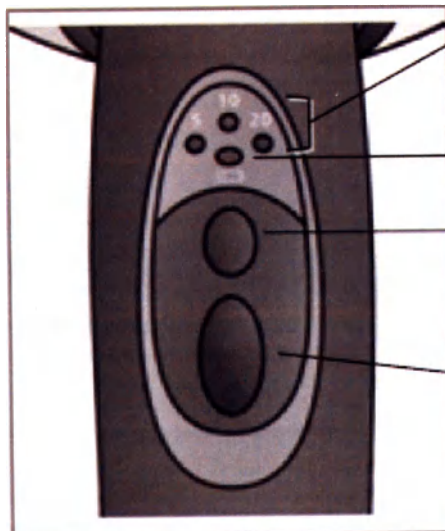Do not autoclave the LED Light Attachment or any part of the curing light system. Demi Ultra parts are not autoclaveable.

1.4. Controls

Curing Time Indicator Lights: Three LEDs represent the three curing modes; from left to right these are: 5 second, 10 second, and 20 second cycles.

Charge Indicator Light: This LED indicates the state of charge of the power source.

Curing Time Selector: Pressing this button sequentially toggles the unit through the three curing modes. If the unit is in sleep mode, pressing this button awakens the unit in the curing mode last used.

LED Light Button: Pressing this button initiates the selected curing cycle. Pressing this button during the cure cycle will interrupt/end the cycle. If the unit is in sleep mode, pressing this button awakens the unit in the curing mode last used.

Figure 2 – Demi Ultra controls

1.5. Operating Modes

NOTE: For each of the modes explained below, it is assumed that the LED Light Attachment is securely attached to the Handpiece:

Active Mode	The "ready" state of the Demi Ultra is Active Mode. In this mode a Curing Time Indicator Light will be illuminated. The user may cycle between 5 second, 10 second, and 20 second curing modes by pressing the Curing Time Selector. Curing cycles are also initiated when the unit is in Active Mode.
Sleep Mode	The Demi Ultra goes into sleep mode after approximately 60 seconds of no activity, and when the unit is initially removed from the Charging Dock when charging is complete. While in sleep mode, all LEDs are turned off and the curing light goes into a lower power state. It can be awakened by pressing either the Curing Time Selector or LED Light Button once, at which time the unit will be in Active Mode. The unit will reflect the last cure mode chosen.
Off Mode	There are two conditions that put the Demi Ultra into Off Mode: (1) When it is seated in the Charging Dock, and (2) When the unit becomes completely drained. For example, if the unit is not in the Charging Dock and is left idle for many days the charge will completely drain. Pressing the control buttons will not awaken the unit if it is in Off Mode. If the charge is drained, it must be first charged and then removed from the Charging Dock.
Silent Mode	The Demi Ultra in Active Mode emits sounds to signal activity, such as intermittent beeps while curing. To mute these sounds, press and hold down the Curing Time Selector Button for 10 seconds. After 10 seconds the unit will beep twice, signaling Silent Mode has been turned on. The unit will remember this setting, and not produce sounds until Silent Mode is turned off using the same procedure as was used to switch it on.

1.6. Periodic Level Shifting (PLS)

The Demi Ultra employs proprietary Periodic Level Shifting technology. PLS is a constant periodic cycling of light from a base output to a higher level every second during a curing cycle. PLS technology provides faster, deeper cures without overheating.

2. Product Safety Warnings and Cautions

Consistent function and safe operation of this curing light system is dependent on the user's compliance with the following safety procedures as well as the specific safety comments and recommendations presented throughout these Instructions for Use. Users must be qualified medical professionals.

2.1. Warnings:

	The Demi Ultra Curing Light is an electric device designed to meet worldwide electrical safety standards, including U.S., Canada, and Europe, to be safe and effective for all dental applications. <u>To avoid electric shock:</u> •Do not attempt to open or alter the unit in any way. Only service centers authorized by Kerr Corporation may open the unit housing and repair the device. •Do not introduce any foreign objects into the housing of the unit. •Use only the Demi Ultra charging dock when recharging this product. Never attempt to use any other device for recharging.
---	--

	•Connect the power plug into a suitably grounded and approved outlet. If using an extension cable, make sure that the ground line is not interrupted. •Always unplug the charging dock before disinfecting. •Never use the power supply if it or the cord has been damaged.
	The Demi Ultra Curing Light produces high-output curing energy. Prolonged exposure to the eyes, exposed skin, oral mucosa, or tissues near the pulp can result in irritation or injury. It is important to observe the following precautions and procedures: Positioning: When curing, position the LED Light Attachment tip directly over the cure material. Do not place light directly on or toward unprotected gingiva or skin. Eye exposure: Do not look directly at the light emitted from this curing unit. Do not operate this device without suitable eye protection for the operator, assistant, and patient. Select eye protection that filters blue-violet and ultraviolet light. Securely attach Light Shield before each use. Avoid prolonged exposure of unprotected eyes to light scatter or reflected light. If light exposure causes blurring/starring of vision or visual after-image that persists, consult a physician. Skin exposure: Avoid prolonged exposure to skin. If skin rash, sensitization or other reaction occurs, discontinue use and seek medical attention. Oral mucosa exposure: Avoid prolonged exposure to oral soft tissues. Prolonged exposure may cause soft tissue irritation or medical attention.

2.2. Cautions:

- Use this product only as specifically explained in this manual. Any use of this product inconsistent with these Instructions for Use is at the discretion, and is the sole responsibility of, the practitioner.

- The Demi Ultra should be operated only with the supplied LED Light Attachment or other Demi Ultra LED Light Attachments when available. Do not attempt to use this product with other Demetron light guides. This product's warranty does not cover any damage resulting from the attempted use with any light guides or light attachments other than original Demi Ultra LED Light Attachments. There are no user-serviceable parts inside the curing unit. Do not open or tamper with the curing unit.

- Check the LED Light Attachment for adequate light output before each procedure. Failure to verify output may allow inadequate curing (see Product Testing section).

Only the Demi Ultra built-in radiometer or a Demetron radiometer (part number 910726) should be used. Other radiometers may give erroneous output data and cannot be relied upon to test Demi Ultra output.

- The Demi Ultra should not be used on patients, or by users, fitted with implanted or external medical devices, such as an implanted heart pacemaker, who have been cautioned in regard to being exposed to small electrical appliances, such as electrical shavers, hair dryers, etc. In all such cases, contact the prescribing medical device practitioner prior to exposing a patient, or user, to the Demi Ultra.

- The System must be maintained in a clean, undamaged condition. An unclean or damaged unit should not be used. Do not attempt to operate a damaged System until it has been thoroughly inspected and serviced by qualified service person.

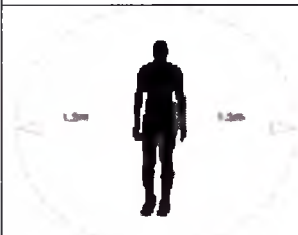
- To prevent cross-contamination, use a disposable plastic sleeve over the LED Light Attachment during each use. Do not allow contact of restorative materials with lens on the end of the LED Light Attachment. Attempts to remove cured material may scratch the lens. A contaminated or chipped/scratched lens will reduce the effectiveness of the unit and could result in a restoration that is not cured completely.

- Items labeled as "Single Use" are intended for single use only. Discard after use. In order to prevent cross-contamination, do not reuse with other patients.

- Do not immerse any system components in water or disinfectant.

- Do not spray liquids directly onto the System components. Spray a towel, then wipe the light. Prevent liquids from entering openings on unit, for example the Magnetic Socket for the LED Light Attachment. Always clean the unit with the LED Light Attachment installed.

- If the Demi Ultra is moved from a cold to a warm environment condensation on the inside of the unit may be a potential risk. In such instances, the product should be brought to ambient temperature before charging and/or operation.

	•Persons who have had surgery for eye cataracts are especially sensitive to light and generally should not use or be exposed to light from the Demi Ultra unless they employ suitable protective measures to prevent eye exposure (as described above).
	•This product was developed to comply with relevant EMC regulations and standards and has been tested to verify its conformity with legal requirements. However, the possibility of EMC disruptions cannot be completely excluded due to variables that may exist within the work environment, such as power supply, wiring, and other ambient conditions. If you should experience problems in the operation of this product, move the unit to a different location. See the appendix for the EMC manufacturer's declaration and the recommended separation distances between the Demi Ultra and portable and mobile RF communications equipment.
	•The dimensions in the figure below represent the minimum patient environment. The Demi Ultra Charging Dock should not be stored within the patient environment as a precaution against inadvertent electric shock.
	

3. Product Setup and First-Time Use

3.1. Power Supply Assembly

The power supply is a universal-input unit that accepts 100V - 240V AC nominal.

NOTE: Only Kerr-supplied power supplies are to be used to power the Demi Ultra. There are two types of power supplies available. The power supplies differ slightly in appearance, but operate the same both electrically and mechanically.

Attach the Adapter to the Power Supply

1. Select the proper adapter. Each adapter has a slot into which the retaining latch on the power supply fits.
2. Place the adapter onto the power supply so that the tabs on the adapter are aligned with their respective slots on the power supply (Figure 3).
3. Slide the adapter into the power supply until a click is heard and the adapter is locked in place.

Figure 3 – Electrical power supply with an adapter

NOTE: When correctly mounted, the adapter will not rock or move. If the adapter feels loose, remove it and re-install.

Remove the Adapter from the Power Supply

1. Press on the latch retainer button; slide the adapter away from the power supply, and remove the adapter (Figure 4).
2. Retain unused adapters for future use.

Figure 4 – Removing the adapter

3.2. Using the Charging Dock

The Demi Ultra System uses a battery-less power source that recharges very quickly. Follow these instructions for effective operation:

1. Plug the power supply into an electrical outlet.
2. Plug the power supply into the Charging Dock. The socket is located at the back of the Charging Dock (Figure 5).
3. Insert the Demi Ultra unit into the Charging Dock with the controls facing the radiometer. When charging, the unit's indicator lights turn off and the charging dock charging indicator lights turn on. These are on the upper and lower surface of the dock's "bridge" (Figure 5). Amber light indicates the unit is charging. Green indicates the unit is fully charged.

When setting up the Charging Dock, always confirm you are using the correct cable. Place the Charging Dock as close as possible to the wall outlet. Never allow the cord to drape across an open space where it can be accidentally pulled. When moving the Charging Dock, unplug it from the wall outlet by hand and remove the Handpiece. Never pull the cord from a distance. Unplug the cord from the Charging Dock only when cleaning.

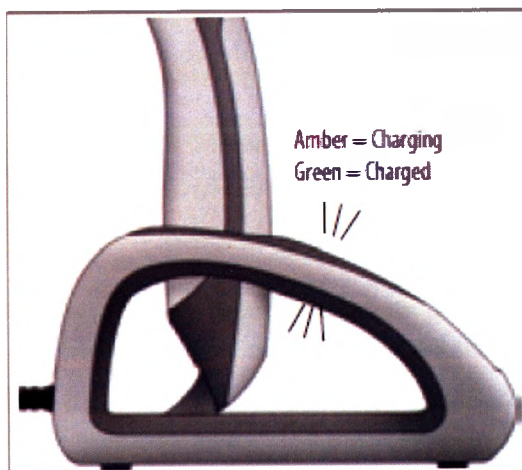
Figure 5 – Charging Indicator lighting

For Optimal Power Source Performance:

For optimal power source performance, it is recommended to leave the Demi Ultra resting in the Charging Dock with an active power connection whenever not in use, including overnight.

When recharged regularly, the Demi Ultra requires only 40 seconds for a full recharge. If the power source fully drains after several days of inactivity, the unit may require up to 70 seconds for a full recharge. After fully

recharging, the unit will again require only 40 seconds for subsequent recharges if operation is regular as stated above.

A full recharge yields approximately 250 seconds of run time, or 25 10-second cures.

4. Product Testing

4.1. Test the LED Light Attachment with the Radiometer

It is highly recommended that curing capability be tested using a hardness disk to ensure a complete cure. When a hardness disk is not available, light output can be verified with a radiometer to estimate curing effectiveness. Failure to perform either of these tests may lead to incomplete cures of dental restorative materials.

The Demi Ultra Charging Dock radiometer should be used only with the Demi Ultra when fitted with the 8mm LED Light Attachment. Use of the radiometer with other curing lights or with larger/smaller diameter light sources may result in inaccurate measurements.

Verify the Demi Ultra performance before each use using the radiometer built into the Charging Dock:

1. Set the curing time interval for a 10 second cycle.
2. Verify that the radiometer sensor is free of debris that could impact the accuracy of the measurement. If the radiometer sensor requires cleaning, clean it per instructions in Product Care and Maintenance.
3. Carefully hold the unit so that the lens of the LED Light Attachment is flush with the radiometer sensor (Figure 6) and centered within the white circle provided.

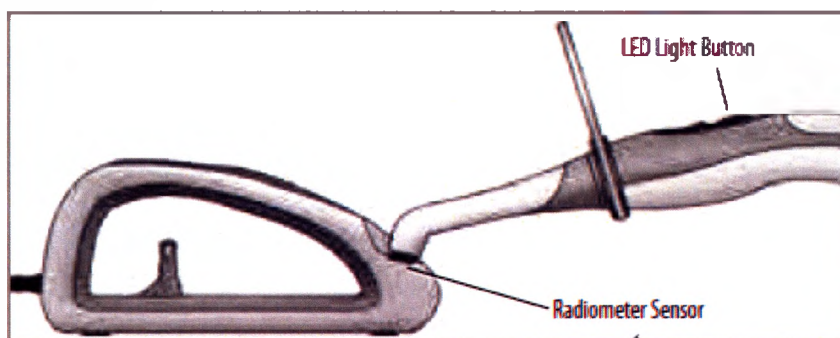
Figure 6 – The LED Light Attachment aligned with the Radiometer Sensor

4. While holding the unit steadily in position, press and release the LED Light Button. In response to the bright light, the radiometer indicator lights will illuminate from the bottom up, providing a reading of the light intensity.

5. Wait until the light goes off before moving the LED Light Attachment away from the radiometer sensor.

Lights	Color	Intensity (mW/cm ²)
5	Blue	1250
4	Blue	1050
3	Blue	950
2	Amber	850
1	Amber	750

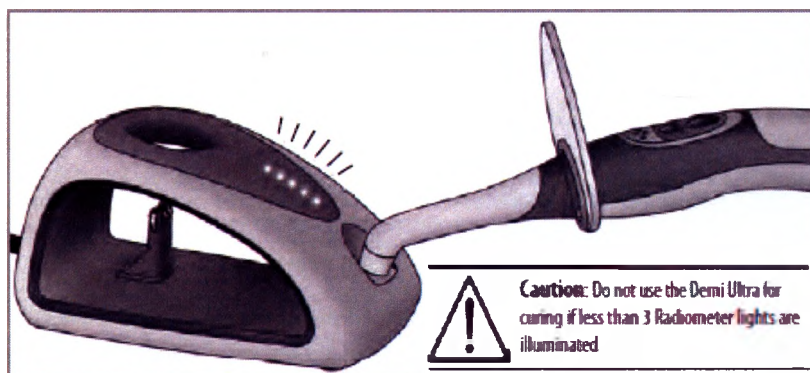
Figure 7 – Radiometer test response

The bottom two lights are amber, the top three lights are blue. At least one blue light must be illuminated for a proper cure. Do not use the Demi Ultra for curing if only the amber lights are illuminated. Reference the Troubleshooting section for possible solutions.

The lower four indicator lights emit a steady glow. The top indicator light is sensitive to the Periodic Level Shifting (PLS) of the LED Light Attachment, causing a blinking or flickering effect.

4.2. Verifying Radiometer Readings with a Hardness Disk Kit

	The radiometer reading is just an estimate of curing performance. Actual curing results depend on the materials and thickness being cured. It is highly recommended that curing of materials be tested using a hardness disk kit to ensure a complete cure. Follow the instructions provided with the kit for the most accurate testing results.
--	---

5. Product Use

	To prevent cross-contamination, a disposal sleeve must be used over the Demi Ultra with each use. This will also prevent transfer of resin to the LED Light Attachment.
---	--

Kerr Corporation recommends the use of the Demi Ultra Barrier Bag to prevent cross-contamination when using the Demi Ultra. See the Parts and Accessories section for ordering information.

5.1. Adjust the LED Light Attachment and Light Shield

Remove the unit from the Charging Dock. Ensure the LED Light Attachment is properly seated in the Magnetic Socket and the Light Shield is firmly attached. Rotate each to the desired position for curing.

Figure 8 – Demi Ultra LED Light Attachment rotation

5.2. Verify the Handpiece is Charged

In Active Mode, a Curing Time Indicator Light is on, and the Charge Indicator Light is off. When there are 40 seconds or less of curing time left, the Charge Indicator Light glows amber (see Figure 9) warning the user that the charge is almost depleted. The Charge Indicator Light turns red when there is not enough charge to complete the selected curing cycle duration. If curing duration is reduced, the light may change back to amber, indicating there is enough charge for a shorter curing cycle.

Figure 9 – Charge Indicator Light "on"

5.3. Select Curing Time

The three topmost LEDs indicate which time interval (cycle) is selected. To change the current selection, press the Curing Time Selector until the desired time interval is indicated (see Figure 10).

5.4. Activate the Curing Light

Position the light as close to the tooth as possible without touching it. Press and release the LED Light Button (see Figure 11). Be sure to hold the Demi Ultra in position until the curing cycle has completed.

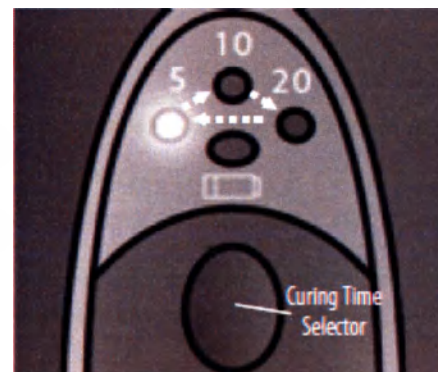
Figure 10 – Selecting a curing time

5.5. Other Product Use Notes

If the LED Light Button is pressed at any time during a curing cycle it will stop the cycle. To disable PLS, press and hold the LED Light Button when starting the curing cycle. PLS will be disabled until the button is released. While PLS is disabled the Demi Ultra will maintain a steady base light output of 1,100 mW/cm² and the unit will beep once per second.

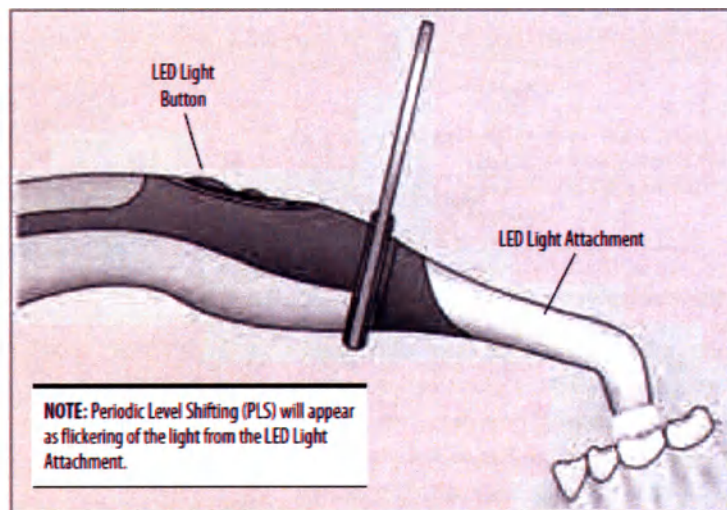
Figure 11 – Curing composite on a tooth

When not in Silent Mode, the unit will emit a short beep when the button is pressed. The beep indicates the curing cycle has begun. A slightly longer beep will sound at the end of the curing cycle. During the 10 and 20- second cycles, a beep is sounded every 5 seconds as well as at the end of the cycle.

5.6. Recommended Curing Times

The Demi Ultra is a high-output curing unit. Kerr testing indicates the following recommended curing times as estimates. Always follow the recommended curing times published by the manufacturer of your dental restorative materials when available.

Material	Curing Time
Universal composite shades A3 and lighter (2 mm depth)	5 seconds
Universal composite shades A3.5 and darker (2mm depth)	10 seconds
SonicFill shades A3 and lighter (5mm depth)	20 seconds*
Bonding agents, universal resin luting cements and self-etch, self-adhesive cements	5 seconds

*Kerr testing indicates that SonicFill attains a 5mm cure in 10 seconds with a light output exceeding 1000mW/cm². Although Demi Ultra provides light output in excess of 1000 mW/cm², Kerr continues to adopt a conservative position and recommends a 20 second cure.

It is highly recommended that curing capability be tested using a hardness disk to ensure a complete cure. When a hardness disk is not available, light output can be verified with a radiometer to estimate curing effectiveness. Failure to perform either of these tests may lead to incomplete cures of dental restorative materials

6. Product Care and Maintenance

6.1. Operating Environment

See Product Specifications section.

6.2. Transport and Storage Environment

- Temperature: -4°F to 104°F (-20°C to 40°C)
- Relative humidity: 10% to 85%
- Atmospheric pressure: 0.5 atm to 1.0 atm (500 hPa to 1060 hPa)

See Product Specifications section for dimensions and weights of system components.

6.3. Troubleshooting

Problem	Resolution
The Handpiece indicator lights don't illuminate.	The electrical charge may be completely drained. Put the Handpiece in the Charging Dock for 70 seconds.
The Handpiece is in the Charging Dock, but the Charging Indicator Lights don't light.	Check to verify that the power cord is inserted into the Dock and into a power receptacle.
	Check the power supply and cord for damage. If there is damage discontinue use of the power supply.
The Handpiece Indicator Lights are blinking.	Verify that the LED Light Attachment is fully seated into the Magnetic Socket.
The Handpiece sounds (beeps) have stopped.	The Handpiece is in Silent Mode. Press and hold the Curing Time Selector button for 10 seconds to turn off Silent Mode. The unit will beep twice to indicate the Silent Mode setting has been changed.
The Handpiece beeps once each second during a curing cycle.	This indicates PLS is not active, which occurs when the LED Light Button is pressed and held. Press and release the button when activating a curing cycle.
A Radiometer test has less than three lights illuminated.	First, confirm that the LED Light Attachment is flush with the radiometer Sensor and centered within the white circle provided.
	Debris may be blocking or clouding the LED Light Attachment lens or the Radiometer Sensor. See cleaning instructions in the Product Care and Maintenance section.
The Charging Indicator Lights on the Charging Dock blink continuously when the Demi Ultra unit is in the Charging Dock.	The unit may not be seated properly in the Charging Dock. Remove and reseat it.
	Check the electrical contacts on the Charging Dock and Handpiece to see if they are fouled. If so, clean the contacts with a dry cloth, and reseat the unit.
	If the above actions do not resolve the problem, the Demi Ultra may require service by a qualified technician. Contact Kerr Customer Care (see back cover) for details on shipping to a service center.

If the resolutions suggested above do not resolve the issue, or for other issues not listed here, please contact customer care (see Contact Customer Care).

6.4. Cleaning

This product must be disinfected as normal preparation for each patient. Read this entire section before cleaning the Demi Ultra unit. Failure to follow these cleaning guidelines could cause damage.

Cleaning the LED Light Attachment and Handpiece

Do not autoclave the LED Light Attachment or any part of the curing light. Demi Ultra parts are not autoclaveable. Never use a blunt instrument to remove composite from the lens. Doing so will damage the lens. Do not immerse or spray cleaning solution directly onto components. Instead, spray cleaning solutions on a cloth to wipe. Do not clean the handpiece when it is resting in the Charging Dock.

1. Select the cleaning solution. Use a quaternary ammonium compound product like CaviCide® solution or CaviWipes® that contain 20% alcohol or less. If using CaviWipes®/CaviWipes™, follow the directions on the package.
2. Remove the Demi Ultra from the Charging Dock and remove the Light Shield from the Handpiece.
3. Wet a cloth or paper towel with cleaning solution and wipe onto the Handpiece, LED Light Attachment, and Light Shield. Avoid the lens at the end of the LED Light Attachment. Do not separate the Handpiece and the LED Light Attachment.

When cleaning this product, prevent liquids from entering any opening. Wipe carefully so that liquid does not run, and always clean the Handpiece with the LED Light Attachment attached to avoid getting moisture or debris in the Magnetic Socket.

Cleaning the Charging Dock and Radiometer

Be sure to disconnect the power cord from the Charging Dock before cleaning. The charging dock should be disinfected before every charge to prevent cross-contamination. Do not spray cleaning solution directly onto components. Instead, spray cleaning solutions on a cloth to wipe.

1. Disconnect the power cord from the Charging Dock.
2. Wet a cloth with cleaning solution and wipe solution onto the Dock surfaces.
3. Do not wipe or spray the electrical contacts.
4. Examine the Radiometer Sensor for debris and wipe as necessary
5. Wipe carefully so that liquid does not run. Do not allow fluid into the port for the electrical cord. Prevent liquids from entering any opening

Additional Cleaning Cautions:

Do not immerse any components of this product in solutions. Use of solutions other than those recommended may damage plastic parts and will void product warranty.
Do not use: Denatured alcohol, Lysol®, phenol, ammonia complex or iodine complex solutions.
Do not use cleaning solutions on the electrical contacts.

7. Product Specifications

Type of Information	Specifications
Light Emitting Diodes:	An array of three LEDs die-mounted on a substrate.
Output peak wavelength range:	450-470 nanometers
Baseline output:	1,100 mW/cm ²
Handpiece with LED Light Attachment dimensions:	Length: 10.7" (27 cm) Width: 1.3" (3.3 cm)
Charging Dock dimensions:	Length: 5.8" (14.7 cm) Width: 3.2" (8.1 cm) Height: 3.0" (7.6 cm)
Weight:	Handpiece: 6.5 oz (184 g) Charging Dock: 16.1 oz (456 g)
Power Supply performance:	25 x 10-second cures on a full charge Charging time: <70 seconds
Overcurrent Protection	Fuse
AC supply connection:	100-240V AC, 1.0-0.5A, 50-60Hz
Power input:	12VDC, nominal
Equipment class:	Class II 
Safety:	IEC 60601 -1
EMC (Electro-Magnetic Compliance):	IEC 60601-1-2
Protection from ingress of liquids:	Handpiece: IPX4 Charging Dock: IPX3
Operation:	Continuous operation patient application, duty cycle 20 seconds ON/1 minute OFF.
Operating environment:	Ambient temperature: 60°F to 104°F (16°C to 40°C) Relative humidity: 10% - 80%, non-condensing Atmospheric pressure: 0.5-atm to 1.0-atm (500 hPa - 1060 hPa)

8. Parts and Accessories

Type	Number	Description
System Parts	35665	8mm LED Light Attachment
	35666	Charging Dock with built-in LED Radiometer
	35667	Demi Ultra Handpiece
	35668	Light Shield
	35815	Power Supply with Adapter Set
		Demi Ultra Disposable Barrier Bags (pack of 5)
		Instruction for Use
Accessories	35837	Disposable Hardness Disk Kit (pack of 1)
	DEMIULTRA-100	Demi Ultra Barrier Bags (pack of 100)

9. Disposal Instruction

When you plan to dispose of the Demi Ultra, please return it to Kerr for environmentally sound recycling. Please contact your sales rep for details. The Waste Electrical and Electronic Equipment (WEEE) Directive forbids the disposal of waste electrical and electronic equipment as unsorted municipal waste, requiring that it be collected and recycled or disposed of separately.

Hazardous substances are present in electrical and electronic equipment and present potential risks to human health and the environment if disposed of in municipal landfills, which are not designed to prevent migration of substances into the soil and groundwater.

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Kerr Corporation, 1717 West Collins Avenue, Orange, CA 92867, США

Plexus (Xiamen) Co LTD., No 6 Xiangxing 2 Road, Modern Logistics Zone, Xiamen, 361006, Китай

Metrex Research LLC, 28210 Wick Road, Romulus, Michigan 48174, США

Suzhou Pac-Dent Technology Co., Ltd, 18, Yuanqi Road Suzhou City, 215133 Jiangsu, China (Китай)

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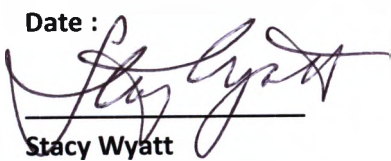
Stacy Wyatt

Senior Product Manager

Supplement for Instruction for Use for Medical Device
Polymerization Dental Lamp DEMI ULTRA with Accessories

Legal Manufacturer:

Kerr Corporation, 1717 West Collins Avenue, Orange, CA 92867, USA.

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Table of contents

1. Indications	3
2. Contraindications	3
3. Potential Complications/Side Effects	3
4. Life Time	4
5. Disposal Instruction	4
6. Protection of the Environment.....	4
7. Terms of Acceptance and Types of Tests	4
8. Sterility	6
9. List of Standards Applied	6
10. Packaging.....	7
11. Labelling	10
12. Product Warranty	13
13. Contact Information.....	13
14. Statement of Relation Among the Facilities	14

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State of California)

County of Orange)

On August 2, 2016 before me, Patricia Contreras, Notary Public
Date Here Insert Name and Title of the Officer

personally appeared STARV WYATT
Name(s) of Signer(s)

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It is recommended that all curing light users and patients employ suitable eye protection and protective clothing during operation of the Demi Ultra.

Persons who have had surgery for eye cataracts are especially sensitive to light and generally should not use or be exposed to light from the Demi Ultra unless they employ suitable protective measures to prevent eye exposure (as described above).

3. Potential Complications/Side Effects

Do not look directly at the light emitted from the LED Light Attachment. As with any bright light source, staring directly at the source may cause temporary effects such as after-image.

Under-cured resin is a potential complication if the proper light dosage is not delivered to the dental material that is why Kerr highly recommend the following curing times as estimated. Always follow the recommended curing times published by the manufacturer of your dental restorative materials when available.

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*Kerr testing indicates that SonicFill attains a 5mm cure in 10 seconds with a light output exceeding 1000mW/cm2. Although Demi Ultra provides light output in excess of 1000 mW/cm2, Kerr continues to adopt a conservative position

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2. Contraindications

The Demi Ultra should not be used on patients, or by users, fitted with implanted or external medical devices, such as an implanted heart pacemaker, who have been cautioned about being exposed to small electrical appliances, such as electrical shavers, hair dryers, etc. In all such cases, contact the prescribing medical device practitioner prior to exposing a patient, or user, to the Demi Ultra.

Persons having a history of photosensitive reactions, including individuals with urticarial solaris or erythropoietic protoporphyria, or who are using photosensitizing drugs, including 8-methoxypsoralen or dimethylchloro – tetracycline, should not be exposed to light from this unit.

It is recommended that all curing light users and patients employ suitable eye protection and protective clothing during operation of the Demi Ultra.

Persons who have had surgery for eye cataracts are especially sensitive to light and generally should not use or be exposed to light from the Demi Ultra unless they employ suitable protective measures to prevent eye exposure (as described above).

3. Potential Complications/Side Effects

Do not look directly at the light emitted from the LED Light Attachment. As with any bright light source, staring directly at the source may cause temporary effects such as after-image.

Under-cured resin is a potential complication if the proper light dosage is not delivered to the dental material that is why Kerr highly recommend the following curing times as estimated. Always follow the recommended curing times published by the manufacturer of your dental restorative materials when available.

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*Kerr testing indicates that SonicFill attains a 5mm cure in 10 seconds with a light output exceeding 1000mW/cm². Although Demi Ultra provides light output in excess of 1000 mW/cm², Kerr continues to adopt a conservative position

and recommends a 20 second cure.

4. Life Time

Kerr Corporation warrants for a period of 3 years from the date of purchase, this instrument shall be free from defect in material and workmanship and will perform satisfactory under normal use and service.

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Demi Ultra is not a consumable and does not produce medical waste during its usage. Final device disposition is recommended to be returned to Kerr. Please contact your representative for details. However, the device is compliant to RoHS Directive, which is related to disposal.

6. Protection of the Environment

Demi Ultra does not make negative effect to human and environmental during usage, storage and transportation.

7. Terms of Acceptance and Types of Tests

The Demi Ultra is intended for use in the electromagnetic environment specified below.

The customer or user of the Demi Ultra shall insure it is used in such an environment.

Guidance and manufacturer's declaration - electromagnetic emissions

EMISSIONS TEST	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
Radiated emissions CISPR 11	Class B	The Demi Ultra uses RF energy only for its internal function. Therefore, its RF emissions are very low and not likely to cause any interference in nearby electronic equipment.
Conducted emissions CISPR 11	Class B	The Demi Ultra is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power distribution network that supplies buildings or for domestic purposes.
Harmonic emissions IEC/EN 61000-3-2	Class A	
Flicker emissions IEC/EN 61000-3-3	Complies	

Guidance and manufacturer's declaration - electromagnetic immunity

IMMUNITY TEST	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
Electrostatic discharge (ESD) IEC/EN 61000-4-2	± 6kV contact ± 8kV air	Floors should be wood, concrete, or ceramic tile.
Conducted radio frequency IEC/EN 61000-4-6 Radiated, radio-frequency IEC/EN 61000-4-3	3V 150KHz to 80MHz 3V/m 80 MHz to 2.5GHz	Portable and mobile RF communications equipment should not be used close to any part of the Demi Ultra unit which includes the cable, other than when used at a recommended distance which is calculated based on the frequency of transmitter. Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol:  NOTE 1: At 80MHz and 800MHz, the higher frequency range applies.
Electrical Fast Transient/Burst IEC/EN 61000 4-4	±2kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC/EN 61000-4-5	±1kV line(s) to line(s) ±2kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Power frequency (50/60Hz) IEC/EN 61000-4-8	3 A/m	Power frequency magnetic field should be at levels characteristic of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC/EN 61000-4-11	<5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5s	Main power quality should be that of a typical commercial or hospital environment. NOTE 2: UT is the ac mains voltage prior to application of the test level.

a - Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobiles radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Demi Ultra is used exceeds the applicable RF compliance level above, the Demi Ultra should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the Demi Ultra.

b - Over the frequency range 150kHz to 80MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the Demi Ultra

The Demi Ultra is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Demi Ultra can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Demi Ultra as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150kHz to 80MHz	80MHz to 800MHz	800MHz to 2.5GHz
	$d=0.4 \sqrt{P}$	$d=0.4 \sqrt{P}$	$d=0.7 \sqrt{P}$
0.01	0.1	0.1	0.1
0.1	0.2	0.2	0.3
1	0.4	0.4	0.7
10	1.3	1.5	2.2
100	4.0	4.0	7.0

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

8. Sterility

According to Directive 93/42/EEC Demi Ultra is not sterile product.

To prevent cross-contamination, a disposal sleeve must be used over the Demi Ultra with each use. This will also prevent transfer of resin to the LED Light Attachment.

9. List of Standards Applied

EN 1041:2008	Information supplied by the manufacturer of medical devices
--------------	---

EN 1640:2009	Dentistry – Medical devices for dentistry - Equipment
BS EN ISO 9687:1996	Dental Equipment – Graphical Symbols
EN ISO 10993-1:2009	Biological evaluation of medical devices –Part 1: Evaluation & Testing
EN ISO 13485:2003	Medical devices – Quality management systems
EN ISO 14155:2011	Clinical investigation of medical devices for human subjects – Good Clinical Practice
EN ISO 14971:2012	Medical devices – Application of risk management to medical devices
ISO 15223-1:2012	Medical devices. Symbols to be used with medical device labels, labeling, and information to be supplied. Part 1: General requirements
EN 50581:2012	Technical documentation for the assessment of electrical and electronic products with respect to the restriction of hazardous substances – (RoHS)
EN 60601-1:2006	Medical electrical equipment –Part 1: General requirements for basic safety and essential performance
EN 60601-1-2:2007	Medical electrical equipment –Part 1-2: General requirements for basic safety and essential performance – Collateral standard: Electromagnetic compatibility-Requirements and Tests
PD IEC/TR 62471-2:2009	Photobiological safety of lamps and lamp systems
EN 62321:2009	Electrotechnical products – Determination of levels of six regulated substances (lead, mercury, cadmium, hexavalent chromium, polybrominated biphenyls, polybrominated diphenyl ethers) – (RoHS)
EN 62366:2008	Medical Devices – Application of Usability Engineering to Medical Devices
ISTA 1G	Package Integrity Performance Test Procedure
IEC 60529:2001, IP X3 Test	Degrees of Protection Provided by Enclosures (IP Code), Water Spray Test
Directive 2011/62/EU	Restriction of the use of certain hazardous substances in electrical and electronic equipment (RoHS)

10. Packaging

Demi® Ultra is packed in a rectangular magnetic box (PN DMC0463), manufactured from 1300g duplex board mounted on 157g copperplate complex dumb film.

Sizes: 154mm x 161mm x 319mm (lid), 149mm x 156mm x 317mm (base). Tolerance: ± 3 mm.

Demi Ultra handpiece/LED Light Attachment is set in a custom molded EVA foam tray (PN DMC0465).

Along with the handpiece/LED Light Attachment, the power supply with adapter set (transformer) is packaged in an individual cardboard box (DMC0590). Size: 127mm x 55mm x 142mm. Tolerance: ± 3 mm.

The charging dock with LED radiometer is packaged in a 450lb polish white paperboard box (DMC0464). Size: 85mm x 89mm x 147mm. Tolerance: ± 3 mm.

Light Shield and barrier bags Demi Ultra (5 pcs) are packaged into transparent zip-lock polybag.

Accessories: hardness disc (1 pcs in pack) is packaged into transparent zip-lock polybag, barrier bags Demi Ultra (DEMIULTRA-100) is packaged into rectangular box (Sizes: 307 mm x 80 mm x 44 mm. Tolerance: ± 3 mm).

Demi Ultra can be transported by all modes of covered (roofed) transport means in accordance with active transportation rules of each transport mode.

Symbols are used on the packaging:


REF	Part number
SN	Serial number
LOT	Lot number
MAN	Manufacturing date
	Date of manufacture
	Name of manufacturer
EC REP	Authorized representative in the European community
	Caution
	Dispose of properly
	Recycle
	ETL Listed – Intertek Electrical Testing Laboratories (ETL) certification mark
	Consult instructions for use
CE	The CE mark certifies that a product has met the EU consumer safety, health, and environmental requirements

	The RoHS mark certifies that a product has met the RoHS Directive 2002/95/EC, which is restricted using specific hazardous materials found in electrical and electronic products
Rx Only	Use only with medical prescription
	Do not reuse – single use only

11. Labelling


Demi Ultra Handpiece

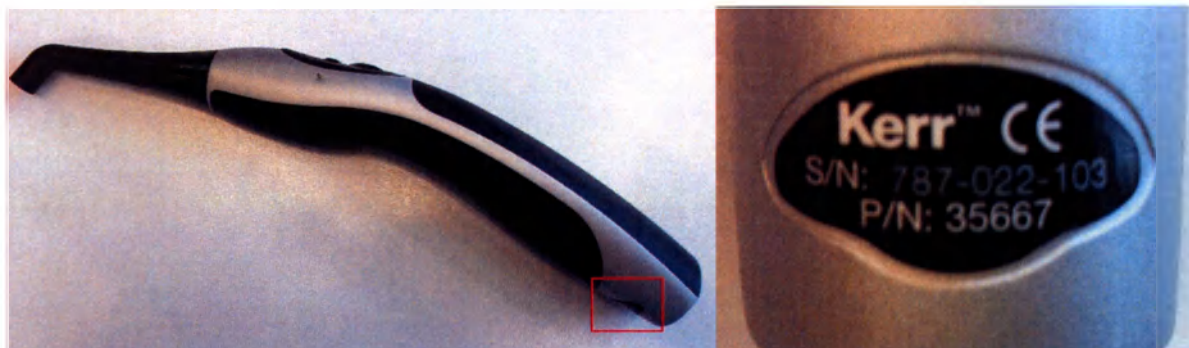An identification label, listing the manufacturer and the part number and serial number, is provided on the back of the Demi Ultra handpiece. A typical sample label is shown below.

Charging dock with LED Radiometer

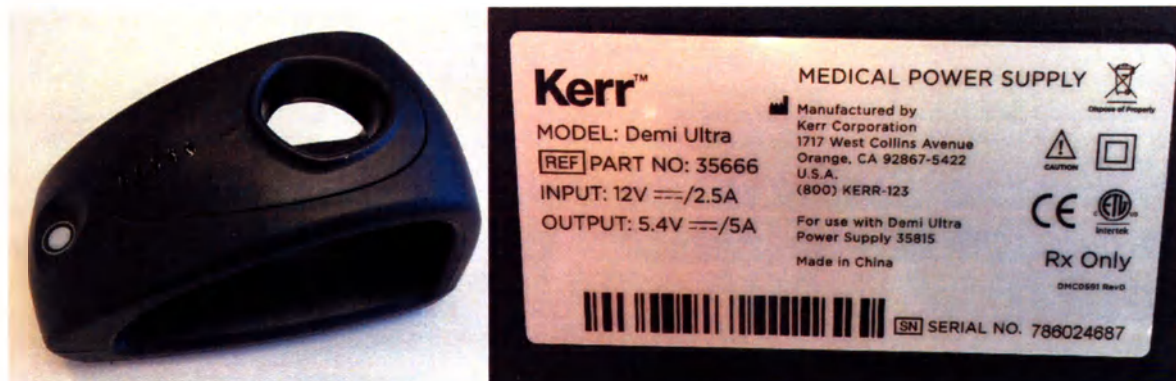Label of medical power supply is provided on the bottom part. This shield is shown the manufacturer, serial and part numbers, input and output voltage and different signs, which meaning you can find below. A typical sample label is shown below.


Power supply with Adapter set

The are 2 labels of AC adapter is provided on the bottom part. The first one is shown the main technical characteristics of power supply the second one is shown the manufacturer. A typical sample label is shown below.

Barrier bags Demi Ultra (packed 5 or 100 pcs)

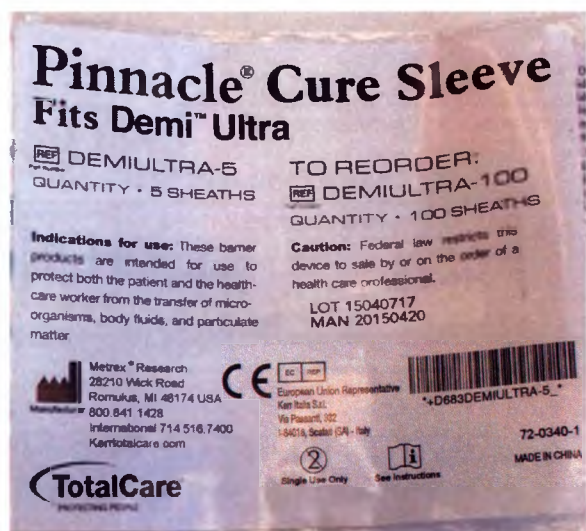Transparent zip-lock polybag with barrier bags (packed 5 pcs), box with identic barrier bags (packed 100 pcs) are equipped with sticker with short instructions for, information about manufacturer, cautions, LOT number and manufacturing date.

Hardness disk

Hardness disc's transparent zip-lock polybag is equipped with sticker with information about manufacturer and main important symbols.

Symbols are used on the device labels:

	Part number
P/N	Part number
	Serial number
S/N	Serial number
	Date of manufacture
	Name of manufacturer
	Authorized representative in the European community
	Caution
	Dispose of properly
	Recycle

	ETL Listed – Intertek Electrical Testing Laboratories (ETL) certification mark
	Consult instructions for use
	The CE mark certifies that a product has met the EU consumer safety, health, and environmental requirements
	The RoHS mark certifies that a product has met the RoHS Directive 2002/95/EC, which is restricted using specific hazardous materials found in electrical and electronic products
Rx Only	Use only with medical prescription
	Do not reuse – single use only
	Protection class II

12. Product Warranty

Kerr hereby warrants that for a period of three years from the date of purchase, this instrument shall be free from defects in material and workmanship and will perform satisfactorily under normal use and service.

To register your warranty, complete the online warranty registration within 30 days of purchase at www.kerrdental.com/warranty. If multiple units have been purchased, fill out one registration form per unit. Please have your dealer invoice available. You can also register your warranty by calling 1.800.KERR.123 (1.800.537.7123).

THE WARRANTY STATED HEREIN IS THE SOLE WARRANTY APPLICABLE TO KERR PRODUCTS. KERR EXPRESSLY DISCLAIMS ANY AND ALL OTHER WARRANTIES EXPRESSED OR IMPLIED, INCLUDING WARRANTIES OR MERCHANTABILITY OR FITNESS FOR A PARTICULAR USE. KERR'S LIABILITY WITH RESPECT TO ITS PRODUCTS IS EXPRESSLY LIMITED TO THE REMEDIES SET FORTH ABOVE. THE REMEDIES ARE THE BUYER'S EXCLUSIVE REMEDIES. KERR SHALL UNDER NO CIRCUMSTANCES BE LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES. THE WARRANTY DOES NOT APPLY TO ANY DAMAGE OR DEFECT CAUSED BY THE USER'S FAILURE TO FOLLOW THE FOLLOWING INSTRUCTIONS FOR USE OR THE OTHER USER ERROR IN OPERATING OR MAINTAINING THE INSTRUMENT. ALTHOUGH KERR OFFERS SERVICES TO REPAIR INSTRUMENTS AND PROVIDES REPLACEMENT PARTS AFTER THE EXPIRATION OF THE WARRANTY, KERR ONLY GUARANTEES THE AVAILABILITY OF SUCH SERVICES AND REPLACEMENT PARTS FOR TWO YEARS AFTER THE INSTRUMENT MODEL HAS BEEN DISCONTINUED.

13. Contact Information

Manufactured by

Kerr Corporation
1717 West Collins Avenue

Distribution facilities

Kerr Australia Pty. Ltd.
112-118 Talavera Road, Unit 10

Orange, CA 92867
U.S.A.
(800) KERR-123
kerrdental.com

North Ryde 2113
New South Wales, Australia
TEL: +61-28-870-3000
FAX: +61-29-870-7600
kerrdental.com

European Union Representative

Kerr Italia S.r.l.
Via Passanti, 332
I-84018 Scafati (SA), Italy
+39-081-850-8311
kerrdental.eu

KerrHawe SA

Via Strecce 4
CH-6934 Bioggio Switzerland
Tel: +41-91-610-5050
Fax: +41-91-610-0514
kerrdental.eu

Contact Customer Care:

Kerr Corporation
1717 West Collins Avenue
Orange, CA 92867 USA

USA Phone	(714) 516-7400
USA FAX	(714) 516-7635
International Fax	(714) 516-7581
Kerr Italy	+39-81-850-8311
KerrHawe Switzerland	+41-91-610-05-05
Kerr Australia	+61-28-870-3000

Representative in Russia:

KaVo Dental Russland LLC
Legal address: 190005, Russia, Saint Petersburg, nab. reki Fontanki, 130A
De facto address: 195112, Russia, Saint Petersburg, Malookhtinsky ave, 64B
Tel. 7(812)331-86-96, Fax 7(812)251-06-55

Direct one's claims to:
E-mail: kavo@kavodental.ru
www.kavodental.ru

14. Statement of Relation Among the Facilities

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Contract Manufacturer:

Plexus (Xiamen) Co LTD., No 6 Xiangxing 2 Road, Modern Logistics Zone, Xiamen, 361006, China
Metrex Research LLC, 28210 Wick Road, Romulus, Michigan 48174, USA
Suzhou Pac-Dent Technology Co.,Ltd, Xiangcheng District Yuanhe Technology Zone, Suchou City, Jiangsu

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