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CERTIFICATE

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China Council for the Promotion of International Trade is China Chamber of International Commerce

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THIS IS TO CERTIFY THAT: the seal of HUIAN XANGDE MEDICAL PRODUCTS CO., LTD. on the annexed document is genuine.

China Council for the Promotion of International Trade

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YANG JIN JIN

Authorized Signature:

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(Date: JAN. 12, 2017)

PEDIATRIC URINE-COLLECTING BAG

Therapeutic Indications:

Designed for urine collecting from infants and young children

A cross functional device designed for male and female children,

single-use

SERVICE MANUAL OF PEDIATRIC URINE-COLLECTING BAG

release pediatric urine-collecting bag from sterile packaging

remove a protective paper band from fixing sticking layer

fix pediatric urine-collecting bag at the (for a boy place

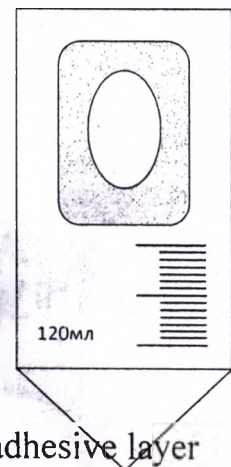
his scrotum (ball sack) inside the urine-collecting bag) making sure

that the adhesive layer has no folds

if necessary, fix pediatric urine-collecting bag at a diaper by its rear side adhesive layer

after urine has been collected unstick the urine-collecting bag and pour

urine over a container

NOTICE! Don't leave the pediatric urine-collecting bag at a child's body for more

than 1 hour.

In case urine was not gathered within 1 hour, it is necessary to remove the pediatric

urine-collecting bag and use a new one.

Measures Of Precaution

don't use in case of sterile packaging integrity loss

don't remove pediatric urine-collecting bag by twitching, by means of any mechanical tools

or chemical solvents

ions and possible side effects:

indications in case urine-collecting bag is used correctly.

recommended for persons suffering hypersensitivity to rubber latex, silicone and
VinylChloride.

pecification

urine-collecting bag is provided with a reverse valve preventing urine drainback

urine-collecting bag is furnished with a special hypoallergenic sticking fixing member

capacity size of 120 ml

possesses sidemount graduated marking from 10 to 120 ml

PEDIATRIC URINE-COLLECTING BAG

graduating division makes 10 ml

product weight without package is 2.61 ± 0.2 g; weight of the product in the package 2.07 ± 0.2 g.

individual sterile packaging

ETO sterilized

made of transparent polyethylene

Main active substances of adhesive material CP15-053926-SZ: caoutchouc, pine pitch.

Material for the waterproof fixing layer: cotton paper.

Storage conditions and transportation:

avoid direct sunlight, do not store at extreme temperatures

store at temperatures from -30 to $+45^{\circ}\text{C}$

transportation must be carried out with use of any roofed transport, protecting the products from climatic influences, in accordance with the rules of goods carriage, which are valid for this type of transport at the temperature from minus 50 to plus 50°C (relative humidity is 80% at 25°C).

during transportation at temperatures below zero the goods should be unpacked after keeping them at a storage temperature for 1 day after the date of receipt.

shelf life makes 5 years from the date of manufacture specified on the packaging

Operating conditions:

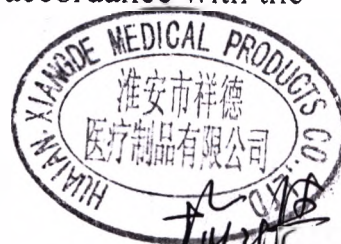
temperature during operation is +10 to +35°C.

humidity: the annual average value of 60% at 20°C, the upper value of 80% at

DISPOSAL: Don't throw urine collecting bag into sewage piping.

After use urine collectors should be collected, temporary stored, moved away and disposed in accordance with the scheme of medical waste disposal adopted in organizations conducting medical activities.

The processing method and the method of disposal shall be in accordance with the requirements of SanPiN 2.1.7.2790-10 for waste of A class.

PEDIATRIC URINE-COLLECTING BAG

REF. catalogue number

Single use only

MFD manufacture date

See User Guide

EXP Expiry date

ETO sterilized

LOT Lot number

Reg.No

Manufactured:

By LLC "Paritet" order

RF, RT, Kazan

IM15

1023

Huaian Xiangde Medical Products Co.,
Ltd, People's Republic of China, 223300,
the city of Huaian, Huanin district,
industrial park, Wayu str., 11
18 F, build. No 1. No 487, Yanmutsi str.,
Ninbo, China

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