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Registration Certificate for medical device of the Federal Agency for Supervision of Health Care Service
№ ФСР 2009/06316

CONTENTS

Part I. Technical Passport

1 Safety Rules	5
2 Purpose	11
3 Technical Parameters	12
4 Package Contents	24
5 Elements of the device	25
6 Sequences Using the Device	34
7 Using Menu	36
8 Maintenance	64
9 Battery Replacement Procedure	66
10 Troubleshooting	67
11 Manufacturer Warranty	72

Part II. Application instruction

1 Conducting treatment procedures	78
Annex A. Accessories	105
Annex B. Zone Atlas for Using Programs Menu	111
Annex C. Zone Atlas for Using Frequency Menu	145
Certificate of Guarantee	225
Proof of Acceptance	228

EN

PART I

Technical passport

The device contains fragile elements. Keep it from blows.

7

Keep the device away from continuous direct sunlight at high ambient temperature (above 25°C). Keep the device away from heating appliances.


The device is not waterproof. Keep it away from humidity.

Specialists qualified by the manufacturer only should perform any repair works of the devices. Any partial or full dismantling of the device, as well as any modification in the device, is prohibited.

EN

8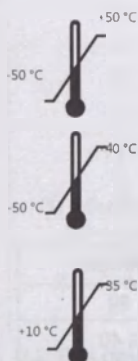

Transport Conditions: temperature of 50°C below zero to 50°C above zero, and atmospheric pressure of 70 to 106 kPa (525-to795 mm Hg).

Storage Conditions: temperature of 50°C below zero to 40°C above zero, and atmospheric pressure of 70 to 106 kPa (525-to795 mm Hg).

Operating Conditions: temperature of 10°C above zero to 35°C above zero, and atmospheric pressure of 70 to 106 kPa (525-to795 mm Hg).

Attention! If the device has been stored at the ambient air temperature below +1°C, leave it at a temperature of application environment for a minimum of two hours before using.

EN

Relative air humidity of 30 to 93 %.

9

Waste Disposal: packaging materials do not have adverse impact on the environment; they could be recycled.


The device and batteries should be disposed of according to the established disposal rules for electronic devices.

The device contains valuable materials which could be used after recycling subject to environment protection requirements. They should be taken to specialized waste collection and disposal centers (consult the relevant agencies at your region).


10 It is technically feasible to dispose of the device. The device does not pose any hazard to life or health of people or to environment after the expiry of its service (product) life and it does not require any special measures to prepare or send its components for disposing.

EN

Handwritten signature or mark.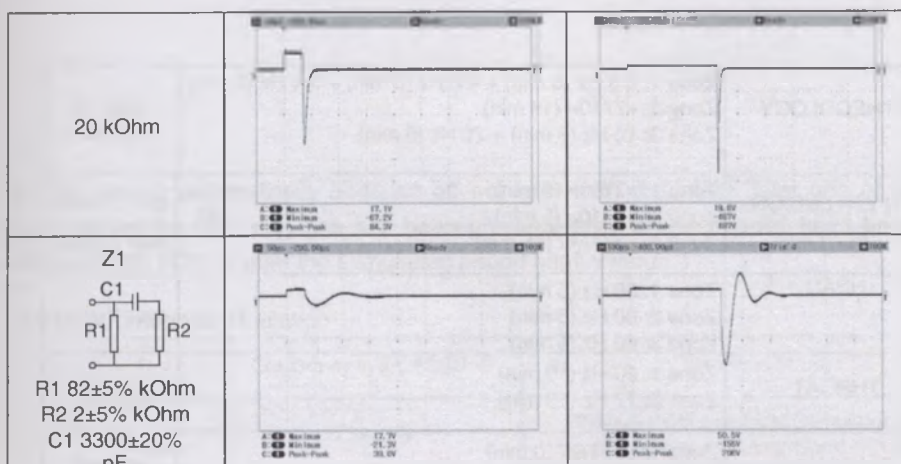

EN

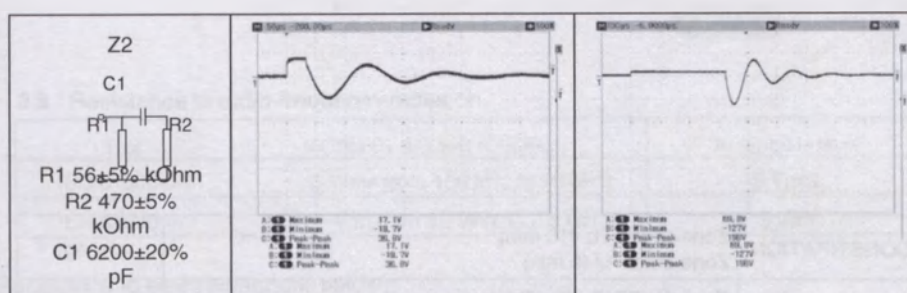3.2 The device allows setting the following impulse frequencies, Hz:

3.2.1 Range 1: 10±2, 20±2, 60±2, 77±3, 125±4, 140±5, 200±5.

3.2.2 Range 2: 1.0 to 9.9 with pitch of 0.1±0.05.

3.2.3 Modes: «7710» (alternation of 77±3 Hz and 10±2 Hz, 2±0.1 Hz frequency modulated), «7720» (alternation of 77±3 Hz within 2.0±1.0 sec and 20±2 Hz within 3.0±1.5 sec), «77AM» (amplitude modulated signal 77±3 Hz with 2±0.1 Hz modulation), «MED» (10±2 Hz), «SCREENING» (10±2 Hz).

3.3 Special stimulation programs.

Program name	Application zone / frequency / duration	Age
ALLERGY	Zone 1: «7710» (5 min) + 3.8 Hz (5 min) Zone 2: 77 Hz (5 min) + 1.7 Hz (5 min) + 3.8 Hz (5 min)	over 7 years
PAIN	Zone 1: Impulses with quasirandom repetition frequency charge within the range of 75 to 135 Hz (15 min)	over 1 year
INTENSE PAIN	Zone 1: Impulse packets with repetition frequency of 77 Hz and variable duration and carrier frequency: from 2.5 msec and 27 kHz at minimum power level up to 1.0 msec and 2.8 kHz at maximum power level (15 min)	over 1 year
BLOATING	Zone 1: «7710» (5 min) + 125 Hz (5 min)	any age
DYSTONIA	Zone 1: «7710» (10 min) + 10 Hz (5 min) + 2,5 Hz (5 min)	over 4 years
GYNECOLOGICAL PAIN	Zone 1: «7710» (5 min) + 3.8 Hz (5 min) + 4.0 Hz (5 min)	over 13 years

EN

Obryt

3.7 The device automatically switches off not later than 5 minutes after one of its keys (except for  key) has last been pressed, after the electrodes have been released from skin, or after the stimulation period has finished.

3.8 Electromagnetic Radiation

Test	Conformity to IEC 60601-1-2	Conditions of Use
Radio-frequency radiation CISPR 11	Class B	DENAS-PCM Electrical Stimulator can be used in all types of institutions and at home as well
	Group 1	DENAS-PCM device emits very little radio-frequency radiation that cannot lead to failure of electronic equipment located nearby

3.9 Resistance to radio-frequency radiation.

Test	IEC 60601-1-2 test conditions	Acceptable level
IEC 61000-4-6	3 Vrms from 150 kHz to 80 MHz	3 Vrms
IEC 61000-4-3	3 V/m from 80 MHz to 2,5 GHz	3 V/m


3.10 Resistance to electromagnetic fields.

Magnetic fields	Test level	Conformity level	Conditions of Use
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact; ±8 kV air	±4 kV contact; ±8 kV air	The floor should be made of wood, concrete or ceramic tiles. If the floor is covered with synthetic material, the relative air humidity should be not less than 40%.
Magnetic fields IEC 61000-4-8	3 A/m	3 A/m	Magnetic field parameters should be within the normal values for commercial buildings and healthcare facilities.

5 ELEMENTS OF THE DEVICE

25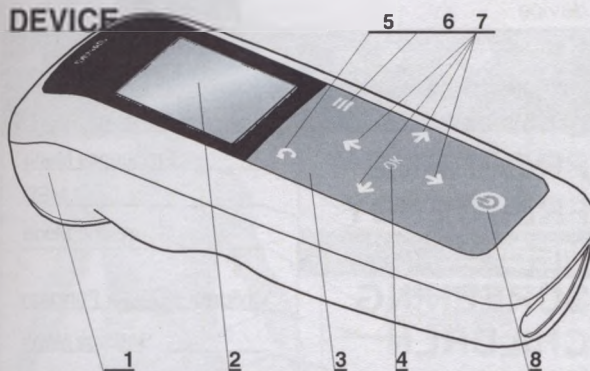


1. Body.
2. Display — LCD indicator.
3. Keyboard.
4. Select Menu Key.
5. Previous Menu Key.
6. Reset Key.
7. Navigation keys for the device menu.
8. On/Off Key.

Figure 1 — DENAS-PCM device (top view)

EN

26

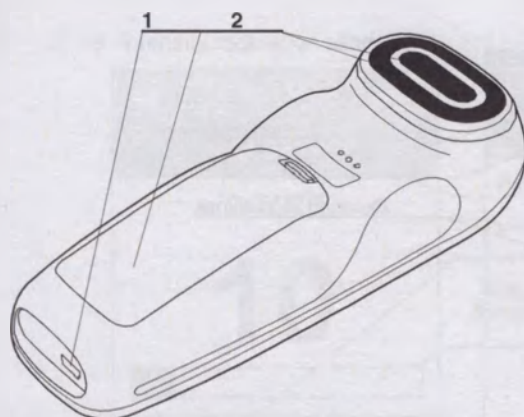1. Connector for external electrodes (closed with protective cap).
2. Battery case lid.
3. In-built electrode.

EN

Figure 2 — DENAS-PCM device (bottom view)

27

The device can be connected to external therapeutic electrodes of manufacturer's model range (see Annex A).

Attention! External therapeutic electrodes can only be used for Frequency and Programs modes.


In order to improve the contact you can moisture skin with water or with Malavtilin cream, or with Baby-DENS cream for children before applying the electrodes.

EN

Handwritten signature

5.1.4 Stimulation window


31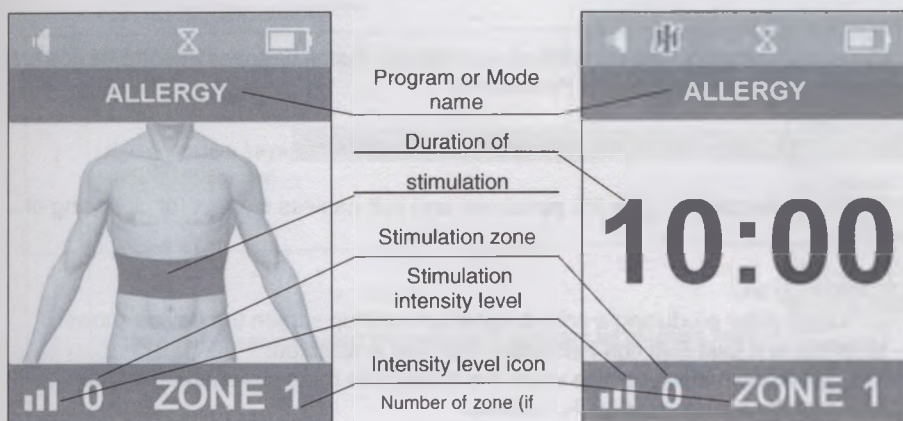

EN

32


5.1.5 Intensity regulation window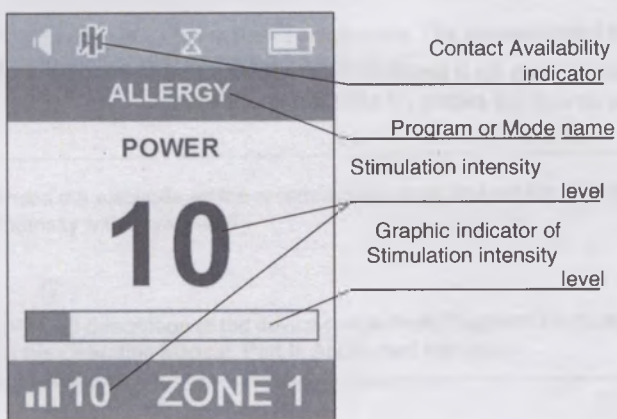

EN

5.1.6 MED and SCREENING Modes window



33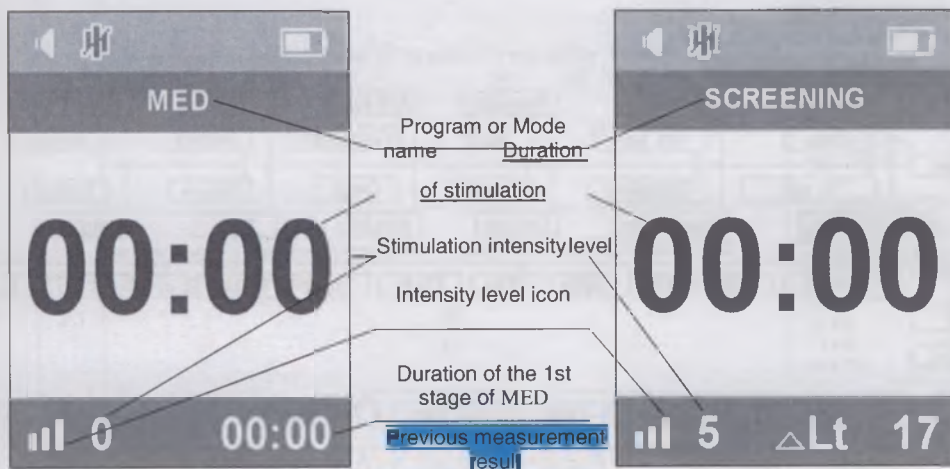

EN

Handwritten signature

7.2. Menu item selection.

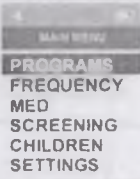
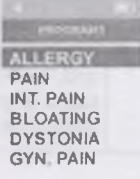
Programs Menu		
1	Use navigation keys  and  to select "Programs" in Main Menu and press  to confirm.	
1.1	In the opened section select the necessary program by pressing keys  and  . Press	

EN

38

1.1.1	The device will switch to the selected mode. The recommended zone for application will be shown on the display.	
1.1.2	Place the electrode on the recommended zone and set the stimulation intensity with keys  and  .  Detailed description of the device operation in Programs Menu is given in the Operating Manual. Part II. Application instruction.	



Frequency Menu

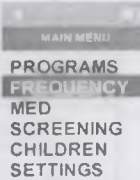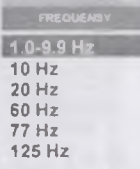2	Use navigation keys  and  to select "Frequency" in Main Menu and press  to confirm.	
2.1	In the opened section select the necessary frequency by pressing keys  and  . Press 	



Handwritten signature

4.1.1	If you make a repeated measurement, the previous ΔLT value will be shown in the lower right corner. Detailed description of the device operation in SCREENING mode is given in the Operating Manual. Part II. Application instruction.	
-------	--	--

EN

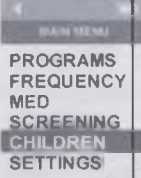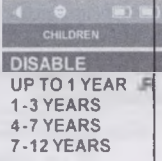Children's Doctor Function		
5	Use navigation keys and to select "Children's Doctor" section in Main Menu and press to confirm.	
5.1	The device has a function of limited intensity for sessions with children. Use keys and to set the child's age and press to confirm. If you press keys or instead of key or if you power off the device, the "Children's Doctor" settings will return to the earlier saved values. If the "Children's Doctor" function is on, symbol will be shown in the status line.	

EN

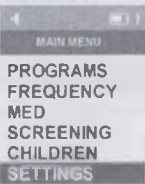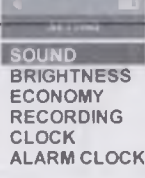Sound Menu (Sound settings)		
6	Use navigation keys and to select "Settings" in Main Menu or in Stimulation Window and press to confirm. The number of items in Settings Menu depends on the current status of the device.	
6.1	Use keys and to select "Sound" in Settings Menu and press to confirm.	

EN

Handwritten signature

6.2.1	The device has brightness adjustment option within the range of 5 to 100 with a pitch of 5. Use keys and to select the comfortable brightness level and press to confirm.	
	If you press keys or instead of key or if you power off the device, the Brightness settings will return to the earlier saved values. Attention! If you use increased brightness it may lead to increased current consumption and more frequent change of batteries. Set the minimum brightness to use the installed batteries for longer time.	

EN

"Economy" Menu (Economical use of batteries)		
6	Use navigation keys and to select "Settings" in Main Menu and press to confirm.	
6.3	Use keys and to select "Economy" in Settings Menu and press to confirm	

EN

6.3.1	The device will switch to "Economy" Menu, which allows changing the "Economy" function: activating lower energy consumption by reducing backlit duration (without changing the brightness level). Use keys and to set this function on and press to confirm. If you press keys or instead of key or if you power off the device, the "Economy" mode settings will return to the earlier saved values.	
"Last setting record" Menu (Recording of last settings)		
6	Use navigation keys and to select the "Settings" in Main Menu and press to confirm.	

EN

Handwritten signature

6.6	Use keys and to select "Alarm Clock" in Settings Menu and press to confirm.	
6.6.1	The device will switch to Alarm Clock Menu where you can set the frequency and time of alarm clock signal. In the menu, use keys and to select one of the items to be changed and press to confirm.	

EN

6.6.1.1	In order to set the alarm clock frequency use keys and to select one of the modes: off, one time or every day. Press to save the selected mode. If you press keys or instead of key if you power off the device the current alarm clock settings will not be changed. If the alarm clock is on, symbol will be displayed in the status line. In order to stop the alarm clock when it goes off press key one time.	
6.6.2.1	In order to set the alarm clock time, use keys and to set the cursor on the value to be changed, and use keys and to change the values accordingly. Press key to confirm. Turn the device off. If you press keys or instead of key or if you power off the device the current alarm clock settings will not be changed.	

EN

"Language" Menu		
6	Use navigation keys and to select "Settings" in Main Menu and press to confirm.	
6.7	Use keys and to select "Language" in Settings Menu and press to confirm.	

EN

Handwritten signature

7.1	The device will switch to "Contact" Menu. Use keys and switch the function on and press confirm. If you press key instead of key in order to close Therapy without Contact Control Function, its settings will not be changed.	
	Contact status indication: - indication is on without interruption – electrodes contact the skin, Therapy without Contact Control Function is off; - electrodes do not contact the skin; it is recommended to slightly moisture the skin for better contact or to switch Therapy without Contact Control Function on; - indication flashes on and off – Therapy without Contact Control Function is on.	

EN

62

"Timer" Menu (function is available in Frequency mode only)		
8	Timer Function is necessary when it is not possible to control the stimulation duration (in case of electrical stimulation of hard-to-reach zones) or when the stimulation duration is strictly specified. When this function is on, the stimulation duration will be counted in the form of a countdown timer. When this function is off, the maximum session duration is 30 minutes. Timer function settings are available only in Frequency Menu. The duration in Programs, MED, and Screening Menus is fixed. Use navigation keys and to select "Timer" and press to confirm.	

EN

8.1	The device will switch to Timer Menu. Use keys and to set the desired timer value from 1 to 30 minutes (with pitch of 1 minute) and press to confirm. If you press key instead of key in order to close the Timer, its settings will not be changed. If timer is set on 0 minutes, this function will be off and there will be direct timing of sessions in stimulation modes. If Timer function is on, symbol will be shown in the status line.	
-----	---	--

EN

Handwritten signature

8 MAINTENANCE

8.1 Daily maintenance includes the following operations:

- visual inspection of the device;
- disinfection of the electrode.

Use 3% penozone and soft napless diapers for cleansing of electrodes.

8.2 Operability of the device is checked according to recommendations stipulated in Section 6.

8.3 If you are supposedly not going to use the device for a long time, remove the set of batteries from the battery case (see Section 9).

8.4 If the battery symbol flashes and the device generates intermittent alarm sound it is necessary to replace the set of batteries of the device (see Section 9).

EN

Attention! If the batteries are removed from the device for more than 5 minutes, User's individual settings and any set reminders will clear off and information about current time will be lost as well.


EN

9 BATTERY REPLACEMENT PROCEDURE

Replacement of the set* of batteries:

- open the battery case (1) and remove the batteries;
- install new batteries** observing the polarity (2);
- close the battery case (3).

Attention! Observe the polarity while installing batteries.

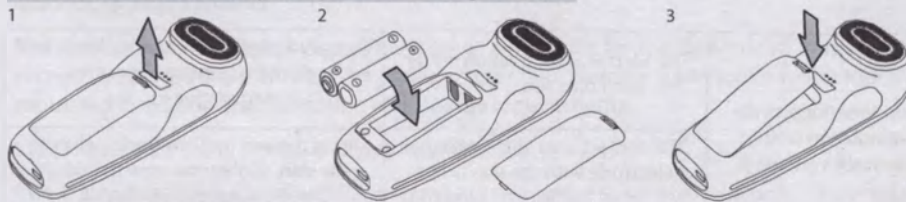* The set consists of two batteries.

** Install only those batteries which are designed for this type of the device: LR6/AA type with nominal voltage of 1.5 V or relevant storage batteries with nominal voltage of 1.2 V.

EN

Handwritten signature

8) The device switches off or battery symbol flashes and the device produces intermittent alarm sound during the operation.	Voltage of the set of batteries is less than 2.1 V.	Replace the set of batteries (see Section 9, Battery Replacement Procedure).
9) The device does not switch on in time set for Alarm Clock.	Alarm Clock is Off.	Switch on the Alarm Clock. Go to Settings --- Alarm Clock --- Off and select One time or Daily mode.
	Voltage of the set of batteries is less than 2.1 V.	Replace the set of batteries (see Section 9, Battery Replacement Procedure).
10) The device switches off without command.	Voltage of the set of batteries is less than 2.1 V.	Replace the set of batteries (see Section 9, Battery Replacement Procedure).

11) Fast consumption of batteries.	Low-quality batteries.	Use high-quality batteries (it is recommended to use alkaline batteries) or storage batteries of relevant typical size with voltage not less than 1.5 V.
	Maximal backlit brightness level.	Reduce backlit brightness. Go to Settings --- Brightness.
	Economical Use of Batteries Function is disabled.	Enable Economical Use of Batteries Function. Go to Settings --- Economy.
12) The device does not provide intensity more than 5 units.	Children's Doctor Function is enabled.	Disable Children's Doctor Function (see item 7.2.5 Children's Doctor Function).
13) The Programs Menu lacks some programs.	Children's Doctor Function is enabled.	Disable Children's Doctor Function (see item 7.2.5 Children's Doctor Function).

Attention! Any other defects should be remedied by the company-manufacturer.

11 MANUFACTURER WARRANTY

11.1 The manufacturer guarantees that the device complies with Technical Regulation TY 9444-009-44148620-2009 on condition that it is used subject to operation, transportation and storage requirements.

11.2 Product service life is 5 years. The product service life for the designated purpose can exceed the period stipulated by the manufacturer considerably if consumer follows all the stipulated operation, transportation and storage requirements.

11.3 Product warranty period is 12 months after sale.

11.4 If any defects are found within the warranty period the seller (manufacturer) undertakes to satisfy the consumer's claim subject to Law of the Russian Federation "On Protection of Consumers' Rights".

The seller (manufacturer) or any organization performing functions of the seller (manufacturer) under the relevant contract will not be responsible for any defects if they emerge after the product has been delivered to consumer and relate to:

1 CONDUCTING TREATMENT PROCEDURES

1.1 General Information.

Stimulation of reflexogenic zones and points with impulse current for prevention, treatment and recovery of body functions is an efficient method of health improvement. Transcutaneous dynamic electrical nerve stimulation, which can be applied both together with other treatment methods and as a separate health improvement method, promotes positive dynamics of treating many acute and chronic diseases. Using of the devices enhances recovery process, reduces pharmacological load on the body and promotes fast health improvement. Clinical research authentically established that transcutaneous dynamic electrical nerve stimulation is efficient for treating patients with cardiovascular disorders, locomotor system diseases, injuries, gynecological and urological diseases. It showed efficiency of treatment patients after surgeries, or with ophthalmologic problems, bronchopulmonary and gastrointestinal diseases. We have an extensive experience of using DENS for a wide range of neurological diseases. A series of studies evidences indisputable advisability of dynamic electrical nerve stimulation as an effective component of combination treatment and rehabilitation programs in pediatrics.

EN

71

1.2 Indications and contraindications to application.

The device can be used by patients of any age starting the newly born children. Subject to treatment purposes, the device can be used for analgesic, spasmolytic, vascular, anti-inflammatory, trophic, and general regulatory effects.

1.2.1 Indications:

- acute and chronic pain syndromes;
- injuries;
- diseases of respiratory and gastrointestinal (GIT) organs, ENT, and cardiovascular, musculoskeletal, nerve, endocrine, and genito-urinary systems, and eyes and skin;

EN

72

- rehabilitation after diseases, surgical interventions, and injuries;
- enhancement of adaptive capacities of the body against pressures of negative pathogen factors, intensive physical and mental activities, physical and mental exhaustion, and chronic fatigue syndrome; when it is hard for the patient to wake up in the morning and he/she feels drowsiness during the day, or he/she has disorder of falling asleep at night or insomnia; when there is hyperirritability, depression, or sexual function disorder, and for prevention of cold-related diseases.

1.2.2 Contraindications to application

Absolute:

- idiosyncrasy of electric current;
- implanted heartpacemaker.

Average duration of one session according to patient's age:

- for children under 1 year old — 5–10 minutes (1–3 minutes per one zone);
- for 1–3 years old children — 10–15 minutes (3–5 minutes per one zone);
- for 3–5 years old children — 15–20 minutes (5–7 minutes per one zone);
- for 5–12 years old children — 20–25 minutes (7–9 minutes per one zone);
- for children above 12 years old and adults — up to 30–40 minutes (10–15 minutes per one zone).

1.4.3 Stimulation Intensity.

Intensity of electrical stimulation is determined on individual basis according to the patient's subjective feelings. Stimulation Intensity can be divided into three conventional levels.

1.4.3.1 Minimal level — there is no sensation or there is sensation of light vibration under the electrodes.

Select minimal level if stimulation should have low intensity: for children under 12 years old and adults with cardiovascular diseases (arterial hypertension, arterial hypotension, vegetal dystonia). Those patients who suffer from frequent headaches or giddiness should also start treatment setting the minimal intensity level. Specifically for children the device has Children's Doctor Function which provides minimal intensity level: 1 to 5 units.

1.4.3.2 Comfort level — sensations are above the sensitivity threshold. There is painless vibration, prickly sensation or light burning sensation under the electrodes. This intensity level is used more often.

1.4.3.3 Maximal level — sensations are on the pain threshold. There is painful burning or prickly sensation under the electrode. Muscles of the stimulated zone may jerk. It can be applied for treatment of pain syndromes with intensive pain.

Attention! Electrical stimulation intensity level may be increased or decreased as patient's individual sensitivity changes and his/her pain reduces at different treatment stages.

Attention! Control the intensity level subjectively, according to patient's sensations. It is not advisable to exceed the tolerance threshold.

Attention! Patients suffering from arterial hypertension, arterial hypotension, vegetal dystonia syndrome, frequent headaches or giddiness should have treatment at minimal intensity level.

1.4.4 Stimulation Modes.

The device menu contains the following items designed for treatment: Programs, Frequency, MED, and Screening.

Handwritten signature

HYPOTONIA	Neck and collar area				+	+
HEAD	2nd cervical vertebra area					
	Neck and collar area				+	+
	Hands, head area according to Su Jok					
THROAT	Hands, neck area according to Su Jok			+	+	+
	Submaxillary area					
GYNECOLOGICAL PAIN	Painful abdominal area					+
GYNECOLOGY	Soles					
	Neck and collar area					+
	Lumbosacral area					
DIARRHEA	Anterior abdominal wall	+	+	+	+	+
	Lumbosacral area					
ABDOMINAL PAIN	Painful abdominal area			+	+	+

CONSTIPATION	Anterior abdominal wall	+	+	+	+	+
	Lumbosacral area					
COUGH	Submaxillary area		+	+	+	+
	Suprasternal notch					
MUSCLES	Injured or painful area		+	+	+	+
COLD	Hands, nose area according to Su Jok	+	+	+	+	+
	Nose area and sinuses projections					
POTENCY	Lumbosacral area					
	Neck and collar area					+
	Soles area					
KIDNEYS	Lumbosacral area				+	+
	Soles area					


BACK	Spine projection, painful area			+	+	+
JOINTS	Painful joint				+	+
	Segmental area					
NAUSEA	Upper abdomen, right and left hypochondriums			+	+	+
TRAUMA	Injured part of the body		+	+	+	+
ASPHYXIA	Suprasternal notch	+	+	+	+	+
	7th cervical vertebra area					


Handwritten signature

60 and 77	Frequencies of 60 Hz and 77 Hz are universal and can be widely used for correction of internal organs disorders when there are expressed local complaints. They can be recommended for pain relief, improvement of local and regional blood flow, treatment of inflammations, etc. in acute and rehabilitation period.
125 and 140	Frequencies of 125 Hz and 140 Hz are recommended for treatment of locomotor system diseases and neuralgic pain accompanied with expressed pain syndrome.
200	Frequency of 200 Hz is recommended for quick pain relief when there are locomotor system diseases, neuralgic pain, or injuries.
7710	7710 Frequency is recommended for people with symptoms of arterial hypertension, vegetative dystonia syndrome of hypertensive type, and patients with sleep disorders.
77AM	77AM Frequency can be recommended when there are symptoms of hypotonic or atonic state of internal organs.
7720	7720 Frequency is recommended for vegetative support normalization and reducing of puffiness.

1.4.4.3 MED

Minimal Effective Dose (MED) is an automatic program which can be used for rehabilitation and prevention when there is physical stress, psychic and emotional overstrain, chronic fatigue syndrome or internal organs diseases, including for prevention of cold-related diseases in epidemic season. It is used minimum once a day for a course of 8–12 days.

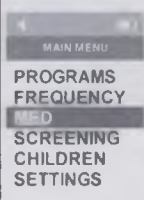
Sequences Using the Menu	
Use keys and to select MED in Main Menu and press   	

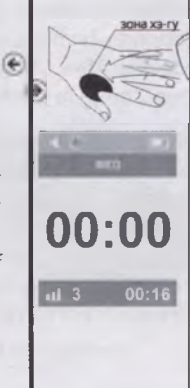Apply the electrode on He-Gu zone. Set the stimulation intensity by using and keys. It is recommended to set minimal or comfort intensity level. Attention! Stimulation when using MED Program should be performed with in-build electrode only. Attention! When the device is operated in MED mode, electrode should be kept stable on the patient's skin i.e. it is not allowed to move the electrode of the device during its operation. Attention! Control the intensity level according to patient's subjective sensations of electrode contacting the skin. It is not advisable to exceed the pain threshold. MED Program consists of two phases. The device provides a sound signal at the end of each phase.	
--	---

Handwritten signature

ANNEX A

Accessories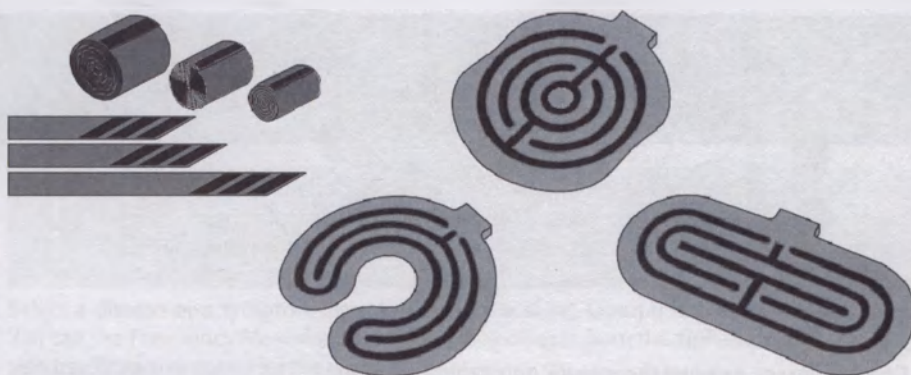


External zonal therapeutic electrodes*

Designed for therapeutic non-invasive treatment of painful areas, lesion focuses, and stimulation of reflexogenic zones (where skin is not damaged).

* External zonal therapeutic electrodes should be fixed in the stimulation zone, current-conducting strips to the skin.

Handwritten signature

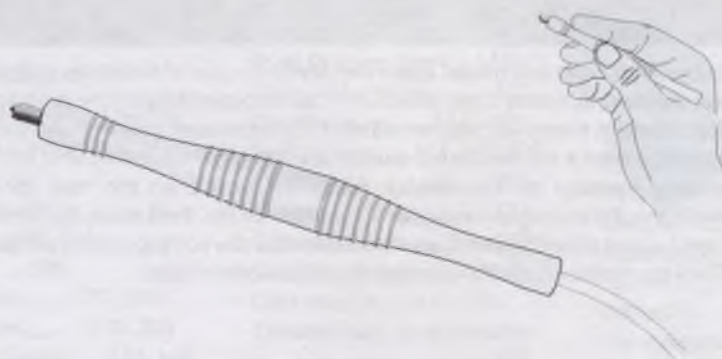External therapeutic point electrode.
Designed for stimulation of biologically active points.

ANNEX B

Zone Atlas for Using Programs Menu

Attention! This Manual contains zones and modes of the common illnesses and diseases. You can find additional information about treatment and prevention capacities of dynamic electrical nerve stimulation devices in Practical Guidance for Electrical Nerve Stimulation.

Select a disease or a symptom from the alphabetical list. Open the relevant Atlas chapter. You can use Frequency Menu for treatment of any disease from the alphabetical list. You can also use Programs menu for the most of diseases.

For treatment by using Frequency mode, apply the device on one of the zones indicated in the Atlas (there can be several stimulation zones depending on disease). Select time and intensity of stimulation. After finishing stimulating one selected zone, start treatment of the next recommended zone.

112 Alphabetical List of Diseases

Abdominal pain.....132, 158	Bone fracture (pain)...142	Diarrhea, loose stool.....131
Acute respiratory diseases.....130, 148	Bronchial asthma.....152	Diskogenic radiculopathy
Algodismenorrhea (Gynecological	Bronchitis.....134, 150	(pain).....122, 139
pain).....125, 182	Burns (pain).....121, 142, 172	Disorder of micturition.....174
Allergy.....120, 186	Cardialgia.....200	Dystonia.....123
Angina, tonsillitis 130, 154	Caugh.....134	Earache.....121, 156
Anxiety, stress198	Cataract.....210	Eczema.....186
Arterial hypertension.....127, 202	Cellulite, obesity.....190	Edema.....206
Arterial hypotension.....128, 204	Cold, rhinitis....136, 148	Esthetics.....188, 190
Arthritis, arthrosis (joints).....140, 166	Constipation, stool retention	Eye overstrain.....210
Asphyxia.....143, 152	133	Face pain.....121, 194
Astigmatism.....210	Coronary heart disease..200	Facial care.....188
Blind headache.....129, 198	Cystitis (pain)121, 220	Fever.....218
Bloating.....124, 158	Dermatitis.....186	Frequent and
	Diabetic angiopathy.....208	painful urination.....221

EN

Alphabetical List of Diseases 113

Frontal sinusitis148	Joint dislocations.....170	Menstrual disorder (gynecology)..126
Giddiness.....192	Joint traumas (pain)122	Myocardiodystrophy.....200
Glaucoma.....210	Hypogalactia.....126	Nasal stuffiness.....136, 148
Headache, vasoneurosis...123, 192	Laryngitis154	Nausea.....141
Hearing loss, sonitus.....156	Leg pain.....121, 220	Neck pain
Heartache.....200	Logoneurosis, stammering	and backache.....139, 220
Hemorrhoids.....164	196	Neurodermatitis.....186
Hypertension.....127, 202	Long sight.....210	Neurosis, stress, vasoneurosis...123
Infertility (gynecology) ...126,137	Loss of consciousness 216	Osteoarthritis (joints)140, 166
Insomnia.....198	Lumbar pain.....139, 220	Pain in the joints.....121, 140
Ischias, lumbodynia.....122	Lumbodynia (pain).....122	Pharyngitis148
	Mastitis.....180	
	Maxillary sinusitis.....148, 222	

EN

114 Alphabetical List of Diseases

Plugged duct.....180	Soft tissue bruise142	Trauma of soft tissues (trauma)
Potency disorder.....137	Sore throat.....130, 148	142
Prostatitis.....137, 184	Stammering.....196	Trifacial neuralgia.....194
Pyelonephritis.....138	Stress, vasoneurosis....	Urethritis.....174, 221
Refringence disorder.....210	123, 198	Urolithiasis (pain)121, 122
Rhinitis, cold.....148	Tension of ligaments.....170	Varicose vein disease.....206
Sensorineural hearing loss.....156	Toothache.....122, 212	Wounds (trauma)142, 172
Short sight.....210	Tracheitis (throat)130, 150	

Zone 2. 7th cervical vertebra area

LS :S:

EN

Zone: Painful area. The most painful area should be stimulated with in-built or external electrodes

LS :S:

Zone: Painful area. The most painful area should be stimulated with in-built or external electrodes

LS

- intense pain


- dull nagging pain

EN

Zone 1. Feet

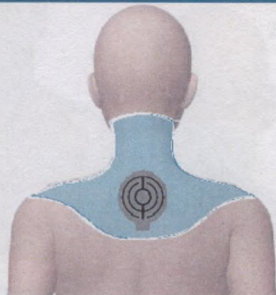Zone 2. Neck and collar area


Zone 3. Lumbosacral area


Attention! The program is not available if Children's Doctor Function is enabled.

EN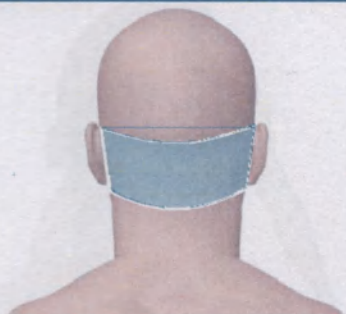

Zone 1. 2nd cervical vertebra area

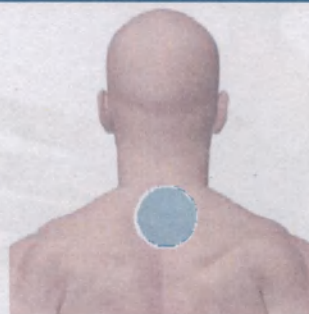Zone 2. 7th cervical vertebra area

EN

Zone: Neck and collar area


EN

ABDOMINAL PAIN Program (stomachache)

127

Zone: Painful area. Apply clockwise.


LS :S: 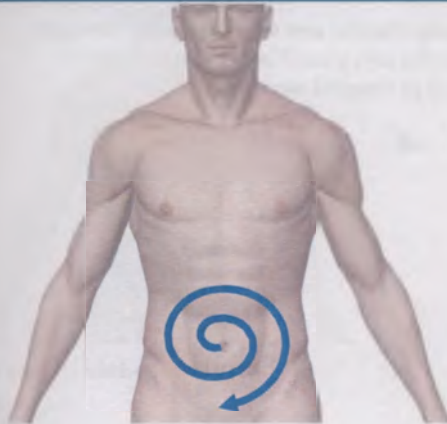

EN

128

CONSTIPATION Program

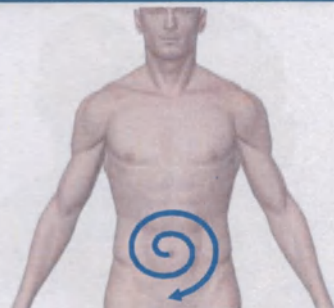Zone 1. Anterior abdominal wall.
Apply clockwise

S 

Zone 2. Lumbosacral area

S 

COUGH Program

129

Zone 1. Submaxillary area

S 

Zone 2. Suprasternal notch

S 

EN

KIDNEYS Program 133

Zone 1. Lumbosacral area

S

Zone 2. Feet

S

EN


134 BACKProgram

Neck pain — cervical spine

LS — intense pain
 — dull nagging ache

EN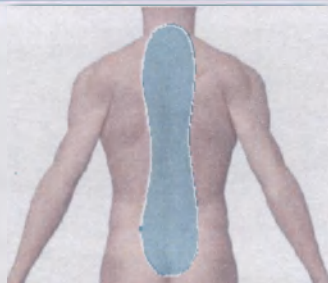

Backache — thoracic spine, lumbar spine, sacral spine

LS — intense pain
 — dull nagging ache

JOINTS Program 135

Zone 1. Painful joint

LS :S:

Zone 2. Lower back – if leg and feet joints are painful; neck and collar area – if arm and hand joints are painful

LS :S:

EN

Handwritten signature

ANNEX C

Zone Atlas for Using Frequency Menu

140

Attention! The treatment methods described in the Atlas are given for reference only and they cannot substitute a consultation with a medical specialist.


Attention! Due to predefined parameters the Programs can be used for adults and children above 12 years old.

EN

Conventional Symbols

141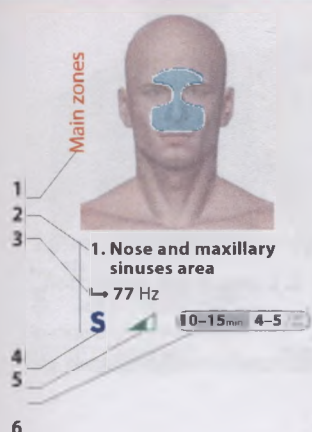

1. **Main zones** are designated for obligatory stimulation. **Additional zones** are stimulated in order to enhance the impact of treatment.
2. **Consequence of zones stimulation**
3. **Mode selection in the device menu**
 - ↳ 77 Hz
 - ↳ or other modes
4. **Treatment method** (see item 1.4.1, page 82)
 - L** Instable
 - S** Stable
 - LS** Stable/Instable
 - :S:** Stable with applicator electrode
5. **Stimulation intensity level / Patient's sensations** (see item 1.4.3, page 85)
 - Minimal / No sensations
 - Comfort / Pleasant prickly sensation.
 - Maximal / On the sensitivity threshold
6. **Duration of treatment** (see item 4.2, page 15)
 - 10-15 min, 4-5 Stimulate the zone during 10 to 15 minutes 4-5 times a day.

EN

1. Area according to Su Jok

↪ 77 Hz

S

5 min.

1-2

times a day

Note. Course duration — until full recovery

2. Area between the shoulder blades

↪ 10 Hz

:S:

5-10 min

1-2

times a day

Additional zones

EN

Bronchial asthma

Main zones

1. Suprasternal notch

↪ 77 Hz

S

5-10 min.

In case of asthma attack

2. Lungs front projection


↪ 77 Hz

LS

5-10 min.

In case of asthma attack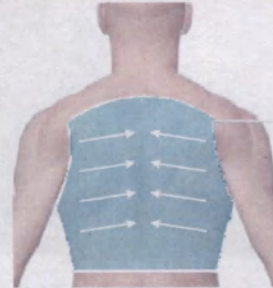

3. Lungs back projection

↪ 77 Hz

LS

5-10 min.

In case of asthma attack

EN

Bronchial asthma

1. Area according to Su Jok


↪ 10 Hz

S

5 min.

In case of asthma attack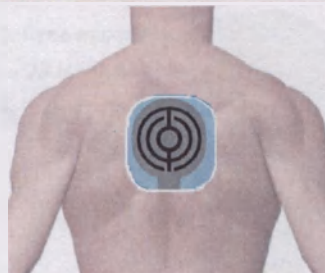

2. Area between the shoulder blades

↪ 77 Hz

:S:


5-10 min.

In case of asthma attack

Additional zones

EN

Note. If one session is not enough repeat the session. Continue stimulation until the attack stops.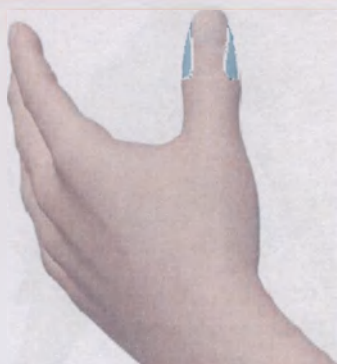

1. Area according to Su Jok

↪ 77 Hz

S  5–10 min.


In case of otitis, conduct stimulation 1-2 times a day, in case of sensorineural hearing loss — once in 2-3 days.

Additional zones

Note. Course duration in case of otitis — until full recovery. Course duration in case of sensorineural hearing loss — 10–14 days. Retreatment in 1–1.5 months.

EN

Main zones

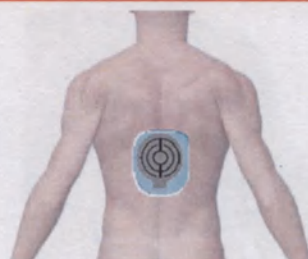1. Stomach and pancreas gland projection

↪ 77 Hz

LS  5–10 min.

Stimulation 3–4 times a day until symptoms abate.

2. Thoracic section (at stomach level)

↪ 10 Hz

:S:  10–15 min.

Stimulation 3–4 times a day until symptoms abate.

EN

1. Area according to Su Jok


↪ 77 Hz, 9,4 Hz

S  5 min. 1–2

Additional zones

Note. Course duration — until full recovery.

EN

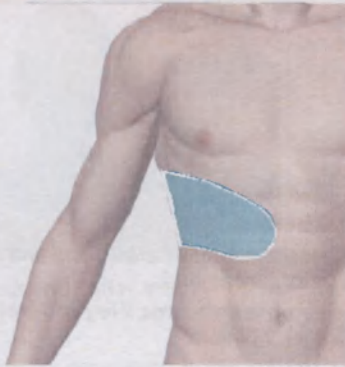1. Liver front and side projections.

↪ 77 Hz

L

5 min.

1-2 times a day

Additional zones

Note. Course duration — 10–14 days.

EN

Main zones

1. Periproctical area

↪ 77 Hz

S

5–10 times a day

EN

Several times a day as the patient's complaints increase.

2. Lumbosacral area

↪ 10 Hz

:S:

10–15 min.

Several times a day as the patient's complaints increase.

1. Area according to Su Jok

↪ 77 Hz, 9,4 Hz

S

5 min.

1–2 times a day

Additional zones

Note. Course duration — until recovery, but for a minimum of 7 days.

EN

Handwritten signature

6+

Neck pain and backache (vertebral osteochondrosis, diskogenic radiculopathy, lumbodinia, ischias)

Additional zones

1. Neck and collar area

5-15 min. 1-2

9,6 Hz — if there is pain in the neck

LS

Note. Course duration — until recovery.

2. Pain center projection

5-15 min. 1-2

9,6 Hz — if there is pain in the thoracic or lumbar spine

LS

EN

Micro-ruptures (tension) of ligaments, tendons, and muscles, joint dislocations, meniscus injuries

Main zone

1. Direct projection of injured area (5-30 min. 1-2)

77 Hz

- intense pain
- dull nagging ache

Several times a day as the patient's complaints increase.

EN

Micro-ruptures (tension) of ligaments, tendons, and muscles, joint dislocations, meniscus injuries

Additional zones

1. Neck and collar area if upper limbs are injured

10 Hz
10-15 min. 1-2

Note. Course duration — until full recovery.

2. Lumbosacral area if lower limbs are injured

10 Hz, «7710»
10-15 min. 1-2

EN

Handwritten signature

Nephrocolica, frequent and painful urination
(pyelonephritis, cystitis, urethritis)

169

1. Neck and collar area

↳ 10 Hz

:S: 10-15 min. 1-2

Note. Course duration — for a minimum of 10-12 days.

Additional zones

2. Area according to Su Jok

↳ 77 Hz

S 5 min. 1-2

EN

170

Menstrual disorder, infertility

Main zones

1. Pubic area (lower abdomen)

↳ 77 Hz

LS 5-10 min. 1

EN

2. Lumbosacral area

↳ 10 Hz

:S: 10-15 min. 1

Menstrual disorder, infertility

171

1. Area according to Su Jok


↳ 77 Hz

S 5 min. 1

Additional zones

Note. Course duration – 14 days. Repeat the course in a month.

EN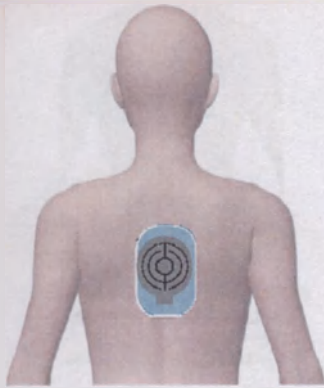

1. Thoracic section (at mammary glands level)

↳ 77 Hz, 10 Hz

S: 10-15 min

Several times a day before breast-feeding.

Additional zones


Note. Course duration — until full recovery.

EN

176

Lower abdominal pain and genitals pain
(prostatitis, urethritis, orchitis, algodismenorrhea)

Main zones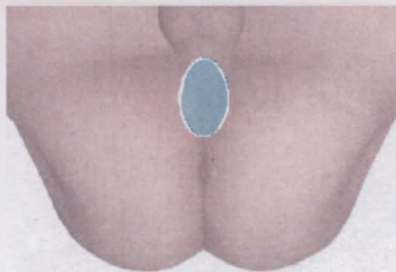


1. Perineum area

↳ 77 Hz

S: 5-10 min 3-4

3-4 times a day when there is pain.

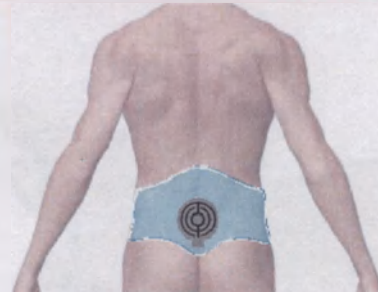2. Lumbosacral area

↳ 10 Hz

S: 10-15 min 3-4

3-4 times a day when there is pain.

EN

Lower abdominal pain and genitals pain
(prostatitis, urethritis, orchitis, algodismenorrhea)

177

1. Area according to Su Jok

↳ 77 Hz

S: 5 min. 1-2 times a day

Additional zones

Note. Course duration — until full recovery.


EN

1. 7th vertical vertebra area

↳ 1,7 Hz

S 5 min. 1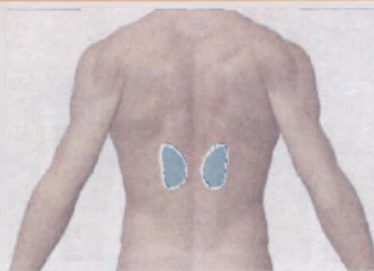

2. Adrenal glands area

↳ 1,7 Hz

S 5 min. 1

Additional zones

Note: Course duration — for a minimum of 10–14 days. Repeat the course in 1–1.5 months.

EN

Main zones

1. Facial skin according to cosmetic lines

↳ 77 Hz

S 5–10 min. 1

2. Submaxillary area

↳ 77 Hz

S 5–10 min. 1

EN

1. Liver and intestines area according to Su Jok

↳ 77 Hz

S min. 1

2. Neck and collar area

↳ 10 Hz

S: 10–15 min. 1

Additional zones

Note: Conduct sessions 3 times a week. Course duration — 12–15 treatments.

EN

Handwritten signature


1. Area according to Su Jok

↳ 9,0 Hz

S 5 min. 1 time a day

Note: Course duration — 10 days.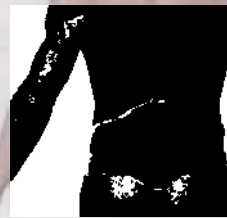

2. Lumbosacral area

↳ 10 Hz


:S: 10-15 min. 1 time a day

Additional zones

EN

☀ Face pain (trifacial neuralgia, Bell's palsy)

Main zones

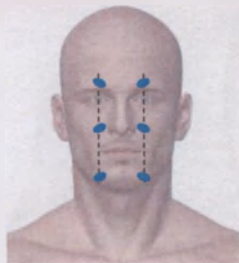1. Trigeminal branches output points

↳ 77 Hz

S 2-3 min.

2. Area under earlobe

↳ 77 Hz

S 2-3 min.

3. Neck and collar area

↳ 10 Hz

:S: 10 min.

EN

Until the pain is relieved or once a day during course treatment.

Face pain (trifacial neuralgia, Bell's palsy) ✨

1. Area according to Su Jok

↳ 9,0 Hz

S 5 min. 1 time a day

Additional zones

EN

Note: Course duration — 10 days.

Note: Course duration — 5–10 days.

2. Lumbosacral area

↪ 10 Hz

S: 10–15 min. 1–2

Conduct treatment in the morning and/or in the evening.


Additional zones

EN

194

♥ Heartache (cardialgia, myocardiodystrophy, coronary heart disease)

Main zones

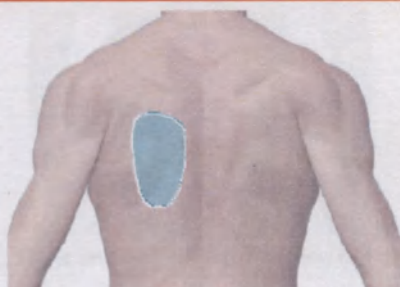1. Back projection of heart

↪ 77 Hz

S: 5–10 min.

Until complaints decrease / stop.

2. Area between the shoulder blades

↪ 10 Hz

S: 10 min.

Until complaints decrease / stop.

EN

Heartache (cardialgia, myocardiodystrophy, coronary heart disease) ♥

195

1. Area according to Su Jok

↪ 10 Hz

S: 5–10 min.

Until complaints decrease / stop.

Additional zones

EN

Handwritten signature

Additional zones

1. Area according to Su Jok

↳ 9,2 Hz

S

5 min.

1

1 day

Note: Course duration — 10 days.

2. Lumbosacral area

↳ 10 Hz

:S:

10-15 min.

1

1 day

EN

Heavy legs, edema, varicose veins

(chronic venous insufficiency syndrome, varicose lower limbs disease)

Main zones

1. Shins and feet area (from toes to knees)

↳ 77 Hz

L

10-15 min.

1

1 day

Attention! It is prohibited to apply the electrodes of the device on shins in direct projection of the modified veins (varicose, tortuous veins).

2. Lumbosacral area

↳ 10 Hz

:S:

10-15 min.


1

1 day

EN

Heavy legs, edema, varicose veins

(chronic venous insufficiency syndrome, varicose lower limbs disease) ♥

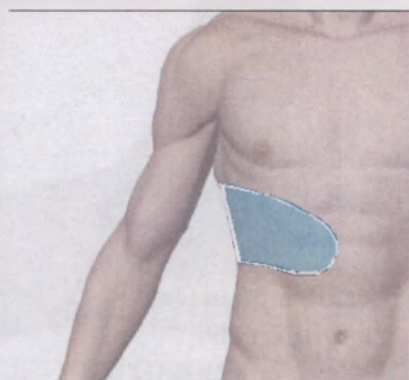1. Area of liver projection

↳ 77 Hz

S

5-10 min.

1

1 day

Once a day or once in two days

Additional zones

Note: Course duration — 2-3 weeks. Repeat the course in 1-2 months.

EN

Handwritten signature

1. Area according to Su Jok

→ 77 Hz

S 10 min.

2. Temporal region

→ 77 Hz

S 5 min. 1 time a day

Additional zones

Note: Course duration — 10–14 days. Repeat the course in 1–2 months.

Toothache, stomatitis, periodontitis

Main zones

1. Skin projection of painful tooth or inflamed mucous membrane of mouth cavity in case of stomatitis

→ 77 Hz

S 10–20 min. 2–3 times a day

Until the pain is relieved.

2. Tongue

→ 77 Hz

S 1–2 min. 2–3 times a day

Until the pain is relieved.

Toothache, stomatitis, periodontitis

1. Area according to Su Jok

→ 77 Hz

S 10 min. 2–3 times a day

Additional zones

1. Area according to Su

↳ Jok 77 Hz

S: 1-2 min.
One time.

2. Neck and collar

↳ area 10 Hz

S: 5-10 min.
One time after a faint.

Additional zones

EN

Main zones

1. Projection of carotid, brachial, ulnar and femoral arteries

↳ 77 Hz

S 30 sec. per each point
One time.

2. Forehead area and temporal

↳ regions 77 Hz

S 2-3 min.
One time.

EN

1. Forearms area (bracelet) 77 Hz

S 1 min.
One time.


2. Neck and collar area 10 Hz

S: 5-10 min.
One time.

Additional zones

EN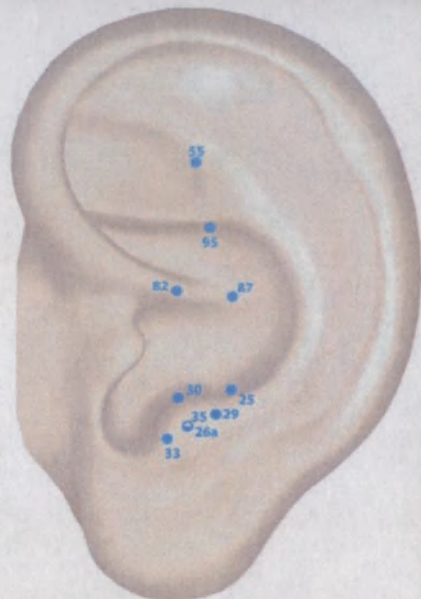


Headache.
Points of
auricle.

S  1-2 min.

Treat 1-2 pairs of points during
one session.

216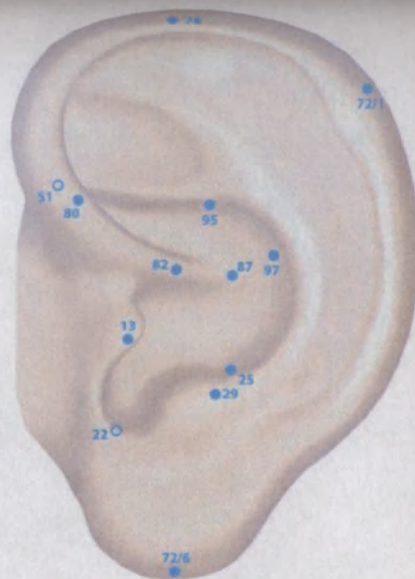


Inflammation.
Points of auricle.

S 77 Hz
 1-2 min.

Treat 1-2 pairs of points during one
session.

217

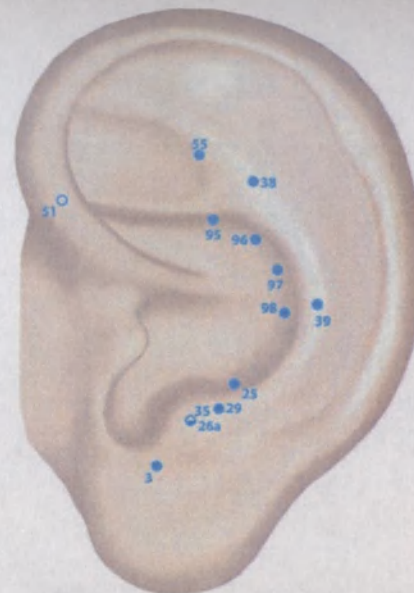General pain relief. Points
of auricle

77, 20 Hz
S  1-2 min.

Treat 1-2 pairs of points during
one session.

214

215

This translation from Russian into English is made by Oksana Churpita, a translator. I certify the identity of this translation.

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