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VelaShape III

Quick Reference Guide Circumference and Cellulite Treatment

Selecting the Appropriate Treatment Type

The VContour applicator combines RF energy and vacuum, while the VSmooth applicator combines IR energy, vacuum and massage.

#	Treatment Type	Treatment Area	Applicator	Cover	Initial Parameters	Treatment Protocol
1	Circumferential Reduction	Abdomen ¹	VContour	Medium	IR = 0 RF = 3 Vacuum = 1	Option 1: three to weekly sessions Option 2: one long treatment followed by two to five shorter sessions
2	Circumferential Reduction - Small Areas	Waist/Small Area ²	VContour	Small	IR = 0 RF = 1 Vacuum = 1	Three to weekly sessions
3	Localized Fatty Deposits	Arms/thighs ³	VContour	Large/Arms/thighs/Markings	IR = 0 RF = 3 Vacuum = 0	Three to weekly sessions
4	Cellulite	Arms/thighs/Trunk/Markings	VSmooth	VSmooth	IR = 1 RF = 1 Vacuum = 1	Three to weekly sessions
5	Localized Fatty Deposits plus Cellulite	Arms/thighs/Trunk/Markings	VContour followed by VSmooth	Large/Arms/thighs/Markings	See above for VContour and VSmooth	Three to weekly sessions

1. Select target treatment area and the appropriate treatment protocol.
2. Measure areas and take photos.
3. Position patient in the optimal orientation and comfortably on the treatment table.

Important

For all treatment modes the end-point temperature is 43-45°C

- (a) Circumference Reduction treatment may be performed on similarly sized areas such as thighs, arms, etc.
 (b) Circumference Reduction treatment may be performed on areas such as Submental.
 (c) Localized Fatty Deposits treatment may be performed on similarly sized areas such as: flanks, thighs, middle back, etc.

Syneron

www.Syneron-Candela.com

CANDELA

For more detailed instructions refer to the Clinical Guide chapter in the VelaShape III User Manual.

SYNERON CANDELA

Syneron Medical Ltd.
Industrial Zone, Tavor Building
Yokneam Illit, P.O. Box 560
Israel 20692

MODEL : VelaShape III

FG71001

REF

SN

U16113153

Series No: 10

POWER: 230V~2A
50/60 Hz

THIS DEVICE COMPLIES WITH PART 18 OF THE FCC RULES.
OPERATION IS SUBJECT TO THE FOLLOWING TWO
CONDITIONS: (1) THIS DEVICE MAY NOT CAUSE HARMFUL
INTERFERENCE, AND (2) THIS DEVICE MUST ACCEPT ANY
INTERFERENCE RECEIVED, INCLUDING INTERFERENCE
THAT MAY CAUSE UNDESIRABLE OPERATION. CONNECT
ONLY TO GROUNDED OUTLET.
U.S. FEDERAL LAW RESTRICTS THIS DEVICE TO SALE AND
USE BY OR ON THE ORDER OF A LICENSED PHYSICIAN.

LB79301

712023

0344

QC Syneron

Segurança

CCF 0089

Compulsório

LB83351

2 x T2A, 250V ~

LB79462

EC REP

European (EU/EC) Authorized Representative:
CEpartner4U B.V.
Eedoomlaan 13,
3951DB Maarn www.CEpartner4u.eu
The Netherlands
Tel: +31 (0)343 442 524
Fax: +31 (0)343 442 182

This product is covered by
patents internationally.
For a list of applicable
patents, visit:

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