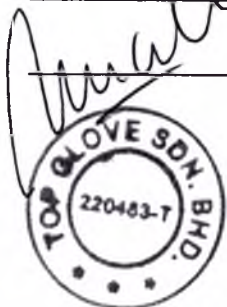
APPROVED BY:

QA Deputy General Manager

« TOP GLOVE SDN.BHD. », Malaysia

Lot 4969, Jalan Teratai, Batu 6, Off Jalan Meru, 41050 Klang,Selangor D.E., MalaysiaTel.: + 603.3362.3098 sales@topglove.com.my

Pn. Noor Akilah Saidin,

Medical product information data sheet

Gloves surgical sterile disposable "BENOVY"

DR. DATO' DR. KARIM
MEDICAL OFFICER
MALAYSIAN PUBLIC
HEALTH BOARD
KUALA LUMPUR
MALAYSIA
100, Jalan Sultan Ismail,
Kuala Lumpur 50000 Shah Alam
Jalan Sultan Ismail, Seksyen 11
40100 Shah Alam, Selangor
MALAYSIA
Tel : 603-5513 5610
Fax : 603-5513 5609

10 MAR 2017

1. Medical product name

Gloves surgical sterile disposable "BENOVY" have various design variants:

Design variants:

1. Gloves surgical sterile disposable "BENOVY" from natural latex hand specific curved fingers textured powdered (beaded cuff/ beadless cuff); colour: natural;
2. Gloves surgical sterile disposable "BENOVY" from natural latex hand specific curved fingers textured powder free with polymer coating (beaded cuff); colour: natural;
3. Gloves surgical sterile disposable "BENOVY" from natural latex hand specific curved fingers textured powder free with polymer coating with indication of the puncture (beaded cuff); colour: natural, green;
4. Gloves surgical sterile disposable "BENOVY" from natural latex hand specific curved fingers textured powder free with double chlorination (long beaded cuff); colour: natural;

2. Medical product application

2.1.Purpose

Medical product designation specified by the manufacturer:

– gloves are designed to protect health worker's hands in a surgical field.

2.2.Scope of use

Gloves are mainly used as a two-sided barrier to protect a patient and health workers from different microbial contaminations.

3. Medical product use

3.1. Indications

- Medical surgical sterile gloves should be used in case of:
- any surgery; any major or minor surgical operations, minor skin surgical operations and other procedures associated with penetration in subdermal tissues or mucous membranes;
- protection of health workers and patients during procedures associated with penetration in sterile body areas (vessels, urogenital tract, tracheobronchial tree), rectal, pelvic exams, contact with mucous membranes, etc;
- drugs preparation for total injection;
- sterile work surface preparation and putting instruments and endoscopes on sterile drape after sterilization;
- protection of health workers and patients in case of contamination risk during direct contact with a patient (patient's biological medium) or potential contact with mucous membranes, damaged skin, blood or other biological substrates potentially or clearly contaminated with microorganisms.

When choosing a specific design variant of the gloves it is necessary to consider the following factors to ensure their most comfortable use:

- material the gloves
- cover and coating of internal surface (powdered or powder free),
- length of the glove and design of the cuff,
- utility and comfort (curved fingers, anatomic or non anatomic shape).
- size of the glove.

Attention! The gloves do not provide 100% hand protection from perforation and cuts.
The gloves are designed for single use only.

MOHD. JACOB KARIM
NOTARY PUBLIC
SILANGOR
MALAYSIA

10 MAR 2017

3.2. Contraindications

- Attention! Latex gloves may cause allergic reaction.
- Attention! Powdered gloves may cause allergic reaction.

3.3. Medical product application rules

- Every health worker aware of all features of his/her work should sum up individual requirements for the gloves taking into account:
 - actual occupational hazards (evaluation of biological, mechanical and chemical risks);
 - sanitary and epidemiological conditions (sanitary demand);
 - operational features (types of performed procedures and actions, physical efforts, duration of work in gloves; tactile sensitivity need, etc.);
 - skin health of the health worker, allergological history of the health worker and a patient and some other factors.
- The gloves should be worn in case of any surgical operations, procedures associated with penetration in subdermal tissues or mucous membranes, procedures performed in sterile boxes and with sterile medium, with materials and instruments, in case of possible contact with blood, other biological substrates and other external environment objects to reduce the risk of microbial contamination.
- Gloves are no substitute for proper hand hygiene. Handwash should be performed in accordance with the mode of procedures.
- It is necessary to choose gloves of appropriate size.
- Gloves should be chosen depending on procedure type, they should be worn easily, should not hamper movements or compress hands.
- When putting on gloves a person should not make great efforts.
- Special techniques to put on the gloves should be followed in order to guarantee their sterility.
- Before putting on gloves for medical procedures, working area should be prepared: sterile instruments and consumables should be put on a manipulation table ready for use. It is necessary to ensure their accessibility without breaching the sterility of hands.
- Using additional pair of gloves may reduce gloves damage rate when working with sharp instruments by 2-4 times.
- It's not allowed to use gloves from a damaged package, because their sterility can be breached.
- After gloves removal it is necessary to wash hands with soap and disinfect them with dermal antiseptic.

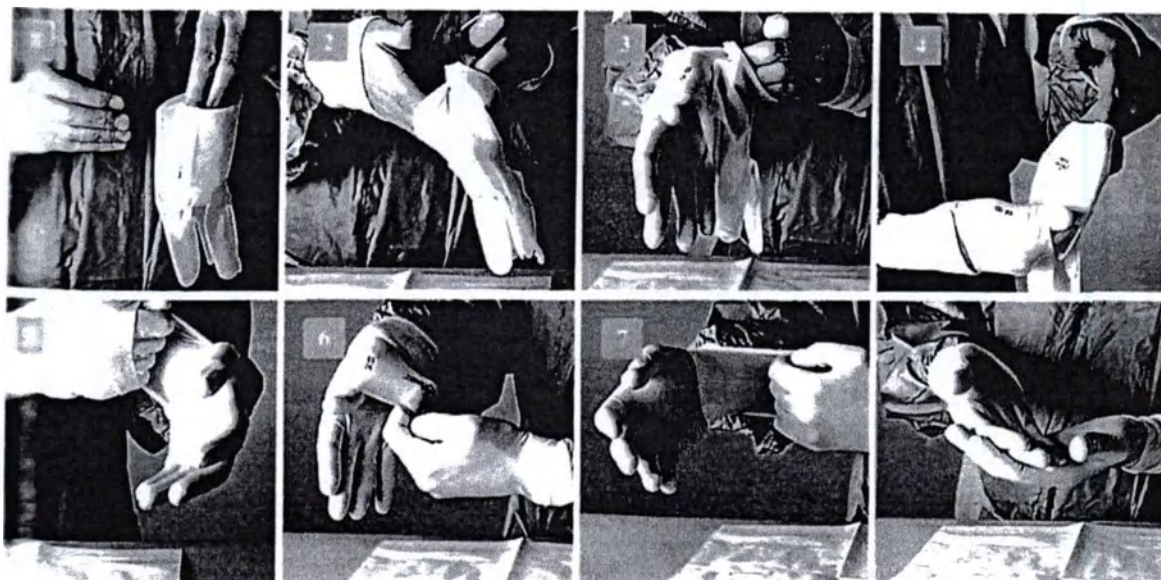
3.4. Medical product use

Before using the gloves please check package integrity. Open secondary package. Primary package of the gloves is sterile. Before opening the primary package and getting the gloves in order to use them as intended, please disinfect your hands.

3.4.1. Putting on gloves

It is recommended to follow the techniques described below in order to avoid external surface contamination when putting on the gloves. There are 2 techniques to put on the gloves: open and closed. Whatever technique is chosen, the most important is to avoid external surface contamination.

Open technique steps

- Open sterile package and take two gloves with your finger tips of the left hand from inside of the left glove, so that the thumb tip is inside the right glove, and hold them together vertically. Make sure the fingers contact only with an internal surface of the gloves.
- Pass the right hand through the cuff holding the gloves in your left hand.
- Pull the folded glove until your right hand is inside the glove.
- Put right hand fingers into the left glove cuff and draw it on the left hand.
- Draw the left hand glove cuff on a surgical coat cuff with four fingers.
- Put your left hand fingers in the glove under a folded glove cuff and draw it on a surgical coat cuff.
- Draw on the cuff with all four fingers and fix the gloves for comfort use.
- In case of powered gloves, the powder should be aseptically removed from the glove surface.

Close technique steps

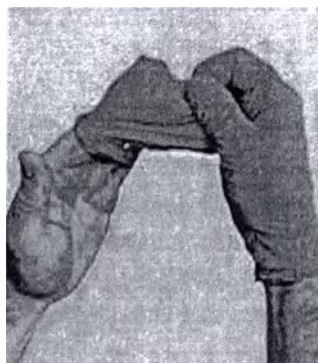
- Open secondary package before washing your hands. Primary package is sterile. Before opening and getting the gloves please put on a surgical coat.
- Draw coat cuffs on your hands, grab the left glove with the right hand. Hold the hand supine, fingers straight. Put the glove on the left wrist so that its fingers are pointed at you and the thumb lays on the wrist. Press down the cuff by your left thumb.
- Put the thumb on the end inside the cuff. Jam your right hand and draw the glove on the fingers.
- Straightening out your left hand fingers and holding the cuff in the right hand, draw on the glove with four right hand fingers till it is drawn on the left hand fingers.
- Repeat the steps with the second glove. In particular: Put the right glove on the right wrist using the left hand in the glove so that its fingers are pointed at you and the thumb lays on the wrist. Press down the cuff by the right thumb.
- Put the left thumb on the end inside the cuff. Jam your right hand and draw the glove on the fingers.
- Straightening your right hand fingers and covering the wrist by the cuff of the surgical coat draw the glove on with four left hand fingers till it is drawn on the right hand fingers.
- Now, when the gloves are put on both hands they can be fixed without the risk of contamination.
- To avoid gloves slipping from the coat cuffs, please ensure the sufficient length of the coat sleeves and sufficient cuff length up to the heel of the hand, as well as avoid folds on the glove cuff.
- In case of powered gloves, the powder should be aseptically removed from the glove surface.

3.4.2. Requirements for working in the gloves

- The gloves should be changed when switching from “dirty procedures” to “clean procedures” or going from one patient to another.
- It is necessary to change the gloves:
 - if any visual defect is observed;
 - after potential or actual perforation by a needle, suture material, bone, etc.;
 - immediately after a direct contact with unhardened methyl methacrylate (bone cement);
 - after accidental electric shock to a hand from an electrosurgical unit;
 - if a glove starts to bloat, strain, stick or stays loosely on the hand due to absorption of liquids or lubricants;
- In case of using powdered gloves it is necessary to aseptically remove powdering agent by: washing it off with sterile water or washing hands in a sterile tank or using a sterile drape. Powder free gloves do not require additional treatment.
- The gloves can not be cleaned during operations (procedures). They should be changed if necessary.

- Damaged or perfused gloves can not be used. If a glove is damaged, both gloves should be changed immediately because it is impossible to remove one glove without contamination of another.
- Remove the gloves carefully avoiding hand skin contamination and spillage of liquids.
- It is not recommended to clean disposable latex gloves with alcohol-containing solutions, because these solutions may damage a protective cover of the gloves. Gloves made of synthetic materials (nitrile, etc.) are proof against the action of antiseptic chemical agents.
- Strictly observe safety precautions while working with sharp piercing objects and instruments.

3.4.3. Removing the gloves

It is recommended to comply with the following glove removal algorithm:

- Take the glove near the heel of the hand with the fingers of the other hand.
- Remove the glove and hold it with the other hand.
- Then put the fingers of your free hand into the glove cuff on another hand and remove the glove as the first one.
- Turn the glove inside out and use it as a bag for the second glove.
- Put the gloves in a special container. Then clean the hands.

4. Medical product description and its specification.

Sterile disposable surgical gloves “BENOVY” are disposable medical products made of natural rubber latex with or without powdering (modified cornstarch), with different internal coating, sterilized and packed in pairs in sterile packages.

Class according to the degree of the potential usage risk — 2a.

4.1. Material used for gloves manufacture is natural rubber latex

4.2. There are the following types of gloves depending on the cover:

- texture surface;
- powdered surface;
- non-powdered surface;
- double chlorination;
- with inner polymer coating.

4.3. Flexicoat 88 polymer is used for internal coating of gloves. This polymer coating ensures additional durability of gloves, isolates skin from natural latex proteins and considerably facilitates the putting on, as well as prevents sticking of gloves.

Gloves can be processed using gaseous chlorine (chlorination). The chlorination can be single or double (both sides).

USPXXI grade modified cornstarch is used for gloves powdering.

4.4. Gloves have ten nominal sizes:

- size 5;
- size 5.5;

- size 6;
- size 6.5;
- size 7;
- size 7.5;
- size 8;
- size 8.5;
- size 9;
- size 9.5.

4.5. The following dyes are used during gloves manufacture:

Green Colour – Farsperse Green PG 708R

4.6. Gloves with perforation indication function are used to protect a physician and a patient in conditions of a high infection risk. Their two-layer design is made in two colours. It does not only provide better protection due to the increased density, but also indicates the deformation of the top layer and the contract with blood and lavage fluids with the appearance of a contrast spot.

The main feature of these gloves is the control and indication of perforation, cuts and mechanical damage. Packaging of the gloves with perforation indication: two pairs of gloves in one sterile package. Inner glove: green colour with inner polymer coating. Outer glove: natural colour with inner polymer coating. The inner glove is smaller than the outer glove by 1/2 of size.

4.7. The product is sterile.

4.8. The products are designed for single use only.

4.9. Class according to the degree of the potential usage risk – 2a.

5. Specification

5.1. Width, length and thickness of gloves, depending on the size, shall correspond to values specified in Table 1.

If measured according to the points (shown in Figure 1), glove size should comply with width and length (see Table 1) in case of inspection level and acceptance quality level (AQL) specified in Table 2.

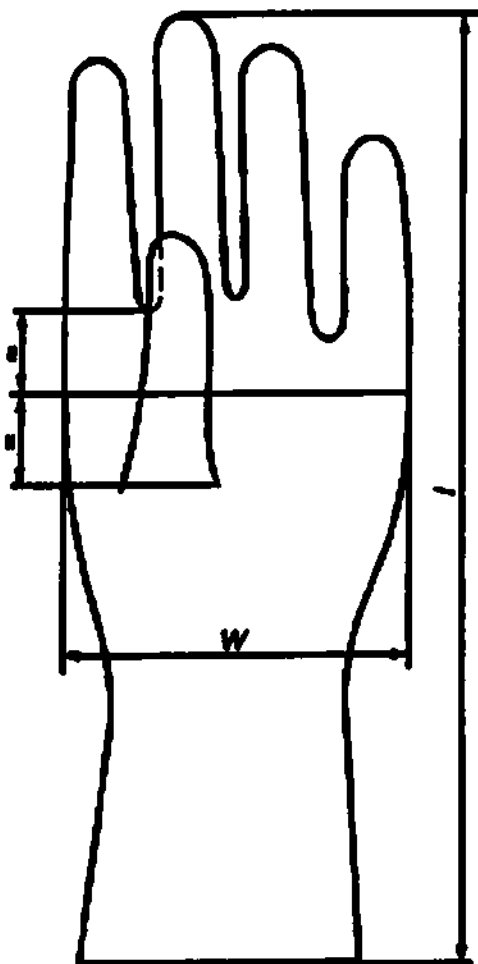
Figure 1

Table 1 Gloves sizes

Size code	Width, mm	Minimum length (beaded cuff), mm	Minimum length (beadless cuff), mm	Length (long beaded cuff), mm	Minimum thickness at measurement points (Figure 2), mm
5	67±4	250	250	300±10	0.10 for non-textured sections 0.13 for textured sections
5.5	72±4	250	250	300±10	
6	77±5	260	260	300±10	
6.5	83±5	260	260	300±10	
7	89±5	270	270	300±10	
7.5	95±5	270	270	300±10	
8	102±6	270	270	300±10	
8.5	108±6	280	280	300±10	
9	114±6	280	280	300±10	
9.5	121±6	280	280	300±10	

The length is measured as the shortest distance from the tip of the middle finger to the edge of the cuff.

The width is measured on a flat surface using the midpoint between the base of the index finger and the thumb.

Thickness of two sides of an intact glove is measured using a thickness gage with pressure on a base of (22±5) kPa in three points shown in the Figure 2 at a distance of (13±3) mm from the tip of the middle

finger, in the centre of a palm, and at a distance of (25 ± 5) mm from the edge of the cuff. Thickness of one side in each point shall be equal to a half of thickness value for two sides and has to meet the standard given in Table 1 in case of inspection level and acceptance quality level (AQL) specified in Table 2. If, in case of visual inspection, thinning-out in any section of a glove is revealed, thickness of one side of a glove in this section is measured; it shall be at least 0.10 mm for non-textured and at least 0.13 mm for textured surface.

Thickness of glove edge shall be not more than 2.50 mm.

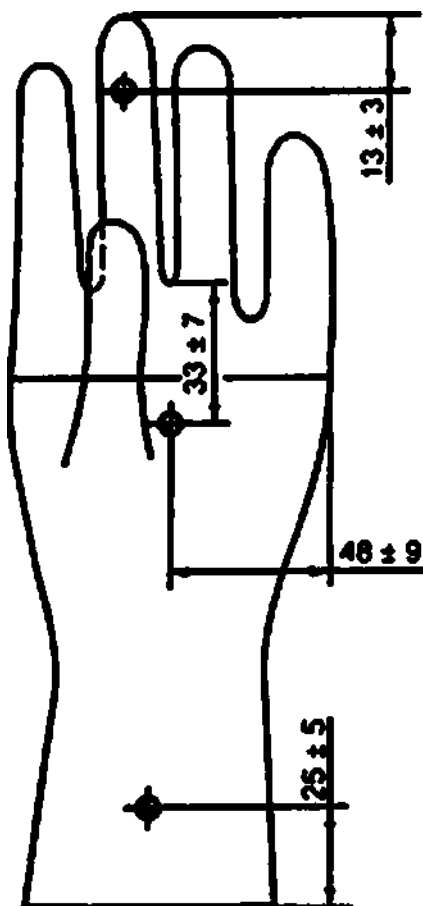
Figure 2

1-finger tip, 2-palm, 3-cuff edge

$A=(13 \pm 3)$ mm, $B=(33 \pm 5)$ mm, $C=(48 \pm 9)$ mm

For the purpose of control, gloves are selected and checked in accordance with the requirements of the regulatory document. Inspection levels and acceptance quality levels (AQL) for gloves are provided in Table 2.

Table 2 Inspection levels and acceptance quality levels (AQL)

Property	Test defect	Inspection level	AQL
Sterility	Lack of sterility	A	N/A
Size (length, thickness, width)	Width, length, thickness	S-2	4.0
Hermeticity	Presence of holes	G-1	1.5
Force at break and elongation at break (before and after accelerated ageing), force at 300% elongation (before accelerated ageing)	Before and after accelerated ageing	S-2	4.0
Physical and mechanical parameters	Before/after accelerated ageing	S-2	4.0
Residual powdering agent	Exceeds the recommended limit	N=5	N/A
Powdering agent content	Exceeds the recommended limit	N=2	N/A
Protein content	Exceeds the recommended limit	N=3	N/A

5.2. Gloves textured area.

The area of texture for textured products includes fingers and palm. The palm and fingers of the gloves working side are textured (i.e. only gloves inner side of the hand).

5.3. Medical product weight

Medical product weight

Name/ size	weight of one glove for the size (minimum)									
	5.0	5.5	6.0	6.5	7.0	7.5	8.0	8.5	9.0	9.5
Gloves surgical sterile disposable “BENOVY” from natural latex hand specific curved fingers tex-tured powdered (beaded cuff); colour: natural	5.0	5.5	6.0	6.5	7.0	7.5	8.0	8.5	9.0	9.5
Gloves surgical sterile disposable “BENOVY” from natural latex hand specific curved fingers tex-tured powdered (beadless cuff); colour: natural	5.0	5.5	6.0	6.5	7.0	7.5	8.0	8.5	9.0	9.5
Gloves surgical sterile disposable “BENOVY” from natural latex hand specific curved fingers tex-tured powder	5.0	5.5	6.0	6.5	7.0	7.5	8.0	8.5	9.0	9.5

free with polymer coating (beaded cuff); colour: natural;										
Gloves surgical sterile disposable "BENOVY" from natural latex hand specific curved fingers tex-tured powder free with polymer coating with indication of the puncture (beaded cuff); inner glove, color: green	5.0	5.5	6.0	6.5	7.0	7.5	8.0	8.5	9.0	9.5
Gloves surgical sterile disposable "BENOVY" from natural latex hand specific curved fingers tex-tured powder free with polymer coating with indication of the puncture (beaded cuff); outer glove, color: natural	5.0	5.5	6.0	6.5	7.0	7.5	8.0	8.5	9.0	9.5
Gloves surgical sterile disposable "BENOVY" from natural latex hand specific curved fingers tex-tured powder free with double chlorination (long beaded cuff); colour: natural	10.1	11.0	11.9	12.8	13.7	15.0	15.8	16.9	18.3	19.2

6. Storage and transportation conditions:

6.1. The product may be transported by any means of transport, in accordance with transport regulations in force for this type of transport, in its original transport packages (pressboard boxes) protected from direct sunlight.

Gloves shall retain their properties under operating conditions in case of the following environmental condition: temperature and relative humidity of air:

temperature – +10-+30 °C;

relative air humidity – up to 80% at
+25 °C.

Gloves, packed into transport containers, shall be resistant to the following climatic effects:
temperature – -30-+50 °C;

relative air humidity – up to 98% at 35 °C, but without moisture condensing.

6.2. The gloves shall be stored in original multiple pressboard packages, containing individual sterile paper packages, in a well-ventilated room away from heat sources, direct sunlight, direct exposure of light and electrical equipment. Condensation may occur if storage temperature is less than 10 °C. If storage temperature is less than 10 °C, the gloves shall be warmed up to 25-30 °C before use, otherwise difficulties may occur due to elasticity degradation of the gloves.

6.3. Store in a dry place protected from direct sunlight and high humidity at 10 °C-30 °C.

6.4. Store the gloves away from sources of ignition.

7. Packaging

7.1. Packaging shall provide protection from mechanical and climatic factors during transportation and storage, as well as ensure complete vehicle load (capacity) use and convenience of handling operations.

7.2. Packing of products shall conform to the requirements of the manufacturer and terms of the contract between the manufacturer and the authorized representative.

Individual package

A glove or pair of gloves are packed into an inner package that must be large enough to ensure the easy putting on of the gloves on its opening.

A pair of gloves or two pair of gloves in an inner package are placed into an individual package (containerette) to ensure product sterility.

Consumer (multiple) packaging.

Individual packages with gloves are packed in cardboard boxes. Consumer packaging protects gloves from external mechanical damage and allows to visually determine the size of the gloves.

Transport package

Consumer packages with gloves are packed in corrugated cardboard boxes. Cardboard boxes shall be durable enough to ensure the quality and sterility of the product during transportation and storage.

Instructions for use are printed on the consumer packaging.

Trademark of manufacturer and/or authorized representative of manufacturer can be marked on any type of packaging.

7.3. Design variants and individual packaging material:

1) *Paper Paper*. The main part of the packaging is made of paper processed using a special material (polymer).

Base: Laminated paper PAPER PE 40/ PE file 18G

Top: Polymeric material Medicoat 60

2) *Paper Plastic*. Combined packaging is made of paper and polymeric materials.

Base: Laminated film PET12/PE45

Top: Polymeric material Medicoat 60

3) *Thermoform*. Packaging manufactured by forging using polymeric materials

Base: Polyamide film NYLON FILM -70 MICRONS PE/PA/PE

Top: High density (low pressure) polyethylene HDPE 70 MICRONS

7.4. Net Quantity of Contents

1 pair of gloves (2 pcs) in an individual package, 50 pairs of gloves (100 pcs) in a multiple package, 400 pairs of gloves (800 pcs) in a transport package.

The following net quantity of contents is applicable for gloves with perforation indication: 2 pairs of gloves in an individual package (each pair packed in an immediate packaging, each color separately), 25 individual packages (total quantity – 100 pcs) in a multiple package, 400 pairs of gloves (800 pcs) in a transport package.

7.5. Packages dimensions

7.5.1. Paper Paper package variants. Unit of measurement: millimetres (mm).

1) Immediate package dimension: 460x275

Dimension of Paper Paper individual package: 150x175

Consumer package dimension, for Paper Paper package: 175x150x190

Transport package dimension, for Paper Paper package: 365x312x395

2) Immediate package dimension: 460x275

Dimension of Paper Paper individual package: 150x175

Consumer package dimension, for Paper Paper package: 175x150x225

Transport package dimension, for Paper Paper package: 365x312x465

3) Packages dimension for gloves surgical sterile disposable "BENOVY" from natural latex hand specific curved fingers textured powder free with polymer coating with indication of the puncture (beaded cuff); of measurement: millimetres (mm).

Variant A) Immediate package dimension: 460x275

Dimension of Paper Paper individual package: 150x175

Consumer package dimension, for Paper Paper package: 175x150x190

Transport package dimension, for Paper Paper package: 365x312x395

Variant B) Immediate package dimension: 460x275

Dimension of Paper Paper individual package: 150x175

Consumer package dimension, for Paper Paper package: 175x150x225

Transport package dimension, for Paper Paper package: 365x312x465

4) Packages dimension for gloves surgical sterile disposable "BENOVY" from natural latex hand specific curved fingers textured powder free with double chlorination (long beaded cuff); Unit of measurement: millimetres (mm).

Immediate package dimension: 460x275

Dimension of Paper Paper individual package: 140x270

Consumer package dimension, for Paper Paper package: 270x140x235

Transport package dimension, for Paper Paper package: 582x282x492

7.5.2. Paper Plastic package variants. Unit of measurement: millimetres (mm)

Immediate package dimension: 460x275

Dimension of Paper Plastic individual package: 142x270

Consumer package dimension, for Paper Plastic package: 270x140x165

Transport package dimension, for Paper Plastic package: 560x290x345

7.5.3. Thermoform package variants. Unit of measurement: millimetres (mm).

1) Immediate package dimension: 460x275

Dimension of Thermoform individual package: 131.25x160

Consumer package dimension, for Thermoform package: 131x131x240

Transport package dimension, for Thermoform package: 545x275x255

2) Immediate package dimension: 460x275

Dimension of Thermoform individual package: 131.25x160

Consumer package dimension, for Thermoform package: 131x131x270

Transport package dimension, for Thermoform package: 545x275x285

3) Packages dimension for gloves surgical sterile disposable "BENOVY" from natural latex hand specific curved fingers textured powder free with polymer coating with indication of the puncture (beaded cuff);

Variant A) Immediate package dimension: 460x275

Dimension of Thermoform individual package: 131.25x250

Consumer package dimension, for Thermoform package: 225x131x205

Transport package dimension, for Thermoform package: 545x235x420

Variant B) Immediate package dimension: 460x275

Dimension of Thermoform individual package: 131.25x250

Consumer package dimension, for Thermoform package: 225x131x230

Transport package dimension, for Thermoform package: 545x235x465

Variant C) Immediate package dimension: 460x275

Dimension of Thermoform individual package: 131.25x250

Consumer package dimension, for Thermoform package: 225x131x170

Transport package dimension, for Thermoform package: 545x235x350

8. Specifications for terminally sterilized package for medical products.

Requirements for specifications of terminally sterilized package for medical products.

8.1. Acceptance requirements. General purpose packaging material, for example wrapping materials (paper, polymer film or nonwoven fabric made of high density polyethylene HDPE, shall comply with the following requirements:

8.1.1. Materials shall be nontoxic and have no odour to the extent that it could deteriorate characteristics and safety of material, or influence negatively medical products they contact with.

8.1.2. The package shall have no pinholes or other damage;

8.1.3. The package shall be hermetic, the width of non penetrating adhesive during dye penetration test shall be at least 5 mm;

8.1.4. Material physical characteristics: density (total weight) and thickness shall correspond to the specified by the manufacturer

a) Paper Paper package

Top part material

Name	Medical paper
Construction	Top layer: Porous medical paper, 60 g/m ²
	Bottom layer: Adhesive coating, weight 5g±2g per sq.m. in coated area
Total weight	63g±2g per sq.m.

Bottom part material

Name	Film coating 18/ Paper 40 Coating Paper
Construction	Top layer: Polyethylene film coating, 18g
	Bottom layer: medical paper, 40g
Total weight	60g±5g per sq.m.

b) Paper Plastic package

Top part material

Name	Medical paper
Construction	Top: Porous medical paper, 60 g/m ²
	Bottom layer: Adhesive coating, 5g±2g per sq.m. in coated area
Total weight	63g±2g per sq.m.

Bottom part material

Name	PET12/PE45 Film
Construction	Top layer: PET Film, 12 microns thickness
	Middle layer: bonding adhesive, 3 microns thickness
	Bottom layer: PE Film, 45 microns thickness
Total thickness	55microns±15 %

c) Thermoform package

Top part material

Name	NYLON FILM -70 MICRONS PE/PA/PE
Construction	NYLON FILM -70 MICRONS PE/PA/PE
Total weight	70microns±15 %

Bottom part material

Name	High density (low pressure) polyethylene HDPE 70 MICRONS
Construction	High density (low pressure) polyethylene HDPE 70 MICRONS
Total thickness	70microns±15 %

8.1.5. The package shall be clean and free of any foreign substances.

8.1.6.-The packaging shall be hermetic.

8.1.7. The structure of an adhesive (weld) layer shall be uninterrupted, without tear and gaps, which can cause defects of adhesion;

8.1.8. Adhesive density shall correspond to $5g \pm 2g$;

8.1.9. Package adhesive (weld) bonding shall open without excessive tearing. Minimum strength of package bonding shall be 1.3-3.5N/15mm.

8.1.10. Adhesive (weld) layer width shall be from 6mm to 14mm

8.1.11. Package opening by peeling shall be uninterrupted and homogeneous, without delamination or tearing of material that could result in deterioration of aseptic properties.

9. Safety requirements

- Materials the gloves are made of shall be toxicologically safe.
- Gloves shall be sterile. Before using the gloves please check package integrity.
- In case of powered gloves, the powder should be aseptically removed from the glove surface: wash with sterile water or wash hands in a sterile tank or using a sterile drape. Powder free gloves do not require additional treatment.
- The gloves are designed for single individual use only.

10. Labelling.

- 1) Labelling of a containerette of surgical gloves includes the labelling on an immediate package (immediate packaging) and labelling on an individual package (containerette). The information is printed in Russian and, if necessary, may be repeated in other languages.
 - a) Inner package labelling shall include:
 - 1) Indication of left and right glove
 - 2) Size information
 - 3) Recommendations for powder removal (if any)
 - 4) Trademark of the manufacturer and/or manufacturer's legal representative
 - b) Containerette packaging labelling shall include:
 - 1) Name of medical product.
 - 2) Name of the manufacturer/ maker and manufacturer's legal representative in the Russian Federation.
 - 3) Address of the manufacturer/ maker and manufacturer's legal representative in the Russian Federation.
 - 4) Trademark of the manufacturer and/or manufacturer's legal representative in the Russian Federation.
 - 5) Indication of material used
 - 6) Words "straight fingers" or "curved fingers" or other words describing gloves design
 - 7) Words "Textured" or "Non-textured", "Powdered" or "Powder free"
 - 8) Size information
 - 9) Batch number
 - 10) "Date of manufacture" as well as year and month of manufacture
 - 11) Words "Sterility of the product is guaranteed only by package integrity"
 - 12) Words "Disposable"
 - 13) Words "Surgical gloves"
 - 14) Words "The product is made of natural rubber latex, it may cause allergic reactions" (for products made of natural latex), Words "Powdered gloves may cause allergic reaction" (for powdered gloves).
 - 15) Information on sterility and sterilization method.
 - 16) Information of cuffs
 - 17) Shelf life
 - 18) Number of items in a package.
 - 19) Storage conditions
 - 20) Part number
 - 21) Bar code
 - 22) Conformity assessment information
 - 23) Indication of regulatory document in force in a country of the legal representative (Russian Federation)
 - 24) Number and date of marketing authorisation (and date of its issue) necessary for product distribution in a country of the legal representative (Russian Federation)
 - 25) Medical product designation
 - 26) It is possible to use the symbols for medical products of ISO 15223-1:2012 "Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements" (and other symbols approved in the country of medical product distribution) (if necessary)
- 2) Consumer packaging labelling shall include:

1. Name of medical product.
2. Name of the manufacturer/maker and manufacturer's legal representative in the Russian Federation.
3. Address of the manufacturer/ maker and manufacturer's legal representative in the Russian Federation
4. Trademark of the manufacturer and/or manufacturer's legal representative in the Russian Federation.
5. Indication of material used
6. Words "straight fingers" or "curved fingers" or other words describing gloves design.
7. Words "Textured" or "Non-textured", "Powdered" or "Powder free"
8. Size information
9. Batch number
10. "Date of manufacture" as well as year and month of manufacture
11. Words "Sterility of the product is guaranteed only by package integrity"
12. Words "Disposable"
13. Words "Surgical gloves"
14. Words "The product is made of natural rubber latex, it may cause allergic reactions" (for products made of natural latex), Words "Powdered gloves may cause allergic reaction" (for powdered gloves).
15. Information on sterility and sterilization method
16. Information of cuffs
17. Shelf life
18. Number of items in a package.
19. Storage conditions
20. Part number
21. Bar code
22. Conformity assessment information
23. Indication of regulatory document in force in a country of the legal representative (Russian Federation)
24. Number and date of marketing authorisation (and date of its issue) necessary for product distribution in a country of the legal representative (Russian Federation)
25. Medical product designation
26. Information needed for use, storage and disposal of the product.
27. It is possible to use the symbols for medical products of ISO 15223-1:2012 "Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements" (and other symbols approved in the country of medical product distribution) (if necessary)
28. The information is printed in Russian and, if necessary, may be repeated in other languages.

11. Medical product labelling layout

11.1. Gloves surgical sterile disposable “BENOVY” from natural latex hand specific curved fingers textured powdered (beaded cuff/ beadless cuff); color: natural

11.1.1. Gloves surgical sterile disposable “BENOVY” from natural latex hand specific curved fingers textured powdered (beaded cuff); color: natural

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LOT

3) Consumer (multiple) packaging

12. Symbols and images specified on medical product labelling and their explanation.

12.1. Package layouts have blank spaces for the following marks (numbering according to the figures in the layouts):

№1 – place for indication of sterilization method;

Depending on sterilization method, the following symbols will be used:

Symbol “Sterilization with ethylene oxide”

┌ ┐

└ ┘

Symbol “Sterilization with radiation”

┌ ┐

└ ┘

№2 – place for conformity mark when product certification is obligatory; The sign of confirmation of products compliance in the obligatory certification form (this confirmation form is valid at the moment of medical product registration and can be changed considering the requirements of legislation at the time of product distribution)

№3 – place for product item number;

№4 – place for product bar code.

№5 – place for product size indication

№6 – place for information in English language

№7 – place for information in Kazakh language

12.2. Areas indicated as

XXXX

 shall be filled with numeric information.

12.3. Symbol

12.3.1. The symbol "Batch code"

12.3.2. The symbol "Do not reuse".

12.3.3. The symbol "Date of manufacture"

The area indicated as

XXXX

 shall be filled with numeric information.

12.3.4. The symbol "Contains natural latex"

12.3.5. The symbol "Keep dry"

12.3.6. The symbol "Protect from direct sunlight"

12.3.7. Symbol "Sterilization with ethylene oxide"

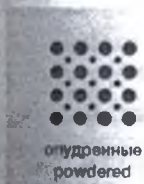
12.3.8. Symbol "Sterilization with radiation"

12.4. Image (A, B, C, D) specified on the package are the part of the design and repeat fragments from the name of the medical product

A) The image means that the gloves are textured.

B) The image means that the gloves are powder free.

C) The image means that the gloves are surgical.

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D) The image means that the gloves are anatomical.

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13. Warranty and shelf life

The manufacturer guarantees sterility of the gloves and integrity of all their characteristics and parameters, described in this document, throughout the shelf life (5 years), if individual packages are not damaged and storage, transportation and operating instructions are followed..

14. Disposal and destruction procedures

The system of collection, temporary storage and transportation of medical waste should consist of the following steps:

- internal waste collection in a healthcare and/or pharmaceutical organization;
- transportation of waste from the divisions and its temporal storage on the territory of the organization generating waste;
- disinfection/neutralization;
- transportation of waste from the territory of the organization generating waste;
- dumping or disposal of medical waste.

Chemical disinfection of waste at place of its generation is performed as a mandatory interim measure in the absence of medical waste handling area of healthcare and/or pharmaceutical organizations, or in the absence of a centralized system of medical waste neutralization in the area.

Methods of safe disinfection and/or neutralization waste shall be chosen depending on the capacity and profile of the healthcare organization, availability of facilities for disinfection/neutralization of waste, methods of waste disinfection/neutralization adopted in the area (incineration, removal, recycling).

Collection, temporal storage and removal of waste shall be performed in accordance with a medical waste handling plan of a healthcare and/or pharmaceutical organization.

15. Medical product manufacturer data

Manufacturer:

TOP GLOVE SDN. BHD, Malaysia

Address: Lot 4969, Jalan Teratai, Batu 6, Off Jalan Meru, 41050 Klang, Selangor D.E., Malaysia

Manufacturer and production site address:

TOP GLOVE SDN. BHD, Malaysia

Address: Lot 4969, Jalan Teratai, Batu 6, Off Jalan Meru, 41050 Klang, Selangor D.E., Malaysia

TG MEDICAL SDN. BHD, Malaysia

Address: Lot 5091, Jalan Teratai, Batu 5, Off Jalan Meru, 41050 Klang, Selangor D.E., Malaysia

16. Claims

For all issues related to gloves surgical sterile disposable "BENOVY" manufactured by TOP GLOVE SDN. BHD, Malaysia, please contact the legal representative of the manufacturer in the Russian Federation: «BENOVY» Company Limited, legal and postal address: 1, 5-y Verkhniy pereulok, lit. J, St. Petersburg 194292 Tel.: (812)309-16-32. E-mail: benovy@benovy.ru.

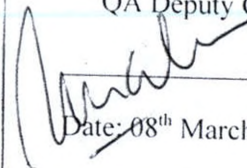
MOHD. YACOB KARIM
NOTARY PUBLIC
SELANGOR
Lot 10, Jalan 10,
Pusat Perniagaan 10 Shah Alam
40100 Shah Alam, Selangor
MALAYSIA
Tel : 603-5513 5610
Fax : 603-5513 5609

01 MAR 2017

List of enclosed documents are as below:

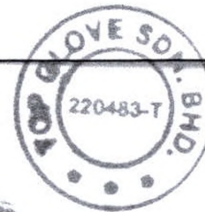
1. Medical product information data sheet (Surgery latex)
(Total: 37 pages)

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