APPROVED BY

Advanced Healthcare Products SDN BHD

(signature)

Tuan Hj Syed Al Aljffri, CEO
(name, position)

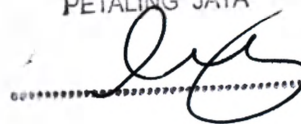
«15» November 2015

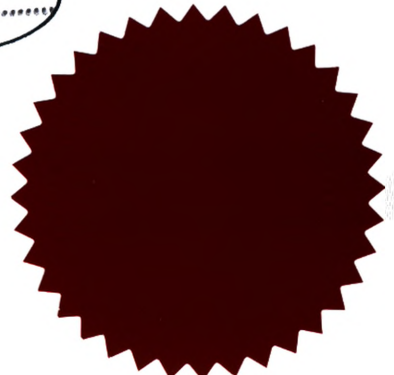
USER MANUAL

Latex surgical gloves, sterile
(Advanced Healthcare Products SDN BHD)

CERTIFIED TRUE COPY
OF THE ORIGINAL

No:..... 13 OCT 2016
LEE MOI YIN
NOTARY PUBLIC
PETALING JAYA

2015

Contents

Description	3
Manufacturer	3
Intended use	3
Specification.....	4
Complete set.....	9
Packing and marking.....	9
Product range	13
Information on sterilisation	13
Transportation and storage	13
Information for healthcare professional	14
Warning.....	14
Materials (application information for healthcare professional).....	14
Directions for use	14
Service and maintenance.....	14
Disposal procedure.....	15
Warranty.....	15
Contact details.....	15

Description

Latex surgical gloves, sterile.

Manufacturer

Advanced Healthcare Products SDN BHD
Lot 60 & 61, Lorong Senawang 3/2, Senawang Industrial Estate, 70450 Seremban, Negeri Sembilan Darul Khusus, Malaysia.

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The manufacturer possesses a certified quality managements system, the manufacturing process and final quality control fully comply with the requirements established by EN ISO 13485:2012, EN ISO 9001:2008, Directive 93/42 EEC, that is confirmed by the following certificates:

No. MD 99288, date of issue – 29.10.2014, valid until – 20.12.2017, Notified Body: BSI;

No. FM 13934, date of issue – 26.10.2014, valid until – 20.12.2017, Notified Body: BSI;

No. CE 00742, date of issue – 08.08.2014, valid until – 06.12.2018, Notified Body: BSI.

Classification Code in accordance with Russian National Classification of Products OK-005-93: **251465**.

Risk class in accordance with Classification List of Medical Devices approved by Order of Ministry of Health of the Russian Federation No. 4n dated 06 June, 2012 – **2a**.

Classification Code in accordance with Classification List of Medical Devices approved by Order of Ministry of Health of the Russian Federation No. 4n dated 06 June, 2012 – **122630, 122640**.

Human body contact type: short-time contact with skin cover, mucous membranes and wound surface.

Application environment: healthcare facilities, medical and preventive treatment facilities.

Intended use

The gloves are used as two-way barrier to prevent healthcare consumer and healthcare professional from mutual introduction of infections during surgical operations at healthcare facilities.

Specification

Materials used:

Design style	Material
Powdered latex surgical gloves, sterile connect elite	Natural Rubber Latex CAS # 9006-04-6, produced by Chip Yoo Latex Co., Ltd, Thailand Powder: Absorbable cornstarch 21CFR 878.4480
Powdered latex surgical gloves, sterile connect pro	Natural Rubber Latex CAS # 9006-04-6, produced by Chip Yoo Latex Co., Ltd, Thailand Powder: Absorbable cornstarch 21CFR 878.4480
Powder-free latex surgical gloves, sterile connect mt	Natural Rubber Latex CAS # 9006-04-6, produced by Chip Yoo Latex Co., Ltd, Thailand
Packing:	
Inner wrap	Medical paper, produced by Billerud AB, country of origin – Sweden
Peel Pouch	White peelable film, 70 um + PA/PE 70 um Nylon, produced by Amcor, country of origin – Singapore
Dispenser	Fluted cardboard 250BB/120M/120TL E flute, produced by Percetakan Tenaga Sdn Bhd, country of origin – Malaysia

Human body contact type

Short-time contact with skin cover, mucous membranes and wound surface.

Design

Technical characteristics are shown in Table 1.

Overall dimensions are shown in Table 2, Figure 1.

External appearance is shown in Table 3.

Physical properties are shown in Table 4.

Table 1. Technical characteristics

Parameter/Characteristic		Design style		
		connect elite	connect pro	connect mt
Size		5½, 6, 6½, 7, 7½, 8, 8½, 9	5½, 6, 6½, 7, 7½, 8, 8½, 9	5½, 6, 6½, 7, 7½, 8, 8½, 9
Reorder No. depending on the size	5½	W31355	W35455	S41155
	6	W31360	W35460	S41160
	6½	W31365	W35465	S41165
	7	W31370	W35470	S41170
	7½	W31375	W35475	S41175
	8	W31380	W35480	S41180
	8½	W31385	W35485	S41185
	9	W31390	W35490	S41190
Type (depending on the material)		Natural rubber latex	Natural rubber latex	Natural rubber latex
Design		Hand-specific gloves, solid, straight fingers, beaded cuff	Hand-specific gloves, solid, straight fingers, beaded cuff	Hand-specific gloves, solid, straight fingers, beaded cuff
Surface finish	textured/smooth	Smooth	Smooth	Smooth
	powdered/powder-free	Powdered	Powdered	Powder-free
Powder		Cornstarch	Cornstarch	-
Colouring		Uncoloured, of natural colour of the material used	Uncoloured, of natural colour of the material used	Uncoloured, of natural colour of the material used

Table 2. Overall dimensions

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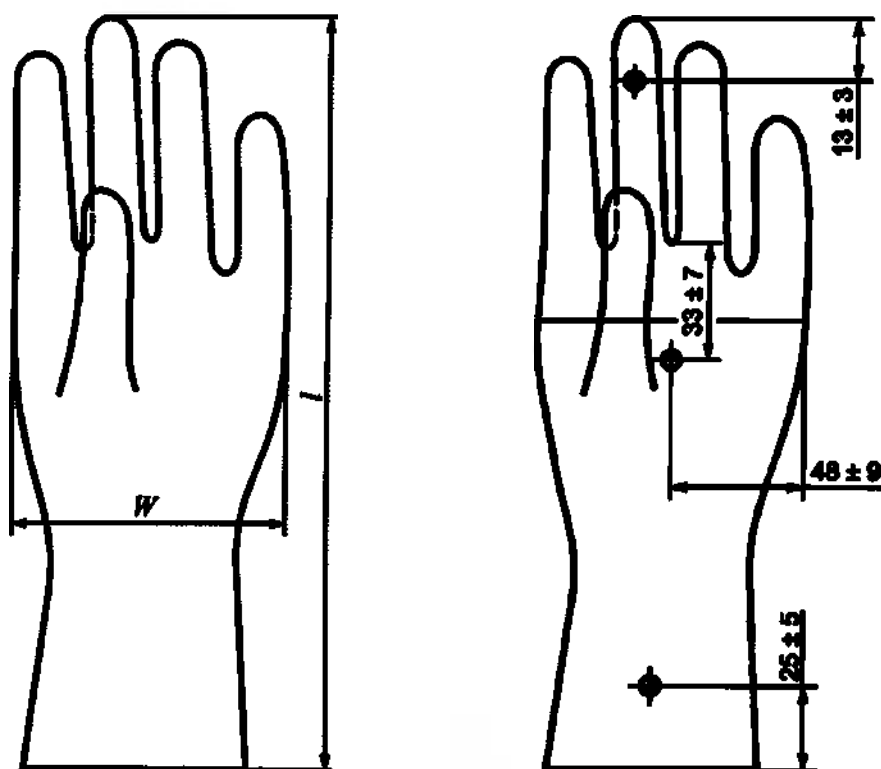
Figure 1 – Measurement points

Design style	Surface finish	Powder content	Protein content
Powdered latex surgical gloves, sterile connect elite	Powdered with absorbable cornstarch	$<15 \text{ mg/dm}^2$	$\leq 200 \text{ } \mu\text{g/dm}^2$
Powdered latex surgical gloves, sterile connect pro	Powdered with absorbable cornstarch	$<15 \text{ mg/dm}^2$	$\leq 200 \text{ } \mu\text{g/dm}^2$
Powder-free latex surgical gloves, sterile connect mt	Chlorination	Powder residual – $\leq 2 \text{ mg per glove}$	$\leq 50 \text{ } \mu\text{g}$ of total extractable protein per gram

Table 3. External appearance

Design style	Photographic image
Powdered latex surgical gloves, sterile connect elite	
Powdered latex surgical gloves, sterile connect pro	
Power-free latex surgical gloves, sterile connect mt	

Table 4. Physical properties.

Characteristic	Design style					
	connect elite		connect pro		connect mt	
	Before aging	After aging	Before aging	After aging	Before aging	After aging
Elongation (%)	750	560	750	560	750	560
Modulus 500% (MPa)	5.5	-	5.5	-	5.5	-
Force at break, N, not less	12.5	9.5	12.5	9.5	12.5	9.5

Inspection level, AQL requirements

Characteristic	Inspection level	AQL
Dimensions (length, thickness, width)	S-2	4.0
1000 water leak	G-1	1.5
Force and elongation at break (before and after aging), Modulus 500% (before aging)	S-2	4.0

Complete set

Delivery set:

- dispenser – 1 piece;

User manual is printed on the dispenser.

Packing and marking

1 pair of gloves (1 right glove and 1 left glove) per inner wrap (inner individual package).
1 inner wrap per peel pouch (outer individual package). Peel pouches are hermetically sealed.

50 peel pouches per dispenser (consumer package).

6 dispensers per outer carton (transportation package).

Glove marking is as follows:

- marking “Size” is embossed on the cuff of each glove.

Inner wrap marking is as follows:

- gloves name;
- «Powdered latex surgical gloves, sterile» in Russian and English (for powdered latex surgical gloves, sterile connect elite, powdered latex surgical gloves, sterile connect pro)
- «Powder-free latex surgical gloves, sterile» in Russian and English (for Powder-free latex surgical gloves, sterile connect mt)
- size;
- indication «Open here»;
- indication «Left» and «Right»;
- indication «Cuff end»;
- warning «Remove powder before use» (for powdered latex surgical gloves, sterile connect elite, powdered latex surgical gloves, sterile connect pro)

Peel pouch marking is as follows:

- name and address of the manufacturer in Russian and English;
- name and address of the manufacturer's representative in the Russian Federation in Russian and English;
- gloves name;
- «Surgical gloves»;
- «Material: natural rubber latex»;
- «Disposable»;
- «Sterile»;
- «Hand specific»;
- «Straight fingers»;
- «Liquid-tight»;
- «Solid»;
- «Beaded cuff»;
- «Surface finish: smooth»;
- «Powdered» (for powdered latex surgical gloves, sterile connect elite, powdered latex surgical gloves, sterile connect pro)
- «Powder-free» (for powder-free latex surgical gloves, sterile connect mt)
- «Powder – cornstarch» (for powdered latex surgical gloves, sterile connect elite, powdered latex surgical gloves, sterile connect pro)
- size;
- colour;
- quantity of gloves per peel pouch;
- lot number;
- reorder number;
- date of manufacture;
- shelf life;
- warning «The product is made of natural rubber latex which may cause allergic response»;
- «Made in Malaysia»;
- «GOST R 52238-2004»;
- symbols:

REF

– reorder number

LOT

– lot number

– date of manufacture

– single use only

– contains natural rubber latex

STERILE R

– irradiation sterilisation

– do not use if the package is damaged

– manufacturer

– avoid sunlight

– avoid moisture

– temperature limits (from 0 through +25 °C)

– read user manual

– warning! Read user manual

Variable information on the size of the gloves, lot number, reorder number, date of manufacture is printed on peel pouches with special equipment immediately after packing the gloves.

Dispenser marking is as follows:

- name and address of the manufacture in Russian and English;
- name and address of the manufacturer's representative in the Russian Federation;
- gloves name;
- «Surgical gloves»;
- «Material: natural rubber latex»;
- «Disposable»;
- «Sterile»;
- «Hand specific»;
- «Straight fingers»;
- «Liquid-tight»;
- «Solid»;
- «Surface finish: smooth»;
- «Powdered» (for powdered latex surgical gloves, sterile connect elite, powdered latex surgical gloves, sterile connect pro)
- «Powder-free» (for powder-free latex surgical gloves, sterile connect mt)
- «Powder – cornstarch» (for powdered latex surgical gloves, sterile connect elite, powdered latex surgical gloves, sterile connect pro)
- «Beaded cuff»;
- «Each pair is in an individual sterile package»;
- «Sterility is guaranteed subject to peel pouch being intact»;
- size;
- colour;
- quantity of pairs per dispenser;
- lot number;

- reorder number;
- date of manufacture;
- shelf life;
- registration certificate number;
- warning «The product is made of natural rubber latex which may cause allergic response»;
- application rules/warning;
- directions for use;
- «Made in Malaysia»;
- «GOST R 52238-2004»;
- symbols:

	– reorder number
	– lot number
	– date of manufacture
	– single use only
	– contains natural rubber latex
	– irradiation sterilisation
	– do not use if the package is damaged
	– manufacturer
	– avoid sunlight
	– avoid moisture
	– temperature limits (from 0 through +25 °C)
	– read user manual
	– warning! Read user manual

Variable information on the size of the gloves, lot number, reorder number, date of manufacture is contained on a special sticker-label.

Product range

Latex surgical gloves, sterile, design styles:

1. Powdered latex surgical gloves, sterile connect elite, sizes: 5,5; 6; 6,5; 7; 7,5; 8; 8,5; 9 (code 122640).
2. Powdered latex surgical gloves, sterile connect pro, sizes: 5,5; 6; 6,5; 7; 7,5; 8; 8,5; 9 (code 122640).
3. Powder-free latex surgical gloves, sterile connect mt, sizes: 5,5; 6; 6,5; 7; 7,5; 8; 8,5; 9 (code 122630).

Information on sterilisation

The gloves are sterile.

Method of sterilisation – irradiation sterilisation, sterilising agent – gamma rays.

The minimum dose is 25 kGy. The dose setting is performed per ISO 11137-2 (2012) using the VD_{max}^{25} method.

The Sterility Assurance Level (SAL) is 10^{-6} .

Single use only. Reuse is prohibited.

Sterility expiry dates: 3 years from the date of sterilisation for powder-free latex surgical gloves, sterile connect mt; 5 years from the date of sterilisation for powdered latex surgical gloves, sterile connect elite, powdered latex surgical gloves, sterile connect pro.

Transportation, storage and in-service conditions

Storage conditions: temperature – from 0°C through +25°C, relative humidity – $\leq 85\%$ (non-condensing) and atmospheric pressure – 500–1060 hPa, at a distance of not less than one meter from heat-radiating appliances.

While storing the gloves must be protected against exposure to direct sunlight, fluorescent lighting, atmospheric agents, ozone, greases, petroleum, paraffin and other substances destroying latex; the gloves should not be in direct contact with metals such as copper, manganese and iron.

Transportation conditions:

Transportation of gloves packed into transportation package is effected by any means of roofed transport subject to all warnings indicated on the transportation package and haulage rules for the means of transport used.

After transportation at subzero temperatures gloves can be unpacked and used only after having been kept in the transportation package at the storage temperature for a period of 24 hours.

In-service conditions: temperature – from 5°C through +35°C, relative humidity – from 35% through 60% (non-condensing) and atmospheric pressure – 500–1060 hPa.

Do not use the gloves after expire date.

Single use only.

Make sure the peel pouch is intact (hermetically sealed) before use.

Information for healthcare professional

Indications:

The gloves are used as two-way barrier to prevent healthcare consumer and healthcare professional from mutual introduction of infections during surgical operations at healthcare facilities.

Counter indications:

Idiosyncrasy or allergy to natural rubber latex.

Potential complications:

Allergic response.

Warning

- Sterility is guaranteed subject to peel pouch being intact.
- The gloves are intended for use as personal hand protective device.
- Do not use damaged gloves.
- Do not use gloves of improper size.
- Do not use gloves after the expiry date.
- The gloves do not guarantee protection from punctures and cuts.

Materials (application information for healthcare professional)

The gloves are made of natural rubber latex.

Directions for use

Application rules/warning:

- Sterility is guaranteed subject to peel pouch being intact.
- The gloves are intended for use as personal hand protective device.
- Do not use damaged gloves.
- Do not use gloves of improper size.
- Do not use gloves after the expiry date.
- The gloves do not guarantee protection from punctures and cuts.

Directions:

- Sanitize hands.
- Open the peel pouch where indicated.
- Take the inner wrap out of the peel pouch.
- Take the gloves out of the inner wrap.
- Make sure the gloves are intact.
- Carefully put on the right and left gloves so that the gloves envelope the hands.
- Take off the gloves after use and put them into a special container suitable for transportation of used disposable medical devices.

Service and maintenance

Service and maintenance are not applicable to the medical device.
Single use only.

Disposal procedure

The gloves are to be disposed of in accordance with Sanitary Regulations and Rules 2.1.7.2790-10 for class B waste (epidemiologically hazardous waste). After disinfection with special physical methods equipment and external appearance change ensuring impossibility of reuse of the device, class B waste can be stocked up, temporarily stored, transported, destroyed and buried together with class A waste (epidemiologically non-hazardous waste, close in composition to household waste). Package of disinfected class B waste must bear marking showing the waste is disinfected.

Warranty

Shelf life:

3 years from the date of manufacture (for powder-free latex surgical gloves, sterile connect mt);

5 years from the date of manufacture (for powdered latex surgical gloves, sterile connect elite, powdered latex surgical gloves, sterile connect pro).

The manufacturer guarantees the gloves sterility and full compliance with the specified requirements during the shelf life subject to peel pouches being intact (hermetically sealed) and consumer following application, storage and transportation rules.

Contact details

Powdered latex surgical gloves, sterile connect elite, powdered latex surgical gloves, sterile connect pro, powder-free latex surgical gloves, sterile connect mt manufactured by Advanced Healthcare Products SDN BHD. On quality issues please contact:

Manufacturer authorized representative in the Russian Federation:

RTL-med, LLC

Russia, 117198, Moscow, Leninsky prospect, 113/1,

Tel.: +7 (495) 783-08-88, e-mail: general@rtl-med.com.

Manufacturer:

Advanced Healthcare Products SDN BHD, Malaysia.

Lot 60 & 61, Lorong Senawang 3/2, Senawang Industrial Estate, 70450 Seremban, Negeri Sembilan Darul Khusus, Malaysia.

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