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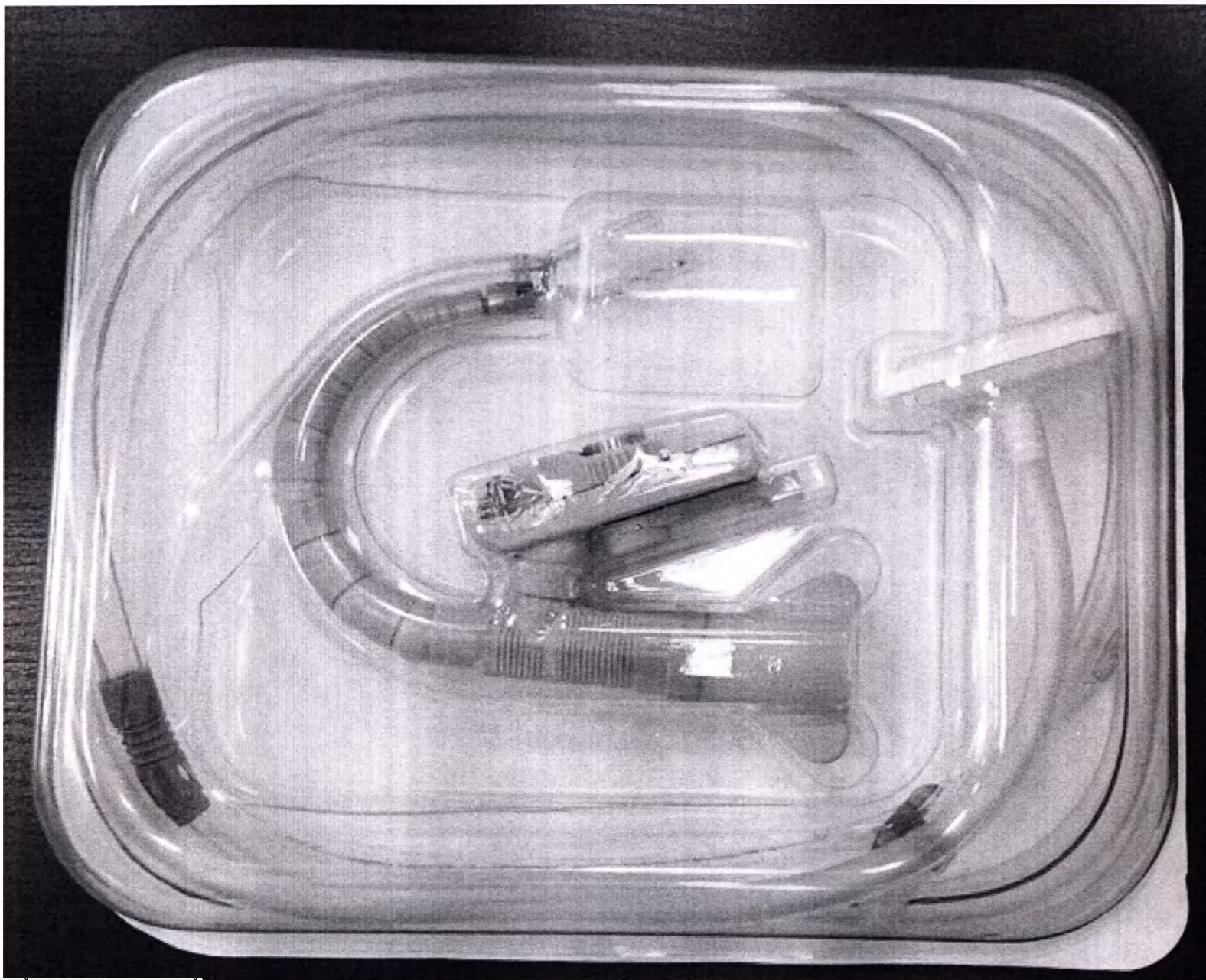
- Product Name : HUMETRON STABILIZER 1.0
- Model Name : 5385-001
- Sterilization Methods : E.O Gas
- Packing Unit : Refer to box label (Unit: EA)
- Purposes, Instruction, Cautions, Storage : Please refer to the instruction manual

- Serial No : Same as box
- Manufacturing Date : Same as rd
- Expiration Date : Same as g
- Disposable Sterile Medical Device

Aami world Bugenhagenstr. 9, 10551 Berlin, Germany TEL : +0049-(030)-8620-3461

• Manufacturer : Jinhwa Medical Co., Ltd. • Tel : +82-43-232-7554

125, Osong Saengmyeong 2-ro, Osong-eup, Heungdeok-gu, Cheongju-si, Chungcheongbuk-do, Rep of Korea

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
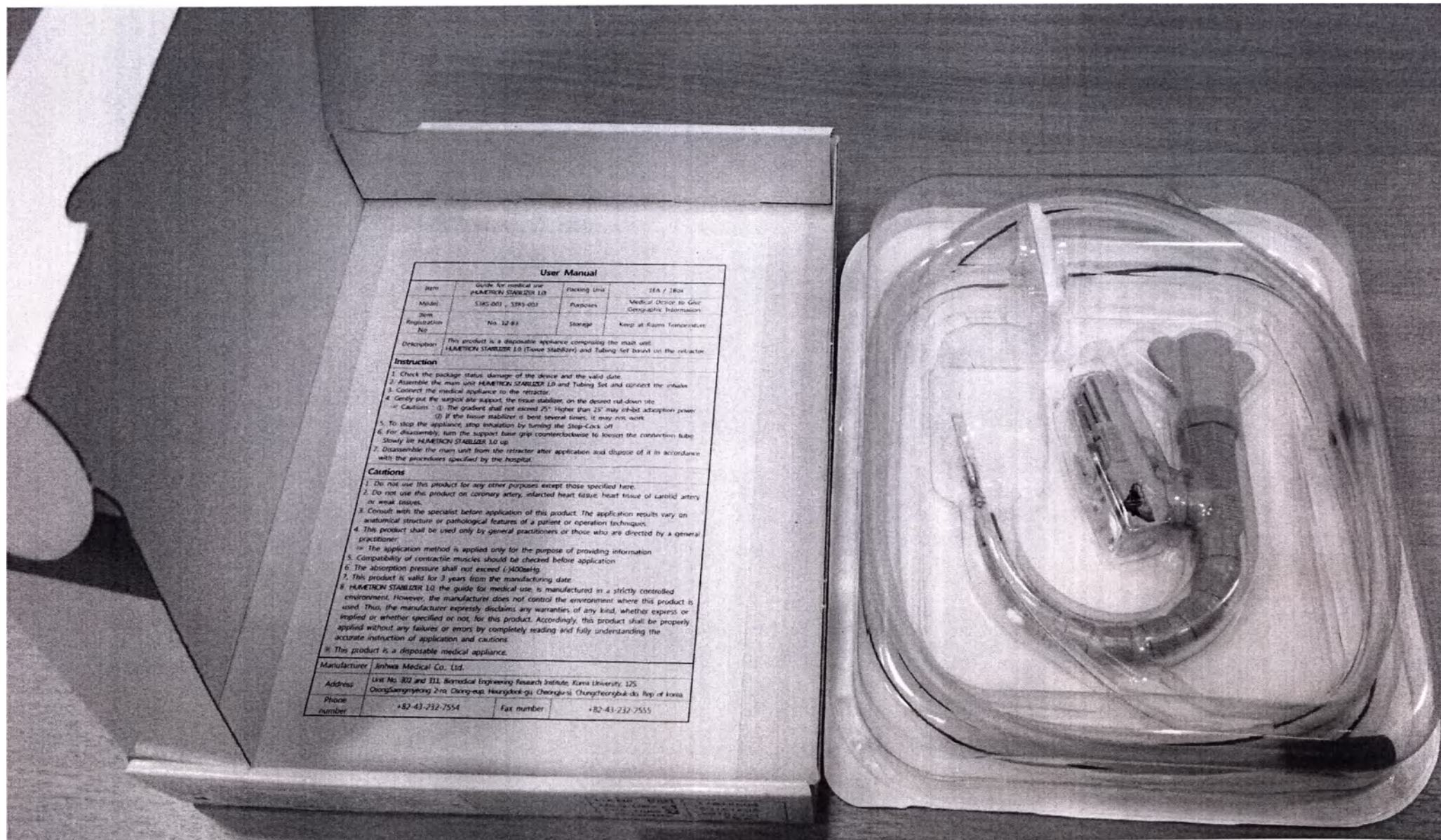
User Manual			
Item	Guide for medical use HUMETRON STABILIZER 1.0	Packing Unit	SEA / 2Box
Model	5385-001 , 5385-003	Purpose	Medical Device to Give Geographic Information
Item Registration No.	No. 12-81	Storage	Keep at Room Temperature
Description	This product is a disposable appliance comprising the main unit HUMETRON STABILIZER 1.0 (Tissue Stabilizer) and Tubing set housed on the retractor.		
Instruction			
1. Check the package status, damage of the device and the valid date. 2. Assemble the main unit HUMETRON STABILIZER 1.0 and Tubing Set and connect the intake. 3. Connect the medical appliance to the retractor. 4. Gently put the support also support, the tissue stabilizer, on the desired set-down site. Caution : (1) The gradient shall not exceed 25°. Higher than 25° may inhibit adaption power. (2) If the tissue stabilizer is bent several times, it may not work. 5. To stop the appliance, stop inhalation by turning the Stop-Clack on. 6. For disassembly, turn the support base grip counterclockwise to loosen the connection tube. Slowly lift HUMETRON STABILIZER 1.0 up. 7. Disassemble the main unit from the retractor after application and dispose of it in accordance with the procedures specified by the hospital.			
Cautions			
1. Do not use this product for any other purposes except those specified here. 2. Do not use this product on coronary artery, infarcted heart tissue, heart tissue of carotid artery or weak tissues. 3. Consult with the specialist before application of this product. The application results vary on anatomical structure or pathological features of a patient or operation techniques. 4. This product shall be used only by general practitioners or those who are directed by a general practitioner. a. The application method is applied only for the purpose of providing information. 5. Compatibility of contractile muscles should be checked before application. 6. The absorption pressure shall not exceed 1,400mmHg. 7. This product is valid for 3 years from the manufacturing date. 8. HUMETRON STABILIZER 1.0, the guide for medical use, is manufactured in a strictly controlled environment. However, the manufacturer does not control the environment where this product is used. Thus, the manufacturer expressly disclaims any warranties of any kind, whether express or implied or whether specified or not, for this product. Accordingly, this product shall be properly applied without any failures or errors by completely reading and fully understanding the accurate instruction of application and cautions. 9. This product is a disposable medical appliance.			
Manufacturer	Jinhwa Medical Co., Ltd.		
Address	Unit No. 802 and 811, Biomedical Engineering Research Institute, Korea University, 125, Ongseongryong 2-ro, Dong-gu, Hwangshik-gu, Chungcheongbuk-do, Rep. of Korea		
Phone number	+82-43-252-7554	Fax number	+82-43-252-7555

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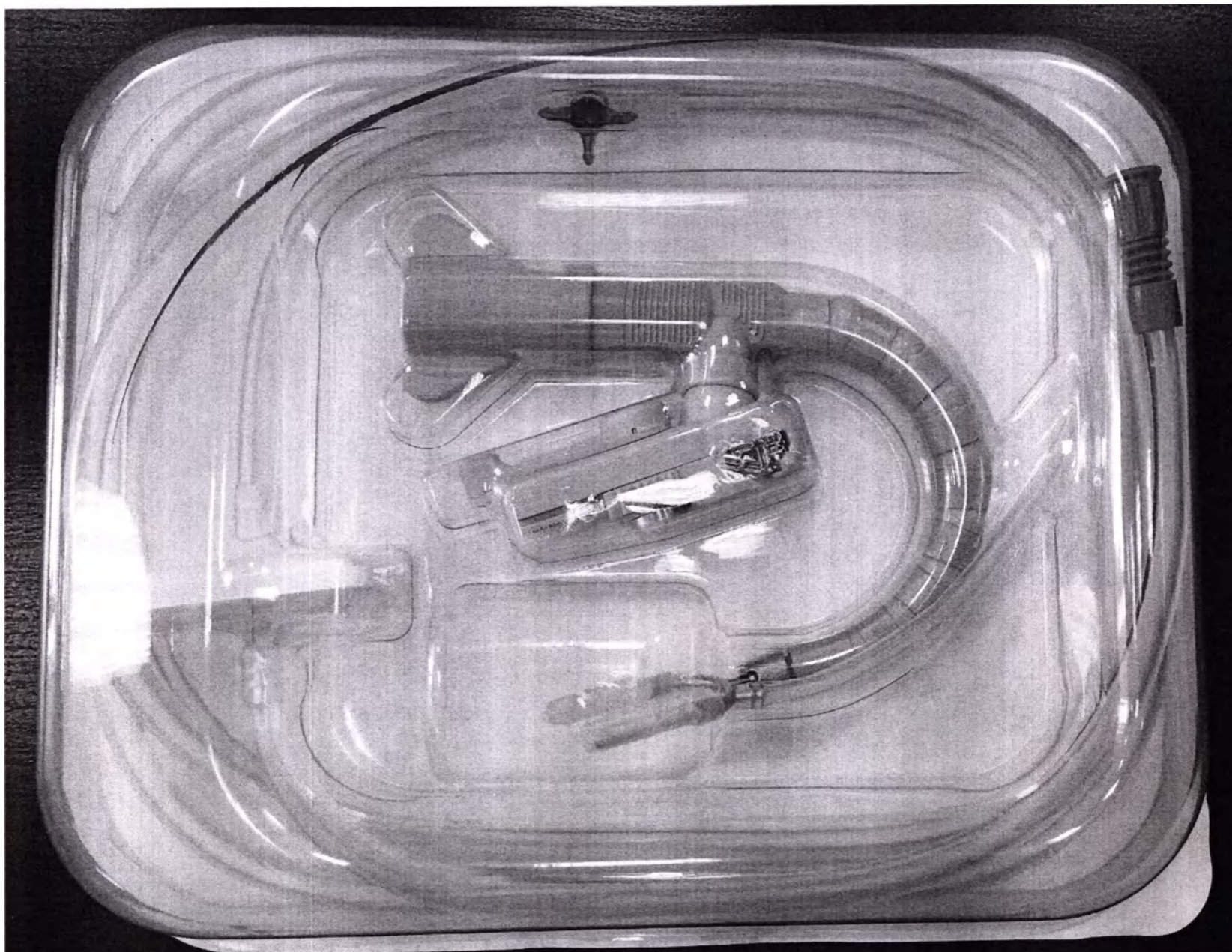• Model Name : 5385-003

• Sterilization Methods : E.O Gas

• Packing Unit : Refer to box label (Unit: EA)

• Purposes, Instruction, Cautions, Storage : Please refer to the instruction manual

• Serial No : Same as box

• Manufacturing Date : Same as box

• Expiration Date : Same as box

• Disposable Sterile Medical Device

EC REP

Aemi world Eugenhefenstr. 9, 10551 Berlin, Germany TEL : +0049-(030)-8620-3461

• Manufacturer : Jinhwa Medical Co., Ltd. • Tel : +82-43-232-7554

125, Osong Saengmyeong 2-ro, Osong-eup, Heungdeok-gu, Cheongju-si, Chungcheongbuk-do, Rep of Korea

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
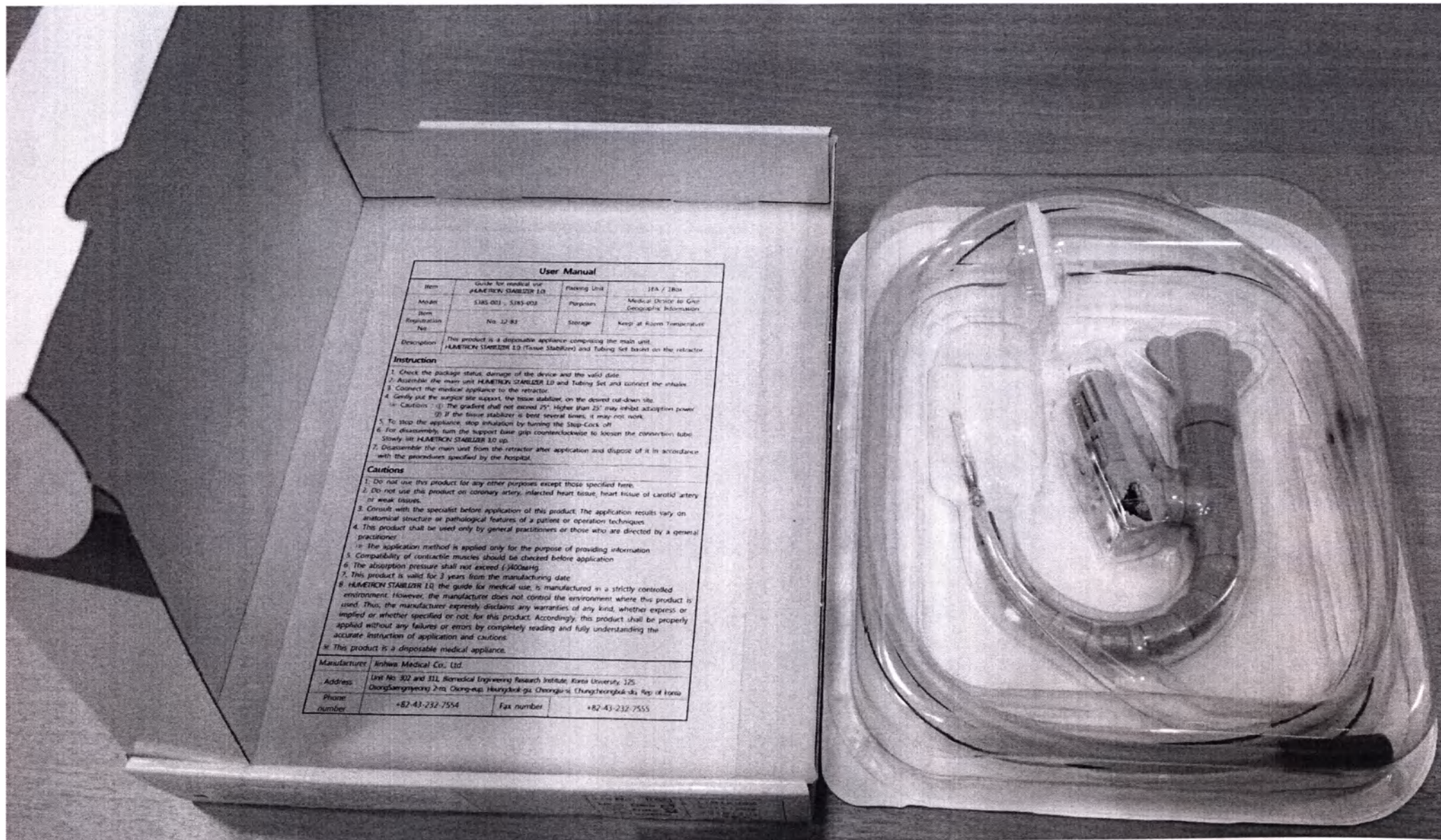
User Manual			
Item	Guide for medical use HUMETRON STABILIZER 1.0	Flasking Unit	1EA / 2Box
Model	5385-003 - 5385-003	Purpose	Medical Device for Cardiac Circulatory Information
Item Registration No.	No. 12-83	Storage	Keep at Room Temperature
Description	This product is a disposable appliance comprising the main unit, HUMETRON STABILIZER 1.0 (Tissue Stabilizer) and Tubing Set located on the retractor.		
Instruction			
1. Check the package status, damage of the device and the valid date. 2. Assemble the main unit, HUMETRON STABILIZER 1.0 and Tubing Set and connect the intake. 3. Connect the medical appliance to the retractor. 4. Gently give the surgical site support, the tissue stabilizer, on the desired cut-down site. * Caution : (1) The gradient shall not exceed 25". Higher than 25" may inhibit absorption power. (2) If the tissue stabilizer is bent several times, it may not work. 5. To stop the appliance, stop inflation by turning the Stop-Cock off. 6. For disassembly, turn the support base grip counterclockwise to loosen the connection tube. * Stowly lift HUMETRON STABILIZER 1.0 up. 7. Disassemble the main unit from the retractor after application and dispose of it in accordance with the procedures specified by the hospital.			
Caution			
1. Do not use this product for any other purposes except those specified here. 2. Do not use this product on coronary artery, infarcted heart tissue, heart tissue of carotid artery or weak tissues. 3. Consult with the specialist before application of this product. The application results vary on anatomical structure or pathological features of a patient or operation techniques. 4. This product shall be used only by general practitioners or those who are directed by a general practitioner. * The application method is applied only for the purpose of providing information. 5. Compatibility of contractile muscles should be checked before application. 6. The absorption pressure shall not exceed 6-1400mmHg. 7. This product is valid for 3 years from the manufacturing date. 8. HUMETRON STABILIZER 1.0 the guide for medical use, is manufactured in a strictly controlled environment. However, the manufacturer does not control the environment where this product is used. Thus, the manufacturer expressly disclaims any warranties of any kind, whether express or implied or whether specified or not, for this product. Accordingly, this product shall be properly applied without any failures or errors by completely reading and fully understanding the accurate instruction of application and cautions. * This product is a disposable medical appliance.			
Manufacturer	Arlhwa Medical Co., Ltd.		
Address	Unit No. 302 and 311, Biomedical Engineering Research Institute, Korea University, 125, ChongSangmyeong 2-ro, Olsong-eup, Heungdeok-gu, Daegu-si, Chungcheongbuk-do, Rep. of Korea		
Phone number	+82-43-232-7554	Fax number	+82-43-232-7555

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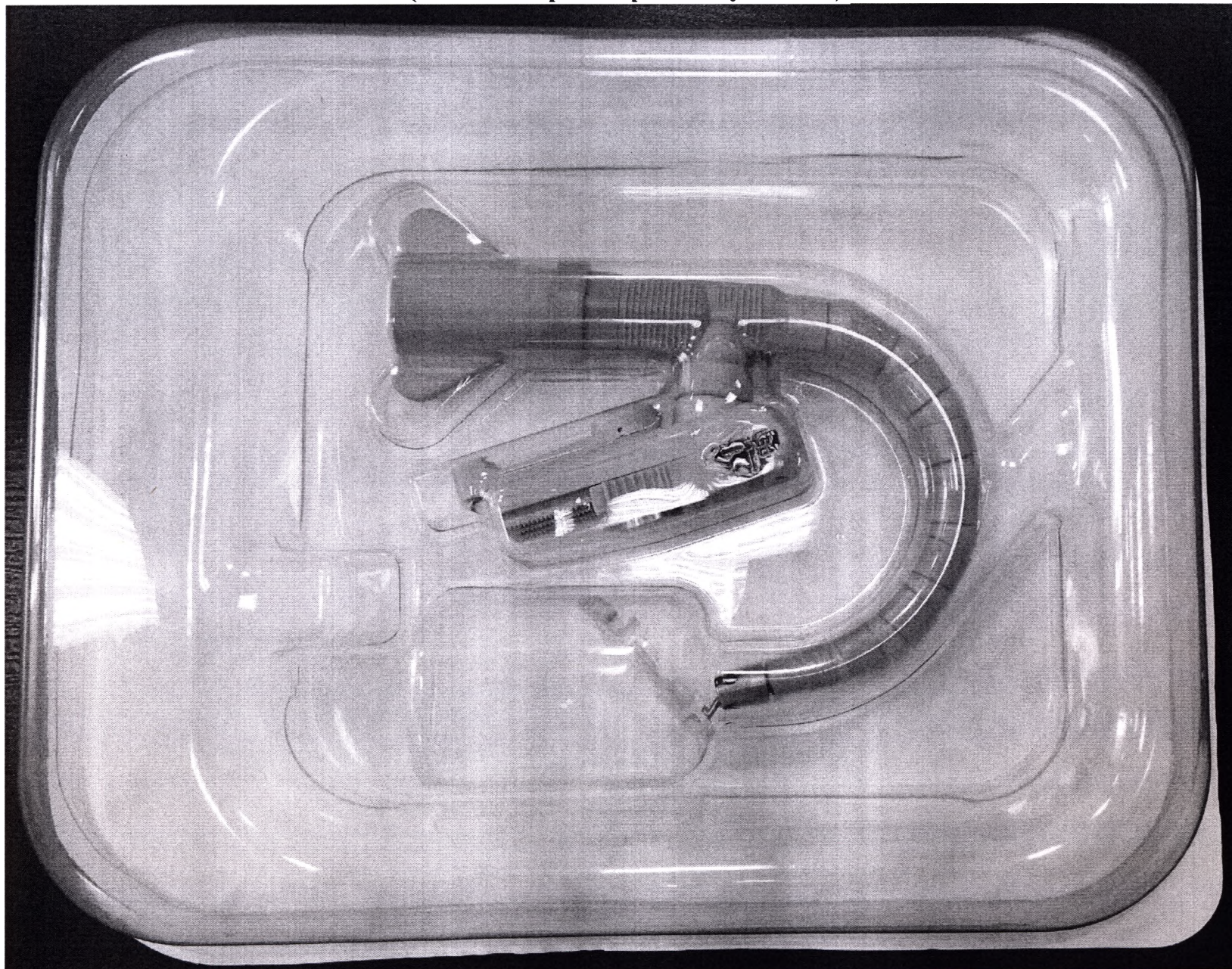• Model Name : 5385-002

• Sterilization Methods : E.O Gas

• Packing Unit : Refer to box label (Unit : EA)

• Purposes, Instruction, Cautions, Storage : Please refer to the instruction manual

• Serial No : Same as 

• Manufacturing Date : Same as 

• Expiration Date : Same as 

• Disposable Sterile Medical Device

Aemi world Bugenhagenstr. 9, 10551 Berlin, Germany TEL : +0049-(0)30-8620-3461

• Manufacturer : Jinhwa Medical Co., Ltd. • Tel : +82-43-232-7554

125, Osong Saengmyeong 2-ro, Osong-eup, Heungdeok-gu,

Cheongju-si, Chungcheongbuk-do, Rep of Korea

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JINHWAMEDICAL CO.,LTD.