State of Minnesota

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SECRETARY OF STATE

APOSTILLE

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1. Country: United States of America

This Public Document: 3M Tegaderm Diamond Pattern Film Dressing

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3M Center, Bldg. 0275-05-W-06
St. Paul, MN 55144-1000
651 733 1110

To Whom It May Concern:

The attached is a copy of the instructions for use for 3M™ Tegaderm™ Diamond Pattern Film Dressing.

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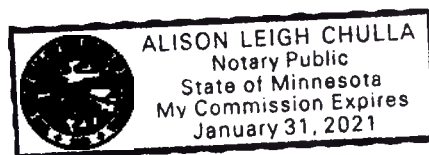
3M™ Tegaderm™ Diamond Pattern Film Dressing

Description: Tegaderm Diamond Pattern Film Dressings are without borders and have a diamond pattern coated film adhesive that provides better adherence to moist skin. The dressing is breathable, allowing good oxygen and moisture vapor exchange.

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*In vitro testing shows that 3M™ Tegaderm™ Diamond Pattern Film Dressing provides a viral barrier from viruses 27 nm in diameter or larger while the dressing remains intact without leakage.

Indications for Use

3M™ Tegaderm™ Diamond Pattern Film Dressing can be used to cover and protect catheter insertion sites and minor wounds. It can also be used as a secondary dressing, as a protective cover over at-risk skin, to secure devices to the skin and as a protective eye covering. Do not use as a replacement for sutures and other primary wound closure methods.

3M™ Tegaderm I.V. Diamond Pattern Film Dressing can be supplied in the following sizes: 10 cm x 11.5 cm, 6 cm x 7 cm, 10 cm x 12 cm.

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Precautions

1. Stop any bleeding at the site before applying the dressing.
2. Do not stretch the dressing during application as tension can cause skin trauma.
3. Make sure the skin is clean, free of soap residue and lotion and allowed to dry thoroughly before applying the dressing to prevent skin irritation and to ensure good adhesion.
4. The dressing may be used on an infected site, only when under the care of a health care professional.
5. Antimicrobial ointments containing polyethylene glycols may compromise the strength of the Tegaderm™ Diamond Pattern Film Dressing.
6. Tegaderm™ Diamond Pattern Film Dressing should not be re-sterilized by gamma, E-beam or steam methods.

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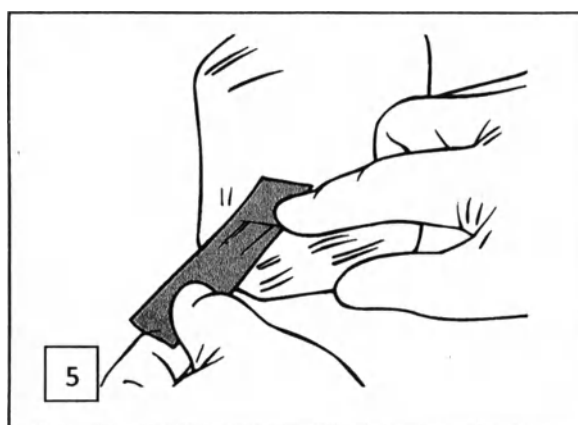
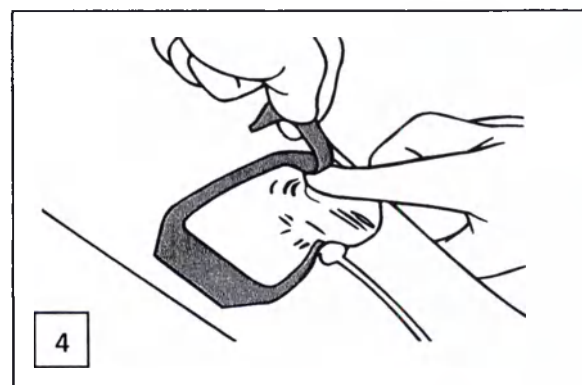
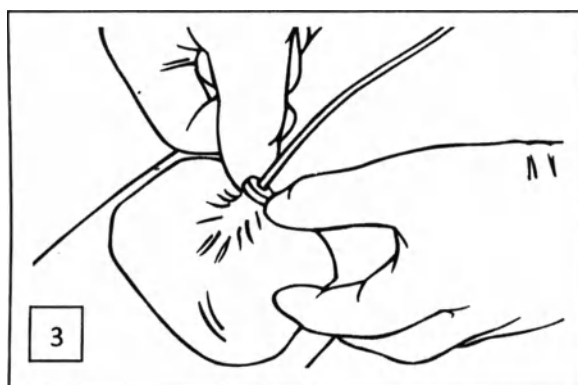
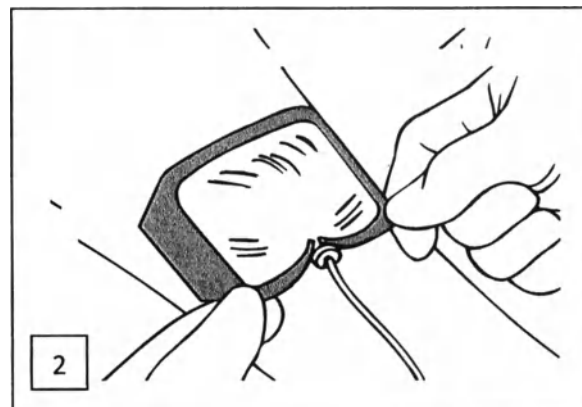
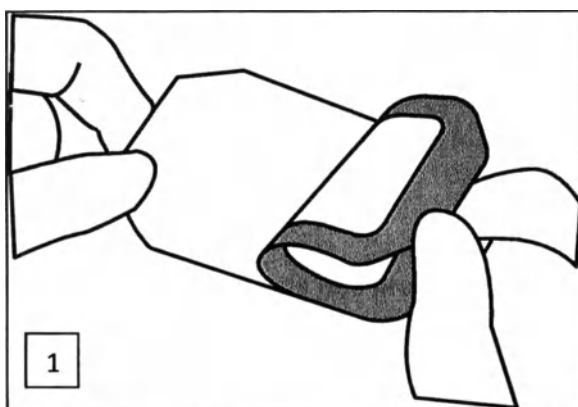
Application Technique

1. Open package and remove sterile dressing. Remove liner to expose adhesive. Place dressing adhesive side up on sterile field or inside of dressing package.

2. Position notch edge of dressing over catheter hub, placing stabilization border over catheter "wings." Do not stretch dressing over skin surface. Apply dressing to skin applying firm, gentle pressure to window film. Pinch film firmly around catheter tubing connection. Gently lift tubing connection and press border firmly under catheter lumens.
3. Apply dressing to skin applying firm, gentle pressure to window film. Pinch film firmly around catheter tubing connection. Gently lift tubing connection and press border firmly under catheter lumens.
4. Slowly peel frame away from border edge while simultaneously applying firm pressure to border edge. Smooth entire dressing into skin to ensure optimal adhesion.
5. Prepare and apply documentation label over the top of the exposed molded junction. Press first half of strip against the junction; then adhere the other side.

Removal Technique

1. Remove all securement tape strips from dressing.
2. Gently grasp soft cloth edges below keyhole notch.
3. Lift dressing on each side from skin using low and slow technique.

4. Peel dressing toward catheter insertion site until removed

Storage/Shelf Life

3M™ Tegaderm Diamond Pattern Film Dressing are considered validated to a two (2) year shelf life. Do not use after the expiration date, which is printed on each package.

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Transport conditions

Shelf life of the products is guaranteed as labeled, if stored according to the recommended storage conditions. During transportation the products may experience short term temperature variations without affecting product safety and performance.

Transportation of medical device is carried out by all type of transport.

Warranty

3M Health Care warrants this product will be free from defects in material and manufacture. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Health Care sole obligation shall be repair or replacement of the 3M Health Care product.

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CE 0086

Made in U.S.A. by
3M Health Care
2510 Conway Ave.
St. Paul, MN 55144 USA
1-800-228-3957
www.3M.com/Tegaderm
www.3M.com/Patents

3M Deutschland GmbH
Health Care Business
Carl-Schurz-Str. 1
41453 Neuss
Germany

3M and Tegaderm are trademarks of 3M.
© 2013, 3M. All rights reserved.
Issue Date: 2012-06
34-8712-2243-5

Authorised representative of the
manufacturer in Russia:
ZAO "3M Russia", Russia 121614
Moscow, Krylatskaya str.
bldg.17/3
T.+74957847474

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1. Country: United States of America

This Public Document: Tegaderm I.V. Advanced Securement Dressing

2. has been signed by Alison Leigh Chulla

3. acting in the capacity of Notary Public, State of Minnesota

4. bears the seal/stamp of by Alison Leigh Chulla, Notary Public, State of Minnesota

CERTIFIED

5. at: St. Paul, Minnesota

6. Date: November 18, 2016

7. by: Secretary of State, State of Minnesota

8. File No: 9161061-4

9. Seal/Stamp:

10. Signature:

Steve Simon
Secretary of State

3M Health Care Business

3M Center
2510 Conway Ave., Bldg. 275-05-W-06
St. Paul, MN 55144 U.S.A.
651 733 1110

To Whom It May Concern:

The attached is a copy of the instructions for use for 3M™ Tegaderm™ I.V. Advanced Securement Dressing.

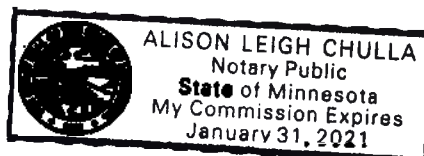
Regina Feferman-Savvateev
International Regulatory Affairs Associate
3M Health Care Business Group

11/17/2016
Date

State of Minnesota)
County of Dakota)^{ss}

Subscribed and sworn to before me this 17th day of November 2016.

Alison Leigh Chulla, Notary Public
Commission Expiration Date: January 31, 2021

Tegaderm™ I.V. Advanced Advanced Securement Dressing

Description: 3M™ Tegaderm I.V. Advanced Securement Dressing consists of a thin film backing with a non-latex, hypoallergenic adhesive. Tegaderm™ I.V. Advanced Securement Dressing is bordered, notched and reinforced with soft cloth tape to provide a better seal around catheters and other devices.

The dressing is breathable, allowing good oxygen and moisture vapor exchange.

It is waterproof and impermeable to liquids, bacteria and viruses*. An intact dressing protects the site from outside contamination.

*In vitro testing shows that 3M™ Tegaderm™ I.V. Advanced Securement Dressing provides a viral barrier from viruses 27 nm in diameter or larger while the dressing remains intact without leakage.

Indications for Use

3M™ Tegaderm™ I.V. Advanced Securement Dressing can be used to cover and protect catheter insertion sites and minor wounds and to secure devices to the skin. Do not use the dressing as a replacement for sutures and other primary wound closure methods.

3M™ Tegaderm I.V. Advanced Securement Dressing can be supplied in the following sizes: 3.8 cm x 4.5 cm, 7 cm x 8 cm, 5 cm x 5.7 cm, 6.5 cm x 7 cm, 8.5 cm x 11.5 cm, 10 cm x 12 cm, 10 cm x 15.5 cm.

Contraindications

No contraindications have been identified for these products.

Potential complications

No potential complications have been identified for these products.

Precautions

1. Stop any bleeding at the site before applying the dressing.
2. Do not stretch the dressing during application as tension can cause skin trauma.
3. Make sure the skin is clean, free of soap residue and lotion and allowed to dry thoroughly before applying the dressing to prevent skin irritation and to ensure good adhesion.
4. The dressing may be used on an infected site, only when under the care of a health care professional.
5. Antimicrobial ointments containing polyethylene glycols may compromise the strength of the Tegaderm™ I.V. Advanced Securement Dressing.
6. Tegaderm™ I.V. Advanced Securement Dressing should not be re-sterilized by gamma, E-beam or steam methods.

Directions for Use

Dressing Selection

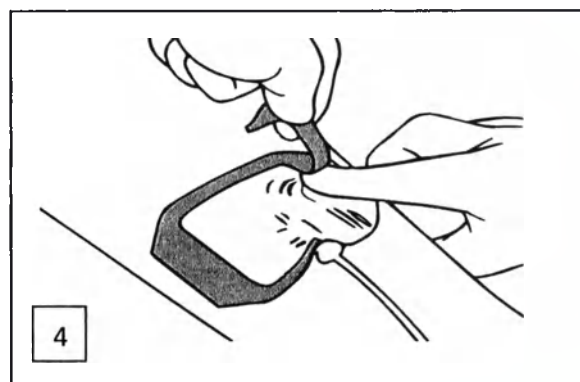
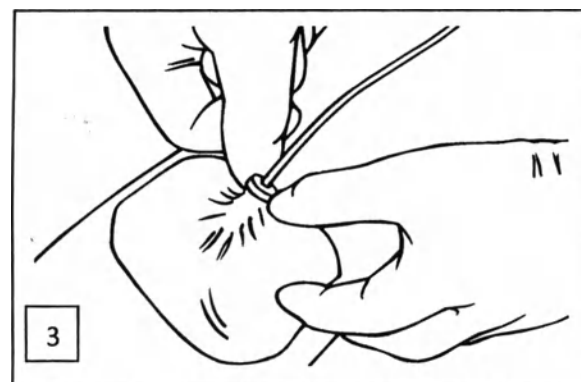
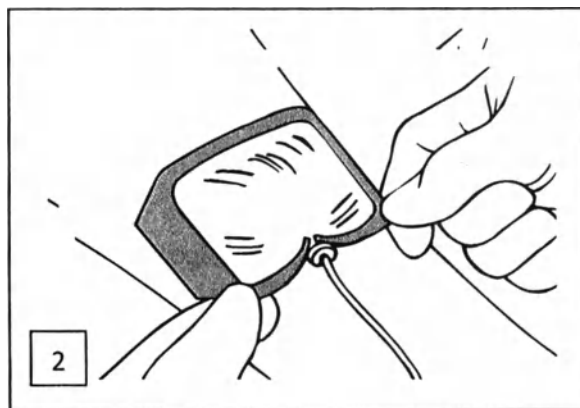
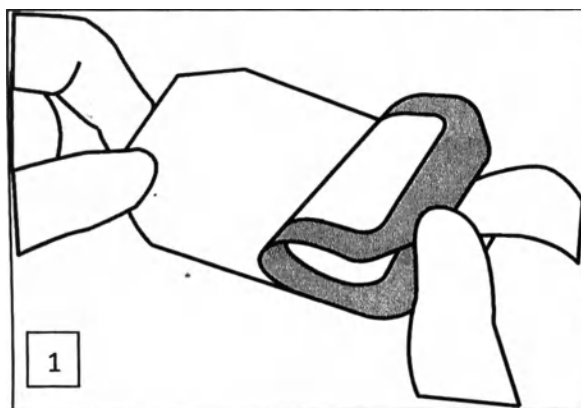
Choose a dressing large enough to provide at least a one-inch margin of adherence on dry, healthy skin around the catheter site.

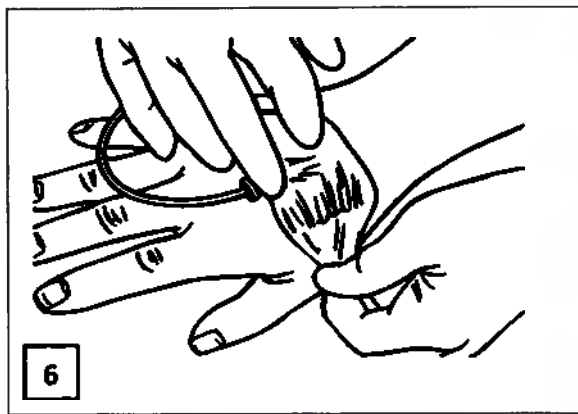
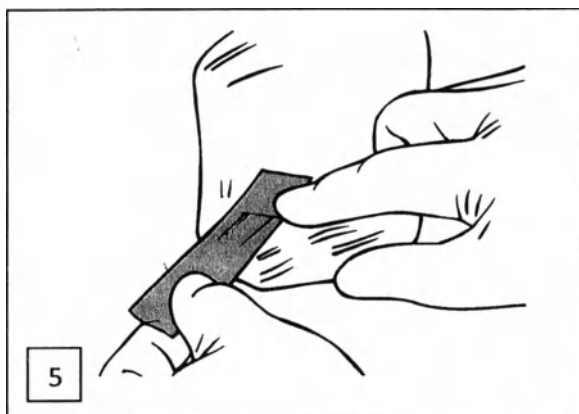
Site Preparation

1. Prepare the site according to your facility's protocol.
2. Clipping hair at the site may improve dressing adhesion. Shaving is not recommended.
3. Allow all preps and skin protectants to dry completely.

Application Technique

1. Open package and remove sterile dressing. Remove liner to expose adhesive. Place dressing adhesive side up on sterile field or inside of dressing package.
2. Position notch edge of dressing over catheter hub, placing stabilization border over catheter "wings." Do not stretch dressing over skin surface. Apply dressing to skin applying firm, gentle pressure to window film. Pinch film firmly around catheter tubing connection. Gently lift tubing connection and press border firmly under catheter lumens.
3. Apply dressing to skin applying firm, gentle pressure to window film. Pinch film firmly around catheter tubing connection. Gently lift tubing connection and press border firmly under catheter lumens.
4. Slowly peel frame away from border edge while simultaneously applying firm pressure to border edge. Smooth entire dressing into skin to ensure optimal adhesion.
5. Apply tape strip's pre-cut notch completely underneath catheter's molded junction. The notch opening should face the insertion site. Peel paper tabs from liner corners. Smooth entire dressing surface; working from catheter insertion site out toward dressing edge.
6. Prepare and apply documentation label over the top of the exposed molded junction. Press first half of strip against the junction; then adhere the other side.

Removal Technique

1. Remove all securement tape strips from dressing.
2. Gently grasp soft cloth edges below keyhole notch.
3. Lift dressing on each side from skin using low and slow technique.
4. Peel dressing toward catheter insertion site until removed

Storage/Shelf Life

3M™ Tegaderm I.V. Advanced Securement Dressing are considered validated to a two (2) year shelf life. Do not use after the expiration date, which is printed on each package.

Store the product at 18-24°C.

This medical device isn't subject to maintenance and repair.

Utilization

Dispose of the device in accordance with applicable local/regional/national/international regulations.

Transport conditions

Shelf life of the products is guaranteed as labeled, if stored according to the recommended storage conditions. During transportation the products may experience short term temperature variations without affecting product safety and performance. Transportation of medical device is carried out by all type of transport.

Warranty

3M Health Care warrants this product will be free from defects in material and manufacture.

User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Health Care sole obligation shall be repair or replacement of the 3M Health Care product.

Explanation of Symbols

Do not use if package is damaged or open

This product and package do not contain natural rubber latex.

Caution, see instructions for use

Do not reuse

Use by date

Batch code

Manufacturer

Sterilized using irradiation

For further information, please contact your local 3M representative or contact us at www.3M.com and select your country.

CE 0086

EC

REP

Made in U.S.A. by
3M Health Care
 2510 Conway Ave.
 St. Paul, MN 55144 USA
 1-800-228-3957
www.3M.com/Tegaderm
www.3M.com/Patents
 3M and Tegaderm are trademarks of 3M.
 © 2013, 3M. All rights reserved.
 Issue Date: 2012-06
 34-8712-2243-5

3M Deutschland GmbH
Health Care Business
 Carl-Schurz-Str. 1
 41453 Neuss
 Germany

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Authorised representative of the
 manufacturer in Russia:
 ZAO "3M Russia", Russia 121614,
 Moscow, Krylatskaya str.
 bldg.17/3
 T.+74957847474

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