in original

Directions for use

For dental use only

**Palodent V3 Dental Matrices in Kits and Refill Packages,
with accessories**

Manufactured by:

**DENTSPLY Caulk, 38 West Clarke Avenue, Milford, DE 19963,
Tel: 1-302-422-4511**

1 Description

1.1.Name:

Palodent V3 Dental Matrices in Kits and Refill Packages, with accessories

1.2.Information about medical device manufacturer/developer:

DENTSPLY Caulk

38 West Clarke Avenue, Milford, DE 19963, USA

Tel: 1-302-422-4511.

1.3.Addresses of medical device manufacturing sites:

DENTSPLY Caulk

38 West Clarke Avenue, Milford, DE 19963, USA

Tel: 1-302-422-4511.

1.4.Description:

The Palodent V3 Dental Matrices employ a nickel-titanium retaining ring, interproximal wedge, WedgeGuard and sectional matrices for placement of restorations in the posterior region. Sectional matrices are suitable for use with all posterior restorative materials. Palodent V3 EZ Coat matrices have a micro-thin, non-stick finish that minimizes the chance that bonding agent or composite sticks to the band, making matrix removal easier. The retaining rings are available in narrow, for most premolar and small molars or universal, for larger molars. Rings may be used either singly or in tandem, for simultaneous restoration of multiple surfaces. The advantages of the sectional matrices and rings include natural contours for better control of contact areas and embrasures, ease of placement, better visualization of the operative field, and more comfort for the practitioner and patient. Eliminating the leverage of a conventional matrix retainer allows the practitioner to use less tension, avoiding excessive compression of tooth and easier formation of anatomical proximal contact areas. When placed prior to preparation, the innovative WedgeGuard combination wedge and shield protects approximating surfaces from inadvertent damage during preparation. Using the pin tweezers, the shield easily slides away, leaving the wedge securely in place.

1.5.Delivery forms

Sectional matrices are available in regular polished stainless or color-coded, non-stick Palodent V3 EZ Coat in 5 sizes: 3.5mm, 4.5mm, 5.5mm, 6.5mm and 7.5mm. These matrices are suitable for use with all posterior restorative materials.

Retaining rings are available in narrow (dark blue) and universal (light blue) sizes suitable for use in the posterior region either singly or in tandem.

Wedges and WedgeGuards are available in small (dark blue), medium (medium blue) and large (light blue) sizes.

1.6.Indications:

The Palodent V3 Dental Matrices employ a nickel-titanium retaining ring, interproximal wedge, WedgeGuard and sectional matrices to facilitate the contouring of a direct placement restorative materials.

Scope: dentistry.

These products are indicated for Class II restorations to facilitate the contouring of a direct placement restorative materials performed by qualified dental personnel only in clinical environments.

The Palodent® V3 WedgeGuard is indicated for pre-operative placement to protect surfaces adjacent to Class II, Class III, veneer and crown preparations.

1.7. Contraindications

None known.

1.8. Delivery forms:

Palodent V3 Sectional Matrix System:

I. Palodent V3 Sectional Matrix System in Kits:

1. Palodent V3 Sectional Matrix System, Introductory kit, including:

- Palodent V3 Matrices 3.5 mm – 25 pieces.
- Palodent V3 Matrices 4.5 mm – 25 pieces.
- Palodent V3 Matrices 5.5 mm – 25 pieces.
- Palodent V3 Matrices 6.5 mm – 25 pieces.
- Palodent V3 Wedges, Small – 25 pieces.
- Palodent V3 Wedges, Medium – 25 pieces.
- Palodent V3 Wedges, Large – 25 pieces.
- Palodent V3 Wedge Guard, Small – 10 pieces.
- Palodent V3 Wedge Guard, Medium – 10 pieces.
- Palodent V3 Wedge Guard, Large – 10 pieces.
- Palodent V3 Forceps.
- Palodent V3 Pin Tweezers.
- Palodent V3 Ring: universal, narrow.

2. Palodent V3 Sectional Matrix System, Starter Kit, including:

- Palodent V3 Ring: universal – 2 pieces.
- Palodent V3 Matrices, 4.5 mm – 5 pieces.
- Palodent V3 Matrices 5.5 mm – 15 pieces.
- Palodent V3 Wedges, small – 5 pieces.
- Palodent V3 Wedges, medium – 5 pieces.
- Palodent V3 Wedges, large – 5 pieces.
- Palodent V3 Wedge Guard medium – 5 pieces.
- Palodent V3 Forceps.
- Palodent V3 Pin Tweezers.
- Illustrated Technique guide – 1 pieces
- Indications for Use – 1 pieces.

II. Palodent V3 matrices Refills:

1. Palodent V3 Sectional Matrix System, Matrices Refill available in sizes 3.5 mm, 4.5 mm, 5.5 mm, 6.5 mm, 7.5 mm in packs of 50 and 100 pieces.
2. Palodent V3 Sectional Matrix System, Wedges Refill available in sizes small, medium, large in packs of 100 pieces.
3. Palodent V3 Sectional Matrix System, Ring Refill available in 1 or 2 pieces per pack, sizes: universal, narrow.
4. Palodent V3 Sectional Matrix System, wedge guard refill available in sizes small, medium, large in packs of 50 pieces.
5. Palodent V3 EZ Coat Sectional Matrix System, Matrices Refill available in sizes 3.5 mm, 4.5 mm, 5.5 mm, 6.5 mm, and 7.5 mm in packs of 5, 50 or 90 pieces.

III. Accessories:

1. Palodent V3 Sectional Matrix System, Forceps Refill, not more than 10 pieces.
2. Palodent V3 Sectional Matrix System, Pin Tweezers Refill, not more than 10 pieces.

2 Images

1) Palodent V3 Sectional Matrix System, Starter Kit:

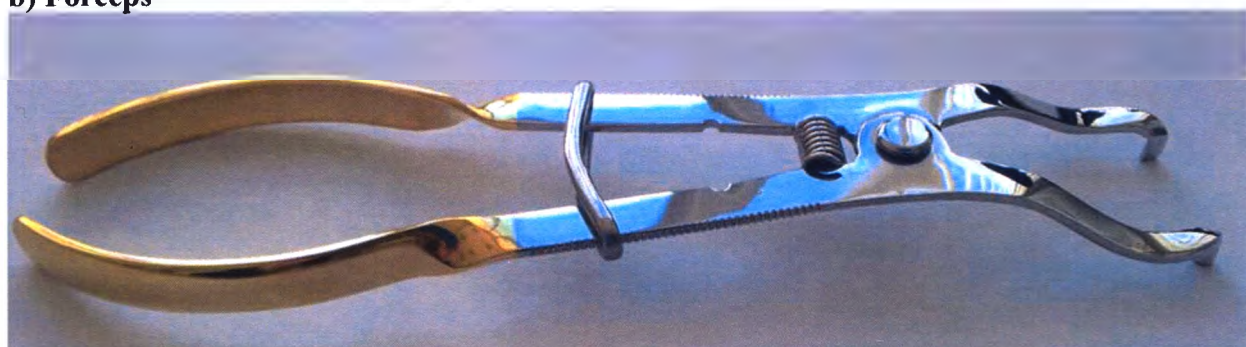
a) Contents of the box

Palodent V3 Sectional Matrix System, Starter Kit, including:

1) Palodent V3 Ring: universal – 2 pieces; 2) Palodent V3 Matrices, 4.5 mm – 5 pieces; 3) Palodent V3 Matrices 5.5 mm – 15 pieces; 4) Palodent V3 Wedges, small – 5 pieces; 5) Palodent V3 Wedges, medium – 5 pieces; 6) Palodent V3 Wedges, large – 5 pieces; 7) Palodent V3 Wedge Guard medium – 5 pieces.

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- 1) Small WedgeGuard
- 2) Medium WedgeGuard
- 3) Large WedgeGuard

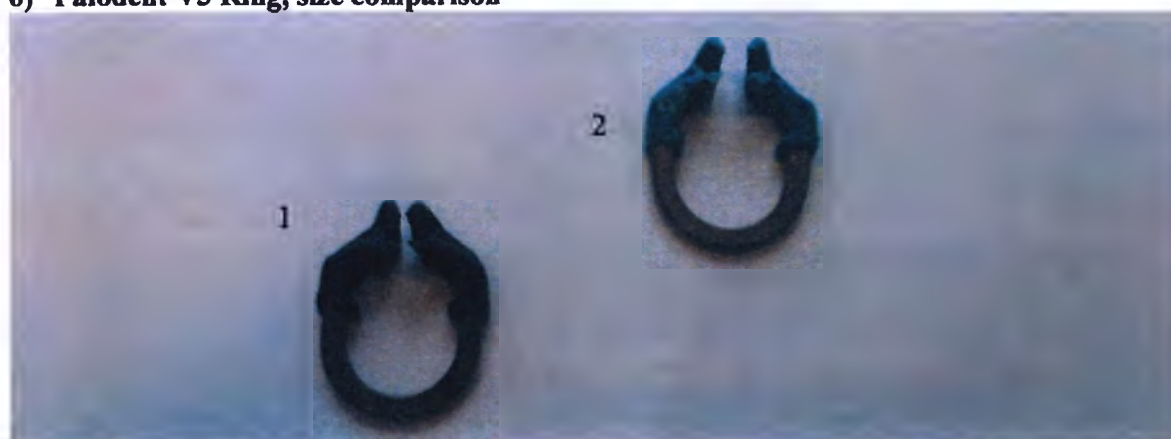
5) Palodent V3 Wedges, size comparison

Palodent V3 Wedges:

- 1) Small
- 2) Medium
- 3) Large

6) Palodent V3 Ring, size comparison

- 1) Palodent V3 Ring, narrow.
- 2) Palodent V3 Ring, universal.

7) Palodent V3 EZ Coat Sectional Matrix System, Matrices Refill

Palodent V3 EZ Coat Sectional Matrix System, Matrices Refill, Size 3.5 mm, pack of 50 pieces.

3 Composition and materials

Item	Parts	Material
Palodent V3 EZ Coat Matrices	Matrice	Stainless Steel 304L
	Coating	Polytetrafluoroethylene XYLAN 8820 HR/D12416 Light Grey (PMS) #428
Palodent V3 Matrices	Matrice	Stainless Steel 304L

4 Technical characteristics

1) **Palodent V3 Sectional Matrix System Matrices** - matrix shim used in matrix dental procedures.

Palodent V3 Sectional Matrix System Matrices are available in sizes: 3.5mm, 4.5mm, 5.5mm, 6.5mm, 7.5 mm.

Material: Stainless Steel 304L

Appearance (visual): Shiny and smooth. Free of dents and kinks.

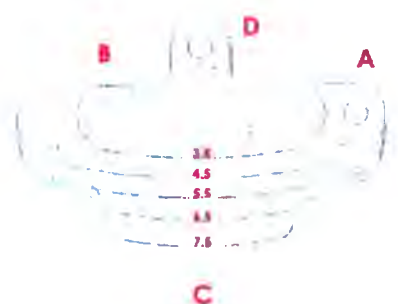
2) **Palodent V3 Sectional Matrix System EZ Coat Matrices** - Teflon Coated Matrix shim used in matrix dental procedures.

Palodent V3 Sectional Matrix System EZ Coat Matrices are available in sizes: 3.5mm, 4.5mm, 5.5mm, 6.5mm, 7.5 mm.

Material: Stainless steel 304L

Coating: Polytetrafluoroethylene XYLAN 8820 HR/D12416 Light Grey (PMS) #428

Appearance (visual): Shiny and smooth. Free of dents and kinks.

A – front side

B – edge of a matrix

C – rounded edges

D – eyelet

3) **WedgeGuards** are available in sizes: Small, Medium, Large.

Material:

- WedgeGuard shield: Polished stainless steel;
- WedgeGuard: Polyacetal Thermoplastic.

4) **Wedges** available in sizes: Small, Medium, Large.

Material: Polyacetal Thermoplastic.

5) **Retaining rings** are used to hold Matrix shim in dental procedures and are available in two sizes:

Universal and Narrow.

Material: Nickel-Titanium alloy with Glass-Fiber Reinforced Tines

Appearance:

- Palodent V3 Sectional Matrix System Universal Ring – Color: Sky Blue.

- Palodent V3 Sectional Matrix System Narrow Ring – Color: Dark Blue.
-

6) Placement forceps:

Material: Stainless steel 420.

Appearance: Steel forceps may have polished or matte surface.

7) Pin tweezers:

Material: Stainless steel 420.

Appearance: Steel Tweezers may have polished or matte surface.

5 Warnings

General Safety Notes

Be aware of the following general safety notes and the special safety notes in other chapters of these directions for use.

Safety alert symbol

Safety alert symbol.

This is a safety alert symbol. It is used to alert you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury.

Warnings

1. The Palodent V3 System consists of small, sharp parts. As with all intraoral instruments, care must be exercised to ensure stable handling and placement. Placement of rubber dam or other suitable isolation is advised prior to operative procedures. Wear suitable protective eyewear and gloves.

Aspiration: is aspiration of matrix, wedge, WedgeGuard, or retaining ring occurs seek immediate medical attention.

Ingestion: if accidental swallowing occurs, encourage patient to seek medical assistance to locate swallowed part. If nausea or illness develop, seek medical attention immediately.

Laceration: The palodent V3 matrices and WedgeGuard shields are sharp, and may cut unprotected skin. Handling matrices and WedgeGuard shields with gloved hands and/or pin tweezers or similar instrument is recommended. In the event a clean matrix or WedgeGuard shield causes a skin laceration, employ usual first aid measures. Discard and properly dispose matrix or WedgeGuard shield.

2. Used matrices and accessories are contaminated. Follow proper exposure control plans. In the event of an accidental exposure, follow recommended post-exposure procedures. Following use of the Palodent V3 System, proper handling and disposal techniques are required. Used matrices are contaminated and sharp. When handling or disposing, use precautions as when handling or disposing other contaminated sharps.

3. Care should be taken to have good control on any instrument to protect patients from injury.

Precautions

1. This product is intended to be used only as specifically outlined in the Directions For Use. Any use of this product inconsistent with the Directions For Use is at the discretion and sole responsibility of the practitioner.

2. Care must be taken when placing the retaining ring, due to the spring nature of the ring. Ensure ring is fully seated in forceps before placement. Before withdrawing forceps, check that the ring is seated securely apically (see Step-by-step Instructions). Tie dental floss to the Palodent ® V3 ring for added security.

3. Excessive pressure on matrix, wedge, WedgeGuard, or retaining ring in an apical direction may result in injury to periodontal tissues.

4. Devices marked “single use” on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
5. Do not use accessory tools other than those designed, supplied, and/or outlined for use with the Palodent ® V3 System. Injury may result. (See Step-by-Step Instructions)

6 Adverse events

- Product misuse may cause failure resulting in respiratory, digestive, eye, skin, or mucosa injury. (See Contraindications, Warnings and Precautions).
- Failure to properly employ accepted handling and disposal techniques may present biohazard. (See Warnings and Precautions).

7 Step-by-step instructions

1. Pre-operative Sterilization

Sterilization Matrices, wedges and WedgeGuards are intended for single use. If desired, these components may be steam autoclaved one time (134°C, 3 minutes minimum holding time) prior to clinical use. Retaining rings, forceps and pin tweezers are not supplied sterile, and should be steam autoclaved prior to first and each subsequent use (see Hygiene section).

2. Place rubber dam or other suitable isolation technique.

3. Pre-operative Placement (optional)

1. WedgeGuard placement

Prior to preparation, select the appropriately sized WedgeGuard, and place interproximally. Palodent V3 WedgeGuards contain a placement tab with hole, which is securely grasped by the pin tweezers. Do not use excessive force.

OR

2. Ring Placement

To effectively use a contoured sectional matrix, there must be enough interproximal space to position the matrix without force or distortion. The retaining ring, when placed prior to and left on during cavity preparation, provides the necessary separation with no additional chair time. **NOTE:** Do not place retaining ring pre-operatively if WedgeGuard is used. Follow these ring placement instructions after preparation completion, shield removal and matrix placement (step 7).

To place the ring, grasp it within at its widest diameter with the provided forceps, being sure the ring rests within the channel on the forceps beaks. Spread the forceps to expand ring only enough to clear tooth height of contour. Do not over-expand. Over-expansion may result in permanent ring deformation. The locking slide on the forceps may be engaged to facilitate carrying and placement of the ring.

Technique Tip: Although Ni-Ti material is known for excellent elastic properties, do not over-stretch the ring to avoid permanent deformation. If ring becomes excessively opened, size and tension may be reset using the forceps (see Hygiene, Sterilization and Maintenance section).

Place the ring with one tine in each interproximal space from buccal to lingual adjacent to the surface to be restored. The direction the ring points, mesial or distal, does not matter, though placing it so that the ring points mesially is usually more easily achieved due to easier access.

With the tines properly positioned interproximally, release the locking slide, and slowly release the forceps tension. When ring is securely seated, release the forceps completely and withdraw forceps.

Small parts under tension.

- Be sure ring is securely seated in the forceps before introducing into the patient field.
- Be sure ring is securely seated apically before withdrawing forceps.

4. Cavity Preparation

Complete cavity preparation as usual. Once the preparation is complete, remove the ring or the metal shield of the WedgeGuard (if applicable). Use the pin tweezers to grasp the hole on the WedgeGuard shield, and slide shield laterally to remove. The wedge may remain in place if apical to the gingival cavosurface margin. Properly dispose of the shield with contaminated sharps.

5. Matrix Selection

The 5.5mm Palodent V3 contoured sectional matrix is recommended for most applications. The 4.5mm and 3.5mm are designed for use with adolescents and patients with poorly erupted posterior teeth. They are also ideal for the direct-indirect technique. The 6.5mm matrix is designed for larger cavities occlusogingivally. It augments a contoured matrix with a gingival apron and an adjustable continuation of the contour at the marginal ridge. The 7.5mm matrix is useful for patients with periodontal issues that require more coverage toward the gingival margin.

Technique Tip: Rings, Matrices, Wedges and WedgeGuards may be used with most popular matrix systems, including other sectionals. The Rings, Wedges and WedgeGuards may be used with most circumferential matrices. The benefits of the Ring, Wedge and WedgeGuard contours assist proper proximal contour.

6. Application of Matrix and Wedge

1. Grasp the matrix with the pin tweezers using the hole located on the occlusal tab. If the ring was placed properly before tooth preparation, then the matrix should slide easily into place adjacent to the space to be restored from the occlusal approach. Ensure matrix is apical to preparation margin with no gap. **Technique Tip:** If WedgeGuard wedge prevents matrix from fully seating gingivally, slide wedge out slightly, place matrix, then reposition wedge.
2. Close the gingival margin by placing a contoured wedge or anatomic wooden wedge. Palodent V3 wedges contain a placement tab with hole, which is securely grasped by the pin tweezers. This assures secure placement and minimizes the likelihood of dropping small accessories. **Technique Tip:** Palodent V3 wedges' hollow design allows for easy placement of a second wedge from the opposing side (if needed) without dislodging the first. **Technique Tip:** Alternatively, the fork design allows placement of the Palodent V3 retaining ring first if desired. The wedge can then be easily placed through the fork tine without dislodging the ring.
7. Replace the Palodent V3 retaining ring in the proximal plane, engaging the matrix and the wedge, thus stabilizing it against tooth structure. The forks of the tines should be sitting over the wedge.
8. Lightly burnish the matrix against the adjacent tooth both buccally and lingually to further adapt it to form proper contours. **NOTE:** Over burnishing may distort matrix and compromise

contours. The pin tweezer has a built-in ball burnisher for this purpose. The matrix placement tab may be bent over the adjacent marginal ridge to facilitate restoration marginal ridge contour. **Technique Tip:** When restoring an MOD preparation, two rings may be placed facing opposite directions, one mesially and one distally or both pointing in the same direction, usually mesially, one over the other placing the mesial matrix and ring first.

9. Placement of Restorative

Place restorative material and establish proximal contact as per usual technique. Light cure and/or allow material to reach initial set. Gross contouring may be accomplished with matrix in place.

10. **Matrix Removal** Remove ring(s) using forceps, remove wedge(s) and sectional matrices using pin tweezers. Finish contouring and verify appropriate proximal contact. **Technique Tip:** It is advisable to “post-cure” light cured restorative materials following matrix removal from all surfaces, including proximo-facial and lingual line angles.

8 Hygiene

Cross-contamination

- Do not reuse single use products. Dispose of in accordance with local regulations. During incineration, Wedges and WedgeGuards may produce irritating and highly toxic gasses.
- Reprocess reusable products as described below.

9 Cleaning, disinfection and sterilization

Cleaning

Manual cleaning: as an alternative to automated cleaning and disinfection, the Palodent V3 placement forceps, pin tweezers and retaining rings can be cleaned by scrubbing with hot water and soap or detergent. After thorough cleaning all device surfaces should be wiped with a single-use cloth in combination with an alcohol-based, bactericidal, virucidal and fungicidal disinfection solution approved according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use. Disinfectant solution must be compatible with cleaning solution. Disinfectant solution residue should be removed with a cloth soaked with water. At the end of cleaning the device should be dried with a lint-free single-use cloth.

Automated washer/disinfector.

Palodent V3 placement forceps, pin tweezers and retaining rings can be cleaned in an automated washer/disinfector.

1. Use only a properly maintained, inspected, calibrated, and approved washer/disinfector (according to ISO 15883).
2. Place all accessories in washer/disinfector in such a way that water with detergent has access to them and is washed away in the disinfector holes according to the instructions.
3. Perform a cleaning program cycle: recommended thermal disinfection with minimum A0 of 3000 (e.g. 5 minutes at $\geq 90^{\circ}\text{C}$) using appropriate detergent.

Palodent V3 placement forceps, pin tweezers and retaining rings can be subjected to the cleaning and automated disinfection up to 500 times.

Sterilization and Maintenance

The Palodent V3 sectional matrix band, Wedges and WedgeGuards are intended for single use only - do not attempt to reprocess and/or re-use. Properly dispose used wedges, WedgeGuards and matrices with other contaminated sharps.

**Cross-contamination**

- Do not reuse single use products such as wedges, matrices and WedgeGuards.
- Dispose of used and/or contaminated matrices, wedges and WedgeGuards in accordance with local regulations.
- During incineration, Wedges and WedgeGuards may produce irritating and highly toxic gasses.
- Reprocess reusable products (i.e., placement forceps, pin tweezers and retaining rings) as described below.

Palodent V3 placement forceps, pin tweezers and retaining rings are subject to sterilization in the autoclave at $\geq 134^{\circ}\text{C}$ for minimum 3 minutes. The Palodent V3 placement forceps, pin tweezers and retaining rings can be subjected to the sterilization process up to 1000 times. Any use of this product inconsistent with the Directions For Use is at the discretion and sole responsibility of the practitioner. It is not recommended to submerge Palodent V3 placement forceps, pin tweezers and retaining rings into disinfection solutions. Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use.

Sterilization of new matrices, Wedges and WedgeGuards pre-use (if desired) may be accomplished (one time) by steam autoclaving, 134°C , 3 minutes without the packaging and 12 minutes inside the package.

Palodent V3 placement forceps, pin tweezers and retaining rings should be disinfected/sterilized and dried prior to storage.

The Palodent V3 forceps and pin tweezers require no additional maintenance and should not be oiled or greased.

If a Palodent V3 retaining ring is deformed by over stretching during placement or removal, place ring within the notches behind the fulcrum of the forceps, and squeeze forceps closed to return ring to its original shape and opening width. This assures proper separation force and secure retention for next use.

10 Packaging

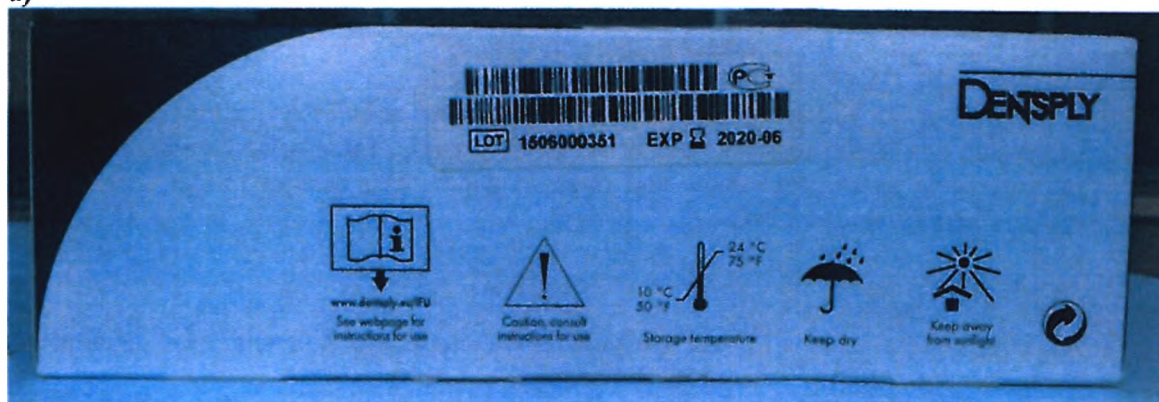
All components of the kit are packaged in a cardboard box and tray. Separately supplied components are packed in a cardboard box.

1. Front view: cardboard packaging of Palodent V3 Sectional Matrix System, Starter Kit


2. Side view:


a)

b)

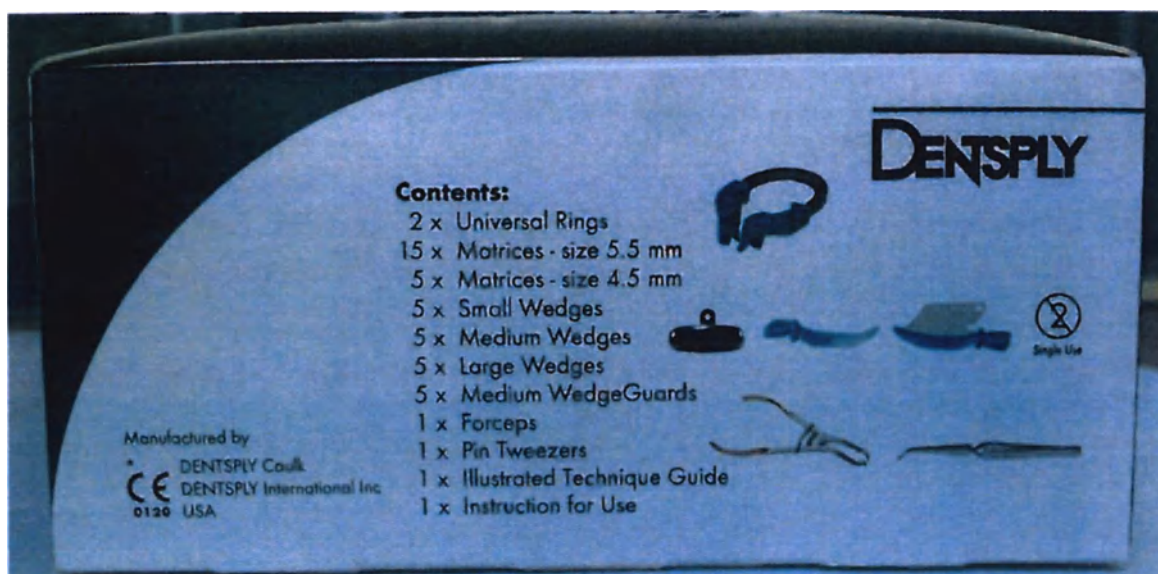c)

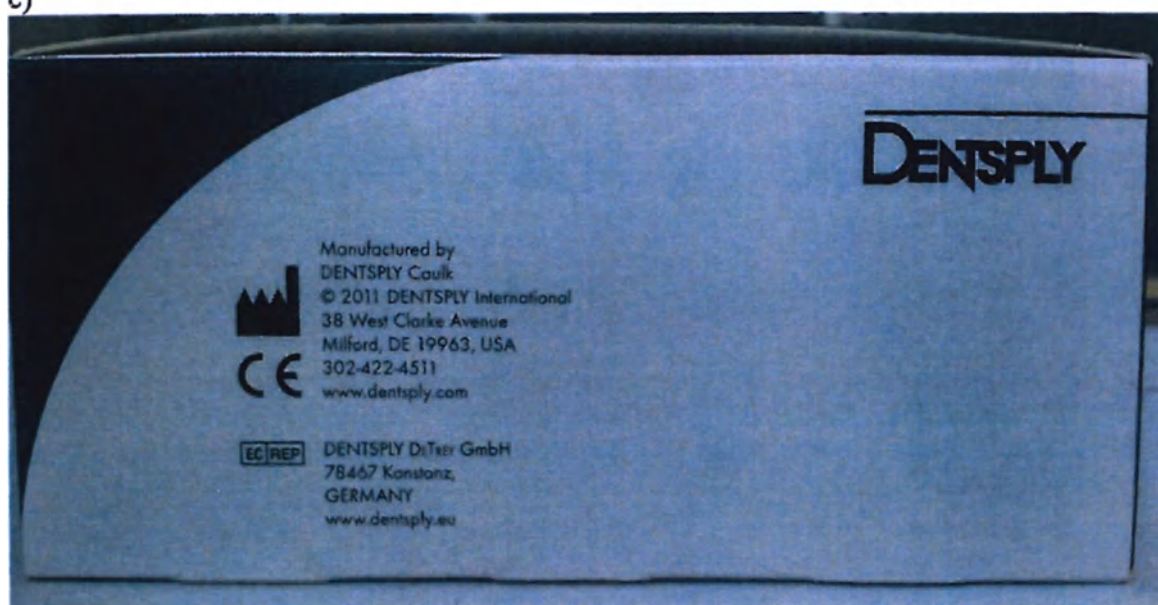4) Inside of the box:

5) Palodent V3 Forceps and Pin tweezers packed in separate boxes inside the Starter Kit

6) Forceps box:

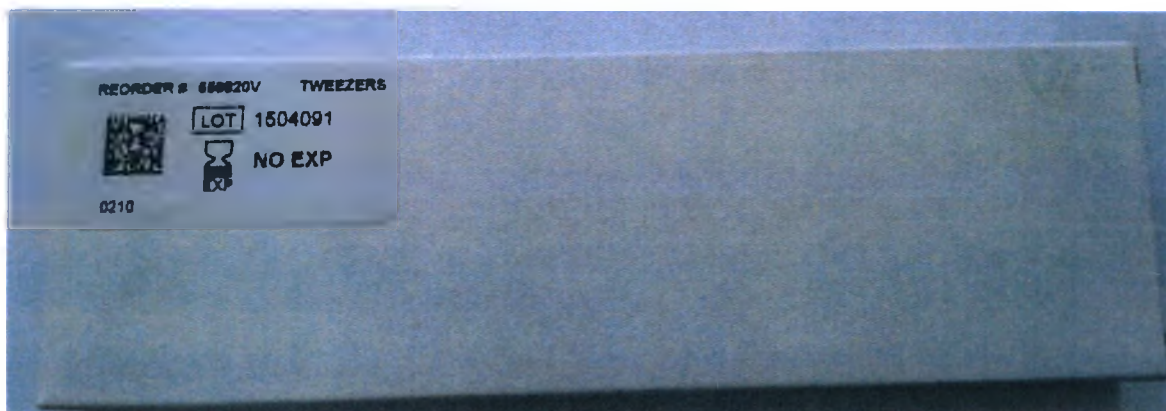
7) Rear view of the forceps box:


8) Front view of pin tweezers box:


9) Rear view of pin tweezers box:

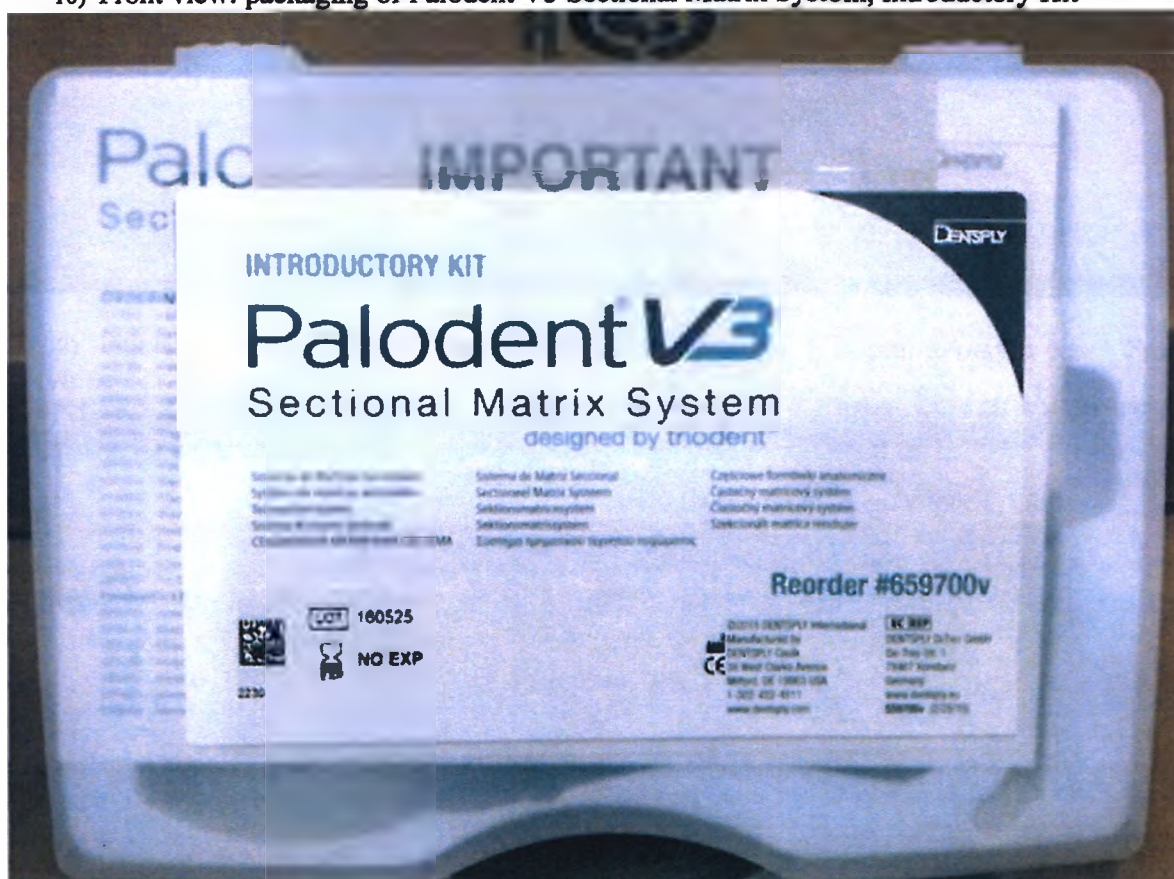
10) Front view: packaging of Palodent V3 Sectional Matrix System, Introductory Kit

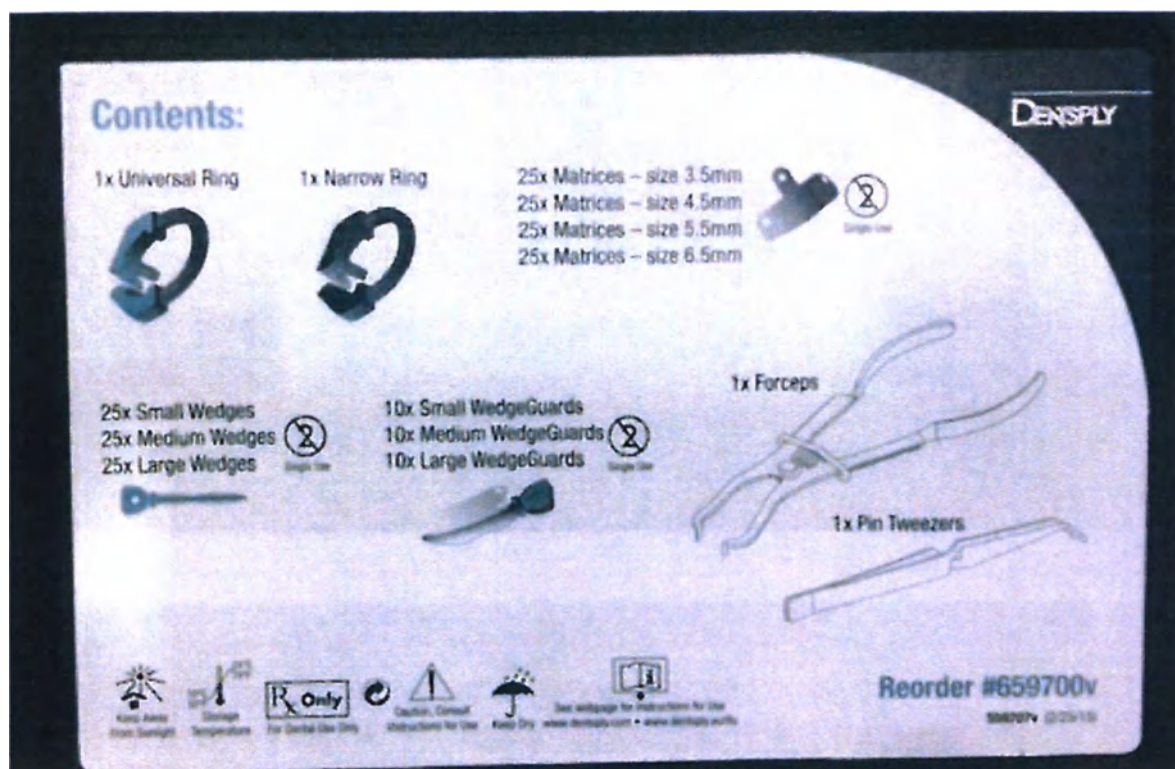11) Contents of Palodent V3 Sectional Matrix System, Introductory Kit

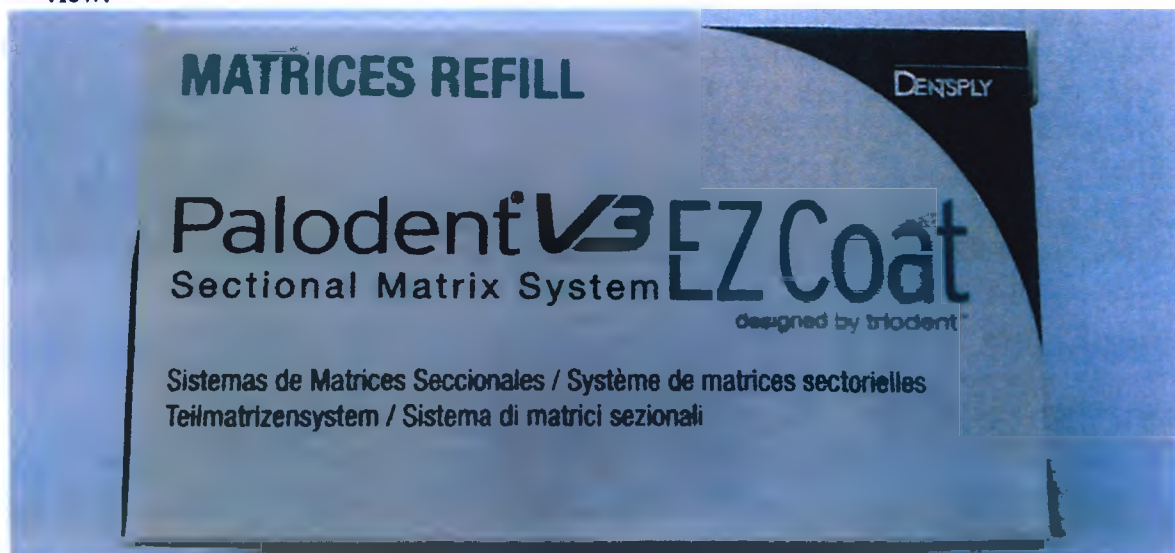12) Palodent V3 EZ Coat Sectional Matrix System, Refill – 3, 5 mm, in carton box – Front view:

11) Side view:

a)

b)

13) Rear view:

11 Symbols used in labelling:

I. Symbols used in labeling

Symbol	Description
	Serial number (batch/lot number)
	Expiration date
	For single use only (for matrices, matrix wedges and WedgeGuards)
	Keep dry
	Manufacturer's name and address
	Warning: read instructions for use
	See webpage for instructions for use: www.dentsply.com , www.dentsply.eu/ifu
	Protect from sunlight
	Temperature limits; both lower and upper temperature limits are noted next to horizontal lines

	CE mark
	PCT mark
	For Dental use only

II. Maquette of labeling in Russian

1. Labelling for kits (for example, Introductory Kit)

Palodent V3 Sectional Matrix System, Introductory Kit

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12 Storage and shelf life

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store the product in a well-ventilated area and protect from sunlight.
- Store in sealed package at temperatures between 10 °C and 24 °C.
- Use Palodent V3 at room temperature.
- Keep out of direct sunlight and protect from moisture.
- Do not freeze.
 - Rings, forceps and pin tweezers can be reused and processed up to 1000 times.

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Sterilization of new matrices, Wedges and WedgeGuards pre-use (if desired) may be accomplished (one time) by steam autoclaving, 134°C, 3 minutes minimum prior to storage and/or use.

13 Transportation

The devices should be transported according to the rules applicable to each transport mode at temperatures between 10 °C and 24 °C and relative humidity <80%.

14 Environmental requirements

The product does not have an adverse impact on the environment during its life cycle.

15 Conditions of use

The product is properly maintained in the mouth, in the presence of the body fluids at nominal temperatures from 32° to 42° C.

16 Maintenance and repair

Products cannot be repaired. Special maintenance is not required.

If the Palodent V3 placement forceps, pin tweezers or retaining rings are damaged, worn, or distorted they should be discarded. Before use, visually inspect the tines for damage, such as cracking or pitting. The Palodent V3 forceps and pin tweezers require no additional maintenance and should not be oiled or greased

17 Disposal

Dispose in accordance with the sanitary rules and regulations of the Russian Federation 2.1.7.2790-10 SanPiN "Sanitary requirements for handling of medical waste" Hazard category of medical waste - B.

18 Manufacturer's warranties

The manufacturer is not responsible and shall not indemnify for any possible damage or accidents caused by:

- Use of components unrelated to the product which can affect normal operation.
- Failure to follow the instructions for use.

The user is responsible for testing of materials for fitness and use for any purpose not expressly specified in the instructions!

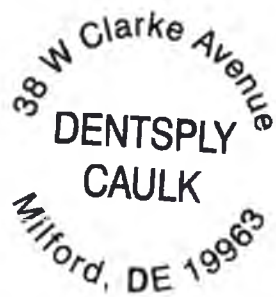
For product quality issues, please contact the Authorized Manufacturer's Representative.

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