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For your own safety and that of your patients

Please be sure to observe Instructions for Use. These Instructions must be thoroughly understood and exactly observed by anyone who uses this device in any way. The relevant Instruction for Use must also be observed when operating the device.

Not for areas of explosion hazard.

It is not allowed to use the device in areas where combustible or explosive gas mixtures are likely to occur.

Liability for proper function or damage.

The liability for proper function of the device is irrevocably transferred to the owner or operator to the extent that the device is serviced or repaired by personnel not employed or authorized by DrägerService or if the device is not used in a manner not conforming to its intended use. Dräger cannot be responsible for damage caused by non-compliance with recommendations given above.

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Intended use

Linea – Wall supply system for rooms used for medical purpose.

- To supply electric power and medical gases to medical appliances.
- For communication and data transmission.
- To provide the indirect and reading light.
- For the ergonomic positioning of medical appliance directly on the carrier rails or on the brackets supported on the rails.

Description of the system

The **Linea** is a prefabricated device, which is permanently installed for application in medical areas. The **Linea** system consists of modular basic units which can be combined with each other and thus it provides a feature of easy to install and easy to maintain. The **Linea** system shall be installed, tested and serviced in accordance with EN 737-3 and thus Instruction in Use.

The **Linea** system is delivered in the proper combination as fully fitted unit.

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Instruction for use Operation

Electric supply

- . The current rate of each electrical socket is max. 16 A

Gas supply

- . As a safety measure, the identification on the plunger must be ensured to be identical with that on the release ring of the terminal unit.
- . Care must be taken to ensure the gas supplying hose on the plunger is properly connected to medical equipment.
- . The connector assembly and plunger for oxygen and nitrous oxide must not come into the contact with oil, grease or any flammable materials. No naked flames, no smoking – increased risk of fire.

Bring the plunger in correct position

- . Push the plunger into terminal unit reach first click-stop.

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
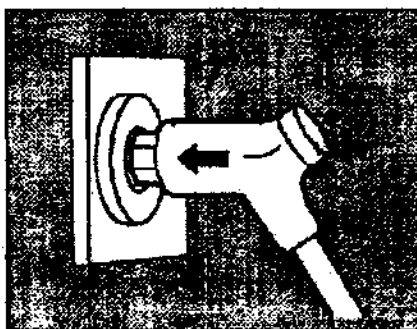
Gas supply

Push the plunger until reach second click-stop.

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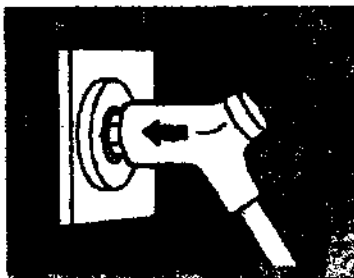**Interruption of gas supply**

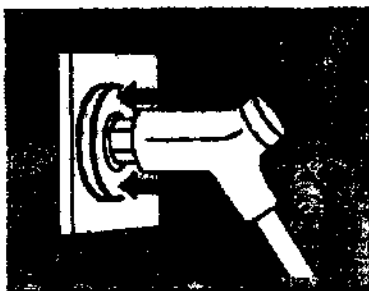
. Lightly push the release ring:

The plunger returns to the first click-stop (parking position)

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
**Disconnect the plunger entirely**

. Keep the release ring being pushed and simultaneously pull the plunger out the terminal unit

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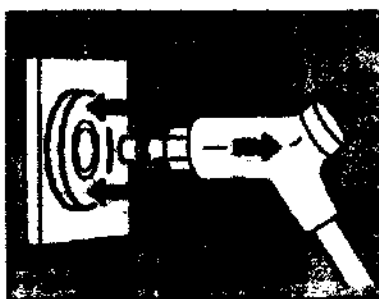Maintenance intervals

It is recommended to conclude a service contract with your local Dräger Service.

Regularly

. Functional and visual check of the entire supply unit to be performed by experts.

Every 6 years

. Visual check by experts of all copper pipes and cables

Every 12 years

. General overhaul and replacement of cables.
. Only original Dräger spare parts may be used for maintenance.

Change the fluorescent tube

- . Screw off the dowel pin.
- . Take out the light cover.
- . Change the fluorescent tube.
- . Re-install the light cover.
- . Re-install the dowel pin.

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Cleaning

Do not use cleaning agents, which release chlorine or oxygen.

- . Wipe the surface of terminal unit, plunger and outer cover of the supply unit with clean cloth which has been soaked in water and liquid detergent before and then wrung out.
- . Don't let any liquid get into the terminal or into the duct of the supply unit.
- . After cleaning, the supply unit must be carefully wiped quite dry with a clean cloth.

Disinfections

. Do not use any phenol or halogen based disinfectants or oxygen generating agents

. Use surface disinfectants on material bases of aldehydes and quaternary ammonium compounds, as recommended by the "Application of Disinfections medium" which is contained in the Actual list of German Association for Hygiene and microbiology (DGHM). The material bases of each medium were also enumerated in the DGHM list.

. Wipe the surface of terminal unit, plunger and outer cover of the supply unit with clean cloth which has been soaked in disinfectant before and then wrung out.

. Don't let any liquid get into the terminal or into the duct of the supply unit.

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