To be submitted to the Competent Authority of Russia

Directions for use

For dental use only

Spectrum Bond dental single-component adhesive

Manufactured by:

DENTSPLY DeTrey GmbH, De-Trey-Strasse 1, 78467 Konstanz, Germany
Tel: +49-(0)7531-583-0; +49-(0)7531-583-104

1	Description	2
2	Composition and materials	2
3	Technical specifications of Spectrum Bond adhesive.....	3
4	Images	3
5	Indication for use	5
6	Contradictions	5
7	Warnings	5
8	Adverse events	6
9	Step-by-step instructions	6
10	Cleaning, Disinfection and Sterilization.....	9
11	Symbols used for labelling.....	9
12	List of applied international standards	11
13	Storage and shelf life	11
14	Transportation.....	11
15	Environmental requirements	11
16	Condition of use.....	11
17	Package contents	11
18	Maintenance and repair.....	11
19	Disposal.....	11
20	Manufacturer's warranties.....	11
21	Authorized Representative.....	11

DENTSPLY DETREY GmbH

De-Trey-Straße 1
D-78467 Konstanz

Ben Fiesel

1 Description

1.1 Name:

Spectrum Bond dental single-component adhesive.

1.2 Information about medical device manufacturer/developer:

«DENTSPLY DeTrey GmbH»,

De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-0; +49-(0)7531-583-104; e-mail: info@dentsply.de

1.3 Addresses of medical device manufacturing sites:

«DENTSPLY DeTrey GmbH», Германия

De-Trey-Strasse 1, 78467 Konstanz, Germany

Tel: +49-(0)7531-583-0; +49-(0)7531-583-104; e-mail: info@dentsply.de

1.4 Description:

Spectrum Bond is a universal, self-priming dental adhesive designed to bond light-cured composite and compomer materials to enamel and dentin as well as to metals and ceramic.

1.5 Purpose:

Dental material is to be used for bonding light-cured composite materials to enamel and dentin or to metals and ceramic in direct or non-direct restorations.

The product must be used by qualified personnel in clinical environment only.

1.6 Delivery forms:

Bottle 3.5 ml.

2 Composition and materials

2.1 Spectrum Bond adhesive composition:

- R5-62-1-dimethacrylate resin
- PENTA
- Urethane dimethacrylate resin (UDMA)
- TMPTMA resin
- Diethylene glycol dimethacrylate
- Functionalized amorphous silica
- Butylated hydroxytoluene
- Camphorquinone
- 4-dimethyl amino- ethyl benzoate
- Cetylamine hydrofluoride
- Acetone

2.2 Bottle for adhesive

Component	Material
Bottle	low density polyethylene, high density polyethylene, EVOH, ADMER
Cap	polypropylene
Valve	polyethylene

3 Technical specifications of Spectrum Bond adhesive

3.1 Volume of adhesive in a bottle

Volume of adhesive in a bottle: 3.5 ± 0.001 ml.

3.2 **Visual characteristics:** slightly turbid, yellow liquid without impurities

3.3 **Odour:** acetone like.

3.4 **pH:** not applicable since the adhesive does not contain water¹.

3.5 Shear bond strength to enamel and dentin:

	Adhesion strength, MPa
Adhesion to enamel	31.2 ± 3.9
Adhesion to dentin	26.8 ± 4.0

3.6 Tensile strength:

	Tensile strength, MPa
Adhesion to enamel *	23.8 ± 5.2
Adhesion to dentin **	24.1 ± 6.5

3.7 Polymerization time²:

Not less than 10 sec. Power of the light source is not less than $300 \text{ mW} / \text{cm}^2$, wave length including 470 nm^3 .

4 Images

Spectrum Bond dental single-component adhesive is packed into 3.5 ml easy to squeeze bottle.

Front view

Rear view

¹ The value of pH is determined by a numerical scale of acidity or alkalinity of a dilute aqueous solution.


² Recommended.

³ Peak absorption of the photoinitiator

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Spectrum Bond dental single-component adhesive in a 3.5 ml bottle is packed into a carton box:

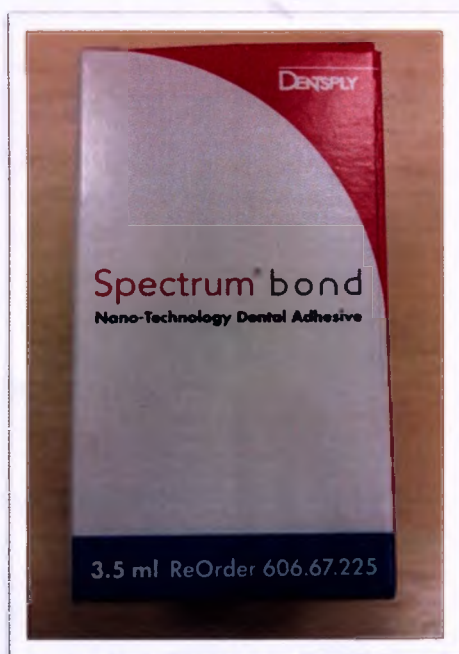Front view

Rear view

Side view

5 Indication for use

Self-priming adhesive for:

- Composite restorations.
- Dyract compomer materials.
- Cementation in indirect restorations.
- Adhesive repairs/restorations.
- Cavity varnish for use with fresh amalgam.

6 Contradictions

- Patients who have a history of allergic reaction to dimethacrylate plastics or any of the other adhesive components.
- Direct or indirect application to dental pulp (direct or indirect pulp capping).

7 Warnings

Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol

	• This is the safety alert symbol. It is used to alert you to potential personal injury hazards. • Obey all safety messages that follow this symbol to avoid possible injury.
---	--

7.1 Warnings

The material contains polymerizable acrylate and methacrylate monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons. The material is an acid agent and may cause burns.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.

Spectrum Bond adhesive contains acetone.

- **Do not breathe vapor.** Keep away from sources of ignition.

7.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole

responsibility of the dental practitioner.

- Use protective measures for the dental personnel and patients such as glasses and rubber dam in accordance with recommendations of your local dental association.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. It is recommended to use adequate isolation such as rubber dam.
- Tightly close the bottle immediately after use.
- Apply adhesive quickly to avoid solvent evaporation. Protect from ambient light.
- Use in a well-ventilated area. Avoid inhaling vapor.
- Flammable: Keep away from sources of ignition.
- Avoid Spectrum Bond adhesive saturating the gingival retraction cord. If Spectrum Bond adhesive soaks into the cord, it may set hard and bond the cord to the underlying tooth surface making removal difficult.
- To prevent bottles from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the instruments and accessories are handled with clean/disinfected gloves.

7.3 Interactions:

Do not use eugenol- and hydrogen peroxide-containing materials in conjunction with Spectrum Bond adhesive since they may cause softening of polymer components and/or interfere with hardening of the product.

- If mineral-impregnated (e.g., ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.

8 Adverse events

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.

9 Step-by-step instructions

9.1 Light-cured composite restorations

9.1.1 Cleaning

1. Clean uninstrumented enamel and dentin with a rubber cup and pumice or a non-fluoride cleaning paste such as Nupro prophylaxis paste.
2. Wash thoroughly with water spray and air dry.
3. Clean freshly instrumented enamel and dentin with water spray, then air dry.

9.1.2 Pulp protection

1. In deep cavities cover the dentin close to the pulp (less than 1 mm) with a hard-setting calcium hydroxide liner (Dycal calcium hydroxide liner, see Instructions for Use), leaving the rest of the cavity surface free for bonding.

9.1.3 Tooth conditioning/dentin pre-treatment

When used as a bonding agent for direct restorative materials, it is recommended to follow the total etch technique described in this chapter. Prior to use, please see the complete Instructions for Use of the respective conditioner.

Application of 34%-36% phosphoric acid conditioner

1. After application of rubber dam or other suitable isolation technique, apply the respective conditioner.
2. Apply the respective conditioner to the cavity surfaces starting at the enamel margins.
3. For best results, condition enamel for at least 15 seconds and dentin for 15 seconds or less.

Rinsing and blot drying

1. Remove conditioner with aspirator tube and/or vigorous water spray and rinse conditioned areas thoroughly for at least 15 seconds.
2. Remove rinsing water completely by blowing gently with an air syringe or by blot drying with a cotton pellet. Do not desiccate dentin.
3. Proceed immediately to application of Spectrum bond adhesive.

Contamination.

Restoration failure.

1. Once the surfaces have been properly treated, they must be kept uncontaminated.
 2. If salivary contamination occurs, thoroughly clean with vigorous water spray, dry, and repeat conditioning procedure of enamel for 5 seconds only.
- Rinse and dry as described above.

Desiccation.

Inferior adhesion.

1. Do not desiccate the tooth substance – cavity surface should be adequate moist.
2. Avoid water pooling, leaving a moist, glistening surface. Do not rub the tooth surface when blot drying.

9.1.4 Dosage

1. To prevent the bottle from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the bottle is handled offside the dental unit with clean/disinfected gloves.
2. Dispense Spectrum bond adhesive directly onto a clean applicator tip (or disposable brush), making sure that the bottle does not come in direct contact with the brush, or place 2-3 drops of Spectrum bond adhesive into a clean well.
3. Replace cap promptly.

Evaporation of solvent.

Inferior adhesion and sealing properties.

1. Immediately apply adhesive to maintain correct solvent.

9.1.5 Application and curing

1. Using the disposable brush or applicator tip supplied, immediately apply generous amounts of Spectrum bond adhesive to thoroughly wet all the tooth surfaces. This surface should remain fully wet for 20 seconds and may necessitate additional applications of Spectrum bond adhesive.
2. Remove excess solvent by gently drying with clean, dry air from a dental syringe for at least 5 seconds. Surface should have a uniform glossy appearance. If not, repeat application and air dry.
3. Cure Spectrum bond adhesive for 10 seconds 1 using a curing light.

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check curing output.

4. Immediately place restorative material over the cured Spectrum bond adhesive as per restorative material manufacturer's Instructions for Use.

9.2 Dyract compomer materials

Prepare the cavity according to:

1. Cleaning [7.1.1 Cleaning].
2. Pulp protection [7.1.2 Pulp protection].

9.2.1 Conditioning

For most restorative procedures with Dyract, it is not necessary to condition the prepared tooth. In this case, proceed to the next chapter 7.2.2 Application of this section. In the case of stress-bearing Class I and II permanent as well as Class IV Dyract AP restorations and in situations where for cosmetic reasons bevelled enamel margins are preferred conditioning with 34-36% phosphoric acid is recommended [7.1.3 Tooth conditioning/dentin pre-treatment]. Alternatively to phosphoric acid NRC Non-Rinse Conditioner may be applied.

9.2.2 Application

Apply Spectrum bond adhesive according to:

1. Dosage [7.1.4 Dosage].
2. Application and curing [7.1.5 Application and curing].

9.3 Cementation of light-transmissive indirect restorations

9.3.1 Cleaning

See chapter “Cleaning” above.

9.3.2 Pulp protection

See chapter “Pulp protection” above.

9.3.3 Tooth conditioning/dentin pre-treatment

See chapter “Tooth conditioning/dentin pre-treatment” above.

9.3.4 Application and curing

Apply and light-cure Spectrum bond adhesive according to:

1. Dosage. See chapter “Dosage” above.
2. Application and curing. See chapter “Application and curing”.

9.3.5 Preparation of restoration

1. Treat internal bonding surface of restoration according to manufacturer’s or dental laboratory’s Instructions for Use, i.e., etching, mechanical roughening and/or silanization.
2. Apply Calibra Silane Coupling Agent to porcelain surfaces to be repaired following manufacturer’s Instructions for Use.
3. Apply a single coat of Spectrum bond adhesive to the internal bonding surface of the restoration. Immediately air dry for 5 seconds.
4. Cure Spectrum bond adhesive for 10 seconds 1 using a curing light. Ensure uniform exposure of all cavity surfaces.

9.3.6 Cementation

1. Prepare and apply light-cured resin cement according to manufacturer’s Instructions for Use.

9.4 Composite, ceramic and amalgam repairs (light-cured composite resin and compomers)

9.4.1 Preparation and cleaning

1. Roughen the fractured surface as much as possible with a diamond bur. Create mechanical retention where possible. For best results, sandblast the area to be repaired with an intraoral microetcher (50 µm alumina). Rubber dam is recommended with high speed evacuation.
2. Rinse microetched areas for 15 to 20 seconds with water.
3. Air dry.

Note: Bevel porcelain margins before etching.

9.4.2 Tooth conditioning/dentin pre-treatment

1. Etch tooth with 34%-36% Tooth Conditioner Gel as outlined in 7.1.3 Tooth conditioning/dentin

pre-treatment.

2. Etch ceramic restoration repair area with hydrofluoric acid according to manufacturer's Instructions for Use.

9.4.3 Treatment of restoration

1. Rinse with water for 10 seconds.

2. Air dry.

Apply Calibra Silane Coupling Agent to porcelain surfaces to be repaired following manufacturer's Instructions for Use.

9.4.4 Application and curing

Apply and light-cure Spectrum bond adhesive according to:

1. Dosage. See chapter "Dosage" above.

2. Application and curing. See chapter "Application and curing" above.

9.4.5 Completion

1. Complete repair with placement and curing of desired shade(s) of light-cured composite restorative following manufacturer's Instructions for Use.

9.5 Cavity varnish for use with fresh amalgam

9.5.1 Preparation

1. Finish preparation by removing existing restorations and/or caries

9.5.2 Cleaning

Clean preparation and place a hard-setting calcium hydroxide liner (Dycal calcium hydroxide liner, see complete Instructions for Use) if needed, according to:

1. Cleaning. See chapter "Cleaning" above.

2. Pulp protection. See chapter "Pulp protection" above.

9.5.3 Tooth conditioning/dentin pre-treatment

1. Rinse and carefully air dry cavity preparation, but do not desiccate exposed dentin.

Note: Acid etching of prepared cavity (enamel or enamel/dentin) is optional prior to placement of Spectrum bond adhesive when used as an adhesive cavity varnish. If desired, see chapter "Tooth conditioning/dentin pre-treatment".

9.5.4 Application and curing

Apply and light-cure Spectrum bond adhesive according to:

1. Dosage. See chapter "Dosage" above.

2. Application and curing. See chapter "Application and curing" above.

9.5.5 Completion

1. Place and condense amalgam (e.g. Dispersalloy Dispersed Phase Alloy), following manufacturer's Instructions for Use.

Adhesive cavity varnish.

When used as a cavity varnish, Spectrum bond adhesive is not an amalgam adhesive. Acid etching of prepared cavity (enamel or enamel/dentin) is optional prior to placement of Spectrum® bond adhesive when used as an adhesive cavity varnish.

10 Cleaning, Disinfection and Sterilization

These products are supplied non-sterile. Cleaning, disinfection and sterilization are not required.

11 Symbols used for labelling

I. Symbols, used for labelling:

СИМВОЛ	Расшифровка
	Serial number (batch/lot number)
	Expiration date
	Manufacturer's name and address
	Caution: consult the Instructions for Use
	Instructions for Use is on official site: www.dentsply.eu/ifu
	Protect from sunlight
	Temperature limits; both lower and upper temperature limits are noted next to horizontal lines
	CE mark
	PCT mark

II. Maquette of labelling in Russian:

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12 List of applied international standards

ISO 13485, Directive 93/42/EEC.

13 Storage and shelf life

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store the product in a well-ventilated area and protect from sunlight.
- Store at temperatures between 10 °C and 24 °C. Use the product at room temperature.
- Protect from moisture.
- Do not freeze.
- Do not use after expiration date.

Shelf-life of Spectrum Bond adhesive – 3 years.

14 Transportation

The devices should be transported according to the rules applicable to each transport mode at temperatures between 10 °C and 24 °C.

14.1 Transportation packaging

Adhesive in carton boxes is packed into a cardboard box (bubble wrap is also used if necessary), which is closed with a sealing tape.

15 Environmental requirements

The product does not have an adverse impact on the environment during its life cycle.

16 Condition of use

The product is properly maintained in the mouth, in the presence of the body fluids at nominal temperatures from 32 ° to 42 ° C. The products cannot be repaired. Special maintenance is not required.

17 Package contents

Carton box contains a bottle with adhesive and Instructions for Use.

18 Maintenance and repair

Devices are not subject to repair. Special maintenance is not required.

19 Disposal

Dispose in accordance with the sanitary rules and regulations of the Russian Federation 2.1.7.2790-10 SanPiN "Sanitary requirements for handling of medical waste." Hazard category of medical waste - B. The product must be used by qualified personnel in clinical environment only.

20 Manufacturer's warranties

The manufacturer is not responsible and shall not indemnify for any possible damage or accidents caused by:

- Use of components unrelated to the product which can affect normal operation.
- Noncompliance with following the instructions for use.

The user is responsible for testing of materials for fitness and use for any purpose not expressly specified in the instructions!

For product quality issues, please contact the Authorized Manufacturer's Representative.

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