To be submitted to the Competent Authority of Russia

Directions for use

For dental use only

**ceram.x SphereTEC one Universal dental light-cured restorative material
with accessories
manufactured by**

**«DENTSPLY DeTrey GmbH», De-Trey-Strasse 1, 78467 Konstanz, Germany
Tel: +49-(0)7531-583-0; +49-(0)7531-583-104**

1	Description	2
2	Intended use.....	3
3	Composition and materials	3
4	Technical specifications.....	4
5	Images	6
6	Indications.....	10
7	Contraindications.....	10
8	Warnings	11
9	Adverse events	12
10	Step-by-step instructions	12
11	Cleaning and disinfection	15
12	Packaging	17
13	Symbols used in labelling:	23
14	List of applied international standards	25
15	Storage and shelf life	25
16	Transportation.....	25
17	Environmental requirements	25
18	Condition of use.....	25
19	Disposal.....	25
20	Manufacturer's warranties.....	25
21	Authorized Representative.....	26

DENTSPLY DETREY GmbH
De-Trey-Straße 1
D-78467 Konstanz

1 Description

1.1 Name

ceram.x SphereTEC one Universal dental light-cured restorative material with accessories.

1.2 Description

ceram.x SphereTEC one Universal dental light-cured restorative material – is a nano-ceramic, light-cured, radiopaque, universal composite for restoration of anterior and posterior teeth. ceram.x SphereTEC one is based on the novel SphereTEC filler technology indicated for both direct and indirect restorations as well as indirect fabrication of inlays, onlays and veneers. The spherical, granulated filler technology called SphereTEC™ leads to a ball-bearing effect that takes the handling properties of ceram.x SphereTEC one to the next level:

- ceram.x SphereTEC one easily adapts to cavity surfaces but does not stick to hand instruments.
- ceram.x SphereTEC one is easy to sculpt precisely and has a high slump resistance.
- ceram.x SphereTEC one provides a firm consistency but is at the same time easy to extrude from syringes and compules tips.

ceram.X SphereTEC one comprises of five universal CLOUD shades of intermediate translucency named A1 to A4.

Each CLOUD shade is designed to achieve shade match with a range of VITA¹ shades. As a whole, the five CLOUD shades cover all 16 VITA classic shades and allow for aesthetic clinical results. Additionally, one bleach shade (BW) is available for bleached teeth in compule version only.

The spherical, granulated fillers consisting of submicron primary glass fillers provide a fast and easy polishing procedure resulting in an outstanding gloss.

Filler system based on SphereTEC™.

The filler system comprises barium glass, pre-polymerized filler and ytterbium fluoride (77-79 weight-% or 59-1 volume-%). Additional ingredients are catalysts, additives, stabilizers and pigments (~1 weight-%, according to shade). The total amount of inorganic fillers is 72-73 weight-% (48-50 volume-%). The particle size of the inorganic fillers ranges from 0.1 to 3.0 µm.

Accessories

1. **ceram.x SphereTEC one Original Material Shade Guide** - a shade guide for selection of ceram.x SphereTEC one Universal shades.
2. **Compules Tips Gun** – an applicator gun for all DENTSPLY filling materials delivered in Compula or Compules Tips.

1.3 Compatible adhesives

The material is to be used in combination with a (meth)acrylate-based dentin/enamel adhesive such as the DENTSPLY adhesives Spectrum® bond, prime&bond one® ETCH & RINSE or prime&bond one® SELECT (manufactured by Dentsply DeTrey GmbH, Germany).

1.4 Information about medical device manufacturer/developer

«DENTSPLY DeTrey GmbH», De-Trey-Strasse 1, 78467 Konstanz, Germany
Tel: +49-(0)7531-583-0; +49-(0)7531-583-104; e-mail: info@dentsply.de

1.5 Addresses of medical device manufacturing sites

«DENTSPLY DeTrey GmbH», De-Trey-Strasse 1, 78467 Konstanz, Germany

¹ Note: VITA® is a registered trademark of VITA Zahnfabrik H.Rauter GmbH & Co. KG.

1.6 Delivery forms

Ceram.x SphereTEC one Universal dental light-cured restorative material is delivered in:

I. Ceram.x SphereTEC one Universal dental light-cured restorative material with accessories, configurations:

1. ceram.x SphereTEC one Universal Syringe Refill Package, 3 g syringes, 1 piece per pack, in different shades: A1, A2, A3, A3.5, A4.
2. ceram.x SphereTEC one Universal Compule Refill Package, 0.25 g compules, 12 pieces per pack; in different shades: A1, A2, A3, A3.5, A4, BW.
3. ceram.x SphereTEC one Universal Sample: 0.25 g compules, 4 pieces per pack, shade: A2.
4. ceram.x SphereTEC one Universal Intro Kit, including:
 - ceram.x SphereTec one Universal Compules, 0.25 g compules, 24 pieces total per kit, in different shades: A1 (4 pcs), A2 (4 pcs), A3 (8 pcs), A3.5 (4 pcs), A4 (4 pcs);
 - Compules Tips Gun (1 piece);
 - Instructions for use (1 piece);
 - iShade label (1 piece);
 - Order leaflet for ceram.x SphereTEC one Universal Shade Guide (1 piece)
 - Illustrated Technique Guide (1 piece).

II. Accessories:

1. ceram.x SphereTEC one Universal Shade Guide (1 piece).
2. Compules Tips Gun (1 piece).

2 Intended use

Direct restorations of all cavity classes in anterior and posterior teeth as well as indirect fabrication of inlays, onlays and veneers.

Scope: dentistry.

These products should be used only in clinical environments by qualified dental personnel.

3 Composition and materials

3.1 ceram.x SphereTEC one Universal dental light-cured restorative material.

Components	Chemical Name
EBPADMA Urethane Resin	EBPADMA Urethane Resin
Methacrylate modified polysiloxane (organically modified ceramic)	Polycondensation product of N-(3-Triethoxysilylpropyl)-O-(2-methacryloyloxyethyl)urethane
BHT	Butylated hydroxyl toluene
Lumilux	2,5-dihydroxybenzene-1,4-dicarboxylic acid diethyl ester
2-hydroxy-4-methoxybenzophenone	2-hydroxy-4-methoxybenzophenone
Camphorquinone	(±)-2,3-Dioxo-1,7,7-trimethyl-bicyclo-[2,2,1]-heptan
Ethyl-4(dimethylamino) benzoate (DMABE)	Ethyl-4(dimethylamino)benzoate
Me ₂ -DPI	Bis(4-methylphenyl)iodonium hexafluorophosphate
Fluoride containing Barium-aluminum-borosilicate glass, silanated	Fluoride containing Barium-aluminum-borosilicate glass, silanated
Fluoride containing Barium-aluminum-borosilicate glass, silanated (SphereTEC filler)	Fluoride containing Barium-aluminum-borosilicate glass, silanated

Ytterbium fluoride	Ytterbium trifluoride
Pigments according to shade	see table below

Pigments
1. Titanium dioxide
2. Black iron oxide
3. Yellow iron oxide
4. Red iron oxide

3.2 Syringe materials:

Component	Material
Cylinder, piston, cap, sleeve	Polypropylene
Screw	Polybutylene terephthalate

3.3 Compules materials:

Component	Material
Body	Polybutylene Terephthalate, glass fiber reinforced
Cap	Polyethylene

3.4 ceram.x SphereTEC one Universal Shade Guide materials:

Component	Material
Shade Guide Holder	Polybutylenterephthalate
Shade (fingers)	Polycoarbonate

3.5 Compules Tips Gun materials:

Component	Material
Handle, rear part	Polyamide
Handle, front part	Polyamide
Connection piece	Polyamide
Plunger	Polyamide
Retaining spring	Stainless Steel
Fixing bolt	Stainless Steel
Extruding peak	Stainless Steel

3.6 i-Shade label

This is a self-adhesive label (152 x 81 mm), color printed with the scale of colors which allows for simplified adaptation of VITA classical shades into the color scale of ceram.x SphereTEC one Universal.

3.7 Illustrated technical guide

Illustrated technical guide contains a description of step by step process of working with the material, supplemented with visual illustrations, colors and recommendations on use. The guide (150 x 80 cm) has double-sided color printing.

3.8 Instructions for use

ceram.x SphereTEC one Universal dental light-cured restorative material in ceram.x SphereTEC one Universal Intro Kit contains instructions for use for the following components:

- ceram.x SphereTec one Universal dental light-cured restorative material in compules, 0.25 g each (including ceram.x SphereTEC one Universal Shade Guide, i-Shade label). Instructions are printed on paper in black and white color.

4 Technical specifications

4.1 Material

Ceram.x SphereTEC one Universal dental light-cured restorative material in syringes of different shades: A1, A2, A3, A3.5, A4.

Ceram.x SphereTEC one Universal dental light-cured restorative material in compules, 0.25 g of different shades: A1, A2, A3, A3.5, A4, BW.

4.2 Weight of material in the syringe: 3 ± 0.01 g

4.3 Weight of material in the compule: 0.25 ± 0.01 g

4.4 Appearance:

Homogeneous paste without impurities.

4.5 Consistency:

Material	Shades	Consistency
ceram.x SphereTEC one (syringes)	A1 – A4	15-23 mm
ceram.x SphereTEC one (compules)	A1 - BW	15-23 mm

4.6 Curing time and Depth of cure (according to ISO 4049:2009):

Material	Shades	Curing time		Depth of cure
		Power ≥ 500 mWatt/cm ²	Power ≥ 800 mWatt/cm ²	
ceram.x SphereTEC one (syringes)	A1 – A4	20 s	10 s	≥ 2 mm
ceram.x SphereTEC one (compules)	A1 - BW	20 s	10 s	≥ 2 mm

4.7 Sensitivity to Ambient Light (according to ISO 4049:2009):

Sensitivity to Ambient Light: ≥ 100 sec for all shades.

4.8 Opacity:

Material	Shades	Opacity, %
ceram.x SphereTEC one	A1	41 ± 2
	A2	41 ± 2
	A3	41 ± 2
	A3.5	41 ± 2
	A4	41 ± 2
	BW (compules only)	50 ± 2

4.9 Main technical properties of the polymerized material:


Parameter	Units	Value
Flexural strength	MPa	≥ 100
Water Sorption	$\mu\text{g}/\text{mm}^3$	≤ 40 (ISO 4049)
Water Solubility	$\mu\text{g}/\text{mm}^3$	≤ 7.5 (ISO 4049)
Radiopacity	mm eq. Al	≥ 2 (ISO 4049)
Extrudability	N	70 - 110

5 Images

5.1 **ceram.x SphereTEC one** dental light-cured restorative material in 3 g syringes of different shades: A1, A2, A3, A3.5, A4.

5.2 i-Shade label.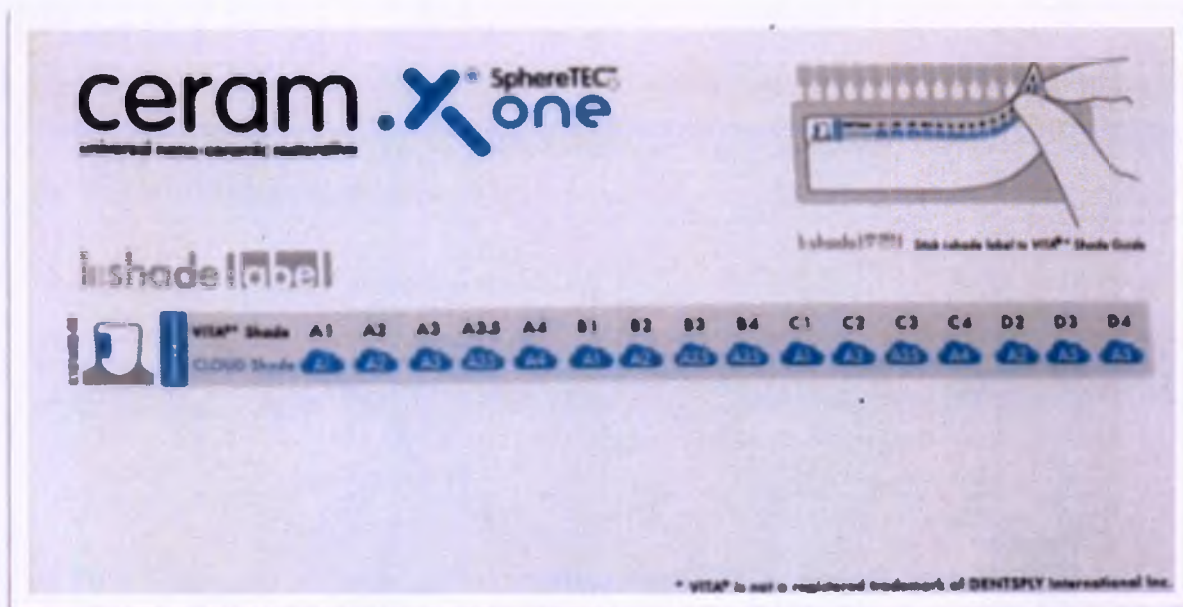

5.3 Ceram.x SphereTEC one Universal Intro Kit

5.3.1 Kit and Compules Tips Gun


5.3.2 Ceram.x SphereTEC one Universal dental light-cured restorative material in compules inside the sphere package

5.4 0.25 g Compule, A1 shade (example).


5.5 ceram.x SphereTEC one Universal Shade Guide².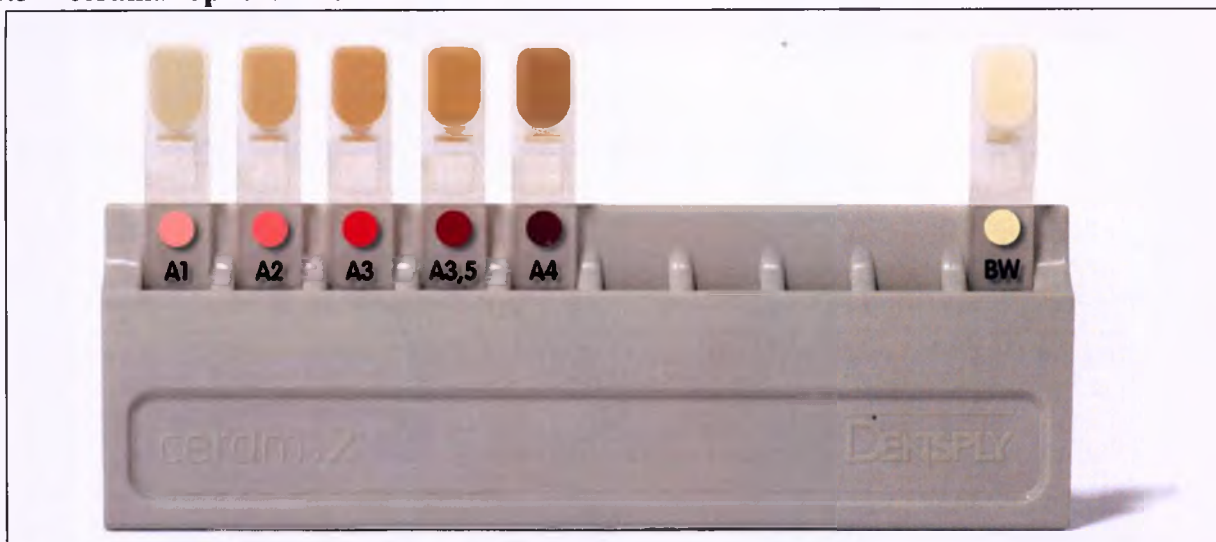

² BW shade is available in compules only

5.6 Compules Tips Gun (Refill)

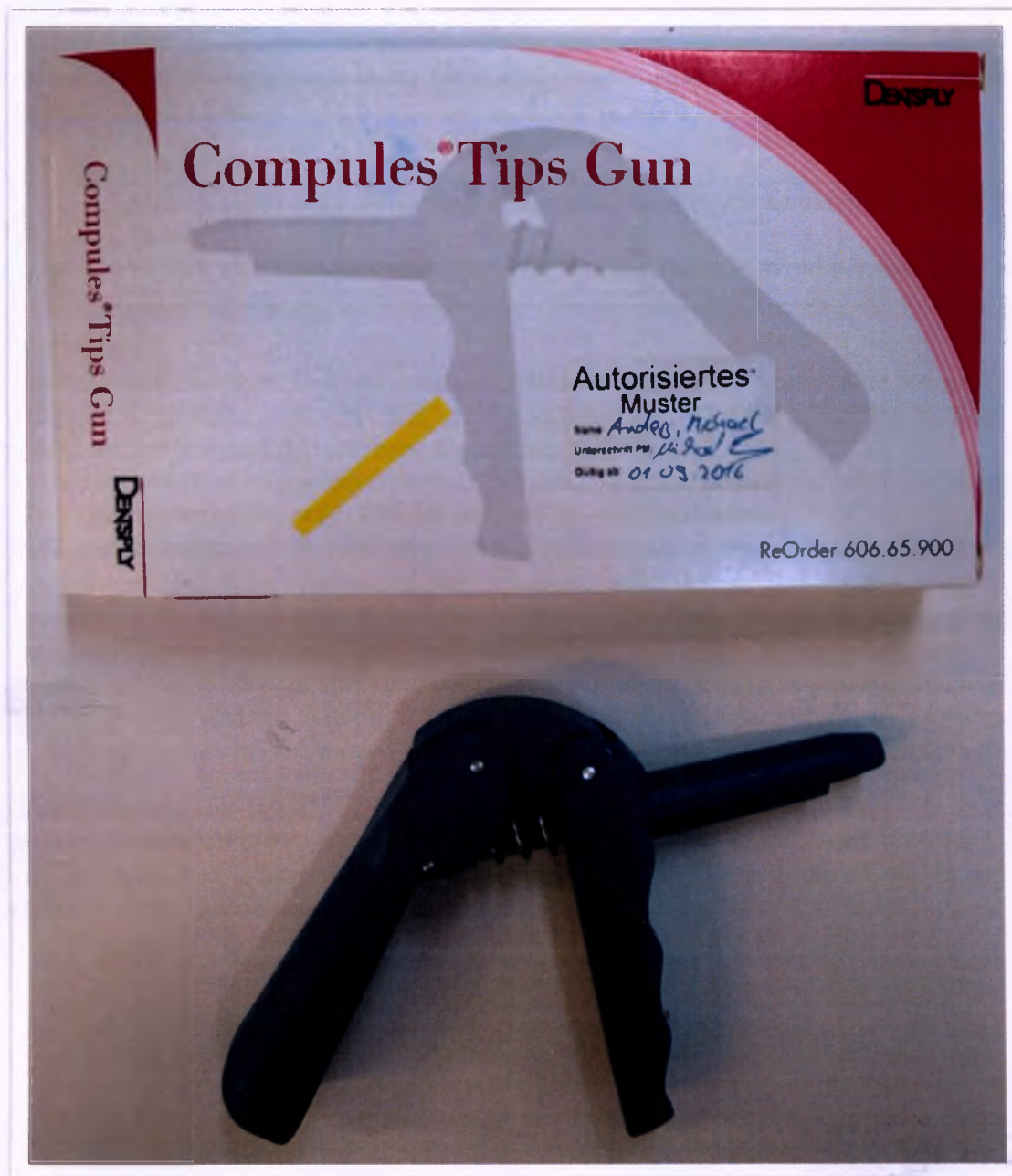6 Indications

Material is intended for direct restorations of all cavity classes in anterior and posterior teeth, as well as indirect fabrication of inlays, onlays and veneers.

Scope: dentistry.

These products should be used only in clinical environments by qualified dental personnel.

7 Contraindications

Patients who have a history of severe allergic reaction to methacrylate resins or any of the other components.

8 Warnings

8.1 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

8.1.1 Safety alert symbol

	• This is the safety alert symbol. It is used to alert you to potential personal injury hazards. • Obey all safety messages that follow this symbol to avoid possible injury.
---	---

8.2 Warnings

The material ceram.x SphereTEC one contains methacrylates and polymerizable monomers which may be irritating to skin, eyes and oral mucosa and may cause sensitization by skin contact and allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

8.3 Precautions:

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- If refrigerated, allow material to reach room temperature prior to use.
- Use protective measures for the dental personnel and patients such as glasses and rubber dam in accordance with local best practice.
- The syringes and Compules Tips cannot be reprocessed. To prevent syringes (and Compules Tips when used for indirect application) from exposure to spatter or spray of biological fluids or spread of material from contaminated hands it is mandatory that the syringes are handled offside the dental unit with clean/disinfected gloves. Discard devices if contaminated.
- Compules Tips may be used for direct intraoral application of restorative material into a cavity or for indirect application by first placing the restorative material on a pad. After intraoral use, discard Compules Tips and do not reuse in other patients in order to prevent cross-contamination.
- Use of Compules Tips with the Compules Tips Gun is recommended.
- The Compules Tips Gun is an applicator gun which is only intended to be used with DENTSPLY Compula® and Compules Tips.
- Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.
- The shade guide and the individual shade guide tabs cannot be reprocessed. To prevent the shade guide and the individual shade guide tabs from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide and the individual shade guide tabs are handled with clean/disinfected gloves.

- The material should extrude easily. **DO NOT USE EXCESSIVE FORCE.** Excessive pressure may cause the Compules Tips to rupture or to eject from the Compules Tips Gun.
- Tightly close syringes immediately after use.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.

8.4 Interactions:

- Do not use eugenol- and hydrogen peroxide-containing materials in conjunction with this dental material since they may interfere with hardening of the product.

9 Adverse events

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

10 Step-by-step instructions

10.1 Shade selection

1. Prior to shade selection, clean the tooth surface with prophylaxis paste to remove plaque or surface stain.
2. Select the shade while teeth are hydrated.

For shade selection, two tools may be used:

- The **ceram.x SphereTEC one shade guide**.
The **ceram.x SphereTEC one shade guide** consists of tabs, which are manufactured from composite with optical properties close to those of **ceram.x SphereTEC one**.
- The **VITA classical shade guide** in combination with the **i-Shade label**. The **ceram.x SphereTEC one i-shade label** facilitates shade selection by assigning one of the five **CLOUD** shades to each of the 16 **VITA®** shades.

10.1.1 Shade selection with the **ceram.x SphereTEC one shade guide**.

- Use the shade tabs of the **ceram.x SphereTEC one shade guide** to select the **CLOUD** shade closest to the shade of the area to be restored.

Avoid contamination of the shade guide.

To prevent the shade guide from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. For shade selection, remove individual tabs from shade guide holder.

10.1.2 Shade selection with **VITA classical shade guide** and **i-shade label**

Have the i-Shade label ready.

To make sure that the **i-shade label** is available chairside, we recommend sticking it on the rear side of the **VITA classical shade guide**.

1. Select the **VITA classical reference tooth**, the central part of which best matches the area of the tooth to be restored.
2. Use the **i-shade label** (see the table below) to identify the matching **ceram.x SphereTEC one CLOUD** shade.

Tooth shade according to the **VITA Classical Shade Guide**

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10.2 Cavity preparation

1. Clean the tooth surface with a fluoride free prophylaxis paste (e.g. Nupro prophylaxis paste (Dentsply Professional, USA)).
2. Prepare cavity (unless not necessary, e.g. cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

10.3 Matrix placement and wedging (Class II restorations)

1. Place a matrix (e.g. AutoMatrix[®] or Palodent[®] matrix system (Dentsply Caulk, USA)) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or BiTine ring (Dentsply Caulk, USA) placement is recommended.

10.4 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application.

Refer to adhesive manufacturer's Instructions for Use for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of the material.

10.5 Application

Apply material in increments immediately after the application of the adhesive. The material is designed to be cured in increments up to a 2 mm depth/thickness.

10.5.1 Using Compules Tips

1. Insert Compules Tip into the notched opening of the Compules Tips Gun. Be certain that the collar on the Compules Tip is inserted first.
2. Remove the colored cap from the Compules Tip. The Compules Tip may be rotated 360° to gain the proper angle of entrance into the cavity or to the mixing pad.

DO NOT USE EXCESSIVE FORCE.

Indirect use: to prevent Compules Tip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the Compules Tip is handled offside the dental unit with clean/disinfected gloves. Dispense the necessary amount of material onto a mixing pad and protect against light. Apply material into the cavity with a plastic instrument.

– or –

Direct intraoral use: dispense the material directly into the cavity preparation using a slow, steady pressure.

3. To remove the used Compules Tip, be sure that the Compules Tips Gun plunger is pulled back completely by allowing the handle to open to its widest position. Apply a downward motion to the front end of the Compules Tip and remove.

Excessive force.

Injury.

1. Apply slow and steady pressure on the Compules Tips Gun.
2. Do not use excessive force. Compules Tip rupture or ejection from Compules Tips Gun may result.

10.5.2 Using syringes.

Avoid contamination of the syringe.

To prevent the syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves.

1. Remove the cap.
2. Turn the handle of the syringe slowly in a clockwise direction and dispense the necessary amount of the material onto a mixing pad.
3. Point the front tip of the syringe upwards and turn the handle anti-clockwise to prevent oozing of the material.
4. Immediately close the syringe with the cap.
5. Protect the restorative material on the mixing pad against light.

10.6 Applying ceram.x SphereTEC one restorative

1. Fill the whole cavity incrementally. Each layer ≤ 2 mm.
2. Cure each layer according to the table below (see **10.7 Light-curing**).

10.7 Light-curing

Light-cure each increment with a suitable curing light³. Cure each increment for 10 seconds when using a curing light as DENTSPLY SmartLite PS or SmartLite FOCUS that has a minimum irradiance of 800 mW/cm².

Cure composite **for 20 seconds** when minimum irradiance is between 500 mW/cm² and 800 mW/cm².

Refer to curing light manufacturer's Instructions for Use for compatibility and curing recommendations.

Curing time for 2 mm increments:

Shade	mW/cm ²	sec
A1-A4, BW(compules only)	≥ 500 mW/cm ²	20 sec
	≥ 800 mW/cm ²	10 sec

Curing time table.

The curing time table above also appears on all outer packages.

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check minimum irradiance.
4. Cure each area of each increment for the recommended curing time.

10.8 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.

³ Curing light designed to cure materials containing camphoroquinone (CQ) initiator. Peak of spectrum in the range of 440-480 nm.

2. Use Enhance finishing devices (Dentsply Caulk, USA) for additional finishing.
3. Prior to polishing, check that surface is smooth and free of defects.
4. For achieving a very high luster of the restoration, we recommend Enhance and PoGo, finishing and polishing instruments, Prisma Gloss Regular and Extrafine polishing paste (Dentsply Caulk, USA).

For contouring, finishing, and/or polishing, follow the manufacturer's Instructions for Use.

11 Cleaning and disinfection

The products are supplied non-sterile. Un-sealed syringes containing the material can be used within 3 months. Sterilisation of the material and accessories is not acceptable. Compules are intended for single use on one patient use and are subject to disposing after one use.

11.1 Cleaning and disinfection of Compules Tips.

NOTICE

Cross contamination.

Infection.

1. The syringes cannot be reprocessed.
2. Do not reuse Compules Tips after intraoral use and/or if contaminated.
3. Dispose the intraorally used and/or contaminated Compules Tips in accordance with local regulations.

11.2 Cleaning and disinfection of syringes

NOTICE

Cross contamination.

Infection.

1. The syringes cannot be reprocessed.
2. To prevent the syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringes are handled with clean/disinfected gloves. Do not reuse syringes if contaminated.
3. Dispose contaminated syringes in accordance with local regulations.

11.3 Cleaning and disinfection of Shade Guide and individual shade guide tabs

NOTICE

Cross contamination.

Infection.

The shade guide and the individual shade guide tabs cannot be reprocessed.

1. To prevent the shade guide and the individual shade guide tabs from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide and the individual shade guide tabs are handled with clean/disinfected gloves. Do not reuse shade guide and the individual shade guide tabs if contaminated.
2. Dispose contaminated shade guide and the individual shade guide tabs in accordance with local regulations.

11.4 Cleaning and disinfection of Compules Tips Gun.

NOTICE

Compules Tips Gun.

Wrong cleaning or disinfection method.

Damage to the device.

1. Always disassemble Compules Tips Gun before reprocessing.
2. Do not clean or disinfect with aggressive agents (e.g. solutions based on orange oil or acetone).

Disassembly and preparation for reprocessing.

1. Partially close the Compules Tips Gun and place your thumb on the rear part of the hinge.
2. Push upwards and lift hinge. The Compules Tips Gun is separated into two parts and the plunger is exposed.
3. In case of gross contamination of the Compules Tips Gun, remove it using a soft brush and running water.
4. Remove restorative material, if any, with a soft paper tissue and alcohol of 70% vol.

Manual cleaning and disinfection.

1. Discard gloves according to local regulations.
2. Disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's Instructions for Use.
3. Use a new clean pair of examination gloves.
4. Clean Compules Tips Gun with a soft brush, warm water, and an appropriate cleaning solution until free of visible contamination. Pay special attention to device seams and insertions.
5. Remove cleaning solution residue with a cloth soaked with water.
6. Dry device with a lint-free single-use cloth.
7. Thoroughly wipe all surfaces of the Compules Tips Gun with a single-use cloth in combination with an alcohol-based, bactericidal, virucidal and fungicidal disinfection solution approved according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions.
8. Remove disinfectant solution residue with a cloth soaked with water.
9. Dry Compules Tips Gun with a lint-free single-use cloth.

Automatic Washer-Disinfector.

As an alternative to manual cleaning and disinfection, the Compules Tips Gun can also be processed in a washer-disinfector⁴. Use only properly maintained, calibrated, and approved washer-disinfector according to ISO 15883-1.

1. Place Compules Tips Gun in the washer-disinfector allowing water and detergent to enter into and drain off from Compules Tips Gun openings during washer-disinfection program and follow washer-disinfector's Instructions for Use.
2. Run washer-disinfection program with A0 value ≥ 3000 (e.g. 5 min at ≥ 90 °C) using appropriate detergents⁴.

Autoclaving (optional).

The Compules Tips Gun can also be autoclaved.

1. Steam autoclave device unwrapped at 134 °C/2 bar = 273 °F/29 psi for 3 minutes 30 seconds.

Reassembly.

1. After reprocessing insert plunger into the Compules Tips Gun cylinder.
2. Press components together and snap mechanism in place.

Do not continue to use damaged or soiled Compules Tips Guns.

Compules Tips Gun can be reprocessed for up to 200 times.

⁴ For example, Miele Washer-Disinfector G7835 CD using program Vario TD with neodisher Mediclean (alkaline detergent) and neodisher Z (Acid neutralization and cleaning detergent); both detergents from Dr. Weigert, Hamburg, Germany (no products or registered trademarks of DENTSPLY International).

12 Packaging

12.1 **ceram.x SphereTEC one Universal (Syringe Refill Package), one 3 g syringe per pack, is packed inside sealed film pouches (see A1 shade example below).**

12.2 ceram.x SphereTEC one Universal (CompuLe Refill Package), 12 compules per pack, is packed inside sealed film pouches (see A3 shade example below).

12.3 ceram.x SphereTEC one Universal Sample A2: 0.25 g compules, 4 pieces per pack, shade: A2.

12.3.1 Front of the package

12.3.2 Back of the package

12.3.3 Inside of the package

12.4 Compules are packed inside sealed film pouches, which are stored in the Sphere-Box as part of ceram.x SphereTEC one Universal Compules Tips Intro Kit.


12.4.1 Side view (A)

12.4.2 Side view (B)

12.4.3 Side view (C)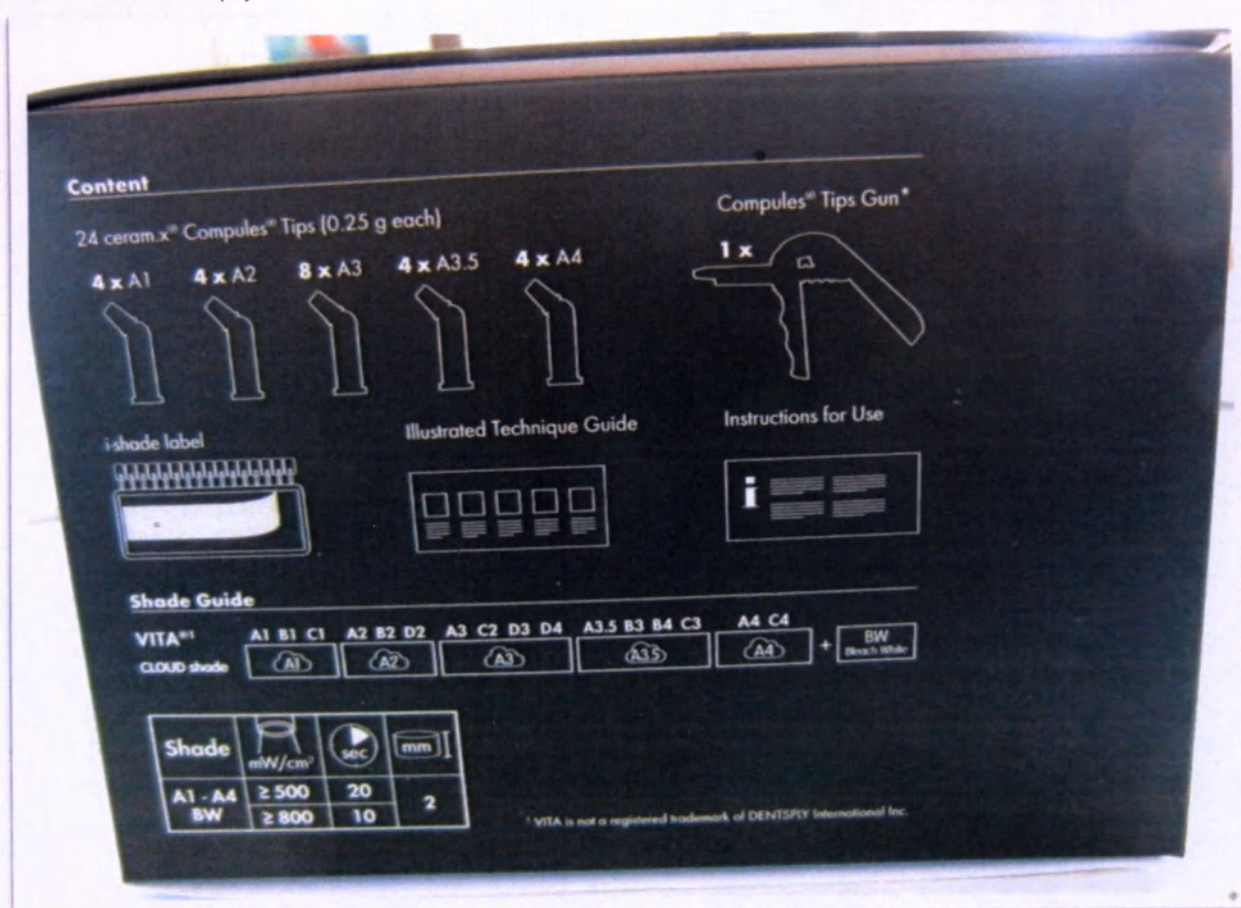

12.4.4 Side view (D)

12.4.5 Top view

2.4.6 Bottom view

13 Symbols used in labelling:

I. Symbols used in labeling

Symbol	Description
	Serial number (batch/lot number)
	Expiration date
	Reference number
	For single use only (for Applicator Tips and all Compules refills)
	Keep dry
	Manufacturer's name and address
	Warning: read instructions for use
	See webpage for instructions for use: www.dentsply.eu/ifu

	Protect from sunlight
	Temperature limits; both lower and upper temperature limits are noted next to horizontal lines
	CE mark
	PCT mark
	Depth of cure
	Curing light irradiance
	Required curing time

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14 List of applied international standards

- Directive 93/42/EC
- ISO 13485
- ISO 14971
- ISO 10993
- EN ISO 7405
- ISO 4049

15 Storage and shelf life

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store in sealed packs at temperatures between 2 °C and 28 °C (36 °F and 82 °F).
- Keep out of direct sunlight and protect from moisture.
- Do not freeze.
- Do not use after expiration date.

The shelf life of ceram.x SphereTEC one Universal dental light-cured restorative material is 24 months from the date of manufacture.

Humidity can adversely affect the properties of the material. For un-sealed syringes, the following is recommended:

- Store in a dry environment (< 80% relative humidity).
- Use within 3 months.

Do not use Compules Tips Gun which is reprocessed for more than 200 times or in use for longer than two years.

16 Transportation

The devices should be transported according to the rules applicable to each transport mode at temperatures between 2 °C and 28 °C and relative humidity <80%.

16.1 Transportation packaging

For the transportation purposes ceram.x SphereTEC one Universal dental light-cured restorative material in syringes and compules is packed into a cardboard box (bubble wrap is also used if necessary), which is closed with a sealing tape.

17 Environmental requirements

The product does not have an adverse impact on the environment during its life cycle.

18 Condition of use

Devices are subject to intraoral use in presence of biological fluids under nominal temperature values of: +10 °C - +42 °C; accessories, which are not used intraorally: +10 °C - +35 °C. The devices are not subject to repair. No special maintenance is required. Products should be used only by qualified medical personnel in clinical institutions.

19 Disposal

Dispose in accordance with the sanitary rules and regulations of the Russian Federation 2.1.7.2790-10 SanPiN "Sanitary requirements for handling of medical waste".

Hazard category of medical waste - B. Products should be used only by qualified medical personnel in clinical institutions.

20 Manufacturer's warranties

The manufacturer is not responsible and shall not indemnify for any possible damage or accidents caused by:

- Use of components unrelated to the product which can affect normal operation.
- Noncompliance with following the instructions for use.

The user is responsible for testing of materials for fitness and use for any purpose not expressed in the instructions!

For product quality issues, please contact the Authorized Manufacturer's Representative.

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