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certified copy

Carl-Schurz-Straße 1
41453 Neuss
Germany

Postal Address:
ESPE Platz
82229 Seefeld
Germany

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+49 (0)8152 700-0
+49 (0)8152 700-1366
Internet: www.3MESPE.com
E-Mail: info3MESPE@3M.com
WEEE-Reg.-Nr. DE 36963167
VAT-ID: DE 120679179

To whom it may concern

February 21, 2017

Registration documents for Adper Easy One - IFU

We 3M Deutschland GmbH, Carl-Schurz-Str. 1, 41453 Neuss, herewith confirm that the following pages are an excerpt of the Information for Use.

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Christine Schalk
Regulatory Affairs

Adper Easy One Self-Etch Adhesive

Instructions for Use

Product Description

Adper™ Easy One is a light-curing self-etch adhesive. It is used in combination with light-curing composite or compomer filling materials, cements and core build-up materials. The adhesive can be polymerized using halogen, LED, or plasma curing lights. Delivery option: vials for free dosing.

 These instructions for use must be kept for reference for the duration of product use.

Indications

- All classes of fillings (according to Black) with light-curing composite or compomer
- Cementation of indirect restorations made of composite or compomer, ceramic, and metal using RelyX™ ARC Adhesive Resin Cement
- Core build-ups made of light-curing composite
- Root surface desensitization
- Repair of composite or compomer fillings
- Intraoral repair of existing composite, porcelain fused to metal, and all ceramic restorations

Contraindications:

- Cementation of veneers and adhesive cementation of self-curing and dual-curing composites

Precautionary Measures

For Patients and Dental Personnel

Adper Easy One: contact with eyes may cause severe eye damage. Wear eye protection to prevent injury. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. **For Patients**

Adper Easy One contains substances that may cause allergic reactions by skin contact in certain individuals.

Avoid use of this product in patients with known acrylate allergies. If prolonged contact with oral tissue occurs, flush with large amounts of water. If an allergic reaction occurs, remove the product, and discontinue future use; seek medical attention as needed.

For Dental Personnel

Adper Easy One contains substances that may cause allergic reactions by skin contact in certain individuals. To reduce the risk of allergic response, avoid exposure to these materials; in particular, avoid exposure to uncured materials. If skin contact occurs, wash skin with soap and water.

The use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If the liquid contacts the glove, remove and discard the gloves. Wash the hands immediately with soap and water and re-glove.

If an allergic reaction occurs, seek medical attention as needed.

Adper Easy One is flammable.

M MSDSs can be obtained at www.mmm.com or from your local subsidiary.

Curing Use

- A rubber dam is recommended for processing the adhesive. Superficial changes of the gingiva, which may occur in some cases, are reversible.
- Temporaries should be seated with non-eugenol temporary cements, as eugenol containing temporary cements can adversely affect the setting of Adper Easy One.
- **Pulp Protection:** To prevent pulpal irritation, cover areas in close proximity to the pulp by applying small amounts of hard-setting calcium hydroxide material.

Preparatory Steps

For Direct Restorations and Core Build-Ups

- Thoroughly remove loose preparation debris by spraying with water.

- Dry the cavity with gentle stream of air free of water and oil, or by blotting with cotton pellets. Do not overdry!

Before Cementing Indirect Restorations with RelyX ARC

- Completely remove any temporary cement remaining on the cavity and the restoration.
- Prepare the restoration for cementation as is needed for the respective material.
- Dry the cavity with gentle stream of air free of water and oil, or by blotting with cotton pellets. Do not overdry!

For Uncut Enamel

- Clean the enamel as usual (e.g., with polishing paste or powder blasting), and thoroughly rinse by spraying with water.
- Etch for 15 sec with a commonly used phosphoric acid etching gel; then thoroughly rinse again.

For Cut Enamel

Cut enamel can selectively be etched to further increase the enamel bond.

- Apply a commonly used phosphoric acid etching gel to the entire enamel surface - incl. uncut enamel if present - and etch for 15 sec.
- Then thoroughly rinse by spraying with water.
- Dry the cavity with gentle stream of air free of water and oil, or by blotting with cotton pellets. Do not overdry!

If accidental etching of dentin occurs, the bond strength is not adversely affected.

Before Desensitization of Root Surfaces

- Clean the root surface as usual (e.g., with polishing paste or powder blasting), thoroughly rinse by spraying with water, and dry with air free of water and oil.

For the Repair of Fillings or Restorations

- Roughen the composite or compomer filling, or the composite or ceramic veneer respectively; with ceramic remove the glaze 1 mm beyond the margin.
- Thoroughly clean the surface with water and dry it with air.
- Silanate ceramic veneers using RelyX™ Ceramic Primer. Please refer to the Instructions for Use of the respective product during processing.

Cavity liner/Base

Self-adhesive materials such as glass ionomer cements (e.g., Ketac™ Bond) or resin-modified glass ionomer cements (e.g., Vitrebond™) can be used as a cavity liner or as a base.

After applying Adper Easy One, flowable composites and flowable compomers can be used as cavity liner.

Dosing from the Vial

- Unscrew the cap of the vial.
- Dose the necessary amount of Adper Easy One to a mixing well and protect the liquid from light.
- Carefully close the vial tightly after dosing. If necessary, disinfect the vial in accordance with local hygiene regulations.

Application

- Avoid any contamination with blood, saliva, or sulcus fluid during application. A rubber dam is recommended to keep the treated area sufficiently dry.
- Apply the adhesive with the disposable applicator to all surfaces of the cavity and rub it for 20 seconds.
- Rewet the disposable applicator as needed during application. Avoid contact of the adhesive with mucosal tissue.
- Subsequently, air thin the liquid for approx. 5 sec until the film no longer moves, indicating complete vaporisation of the solvent.
- Cure the adhesive with a commonly used curing light for 10 sec. Depending on the indication, continue to work with the desired composite
- or compomer material as indicated in the pertinent Instructions for Use.

Disposal

Only dispose of completely empty packages. The residue of the etching fluids - if heavily diluted with water or neutralized - can be poured down the drain.

Storage and Shelf Life

To achieve the maximum amount of shelf life, store the product at 2-8°C/ 36- 46°F.

Refrigeration is not required if the product is depleted within six months.

Do not use after the expiration date.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Information valid as of January 2012

RUS: Authorised representative of the manufacturer in RF: ZAO "3M Russia", Russia 121614, Moscow,

Krylatskaya str. bldg.17/3 T. +74957847474

44000768667103

3M Deutschland GmbH
Dental Products
Carl-Schurz-Str. 1
41463 Neuss - Germany

3M, ESPE, Adper, Ketac, RelyX and Vitrebond are trademarks of
3M or 3M Deutschland GmbH.
© 2016, 3M. All rights reserved.

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